



Important Software Update Information

COGNIS 100-D

TELIGEN 100

Before interrogating this pulse generator, verify that you have the Model 2868 PRM software application version 1.03 or later installed on the PRM. If the pulse generator is interrogated with an earlier version of the software, the PRM may detect invalid parameter values.

If this occurs, the PRM will display a message stating that the pulse generator's settings could not be displayed. Upon closing the message window, all programmable parameters will be automatically programmed to default values and the PRM will reinterrogate the pulse generator to ensure the default values were successfully programmed.

Once the default values have been set, review and, as is necessary, reprogram the pulse generator settings to be clinically appropriate for your patient. You will not be able to reprogram the Post-Shock Pacing Delay, which will be permanently changed from 2.25 seconds to 3 seconds.

If this continues to occur, please contact Boston Scientific.

Boston Scientific
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA

1.800.CARDIAC (227.3422)
+1.866.484.3268

Guidant Europe NV/SA; Boston Scientific
Green Square, Lambroekstraat 5D
1831 Diegem, Belgium

1.800.CARDIAC (227.3422)
+1.651.582.4000

© 2008 Boston Scientific or its affiliates.
All rights reserved.

358009-001 EN EU 06/08

