

REFERENCE GUIDE

ENERGEN™ ICD

IMPLANTABLE CARDIOVERTER HIGH ENERGY
DEFIBRILLATOR

REF F140, F141, F142, F143

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verze. Nepoužívat.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

ABOUT THIS MANUAL

This family of implantable cardioverter defibrillators (ICDs) contains both single- and dual-chamber pulse generators that provide ventricular tachyarrhythmia therapy, bradycardia pacing, and a variety of diagnostic tools.

The organization of the manuals provided for Boston Scientific pulse generators has changed. System Guides have been replaced by Reference Guides, and the Physician's Technical Manual has been expanded.

The Reference Guide contains the following sections formerly covered in the System Guide: Using the Programmer/Recorder/Monitor, Tachyarrhythmia Detection, Tachyarrhythmia Therapy, Pacing Therapies, System Diagnostics, Patient Diagnostics and Follow Up, Electrophysiological Testing, Programmable Options, and Pacemaker Interactions.

Information for Use, Implant Information, and Post Implant Information, topics formerly covered in the System Guide, can now be found in the Physician's Technical Manual.

To view and download any of these documents, go to www.bostonscientific-international.com/manuals.

The text conventions discussed below are used throughout this manual.

- PRM KEYS** The names of Programmer/Recorder/Monitor (PRM) keys appear in capital letters (e.g., PROGRAM, INTERROGATE).
- 1., 2., 3. Numbered lists are used for instructions that should be followed in the order given.
- Bulleted lists are used when the information is not sequential.

This product family includes single- and dual-chamber models, with feature variations. This manual describes the full-featured model (e.g., a dual-chamber model with ZIP telemetry). Some models will contain fewer features; for those devices, disregard descriptions of the unavailable features.

The screen illustrations used in this manual are intended to familiarize you with the general screen layout. The actual screens you see when interrogating or programming the pulse generator will vary based on the model and programmed parameters.

A complete list of programmable options is provided in the appendix ("Programmable Options" on page A-1). The actual values you see when interrogating or programming the pulse generator will vary based on the model and programmed parameters.

The following acronyms may be used in this Reference Guide:

A	Atrial
AF	Atrial Fibrillation
AFib	Atrial Fibrillation
AFR	Atrial Flutter Response
AGC	Automatic Gain Control
AIVR	Accelerated Idioventricular Rhythm
AT	Atrial Tachycardia
ATP	Antitachycardia Pacing
ATR	Atrial Tachy Response
AV	Atrioventricular
BCL	Burst Cycle Length
BPEG	British Pacing and Electrophysiology Group

The following are trademarks of Boston Scientific or its affiliates: ENDOTAK, ENERGEN, LATITUDE, Onset/Stability, QUICK CONVERT, QUICK NOTES, Rhythm ID, RYTHMIQ, Safety Core, Smart Blanking, VENTAK, VITALITY, ZIP, ZOOM, ZOOMVIEW.

CPR	Cardiopulmonary Resuscitation
DFT	Defibrillation Threshold
EAS	Electronic Article Surveillance
ECG	Electrocardiogram
EF	Ejection Fraction
EGM	Electrogram
EMI	Electromagnetic Interference
EP	Electrophysiology; Electrophysiologic
HE	High Energy
IBP	Indications-Based Programming
ICD	Implantable Cardioverter Defibrillator
LRL	Lower Rate Limit
MI	Myocardial Infarction
MPR	Maximum Pacing Rate
MRI	Magnetic Resonance Imaging
MSR	Maximum Sensor Rate
MTR	Maximum Tracking Rate
NASPE	North American Society of Pacing and Electrophysiology
NSR	Normal Sinus Rhythm
PAC	Premature Atrial Contraction
PAT	Paroxysmal Atrial Tachycardia
PES	Programmed Electrical Stimulation
PMT	Pacemaker-Mediated Tachycardia
PRM	Programmer/Recorder/Monitor
PSA	Pacing System Analyzer
PTM	Patient Triggered Monitor
PVARP	Post-Ventricular Atrial Refractory Period
PVC	Premature Ventricular Contraction
RADAR	Radio Detection and Ranging
RF	Radio Frequency
RTTE	Radio and Telecommunications Terminal Equipment
RV	Right Ventricular
RVRP	Right Ventricular Refractory Period
SCD	Sudden Cardiac Death
SRD	Sustained Rate Duration
SVT	Supraventricular Tachycardia
TARP	Total Atrial Refractory Period
TENS	Transcutaneous Electrical Nerve Stimulation
V	Ventricular
VF	Ventricular Fibrillation
VFib	Ventricular Fibrillation
VRP	Ventricular Refractory Period
VRR	Ventricular Rate Regulation
VT	Ventricular Tachycardia
VTR	Ventricular Tachycardia Response

TABLE OF CONTENTS

USING THE PROGRAMMER/RECORDER/MONITOR.....	1-1
CHAPTER 1	
ZOOM LATITUDE Programming System	1-2
Software Terminology and Navigation	1-2
Main Screen	1-2
PRM Mode Indicator	1-3
ECG/EGM Display	1-3
Toolbar	1-4
Tabs	1-5
Buttons	1-5
Icons	1-5
Common Objects	1-6
Use of Color	1-7
Demonstration Mode	1-7
Communicating with the Pulse Generator	1-7
ZIP Telemetry	1-8
Starting a Wanded Telemetry Session	1-8
Starting a ZIP Telemetry Session	1-8
Ending a Telemetry Session	1-9
ZIP Telemetry Security	1-9
Indications-Based Programming (IBP)	1-11
Manual Programming	1-13
DIVERT THERAPY	1-13
STAT SHOCK	1-14
STAT PACE	1-14
Data Management	1-15
Patient Information	1-15
Data Storage	1-15
Device Memory	1-16
Print	1-16
Safety Mode	1-17
Backup Pacemaker	1-17
Backup Defibrillator	1-17
TACHYARRHYTHMIA DETECTION	2-1
CHAPTER 2	
Device Mode	2-2
Ventricular Tachy Mode	2-2
Electrocautery Protection Mode	2-2
Rate Sensing	2-3
Calculating Rates and Refractory Periods	2-3
Ventricular Rate Thresholds and Zones	2-4
Use of Atrial Information	2-4
Ventricular Detection	2-5
Ventricular Detection Enhancement Suites	2-6
Ventricular Redetection	2-9

Ventricular Post-shock Detection Enhancements	2-9
Ventricular Detection Details	2-10
TACHYARRHYTHMIA THERAPY	3-1
CHAPTER 3	
Ventricular Therapy	3-2
Ventricular Therapy Prescription	3-2
Ventricular Therapy Selection	3-2
Ventricular Redetection after Ventricular Therapy Delivery	3-6
Ventricular Redetection after Ventricular ATP Therapy	3-6
Ventricular Redetection after Ventricular Shock Therapy	3-7
Antitachycardia Pacing Therapies and Parameters	3-7
Burst Parameters	3-8
Coupling Interval and Coupling Interval Decrement	3-9
Burst Cycle Length (BCL)	3-10
Minimum Interval	3-10
Burst Scheme	3-11
Ramp Scheme	3-11
Scan Scheme	3-12
Ramp/Scan Scheme	3-12
ATP Pulse Width and ATP Amplitude	3-13
Ventricular ATP Time-out	3-13
QUICK CONVERT ATP	3-14
Ventricular Shock Therapy and Parameters	3-15
Ventricular Shock Vector	3-15
Ventricular Shock Energy	3-15
Charge Time	3-15
Waveform Polarity	3-16
Committed Shock/Reconfirmation of the Ventricular Arrhythmia	3-17
PACING THERAPIES	4-1
CHAPTER 4	
Pacing Therapies	4-2
Basic Parameters	4-2
Brady Mode	4-3
Lower Rate Limit (LRL)	4-4
Maximum Tracking Rate (MTR)	4-4
Maximum Sensor Rate (MSR)	4-6
Runaway Protection	4-7
Pulse Width	4-7
Amplitude	4-8
Sensitivity	4-8
Post-Therapy Pacing	4-11
Post-Shock Pacing Delay	4-12
Post-Therapy Period	4-12
Temporary Brady Pacing	4-12
Rate Adaptive Pacing and Sensor Trending	4-13
Rate Adaptive Pacing	4-13
Accelerometer	4-13
Sensor Trending	4-18
Atrial Tachy Response	4-19
ATR Mode Switch	4-19
Ventricular Rate Regulation (VRR)	4-22

Atrial Flutter Response (AFR).....	4-23
PMT Termination.....	4-23
Rate Enhancements	4-25
Rate Hysteresis.....	4-25
Rate Smoothing	4-25
Rate Smoothing Example Based on a Dual-Chamber Tracking Mode.....	4-27
Lead Configuration	4-28
AV Delay.....	4-28
Paced AV Delay	4-28
Sensed AV Delay	4-29
AV Search+	4-31
RYTHMIQ	4-32
Refractory	4-33
A-Refractory - PVARP.....	4-33
A Refractory - same chamber.....	4-35
RV-Refractory (RVRP).....	4-35
Cross-Chamber Blanking.....	4-36
Noise Response	4-39
Ventricular Tachy Sensing Interactions.....	4-41
SYSTEM DIAGNOSTICS.....	5-1
CHAPTER 5	
Summary Dialog.....	5-2
Battery Status.....	5-2
Capacitor Re-formation.....	5-6
Charge Time Measurement.....	5-6
Last Delivered Ventricular Shock.....	5-6
Leads Status.....	5-7
Lead Tests	5-10
Intrinsic Amplitude Test.....	5-11
Lead Impedance Test.....	5-11
Pace Threshold Test.....	5-12
PATIENT DIAGNOSTICS AND FOLLOW UP.....	6-1
CHAPTER 6	
Therapy History	6-2
Arrhythmia Logbook	6-2
Histograms	6-7
Counters.....	6-8
Ventricular Tachy Counters.....	6-8
Brady Counters.....	6-9
Trends	6-9
Post Implant features.....	6-12
Patient Triggered Monitor.....	6-12
Beeper Feature	6-14
Magnet Feature.....	6-15

ELECTROPHYSIOLOGIC TESTING.....	7-1
CHAPTER 7	
EP Test Features.....	7-2
Temporary EP Mode.....	7-2
EP Test Screen.....	7-2
Induction Methods.....	7-4
VFib Induction.....	7-4
Shock on T Induction.....	7-5
Backup Ventricular Pacing During Atrial EP Testing.....	7-5
Programmed Electrical Stimulation (PES).....	7-6
50 Hz/Manual Burst Pacing.....	7-7
Commanded Therapy Methods.....	7-8
Commanded Shock.....	7-8
Commanded ATP.....	7-9
PROGRAMMABLE OPTIONS.....	A-1
APPENDIX A	
PACEMAKER INTERACTION.....	B-1
APPENDIX B	
SYMBOLS ON PACKAGING.....	C-1
APPENDIX C	
Symbols on Packaging.....	C-1

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolete. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verze. Nepoužívat.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

USING THE PROGRAMMER/RECORDER/MONITOR

CHAPTER 1

This chapter contains the following topics:

- "ZOOM LATITUDE Programming System" on page 1-2
- "Software Terminology and Navigation" on page 1-2
- "Demonstration Mode" on page 1-7
- "Communicating with the Pulse Generator" on page 1-7
- "Indications-Based Programming (IBP)" on page 1-11
- "Manual Programming" on page 1-13
- "DIVERT THERAPY" on page 1-13
- "STAT SHOCK" on page 1-14
- "STAT PACE" on page 1-14
- "Data Management" on page 1-15
- "Safety Mode" on page 1-17

ZOOM LATITUDE PROGRAMMING SYSTEM

The ZOOM LATITUDE Programming System is the external portion of the pulse generator system and includes:

- Model 3120 Programmer/Recorder/Monitor (PRM)
- Model 2868 ZOOMVIEW Software Application
- Model 6577 Accessory Telemetry Wand

The ZOOMVIEW software provides advanced device programming and patient monitoring technology. It was designed with the intent to:

- Enhance device programming capability
- Improve patient and device monitoring performance
- Simplify and expedite programming and monitoring tasks

You can use the PRM system to do the following:

- Interrogate the pulse generator
- Program the pulse generator to provide a variety of therapy options
- Access the pulse generator's diagnostic features
- Perform noninvasive diagnostic testing
- Access therapy history data
- Access an interactive Demonstration Mode or Patient Data Mode without the presence of a pulse generator
- Print patient data including pulse generator therapy options and therapy history data
- Save patient data

You can program the pulse generator using two methods: automatically using IBP or manually.

For more detailed information about using the PRM, refer to the PRM Operator's Manual.

SOFTWARE TERMINOLOGY AND NAVIGATION

This section provides an overview of the PRM system.

Main Screen

The main PRM screen is shown below, followed by a description of the components (Figure 1-1 on page 1-3).

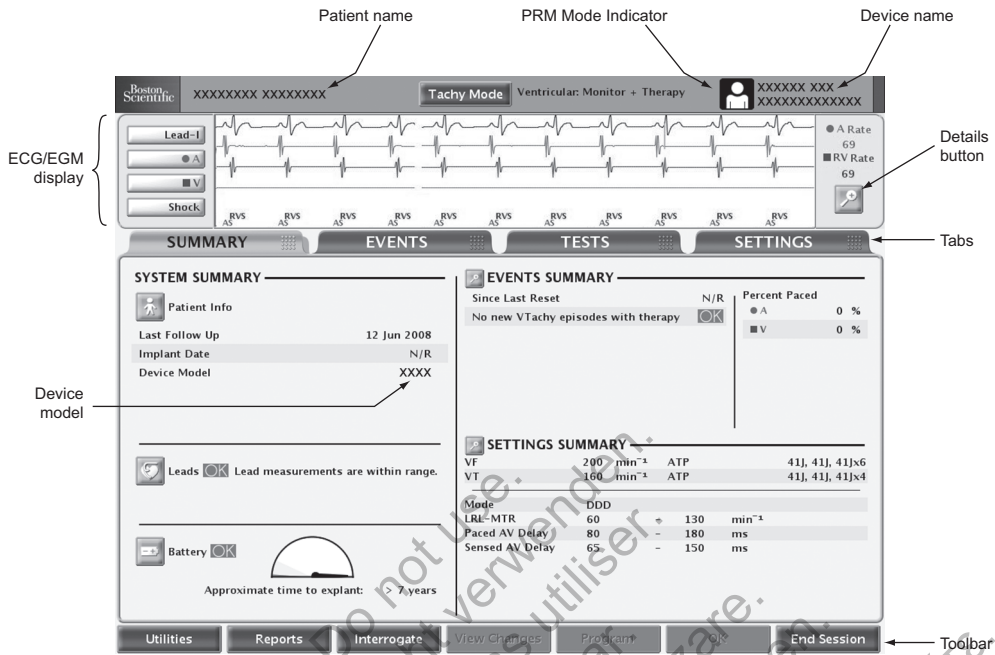


Figure 1-1. Main Screen

PRM Mode Indicator

The PRM Mode Indicator displays at the top of the screen to identify the current PRM operational mode.



Patient—indicates that the PRM is displaying data obtained by communicating with a device.



Patient Data—indicates that the PRM is displaying stored patient data.



Demo Mode—indicates that the PRM is displaying sample data and operating in demonstration mode.

ECG/EGM Display

The ECG area of the screen shows real-time status information about the patient and the pulse generator that can be useful in evaluating system performance. The following types of traces can be selected:

- Surface ECGs are transmitted from body surface lead electrodes that are connected to the PRM, and can be displayed without interrogating the pulse generator.

- Real-time EGMs are transmitted from the pace/sense or shocking electrodes, and are often used to evaluate lead system integrity and help identify faults such as lead fractures, insulation breaks, or dislodgments.

Real-time EGMs can only be displayed upon interrogation of the pulse generator. Because they rely on ZIP or wanded telemetry, they are susceptible to radio frequency interference. Significant interference may cause a break or drop-out of real-time EGMs ("ZIP Telemetry Security" on page 1-9).

NOTE: *If the PRM is left idle for 15 minutes (or 28 minutes if the pulse generator was in Storage Mode at interrogation) real-time EGMs are shut off. The PRM provides a dialog box allowing real-time EGMs to be restored.*

NOTE: *In the presence of telemetry interference, the real-time intracardiac EGM traces and markers may become misaligned from the real-time surface ECG traces. When the telemetry link has improved, re-select any of the intracardiac EGM traces to cause re-initialization.*

You can select the Details button to enlarge the ECG/EGM screen. The following options are available:

- Show Device Markers—displays annotated event markers, which identify certain intrinsic cardiac and device-related events, and provide information such as sensed/paced events, decision of detection criteria, and therapy delivery
- Enable Surface Filter—minimizes noise on the surface ECG
- Display Pacing Spikes—shows detected pacing spikes, annotated by a marker on the surface ECG waveform

You can print real-time EGMs, which include annotated event markers, by performing the following steps:

1. Press one of the print speed keys on the PRM (e.g., speed key 25) to begin printing.
2. Press the 0 (zero) speed key to stop printing.
3. Press the paper-feed key to fully eject the last printed sheet.

You can print definitions of the annotated markers by pressing the calibration key while the EGM is printing. Alternatively you can print a full report containing the definitions of all of the annotated markers by performing the following steps:

1. From the toolbar, click the Reports button. The Reports window displays.
2. Select the Marker Legend checkbox.
3. Click the Print button. The Marker Legend Report is sent to the printer.

Toolbar

The toolbar allows you to perform the following tasks:

- Select system utilities
- Generate reports
- Interrogate and program the pulse generator
- View pending or programmed changes
- View attentions and warnings
- End your PRM session

Tabs

Tabs allow you to select PRM tasks, such as viewing summary data or programming device settings. Selecting a tab displays the associated screen. Many screens contain additional tabs, which allow you to access more detailed settings and information.

Buttons

Buttons are located on screens and dialogs throughout the application. Buttons allow you to perform various tasks, including:

- Obtain detailed information
- View setting details
- Set programmable values
- Load initial values

When a button selection opens a window in front of the Main Screen, a Close button displays in the upper-right corner of the window to allow you to close the window and return to the Main Screen.

Icons

Icons are graphic elements that, when selected, may initiate an activity, display lists or options, or change the information displayed.



Details—opens a window containing detailed information.



Patient—opens a window with patient information details.



Leads—opens a window with details on leads.



Battery—opens a window with details on the pulse generator battery.



Check—indicates that an option is selected.



Event—indicates that an event has occurred. When you view the Trends timeline on the Events tab, event icons display wherever events have occurred. Selecting an events icon displays details about the event.



Information—indicates information that is provided for reference.

Action Icons



Run—causes the programmer to perform an action.



Hold—causes the programmer to pause an action.



Continue—causes the programmer to continue an action.

Slider Icons



Horizontal Slider—indicates that a slider object can be clicked and dragged left or right.



Vertical Slider—indicates that a slider object can be clicked and dragged up or down.

Sort Icons



Sort Ascending—indicates that Ascending sort is currently selected on a table column sort button. (e.g., 1, 2, 3, 4, 5)



Sort Descending—indicates that Descending sort is currently selected on a table column sort button. (e.g., 5, 4, 3, 2, 1)

Increment and Decrement Icons



Increment—indicates that an associated value can be incremented.



Decrement—indicates that an associated value can be decremented.

Scroll Icons



Scroll Left—indicates that an associated item can be scrolled left.



Scroll Right—indicates that an associated item can be scrolled right.



Scroll Up—indicates that an associated item can be scrolled up.



Scroll Down—indicates that an associated item can be scrolled down.




Common Objects

Common objects such as status bars, scroll bars, menus, and dialogs are used throughout the application. These operate similarly to the objects found in web browsers and other computer applications.

Use of Color

Colors and symbols are used to highlight buttons, icons, and other objects, as well as certain types of information. The use of specific color conventions and symbols is intended to provide a more consistent user experience and simplify programming. Refer to the table below to understand how colors and symbols are used on the PRM screens (Table 1-1 on page 1-7).

Table 1-1. PRM color conventions

Color	Meaning	Examples	Symbol
Red	Indicates warning conditions	The selected parameter value is not allowed; click the red warning button to open the Parameter Interactions screen, which provides information about corrective action.	
		Device and patient diagnostic information that requires serious consideration.	
Yellow	Indicates conditions requiring your attention	The selected parameter value is allowed, but not recommended; click the yellow attention button to open the Parameter Interactions screen, which provides information about corrective action.	
		Device and patient diagnostic information that should be addressed.	
Green	Indicates acceptable changes or conditions	The selected parameter value is allowed, but is still pending.	
		There is no device or patient diagnostic information requiring your specific attention.	
White	Indicates the value that is currently programmed		

DEMONSTRATION MODE

The PRM includes a Demonstration Mode feature, which enables the PRM to be used as a self-teaching tool. When selected, this mode allows you to practice PRM screen navigation without interrogating a pulse generator. You can use Demonstration Mode to familiarize yourself with many of the specific screen sequences that will display when interrogating or programming a specific pulse generator. You can also use Demonstration Mode to examine available features, parameters, and information.

To access Demonstration Mode, select the appropriate PG from the Select PG screen, and then select Demo from the Select PG Mode dialog. When the PRM is operating in Demonstration Mode, the PRM Mode Indicator displays the Demo Mode icon. The pulse generator cannot be programmed when the PRM is operating in Demonstration Mode. Exit the Demonstration Mode before attempting to interrogate or program the pulse generator.

COMMUNICATING WITH THE PULSE GENERATOR

The PRM communicates with the pulse generator using a telemetry wand.

After initiating communication with the wand, some pulse generator models can use wandless ZIP telemetry (two-way RF communication) to interface with the PRM.

Telemetry is required to:

- Direct commands from the PRM system, such as:
 - INTERROGATE
 - PROGRAM
 - STAT SHOCK
 - STAT PACE
 - DIVERT THERAPY
- Modify device parameter settings
- Conduct EP testing
- Conduct diagnostic tests including the following:
 - Pacing impedance tests
 - Pacing threshold tests
 - Intrinsic amplitude tests
- Perform manual capacitor re-form

ZIP Telemetry

ZIP telemetry is a wandless, two-way RF communication option that allows the PRM system to communicate with some pulse generator models. When a wanded telemetry session is initiated, the PRM checks the pulse generator's telemetry capability. If the PRM detects a pulse generator with ZIP telemetry capability, a message will display indicating that ZIP telemetry is available and the wand can be removed. Otherwise, the session will continue with wanded telemetry.

ZIP telemetry offers the following advantages over traditional wanded telemetry:

- The faster data transmission speed means less time is required for device interrogation
- Data transmission over a longer distance (within 3 m [10 ft]) minimizes the need to keep the wand in the sterile field during implant, which may reduce the risk of infection
- Continuous telemetry is possible during the entire implant procedure, allowing monitoring of pulse generator performance and lead integrity during implant

Regardless of whether ZIP telemetry is being used, wanded communication is still available.

Starting a Wanded Telemetry Session

Follow this procedure to begin a wanded telemetry communication session:

1. Make sure the telemetry wand is connected to the PRM system and is available throughout the session.
2. Position the wand over the pulse generator at a distance not greater than 6 cm (2.4 inches).
3. Use the PRM to Interrogate the pulse generator.
4. Retain the wand position whenever communication is required.

Starting a ZIP Telemetry Session

Follow this procedure to begin a ZIP telemetry communication session:

1. Start a wanded telemetry session. Verify that the wand cord is within reach of the pulse generator to enable the use of wanded telemetry should it become necessary.
2. Keep the telemetry wand in position until either a message appears, indicating that the telemetry wand may be removed from proximity of the pulse generator, or the ZIP telemetry light illuminates on the PRM system.

Ending a Telemetry Session

Select the End Session button to quit a telemetry session and return to the startup screen. You can choose to end the session or return to the current session. Upon ending a session, the PRM system terminates all communication with the pulse generator.

ZIP Telemetry Security

The pulse generator is a compliant low-power transceiver. The pulse generator can only be interrogated or programmed by RF signals that employ the proprietary ZIP telemetry protocol. The pulse generator verifies that it is communicating with a ZOOMVIEW system before responding to any RF signals. The pulse generator stores, transmits, and receives individually identifiable health information in an encrypted format.

ZIP telemetry is possible when all of the following conditions are met:

- ZIP telemetry for the PRM is enabled
- The pulse generator has RF communication capabilities
- The ZIP telemetry channel is available for use
- The pulse generator is within range of the PRM system
- The pulse generator has not reached Explant; note that a total of 1.5 hours of ZIP telemetry will be available after the pulse generator reaches Explant
- The pulse generator battery capacity is not depleted

In order to meet local communications rules and regulations, ZIP telemetry should not be used when the pulse generator is outside its normal operating temperature of 20°C–43°C (68°F–109°F).

Communication is supported between one PRM and one pulse generator at a time. If there is a PRM–pulse generator communication session already occurring in the vicinity, a second session will not be allowed to start; wanded communication will be necessary in this case.

The PRM notifies you if ZIP telemetry is unavailable because of other sessions already in progress.

RF signals in the same frequency band used by the system may interfere with ZIP telemetry communication. These interfering signals include:

- Signals from other pulse generator/PRM system RF communication sessions after the maximum number of independent sessions has been reached. Other nearby pulse generators and PRMs using ZIP telemetry may prevent ZIP telemetry communication.
- Interference from other RF sources.

CAUTION: RF signals from devices that operate at frequencies near that of the pulse generator may interrupt ZIP telemetry while interrogating or programming the pulse generator. This RF interference can be reduced by increasing the distance between the interfering device and the PRM and pulse generator. Examples of devices that may cause interference include:

- Cordless phone handsets or base stations
- Certain patient monitoring systems

Radio frequency interference may temporarily disrupt ZIP telemetry communication. The PRM will normally reestablish ZIP communication when the RF interference ends or subsides. Because continued RF interference may prevent ZIP telemetry communication, the system is designed to use wanded telemetry when ZIP telemetry is not available.

If ZIP telemetry is not available, wanded telemetry communication with the PRM can be established. The system provides the following feedback to indicate that ZIP telemetry is not available:

- The ZIP telemetry indicator light on the PRM turns off
- If event markers and/or EGMs are activated, transmission of the event markers and/or EGMs is interrupted
- If a command or other action has been requested, the PRM displays a notification indicating the wand should be placed in range of the pulse generator

ZIP telemetry operates consistently with wanded telemetry—no programming step can be completed unless the entire programming command has been received and confirmed by the pulse generator.

The pulse generator cannot be misprogrammed as a result of interrupted ZIP telemetry. Interruptions of ZIP telemetry may be caused by RF signals that operate at frequencies near that of the pulse generator and are strong enough to compete with the ZIP telemetry link between the pulse generator and the PRM. Significant interference may result in a break or drop-outs of real-time EGMs. If commands are interrupted, the PRM displays a message to place the wand on the pulse generator. Repeated displays of this message may indicate the presence of intermittent interference. These situations can be resolved by repositioning the PRM or using standard wanded telemetry. There will be no interruption of device functionality or therapy during this period.

NOTE: When both ZIP and wanded telemetry are being used (for example, switching from ZIP to wanded because of the presence of interference), the pulse generator will communicate with the programmer by ZIP telemetry when possible. If wanded telemetry only is desired, set the Communication Mode (accessed via the Utilities button) to use the wand for all telemetry.

NOTE: To conserve battery longevity, a ZIP telemetry session will be terminated if the pulse generator completely loses communication with the PRM for a continuous period of one hour (or 73 minutes if the device was in Storage Mode at interrogation). Wanded telemetry must be used to re-establish communication with the pulse generator after this period has elapsed.

NOTE: The PRM operates on a country-specific frequency range. The PRM determines the ZIP frequency range that the pulse generator uses based on the specific device model. If the PRM and pulse generator ZIP frequency ranges do not match, it indicates that the patient has traveled outside the country in which the pulse generator was implanted. The PRM will display a message indicating that ZIP telemetry cannot be used; however, the patient's pulse generator can be interrogated by using the wand. If out-of-country interrogation is needed, contact Boston Scientific using the information on the back cover of this manual.

Considerations for Reducing Interference

Increasing the distance from the source of interfering signals may enable the use of the ZIP telemetry channel. A minimum distance of 14 m (45 ft) is recommended between the source of interference (having an average output of 50 mW or less) and both the pulse generator and PRM.

Repositioning the PRM antenna or repositioning the PRM may improve ZIP telemetry performance. If ZIP telemetry performance is not satisfactory, the option of using wanded telemetry is available.

Depending on the environment and PRM orientation relative to the pulse generator, the system is capable of maintaining ZIP telemetry communication at distances up to 12 m (40 ft). For optimum ZIP telemetry communication, position the PRM antenna within 3 m (10 ft) of the pulse generator and remove any obstruction between the PRM and the pulse generator.

Positioning the PRM at least 1 m (3 ft) away from walls or metal objects and ensuring the pulse generator (prior to implant) is not in direct contact with any metal objects may reduce signal reflection and/or signal blocking.

Ensuring there are no obstructions (e.g., equipment, metal furniture, people, or walls) between the PRM and pulse generator may improve signal quality. Personnel or objects that momentarily move between the PRM and pulse generator during ZIP telemetry may temporarily interrupt communication, but will not affect device functionality or therapy.

Checking the time required to complete an interrogation after ZIP telemetry is established can provide an indication of whether interference is present. If an interrogation using ZIP telemetry takes less than 20 seconds, the current environment is likely free of interference. Interrogation times longer than 20 seconds (or short intervals of EGM drop-outs) indicate that interference may be present.

INDICATIONS-BASED PROGRAMMING (IBP)

IBP is a tool that provides specific programming recommendations based on the patient's clinical needs and primary indications.

IBP is a clinical approach to programming that was developed based on physician consultation and case studies. The intent of IBP is to enhance patient outcomes and save time by providing base programming recommendations that you can customize as needed. IBP systematically presents the specific features intended for use with the clinical conditions you identify in the IBP user interface, and allows you to take maximum advantage of the pulse generator's capabilities.

IBP can be accessed from the Settings tab on the main application screen (Figure 1-2 on page 1-11).

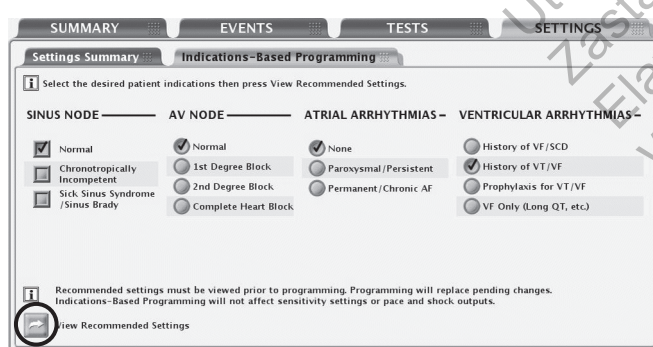


Figure 1-2. Indications-based Programming screen

Indications are clustered in general categories as illustrated above. The intent for each category of indications is described below:

- Sinus Node
 - If Normal is selected, the intent is to allow intrinsic atrial events and provide RV pacing when necessary.
 - If Chronotropically Incompetent is selected, the intent is to provide rate-adaptive pacing.
 - If Sick Sinus Syndrome is selected, the intent is to provide atrial pacing support.
- AV Node
 - If Normal or 1st Degree Block is selected, the intent is to allow intrinsic AV conduction and provide RV pacing when necessary.
 - If 2nd Degree Block is selected, the intent is to allow intrinsic AV conduction and provide AV sequential pacing when conduction is not present.
 - If Complete Heart Block is selected, the intent is to provide AV sequential pacing.

NOTE: The selected settings for AF and Sinus Node may affect the suggested value for the Normal/1st Degree Block setting of AV Node.

- Atrial Arrhythmias
 - If Paroxysmal/Persistent is selected, the intent is to avoid tracking atrial arrhythmias by using ATR Mode Switch when a dual-chamber pacing mode is suggested.
 - If Permanent/Chronic AF is selected, the intent is to provide rate-adaptive RV pacing.
- Ventricular Arrhythmias
 - When History of VF/SCD or Prophylaxis for VT/VF is selected, a 2-zone configuration with the following rate thresholds and therapies is provided:
 - 180 min⁻¹ for the VF zone with QUICK CONVERT ATP and Maximum Energy Shocks enabled
 - 160 min⁻¹ for the VT zone with therapy disabled (Monitor Only)
 - When History of VT/VF is selected, a 2-zone configuration with the following rate thresholds and therapies is provided:
 - 200 min⁻¹ for the VF zone with QUICK CONVERT ATP and Maximum Energy Shocks enabled
 - 160 min⁻¹ for the VT zone with ATP and Maximum Energy Shocks enabled
 - Rhythm ID enabled
 - When VF Only is selected, the intent is to provide a single VF zone of 220 min⁻¹ with only Maximum Energy Shocks enabled.

After choosing appropriate patient indications, select the View Recommended Settings button to view a summary of the programming recommendations (Figure 1-3 on page 1-13).

NOTE: You must view the recommended settings before you can program them. Selecting the View Recommended Settings button allows you to view the settings that are recommended based on the indications that you selected. Viewing the recommended settings does not overwrite any pending (i.e., not yet programmed) parameter changes. You must choose to program or reject the recommended settings after viewing them. If you choose to reject the recommended settings, all of your pending settings will be restored. If you choose to program the recommended settings, any pending parameter changes will be overwritten, with the exception of sensitivity and therapy outputs, which are independent of IBP.

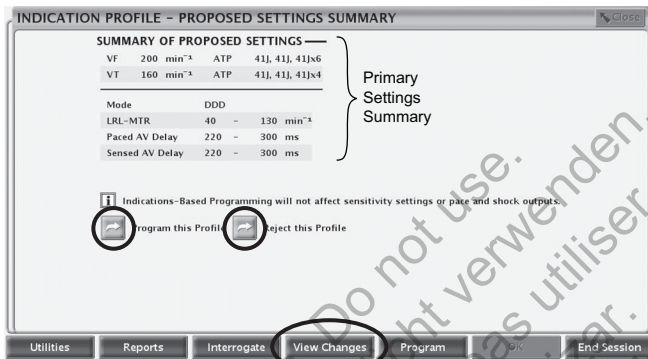


Figure 1-3. Proposed Settings Summary screen

The Proposed Settings Summary screen displays the primary programming recommendations. Additional details about all changed parameters are available by selecting the View Changes button from the toolbar. You have the option to program the proposed settings or reject them, as long as telemetry is still engaged:

- Program—select the Program this Profile button to accept the proposed settings.
- Reject—select the Reject this Profile button to reject the proposed settings; this action will return you to the main IBP screen with no changes made.

MANUAL PROGRAMMING

Manual programming controls such as sliders and menus are available to allow you to individually adjust pulse generator program settings.

Manual programming controls are located on the Settings Summary tab, which can be accessed from the Settings tab or by selecting the Settings Summary button on the Summary tab. Refer to other feature descriptions in this manual for specific manual programming information and instructions. Refer to "Programmable Options" on page A-1 for detailed listings of available settings.

DIVERT THERAPY

When the pulse generator is charging to deliver a shock, the shock delivery may be diverted from the patient. If diverted, the shock does not count as one of the total number of shocks that may be delivered during an episode. If redetection occurs and more shock therapy is required, and if more shocks are available in the therapy prescription, the pulse generator will charge again to deliver subsequent shocks.

Also, the DIVERT THERAPY key can be pressed to divert ATP therapy in midburst. If redetection occurs, the ATP scheme will not be used again and the next programmed therapy in the sequence will be initiated.

1. If you are not already in a session, position the telemetry wand within range of the pulse generator and start a communication session.
2. Press the DIVERT THERAPY key. A message window will appear indicating that a divert attempt is being made.
3. If using wanded telemetry, maintain the wand position until the message window disappears, indicating the shock has been diverted. Prematurely removing the wand (breaking the telemetry link) may allow the pulse generator to continue charging and to deliver the shock.

NOTE: *There is a 500 ms delay between the end of charging and shock delivery designed to provide a minimum period for the DIVERT THERAPY command. After this time, pressing DIVERT THERAPY may not divert the shock.*

The DIVERT THERAPY key can be used to terminate any diagnostic test in progress, as well as Electrocautery Protection Mode (maintain the telemetry wand position until the divert function is complete to avoid interruption to the divert command).

STAT SHOCK

A nonprogrammable, maximum-output STAT SHOCK can be delivered to the patient at any time during a communication session. The STAT SHOCK can be delivered when the pulse generator's Tachy Mode is programmed to any mode. This function does not affect the programmed shock sequences (lower-energy shocks can be delivered following a STAT SHOCK) and does not count as one of the total number of shocks in a therapy sequence for a given episode. The output of the STAT SHOCK is at the maximum-output energy and at the programmed polarity and waveform; STAT SHOCK is always committed regardless of programmed parameters.

1. If you are not already in a session, position the telemetry wand within range of the pulse generator.
2. Press the STAT SHOCK key. A message window appears with information about the shock and instructions to initiate the shock.
3. To initiate the shock, press the STAT SHOCK key again. A different message window appears indicating that STAT SHOCK is in process. When the shock has been delivered, the message window disappears.
4. Subsequent high-energy STAT SHOCKS may be delivered by repeating the previous steps.

NOTE: *The STAT SHOCK may be diverted using the DIVERT THERAPY key.*

NOTE: *Following STAT SHOCK delivery, if the Tachy Mode is programmed to Monitor Only or Monitor + Therapy, post-shock redetection is initiated (initial detection criteria and enhancements are not used). If the Tachy Mode is programmed to Monitor + Therapy and redetection determines that further therapy is required, the programmed sequence of therapy will be resumed or initiated, including ATP and/or low-energy shocks.*

STAT PACE

Emergency bradycardia pacing using the STAT PACE command sets the bradycardia operation to parameters intended to ensure capture.

1. If you are not already in a session, position the telemetry wand within range of the pulse generator.
2. Press the STAT PACE key. A message window displays the STAT PACE values.

3. Press the STAT PACE key a second time. A message indicates that STAT PACE is being performed, followed by the STAT PACE values.
4. Select the Close button on the message window.
5. To stop STAT PACE, reprogram the pulse generator.

NOTE: STAT PACE will terminate Electrocautery Protection Mode.

CAUTION: When a pulse generator is programmed to STAT PACE settings, it will continue to pace at the high-energy STAT PACE values if it is not reprogrammed. The use of STAT PACE parameters will likely decrease device longevity.

The STAT PACE parameter values are listed below (Table 1-2 on page 1-15).

Table 1-2. STAT PACE Parameter Values

Parameter	Values
Mode	VVI
Lower Rate Limit	60 min ⁻¹
Interval	1000 ms
Amplitude	7.5 V
Pulse Width	1.0 ms
Paced Refractory	250 ms
Post-shock Pacing	VVI

DATA MANAGEMENT

The PRM system allows you to view, print, store, or retrieve patient and pulse generator data. This section describes the PRM data management capabilities.

Patient Information

Information about the patient can be stored in pulse generator memory. The information is accessible from the Summary screen by selecting the Patient icon. This information includes, but is not limited to, the following:

- Patient and physician data
- Pulse generator serial number
- Implant date
- Lead configurations
- Implant test measurements

The information can be retrieved at any time by interrogating the pulse generator and viewing it on the PRM screen or printing it as a report.

Data Storage

The PRM system allows you to save pulse generator data to the PRM hard drive or a removable floppy data disk. Data saved to the PRM can also be transferred to a removable USB pen drive.

Saved pulse generator data includes, but is not limited to, the following:

- Therapy history

- Programmed parameter values
- Trending values
- Histogram paced/sensed counters

Select the Utilities button, and then select the Data Storage tab to access the following options:

- Read Disk—allows you to retrieve saved pulse generator data from a floppy disk.
- Save All—allows you to save pulse generator data to either a floppy disk (disk must be inserted) or the PRM hard drive (if no floppy disk is detected). Data saved to a floppy disk can be retrieved using the Read Disk option described above. Data saved to the PRM can be read, deleted, or exported to a USB pen drive from the PRM startup screen. Reports are available in PDF format. Refer to the PRM Operator's Manual for more information.

NOTE: While the data is being saved, a message on the right-hand side of the System Status screen indicates where the data is being saved.

Consider the following when storing and retrieving pulse generator data:

- No more than 400 unique patient records may be saved to the PRM. When a pulse generator is interrogated, the PRM evaluates if there is already a record on file for this pulse generator, or if a new record will need to be created. If a new record is needed, and the PRM is at the 400 record capacity, the oldest record on file will be deleted to create space for the new patient record.
- When performing multiple patient checkups, be sure to start a new session for each patient.
- Be sure to save all pulse generator data to either a floppy disk or USB pen drive before returning a PRM to Boston Scientific, as all patient and pulse generator data will be erased from the PRM when it is returned.
- To protect patient privacy, pulse generator data is encrypted before it is transferred to a USB pen drive.

Device Memory

The Device Memory utility allows you to retrieve, save, and print pulse generator memory data, which is intended for use by a Boston Scientific representative for clinical and troubleshooting purposes. This utility should only be used when directed by a Boston Scientific representative. Digital media with device memory data contains protected health information and therefore should be handled in accordance with applicable privacy and security policies and regulations.

NOTE: Use the Data Storage tab to access pulse generator data for clinician use ("Data Storage" on page 1-15).

Print

You can print PRM reports by using the internal printer, or by connecting to an external printer. To print a report, select the Reports button. Then select the report you wish to print from the following categories:

- Follow-up reports
- Episode reports
- Other reports (includes device settings, patient data, and other information)

SAFETY MODE

The pulse generator is equipped with dedicated Safety Core hardware that is intended to provide life-sustaining therapy if certain nonrecoverable or repeat fault conditions occur and cause a system reset. These types of faults indicate a loss of component integrity in the pulse generator's central processing unit (CPU), including the microprocessor, program code, and system memory. Using minimal hardware (i.e., unipolar lead configuration), Safety Core operates independently and acts as a backup to these components.

Safety Core also monitors the device during normal pacing; if normal pacing does not occur, Safety Core delivers an escape pace, and a system reset is initiated.

If the pulse generator experiences three resets within approximately 48 hours, the device reverts to Safety Mode and device replacement should be considered. The following will also occur:

- The pulse generator will beep 16 times every 6 hours. This beeping is disabled once the device has been interrogated with a PRM.
- ZIP telemetry is unavailable for communicating with the PRM when Safety Mode is active; wanded telemetry must be used instead.
- LATITUDE remote monitoring is unavailable.
- Upon interrogation, a warning screen is displayed indicating that the pulse generator is in Safety Mode, and directing you to contact Boston Scientific.

Backup Pacemaker

Safety Mode provides ventricular pacing, with the following fixed parameters:

- Brady Mode—VVI
- LRL—72.5 min⁻¹
- Pulse Amplitude—5.0 V
- Pulse Width—1.0 ms
- RV Refractory Period (RVRP)—250 ms
- RV Sensitivity—AGC 0.25 mV
- RV lead configuration—Unipolar
- Noise Response—VOO
- Post-Shock Pacing Delay—3 sec

Backup Defibrillator

When Safety Mode is activated, Tachy Mode is automatically programmed to Monitor + Therapy to provide single-zone tachyarrhythmia detection and therapy. Tachy Mode may still be programmed to Off while in Safety Mode.

NOTE: *If additional faults are detected while in Safety Mode, tachyarrhythmia therapy will be disabled.*

While in Safety Mode, the tachyarrhythmia therapy is limited to 5 maximum-energy committed shocks per episode.

Tachyarrhythmia detection and therapy parameters are fixed as follows:

- VF Rate Threshold—165 ppm

- Duration—1 sec
- Shock polarity—initial
- Shock waveform—biphasic
- Shock vector—V-TRIAD

Magnet application will immediately inhibit therapy, although charging may continue. After the magnet has been applied for 1 second, therapy is diverted and detection is inhibited. The magnet must then be removed for 2 seconds in order to allow detection to continue. Also, Safety Mode disables normal beeping behavior following magnet application.

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolete. Ne pas utiliser.
Versión obsoleta. No utilizar.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

TACHYARRHYTHMIA DETECTION

CHAPTER 2

This chapter contains the following topics:

- "Device Mode" on page 2-2
- "Rate Sensing" on page 2-3
- "Ventricular Detection" on page 2-5

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

DEVICE MODE

The Device Mode allows you to program the device to provide the type of therapy and detection desired.

Ventricular Tachy Mode

The Ventricular Tachy Mode controls the availability of the detection and therapy functions in the ventricle (Table 2-1 on page 2-2).

You can program the Ventricular Tachy Mode to the following modes:

- Off—disables ventricular tachyarrhythmia detection and automatic ventricular therapy delivery. This mode is useful during implant or explant, when connecting the leads to or disconnecting them from the pulse generator.
- Monitor Only—enables ventricular tachyarrhythmia detection and episode storage, but does not automatically deliver therapy to the patient. This mode is useful in controlled environments, such as during EP testing, exercise testing, and immediately postoperative, where alternate therapy (e.g., external defibrillation) is available.
- Monitor + Therapy—enables the full range of ventricular detection and ventricular therapy options.

Table 2-1. Device feature availability in the Ventricular Tachy Mode settings

Device features	Ventricular Tachy Mode		
	Off	Monitor Only	Monitor + Therapy
Rate sensing	X ^a	X	X
Bradycardia pacing	X	X	X
Ventricular detection/therapy history	X ^b	X	X
STAT SHOCK	X	X	X
STAT PACE	X	X	X
Real-time annotated EGMs	X	X	X
Ventricular tachyarrhythmia detection		X	X
Commanded ventricular ATP		X	X ^c
Commanded ventricular shock		X	X
Ventricular EP test		X ^d	X ^d
Automatic ventricular tachyarrhythmia therapy			X

- a. In order to enable ventricular sensing when the Ventricular Tachy Mode is programmed to Off, you must program the Brady Mode to a mode with ventricular sensing.
- b. While programmed to Off Mode, the pulse generator will store only STAT SHOCK in history.
- c. When the ventricular tachy mode is Monitor + Therapy, the EP Temp V Mode must be programmed to Monitor Only in order to use the commanded ventricular ATP.
- d. Not all forms of EP Tests are available in this mode.

Electrocautery Protection Mode

Electrocautery Protection Mode provides asynchronous pacing at the programmed outputs and LRL. Tachyarrhythmia detection and therapy features are deactivated.

When Electrocautery Protection is enabled, the Brady Mode switches to an XO0 mode (where X is determined by the programmed Brady Mode). Other pacing parameters remain at the programmed settings (including pacing output). If Brady Mode is Off prior to enabling Electrocautery Protection, it will remain Off during Electrocautery Protection. Once enabled, Electrocautery Protection does not require constant telemetry to remain active.

After cancelling Electrocautery Protection, the following modes will revert to the previously programmed settings:

- Ventricular Tachy Mode
- Brady Mode

After attempting to enable Electrocautery Protection Mode, refer to the message on the PRM screen confirming that Electrocautery Protection is active.

Except for STAT SHOCK and STAT PACE, no commanded therapies, inductions, diagnostic tests, or printing of reports will be allowed while Electrocautery Protection is enabled.

Application of a magnet while the device is in Electrocautery Protection has no effect on Tachy Mode.

To enable and disable Electrocautery Protection Mode, perform the following steps:

1. Select the Tachy Mode button from the top of the PRM screen.
2. Select the check box to Enable Electrocautery Protection.
3. Select the Apply Changes button to enable Electrocautery Protection Mode. A dialog window will appear, indicating that Electrocautery Protection is active.
4. Select the Cancel Electrocautery Protection button on the dialog window to return the device to the previously programmed mode. Electrocautery Protection can also be cancelled by pressing the STAT SHOCK, STAT PACE, or Divert Therapy key on the PRM.

RATE SENSING

Rate sensing is critical to all detection decisions. The pulse generator relies on the following to determine cardiac cycle length:

- Bipolar electrodes in the atrium and right ventricle.
- Automatic gain-controlled sensing circuit for rate sensing. This circuit ensures proper rate sensing by compensating for changing or diminished signal amplitudes.

Calculating Rates and Refractory Periods

The pulse generator evaluates rate on an interval-by-interval basis. Following a sensed depolarization, a cycle length is measured and compared to the programmed detection parameters.

The pulse generator uses refractory periods following paced and sensed intrinsic events; intrinsic events that fall within these periods are ignored for detection purposes. The refractory periods, together with noise windows, may prevent the sensing of nonphysiologic signals and the potential delivery of unwanted therapy. The nonprogrammable refractory periods are as follows:

- 85 ms atrial refractory following an atrial sensed event
- 150 ms atrial refractory following an atrial pace in DDD(R) and DDI(R) modes
- 135 ms RV refractory following an RV sensed event
- 135 ms refractory following a capacitor charge (sensing is ignored in all chambers)
- 500 ms refractory following shock delivery (sensing is ignored in all chambers)

Ventricular Rate Thresholds and Zones

The pulse generator compares each sensed RV cardiac cycle interval against the programmed Ventricular Tachyarrhythmia Rate Threshold.

A Ventricular Tachyarrhythmia Zone is a range of heart rates defined by at least one programmed Ventricular Tachyarrhythmia Rate Threshold. You can program from 1 to 3 Ventricular Tachyarrhythmia Zones, each of which can be treated by a separate therapy prescription (Table 2-2 on page 2-4, Figure 2-1 on page 2-4).

Table 2-2. Nominal values for Ventricular Rate Threshold configurations

Ventricular Zone Configuration	VT-1 Zone	VT Zone	VF Zone
1 Zone	--	--	200 min ⁻¹
2 Zones	--	160 min ⁻¹	200 min ⁻¹
3 Zones	140 min ⁻¹	160 min ⁻¹	200 min ⁻¹

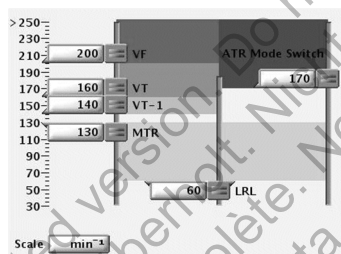


Figure 2-1. Ventricular Tachy Detection settings

- Rate thresholds in adjacent zones must differ by at least 20 min⁻¹
- The lowest Ventricular Tachyarrhythmia Rate Threshold must be at least 5 min⁻¹ higher than the MTR, MSR, and the MPR
- The lowest Ventricular Tachyarrhythmia Rate Threshold must be at least 15 min⁻¹ higher than the LRL

Use of Atrial Information

The atrial rate may be used to:

- Inhibit ventricular therapy in the presence of atrial fibrillation or atrial flutter
- Bypass ventricular therapy inhibitors if the ventricular rate is faster than the atrial rate

The pulse generator will respond to atrial sensing regardless of whether an atrial lead is implanted.

There may be clinical situations in which atrial lead information is not useful (e.g., chronic atrial fibrillation, faulty or dislodged atrial lead, plugged atrial port).

CAUTION: If an atrial lead is not implanted (port is plugged instead), or an atrial lead is abandoned but remains connected to the header, device programming should be consistent with the number and type of leads actually in use.

If an atrial lead will not be used, use the following programming recommendations to ensure appropriate device behavior:

- Program the atrial lead to Off, to prevent atrial sensing and minimize accrual of atrial counters.

NOTE: An atrial EP test should not be performed if the atrial lead is programmed to Off.

- Program the Brady Mode to VVI or VVI(R), to prevent atrial pacing and ensure that atrial information is not used to drive brady pacing.
- Program the following ventricular detection enhancements to Off, to ensure therapy decisions are not based on atrial measurements:
 - Initial and Post-Shock V Rate > A Rate (for Onset/Stability)
 - Initial and Post-Shock AFib Rate Threshold (for Onset/Stability)
 - Atrial Tachyarrhythmia Discrimination (for Rhythm ID)

NOTE: You should also review and, if necessary, adjust Stability settings.

- Program the Atrial Intrinsic Amplitude and Atrial Impedance daily lead measurements to Off to disable atrial diagnostics (e.g., atrial amplitude and impedance).
- During follow-up visits, consider deselecting the atrial real-time EGM:

If an atrial lead is used in the future, these programming adjustments should be reevaluated, and the pulse generator should be programmed appropriately for use with an atrial lead.

VENTRICULAR DETECTION

Ventricular detection consists of the following components:

- Initial ventricular detection
- Reconfirmation/committed shock
- Redetection and post-shock detection

Initial ventricular detection criteria consist of the programmable parameters Rate and Duration. The detection criteria may also include one of the following detection enhancement suites, which may be used during initial and post-shock ventricular detection to add specificity beyond Rate and Duration:

- Onset/Stability
- Rhythm ID

The pulse generator initiates ventricular therapy when it determines that detection is met. Ventricular detection is met when all of the following occur:

- A ventricular zone's detection window becomes and remains satisfied throughout Duration
- The ventricular zone's Duration expires
- A higher ventricular zone's detection window is not satisfied
- Detection enhancements (if programmed to On) indicate therapy
- The last detected interval is in the ventricular zone

If the above criteria are not met, therapy is not initiated and the pulse generator continues to evaluate intervals.

Ventricular Detection Enhancement Suites

One of the following ventricular detection enhancement suites may be programmed to provide specificity beyond Rate and Duration (Table 2-3 on page 2-6):

- Rhythm ID
- Onset/Stability

Detection enhancement suites are not available in the VF zone.

Table 2-3. Detection enhancement suites available per zone

	VT-1 Zone	VT Zone	VF Zone
3-zone configuration ^a	Rhythm ID Onset/Stability	Rhythm ID Onset/Stability ^d	None
3-zone configuration (with Monitor Only zone) ^{b c}	None	Rhythm ID Onset/Stability	None
2-zone configuration		Rhythm ID Onset/Stability	None
2-zone configuration (with Monitor Only zone) ^b		None	None
1-zone configuration			None

- a. If the detection enhancement suite is enabled in a 3-zone configuration, it applies to both the VT-1 and VT zones.
 b. Detection enhancement suites are not available in the lowest zone of a multi-zone configuration when the zone is used as a Monitor Only zone (no therapy programmed for that zone).
 c. For devices programmed to a 3 zone configuration with VT-1 programmed to Monitor Only and detection enhancements On in the VT zone, rhythm discrimination will be applied when a tachycardia meets initial detection in the Monitor Only zone and the rate subsequently accelerates to the VT zone. In this case, initial detection is restarted and detection enhancements are available in the VT zone.
 d. Shock if unstable is the only Onset/Stability detection enhancement available in the VT zone of a 3-zone configuration (applies only to 3-zone configuration without a Monitor Only zone).

NOTE: *There is no clinical data to suggest that one detection enhancement suite is superior to the other for any given patient indication. Therefore, individual programming and evaluation of detection enhancement specificity is recommended.*

Rhythm ID

Rhythm ID uses Vector Timing and Correlation analysis in addition to atrial and ventricular interval analysis to determine if a patient's rhythm should be treated (VT) or if therapy should be inhibited (SVT).

With Rhythm ID, the pulse generator performs a vector timing and correlation analysis using the shock EGM and rate EGM. Based on this data, it saves a reference template of the patient's normal sinus rhythm.

During Rhythm ID analysis, the pulse generator first determines if the ventricular rate is greater than the atrial rate. If so, therapy will be initiated. If the ventricular rate is not greater than the atrial rate, Rhythm ID evaluates the following criteria to determine if therapy should be inhibited or initiated:

- Vector Timing and Correlation analysis during initial detection determines if the rhythm is SVT by comparing it to the previously stored reference template. If the rhythm is declared SVT, therapy is inhibited ("Vector Timing and Correlation" on page 2-19).
- If Vector Timing and Correlation does not declare the rhythm SVT, Stability and AFib Rate Threshold determine if the ventricular rhythm is unstable and the atrial rate is fast. If the ventricular rhythm is unstable and the atrial rate is fast, the rhythm is declared SVT and therapy is inhibited.

Rhythm ID does not consider atrial detection criteria (V Rate > A Rate or A greater than AFib Rate Threshold) for the following configurations:

- Single-chamber devices
- Dual-chamber devices if Atrial Tachyarrhythmia Discrimination is programmed to Off

When configured this way, Stability is not evaluated for initial detection. This may be useful in instances where atrial lead problems have occurred. For these configurations, therapy is inhibited at initial detection if the rhythm is declared SVT (based on Vector Timing and Correlation). Otherwise, therapy is initiated.

Two methods are available for the device to automatically acquire a Rhythm ID reference template: passive and active. The active method may be useful for patients who are frequently ventricular paced.

If the passive method is enabled, the pulse generator will attempt to collect the Rhythm ID reference template every two hours using the programmed brady settings. Updates start between 2 and 4 hours after the device has been removed from Storage mode.

If the active method is enabled and seven days have passed since the last successful collection of a reference template, then every 28 hours the device automatically analyzes the patient's intrinsic rhythm by adjusting the brady parameters. During a Rhythm ID active reference template update, the following will occur:

1. The device verifies that the patient is at rest (as measured by the accelerometer input).
2. The device enables a controlled pacing rate decrease to the programmed Rhythm ID Fallback LRL. During this fallback period, the following occurs:
 - The device temporarily switches the pacing mode to DDI, VDI, VVI, AAI, or Off (according to the programmed brady mode) and extends the AV delay up to 400 ms.
 - Rate Smoothing, ATR, Rate Hysteresis, Rate Search Hysteresis, AV Search +, and dynamic programming (excluding Dynamic VRP) are suspended.
3. After the Fallback period, pacing parameters are restored to normal programmed parameters. Fallback periods occur no more than once per day and will typically last less than one minute.

A method for manually commanding the device to acquire a Rhythm ID reference template is also available.

NOTE: *If Rhythm ID is not enabled, a manual reference template update can still be performed. However, the acquired template will not be used to determine if the patient's rhythm is VT or SVT.*

During a manual Rhythm ID reference template update, the pulse generator will perform the following tasks:

1. Enable a controlled rate decrease to the programmed Rhythm ID Fallback LRL. During the fallback period, the following occurs:
 - The device temporarily switches to the programmed Manual Rhythm ID Brady Mode and extends the AV delay up to 400 ms.
 - Rate Smoothing, ATR, Rate Hysteresis, Rate Search Hysteresis, AV Search +, and dynamic programming (excluding Dynamic VRP) are suspended.
2. After the Fallback interval, pacing parameters are restored to normal programmed parameters. This process will typically last less than one minute.

NOTE: Rhythm ID Fallback LRL settings should be selected such that normal sinus rhythms are promoted (e.g., normal AV node conduction). Care must be used when selecting LRL less than 50 min⁻¹ (rates that approach the patient's ventricular escape rates). Ventricular escape rhythms during Rhythm ID updates may result in inappropriate therapy decisions.

NOTE: An acquired Rhythm ID reference template will be used to perform Vector Timing and Correlation analysis until a newer reference template is acquired.

NOTE: A manual Rhythm ID reference template update should not be commanded immediately after shock therapy. It may take several minutes for irregularities in EGM morphology caused by the shock to subside.

Consider the following when using Rhythm ID:

- Rhythm ID determines whether therapy will be inhibited or not at the end of Duration. If the decision is to inhibit therapy, then Rhythm ID (including Vector Timing and Correlation, V Rate > A Rate, AFib Rate Threshold, and Stability) continues to be reevaluated beat-by-beat. Use of the Sustained Rate Duration (SRD) feature will limit inhibition of therapy by Rhythm ID to the length of the programmed SRD.
- Rhythm ID will not inhibit therapy in the VF zone. Programming the VF rate threshold lower than the rate of fast rhythms will prevent Rhythm ID from inhibiting therapy for those rhythms.
- Programming Atrial Tachyarrhythmia Discrimination to On prevents Rhythm ID from inhibiting therapy if the ventricular rate is faster than the atrial rate.
- If no Rhythm ID reference template has ever been acquired, Rhythm ID uses only Stability and AFib Rate Threshold to discriminate between VT and SVT, because Vector Timing and Correlation analysis cannot be performed. Furthermore, if Rhythm ID does not consider atrial detection criteria, and no reference template has been acquired, no detection enhancements will be evaluated during initial detection.

Onset/Stability

The Onset/Stability detection enhancement suite analyzes the cardiac cycle intervals to determine if a patient's rhythm should be treated (VT) or if therapy should be inhibited (SVT).

Onset/Stability allows you to program detection enhancements by identifying the desired type of rhythm discrimination: atrial tachyarrhythmia, sinus tachycardia, or polymorphic VT (Table 2-4 on page 2-8).

Table 2-4. Onset/Stability rhythm discrimination available per zone

	VT-1 Zone	VT Zone	VF Zone
3-zone Configuration	Atrial Tachyarrhythmia Sinus Tachycardia	Polymorphic VT ^a	None
3-zone Configuration (with Monitor Only zone) ^{b c}	None	Atrial Tachyarrhythmia Sinus Tachycardia Polymorphic VT ^a	None
2-zone Configuration		Atrial Tachyarrhythmia Sinus Tachycardia Polymorphic VT ^a	None

Table 2-4. Onset/Stability rhythm discrimination available per zone (continued)

	VT-1 Zone	VT Zone	VF Zone
2-zone Configuration (with Monitor Only zone) ^b		None	None
1-zone Configuration			None

- a. Polymorphic VT Discrimination is only available in the VT zone.
- b. Rhythm discrimination is not available in the lowest zone of a multi-zone configuration if the zone is used as a Monitor Only zone (no therapy programmed for that zone).
- c. For devices programmed to a 3 zone configuration with VT-1 programmed to Monitor Only and detection enhancements On in the VT zone, rhythm discrimination will be applied when a tachycardia meets initial detection in the Monitor Only zone and the rate subsequently accelerates to the VT zone. In this case, initial detection is restarted and detection enhancements are available in the VT zone.

Reconfirmation/Committed Shock

Reconfirmation refers to the monitoring performed by the device during and immediately following capacitor charging for a shock. When the Committed Shock parameter is programmed to Off, the device is allowed to reconfirm that a shock should be delivered.

Ventricular Redetection

Ventricular Redetection occurs following any:

- Ventricular therapy delivery
- Diverted therapy due to reconfirmation analysis (diverted-reconfirm)
- Manually diverted therapy
- Therapy not available at Detection Met (except when the VT-1 zone is programmed to Monitor Only, in which case initial detection is restarted)

Redetection uses the same ventricular detection window process and programmed tachycardia rate thresholds as initial detection to identify a tachyarrhythmia.

The primary differences between initial detection and redetection are the duration parameters used and the detection enhancements that are available:

- If ventricular shock therapy is delivered, the following will occur:
 - The redetection duration time is determined by the value of the Post-shock Duration parameter
 - Detection enhancements (except for Onset, Shock if Unstable, and Vector Timing and Correlation) are available during redetection
- If ventricular ATP is delivered or if therapy is diverted or unavailable, the following will occur:
 - The redetection duration time is determined by the Redetection Duration parameter
 - Detection enhancements (except for Shock if Unstable) are not available during redetection

Whichever duration is determined to be appropriate, that type of duration (Redetection or Post-shock) will be in effect in all zones at each zone's programmed duration value.

Ventricular Post-shock Detection Enhancements

When programmed to On, the following ventricular post-shock detection enhancements will be in effect following the Post-shock Duration:

- Post-shock V Rate > A Rate
- Post-shock AFib Rate Threshold

- Post-shock Stability
- Post-shock SRD
- Post-shock Rhythm ID (uses AFib Rate Threshold, Stability, V Rate > A Rate, and SRD)

With the exception of Rhythm ID, all post-shock detection enhancements perform the same as the corresponding Initial Detection enhancements (with Rhythm ID, Vector Timing and Correlation is not available post-shock).

Post-shock Stability may be used to prevent shock-induced AF from causing the pulse generator to deliver undesired additional shocks (Figure 2-2 on page 2-10.)

The AFib Rate Threshold can be programmed in conjunction with Post-shock Stability to further discriminate AF and prevent the pulse generator from delivering undesired ventricular shock therapy.

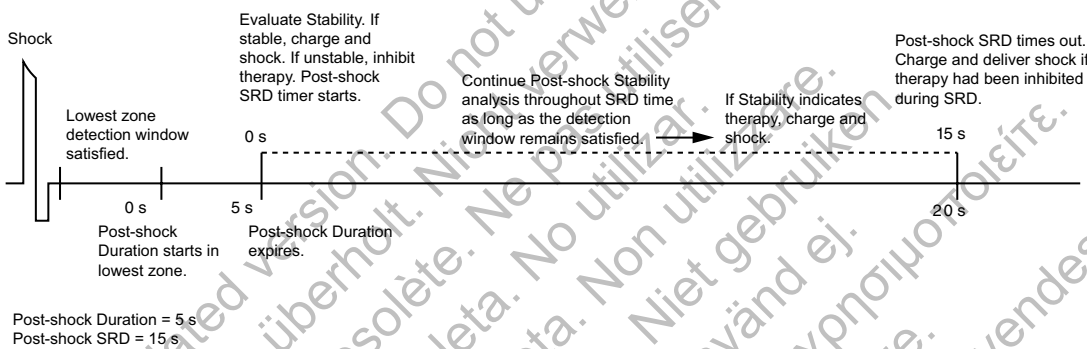


Figure 2-2. Post-shock Duration and Post-shock Stability analysis

Ventricular Detection Details

The pulse generator uses the following information to determine appropriate therapy delivery:

- Ventricular detection windows
- Duration parameter
- Redetection duration and post-shock duration
- Ventricular episodes
- Ventricular detection enhancements

Ventricular Detection Windows

Appropriate therapy delivery is dependent upon accurately classifying a patient's rhythm. To ensure that appropriate therapy is delivered, the pulse generator employs detection windows to differentiate tachycardias.

Each ventricular zone has a detection window that consists of the 10 most recent RV R–R intervals measured by the pulse generator. As each new interval is measured, it is compared to each zone's programmed rate threshold and classified as either fast or slow (i.e., above or below the rate threshold) in each detection window.

The pulse generator prepares for a potential episode when it counts 3 consecutive fast intervals. The detection window is satisfied and an episode is declared when 8 out of 10 fast intervals are counted. The detection window will remain satisfied as long as 6 of 10 intervals remain classified as fast. If the number of fast intervals falls below 6, the zone's detection window is no longer satisfied. The zone's detection window will only become resatisfied when 8 of 10 intervals are again classified as fast (Figure 2-3 on page 2-11).

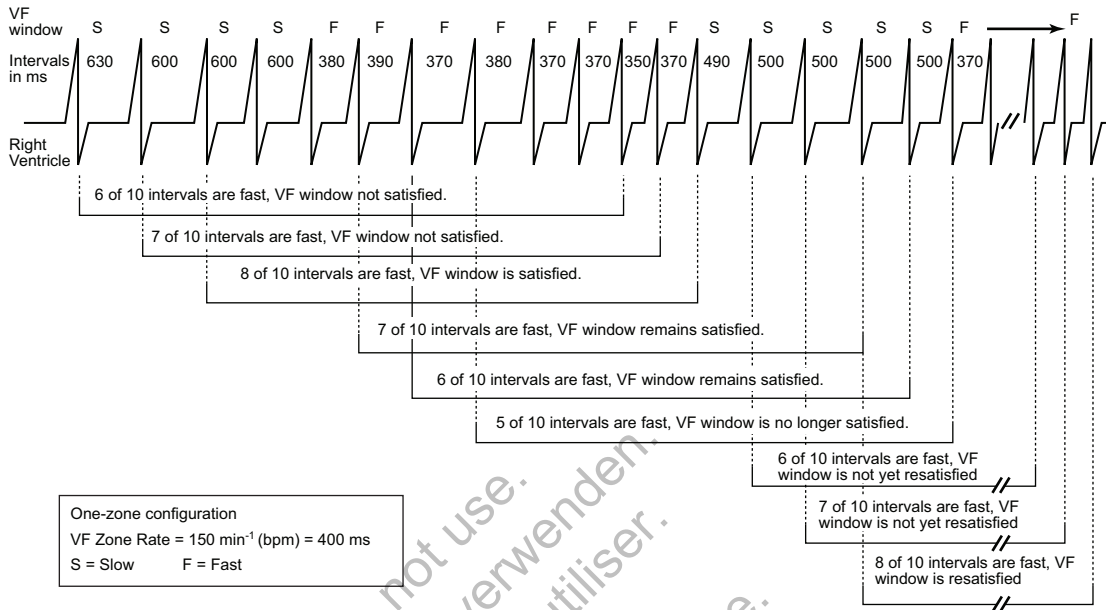


Figure 2-3. Ventricular detection window satisfied

Because Rate Threshold in the higher zones must be programmed at a value greater than Rate Threshold in lower zones, an interval classified as fast in a higher window would also be classified as fast in any lower windows (Figure 2-4 on page 2-11).

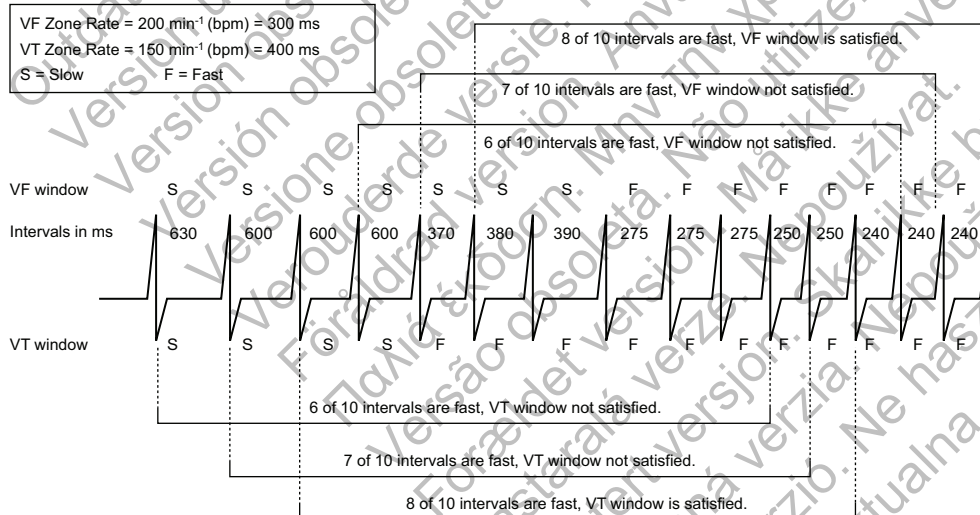


Figure 2-4. Interaction of ventricular detection windows, 2-zone configuration


Duration Parameter

The Duration parameter is a timer that measures the length of time in each zone that a rhythm must be sustained before therapy is delivered.

A Duration timer begins when its respective zone's detection window is satisfied. The programmed Duration time is checked following every cardiac cycle to determine if it has expired.

NOTE: Since the Duration timer is examined synchronously with a cardiac cycle, the programmed Duration may be exceeded by up to one full cardiac cycle.

- As long as the zone's detection window remains satisfied, the Duration timer continues to elapse. If the last detected interval is in the zone when its Duration time expires, detection is considered met and therapy is initiated (assuming no programmed detection enhancements inhibit therapy delivery) (Figure 2-5 on page 2-12).
- If the last detected interval is not in the zone, therapy is not initiated. Each subsequent interval will be checked until an interval is in the original zone, or the window is no longer satisfied (Figure 2-6 on page 2-12).
- If at any point during Duration a zone's detection window detects fewer than 6 of 10 fast intervals, that zone's Duration is reset to 0 (Figure 2-7 on page 2-12). Duration will start again only if the detection window becomes resatisfied.

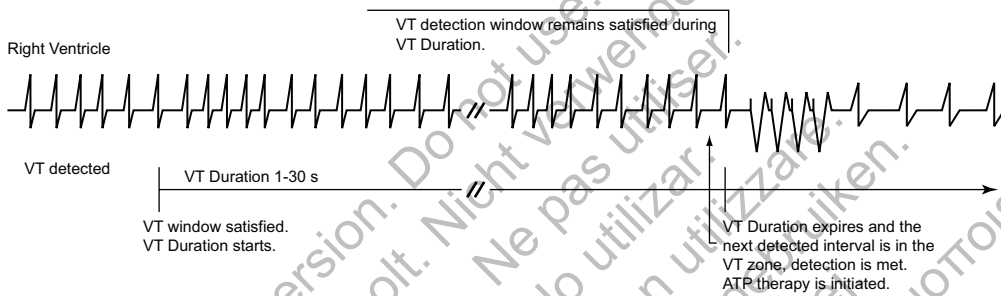

Duration starts when a window becomes satisfied and continues to elapse as long as the ventricular detection window remains satisfied. Detection is met when Duration expires and the next detected interval is in the same ventricular zone.

Figure 2-5. Ventricular Duration timer


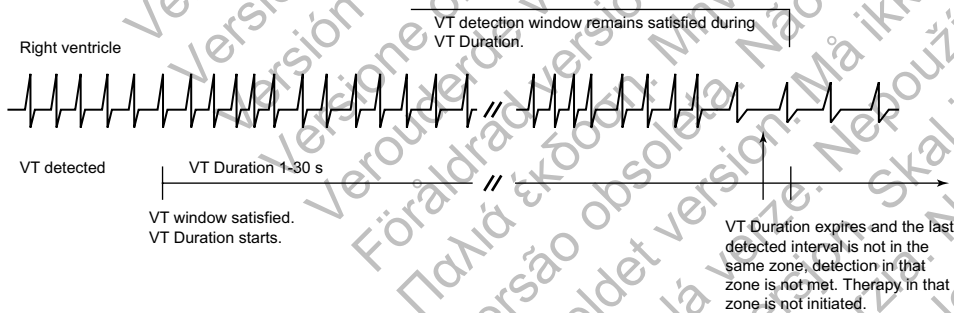


Figure 2-6. Last detected interval

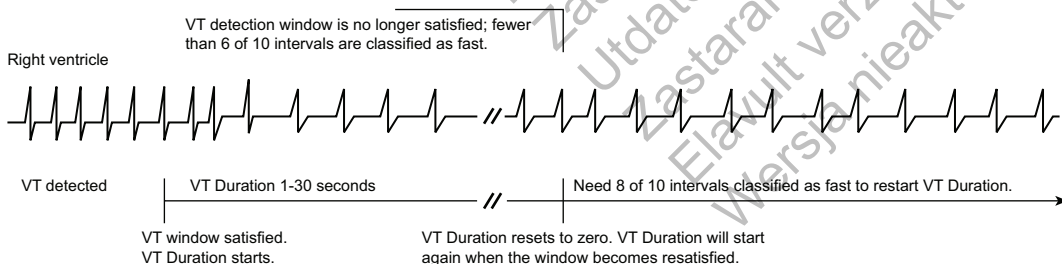

Duration resets when during the Duration period the window is no longer satisfied.

Figure 2-7. Ventricular Duration reset

A Duration is programmed for each ventricular zone. Different values are available depending on the configuration programmed (Table 2-5 on page 2-13). The Duration programmed in lower

ventricular rate zones must be greater than or equal to higher ventricular zones. Longer Durations may be used to prevent the device from initiating treatment of non-sustained arrhythmias.

Table 2-5. Duration programmable ranges by ventricular zone and configuration

Configuration	VT-1 Zone ^a	VT Zone ^a	VF Zone ^b
1 Zone	--	--	1–15 seconds
2 Zones	--	1–30 seconds	1–15 seconds
3 Zones	1–60 seconds	1–30 seconds	1–15 seconds

a. The maximum redetect duration for the VT-1 and VT Zones is 15 seconds.
b. In the VF Zone, the redetect and post-shock duration is fixed at 1 second.

Duration in a Multi-zone Configuration

Duration timers run independently of each other within their respective ventricular zones.

- If the arrhythmia is detected in the highest zone, that zone's Duration timer takes precedence over the lower zones' timers; the lower zones' Duration timers continue to elapse but are ignored while the higher zone's Duration timer runs.
- If the higher zone's Duration expires and detection is met, therapy for that zone will be initiated regardless of whether the lower zones' Duration timers have expired.
- If the higher zone's detection window does not remain satisfied, then the Duration timers for the lower ventricular zones are no longer ignored.

Programmed therapy for lower ventricular zones will be initiated when a lower ventricular zone's duration is met and no higher ventricular zone's window is satisfied (Figure 2-8 on page 2-13, Figure 2-9 on page 2-13).

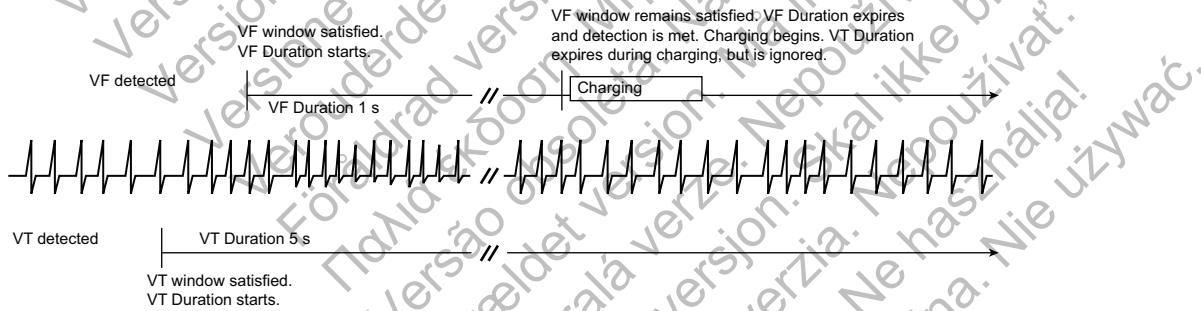


Figure 2-8. Interaction of ventricular Duration, 2-zone configuration, charging

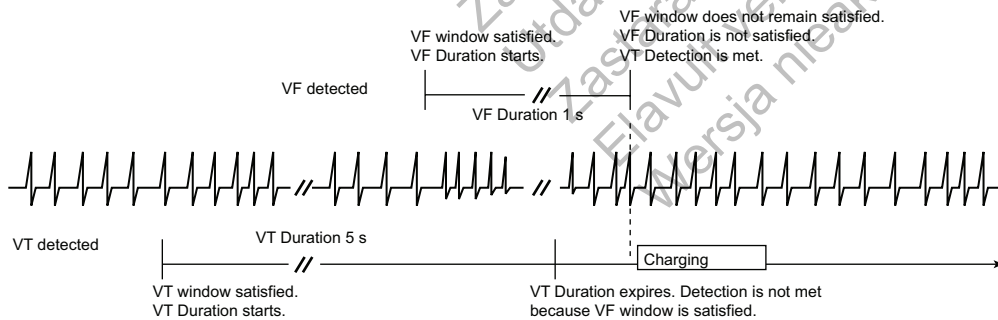


Figure 2-9. Interaction of ventricular Duration, 2-zone configuration, charging delayed

Ventricular Redetection Duration and Post-shock Duration

Duration parameters are used to identify tachyarrhythmias during the ventricular redetection process.

- Redetection Duration is applied following delivery of ATP therapy (except QUICK CONVERT ATP), a diverted-reconfirm, manually diverted therapy, or if therapy is unavailable at Detection Met (Figure 2-10 on page 2-14).
- Post-shock Duration is applied following shock therapy delivery (Figure 2-11 on page 2-14).

Redetection Duration is programmable in the lower ventricular zones of a multi-zone configuration. It is nonprogrammable in the VF Zone. Post-shock Duration can be programmed in the same manner; the values programmed in the lower ventricular rate zones must be greater than or equal to the values programmed in the higher zones.

To help minimize time to potential therapy, it is recommended that Redetection Duration in the VT-1 and VT zones of multi-zone configurations be programmed at less than or equal to 5 seconds.

It is recommended that Post-shock Duration in the VT-1 and VT zones of multi-zone configurations also be programmed at less than or equal to 5 seconds. However, you may program for longer durations if shock-induced, non-sustained, high-rate rhythms, such as accelerated idioventricular rhythm (AIVR) or AF are evident. The longer durations may allow the rhythm to return to a lower rate before redetection is met.

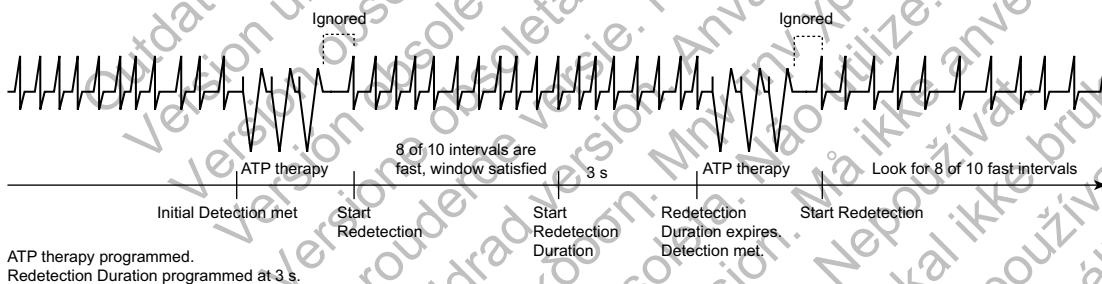


Figure 2-10. Redetection following ventricular ATP delivery

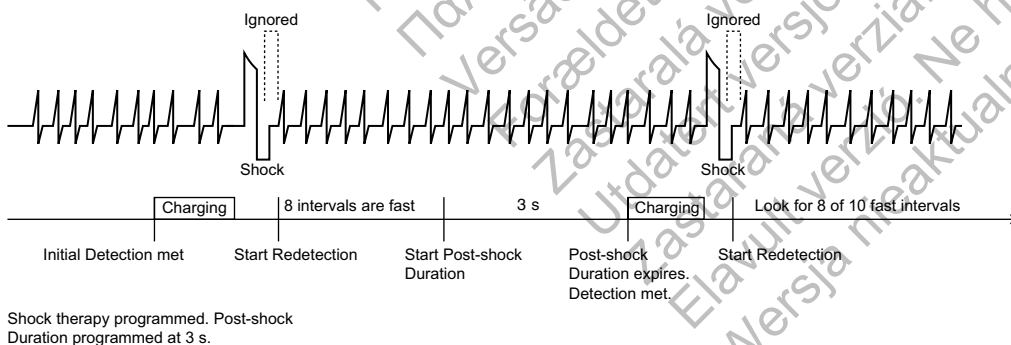


Figure 2-11. Redetection following ventricular shock delivery

Ventricular Episodes

If three consecutive fast ventricular beats are detected, then the pulse generator performs the following:

- Increments the episode number

- Allocates memory for history data and electrogram storage
- Starts monitoring for a detection window to be satisfied

When any zone's detection window becomes satisfied, the start of a ventricular episode is declared and duration timers begin in those zones where detection windows are satisfied. The ventricular episode is declared complete when all detection windows are no longer satisfied and remain unsatisfied for a specified time.

Each ventricular tachy episode is classified as treated or non-treated (Figure 2-12 on page 2-15 through Figure 2-16 on page 2-16).

- A treated episode is one in which therapy is delivered
- A nontreated episode is one in which no therapy is delivered

For a treated episode, an End-of-Episode timer starts after therapy is delivered. For a non-treated episode, an End-of-Episode timer starts at the point that the pulse generator recognizes that all detection windows are no longer satisfied. The End-of-Episode time interval is intended to allow the patient to stabilize before initial detection and initial therapy are used again. The episode is declared complete if no detection window becomes satisfied for a specified time following the last therapy attempt (Table 2-6 on page 2-15). If any window becomes satisfied while the End-of-Episode timer is elapsing, the End-of-Episode timer is reset to zero. It will start again when either therapy is attempted or all windows are not satisfied (Figure 2-16 on page 2-16).

Once an episode has been declared complete, the pulse generator will apply initial detection and therapy to subsequent tachyarrhythmias.

Table 2-6. End-of-Episode Timer

Episode Classification	Ventricular End-of-Episode Timer (elapsed time required to declare episode over)
Nontreated (no therapy delivered)	10 seconds
Treated (only ATP therapy delivered)	10 seconds
Treated (any shock therapy delivered)	30 seconds

NOTE: The episode is terminated immediately if the Tachy Mode is reprogrammed, an induction method or lead test is attempted before the End-of-Episode time out, or any ventricular detection or ventricular therapy parameters are reprogrammed.

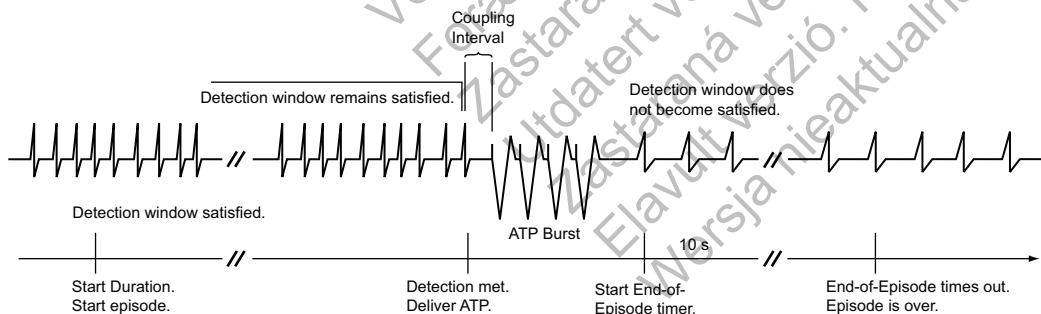


Figure 2-12. Treated episode, ventricular mode is Monitor + Therapy and ATP is delivered

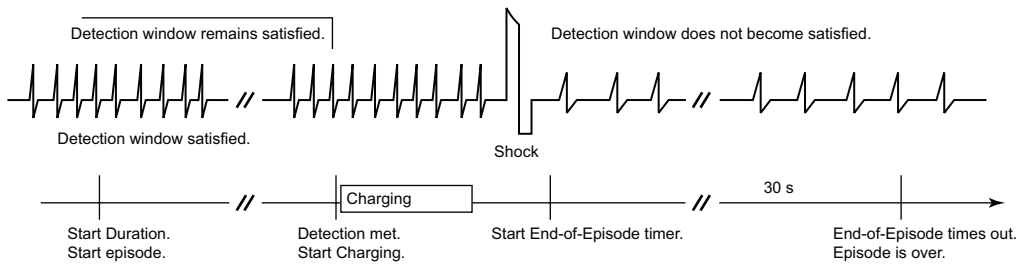


Figure 2-13. Treated episode, ventricular mode is Monitor + Therapy and shock is delivered



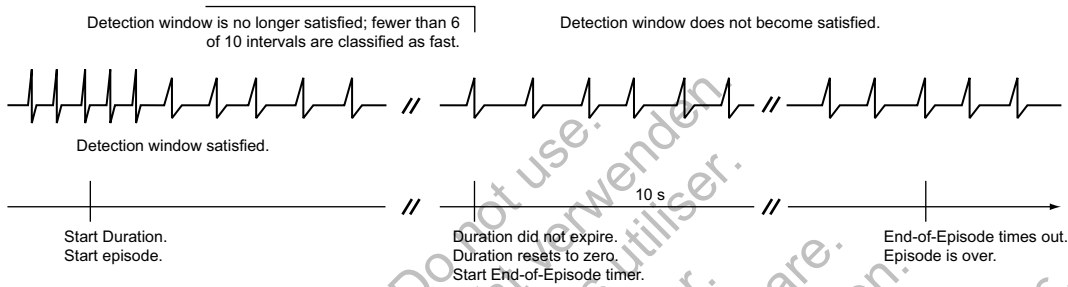


Figure 2-14. Nontreated episode, ventricular mode is Monitor + Therapy or Monitor Only, duration is not expired

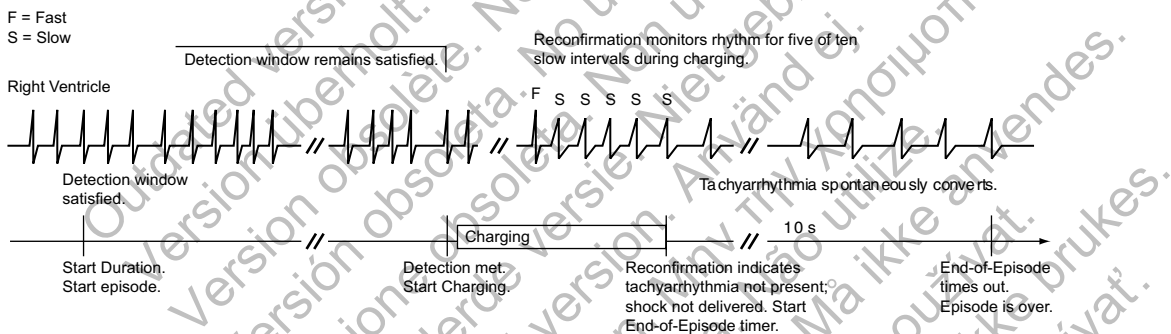

This example assumes Committed Shock is programmed to Off.

Figure 2-15. Nontreated episode, ventricular mode is Monitor + Therapy and charging is stopped prior to shock delivery

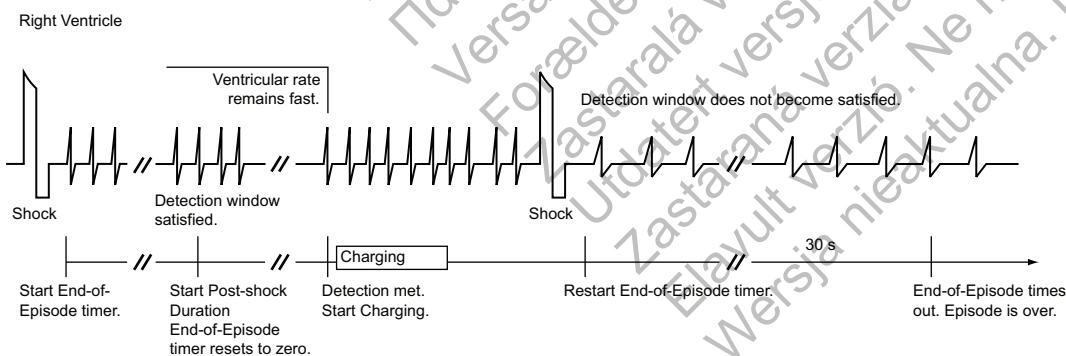

This example illustrates a Treated Episode when Ventricular mode is Monitor + Therapy. The End-of-Episode timer is reset to 0 when a ventricular detection window becomes satisfied after ventricular therapy delivery, but prior to the episode time-out being reached. In this example, 2 shocks were delivered in the episode.

Figure 2-16. Treated episode, ventricular mode is Monitor + Therapy and End-of-Episode timer is reset to 0

Ventricular Detection Enhancements

Ventricular detection enhancements add specificity to the Rate and Duration detection criteria. You may program ventricular detection enhancements to perform the following:

- Delay or inhibit therapy delivery
- Override therapy inhibition
- Bypass a sequence of ATP therapy in favor of shock therapy

Ventricular detection enhancements may be programmed to one of the following:

- Rhythm ID
- Onset/Stability
- Off (i.e., rate only)

If Off is selected, only the ventricular rate and duration are used for therapy decisions.

If either Rhythm ID or Onset/Stability is selected, enhancement parameters are used in addition to ventricular rate and duration for therapy decisions (Table 2-7 on page 2-17) as follows:

- Vector Timing and Correlation inhibits therapy when the conduction vector (EGM morphology and timing) during tachyarrhythmia matches a reference conduction vector of the patient's normal sinus rhythm.
- V Rate > A Rate can be used to override the inhibit decision of Onset, Stability, Vector Timing and Correlation, and/or AFib Rate Threshold. V Rate > A Rate can be used to deliver ventricular therapy anytime the ventricular rate is greater than the atrial rate.
- AFib Rate Threshold can be programmed (together with stability) to inhibit ventricular therapy if the atrial rhythm is fast.
- Stability can be programmed to inhibit ventricular therapy delivery if the ventricular rhythm is unstable.
- Shock if Unstable can be programmed to bypass the ventricular ATP therapy and deliver shock therapy if the ventricular rhythm is declared unstable.
- Onset can be programmed to inhibit ventricular therapy if the patient's heart rate increases gradually.
- SRD enables the pulse generator to override the Stability, Onset, Vector Timing and Correlation, and/or AFib Rate Threshold parameters' decision to inhibit ventricular therapy if the high rate continues throughout the programmed time period.

Table 2-7. Enhancement parameters available with detection enhancements

Enhancement Parameter	Rhythm ID		Onset/Stability	
	Initial	Post-Shock	Initial	Post-Shock
Vector Timing and Correlation ^a	X	--	--	--
V Rate > A Rate (dual-chamber devices only)	X ^{b c}	X ^{b c}	X	X
AFib Rate Threshold (dual-chamber devices only)	X ^{b d}	X ^{b d}	X ^e	X ^e
Stability (to inhibit)	X ^f	X ^f	X	X
Shock if Unstable	--	--	X	--

Table 2-7. Enhancement parameters available with detection enhancements (continued)

Enhancement Parameter	Rhythm ID		Onset/Stability	
	Initial	Post-Shock	Initial	Post-Shock
Onset	--	--	X	--
SRD ^g	X	X	X	X

- a. This enhancement is not individually programmable.
- b. When Rhythm ID is selected, this enhancement is automatically enabled when Atrial Tachyarrhythmia Discrimination is programmed to On. However, it is not available in single chamber devices or when the Atrial Tachyarrhythmia Discrimination is programmed to Off in dual chamber devices.
- c. This enhancement is not individually programmable when Rhythm ID is enabled.
- d. When Rhythm ID is selected, this parameter uses the same value for both initial and post-shock detection. It cannot be independently enabled or disabled for post-shock detection.
- e. When Onset/Stability is selected, this parameter can be enabled and disabled independently for post-shock detection. If enabled, it uses the same value as the initial detection.
- f. When Rhythm ID is enabled and Atrial Tachyarrhythmia Discrimination is programmed to On in dual chamber devices, this enhancement uses the same value for both initial and post-shock detection. In single chamber devices, or when Atrial Tachyarrhythmia Discrimination is programmed to Off, this enhancement is automatically disabled for Initial Detection, but is still enabled for post-shock detection.
- g. SRD is available when detection enhancements, which inhibit therapy, are programmed.

Some of these detection enhancement parameters are also independently programmable as post-shock parameters (Table 2-7 on page 2-17).

The individual detection enhancement parameters that are available depend on the number of tachy zones that are programmed: 3, 2, or 1 (Table 2-8 on page 2-18).

Table 2-8. Individual Ventricular Detection Enhancements available in multizone configurations

	VT-1 Zone	VT Zone	VF Zone
3-zone configuration	Vector Timing and Correlation V Rate > A Rate AFib Rate Threshold Stability to Inhibit Onset SRD	Vector Timing and Correlation ^a V Rate > A Rate ^a AFib Rate Threshold ^a Stability (to Inhibit) ^a Shock if Unstable SRD ^a	--
3-zone configuration (with Monitor Only zone) ^{b c}	--	Vector Timing and Correlation V Rate > A Rate AFib Rate Threshold Stability (to Inhibit) Shock if Unstable ^d Onset SRD	--
2-zone configuration		Vector Timing and Correlation V Rate > A Rate AFib Rate Threshold Stability (to Inhibit) Shock if Unstable ^d Onset SRD	--
2-zone configuration (with Monitor Only zone) ^b		--	--
1-zone configuration			--

- a. Enhancement is available in the middle zone of a 3-zone configuration only when Rhythm ID is enabled.
- b. Detection enhancements are not available in the lowest zone of a multi-zone configuration when it is used as a Monitor Only zone (no therapy programmed for that zone).
- c. For devices programmed to a 3 zone configuration with VT-1 programmed to Monitor Only and detection enhancements On in the VT zone, rhythm discrimination will be applied when a tachycardia meets initial detection in the Monitor Only zone and the rate subsequently accelerates to the VT zone. In this case, initial detection is restarted and detection enhancements are available in the VT zone.
- d. Shock if Unstable cannot be programmed on in the same zone as other detection enhancements that are programmed to inhibit therapy (Onset, Stability, and AFib Rate Threshold).

When a specific rhythm discrimination is selected, you can modify the values for the detection enhancements that are suitable for discriminating that rhythm. Nominal values are shown in the following table; however, you can use those values at your discretion.

Table 2-9. Nominal values for initial detection and redetection enhancements

Parameter	Onset/Stability			Rhythm ID	
	Atrial Tachyarrhythmia Discrimination	Sinus Tachycardia Discrimination	Polymorphic VT Discrimination	Atrial Tachyarrhythmia Discrimination On	Atrial Tachyarrhythmia Discrimination Off
Vector Timing and Correlation	--	--	--	On ^a	On ^a
V Rate > A Rate (dual-chamber models only)	On	On	--	On ^b	--
AFib Rate Threshold (dual-chamber models only)	170 min ⁻¹	--	--	170 min ⁻¹	--
Stability (Inhibit)	20 ms (DR devices) 30 ms (VR devices)	--	--	20 ms (DR devices) 30 ms (VR devices)	30 ms
Onset (initial detection only)	--	9%	--	--	--
SRD Initial	3:00 minutes:seconds	3:00 minutes:seconds	--	3:00 minutes:seconds	3:00 minutes:seconds
SRD Redetection	0:15 minutes:seconds	--	--	0:15 minutes:seconds	0:15 minutes:seconds
Shock if Unstable	--	--	30 ms	--	--

a. Parameter is not individually programmable.

b. Parameter is not individually programmable when Rhythm ID is enabled.

Vector Timing and Correlation

Vector Timing and Correlation compares EGM signals for an unknown rhythm with a stored reference template of the EGM signals of a normal sinus rhythm (NSR). Rhythms that are not similar (i.e. not correlated) to the stored reference template are classified as VT. Rhythms that are correlated with the stored reference template are classified as SVT. Rhythm ID uses this classification during initial detection to make a decision to treat or to inhibit therapy.

When a fast rhythm is sensed, each beat of the rhythm is compared to the stored reference template. The pulse generator measures the correlation of the detected waveform to the stored reference template, and classifies each beat as correlated or uncorrelated. A beat in the VF zone is always counted as uncorrelated, even if it has a high measured correlation value.

The pulse generator then classifies the detected rhythm as SVT or VT based on the calculations:

- If at least 3 out of 10 beats are correlated, the rhythm is classified as SVT, shown as RID+ in the annotated electrograms
- If fewer than 3 out of 10 beats are correlated, the rhythm is classified as VT, shown as RID- in the annotated electrograms

V Rate > A Rate

The V Rate > A Rate (ventricular rate greater than atrial rate) enhancement compares the atrial and ventricular rates to classify the type of fast ventricular rhythm. When the ventricular rate is greater than the atrial rate, therapy will be initiated regardless of the analysis of the other programmed detection enhancements.

Analysis is made by comparing the average rate of the last 10 ventricular intervals prior to the end of Duration to the average rate of the last 10 atrial intervals prior to the end of Duration (Figure 2-17 on page 2-20). If fewer than 10 atrial intervals are available, those intervals will be used to calculate the average atrial rate. This analysis is performed using the following criteria:

- If the average ventricular rate is greater than the average atrial rate by at least 10 min⁻¹, the ventricular rate is declared to be faster than the atrial rate (indicated as True on the Episode Detail Report), and therapy will be initiated.
- If the average ventricular rate is not greater than the average atrial rate by at least 10 min⁻¹ (indicated as False on the Episode Detail Report), therapy may continue to be inhibited. The Episode Detail report will indicate the measured value even though the parameter may be programmed to Off.

If therapy is inhibited, the V Rate > A Rate analysis continues until either the ventricular rate is greater than the atrial rate or other enhancements indicate therapy treatment, at which time therapy will be initiated.

NOTE: V Rate > A Rate is not evaluated during redetection following ATP therapy.

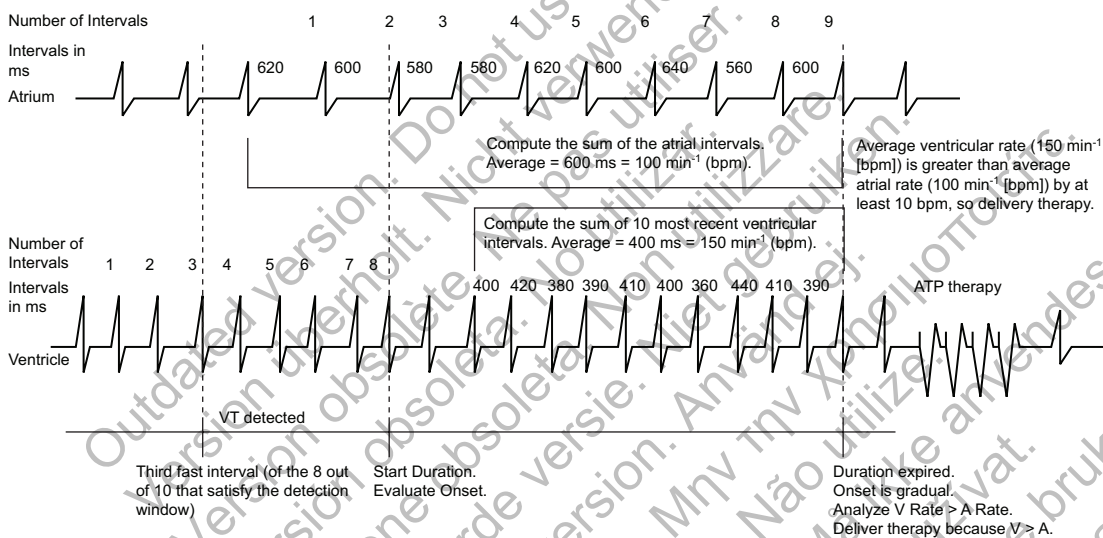


Figure 2-17. V Rate > A Rate analysis

V Rate > A Rate can be programmed to bypass inhibitors (Vector Timing and Correlation, AFib Rate Threshold, Stability, and/or Onset) and initiate therapy in the event that the ventricular rate is faster than the atrial rate.

NOTE: Refer to "Use of Atrial Information" on page 2-4 for additional information about device performance when the atrial lead is programmed to Off.

NOTE: In a Rhythm ID configuration, the evaluation of V Rate > A Rate is linked to the AFib Rate Threshold. If Atrial Tachyarrhythmia Discrimination is programmed to Off, the AFib Rate Threshold and V Rate > A Rate detection enhancements are not evaluated.

AFib Rate Threshold

AFib Rate Threshold analysis identifies AF by comparing the atrial rate to the programmed AFib Rate Threshold.

AFib Rate Threshold cannot be enabled without also enabling the Stability detection enhancement. The device analyzes both parameters to determine whether to withhold or deliver therapy.

If the intrinsic atrial rate is greater than the AFib Rate Threshold and the ventricular rhythm is classified as unstable, the ventricular rhythm is declared to be due to AF.

The intrinsic atrial rate is declared to be above the AFib Rate Threshold in the following manner (Figure 2-18 on page 2-21):

- Atrial analysis begins at initiation of ventricular tachyarrhythmia detection. Each atrial interval is classified as faster or slower than the AFib Rate Threshold Interval.
- When 6 of the last 10 intervals are classified as faster than the AFib Rate Threshold, the device declares AF to be present.
- Ventricular stability is then checked. If unstable, therapy is inhibited.

In the event that ventricular therapy is not delivered, the atrial rate continues to be examined. As long as 4 of 10 intervals remain classified as fast, AF is considered present. Therapy is inhibited by AFib Rate Threshold/Stability until any of the following occur:

- The atrial rate drops below the AFib Rate Threshold
- The ventricular rhythm becomes stable
- If programmed to On, $V \text{ Rate} > A \text{ Rate}$ is true
- SRD times out

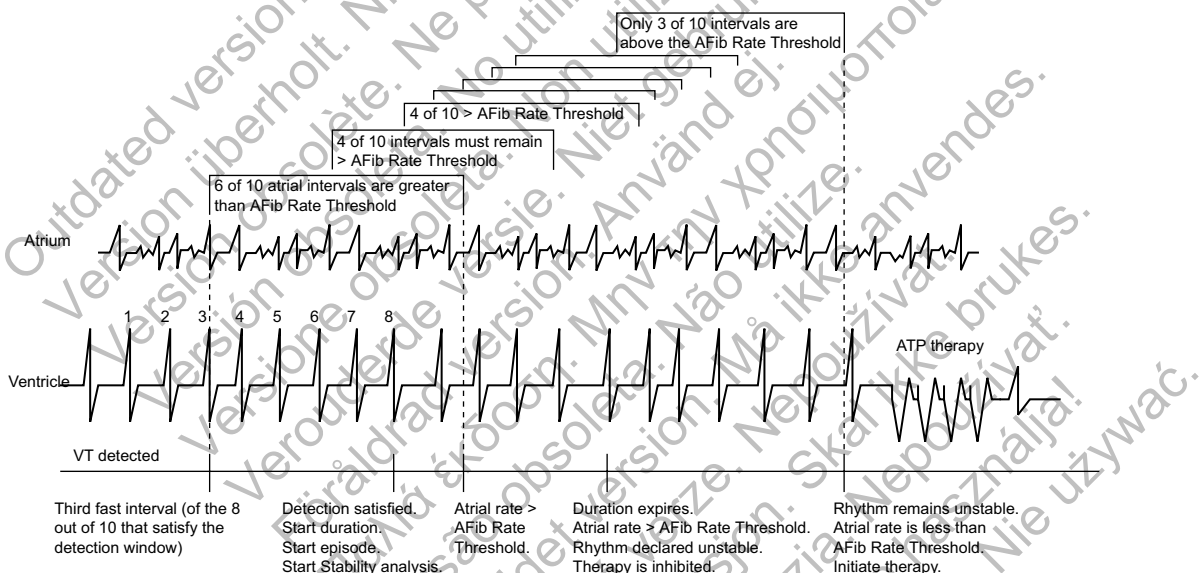


Figure 2-18. Interaction of AFib Rate Threshold and Stability

When AFib Rate Threshold and Stability are used alone, ventricular therapy is initiated when a stable rhythm is declared. Ventricular therapy is initiated for an unstable rhythm when it is determined that the atrial rate is less than the AFib Rate Threshold (Table 2-10 on page 2-22). When AFib Rate Threshold and Stability are used with other inhibitor enhancements, ventricular therapy is not always initiated when no longer inhibited by AFib Rate Threshold/Stability. Therapy may continue to be inhibited by other programmed detection enhancements, such as Onset (when the Onset/Stability detection enhancement suite is enabled) or Vector Timing and Correlation (when the Rhythm ID detection enhancement suite is enabled).

Consider the following information during these interactions:

- The AFib Rate Threshold and $V \text{ Rate} > A \text{ Rate}$ detection enhancements are not evaluated if Atrial Tachyarrhythmia Discrimination is programmed to Off in a Rhythm ID configuration.

- Because the AFib Rate Threshold is not evaluated during redetection (following ventricular ATP therapy delivery, any aborted ventricular therapy, or therapy not available), the Episode Detail report will not display data for the enhancement during redetection, even though the parameter is programmed On.
- The AFib Rate Threshold enhancement is not evaluated for arrhythmia detection in the following cases; however, the Episode Detail report will still display the data for the AFib Rate Threshold enhancement based on a threshold of 170 min⁻¹:
 - The AFib Rate Threshold is programmed to Off
 - Ventricular Zones is programmed to 1
 - No detection enhancement suite is enabled
- An atrial sense event will only be classified as AF while the AFib Rate Threshold is being evaluated for arrhythmia detection.

Table 2-10. AFib Rate Threshold and Stability combinations and resulting therapy

Detected Ventricular Rhythm ^a	Therapy Decision ^b
Unstable, A > AFib Rate Threshold	Inhibit
Stable, A > AFib Rate Threshold	Treat
Unstable, A < AFib Rate Threshold	Treat
Stable, A < AFib Rate Threshold	Treat

a. If the detected ventricular rhythm changes, then the appropriate, corresponding row in the table is evaluated.
 b. Decisions to inhibit can be overridden by V > A or expiration of SRD.

NOTE: Refer to "Use of Atrial Information" on page 2-4 for additional information about device performance when the atrial lead is programmed to Off.

Stability Analysis

Stability analysis distinguishes unstable (irregular) ventricular rhythms from stable (regular) ventricular rhythms. This is accomplished by measuring the degree of variability of the tachycardia R–R intervals.

This degree of variability, when used alone, may allow the device to distinguish conducted AF (which may produce greater R–R variability) from monomorphic VT (which is typically stable). It also may be used to differentiate MVTs (which are pace terminable) from polymorphic VTs and VF (which are typically not pace terminable).

Based on the patient's needs, you may choose to program Stability as an inhibitor to prevent therapy for AF, or use stability analysis to direct the type of therapy to be delivered (Shock if Unstable).

The stability algorithm calculates RV R–R interval differences. These differences are calculated throughout Duration; an average difference is also calculated. When Duration expires, rhythm stability is evaluated by comparing the current average difference to the programmed Stability threshold and/or Shock If Unstable thresholds. If the average difference is greater than the programmed thresholds, the rhythm is declared unstable. Independent thresholds are available for the Stability (to inhibit) or Shock If Unstable functions; you cannot program both in the same ventricular zone.

The pulse generator performs stability calculations for all episodes (even when Stability is programmed to Off) and stores the results in therapy history. This stored data may be used to select an appropriate stability threshold.

Stability to Inhibit

The Stability parameter may help you identify rapid rhythms originating in the atrium, such as AF. These rhythms may result in unstable ventricular rhythms whose rate exceeds the lowest rate threshold and should not be treated. If a rhythm is declared stable when Duration expires, programmed therapy will be delivered. If the rhythm is declared unstable, ventricular therapy will be inhibited.

At the end of initial Duration, if a tachycardia is declared unstable and ventricular therapy is inhibited, the pulse generator continues to evaluate for stability on each new detected interval (Figure 2-19 on page 2-23). Therapy will not be inhibited by Stability if:

- V Rate > A Rate declares the ventricular rate greater than the atrial rate
- The SRD has expired (if programmed to On)

Ventricular therapy is not always initiated when no longer inhibited by Stability. Therapy may continue to be inhibited by other programmed detection enhancements, such as Onset (when the Onset/Stability detection enhancement suite is enabled) or Vector Timing and Correlation (when the Rhythm ID detection enhancement suite is enabled).

NOTE: Ventricular Therapy can also be inhibited through analysis of the Stability algorithm as it is used with the AFib Rate Threshold enhancement.

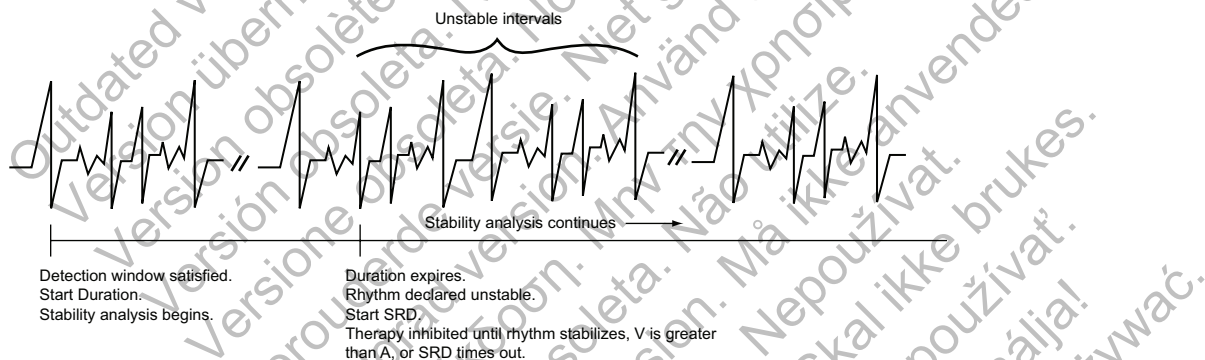


Figure 2-19. Stability evaluation when Duration expires

Shock if Unstable

When programmed to Shock if Unstable, the stability analysis helps determine if ventricular ATP therapy should be bypassed in preference for the first programmed ventricular shock therapy (which may be low- or high-energy) for the ventricular zone (Figure 2-20 on page 2-24).

Dynamic ventricular arrhythmias such as polymorphic VT or VF may be sensed at a rate lower than the highest ventricular rate threshold and can be classified as unstable. Since the sensed rhythm may be detected in a lower ventricular zone in which ATP may be programmed, the stability analysis may be used to skip over the programmed ventricular ATP therapies and instead provide shocks to the patient. Stability is evaluated on each detection/redetection cycle, including evaluation between bursts of an ATP scheme. Once a ventricular shock has been delivered in an episode, the Shock If Unstable function no longer affects therapy selection.

Shock If Unstable may be used only in the VT zone of a 2- or 3-zone configuration. You cannot program it in a 2-zone configuration if Stability or Onset is already programmed to On, or if Post V-Shock Stability or AFib Rate Threshold is programmed to On.

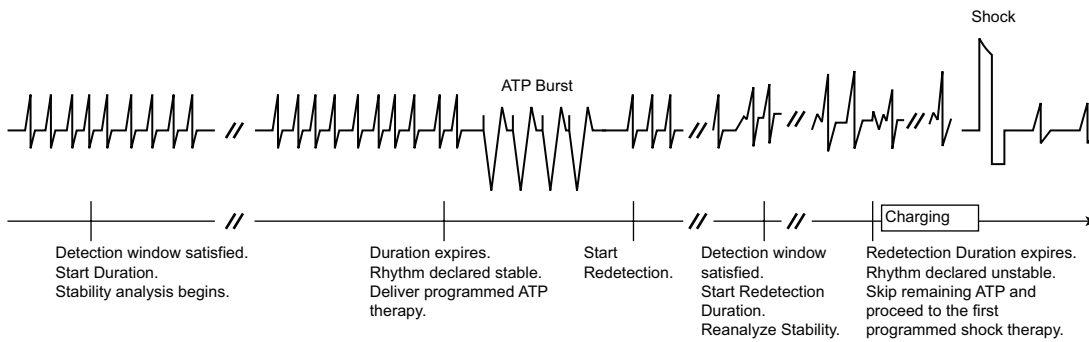


Figure 2-20. Shock if Unstable

Onset

Onset differentiates physiologic sinus tachycardias, which typically begin slowly, from pathologic tachycardias, which typically begin abruptly. It measures the rate of transition in the ventricular rhythm from slow rates to tachycardia. If the rate increase is gradual, it enables the device to inhibit ventricular therapy in the lowest tachycardia rate zone.

When a detection window becomes satisfied, the pulse generator begins calculating for sudden Onset in a two-stage sequence.

- Stage 1 measures the ventricular intervals prior to the start of the episode and locates the pair of adjacent intervals (pivot point) where the cycle length decreased the most. If the decrease in cycle length is equal to or greater than the programmed Onset value, stage 1 declares sudden Onset.
- Stage 2 then compares additional intervals. If the difference between the average interval before the pivot point and 3 out of the first 4 intervals following the pivot point is equal to or greater than the programmed Onset Threshold, stage 2 declares sudden Onset.

If both stages declare the rhythm sudden, therapy will be initiated. If either stage indicates a gradual onset, initial ventricular therapy will be inhibited in the lowest zone. Therapy will not be inhibited by Onset if:

- The rate accelerates to a higher ventricular zone
- Information from the atrial lead determines that the RV rate is faster than the atrial rate ($V \text{ Rate} > A \text{ Rate}$ programmed to On)
- The SRD timer expires

Onset is measured using RV intervals only. It can be programmed as a percentage of cycle length or as an interval length (in ms). It is limited to the lowest therapy zone of a multi-zone configuration. The selected Onset value represents the minimum difference that must exist between intervals that are above and below the lowest programmed rate threshold. The pulse generator performs Onset calculations (even when the feature is programmed to Off) for all episodes except induced or commanded episodes. The measured Onset results from a two-stage calculation are stored in therapy history. This stored data may be used to program an appropriate Onset value.

Sustained Rate Duration (SRD)

Sustained Rate Duration allows delivery of the programmed ventricular therapy when a tachycardia is sustained for a programmed period beyond Duration, but the programmed therapy inhibitors (Vector Timing and Correlation, AFib Rate Threshold, Onset, and/or Stability) indicate to withhold therapy (Figure 2-21 on page 2-25).

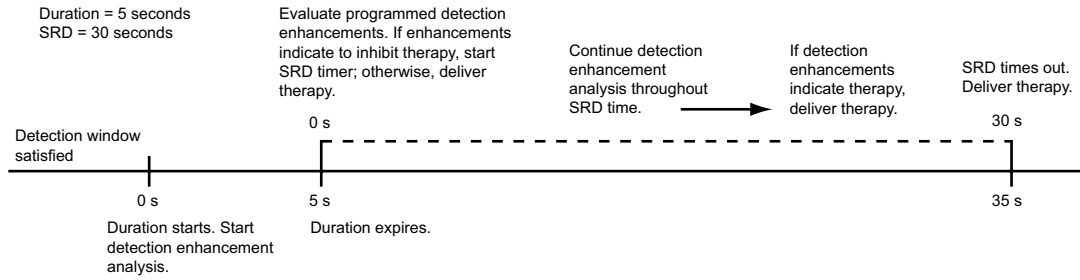


Figure 2-21. Combination of Onset OR Stability, SRD programmed on

SRD is available in a zone only when an inhibitor enhancement is programmed on in that zone. When the Rhythm ID detection enhancement suite is enabled, SRD may be programmed separately for the VT and VT-1 zones.

- A programmed SRD timer begins if ventricular therapy is withheld when the Duration expires in a zone where detection enhancements are programmed On.
- If the detection window in the lowest zone is maintained for the programmed SRD period, the programmed ventricular therapy will be delivered at the end of the VT-1 SRD period if VT-1 SRD is programmed and the rhythm is in the VT-1 zone. Therapy will be delivered at the end of the VT SRD period if VT SRD is programmed and the rhythm is in the VT zone.
- If the rate accelerates to a higher ventricular zone, detection enhancements are not programmed to On in the higher zone, and the Duration for the higher zone expires, therapy is initiated in that zone without waiting for SRD time-out in a lower ventricular zone. If SRD is programmed to Off, an SRD timer will not start when Duration expires, thus allowing detection enhancements to potentially inhibit therapy indefinitely.

An independent Post-Shock SRD value may be programmed.

Combinations of AFib Rate Threshold, Stability, and Vector Timing and Correlation

The combination of AFib Rate Threshold, Stability, and Vector Timing and Correlation add specificity to ventricular detection beyond rate and duration. In addition to using AFib Rate Threshold and Stability to identify AF, this combination of enhancements uses Vector Timing and Correlation analysis to differentiate SVT rhythms from VT rhythms based on conduction patterns within the heart.

The AFib Rate Threshold, Stability, and Vector Timing and Correlation detection enhancement combination also includes V Rate > A Rate; both AFib Rate Threshold and V Rate > A Rate are enabled when Atrial Tachyarrhythmia Discrimination is programmed to On. This combination is only available when the Rhythm ID detection enhancement suite is enabled, and only for Initial Detection (Table 2-11 on page 2-25).

If V Rate > A Rate is programmed to On (by programming Atrial Tachyarrhythmia Discrimination to On) and is True, it will take precedence over all inhibitor enhancements.

Table 2-11. AFib Rate Threshold, Stability, and Vector Timing and Correlation combinations and resulting therapy decision if Atrial Tachyarrhythmia Discrimination is programmed to On

Detected Ventricular Rhythm ^{a b c}	Therapy Decision ^d
Correlated, Unstable, A > AFib Rate Threshold	Inhibit
Correlated, Unstable, A < AFib Rate Threshold	Inhibit
Uncorrelated, Unstable, A > AFib Rate Threshold	Inhibit
Uncorrelated, Unstable, A < AFib Rate Threshold	Treat
Correlated, Stable, A > AFib Rate Threshold	Inhibit

Table 2-11. AFib Rate Threshold, Stability, and Vector Timing and Correlation combinations and resulting therapy decision if Atrial Tachyarrhythmia Discrimination is programmed to On (continued)

Detected Ventricular Rhythm ^{a b c}	Therapy Decision ^d
Correlated, Stable, A < AFib Rate Threshold	Inhibit
Uncorrelated, Stable, A > AFib Rate Threshold	Treat
Uncorrelated, Stable, A < AFib Rate Threshold	Treat

- a. If the detected ventricular rhythm changes, then the appropriate, corresponding row in the table is evaluated.
- b. If a Rhythm ID reference template is not available, the detected ventricular rhythm is considered to be Uncorrelated.
- c. For post shock detection (if enabled), Vector Timing and Correlation is considered to be Uncorrelated.
- d. Decisions to inhibit can be overridden by V > A or expiration of SRD.

When Atrial Tachyarrhythmia Discrimination is programmed to Off, then Vector Timing and Correlation is used for Initial Detection and Stability is used for Post-shock detection. V Rate > A Rate and AFib Rate Threshold are no longer used (Table 2-12 on page 2-26).

Table 2-12. Vector Timing and Correlation and Stability combinations with resulting therapy decision if Atrial Tachyarrhythmia Discrimination is programmed to Off

Detection ^{a b}	Detected Ventricular Rhythm ^{a c}	Therapy Decision
Initial	Correlated	Inhibit ^d
Initial	Uncorrelated	Treat
Post-shock	Unstable	Inhibit ^d
Post-shock	Stable	Treat

- a. If the detected ventricular rhythm changes, then the appropriate, corresponding row in the table is evaluated.
- b. If Atrial Tachyarrhythmia Discrimination is programmed to Off, then Vector Timing and Correlation is used for Initial Detection, and Stability is used for Postshock Detection.
- c. If a Rhythm ID reference template is not available, the detected ventricular rhythm is considered to be Uncorrelated.
- d. Decision to inhibit can be overridden by expiration of SRD.

Combinations of AFib Rate Threshold, Stability, and Onset

The combination of AFib Rate Threshold, Stability, and Onset add specificity to ventricular detection beyond rate and duration. This combination of detection enhancements is available only when the Onset/Stability detection enhancement suite is enabled and is available only for Initial Detection. When detection enhancements are enabled, they will act to recommend or inhibit therapy for a specific zone.

If AFib Rate Threshold, Stability, and Onset parameters are all programmed to On, ventricular therapy will be initiated if the rhythm has a sudden onset provided that either the ventricular rate is stable or the atrial rate is less than the AFib Rate Threshold (Table 2-13 on page 2-26).

Table 2-13. AFib Rate Threshold, Stability, and Onset combinations and resulting ventricular therapy

Detected Ventricular Rhythm ^a	Therapy Decision ^b
Gradual, Unstable, A > AFib Rate Threshold	Inhibit
Gradual, Unstable, A < AFib Rate Threshold	Inhibit
Sudden, Unstable, A > AFib Rate Threshold	Inhibit
Sudden, Unstable, A < AFib Rate Threshold	Treat ^c
Gradual, Stable, A > AFib Rate Threshold	Treat
Gradual, Stable, A < AFib Rate Threshold	Inhibit

Table 2-13. AFib Rate Threshold, Stability, and Onset combinations and resulting ventricular therapy (continued)

Detected Ventricular Rhythm ^a	Therapy Decision ^b
Sudden, Stable, A > A Fib Rate Threshold	Treat
Sudden, Stable, A < AFib Rate Threshold	Treat

- a. If the detected ventricular rhythm changes, then the appropriate, corresponding row in the table is evaluated.
b. Decisions to inhibit can be overridden by V > A or expiration of SRD.
c. If V Rate > A Rate is programmed to On and is False, ventricular therapy will be inhibited because the rhythm is unstable.

If V Rate > A Rate is programmed to On and is True, it will take precedence over all inhibitor enhancements.

Combinations of Onset and Stability

When Stability is programmed to inhibit, it may be combined with Onset to provide even greater specificity in classifying arrhythmias.

This combination of detection enhancements is available only when the Onset/Stability detection enhancement suite is enabled and is available only for Initial Detection. The enhancements can be programmed to initiate ventricular therapy if the following options are selected (Table 2-14 on page 2-27):

- Both Onset And Stability indicate to treat
- Either Onset Or Stability indicates to treat

Based on these programming decisions, ventricular therapy is inhibited when any of the following criteria is met:

- If the combination programmed is Onset And Stability, ventricular therapy is inhibited if either parameter indicates that therapy should be withheld; that is, the rhythm is gradual Or unstable (the And condition to treat is not satisfied).
- If the combination programmed is Onset Or Stability, ventricular therapy is inhibited immediately at the end of Duration only if both parameters indicate that therapy should be withheld; that is, the rhythm is gradual and unstable (the Or condition to treat is not satisfied).

In either case, ventricular therapy will be initiated only if the And/Or conditions to treat are satisfied. When these two combinations (And/Or) are used in conjunction with SRD, and the And/Or conditions are not satisfied, ventricular therapy will be inhibited until V Rate > A Rate is True or SRD times out (Table 2-14 on page 2-27).

Table 2-14. Combinations of Onset And Stability and resulting therapy

Detection Rhythm	Onset And Stability Combination ^{a b c}	Onset Or Stability Combination ^c
Gradual, unstable	Inhibit	Inhibit
Gradual, stable	Inhibit	Treat
Sudden, unstable	Inhibit	Treat
Sudden, stable	Treat	Treat

- a. If the detected ventricular rhythm changes, then the appropriate, corresponding row in the table is evaluated.
b. The And combination is the nominal setting when both are enabled.
c. Decisions to inhibit can be overridden by V > A or expiration of SRD.

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verze. Nepoužívat.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

TACHYARRHYTHMIA THERAPY

CHAPTER 3

This chapter contains the following topics:

- "Ventricular Therapy" on page 3-2
- "Antitachycardia Pacing Therapies and Parameters" on page 3-7
- "Ventricular Shock Therapy and Parameters" on page 3-15

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzate.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

VENTRICULAR THERAPY

The pulse generator can deliver the following types of therapy to terminate VT or VF:

- Antitachycardia pacing (ATP)
- Cardioversion/defibrillation shocks

ATP pacing schemes are bursts of pacing pulses delivered between the ventricular pace/sense electrodes. Shocks are high-voltage biphasic pulses delivered through the shocking electrodes synchronously with detected heart activity.

Ventricular Therapy Prescription

A ventricular therapy prescription determines the type of therapy to be delivered in a particular ventricular rate zone. It consists of ventricular ATP and/or shocks. Each ventricular zone may be programmed with independent ventricular therapy prescriptions (Figure 3-1 on page 3-2).

Within each zone, therapy strength must be in ascending order.

		Lowest strength → Highest strength				
Zone	ATP1 ²	ATP2 ²	QUICK CONVERT ATP	Shock 1 ¹	Shock 2 ¹	Remaining (Maximum) Shocks ¹
VF	Not available		On/Off	0.1-max J	0.1-max J	max J
VT	All ATP types available	All ATP types available	N/A	0.1-max J	0.1-max J	max J
VT-1	All ATP types available	All ATP types available	N/A	0.1-max J	0.1-max J	max J

Between zones, therapy strengths are not restricted.

¹ In the lowest zone of a multi-zone configuration, some or all of the shocks may be programmed to Off, starting with the maximum shocks first. If the maximum shocks are programmed to Off, then Shock 2 can be programmed to Off. If Shock 2 is programmed to Off, then Shock 1 can be programmed to Off. If the arrhythmia persists in the lowest zone when some or all of the shocks are programmed to Off, no further therapy will be delivered unless the arrhythmia accelerates to a higher zone. A Disable Therapy button is available in the VT or VT-1 zones' therapy window to quickly disable all ATP and Shock therapy in that zone.

² Ventricular ATP therapy can be programmed as Off, Burst, Ramp, Scan, or Ramp/Scan in VT-1 and VT zones.

Figure 3-1. Ventricular therapy prescription, 3-zone configuration

The therapies within a ventricular zone must be ordered in ascending therapy strengths. All ventricular ATP therapies are considered to be of equal strength, but are of lower strength than any shock therapy. The strength of the shock therapies is determined by the programmed energy. In a multi-zone configuration, therapies in a higher ventricular zone may be of lesser, greater, or equal strength to those in a lower ventricular zone; however, within each zone the therapies must be programmed in equal or increasing energy output.

Ventricular Therapy Selection

The pulse generator determines which ventricular therapy to deliver based on the following rules:

- Each successive therapy delivery must be greater than or equal to the strength of the previous therapy in a ventricular episode. Whenever a ventricular shock therapy has been delivered, no further ventricular ATP therapy is allowed in that episode since ATP therapy is of lower strength than shock therapy. Each subsequent ventricular shock delivery must be of equal or greater strength regardless of ventricular zone changes during a ventricular episode.
- Each ventricular ATP scheme (which may consist of multiple bursts) can only be delivered once during a ventricular episode.

- Up to 8 shocks may be delivered in a ventricular episode. The first 2 shocks are programmable. The following maximum-energy, non-programmable shocks are available in each zone:
 - VT-1 zone: 3 maximum-energy shocks
 - VT zone: 4 maximum-energy shocks
 - VF zone: 6 maximum-energy shocks

NOTE: In the event a shock is diverted with the *DIVERT THERAPY* programmer command, by magnet application or due to a *Diverted-Reconfirm*, the diverted shock is not counted as one of the available shocks for that tachyarrhythmia episode. Also, commanded therapies and *STAT SHOCK* are not counted as one of the available shocks for an episode and do not affect subsequent therapy selection.

Based on initial ventricular detection criteria, the pulse generator selects the first prescribed therapy in the ventricular zone in which the tachyarrhythmia is detected (i.e., detection is met; refer to "Ventricular Detection" on page 2-5). After delivering the selected therapy, the pulse generator begins redetection to determine whether the arrhythmia has been converted.

- If the arrhythmia is converted to a rate below the lowest programmed threshold, the pulse generator continues monitoring until the end of the episode is declared. When the episode ends, the pulse generator will again use initial ventricular detection criteria for a new episode. When a new episode is declared, the first prescribed therapy will be delivered again.
- If the arrhythmia is not converted and an arrhythmia is redetected in the same ventricular zone, the next programmed therapy in that zone is selected and delivered (Figure 3-2 on page 3-4), followed again by redetection. If the arrhythmia persists in the same zone, the therapy will progress in that zone.
- If an arrhythmia crosses ventricular zones (accelerates or decelerates) following therapy delivery and is redetected in a higher or lower ventricular zone, a therapy of equal or greater strength than the previously delivered therapy is selected from the detected zone and delivered. For shock therapy, the pulse generator determines which shock to deliver prior to capacitor charging based on the detected rate threshold. If during capacitor charging, the tachyarrhythmia accelerates or decelerates from the initial detected rate, the predetermined energy will still be delivered.

Refer to Figure 3-3 on page 3-4 through Figure 3-10 on page 3-6.

Redetection is performed after each therapy deliver to determine if further therapy is required. Use the following information when interpreting the therapy progression figures:

- After each redetection cycle, therapy delivery progresses in the direction indicated by the circled numbers.
- Upward sloping lines indicate acceleration of the arrhythmia to a higher ventricular zone.
- Downward sloping lines indicate deceleration into a lower ventricular zone.
- The lowest strength therapy is in the ATP columns; the therapy strengths increase as you move to the right in the table.

NOTE: In the VT-1 zone of a 3-zone configuration or the VT zone of a 2-zone configuration, one or two ATP schemes may be programmed as the only therapy, with all shocks in the lowest zone programmed to Off. If those pacing schemes do not terminate an arrhythmia detected in the VT-1 zone, no further therapy will be delivered in the episode unless the rate is redetected in a higher zone.

3-4 TACHYARRHYTHMIA THERAPY VENTRICULAR THERAPY

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			On/Off	5 J	11 J	max max max max max max
VT	Burst	Scan	N/A	3 J	9 J	max max max max
VT-1	① Burst	② Ramp	N/A	③ 0.1 J	④ 2 J	⑤ max ⑥ max ⑦ max

Figure 3-2. Therapy delivery progression, arrhythmia remains in same zone as initially detected

ATP1 in the VT zone is delivered because it is considered of equal strength to VT-1 ATP2 therapy.

When the rhythm accelerates to the VF zone, Shock 2 in the VF zone is delivered since Shock 1 is a lower energy level than Shock 1 in the VT zone.

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			On/Off	2 J	⑤ 11 J	max max max max max max
VT	③ Burst	Off	N/A	④ 3 J	9 J	⑥ max ⑦ max ⑧ max ⑨ max
VT-1	① Burst	② Ramp	N/A	0.1 J	2 J	max max max

Figure 3-3. Therapy delivery progression, ATP1 in the VT zone and shock 2 in the VF zone

When the rhythm accelerates back to the VT zone, ATP2 therapy is delivered because ATP1 has already been used during the episode.

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			On/Off	11 J	17 J	⑦ max ⑧ max ⑨ max ⑩ max ⑪ max ⑫ max
VT	① Burst	③ Scan	N/A	④ 5 J	9 J	⑥ max ⑦ max ⑧ max ⑨ max
VT-1	② Burst	Ramp	N/A	3 J	⑤ 5 J	max max max

Figure 3-4. Therapy delivery progression, ATP2 therapy

This is the third shock, since two programmable shocks have been delivered.

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			On/Off	5 J	11 J	⑥ max ⑦ max ⑧ max ⑨ max ⑩ max ⑪ max
VT	② Burst	③ Scan	N/A	④ 1.1 J	9 J	⑦ max ⑧ max ⑨ max
VT-1	① Burst	Ramp	N/A	⑤ 3 J	5 J	max max max

When the rhythm decelerates to the VT-1 zone, ATP2 of the VT-1 zone is not delivered since a shock had already been delivered in the VT zone. So the next higher strength therapy (Shock 1 of the VT-1 zone) is delivered.

Figure 3-5. Therapy delivery progression, shock 1 in the VT-1 zone

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			On/Off	2 J	11 J	max max max max max max
VT	Burst	Scan	N/A	3 J	9 J	max max max max
VT-1	① Burst	② Ramp	N/A	③ 0.1 J	④ 2 J	Off Off Off

If the arrhythmia persists in the VT-1 zone after the second shock delivery, no further shock therapy will be delivered unless the arrhythmia accelerates to a higher zone since Shocks 3-5 are programmed Off in the VT-1 zone.

Figure 3-6. Therapy delivery progression, shocks 3 to 5 programmed to Off in the VT-1 zone

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			On/Off	2 J	11 J	max max ⑦ max ⑧ max ⑨ ⑩ max max
VT	Burst	Off	N/A	3 J	9 J	⑤ max ⑥ max max max
VT-1	① Burst	② Ramp	N/A	③ 0.1 J	④ 2 J	max max max

The arrhythmia accelerated back to the VF zone, the seventh shock is delivered. The arrhythmia persists in the VF zone so the eighth (and final) shock is delivered.

A sixth shock is delivered since the arrhythmia is in the VF zone.

The arrhythmia decelerated to a lower zone, an additional shock would not be delivered until the arrhythmia accelerated back to the VF zone.

Figure 3-7. Therapy delivery progression, sixth shock delivered

If reconfirmation indicates the arrhythmia persists after delivery of QUICK CONVERT ATP, the device immediately begins charging for Shock 1.

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			① On	② 11 J	③ 21 J	max max max max max max
VT	Burst	Scan	N/A	3 J	9 J	max max max max
VT-1	Burst	Ramp	N/A	0.1 J	2 J	max max max

Figure 3-8. Therapy delivery progression, QUICK CONVERT ATP and shock in the VF zone

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			① On	2 J	11 J	max max max max max max
VT	② Burst	③ Scan	N/A	④ 3 J	⑤ 9 J	max max max max
VT-1	Burst	Ramp	N/A	0.1 J	2 J	max max max

ATP1 in the VT zone is delivered because it is considered of equal strength to QUICK CONVERT ATP therapy.

Figure 3-9. Therapy delivery progression, QUICK CONVERT ATP decelerates the rhythm, ATP1 and shock delivered in the VT zone

When the rhythm accelerates to the VF zone, Shock 1 is delivered because QUICK CONVERT ATP is only available as the first therapy in an episode.

Zone	ATP1	ATP2	QUICK CONVERT ATP	Shock 1	Shock 2	Remaining Shocks
VF			On	② 11 J	③ 21 J	max max max max max max
VT	① Burst	Scan	N/A	3 J	9 J	max max max max
VT-1	Burst	Ramp	N/A	0.1 J	2 J	max max max

Figure 3-10. Therapy delivery progression, ATP1 in VT zone accelerates the rhythm, QUICK CONVERT ATP is skipped in VF zone

Ventricular Redetection after Ventricular Therapy Delivery

After ventricular therapy delivery, the pulse generator uses redetection criteria to evaluate the rhythm and determine whether more therapy is appropriate. When redetection criteria are satisfied, the rules for therapy selection then determine the type of therapy to deliver.

Ventricular Redetection after Ventricular ATP Therapy

Ventricular Redetection after ventricular ATP therapy determines if an arrhythmia has been terminated.

As a ventricular ATP scheme is delivered, the pulse generator monitors the cardiac rate after each burst and uses ventricular detection windows (looking for 8 of 10 fast intervals) and the Ventricular Redetection Duration to determine if the arrhythmia has terminated.

The ATP scheme will continue with the next bursts in the sequence until any one of the following conditions is satisfied:

- Redetection declares that the therapy has been successful (end-of-episode)
- The specified number of ATP bursts in the scheme has been delivered
- The ATP Time-out for the ventricular zone has expired
- The detected ventricular arrhythmia rate changes to a different ventricular rate zone, whereby a different therapy is selected
- Shock If Unstable forces the device to skip the remaining ATP therapy and initiate shock therapy

- A DIVERT THERAPY command is received from the PRM during delivery of a burst of a scheme
- A magnet abort occurs during delivery of a scheme
- The temporary Tachy Mode has changed
- A commanded therapy is requested
- The episode ends due to reprogrammed Tachy Mode, reprogrammed ventricular tachy parameters, or attempted induction method or lead test

NOTE: *Aborting an ATP burst terminates the affected ATP scheme. If further therapy is required, the next programmed therapy (either ATP or shocks) in the prescription is initiated.*

Ventricular Redetection after Ventricular Shock Therapy

Ventricular redetection after ventricular shock therapy determines if an arrhythmia has been terminated.

As shock therapy is delivered, the pulse generator monitors the cardiac rate after each shock and uses ventricular detection windows (looking for 8 of 10 fast intervals) and post-shock detection enhancements, if applicable, to determine if the arrhythmia has been terminated. Shock therapy will continue until one of the following conditions is satisfied:

- Redetection declares the therapy has been successful (end-of-episode)
- All available ventricular shocks have been delivered for an episode
- The rhythm is redetected in either the VT or VT-1 zone, the available number of programmed shock(s) in those zones has been delivered and the arrhythmia stays in one of these lower zones

If all available shocks have been delivered for an episode, no further therapy is available until the pulse generator monitors a rate below the lowest rate threshold for 30 seconds and end-of-episode is declared.

ANTITACHYCARDIA PACING THERAPIES AND PARAMETERS

Antitachycardia Pacing (ATP) therapy and parameters enable the pulse generator to interrupt the following fast rhythms by delivering a series of critically timed pacing pulses:

- Monomorphic ventricular tachycardia
- Supraventricular tachycardias

ATP Therapy is delivered when the last sensed event fulfills the programmed detection criteria (Figure 3-11 on page 3-8).

An ATP scheme may be customized with the following parameters:

- Number of bursts delivered
- Number of pulses within each burst
- Coupling Interval
- Burst Cycle Length
- Minimum pacing interval

These parameters can be programmed to produce the following ATP therapy schemes:

- Burst
- Ramp
- Scan
- Ramp/Scan

The ATP amplitude and pulse width are common to all schemes. They are independently programmable from the normal pacing settings. The ATP amplitude and pulse width share the same programmable value as the post-therapy pacing settings.

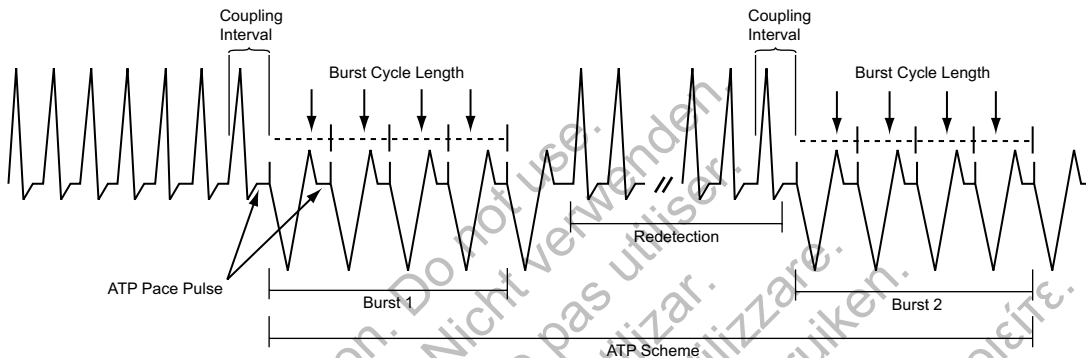


Figure 3-11. ATP therapy basic parameters are Coupling Interval, Burst Cycle Length, Number of Bursts, and Number of Pulses within each burst.

Burst Parameters

A burst is a series of critically timed pacing pulses delivered by the pulse generator during ATP therapy. By programming burst parameters, you can optimize ATP therapy for the patient.

All ATP schemes have several parameters in common. In addition to programming the type of scheme (Off, Burst, Ramp, Scan, Ramp/Scan), the following burst parameters are programmable (Figure 3-12 on page 3-9):

- The Number of Bursts parameter determines the number of bursts used in an ATP scheme and may be programmed independently for each ATP scheme. Programming the parameter to Off will deactivate the ATP scheme.
- The Initial Pulse Count parameter determines the number of pulses delivered in the first burst of a scheme.
- The Pulse Increment parameter determines the number of pulses per burst to be increased for each successive burst in the scheme.
- The Maximum Number of Pulses parameter determines the greatest number of pulses used in an ATP burst and may be programmed independently for each ATP scheme. After the maximum number of pulses is reached in a burst, each additional burst remaining in the scheme contains the programmed Maximum Number of Pulses. The parameter is available only if the Pulse Increment is greater than zero.

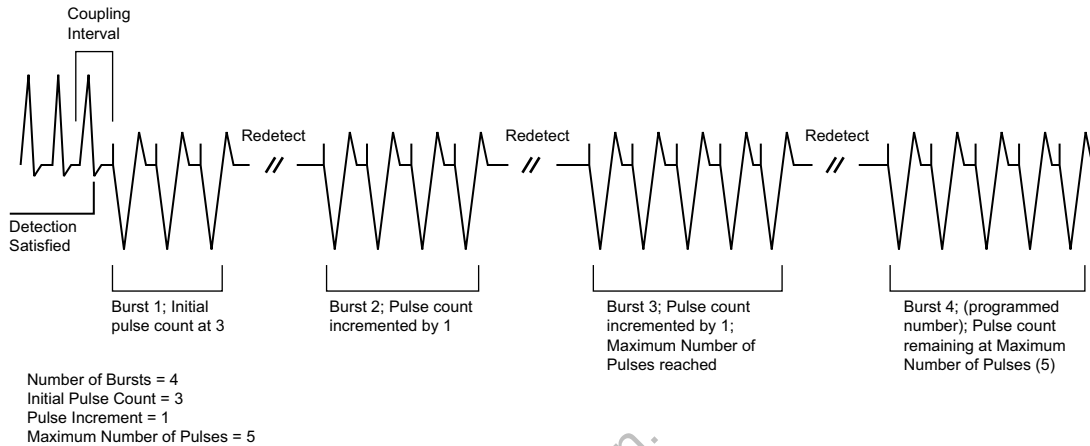


Figure 3-12. Interaction of Maximum Number of Pulses and Number of Bursts

Coupling Interval and Coupling Interval Decrement


The Coupling Interval controls the timing of the first pulse in a burst. It defines the time between the last sensed event that fulfills the detection criteria and delivery of the first pulse in a burst.

The Coupling Interval is programmed independent from the Burst Cycle Length. This allows aggressive ramps and scans to be used without compromising capture of the first pacing pulse in a burst. The Coupling Interval can be programmed as any of the following:

- Adaptive, with timing specified as percentages of the computed average heart rate
- A fixed interval, with timing specified in absolute time (ms) independent of the measured average rate

When programmed as adaptive, the Coupling Interval adjusts to the patient's rhythm based on a four-cycle average (Figure 3-13 on page 3-9). The Coupling Interval Decrement may be programmed such that the Coupling Interval decreases from one burst to the next within a multiple-burst scheme (Figure 3-14 on page 3-10).

NOTE: You cannot program an ATP burst that lasts longer than 15 seconds. The length of an adaptive burst is calculated based on the interval of the ventricular zone in which the ATP is programmed, which means it is based on worst-case timing.

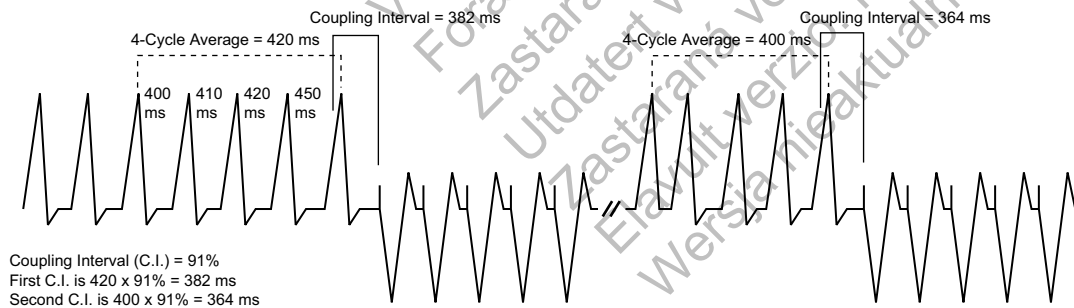

The 4-cycle average is calculated on the four cycles prior to each tachycardia therapy delivery only when no Decrement (Coupling Interval or Scan) is programmed.

Figure 3-13. Adaptive Coupling Interval, Coupling Interval Decrement and Scan Decrement programmed to 0

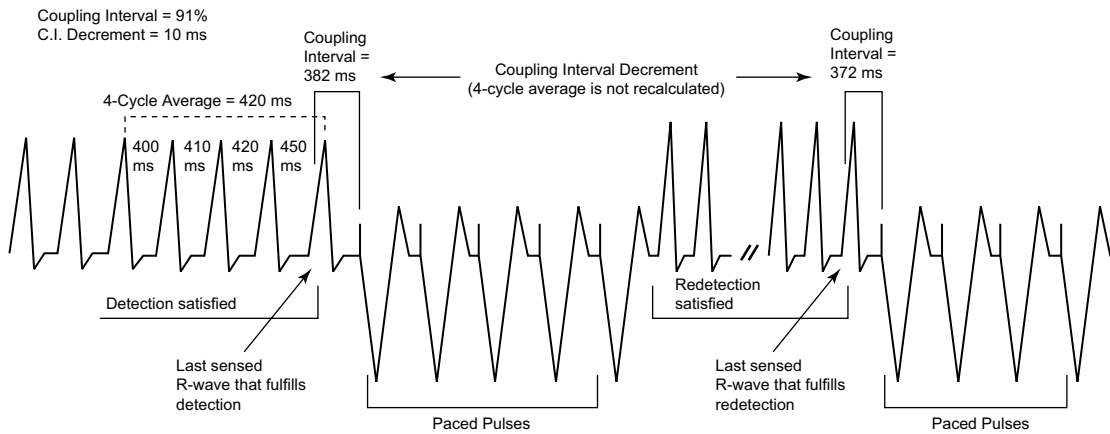


Figure 3-14. Coupling Interval Decrement

The following information should be taken into consideration when programming the Coupling Interval and Coupling Interval Decrement:

- When the Coupling Interval Decrement is programmed to On, the programmed ATP scheme is called a Scan
- When the Coupling Interval is programmed as adaptive, the Coupling Interval will not re-adapt following redetection when the following are programmed to On (greater than zero):
 - Coupling Interval Decrement—the decrement value determines the timing of the first pulse in subsequent bursts
 - Scan Decrement—the decrement value determines the timing of the second pulse in subsequent bursts

Burst Cycle Length (BCL)

The Burst Cycle Length controls the interval between pacing pulses after the Coupling Interval.

This timing is controlled in the same fashion as the Coupling Interval: rate adaptive to the sensed tachycardia or fixed time specified in ms.

NOTE: An adaptive BCL is affected in the same manner as an adaptive Coupling Interval; the average cycle length is not continually recalculated for subsequent bursts if the Scan Decrement or Coupling Interval Decrement are programmed to On.

The following parameters may be programmed to decrement the burst cycle length during an ATP scheme:

- Ramp Decrement controls the pulse timing within a given burst
- Scan Decrement controls the pulse timing between bursts

Minimum Interval

The Minimum Interval limits the Coupling Interval and the BCL in Burst, Ramp, and Scan.

If the Coupling Interval reaches the limit, subsequent Coupling Intervals will remain at the minimum value. Likewise, if the BCL reaches the limit, subsequent BCLs will remain at the minimum value. The Coupling Interval and BCL may reach the limit independently.

Burst Scheme

A Burst scheme is a sequence of critically timed pacing pulses intended to interrupt a reentrant loop, usually delivered at a rate faster than the patient's tachycardia.

An ATP scheme is defined as a Burst (as indicated on the PRM screen) when the timing of all pacing intervals within a burst is the same. The first BCL of each Burst is determined by the programmed BCL. When the number of pulses programmed in a Burst is greater than one, you can use the BCL to control the timing between these paced pulses (Figure 3-15 on page 3-11).

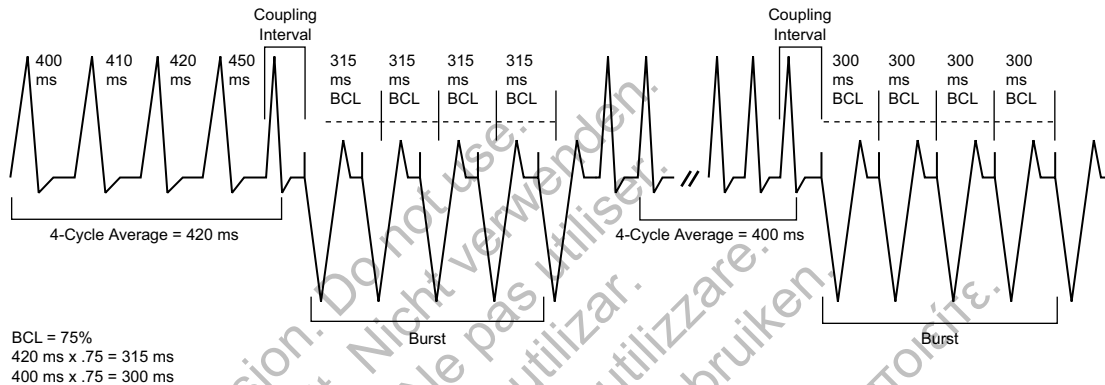


Figure 3-15. Adaptive-rate Burst scheme

Ramp Scheme

A Ramp scheme is a burst in which each paced-to-paced interval within the burst is shortened (decremented).

To program a Ramp scheme, program (in ms) the Ramp Decrement to specify how much the paced-to-paced interval should be shortened, and the Scan Decrement and Coupling Interval Decrement each to 0 ms. As each additional paced pulse in a burst is delivered, its interval is shortened by the programmed Ramp Decrement until either of the following occur:

- The last paced pulse of the burst is delivered
- The Minimum Interval is reached

If subsequent bursts are required, the programmed Ramp Decrement will be applied based on the calculated BCL of that subsequent burst (Figure 3-16 on page 3-12).

Burst Cycle Length = 75%
Ramp Decrement (R-R Within Burst) = 10 ms
Scan Decrement (R-R Between Bursts) = 0 ms
C.I. Decrement = 0 ms
Minimum Interval = 265 ms

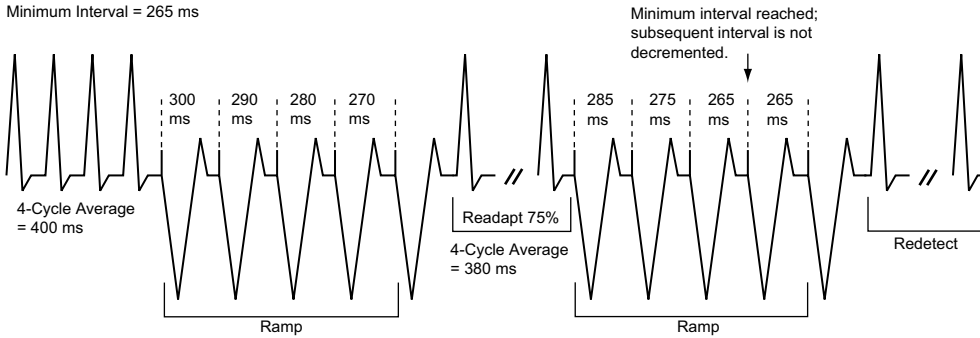


Figure 3-16. Adaptive Ramp Scheme, Coupling Interval Decrement and Scan Decrement programmed to 0

Scan Scheme

A Scan scheme is a burst in which the BCL of each burst in a scheme is systematically shortened (decremented) between successive bursts.

You can program a Scan scheme by programming the Scan Decrement to specify the BCL decrement to a value greater than 0 ms, while the Ramp Decrement is programmed to 0 ms. The BCL of subsequent bursts is determined by subtracting the Scan Decrement from the BCL of the previous burst (Figure 3-17 on page 3-12).

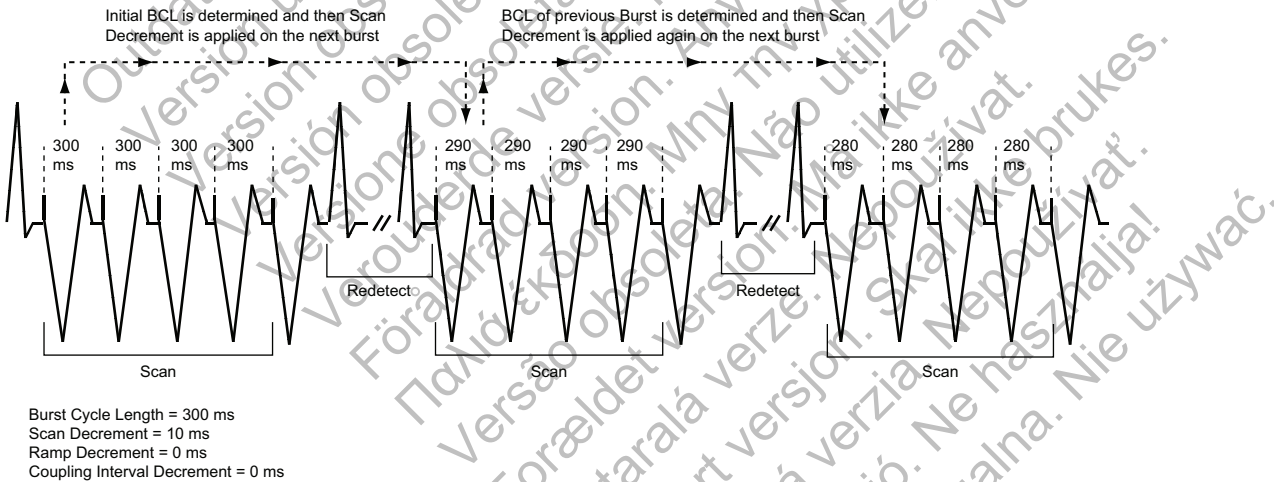


Figure 3-17. Scan scheme, nonadaptive BCL and Scan Decrement programmed on

Ramp/Scan Scheme

A Ramp/Scan scheme is a sequence of bursts. Each scheme contains a Ramp Decrement and a Scan Decrement (Figure 3-18 on page 3-13).

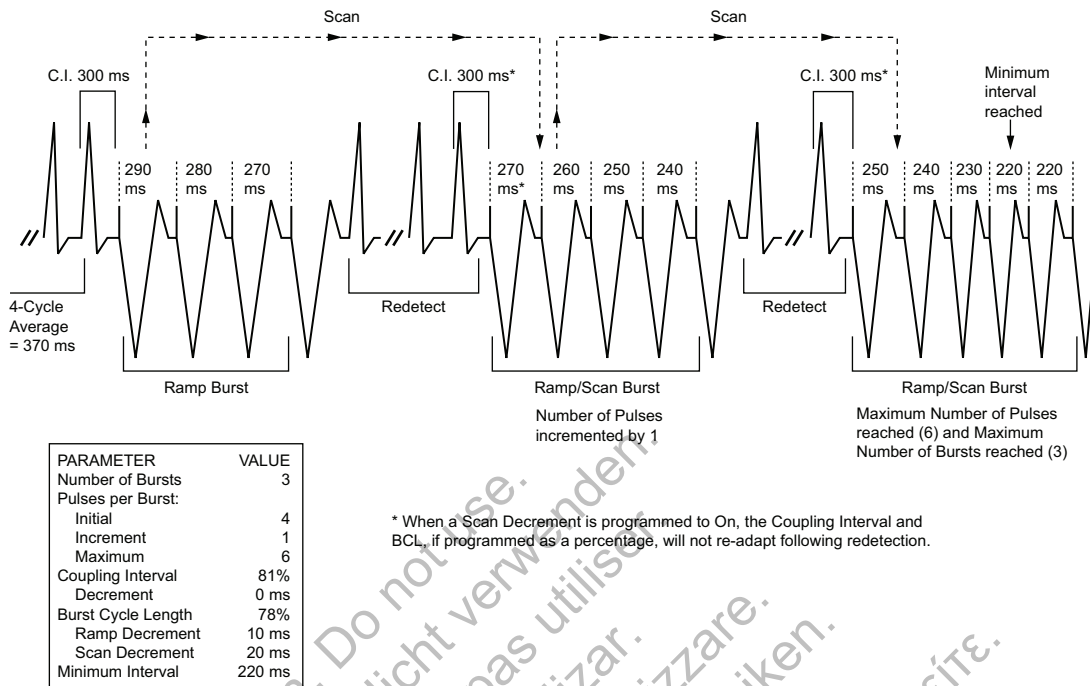


Figure 3-18. Ramp/Scan scheme, interaction of ATP parameters

To program a Ramp/Scan scheme, both the Scan Decrement and Ramp Decrement are programmed to values greater than 0 ms.

ATP Pulse Width and ATP Amplitude

The ATP Pulse Width is the duration of a pacing pulse. The ATP Amplitude is the leading edge voltage of a pacing pulse.

The ATP Pulse Width and ATP Amplitude parameters share the same value as the post therapy pacing Pulse Width and Amplitude. If the programmable value is changed for one parameter, that value will be reflected in the other parameters.

The programmed ATP Pulse Width and ATP Amplitude are shared for all ATP schemes regardless of zone and position in a prescription. The ATP amplitude and pulse width share the same programmable value as the post-therapy pacing settings.

Ventricular ATP Time-out

The Ventricular ATP Time-out forces the pulse generator to skip over any remaining ATP therapy in a ventricular zone to begin delivering ventricular shock therapy programmed in the same zone. This parameter is effective only for ventricular therapy delivery.

The ATP Time-out may be used in the VT or VT-1 zone as long as ATP therapy is programmed to On. Timer values are independent, although VT-1 ATP Time-out must be equal to or greater than the VT ATP Time-out.

The timer starts when the first burst is delivered and continues until any of the following occur:

- The timer expires (Figure 3-19 on page 3-14)
- A ventricular shock is delivered
- The ventricular episode ends

The time-out is examined after each redetection sequence to determine if further ATP bursts can be delivered. If the time-out has been reached or exceeded, further ATP therapy will not be initiated during that ventricular episode. The time-out will not terminate a burst in process.

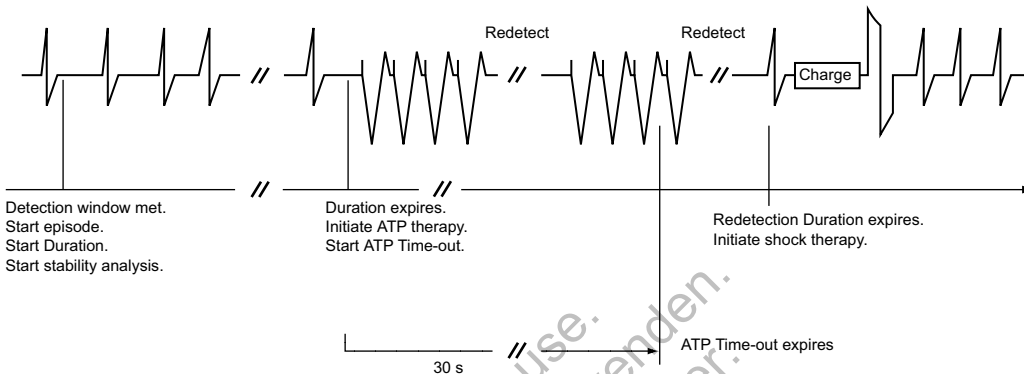


Figure 3-19. ATP Time-out expiration

NOTE: Once a ventricular shock has been delivered during a ventricular episode, ATP will no longer be invoked, irrespective of the time remaining on the ATP Time-out timer.

The timer alone does not invoke therapy; the rate and duration criteria and detection enhancements must still be satisfied in order for a shock therapy to be delivered.

If three zones are programmed, you may program ATP Time-out settings in each of the lower two ventricular zones (Figure 3-20 on page 3-14).

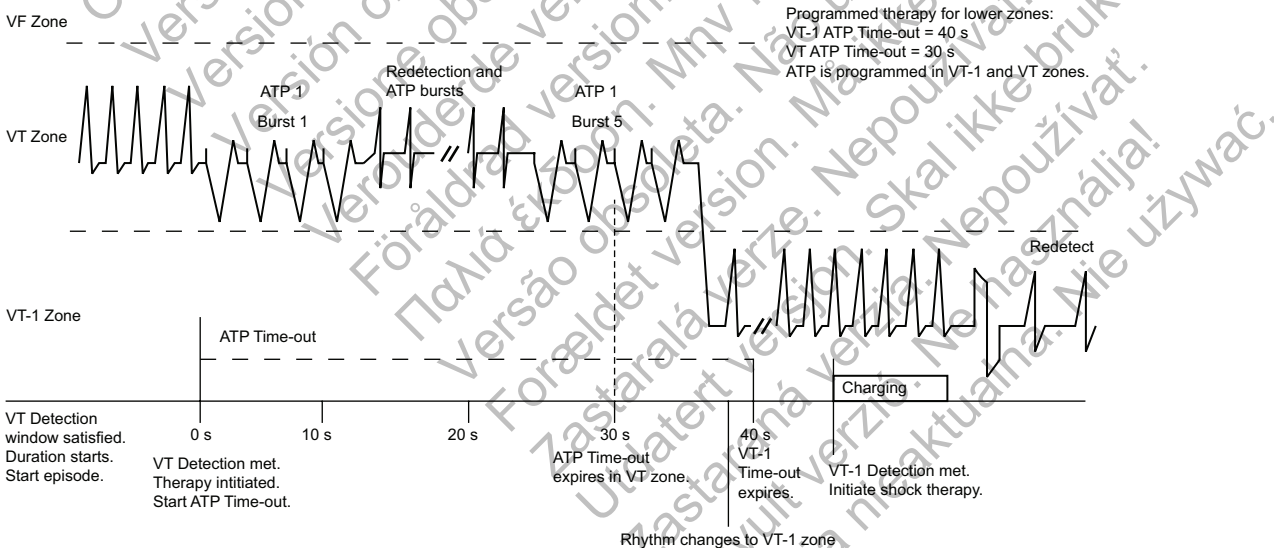


Figure 3-20. ATP Time-outs, 3-zone configuration

QUICK CONVERT ATP

QUICK CONVERT ATP provides an additional option to treat a fast, monomorphic VT that is detected in the VF zone, before advancing to shock therapy.

When QUICK CONVERT ATP is programmed to On, the pulse generator delivers one burst of ATP consisting of 8 pacing pulses with an 88% Coupling Interval and 88% BCL.

QUICK CONVERT ATP is used only as the first therapy attempted in an episode. In the event that QUICK CONVERT ATP is unsuccessful in converting the rhythm and shock therapy is required, the feature's algorithm minimizes the delay to begin charging by using reconfirmation to evaluate whether ATP therapy successfully treated the arrhythmia:

- If 2 out of 3 intervals following delivery of QUICK CONVERT ATP are faster than the lowest rate threshold, the attempt is considered unsuccessful and charging begins for a non-committed shock.
- If 2 out of 3 intervals are slow, shock therapy is diverted and the pulse generator enters redetection. If redetection is satisfied following a diverted shock, the next shock will be committed.

NOTE: QUICK CONVERT ATP is not applied to any rhythm above a maximum rate of 250 min⁻¹.

VENTRICULAR SHOCK THERAPY AND PARAMETERS

The pulse generator delivers shocks synchronous to a sensed event. The shock vector, energy level, and polarity of the shocks are programmable.

Ventricular Shock Vector

The programmed Ventricular Shock Vector indicates the vector of energy delivery for ventricular shock therapy.

The following programmable configurations are available:

- RV Coil to RA Coil and Can—this vector is also known as the V-TRIAD vector. It uses the metallic housing of the pulse generator as an active electrode (“hot can”) combined with the ENDOTAK two-electrode defibrillation lead. Energy is sent via a dual-current pathway from the distal shocking electrode to the proximal electrode and to the pulse generator case.
- RV Coil to Can—this vector uses the metallic housing of the pulse generator as an active electrode (“hot can”). Energy is sent from the distal shocking electrode to the pulse generator case. This configuration should be selected when using a single-coil lead.
- RV Coil to RA Coil—this vector removes the pulse generator case as an active electrode and is also known as a “cold can” vector. Energy is sent from the distal shocking electrode to the proximal electrode. This vector should never be used with a single-coil lead, as a shock will not be delivered.

Ventricular Shock Energy

Ventricular shock energy determines the strength of shock therapy delivered by the pulse generator.

Shock output remains constant over the lifetime of the pulse generator, regardless of changes in lead impedance or battery voltage. The constant output is accomplished by varying pulse width to adjust to changes in lead impedance.

The first two shocks in each ventricular zone can be programmed to optimize charge time, longevity, and safety margins. The remaining shock energies in each zone are nonprogrammable at the maximum-energy value.

Charge Time

Charge time is the time the pulse generator requires to charge for delivery of the programmed shock energy.

Charge time is dependent on the following:

- Programmed output energy level
- Battery condition
- Condition of the energy storage capacitors

Charge times increase as the pulse generator is programmed to higher energy output levels and as the battery depletes (Table 3-1 on page 3-16). If a charge time is greater than 15 seconds, the pulse generator schedules an automatic capacitor re-formation for one hour later. If the charge time during re-formation also exceeds 15 seconds, battery status is changed to Explant.

Capacitor deformation can occur during inactive periods and may result in a slightly longer charge time. To reduce the impact of capacitor deformation on charge time, the capacitors are automatically reformed.

Table 3-1. Typical charge time required at 37 degrees C at beginning of life

Energy Stored (J) ^a	Energy Delivered (J) ^b	Charge Time (seconds) ^c
11.0	10.0	1.9
17.0	15.0	3.0
26.0	22.0	4.8
41.0 ^d	35.0	8.8

- a. Values indicate the energy level stored on the capacitors and correspond to the value programmed for shock energy parameters.
 b. The energy delivered indicates the shock energy level delivered through the shocking electrodes.
 c. Charge times shown are at beginning of life after capacitor re-formation.
 d. HE.

Table 3-2. Typical maximum-energy charge time over life of pulse generator

Charge Remaining (Ah) ^a	Maximum-energy Charge Time Range (seconds)
1.8 to 0.7	8 to 10
0.7 to 0.1	10 to 13

- a. At explant, Charge Remaining is typically 0.15 Ah, and residual capacity is 0.12 Ah for single chamber devices and 0.13 Ah for dual chamber devices. These may vary depending on the amount of therapy delivered over the life of the pulse generator. Residual capacity is used to support device function between Explant and Battery Capacity Depleted indicators.

NOTE: The maximum-energy charge time ranges above are based upon theoretical electrical principles and verified bench testing only.

Waveform Polarity

Waveform polarity reflects the relationship between the leading edge voltages on the defibrillating output electrodes. All shocks will be delivered using a biphasic waveform. The peak shock voltage (V1) is 750 V at 41 J, 535 V at 21 J, and 37 V at 0.1 J (Figure 3-21 on page 3-17).

The selection of the shock polarity applies to all shocks delivered by the device. If the preceding shocks in a zone are unsuccessful, the last shock of that zone will be automatically delivered at an inverted polarity to the previous shock (initial or reversed) (Figure 3-22 on page 3-17).

CAUTION: For IS-1/DF-1 leads, never change the shock waveform polarity by physically switching the lead anodes and cathodes in the pulse generator header—use the programmable Polarity feature. Device damage or nonconversion of the arrhythmia post-operatively may result if the polarity is switched physically.

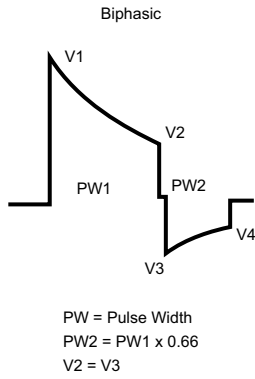


Figure 3-21. Biphasic waveform

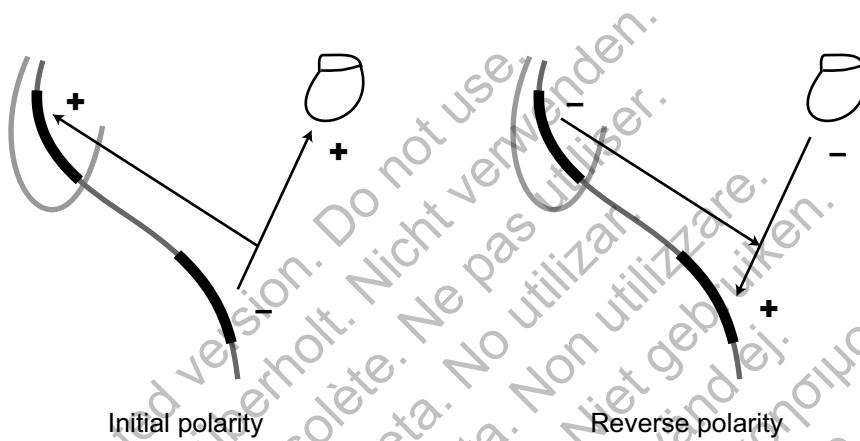


Figure 3-22. Polarity of shock delivery

Committed Shock/Reconfirmation of the Ventricular Arrhythmia

Committed Shock/Reconfirmation refers to the monitoring performed by the pulse generator before delivery of a ventricular shock.

If the patient is subject to non-sustained arrhythmias, reconfirmation may be desirable in order to prevent delivery of unnecessary shocks to the patient.

The device monitors tachyarrhythmias during and immediately following capacitor charging. During this time, it checks for the spontaneous conversion of the tachyarrhythmia and determines whether ventricular shock therapy should be delivered; it does not affect therapy selection.

Ventricular shock therapy can be programmed as committed or non-committed. If the Committed Shock feature is programmed to On, the shock is delivered synchronously with the first sensed R-wave following a 500-ms delay after the capacitors are charged, whether the arrhythmia is sustained or not (Figure 3-23 on page 3-18). The 500-ms delay allows a minimum time for a divert command to be issued from the PRM, if desired. If there is no sensed R-wave detected within 2 seconds following the end of charging, the ventricular shock is delivered asynchronously at the end of the 2-second interval.

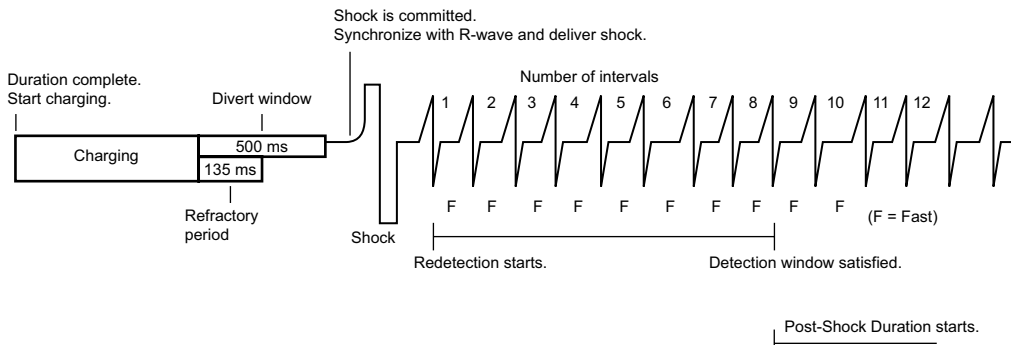


Figure 3-23. Committed Shock is programmed to On, Reconfirmation is Off

NOTE: There is a forced 135-ms refractory period following the end of charging; events that occur during the first 135 ms of the 500-ms delay are ignored.

If the Committed Shock feature is programmed to Off, Reconfirmation consists of the following steps:

1. During capacitor charging, the pulse generator continues to sense the arrhythmia. Sensed and paced beats are evaluated. If 5 slow beats (sensed or paced) are counted in a 10-beat detection window (or 4 consecutive slow beats after an unsuccessful QUICK CONVERT ATP attempt), the pulse generator stops charging and considers this a Diverted-Reconfirm.
2. If 5 of 10 beats are not detected as slow (or less than 4 consecutive slow beats after an unsuccessful QUICK CONVERT ATP attempt) and charging completes, post-charge reconfirmation is performed after charging ends. After the post-charge refractory and the first sensed event, the pulse generator measures up to 3 intervals following charging and compares them to the lowest rate threshold.
 - If 2 of the 3 intervals following charging are faster than the lowest rate threshold, the shock will be delivered synchronously with the second fast event.
 - If 2 of the 3 intervals following charging are slower than the lowest rate threshold, the shock will not be delivered. If no beats are sensed, pacing will begin at the programmed LRL following the 2-second no-sense period. If a shock is not delivered, or if pacing pulses are delivered, this is also considered a Diverted-Reconfirm.

If a shock is required after redetection, the charge time for the shock may be short.

The reconfirmation algorithm will not allow two consecutive Diverted-Reconfirm cycles. If the arrhythmia is detected after a Diverted-Reconfirm, the next shock in the episode is delivered as if Committed Shock were programmed to On. Once a shock has been delivered, the reconfirmation algorithm can be applied again (Figure 3-24 on page 3-18).

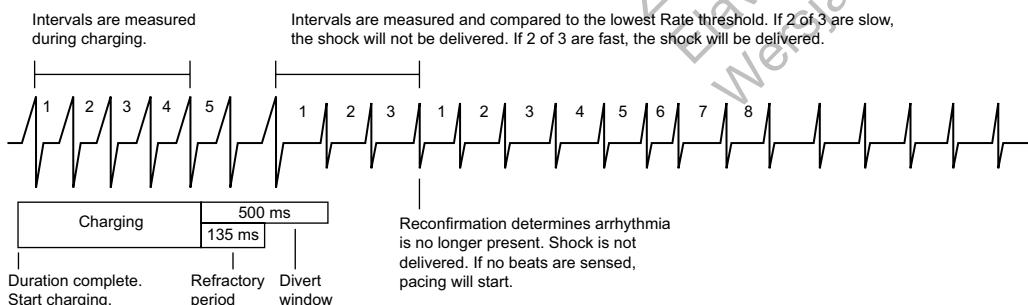


Figure 3-24. Committed Shock is programmed to Off, reconfirmation is On

PACING THERAPIES

CHAPTER 4

This chapter contains the following topics:

- "Pacing Therapies" on page 4-2
- "Basic Parameters" on page 4-2
- "Post-Therapy Pacing" on page 4-11
- "Temporary Brady Pacing" on page 4-12
- "Rate Adaptive Pacing and Sensor Trending" on page 4-13
- "Atrial Tachy Response" on page 4-19
- "Rate Enhancements" on page 4-25
- "Lead Configuration" on page 4-28
- "AV Delay" on page 4-28
- "Refractory" on page 4-33
- "Noise Response" on page 4-39
- "Ventricular Tachy Sensing Interactions" on page 4-41

PACING THERAPIES

Single-chamber ICDs provide ventricular bipolar (pace/sense) normal and post-therapy bradycardia pacing, including adaptive-rate modes. Dual-chamber ICDs provide both atrial and ventricular bipolar (pace/sense) normal and post-therapy bradycardia pacing, including adaptive-rate modes.

The bradycardia pacing function is independent of the tachycardia detection and therapy functions of the device, with the exception of interval-to-interval sensing.

The pulse generator provides the following types of therapies:

Normal Bradycardia Pacing

- If the intrinsic heart rate falls below the programmed pacing rate (i.e., LRL), the device delivers pacing pulses at the programmed settings.
- Sensor-based rate modulation allows the pulse generator to adapt the pacing rate to the patient's changing activity levels.

Post-Therapy Pacing—alternative bradycardia pacing therapy may be delivered for a programmed period to ensure capture after delivery of a shock.

Additional Options

- Temporary Bradycardia Pacing—allows the clinician to examine alternate therapies while maintaining the previously programmed normal pacing settings in the pulse generator memory ("Temporary Brady Pacing" on page 4-12).
- STAT PACE—initiates emergency ventricular pacing at high output settings when commanded via the PRM using telemetry communication ("STAT PACE" on page 1-14).

BASIC PARAMETERS

Normal Settings include the following:

- Pacing parameters, which are independently programmable from post-therapy and temporary pacing parameters
- Pacing and Sensing
- Leads
- Sensors and Trending

Post-Therapy Settings include the following:

- Pacing parameters, which are independently programmable from normal and temporary pacing parameters
- Post-ventricular shock

Interactive Limits

Because many features with programmable parameters interact, programmed values must be compatible across such features. When values requested by the user are incompatible with existing parameters, the programmer screen displays an alert describing the incompatibility and either prohibits the selection or instructs the user to proceed with caution ("Use of Color" on page 1-7).

Brady Mode

Brady modes provide programmable options to help individualize patient therapy.

This pulse generator includes the pacing modes identified in the Programmable Options appendix.

DDD and DDDR

In the absence of sensed P- and R-waves, pacing pulses will be delivered to the atrium and the ventricle at the LRL (DDD) or the sensor-indicated rate (DDDR), separated by the AV Delay. A sensed P-wave will inhibit an atrial pace and start the AV Delay. At the end of the AV Delay, a ventricular pace will be delivered unless inhibited by a sensed R-wave.

DDI and DDIR

In the absence of sensed P- and R-waves, pacing pulses will be delivered to the atrium and the ventricle at the LRL (DDI) or the sensor-indicated rate (DDIR), separated by the AV Delay. A sensed P-wave will inhibit an atrial pace but will not start the AV Delay.

VDD and VDDR

In the absence of sensed P- and R-waves, pacing pulses will be delivered to the ventricle at the LRL (VDD) or the sensor-indicated rate (VDDR). A sensed P-wave will start the AV Delay. At the end of the AV Delay, a ventricular pace will be delivered unless inhibited by a sensed R-wave. A sensed R-wave or a paced ventricular event will determine the timing of the next ventricular pace.

VVI and VVIR

In VVI(R) mode, sensing and pacing occur only in the ventricle. In the absence of sensed events, pacing pulses will be delivered to the ventricle at the LRL (VVI) or the sensor-indicated rate (VVIR). A sensed R-wave or a paced ventricular event will determine the timing of the next ventricular pace.

AAI and AAIR

In AAI(R) mode, sensing and pacing occur only in the atrium. In the absence of sensed events, pacing pulses will be delivered to the atrium at the LRL (AAI) or the sensor-indicated rate (AAIR). A sensed P-wave or a paced atrial event will determine the timing of the next atrial pace.

Dual-Chamber Modes

Do not use DDD(R) and VDD(R) modes in the following situations:

- In patients with chronic refractory atrial tachyarrhythmias (atrial fibrillation or flutter), which may trigger ventricular pacing
- In the presence of slow retrograde conduction that induces PMT, which cannot be controlled by reprogramming selective parameter values

Atrial Pacing Modes

In DDD(R), DDI(R), and AAI(R) modes, atrial pacing may be ineffective in the presence of chronic atrial fibrillation or flutter or in an atrium that does not respond to electrical stimulation. In addition, the presence of clinically significant conduction disturbances may contraindicate the use of atrial pacing.

WARNING: Do not use atrial tracking modes in patients with chronic refractory atrial tachyarrhythmias. Tracking of atrial arrhythmias could result in ventricular tachyarrhythmias.

NOTE: If a separate pacemaker is desired, a dedicated bipolar pacemaker is recommended ("Pacemaker Interaction" on page B-1).

NOTE: Refer to "Use of Atrial Information" on page 2-4 for additional information about device performance when the atrial lead is programmed to Off.

If you have any questions regarding the individualization of patient therapy, contact Boston Scientific using the information on the back cover.

Lower Rate Limit (LRL)

LRL is the number of pulses per minute at which the pulse generator paces in the absence of sensed intrinsic activity.

As long as the ventricle is being paced (or if a PVC occurs), the interval is timed from one ventricular event to the next. Whenever an event is sensed in the ventricle (e.g., intrinsic AV conduction occurs before the AV Delay elapses), the timing base switches from ventricular-based timing to modified atrial-based timing (Figure 4-1 on page 4-4). This switching of timing base ensures accurate pacing rates since the difference between the intrinsic AV conduction and programmed AV Delay is applied to the next V–A interval.

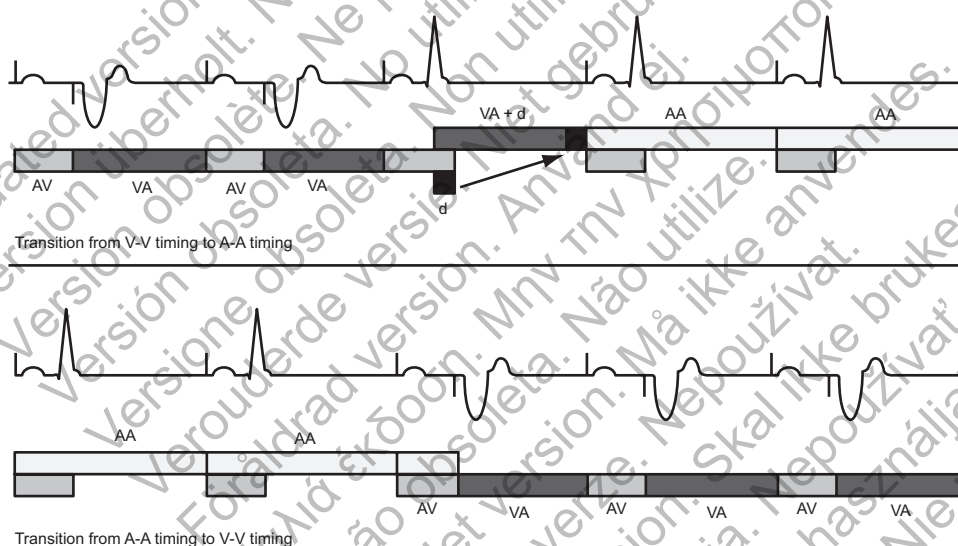


Illustration of timing transitions (d = the difference between AV Delay and the AV interval in the first cycle during which intrinsic conduction occurs. The value of d is applied to the next V–A interval to provide a smooth transition without affecting A–A intervals).

Figure 4-1. LRL timing transitions

Maximum Tracking Rate (MTR)

The MTR is the maximum rate at which the paced ventricular rate tracks 1:1 with nonrefractory sensed atrial events in the absence of a sensed ventricular event within the programmed AV Delay. MTR applies to atrial synchronous pacing modes, namely DDD(R) and VDD(R).

Consider the following when programming MTR:

- The patient's condition, age, and general health
- The patient's sinus node function

- A high MTR may be inappropriate for patients who experience angina or other symptoms of myocardial ischemia at higher rates

NOTE: If the pulse generator is operating in DDDR or VDDR mode, the MSR and MTR may be programmed independently to different values.

Upper Rate Behavior

When the sensed atrial rate is between the programmed LRL and MTR, 1:1 ventricular pacing will occur in the absence of a sensed ventricular event within the programmed AV Delay. If the sensed atrial rate exceeds the MTR, the pulse generator begins a Wenckebach-like behavior to prevent the paced ventricular rate from exceeding the MTR. This Wenckebach-like behavior is characterized by a progressive lengthening of the AV delay until an occasional P-wave is not tracked because it falls into the PVARP. This results in an occasional loss of 1:1 tracking as the pulse generator synchronizes its paced ventricular rate to the next sensed P-wave. Should the sensed atrial rate continue to increase further above the MTR, the ratio of sensed atrial events to sequentially paced ventricular events becomes lower until, eventually, 2:1 block results (e.g., 5:4, 4:3, 3:2, and finally 2:1).

The sensing window should be maximized by programming the appropriate AV Delay and PVARP. At rates close to the MTR, the sensing window can be maximized by programming Dynamic AV Delay and Dynamic PVARP, and Wenckebach behavior will be minimized.

High rate atrial tracking is limited by the programmed MTR and the total atrial refractory period (TARP) ($AV\ Delay + PVARP = TARP$). In order to avoid complete closure of the sensing window at MTR, the PRM will not allow a TARP interval that is longer (lower pacing rate) than the programmed MTR interval.

If the TARP interval is shorter (higher pacing rate) than the interval of the programmed MTR, then the pulse generator's Wenckebach-like behavior limits the ventricular pacing rate to the MTR. If the TARP interval is equal to the interval of the programmed MTR, 2:1 block may occur with atrial rates above the MTR.

The PRM does not consider the AV Delay associated with AV Search+ when calculating the TARP interval ("AV Search+" on page 4-31).

Rapid changes in the paced ventricular rate (e.g., Wenckebach-like, 2:1 block) caused by sensed atrial rates above the MTR may be dampened or eliminated by the implementation of any of the following:

- AFR
- ATR
- Rate Smoothing parameters and sensor input

NOTE:

For the purpose of atrial tachycardia detection and histogram updates, atrial events are detected throughout the cardiac cycle (except during atrial blanking), including AV Delay and PVARP.

Examples

If the atrial rate exceeds the MTR, the AV Delay will be progressively lengthened (AV') until an occasional P-wave is not tracked because it falls into the atrial refractory period (Figure 4-2 on page 4-6). This results in occasional loss of 1:1 tracking as the pulse generator synchronizes its paced ventricular rate to the next tracked P-wave (pacemaker Wenckebach).

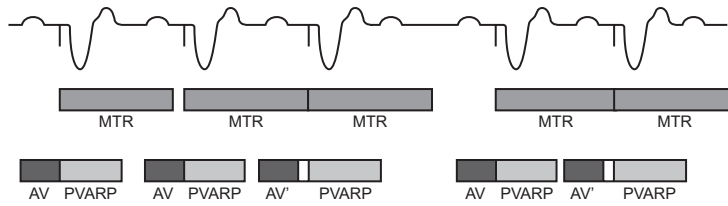


Figure 4-2. Wenckebach behavior at MTR

Another type of pulse generator upper rate behavior (2:1 block) can occur when tracking high atrial rates. In this type of behavior, every other intrinsic atrial event occurs during PVARP and, thus, is not tracked (Figure 4-3 on page 4-6). This results in a 2:1 ratio of atrial-to-ventricular events or a sudden drop in the ventricular paced rate to half of the atrial rate. At faster atrial rates, several atrial events can fall in the TARP period, resulting in the pulse generator tracking only every third or fourth P-wave. The block then occurs at rates such as 3:1 or 4:1.

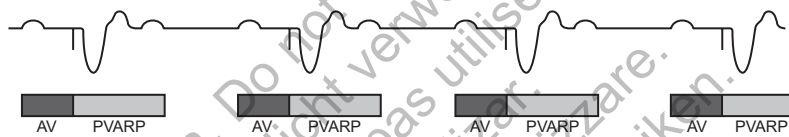


Illustration of pacemaker 2:1 block, in which every other P-wave falls inside the PVARP interval.

Figure 4-3. Pacemaker 2:1 block

Maximum Sensor Rate (MSR)

MSR is the maximum pacing rate allowed as a result of rate-adaptive sensor control from accelerometer input.

Consider the following when programming MSR:

- Patient's condition, age, and general health:
 - Adaptive-rate pacing at higher rates may be inappropriate for patients who experience angina or other symptoms of myocardial ischemia at these higher rates
 - An appropriate MSR should be selected based on an assessment of the highest pacing rate that the patient can tolerate well

NOTE: If the pulse generator is operating in DDDR or VDDR mode, the MSR and MTR may be programmed independently to different values.


MSR is independently programmable at, above, or below the MTR. If the MSR setting is higher than the MTR, pacing above the MTR may occur if the sensor rate exceeds the MTR.

Pacing above the MSR (when programmed lower than the MTR) can only occur in response to sensed intrinsic atrial activity.

CAUTION: Adaptive-rate pacing is not limited by refractory periods. A long refractory period programmed in combination with a high MSR can result in asynchronous pacing during refractory periods since the combination can cause a very small sensing window or none at all. Use dynamic AV Delay or dynamic PVARP to optimize sensing windows. If you are entering a fixed AV delay, consider the sensing outcomes.

With intrinsic conduction, the pulse generator maintains the A–A pacing rate by extending the V–A interval. This extension is determined by the degree of difference between the AV

Delay and the intrinsic ventricular conduction—often referred to as modified atrial-based timing (Figure 4-4 on page 4-7).

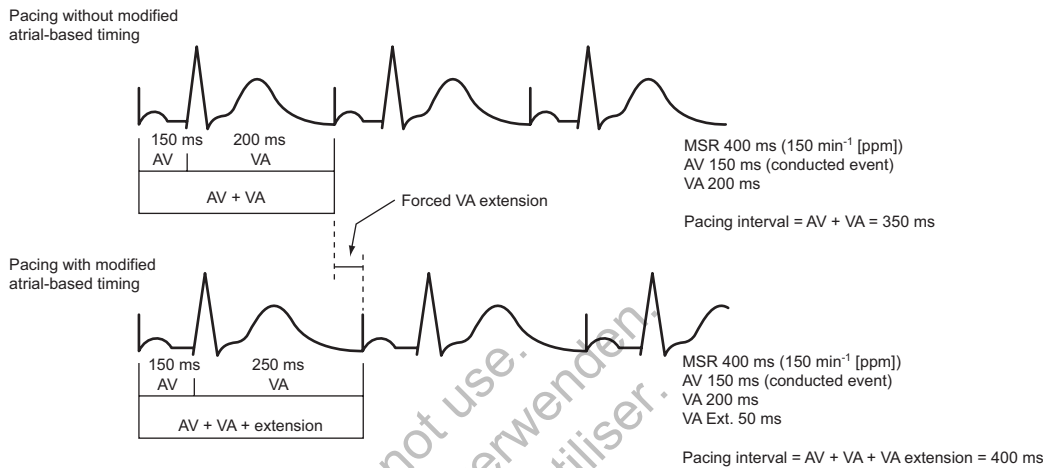

The pulse generator's timing algorithm provides effective pacing at the MSR with intrinsic ventricular conduction. Extending the VA interval prevents the A pace from exceeding the MSR at high rates.

Figure 4-4. VA interval extension and MSR

Runaway Protection

Runaway protection is designed to prevent pacing rate accelerations above the MTR/MSR for most single-component failures. This feature is not programmable and operates independently from the pulse generator's main pacing circuitry.

Runaway protection prevents the pacing rate from increasing above 205 min⁻¹.

NOTE: Magnet application does not affect the pacing rate (pulse interval).

NOTE: Runaway protection is not an absolute assurance that runaways will not occur.

During PES, Manual Burst pacing, and ATP, runaway protection is temporarily suspended to allow for high-rate pacing.

Pulse Width

Pulse Width, also referred to as pulse duration, determines how long the output pulse will be applied between the pacing electrodes.

Consider the following when programming Pulse Width:

- Pulse Widths are independently programmable for each chamber.
- If a pulse width threshold test is performed, a minimum 3X pulse width safety margin is recommended.
- The energy delivered to the heart is directly proportional to the pulse width; doubling the Pulse Width doubles the energy delivered. Therefore, programming a shorter pulse width while maintaining an adequate safety margin may increase battery longevity. To prevent loss of capture, exercise caution when you are programming permanent pulse width values of less than 0.3 ms (Figure 4-5 on page 4-8).

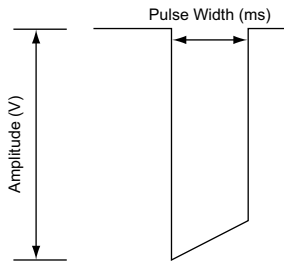


Figure 4-5. Pulse waveform

Amplitude

The pulse Amplitude, or voltage of the output pulse, is measured at the leading edge of the output pulse (Figure 4-5 on page 4-8).

Consider the following when programming Amplitude:

- Amplitudes are independently programmable for each chamber.
- Brady Mode may be programmed to Off via permanent or temporary programming. In effect, this turns Amplitude off to monitor the patient's underlying rhythm.
- A minimum 2X voltage safety margin is recommended for each chamber based on the capture thresholds, which should provide an adequate safety margin and help preserve battery longevity.
- The energy delivered to the heart is directly proportional to the square of the amplitude: doubling the amplitude quadruples the energy delivered. Therefore, programming to a lower Amplitude while maintaining an adequate safety margin may increase battery longevity.

Sensitivity

The Sensitivity feature allows the pulse generator to detect intrinsic cardiac signals that exceed the programmed sensitivity value. Adjusting the Sensitivity value allows you to shift the atrial and/or ventricular sensing range to higher or lower sensitivity. All detection and timing decisions are based on the sensed cardiac signals. Atrial and ventricular Sensitivity values are independently programmable.

- High Sensitivity (low programmed value)—when Sensitivity is programmed to a very sensitive setting, the pulse generator may detect signals unrelated to cardiac depolarization (oversensing, such as sensing of myopotentials).
- Low Sensitivity (high programmed value)—when Sensitivity is programmed to a less sensitive setting, the pulse generator may not detect the cardiac depolarization signal (undersensing).

It is recommended that the Sensitivity parameter settings be left at the nominal values unless troubleshooting determines that another value may be more appropriate. While the nominal value is primarily indicated for both atrial and ventricular sensing, an adjustment can be made if, in a rare situation, atrial or ventricular oversensing/undersensing has been observed (e.g., inhibition of bradycardia pacing or inappropriate therapy)

Should it become necessary to adjust the Sensitivity parameter in a chamber, always choose the setting that provides appropriate sensing of intrinsic activity and best resolves oversensing/undersensing.

If proper sensing cannot be restored with an adjustment or if any undersensing or oversensing is observed after making a change, consider any of the following (taking into account individual patient characteristics):

- Reprogram the AGC sensitivity value
- Reprogram the refractory or cross-chamber blanking period appropriately to address the observed undersensing or oversensing
- Reposition the lead
- Implant a new sensing lead

After any change to Sensitivity, evaluate the pulse generator for appropriate sensing and pacing.

CAUTION: Following any sensing range adjustment or any modification of the sensing lead, always verify appropriate sensing. Programming Sensitivity to the highest value (lowest sensitivity) may result in delayed detection or undersensing of cardiac activity. Likewise, programming to the lowest value (highest sensitivity) may result in oversensing of non-cardiac signals.

Automatic Gain Control

The pulse generator uses digital Automatic Gain Control (AGC) to dynamically adjust the sensitivity in both the atrium and the ventricle. The pulse generator has independent AGC circuits for each chamber.

Cardiac signals can vary widely in size and rate; therefore the pulse generator needs the ability to:

- Sense an intrinsic beat, regardless of rate or size
- Adjust to sense varying amplitude signals, but not overreact to aberrant beats
- Sense any intrinsic activity after a paced beat
- Ignore T-waves
- Ignore noise

The programmable AGC value is the minimum sensitivity value (floor) that could be reached between one beat and the next beat. This programmable value is not a fixed value present throughout the cardiac cycle; rather, the sensitivity level begins at a higher value (based on the peak of a sensed event or a fixed value for a paced event) and decrements towards the programmed floor (Figure 4-6 on page 4-11).

AGC will typically reach the programmable floor during pacing (or with low amplitude signals). But when moderate or high amplitude signals are sensed, AGC will typically be less sensitive and not reach the programmable floor.

The AGC circuit in each respective chamber processes an electrogram signal via a two step process to optimize sensing of potentially rapidly changing cardiac signals. The process is illustrated in the figure below (Figure 4-6 on page 4-11):

- First step
 1. AGC uses a rolling average of previous signal peaks to calculate a search area where the next peak will likely occur.
 - If the previous beat is sensed, it is incorporated into the rolling peak average.
 - If the previous beat is paced, the peak average is calculated using the rolling average and a paced peak value. The paced peak value depends on the settings:
 - For nominal or more sensitive settings, it is a fixed value (initial value 4.8 mV in the RV; initial value 2.4 mV in the RA).
 - For less sensitive settings, it is a higher value calculated using the programmed AGC floor value (for example, if RV sensitivity is programmed to the least sensitive setting or the highest value of 1.5 mV, the paced peak value = 12 mV).

The peak average is then used to bound an area with MAX (maximum) and MIN (minimum) limits.

- Second step
 2. AGC senses the peak of the intrinsic beat (or uses the calculated peak for a paced beat as described above).
 3. It holds the sensitivity level at the peak (or MAX) through the absolute refractory period + 15 ms.
 4. It drops to 75% of the sensed peak or calculated peak average for paced events (ventricular paced events only).
 5. AGC becomes more sensitive by 7/8 of the previous step.
 6. Sensed beat steps are 35 ms for the RV and 25 ms for the atrium. Paced beat steps are adjusted based on the pacing interval to ensure an approximately 50 ms sensing window at the MIN level.
 7. It reaches the MIN (or programmed AGC floor).
 - The programmed AGC floor will not be reached if the MIN value is higher.
 8. The AGC remains at the MIN (or programmed AGC floor) until a new beat is sensed, or the pacing interval times out and a pace is delivered.

NOTE: If a new beat is sensed as the sensitivity level steps down, AGC starts over at Step 1.

NOTE: If the amplitude of a signal is below the sensitivity threshold in effect at the time the signal occurs, it will not be sensed.

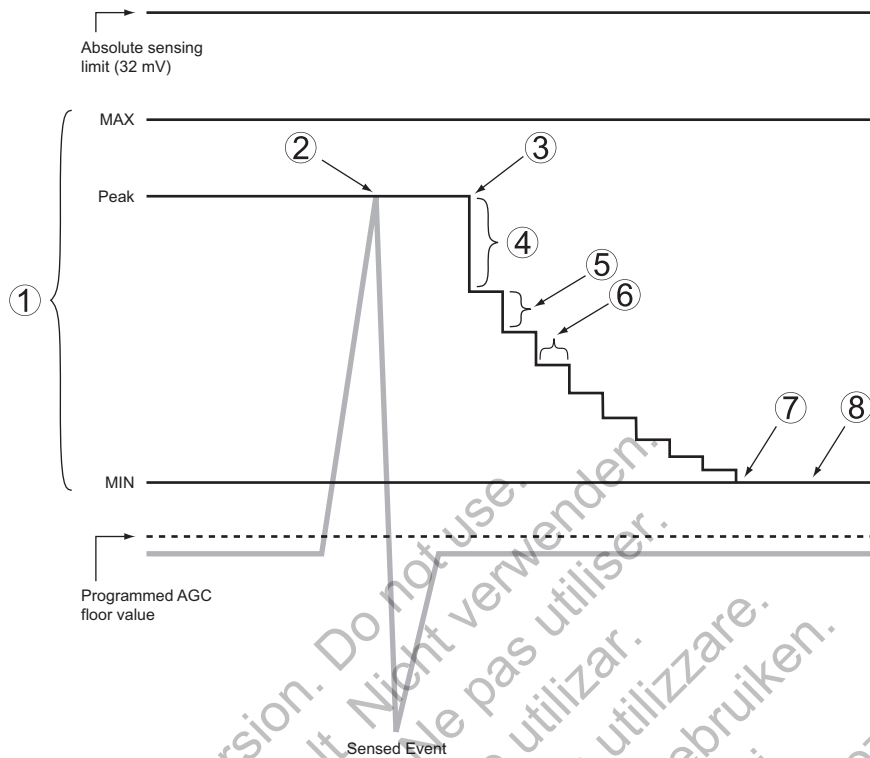


Figure 4-6. AGC sensing

A nonprogrammable Dynamic Noise Algorithm is active in rate channels where AGC sensing is used. The Dynamic Noise Algorithm is intended to help filter out persistent noise. The Dynamic Noise Algorithm is a separate noise channel for each chamber that continuously measures the baseline signal that is present and is designed to adjust the sensitivity floor to minimize the effects of noise.

The algorithm uses the characteristics of a signal (frequency and energy) to classify it as noise. When persistent noise is present, the algorithm is designed to minimize its impact, which may help to prevent oversensing myopotentials and the associated inhibition of pacing. Noise that affects the sensing floor may be visible on the intracardiac EGMs, but would not be marked as sensed beats. However, if the noise is significant, the floor may rise to a level above the intrinsic electrogram and the programmed Noise Response behavior (asynchronous pacing or Inhibit Pacing) will occur ("Noise Response" on page 4-39).

NOTE: The Dynamic Noise Algorithm does not ensure that AGC will always accurately distinguish intrinsic activity from noise.

POST-THERAPY PACING

Post-therapy pacing provides alternate pacing therapy following the delivery of any shock.

The pacing mode and pacing therapies used following a shock are the same as the programmed Normal pacing settings.

The following pacing parameters can be programmed independently from the Normal pacing settings:

- Pacing Parameters—LRL, Amplitude, and Pulse Width
- Post Therapy Period

Post-Shock Pacing Delay

The Post-Shock Pacing Delay determines the earliest possible start of post-shock pacing following the delivery of a ventricular shock and is fixed at 2.25 seconds.

The timing of the initial pacing pulse in the Post-Therapy Period depends on the cardiac activity during the Post-Shock Pacing Delay.

- If R-waves (and/or P-waves for dual-chamber pacing modes) are sensed during the Post-Shock Pacing Delay, the device paces only when the sensed rate is slower than the post-therapy LRL.
- If no R-waves (and/or P-waves for dual-chamber pacing modes) are sensed during the Post-Shock Pacing Delay or if the interval since the preceding P- or R-wave was greater than the escape interval, a pacing pulse is delivered at the end of the Post-Shock Pacing Delay.

Subsequent pacing pulses are delivered as required, depending on the pacing prescription.

Post-Therapy Period

The Post-Therapy Period determines how long the pulse generator operates using the post-therapy parameter values.

The Post-Therapy Period functions as follows:

- The period starts when the Post-Shock Pacing Delay expires
- On completion of this pacing period, the pulse generator reverts to the programmed Normal pacing values
- While in process, the pacing period is not affected by the end of the current episode

TEMPORARY BRADY PACING

The pulse generator can be programmed with temporary pacing parameter values that differ from the programmed Normal Settings. This allows you to examine alternate pacing therapies while maintaining the previously programmed Normal Settings in the pulse generator memory. During the Temporary function, all other bradycardia features not listed on the screen are disabled.

NOTE: *Post-therapy values are not affected.*

To use this function, follow these steps:

1. From the Tests tab, select the Temp Brady tab to display the temporary parameters. When the parameters are initially displayed, they are set to the Normal Settings values ("Programmable Options" on page A-1).

NOTE: *Post-therapy values are not shown even if post-therapy is presently in effect.*

2. Select the desired values; these values are independent from other pacing functions.

NOTE: *Any interactive limits must be corrected before Temporary programming can occur.*

NOTE: *If Off is selected as the Temporary Brady Mode, the pulse generator will not sense or pace while Temporary pacing mode is in effect.*

3. Establish telemetry communication, then select the Start button. Pacing begins at the temporary values. A dialog box indicates that temporary parameters are being used, and a Stop button is provided.

NOTE: Temporary pacing cannot be started while a tachyarrhythmia episode is in progress.

NOTE: Emergency therapy is the only function that can be initiated until the Temporary function is stopped.

4. To stop the Temporary pacing mode, select the Stop button. The Temporary pacing mode also stops when you command emergency therapy from the PRM, when you press the DIVERT THERAPY key, or if telemetry is lost.

Once Temporary pacing mode is stopped, pacing reverts to the previously programmed Normal/Post-Therapy settings.

RATE ADAPTIVE PACING AND SENSOR TRENDING

The Sensors and Trending section of the Normal Settings screen provides programmable options for Rate Adaptive Pacing, the Respiratory Sensor, and Sensor Trending.

Rate Adaptive Pacing

In rate adaptive pacing modes (i.e., any mode ending with R), sensors are used to detect changes in the patient's activity level and increase the pacing rate accordingly. Rate adaptive pacing is intended for patients who exhibit chronotropic incompetence and who would benefit from increased pacing rates that are concurrent with physical activity.

CAUTION: Rate adaptive pacing should be used with care in patients who are unable to tolerate increased pacing rates.

When rate adaptive parameters are programmed, the pacing rate increases in response to increased activity, then decreases as the activity returns to a resting level.

NOTE: Activity involving minimal upper body motion, such as bicycling, may result in only a moderate pacing response from the accelerometer.

NOTE: Rate adaptive pacing has been shown to be potentially proarrhythmic. Use caution when programming adaptive-rate features.

Accelerometer

The accelerometer detects motion that is associated with a patient's physical activity and generates an electronic signal that is proportional to the amount of body motion. Based on accelerometer input, the pulse generator estimates the patient's energy expenditure as a result of exercise, then translates it into a rate increase.

The pulse generator senses body motion by means of an integrated circuit accelerometer. The accelerometer sensor responds to activity in the frequency range of typical physiologic activity (1–10 Hz). The accelerometer evaluates both the frequency and the amplitude of the sensor signal.

- Frequency reflects how often an activity occurs (e.g., the number of steps taken per minute during a brisk walk)
- Amplitude reflects the force of motion (e.g., the more deliberate steps taken while walking)

Once detected, an algorithm translates the measured acceleration into a rate increase above the LRL.

Because the accelerometer is not in contact with the pulse generator case, it does not respond to simple static pressure on the device case.

There are three Accelerometer settings: Off, On, and ATR Only. When you program the respective rate-responsive modes for Normal Settings and ATR Fallback, that action automatically updates the Accelerometer setting. If the pulse generator is permanently programmed to a non-rate adaptive mode, it is possible to program the ATR Fallback mode to an adaptive-rate mode using the accelerometer sensor. In this case, the Accelerometer field will display ATR Only.

The following programmable parameters control the pulse generator's response to the sensor values generated by the Accelerometer:

- Response Factor
- Activity Threshold
- Reaction Time
- Recovery Time

Response Factor (Accelerometer)

Response Factor (accelerometer) determines the pacing rate increase that will occur above the LRL at various levels of patient activity (Figure 4-7 on page 4-14).

- High Response Factor—results in less activity required for the pacing rate to reach the MSR
- Low Response Factor—results in more activity required for the pacing rate to reach the MSR

NOTE: Programming Response Factor for Normal Settings also changes the corresponding selection for Post-Therapy Settings.


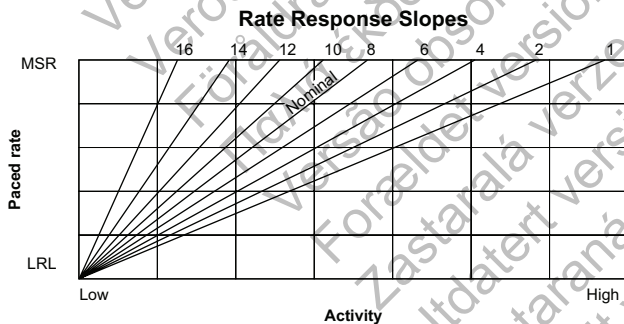


Figure 4-7. Response Factor and paced rate

The pacing rate achieved can be limited either by the detected activity level or the programmed MSR. If the detected activity level results in a steady-state rate below the MSR, the pacing rate can still increase when the detected activity levels increase (Figure 4-8 on page 4-15). The steady-state response is independent of the programmed reaction and recovery times.

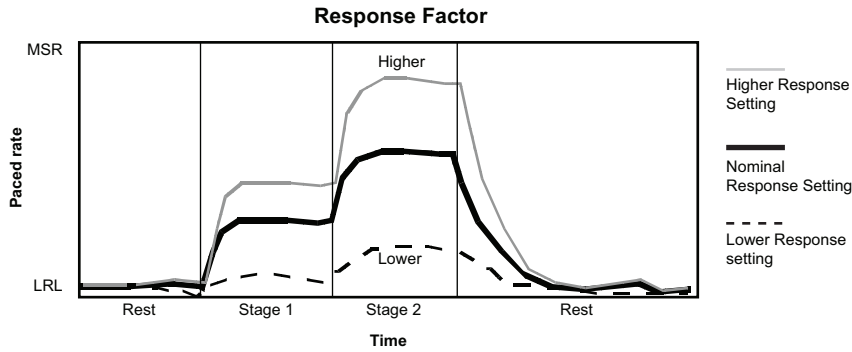

This figure shows the effect of higher and lower settings during a theoretical two-stage exercise test.

Figure 4-8. Response Factor in exercise test

Programming the LRL up or down moves the entire response up or down without changing its shape.

Activity Threshold

Activity Threshold prevents rate increases due to low-intensity, extraneous motion (e.g., motion caused by respiration, heart beat, or in some cases tremor associated with Parkinson's disease).

Activity Threshold represents the activity level that must be exceeded before the sensor-driven pacing rate will increase. The pulse generator will not increase the paced rate above the LRL until the activity signal increases above the Activity Threshold. An Activity Threshold setting should allow a rate increase with minor activity, such as walking, but be high enough so the pacing rate will not increase inappropriately when the patient is inactive (Figure 4-9 on page 4-15 and Figure 4-10 on page 4-16).

- Lower setting—less motion is required to increase the pacing rate
- Higher setting—more motion is required to increase the pacing rate

NOTE: Programming the Activity Threshold for Normal Settings also changes the corresponding selection for Post-Therapy Settings.


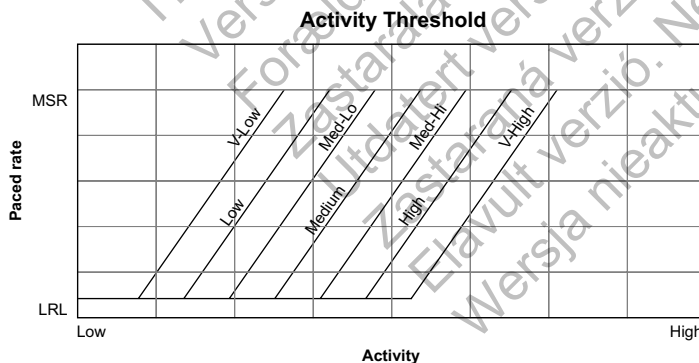


Figure 4-9. Activity Threshold and rate response

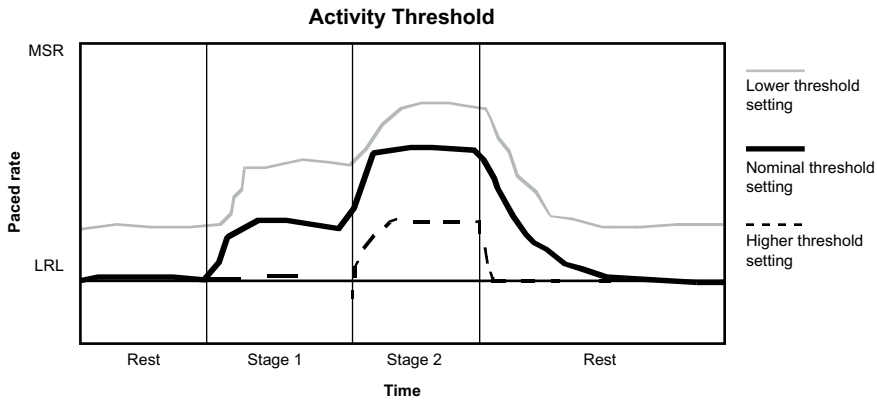

This figure demonstrates the effect of increased or decreased Activity Threshold settings in response to a theoretical two-stage exercise test.

Figure 4-10. Activity Threshold in exercise test

Reaction Time

Reaction Time determines how quickly the pacing rate will rise to a new level once an increase in activity level is detected.

Reaction Time affects only the time required for a rate increase to occur. The value selected determines the time required for the paced rate to move from the LRL to the MSR for a maximum level of activity (Figure 4-11 on page 4-16 and Figure 4-12 on page 4-17).

- Short Reaction Time: results in a rapid increase in the pacing rate
- Long Reaction Time: results in a slower increase in the pacing rate

NOTE: Programming Reaction Time for Normal Settings also changes the corresponding selection for Post-Therapy Settings.

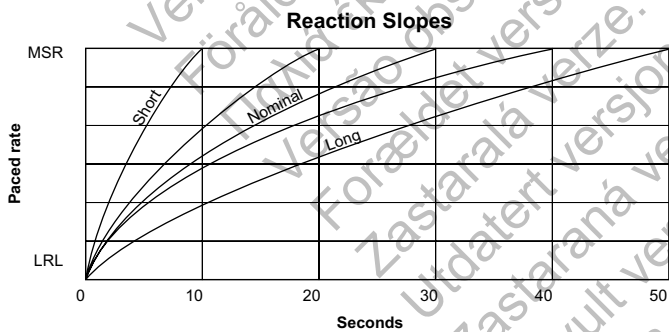


Figure 4-11. Reaction Time and paced rate

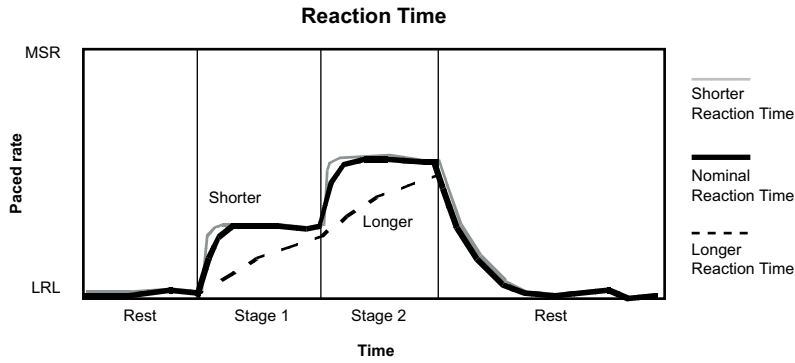



Figure 4-12. Reaction Time in exercise test

Recovery Time

Recovery Time determines the time required for the paced rate to decrease from the MSR to the LRL in the absence of activity. When patient activity concludes, Recovery Time is used to prevent an abrupt decrease in pacing rate (Figure 4-13 on page 4-17 and Figure 4-14 on page 4-18).


- Short Recovery Time—results in a faster decrease in pacing rate after patient activity lowers or stops
- Long Recovery Time—results in a slower decrease in pacing rate after patient activity lowers or stops

NOTE: Programming Recovery Time for Normal Settings also changes the corresponding selection for Post-Therapy Settings.

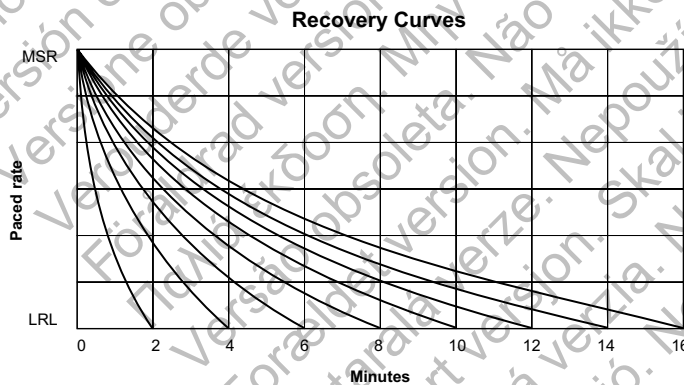

There are 15 settings available; only the even-numbered settings are shown.

Figure 4-13. Recovery Time and paced rate

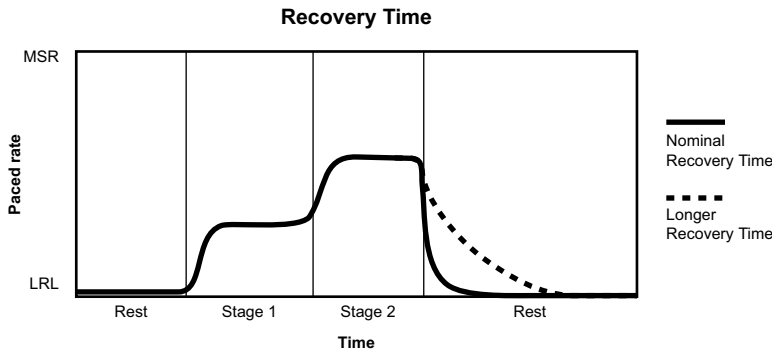

The figure shows the effect of higher and lower settings during a theoretical two-stage exercise test.

Figure 4-14. Recovery Time in exercise test

Sensor Trending

Sensor Trending provides a graphical display of the sensor rate based on sensor data. This feature evaluates the pulse generator's rate response to the patient's detected activity level and provides useful information during exercise testing.

The Sensor Trending screen is accessible from within Normal Settings (Figure 4-15 on page 4-18).

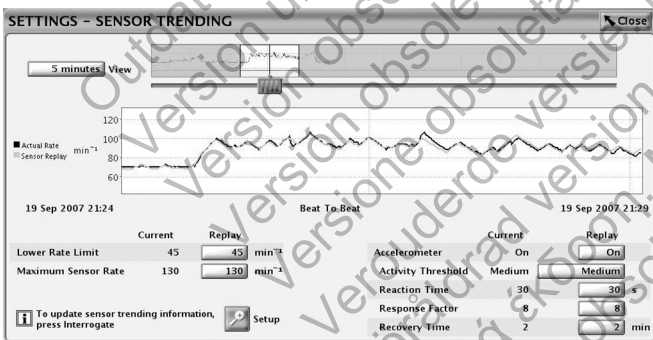


Figure 4-15. Sensor Trending screen

Setup includes the following options:

- Recording Method—programmable:
 - 30-Second Average—records and plots the average rate every 30 seconds.
 - Beat to Beat—records and plots the rate of every beat.

NOTE: *Beat to Beat is recommended when using hall walks or shorter periods of activity to manually optimize sensor rates.*

 - Off—no trending data is gathered.
- Duration—non-programmable and based on the selected Recording Method:
 - When Recording Method is set to Off or 30-Second Average—Duration is approximately 25 hours.
 - When Recording Method is set to Beat to Beat—Duration is approximately 40 minutes at 75 min⁻¹.

- Data Storage—programmable:
 - Continuous—contains the most recent data available. Storage starts when setup is confirmed and continuously records the latest information, overwriting the oldest data until the information is retrieved. This option allows you to view data for the recording duration immediately prior to data retrieval.
 - Fixed—storage starts when setup is confirmed and continues until device memory storage is full. This allows you to view data from initial setup for a fixed amount of time.

To collect accelerometer sensor trending data without a rate response, program the Brady Mode to a non-rate-adaptive mode and program the Recording Method for sensor trending to a value other than Off.

The pulse generator collects and stores rate and sensor data. The rate data represents the programmed parameters. The Sensor Replay option allows you to adjust the parameter values and view the result without having to repeat an exercise test. Select the View button to vary the time period for how much data is visible; options exist for 1–25 hours. To adjust the vertical axis, move the slider bar at the bottom of the display window. The pulse generator also collects and stores data in non-rate-adaptive modes; however, without the sensor data comparison, only rate data will be displayed.

The pulse generator will record Sensor Trending data while wanded or RF telemetry is active.

When the heart rate is completely sensor-driven, small differences between the Actual Rate and Sensor Replay may still be observed because they are calculated independently by slightly different methods.

ATRIAL TACHY RESPONSE

ATR Mode Switch

ATR is designed to limit the amount of time that the ventricular paced rate is at the MTR or exhibits upper-rate behavior (2:1 block or Wenckebach) in response to a pathological atrial arrhythmia.

In the presence of detected atrial activity that exceeds the ATR Trigger Rate, the pulse generator switches the pacing mode from a tracking mode to a nontracking mode as follows:

- From DDD(R) to DDI(R) or VDI(R)
- From VDD(R) to VDI(R)

An example of ATR behavior is shown (Figure 4-16 on page 4-20).

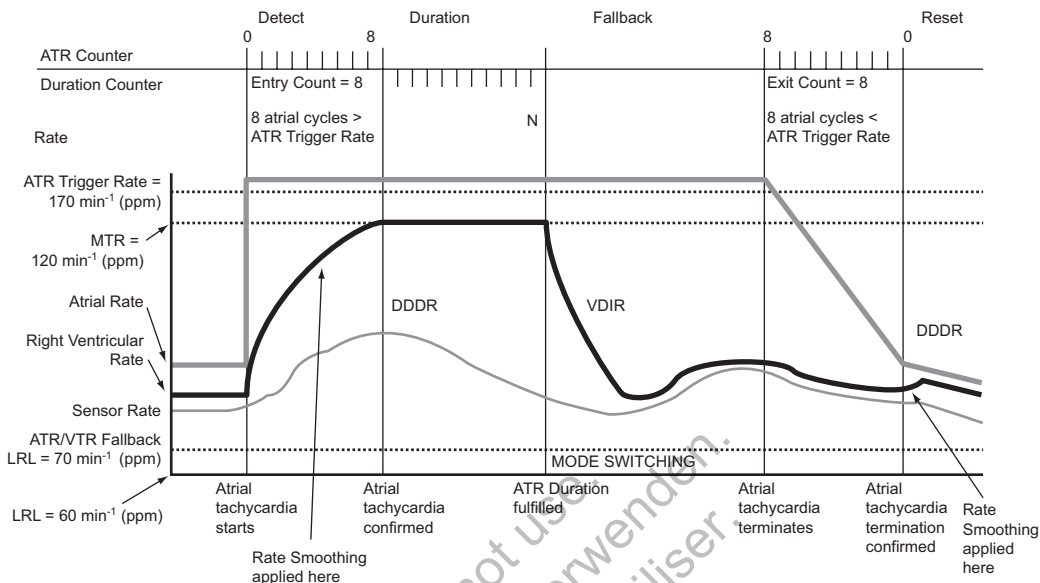


Figure 4-16. ATR behavior

NOTE: Parameter settings that reduce the atrial sensing window may inhibit ATR therapy.

ATR Trigger Rate

The ATR Trigger Rate determines the rate at which the pulse generator begins to detect atrial tachycardias.

The pulse generator monitors atrial events throughout the pacing cycle, except during the atrial blanking period and the noise rejection intervals. Atrial events faster than the Trigger Rate increase the ATR detection counter; atrial events slower than the Trigger Rate decrease the counter.

When the ATR detection counter reaches the programmed entry count, the ATR Duration begins. When the ATR detection counter counts down from the programmed Exit Count value to zero at any point in time, ATR Duration and/or fallback are terminated, and the ATR algorithm is reset. An event marker is generated whenever the ATR detection counter is incremented or decremented.

NOTE: During post-therapy pacing, ATR functions the same as in normal pacing.

ATR Duration

ATR Duration is a programmable value that determines the number of ventricular cycles during which the atrial events continue to be evaluated after initial detection (entry count) is met. This feature is intended to avoid mode switching due to short, nonsustained episodes of atrial tachycardia. If the ATR counter reaches zero during ATR Duration, the ATR algorithm will be reset, and no mode switch will occur.

If the atrial tachycardia persists for the programmed ATR Duration, then mode switching occurs and the Fallback Mode and Fallback Time begin.

Entry Count

The Entry Count determines how quickly an atrial arrhythmia is initially detected.

The lower the programmable value, the fewer the fast atrial events required to fulfill initial detection. Once the number of fast atrial events detected equals the programmable Entry Count, ATR Duration begins, and the Exit Count is enabled.

CAUTION: Exercise care when programming the Entry Count to low values in conjunction with a short ATR Duration. This combination allows mode switching with very few fast atrial beats. For example, if the Entry Count was programmed to 2 and the ATR Duration to 0, ATR mode switching could occur on 2 fast atrial intervals. In these instances, a short series of premature atrial events could cause the device to mode switch.

Exit Count

The Exit Count determines how quickly the ATR algorithm is terminated once the atrial arrhythmia is no longer detected.

The lower the programmed value, the more quickly the pulse generator will return to an atrial tracking mode once an atrial arrhythmia terminates. Once the number of slow atrial events detected equals the programmable Exit Count, ATR Duration and/or Fallback will be terminated, and the ATR algorithm will be reset. The ATR Exit Count is decremented by atrial events slower than the ATR Trigger Rate or by any ventricular event that occurs more than two seconds after the last atrial event.

CAUTION: Exercise care when programming the Exit Count to low values. For example, if the Exit Count was programmed to 2, a few cycles of atrial undersensing could cause termination of mode switching.

Fallback Mode

Fallback Mode is the nontracking pacing mode that the pulse generator automatically switches to when ATR Duration is fulfilled.

After switching modes, the pulse generator gradually decreases the ventricular paced rate. This decrease is controlled by the Fallback Time parameter.

NOTE: *Dual-chamber pacing fallback mode values are only available when the Normal pacing mode is also set to dual-chamber.*

NOTE: *ATR Fallback mode may be programmed rate responsive even if the permanent brady mode is non-rate responsive. In this scenario, the sensor parameters will indicate "ATR Only".*

Fallback Time

Fallback Time controls how quickly the paced rate will decrease from the MTR to the ATR/VTR Fallback LRL during fallback. The paced rate will decrease to the highest of the sensor-indicated rate, VRR rate, or the ATR/VTR Fallback LRL.

During fallback, the following features are disabled:

- Rate Smoothing—disabled until fallback reaches the ATR/VTR Fallback LRL or the sensor-indicated rate. If VRR is enabled, then Rate Smoothing is disabled throughout the mode switch
- Rate Hysteresis
- AV Search +
- PVARP Extension

Fallback LRL

The ATR/VTR Fallback LRL is the programmed lower rate to which the rate decreases during mode switching. The ATR/VTR Fallback LRL may be programmed higher or lower than the permanent brady LRL.

The rate will decrease to the highest among the sensor-indicated rate (when applicable), the VRR rate (if enabled), and the ATR/VTR Fallback LRL.

The ATR/VTR Fallback LRL is also the Backup VVI pacing rate during backup pacing in the presence of detected ventricular arrhythmias.

End of ATR Episode

The End of ATR Episode identifies the point when the pulse generator reverts to AV-synchronous operation because the atrial arrhythmia is no longer detected.

With the termination of the arrhythmia, the ATR Exit Count decrements from its programmed value until it reaches 0. When the ATR Exit Count reaches 0, the pacing mode automatically switches to the programmed tracking mode, and AV-synchronous operation is restored.

Ventricular Tachy Response (VTR)

VTR serves as an automatic mode switch for backup VVI pacing in the presence of detected ventricular tachyarrhythmias.

When detection is satisfied in a ventricular tachycardia zone, the pacing mode switches to VVI (RV) or to Off if the current mode is AAI(R) or Off.

When the mode switches, backup pacing occurs at the programmed ATR/VTR Fallback LRL and uses the programmed ATP ventricular Pulse Width and Amplitude values.

The device automatically accommodates the AV delay for the lowest programmed tachy rate threshold when atrial pacing occurs near the upper rate limit.

Ventricular Rate Regulation (VRR)

VRR is designed to reduce the V–V cycle length variability during partially conducted atrial arrhythmias by modestly increasing the ventricular pacing rate.

The VRR algorithm calculates a VRR-indicated pacing interval based on a weighted sum of the current V–V cycle length and the previous VRR-indicated pacing intervals.

- Paced intervals have more influence than sensed intervals such that paced events cause a decrease in the VRR-indicated rate.
- For sensed intervals, the VRR-indicated rate may be increased; however, the influence is tempered by the previous history.
- The VRR-indicated rate is further bound by the LRL and the VRR MPR.

When VRR is programmed on in tracking modes, it is only active when an ATR mode switch has occurred. Once the tracking mode operation resumes at the termination of the atrial arrhythmia, VRR becomes inactive. In tracking modes where both Rate Smoothing and VRR are programmed on, Rate Smoothing is disabled when VRR is active during ATR and re-enabled once the ATR terminates.

When programmed on in nontracking modes, VRR is continually active and updates the VRR-indicated pacing rate and the smoothed average on each cardiac cycle.

Ventricular Rate Regulation Maximum Pacing Rate (VRR MPR)

The VRR MPR limits the maximum pacing rate for VRR.

VRR operates between the LRL and the MPR.

Atrial Flutter Response (AFR)

Atrial Flutter Response is designed to:

- Prevent pacing into the vulnerable period following an atrial sense. Pacing into the vulnerable period could occur if an atrial pace is scheduled soon after a refractory atrial sense.
- Provide immediate nontracking of atrial rates higher than the AFR Trigger Rate.

The nontracking behavior is maintained for as long as atrial events continually exceed the AFR Trigger Rate.

Example: When AFR is programmed to 170 min^{-1} , a detected atrial event inside the PVARP or a previously triggered AFR interval starts an AFR window of 353 ms (170 min^{-1}). Atrial detection inside the AFR is classified as a sense within the refractory period and is not tracked. Atrial tracking may only occur after both PVARP and the AFR window expire. Paced atrial events scheduled inside an AFR window are delayed until the AFR window expires. If there are fewer than 50 ms remaining before the subsequent ventricular pace, the atrial pace is inhibited for the cycle.

Ventricular pacing is not affected by AFR and will take place as scheduled. The wide programmable range for AFR Trigger rates allows for appropriate sensing of slow atrial flutters. High-rate atrial sensing may continuously retrigger the AFR window, effectively resulting in behavior similar to the VDI(R) fallback mode.

NOTE: For atrial arrhythmias that meet the programmed AFR rate criteria, using the AFR feature will result in slower ventricular pacing rates.

NOTE: When both AFR and ATR are active in the presence of atrial arrhythmias, nontracking ventricular paced behavior may occur sooner, but the ATR mode switch may take longer. This is because the ATR Duration feature counts ventricular cycles for meeting duration and the AFR feature slows the ventricular paced response to fast atrial arrhythmias.

PMT Termination

PMT Termination detects and attempts to interrupt pacemaker-mediated tachycardia (PMT) conditions.

AV synchrony may be lost for many reasons, including atrial fibrillation, PVCs, PACs, atrial oversensing, or loss of atrial capture. If the patient has an intact retrograde conduction pathway when AV synchrony is lost, the unsynchronized beat may conduct retrograde to the atrium, resulting in premature atrial depolarization. In DDD(R) and VDD(R) pacing modes, the device may detect and track retrograde conducted P-waves that fall outside of PVARP. The repeated cycle of sensing and tracking retrograde conduction is known as PMT, which can result in triggered ventricular pacing rates as high as the MTR. Programming certain refractory periods (e.g., PVARP After PVC) can reduce the likelihood of tracking retrograde events. Rate Smoothing can also be useful in controlling the pulse generator's response to retrograde conduction.

When the pulse generator's response to retrograde conduction has not been controlled by device programming, PMT Termination (when programmed to On) is used to detect and terminate PMT within 16 cycles of onset when the following conditions have been met:

- 16 successive ventricular paces are counted at the MTR following atrial sensed events

- All 16 V–A intervals are within 32 ms (preceding or following) of the second V–A interval measured at MTR during the 16 ventricular paced events (to distinguish Wenckebach behavior from PMT)

When both conditions are met, the pulse generator sets the PVARP to a fixed setting of 500 ms for one cardiac cycle in an attempt to break the PMT. If both conditions are not met, the pulse generator continues to monitor successive ventricular paces for the presence of a PMT.

When PMT Termination is programmed to On, the pulse generator stores PMT episodes in the Arrhythmia Logbook.

NOTE: Although the V–A interval evaluation helps discriminate true PMT (stable V–A intervals) from upper rate behavior due to sinus tachycardia or normal exercise response (typically unstable V–A intervals), it is possible that a patient's intrinsic atrial rate can meet PMT detection criteria. In such cases, if PMT Termination is programmed On, the algorithm will declare the rhythm a PMT and extend PVARP on the 16th cycle.

NOTE: Because retrograde conduction times may vary over a patient's lifetime due to their changing medical condition, occasional programming changes may be necessary.

If retrograde conduction is evident in a stored EGM, you can evaluate the electrogram and/or perform a threshold test to confirm appropriate atrial pacing and sensing. If stored EGMs are not available for review, follow these steps to use the PRM to assist in V–A interval evaluation:

1. From the Tests screen, select the Temp Brady tab.
2. Program an appropriate atrial sensing mode that provides atrial markers (VDD, DDD, or DDI).
3. Program the maximum PVARP to a value shorter than the average retrograde conduction time.

NOTE: Scientific literature suggests that the average retrograde conduction time is 235 ± 50 ms (with a range of 110–450 ms)¹.

4. Program the LRL to ensure pacing above the intrinsic atrial rate (e.g., 90, 100, 110...).
5. Begin printing the real-time ECG.
6. Select the Start button to activate the temporary parameters.
7. When testing is complete for the specified LRL value, select the Stop button.
8. Stop printing the real-time ECG.
9. Evaluate the ECG strip for V–A conduction (VP followed by an AS). Look for stable and consistent intervals suggestive of retrograde conduction.
 - If retrograde conduction was identified, compare the retrograde V–A interval time to the programmed refractory period. Consider programming PVARP to the appropriate value so that the retrograde event is not tracked.
 - If retrograde conduction was not identified, the PMT episode may be a result of normal upper rate behavior. Review Histograms to see how often the rate is at the MTR, and consider raising the MTR (if clinically appropriate).

1. Furman S, Hayes D.L., Holmes D.R., A Practice of Cardiac Pacing. 3rd ed. Mount Kisco, New York: Futura Publishing Co.; 1993:74-75.

10. If necessary, repeat this procedure with different LRL values, as retrograde conduction may occur at different rates.

RATE ENHANCEMENTS

Rate Hysteresis

Rate Hysteresis can improve device longevity by reducing the number of pacing stimuli. In dual-chamber models, this feature is available in DDD, DDI, VVI, and AAI modes. In single-chamber models, this feature is available in VVI mode. In DDD, DDI, and AAI modes, Rate Hysteresis is activated by a single nonrefractory atrial sensed event.

NOTE: Rate Hysteresis is activated and deactivated by ventricular events in VVI mode (e.g., intrinsic activity, paced activity).

In DDD, DDI, and AAI modes, Hysteresis is deactivated by a single atrial pace at the hysteresis rate. In DDD mode, Hysteresis is deactivated by an atrial rate above the MTR.

When Rate Smoothing Down is enabled, Rate Hysteresis remains in effect until pacing occurs at the hysteresis rate. This allows Rate Smoothing to control the transition to the hysteresis rate.

Hysteresis Offset

Hysteresis Offset is used to lower the escape rate below the LRL when the pulse generator senses intrinsic atrial activity.

If intrinsic activity below the LRL occurs, then Hysteresis Offset allows inhibition of pacing until the LRL minus Hysteresis Offset is reached. As a result, the patient might benefit from longer periods of sinus rhythm.

Search Hysteresis

When Search Hysteresis is enabled, the pulse generator periodically lowers the escape rate by the programmed Hysteresis Offset in order to reveal potential intrinsic atrial activity below the LRL. The programmed number of search cycles must be consecutively atrial paced for a search to occur.

Example: At a rate of 70 min⁻¹ and a search interval of 256 cycles, a search for intrinsic atrial activity would occur approximately every 3.7 minutes ($256 \div 70 = 3.7$).

During Search Hysteresis, the pacing rate is lowered by the Hysteresis Offset for up to 8 cardiac cycles. If intrinsic activity is sensed during the search period, Hysteresis will remain active until an atrial pace occurs at the hysteresis offset rate.

Rate Smoothing is disabled during the search cycles. If no intrinsic atrial activity is detected during the 8-cycle search, the pacing rate is brought up to the LRL. Rate Smoothing Up, if enabled, controls the pacing rate increase.

Rate Smoothing

Rate Smoothing controls the pulse generator's response to atrial and/or ventricular rate fluctuations that cause sudden changes in pacing intervals. Rate Smoothing is an important enhancement to ATR because it can significantly reduce the rate fluctuations associated with the onset and cessation of atrial arrhythmias.

Without Rate Smoothing, a sudden, large atrial rate increase will cause a simultaneous sudden increase in the paced ventricular rate as high as the programmed MTR. Patients who experience large variations in their ventricular paced rate can feel symptomatic during these episodes. Rate

Smoothing can prevent these sudden rate changes and the accompanying symptoms (such as palpitations, dyspnea, and dizziness).

In a normal conduction system, limited cycle-to-cycle rate variations occur. However, the paced rate can change dramatically from one beat to the next in the presence of any of the following:

- Sinoatrial disease such as sinus pause or arrest, sinoatrial block, and brady-tachy syndrome
- PACs and/or PVCs
- Pacemaker Wenckebach
- Intermittent, brief, self-terminating SVTs, and atrial flutter/fibrillation
- Retrograde P-waves
- Pulse generator sensing of myopotential signals, EMI, crosstalk, etc.

In single-chamber modes, Rate Smoothing operates between:

- The LRL and the MPR when programmed VVI or AAI
- The LRL and the MSR when programmed VVIR or AAIR

In dual-chamber modes, Rate Smoothing operates between:

- The LRL and the greater of the MSR or MTR when programmed DDD(R) or VDD(R)
- The LRL and MPR when programmed to DDI
- The LRL and MSR when programmed to DDIR

Rate Smoothing is also applicable between the hysteresis rate and LRL when hysteresis is active, except during Search Hysteresis.

When Rate Smoothing is programmed to On, it is functional except:

- During the 8 cycles of rate Search Hysteresis
- During ATR Fallback until fallback reaches the ATR LRL, the sensor-indicated rate, or the VRR interval
- During VRR when active
- Upon triggering PMT Termination
- Immediately following programmed LRL increases
- When the intrinsic rate is above the MTR

Programmable Values

Rate Smoothing values are a percentage of the RV R–R interval (3% to 25% in 3% increments) and can be independently programmed for:

- Increase—Rate Smoothing Up
- Decrease—Rate Smoothing Down
- Off

The pulse generator stores the most recent R–R interval in memory. R-waves may be either intrinsic or paced. Based on this R–R interval and the programmed Rate Smoothing value, the device limits the variation in paced rate on a beat to beat basis.

It is important to ascertain the patient's physiologic cycle-to-cycle variation and program the Rate Smoothing parameter to a value that protects against pathologic interval changes, yet allows physiologic interval changes in response to increases in activity or exercise.

Rate Smoothing Up

Rate Smoothing Up controls the largest pacing rate increase allowed when the intrinsic or sensor rate is increasing.

Rate Smoothing Down

Rate Smoothing Down controls the largest pacing rate decrease allowed when the intrinsic or sensor rate is decreasing.

NOTE: When Rate Smoothing Down is programmed on and Rate Smoothing Up is programmed Off, the pulse generator will automatically prevent fast intrinsic beats (e.g., PVCs) from resetting the Rate Smoothing Down escape rate any faster than 12% per cycle.

Rate Smoothing Maximum Pacing Rate (MPR)

The Rate Smoothing Maximum Pacing Rate places a limit on the maximum pacing rate that Rate Smoothing can reach.

The Rate Smoothing Down parameter requires a programmed MPR when in AAI, VVI, or DDI. Rate Smoothing will then be used only between the MPR and the LRL or the hysteresis rate (if applicable).

When both VRR and Rate Smoothing are programmed on in the VVI(R) or DDI(R) mode, VRR will have priority.

Rate Smoothing Example Based on a Dual-Chamber Tracking Mode

Based on the most recent R–R interval stored in memory and the programmed Rate Smoothing value, the pulse generator sets up the two synchronization windows for the next cycle: one for the atrium and one for the ventricle. The synchronization windows are defined below:

Ventricular synchronization window: previous R–R interval \pm Rate Smoothing value

Atrial synchronization window: (previous R–R interval \pm Rate Smoothing value) - AV Delay

The following example explains how these windows are calculated (Figure 4-17 on page 4-28):

- Previous R–R interval = 800 ms
- AV Delay = 150 ms
- Rate Smoothing Up = 9%
- Rate Smoothing Down = 6%

The windows would be calculated as follows:

Ventricular Synchronization Window = 800 - 9% to 800 + 6% = 800 ms - 72 ms to 800 ms + 48 ms = 728 ms to 848 ms

Atrial Synchronization Window = Ventricular Synchronization Window - AV Delay =
728 ms - 150 ms to 848 ms - 150 ms = 578 ms to 698 ms

The timing for both windows is initiated at the end of every ventricular event (R-R interval).

If paced activity is to occur, it must occur within the appropriate synchronization window.

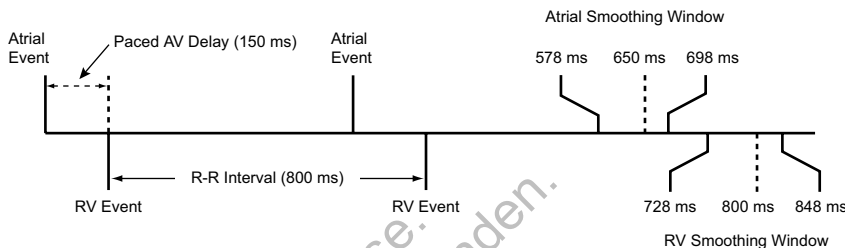


Figure 4-17. Rate smoothing synchronization window

LEAD CONFIGURATION

The pulse generator has independently programmable lead configurations for the following:

- Atrium (in dual-chamber models)
- Right Ventricle

The atrial and RV leads are set to Bipolar pacing and sensing. The atrial lead has the option of being programmed Off.

The input impedance is > 100 KΩ for each sense/pace electrode pair.

AV DELAY

AV Delay is the programmable time period from the occurrence of either a paced or sensed right atrial event to a paced RV event.

AV Delay is designed to help preserve the heart's AV synchrony. If a sensed right ventricular event does not occur during the AV delay following an atrial event, the pulse generator delivers a ventricular pacing pulse when the AV Delay expires.

AV Delay can be programmed to one or both of the following operations:

- Paced AV Delay
- Sensed AV Delay

AV Delay is applicable in DDD(R), DDI(R), DOO or VDD(R) modes.

NOTE: Long fixed AV intervals may be selected to avoid unnecessary RV pacing. However, programming long fixed AV intervals, in some cases, may be associated with PMT, diastolic mitral insufficiency, or pacemaker syndrome. As an alternative to programming long fixed AV intervals, consider AV Search+ to avoid unnecessary RV pacing.

Paced AV Delay

Paced AV Delay corresponds to the AV Delay following an atrial pace.

When the minimum AV Delay value is less than the maximum AV Delay value, then the Paced AV Delay is scaled dynamically according to the current pacing rate. Dynamic AV Delay provides

a more physiologic response to rate changes by automatically shortening the Paced AV Delay or Sensed AV Delay with each interval during an increase in atrial rate. This helps minimize the occurrence of large rate changes at the upper rate limit and allows one-to-one tracking at higher rates.

The pulse generator automatically calculates a linear relationship based on the interval length of the previous A–A or V–V cycle (depending on the previous event type) and the programmed values for the following:

- Minimum AV Delay
- Maximum AV Delay
- LRL
- MTR
- MSR
- MPR

The dynamic AV Delay is not adjusted following a PVC or when the previous cardiac cycle was limited by the MTR.

If the atrial rate is at or below the LRL (e.g., hysteresis), the maximum AV Delay is used. If the atrial rate is at or above the higher of the MTR, MSR, or MPR, the programmed minimum AV Delay is used.

When the atrial rate is between the LRL and the higher of the MTR, MSR, and MPR, the pulse generator calculates the linear relationship to determine the Dynamic AV Delay.

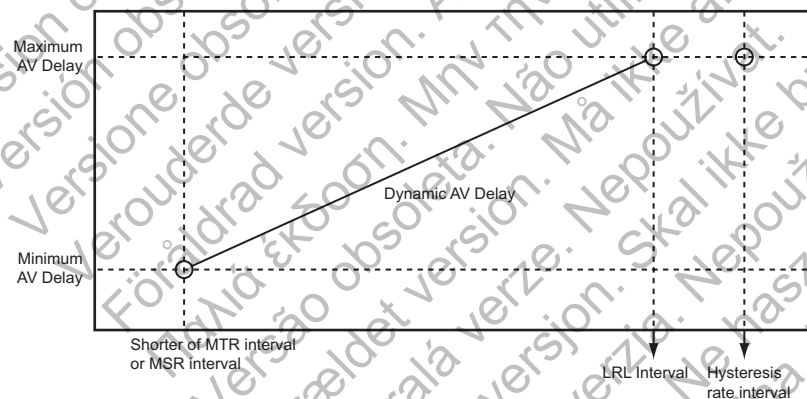


Figure 4-18. Dynamic AV Delay

The AV delay may be programmed to either a fixed or dynamic value as follows:

- Fixed AV Delay—occurs when Paced AV Delay minimum and maximum values are equal
- Dynamic AV Delay—occurs when Paced AV Delay minimum and maximum values are not equal

Sensed AV Delay

Sensed AV Delay corresponds to the AV Delay after a sensed atrial event.

Sensed AV Delay may be programmed to a value shorter than or equal to the Paced AV Delay. A shorter value is intended to compensate for the difference in timing between paced atrial events and sensed atrial events (Figure 4-19 on page 4-30).

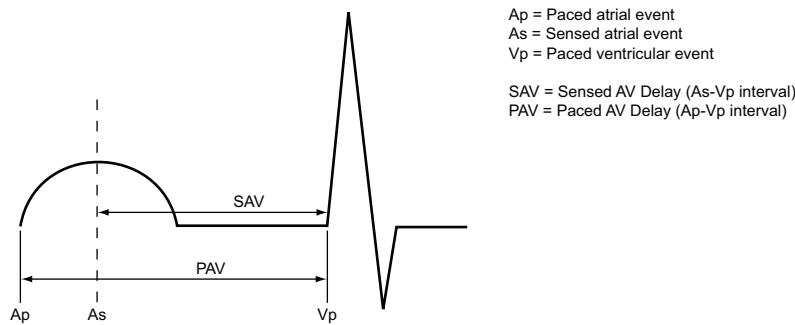


Figure 4-19. Sensed AV Delay

The hemodynamic impact of the Sensed AV Delay depends on the appropriateness of the timing between the atrial and ventricular contractions. Atrial pacing initiates atrial electrical excitation, whereas atrial sensing can only occur after the onset of spontaneous atrial excitation. The delay between initiation and sensing depends on the lead location and conduction. As a result, when Sensed AV Delay is programmed to the same value as Paced AV Delay, the hemodynamic AV interval will differ between paced and sensed atrial events.

When the device is programmed to DDD(R), it is recommended that the patient be tested to determine the optimal AV Delay during atrial sensing and atrial pacing. If the optimal AV Delays are different, this can be reflected by programming different Paced AV Delay and Sensed AV Delay parameter settings.

Using Sensed AV Delay with Paced AV Delay—Fixed

When Paced AV Delay is programmed to a fixed value, then the Sensed AV Delay will be fixed at the programmed Sensed AV Delay value.

Using Sensed AV Delay with Paced AV Delay—Dynamic

When Paced AV Delay is programmed as dynamic, then the Sensed AV Delay will also be dynamic.

Dynamic Sensed AV Delay and Paced AV Delay are based on the atrial rate. To reflect the shortening of the PR interval during periods of increased metabolic demand, the AV Delay shortens linearly from the programmed (maximum) value at the LRL (or hysteresis rate) to a value determined by the ratio of minimum and maximum AV Delay at the higher of the MTR, MSR, or MPR (Figure 4-20 on page 4-31). When Dynamic AV Delay is used, if the maximum Sensed AV Delay value is programmed as shorter than the maximum Paced AV Delay value, then the minimum Sensed AV Delay value will also be shorter than the minimum Paced AV Delay value.

NOTE: The minimum Sensed AV Delay value is programmable only in VDD(R) mode.

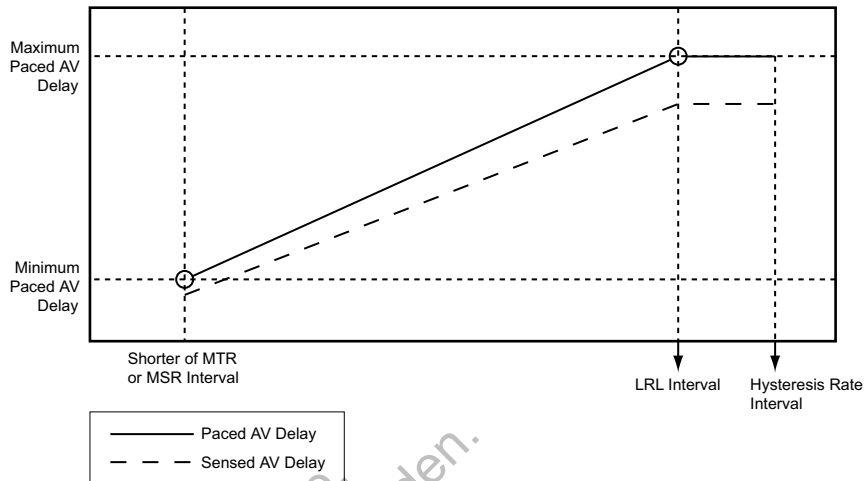


Figure 4-20. Dynamic and Sensed AV Delay function

AV Search+

AV Search+ is designed to promote intrinsic A–V conduction if present by allowing AV conduction to occur beyond the programmed AV Delay. In patients with exercise-dependent, first degree or second degree AV nodal block, this intrinsic AV conduction can improve hemodynamic performance and increase device longevity by reducing the amount of ventricular pacing pulses.

When AV Search+ is enabled, the AV Delay is lengthened periodically (Search Interval) for up to 8 consecutive paced or sensed cardiac cycles. The AV Search+ AV delay remains active as long as the intrinsic PR intervals are shorter than the programmed Search AV Delay value.

The pulse generator reverts to the programmed AV Delay at the following points:

- When the 8-cycle search expires without sensing intrinsic ventricular activity
- When two ventricular paced events occur within a 10-cycle moving window

Search AV Delay

The Search AV Delay parameter determines the length of the sensed and paced AV delays during the search cycles and during the AV hysteresis period.

NOTE: The Search AV Delay value must be programmed to longer than the maximum Paced AV Delay. Dynamic AV Delay and Sensed AV Delay are not applied during AV Search+.

The PRM does not consider the AV Delay associated with AV Search+ when calculating the TARP interval. This is so that longer AV Delays, without interactions, can be programmed for patients with intact AV conduction. Note that if AV Search+ is utilized in this manner, Wenckebach-like behavior may occur at rates lower than the MTR if conduction is lost.

NOTE: Long fixed AV intervals may be selected to avoid unnecessary RV pacing. However, programming long fixed AV intervals, in some cases, may be associated with PMT, diastolic mitral insufficiency or pacemaker syndrome. As an alternative to programming long fixed AV intervals, consider AV Search+ to avoid unnecessary RV pacing.

Search Interval

The Search Interval controls the frequency at which AV Search+ will attempt a search.

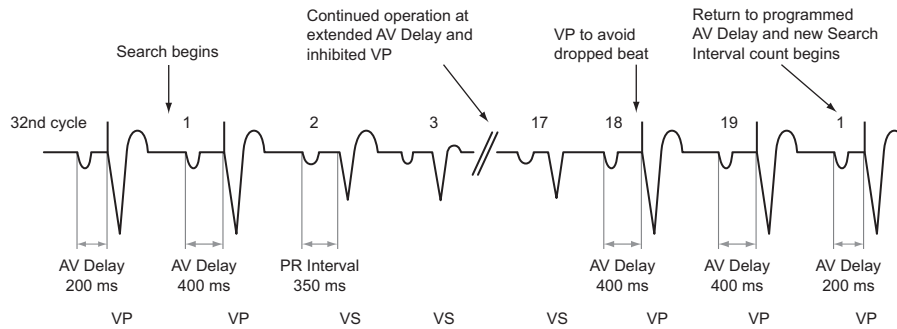


Figure 4-21. AV Search +

RYTHMIQ

RYTHMIQ is designed to eliminate unnecessary ventricular pacing for patients who appear to have normal AV conduction, and to prevent clinically significant pauses as defined by the 2008 ACC/AHA/HRS guidelines². RYTHMIQ operates in an AAI(R) pacing mode with VVI backup during times of normal conduction. If loss of AV synchrony is detected, then the mode automatically switches to DDD(R) to restore AV synchrony. If normal conduction returns, then the mode automatically switches back to AAI(R) with VVI backup. RYTHMIQ does not require dropped ventricular beats to switch to DDD(R) pacing.

RYTHMIQ is available only when the normal brady mode is programmed to DDD(R). If the normal brady mode is DDD, then RYTHMIQ can be set to either AAI with VVI backup or Off. If the normal brady mode is DDDR, then RYTHMIQ can be set to either AAIR with VVI backup or Off.

The following occurs during the RYTHMIQ stage of AAI(R) with VVI backup:

- The device provides AAI(R) pacing at the LRL and/or sensor indicated rate.
- The device provides backup VVI pacing at a rate of 15 min⁻¹ slower than the LRL. The backup VVI pacing rate is limited to no slower than 30 min⁻¹ and no faster than 60 min⁻¹. When there is consistent conduction, ventricular pacing does not occur as the VVI backup mode runs in the background at a reduced LRL.
- The device monitors for loss of AV synchrony. If 3 slow ventricular beats are detected in a window of 11 beats, then the device automatically switches to DDD(R) mode. A slow beat for RYTHMIQ is defined as a ventricular pace or ventricular sensed event that is at least 150 ms slower than the AAI(R) pacing interval.

The following occurs during the RYTHMIQ stage of DDD(R):

- The device provides DDD(R) pacing according to the normal programmed parameters.
- The device uses AV Search+ to periodically check for a return of intrinsic conduction. If AV Search+ remains in AV hysteresis for at least 25 cardiac cycles, and less than 2 of the last 10 cycles are ventricular paced, then the device automatically switches the pacing mode back to AAI(R) with VVI backup.

When RYTHMIQ detects loss of AV synchrony, the device records a RYTHMIQ episode along with 20 seconds of electrogram data (10 seconds before the mode switch, 10 seconds after the mode switch). The RYTHMIQ episode will be noted by the PRM and can be inspected in detail by selecting the appropriate episode from the Arrhythmia Logbook screen. When the DDD(R) stage of RYTHMIQ is active, the RYTHMIQ episode is identified as “in progress”.

2. ACC/AHA/HRS 2008 Guidelines for Device-Based Therapy of Cardiac Rhythm Abnormalities. Journal of the American College of Cardiology, Vol. 51(21), May 27, 2008.

Features available during the DDD(R) stage of RYTHMIQ may not be available during the AAI(R) stage of RYTHMIQ. The exceptions are ATR, rate-adaptive pacing, and rate smoothing. If ATR is programmed on for DDD(R), it will also be active during AAI(R), and may perform an ATR mode switch from either RYTHMIQ stage. When the atrial arrhythmia ends, the pacing mode will resume the RYTHMIQ stage that was active before the ATR mode switch. If rate smoothing is programmed on for DDD(R), then rate smoothing will be active during AAI(R); rate smoothing will not alter the VVI backup-pacing rate.

If you want the switch from AAI(R) with VVI Backup to DDD(R) to only occur once, then program AV Search+ to Off. In this case, the pulse generator remains in DDD(R) mode until reprogramming occurs.

REFRACTORY

Refractory periods are the intervals following paced or sensed events during which the pulse generator is not inhibited or triggered by detected electrical activity. They suppress (or prevent) oversensing of pulse generator artifacts and evoked responses following a pacing pulse. They also promote appropriate sensing of a single, wide, intrinsic complex and prevent the sensing of other intrinsic signal artifacts (e.g., a T-wave or far-field R-wave).

Refer to more information about refractory periods ("Calculating Rates and Refractory Periods" on page 2-3)

NOTE: *Rate adaptive pacing is not inhibited during refractory periods.*

A-Refractory - PVARP

PVARP is defined according to the pacing mode:

- Single-chamber atrial modes: AAI(R)—the time period after a sensed or paced atrial event when an atrial sense event does not inhibit an atrial pace.
- Dual-chamber modes: DDD(R), DDI(R), VDD(R)—the time period after a sensed or paced RV event when an atrial event does not inhibit an atrial pace or trigger a ventricular pace. The atrial refractory period prevents the tracking of retrograde atrial activity initiated in the ventricle.

PVARP can be programmed to a fixed value or to a dynamic value calculated based on the previous cardiac cycles. To program a fixed PVARP, set the minimum and maximum to the same value. PVARP will automatically be dynamic if the minimum value is less than the maximum value.

A long atrial refractory period shortens the brady atrial sensing window. Programming long atrial refractory periods in combination with certain AV Delay periods can cause 2:1 block to occur abruptly at the programmed MTR.

In DDD(R) and VDD(R) pacing modes, the pulse generator may detect retrograde conduction in the atrium, causing triggered ventricular pacing rates as high as the MTR (i.e., PMT). Retrograde conduction times may vary over a patient's lifetime as a function of changing autonomic tone. If testing does not reveal retrograde conduction at implantation, it may still occur at a later time. This problem can usually be avoided by increasing the atrial refractory period to a value that exceeds the retrograde conduction time.

In controlling the pulse generator's response to retrograde conduction, it may also be useful to program the following:

- PVARP after PVC
- PMT Termination

- Rate Smoothing

Dynamic PVARP

Programming of Dynamic PVARP and Dynamic AV Delay optimizes the sensing window at higher rates, allowing upper rate behavior (e.g., 2:1 block and pacemaker Wenckebach) in DDD(R) and VDD(R) modes to be significantly reduced, even at higher MTR settings. At the same time, Dynamic PVARP reduces the likelihood of PMTs at lower rates. Dynamic PVARP also reduces the likelihood of competitive atrial pacing.

The pulse generator automatically calculates the Dynamic PVARP using a weighted average of the previous cardiac cycles. This results in a shortening of the PVARP in a linear fashion as the rate increases. When the average rate is between the LRL and the MTR or applicable upper rate limit, the pulse generator calculates the Dynamic PVARP according to the linear relationship shown (Figure 4-22 on page 4-34). This relationship is determined by the programmed values for Minimum PVARP, Maximum PVARP, the LRL, and the MTR or applicable upper rate limit.

CAUTION: Programming minimum PVARP less than retrograde V–A conduction may increase the likelihood of a PMT.

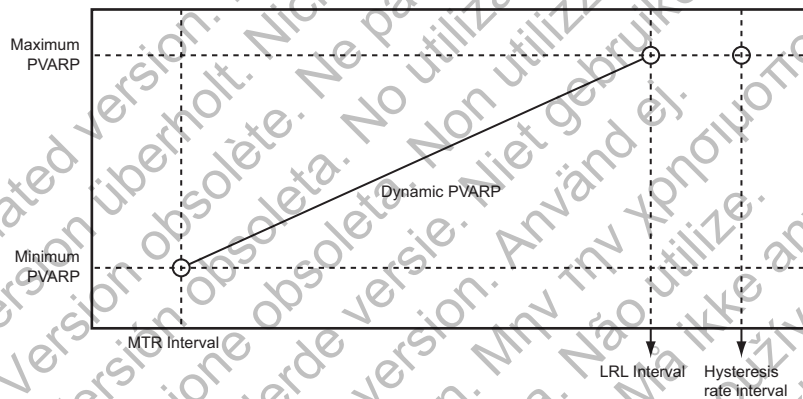


Figure 4-22. Dynamic PVARP

Maximum PVARP

If the average rate is equal to or lower than the LRL (e.g., hysteresis), the Maximum PVARP is used.

Minimum PVARP

If the average rate is equal to or higher than the MTR interval, the programmed Minimum PVARP is used.

PVARP after PVC

PVARP after PVC is designed to help prevent PMT due to retrograde conduction, which can occur due to a PVC.

When the pulse generator detects a sensed RV event without detecting a preceding atrial sensed event (refractory or non-refractory) or delivering an atrial pace, the atrial refractory period automatically extends to the programmed PVARP after PVC value for one cardiac cycle. After a PVC is detected, the timing cycles reset automatically. PVARP extends no more frequently than every other cardiac cycle.

The pulse generator automatically extends the PVARP to the PVARP after PVC value for one cardiac cycle in these additional situations:

- If an atrial pace is inhibited due to Atrial Flutter Response
- After a ventricular escape pace that is not preceded by an atrial sense in VDD(R) mode
- When the device transitions from a non-atrial tracking mode to an atrial tracking mode (e.g., exits ATR Fallback, transitions from temporary non-atrial tracking mode to permanent atrial tracking mode)
- When the device returns from Electrocautery Protection to an atrial tracking mode

A Refractory - same chamber

Dual-chamber Modes

Atrial refractory provides an interval following an atrial paced or sensed event when additional atrial sensed events do not impact the timing of pacing delivery.

The following are nonprogrammable intervals for dual-chamber modes:

- 85 ms atrial refractory following an atrial sensed event
- 150 ms atrial refractory following an atrial pace in DDD(R) and DDI(R) modes

RV-Refractory (RVRP)

The programmable RVRP provides an interval following an RV pace event during which RV sensed events do not impact the timing of pacing delivery.

Additionally, a 135 ms nonprogrammable refractory period provides an interval following an RV sensed event during which further RV sensed events do not impact the timing of pacing delivery.

The programming and function of the ventricular refractory period in VVI(R) mode is the same in dual- and single-chamber devices. Any event which falls into VRP is not detected or marked (unless it occurs within the noise window), and does not impact timing cycles.

RVRP is available in any mode where ventricular sensing is enabled, and RVRP can be programmed to a fixed or dynamic interval (Figure 4-23 on page 4-36):

- Fixed—RVRP remains at the programmed, fixed RVRP value between the LRL and the applicable upper rate limit (MPR, MTR or MSR).
- Dynamic—RVRP shortens as ventricular pacing increases from the LRL to the applicable upper rate limit, allowing adequate time for RV sensing.
 - Maximum—if the pacing rate is less than or equal to the LRL (i.e., hysteresis), the programmed Maximum VRP is used as the RVRP.
 - Minimum—if the pacing rate is equal to the applicable upper rate limit, the programmed Minimum VRP is used as the RVRP.

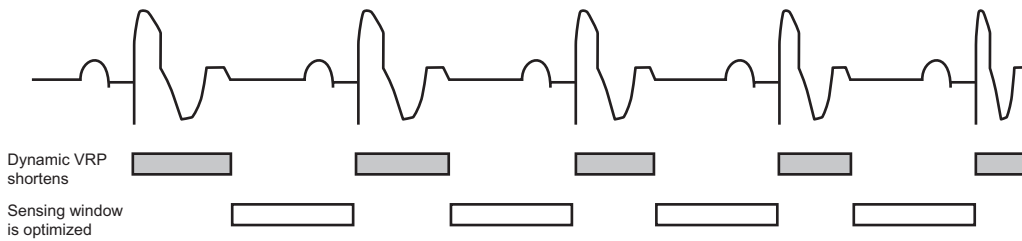


Figure 4-23. Relationship between ventricular rate and refractory interval

To provide an adequate sensing window, the following refractory value (fixed or dynamic) programming is recommended:

- Single-chamber modes—less than or equal to one-half the LRL in ms
- Dual-chamber modes—less than or equal to one-half the applicable upper rate limit in ms

The use of a long RVRP shortens the ventricular sensing window.

Programming the Ventricular Refractory period to a value greater than PVARP can lead to competitive pacing. For example, if the Ventricular Refractory is longer than PVARP, an atrial event can be appropriately sensed following PVARP and intrinsic conduction to the ventricle falls into the Ventricular Refractory period. In this case, the device will not sense the ventricular depolarization and will pace at the end of the AV Delay, resulting in competitive pacing.

Cross-Chamber Blanking

Cross-chamber blanking periods are designed to promote appropriate sensing of in-chamber events and prevent oversensing of activity in another chamber (e.g., cross-talk, far-field sensing).

Cross-chamber blanking periods are initiated by paced and/or sensed events in an adjacent chamber. For example, a blanking period is initiated in the right ventricle each time a pacing pulse is delivered to the right atrium; this prevents the device from detecting the atrial paced event in the right ventricle.

Cross-chamber blanking can be programmed to Smart or a fixed value. Smart Blanking is designed to promote appropriate sensing of in-chamber events by shortening the cross-chamber blanking period (37.5 ms following paced events and 15 ms following sensed events) and prevent oversensing of cross-chamber events by automatically raising the AGC threshold for sensing at the expiration of the Smart Blanking period.

Smart Blanking does not change the programmed AGC sensitivity settings.

NOTE: Smart Blanking periods will be lengthened to 85 ms if a same-chamber blanking period or a retriggerable noise window is active when the Smart Blanking period begins. For example, if an RV sense occurs within the atrial refractory period, the A blank after RV sense cross chamber blank will be 85 ms.

NOTE: Sensitivity adjustments associated with Smart Blanking may not be sufficient to inhibit detection of cross-chamber artifacts if the cross-chamber artifacts are too large. Consider other factors that impact the size/amplitude of cross-chamber artifacts including lead-placement, pacing output, programmed sensitivity settings, shock output, and time since last delivered shock.

RV-Blank after A-Pace

RV-Blank after A-Pace is a cross-chamber blanking period designed to promote the appropriate sensing of RV events and prevent oversensing of cross-chamber events following an atrial pace.

If RV-Blank after A-Pace is programmed to a fixed period, the pulse generator will disregard RV events for the duration selected following an atrial pace. If a fixed period is chosen, then there is the increased potential for undersensing of R-waves (e.g., PVCs) in the cross-chamber blanking period after atrial pacing.

If the value is programmed to Smart, the pulse generator automatically raises the AGC threshold for sensing at the expiration of the Smart Blanking period in order to aid rejection of cross-chamber atrial events. This promotes sensing of R-waves that may have otherwise fallen into the cross-chamber blanking period. Smart Blanking does not change the programmed sensitivity settings.

Smart Blanking is designed to promote sensing of R-waves, and should only be considered when PVCs occur during the cross-chamber blanking period following an atrial pace and are not properly sensed.

When Smart Blanking is used, it is possible that polarization artifacts following atrial pacing may be detected as R-waves. These artifacts are likely a result of voltage build-up on the ventricular sensing lead following tachy therapy or high-output ventricular pacing, and may inhibit ventricular pacing.

When adjusting blanking, consider the following:

- If the patient is pacemaker-dependent, test for proper sensing after shock therapy. If oversensing is occurring post shock, be prepared to use the STAT PACE command.
- To promote continuous pacing for pacemaker-dependent patients, it may be preferable to lessen the potential for ventricular oversensing of atrial paced artifacts by programming a longer blanking period. However, programming a longer blanking period may increase the likelihood of undersensing R-waves (e.g., PVCs, should they occur within the RV-Blank after A-Pace cross-chamber blanking period).
- For patients with a high percentage of atrial pacing and frequent PVCs who are not pacemaker-dependent, it may be preferable to shorten the blanking period to lessen the potential for undersensing a PVC (should it occur in the cross-chamber blanking period following an atrial paced event). However, a shorter blanking period may increase the likelihood for ventricular oversensing of an atrial paced event.

A-Blank after V-Pace

A-Blank after V-Pace is a cross-chamber blanking period designed to promote the appropriate sensing of P-waves and prevent oversensing of cross-chamber events following a ventricular pace.

If the value is programmed to Smart, the pulse generator automatically raises the AGC threshold for sensing at the expiration of the Smart Blanking period in order to aid rejection of cross-chamber ventricular events. This promotes sensing of P-waves that may have otherwise fallen in the cross-chamber blanking period. Smart Blanking does not change the programmed sensitivity settings.

A-Blank after RV-Sense

A-Blank after RV-Sense is a cross-chamber blanking period designed to promote appropriate sensing of P-waves and prevent oversensing of cross-chamber events following an RV-sensed event.

If the value is programmed to Smart, the pulse generator automatically raises the AGC threshold for sensing at the expiration of the Smart Blanking period in order to aid rejection of cross-chamber RV events. This promotes sensing of P-waves that may have otherwise

fallen in the cross-chamber blanking period. Smart Blanking does not change the programmed sensitivity settings.

Refer to the following illustrations:

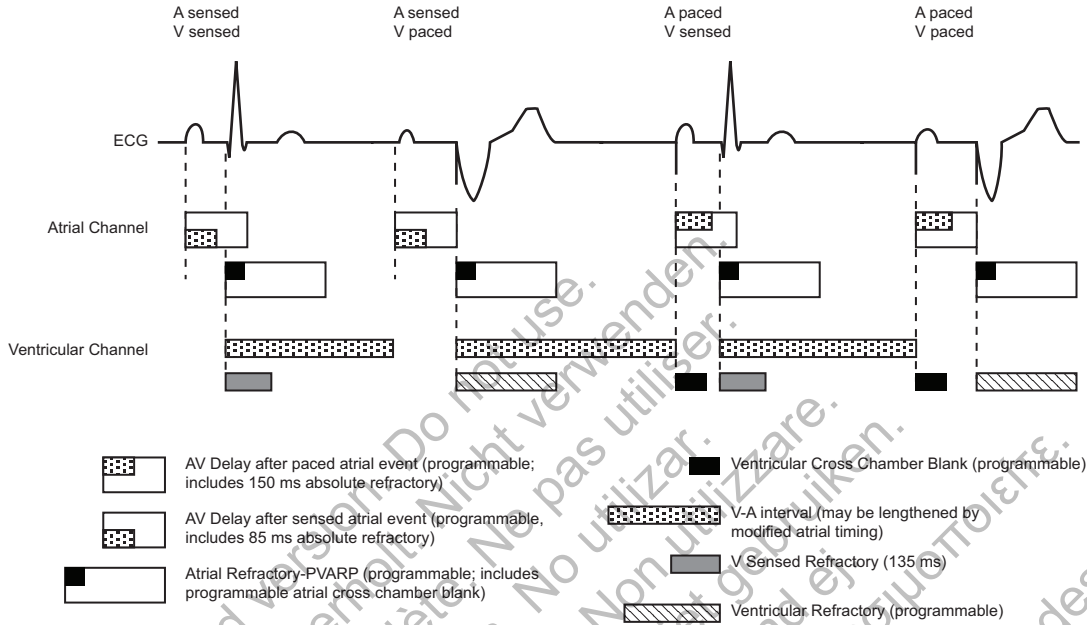


Figure 4-24. Refractory periods, dual-chamber pacing modes

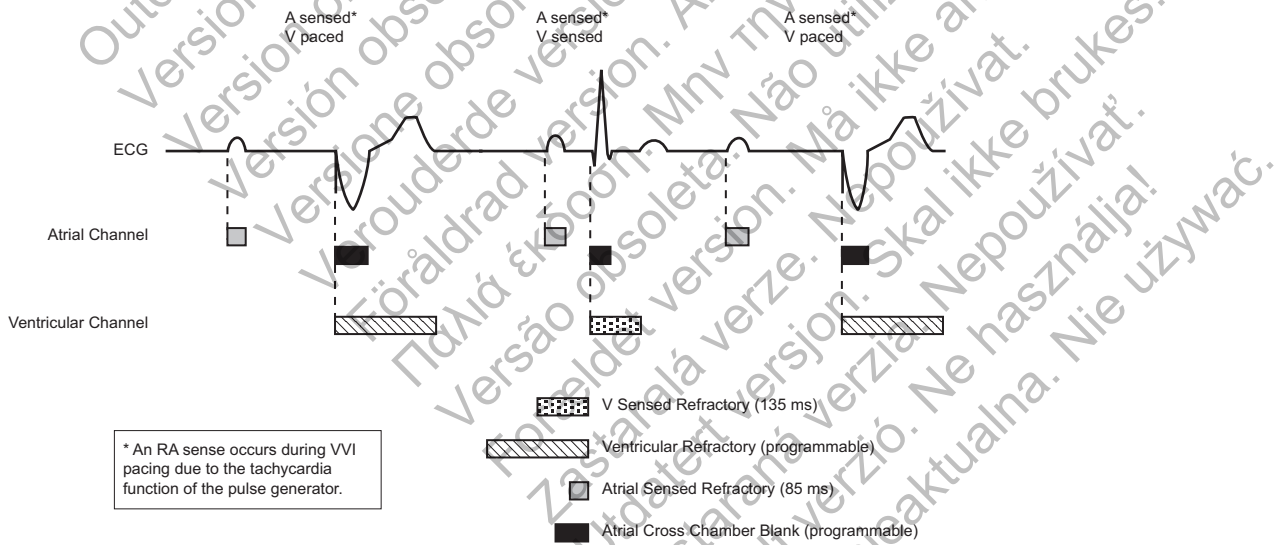


Figure 4-25. Refractory periods, VVI pacing mode

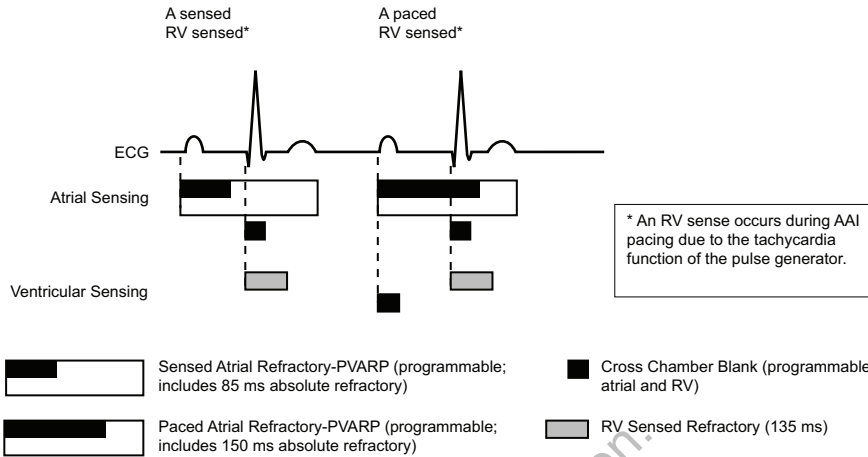


Figure 4-26. Refractory periods, AAI pacing mode; DR

NOISE RESPONSE

Noise windows and blanking periods are designed to prevent inappropriate therapy or pacing inhibition due to cross-chamber oversensing.

Noise Response allows the clinician to choose whether to pace or inhibit pacing in the presence of noise.

A retriggerable, 40-ms noise window exists within each refractory and fixed (non-smart) cross-chamber blanking period. The window is initiated by either a sensed or paced event. Both the noise window and the refractory period must be completed for each cardiac cycle in one chamber before the next event restarts the timing in the same chamber. Recurrent noise activity may cause the noise window to restart, extending the noise window and possibly the effective refractory period or blanking period.

The Noise Response parameter can be programmed to Inhibit Pacing or an asynchronous mode. The available asynchronous mode will automatically correspond to the permanent Brady Mode (i.e., VVI permanent mode will have VOO noise response). If Noise Response is programmed to an asynchronous mode and the noise persists so that the noise window is extended longer than the programmed pacing interval, the pulse generator paces asynchronously at the programmed pacing rate until the noise ceases. If Noise Response is programmed to Inhibit Pacing and persistent noise occurs, the pulse generator will not pace in the noisy chamber until after the noise ceases. The Inhibit mode is intended for patients whose arrhythmias may be triggered by asynchronous pacing.

Refer to the following illustrations.

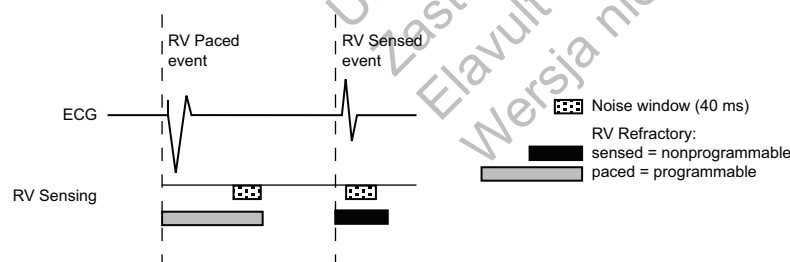


Figure 4-27. Refractory periods and noise windows, RV

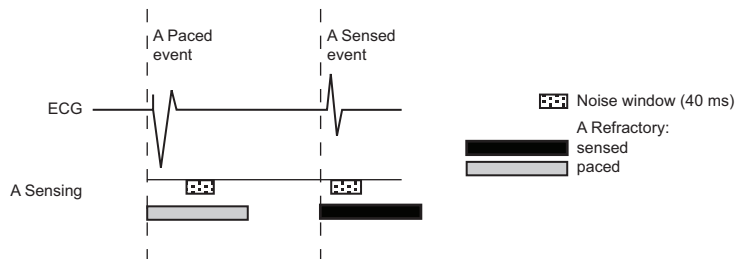


Figure 4-28. Refractory periods and noise windows, RA

In addition, a nonprogrammable Dynamic Noise Algorithm is active in all rate channels.

The Dynamic Noise Algorithm uses a separate noise channel to continuously measure the baseline signal and adjust the sensing floor to avoid noise detection. This algorithm is intended to help prevent oversensing of myopotential signals and the problems associated with oversensing.

The following noise event markers are generated:

Single-Chamber

- The marker [VS] occurs when the noise window is initially triggered following a V pace
- If retriggered for 340 ms, the marker VN occurs
- With continuous retriggers, the marker VN occurs frequently

Dual-Chamber

- Depending on the chamber where noise is occurring, the marker [AS] or [VS] occurs when the noise window is initially triggered following a pace
- If retriggered for 340 ms, the marker AN or VN occurs
- With continuous retriggers, the marker AN or VN occurs frequently
- If asynchronous pacing occurs due to continuous noise, the markers AP-Ns, VP-Ns will occur

NOTE: In pacer-dependent patients, use care when considering setting Noise Response to Inhibit as pacing will not occur in the presence of noise.

Noise Response example

Cross-chamber sensing that occurs early in the AV Delay may be detected by the RV sense amplifiers during the fixed blanking period, but is not responded to except to extend the noise rejection interval. The 40 ms noise rejection interval continues to retrigger until the noise is no longer detected, up to the length of the AV Delay. If noise continues throughout the duration of the AV Delay, the device will deliver a pacing pulse when the AV Delay timer expires, preventing ventricular inhibition due to noise. If a ventricular pacing spike is delivered under conditions of continuous noise, a VP-Ns marker notation appears on the intracardiac electrogram (Figure 4-29 on page 4-41).

If noise ceases prior to the expiration of the AV Delay, the device can detect an intrinsic beat that occurs at any time beyond the 40 ms retriggerable noise interval and initiate a new cardiac cycle.

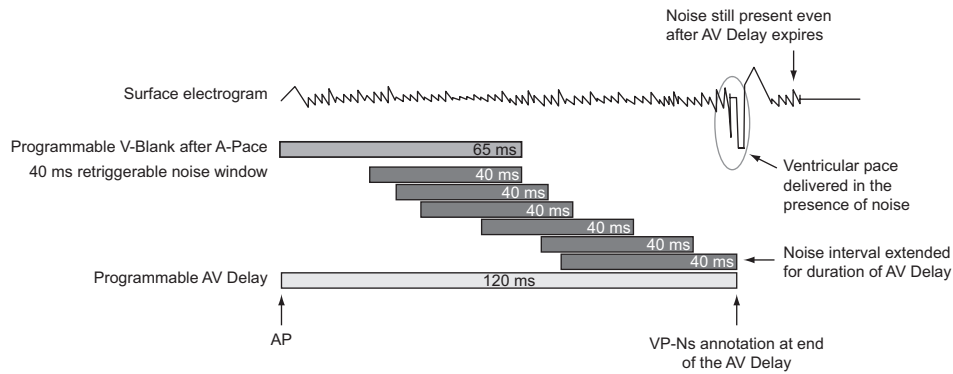


Figure 4-29. Noise Response (fixed blanking)

VENTRICULAR TACHY SENSING INTERACTIONS

Refractory periods and blanking intervals are an integral part of the pulse generator sensing system. They are used to efficiently suppress detection of pulse generator artifacts (e.g., a pace or shock) and certain intrinsic signal artifacts (e.g., a T-wave or far-field R-wave). The pulse generator does not discriminate between events that occur during refractory periods and blanking intervals. As a result, all events (pulse generator artifacts, intrinsic artifacts, and intrinsic events) that occur during a refractory period or blanking interval are ignored for purposes of pacing timing cycles and ventricular tachy detection.

Certain programmed combinations of pacing parameters are known to interfere with ventricular tachy detection. When an intrinsic beat from a VT occurs during a pulse generator refractory period, the VT beat will not be detected. As a result, detection and therapy of the arrhythmia may be delayed until enough VT beats are detected to satisfy the tachy detection criteria ("Ventricular Detection Windows" on page 2-10).

Pacing Parameter Combination Examples

The following examples illustrate the effects of certain pacing parameter combinations on ventricular sensing. When programming pulse generator pacing and tachy detection parameters, consider the possible interactions of these features in light of the expected arrhythmias. In general, the PRM screen displays Parameter Interaction Attentions and advisory messages to inform you about programming combinations that could interact to cause these scenarios; the interactions can be resolved by reprogramming the pacing rate, AV Delay and/or refractory/blanking periods.

Example 1: Ventricular Undersensing Due to Ventricular Refractory Period

If the pulse generator is programmed as follows, a VT that occurs synchronous with the pacing will not be detected:

- Brady Mode = VVI
- LRL = 75 min⁻¹ (800 ms)
- VRP = 500 ms
- VT Zone = 150 min⁻¹ (400 ms)

In this scenario, the pulse generator is VVI pacing at LRL (800 ms). A 500 ms VRP follows each ventricular pace. VT beats that occur during VRP are ignored for purposes of pacemaker timing and ventricular tachy detection/therapy. If a stable VT of 400 ms starts simultaneously with a ventricular pace, the VT will not be detected because every beat will occur during the 500 ms VRP, either concurrent with a ventricular pace or 400 ms after a pace (Figure 4-30 on page 4-42).

NOTE: It is not required for the VT to start concurrently with a pace for undersensing to occur. In this example, all pacing will be inhibited and tachy detection will subsequently occur, as soon as a single VT beat is detected.

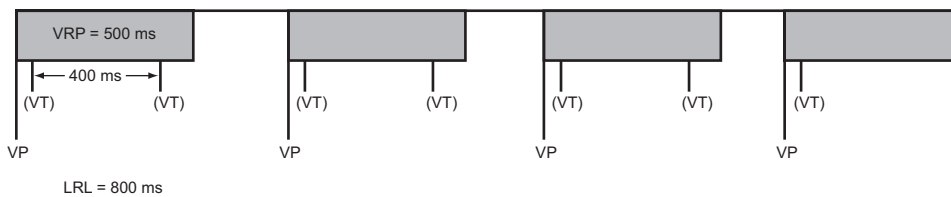


Figure 4-30. Ventricular undersensing due to VRP

When the programming interaction described in this scenario is present, a message will describe the interaction of VRP with LRL. In rate-responsive or tracking modes (e.g., DDDR), similar messages may describe the interaction of VRP with MTR, MSR, or MPR. Along with each message, the pertinent programmable parameters are displayed to assist you in resolving the interaction. Programming Dynamic VRP can be useful in resolving these types of interactions.

Example 2: Ventricular Undersensing Due To V-Blank After A-Pace

Certain programmed combinations of dual-chamber pacing parameters may also interfere with ventricular tachy detection. When dual-chamber pacing occurs, pulse generator refractory periods are initiated by both atrial and ventricular paces. The ventricular refractory period following a ventricular pace is controlled by the VRP parameter; the ventricular refractory period following an atrial pace is controlled by the V-Blank After A-Pace parameter.

Undersensing of a VT due to the pulse generator refractory periods may occur when the pulse generator is pacing at or above LRL. For example, if the pulse generator is rate-adaptive pacing at 100 min^{-1} (600 ms) and is programmed as follows, then a VT that occurs synchronous with the pacing may not be detected:

- LRL = 90 min^{-1} (667 ms), MTR/MSR = 130 min^{-1} (460 ms)
- Brady Mode = DDDR, fixed AV delay = 300 ms
- VRP = 230 ms
- V-Blank After A-Pace = 65 ms
- VT zone = 150 min^{-1} (400 ms)

In this scenario, the pulse generator is DDDR pacing at 600 ms. A VRP of 230 ms follows each ventricular pace; a ventricular refractory period of 65 ms (V-Blank After A-Pace) follows each atrial pace; an atrial pace occurs 300 ms after each ventricular pace. VT beats that occur during either refractory period are ignored for purposes of pacemaker timing and ventricular tachy detection/therapy. If a stable VT of 350 ms starts, then the VT will not be detected because most beats will occur during a ventricular refractory period, either V-Blank After A-Pace or VRP. Some VT beats will be detected, but not enough to satisfy the 8 of 10 tachy detection criteria ("Ventricular Detection Windows" on page 2-10).

NOTE: It is not required for the VT to start concurrently with a refractory period or blanking interval for undersensing to occur. In this example, it is likely that the VT will not be detected until either the VT accelerates to faster than 350 ms or the sensor-driven pacing rate changes from 600 ms.

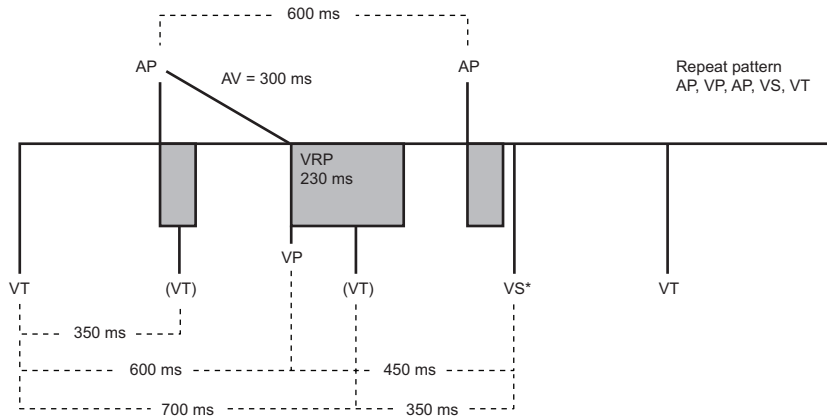


Figure 4-31. Ventricular undersensing due to V-Blank after A-Pace

When the programming interaction described in this scenario is present, a message will describe the interaction of Tachy Rate Threshold with LRL and AV Delay. Similar messages may describe the interaction of V-Blank After A-Pace with MTR, MPR, or LRL. Along with each message, the pertinent programmable parameters are displayed to assist you in resolving the interaction. Programming Dynamic VRP can be useful in resolving these types of interactions.

Programming Considerations

Certain programmed combinations of pacing parameters are known to interfere with ventricular tachy detection. The risk of ventricular tachy undersensing due to device refractory periods is indicated by the interactive warnings on the parameter screen.

As with all device programming, you should evaluate the benefits and the risks of the programmed features for each patient (for example, the benefit of Rate Smoothing with a long AV Delay versus the risk of ventricular tachy undersensing).

The following programming recommendations are provided to reduce the risk of ventricular undersensing due to the refractory period caused by an atrial pace (V-Blank after A-Pace):

- If a dual-chamber pacing mode with Rate Smoothing or Rate Adaptive Pacing is necessary:
 - Reduce the LRL
 - Shorten the AV Delay or use Dynamic AV Delay and reduce the minimum Dynamic AV Delay setting
 - Reduce the Search AV Delay for AV Search +
 - Increase the Down Rate Smoothing percentage to the largest possible value
 - Decrease the recovery time for Rate Adaptive Pacing modes
 - Reduce the MTR or MPR if Down Rate Smoothing is on
 - Reduce the MSR if the pacing mode is rate adaptive
- If Rate Smoothing or Rate Adaptive Pacing are not required for the patient, consider programming these features Off. Programming these features Off can reduce the likelihood of atrial pacing at elevated rates.
- If atrial pacing is not required for the patient, consider using VDD rather than DDD pacing mode.

- In certain usage scenarios, you may elect to program long AV Delays to reduce ventricular pacing for patients with long PR intervals, while providing sensor pacing or rate smoothing to address other patient needs.
- In certain usage scenarios, if a pattern of atrial pacing and VT beats is detected, the AV delay is automatically adjusted to facilitate confirmation of a suspected VT. If no VT is present, the AV delay is returned to the programmed value. For programming scenarios where the automatic AV delay adjustment may occur, a specific Parameter Interaction Attention will not be displayed.

For discussion of details and additional information regarding these or other programmed settings, please contact Boston Scientific using the information on the back cover.

In summary, when programming the pulse generator pacing and tachy detection parameters, it is useful to consider the possible interactions of these features in light of the expected arrhythmias of a particular patient. In general, the interactions will be brought to your attention through Parameter Interaction Attention messages on the PRM screen and can be resolved by reprogramming the pacing rate, AV delay, and/or refractory/blanking periods.

Outdated version. Do not utilize.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

SYSTEM DIAGNOSTICS

CHAPTER 5

This chapter contains the following topics:

- "Summary Dialog" on page 5-2
- "Battery Status" on page 5-2
- "Leads Status" on page 5-7
- "Lead Tests" on page 5-10

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzate.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

SUMMARY DIALOG

Upon interrogation, a Summary dialog is displayed. It includes Leads and Battery status indications, approximate time to explant, and an Events notification for any episodes since the last reset. In addition, a magnet notification will appear if the pulse generator detects the presence of a magnet.

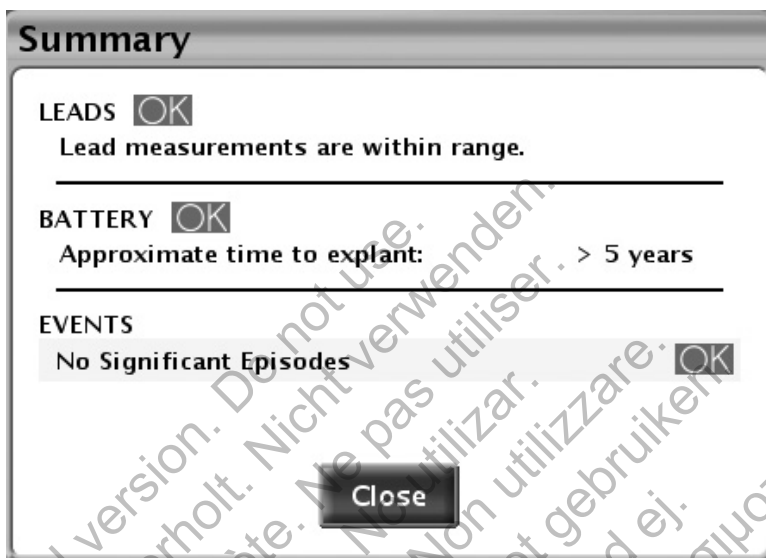


Figure 5-1. Summary dialog

Potential status symbols include OK, Attention, or Warning ("Use of Color" on page 1-7). Potential messages are described in the following sections:

- Leads—"Leads Status" on page 5-7
- Battery—"Battery Status" on page 5-2
- Events—"Therapy History" on page 6-2

Once the Close button is selected, the Warning or Attention symbols for Leads and Battery will not appear on subsequent interrogations until additional events triggering an alert condition occur. Events will continue to appear until any history counter Reset button is selected.

BATTERY STATUS

The pulse generator automatically monitors battery capacity and performance. Battery status information is provided via several screens:

- Summary dialog—displays a basic status message about remaining battery capacity ("Summary Dialog" on page 5-2).
- Summary tab (on the Main Screen)—displays the same basic status message as the Summary dialog, along with the battery status gauge ("Main Screen" on page 1-2).
- Battery Status Summary screen (accessed from the Summary tab)—displays additional battery status information about remaining battery capacity and charge time ("Battery Status Summary Screen" on page 5-3).

- Battery Detail screen (accessed from the Battery Status Summary screen)—provides detailed information about battery use, capacity, and performance ("Battery Detail Summary Screen" on page 5-5).

Battery Status Summary Screen

The Battery Status Summary screen provides the following key information about battery capacity and performance.

Time Remaining

This section of the screen displays the following items:

- Battery status gauge—displays a visual indication of the battery capacity status.
- Approximate time to explant—displays the estimate of calendar time remaining until the pulse generator reaches the Explant status.

This estimate is calculated using battery capacity consumed, charge remaining, and power consumption at current programmed settings.

When insufficient usage history is available, Approximate time to explant may change between interrogation sessions. This fluctuation is normal, and occurs as the pulse generator collects new data and can calculate a more stable prediction. Approximate time to explant will be more stable after several weeks of usage. Causes of fluctuation may include the following:

- If certain brady features that affect pacing output are reprogrammed, the Approximate time to explant will be forecasted based on the expected changes in power consumption from the reprogrammed features. The next time the pulse generator is interrogated, the PRM will resume displaying Approximate time to explant based on recent usage history. As new data is collected, Approximate time to explant will likely stabilize near the initial forecast.
- For several days post-implant, the PRM will display a static Approximate time to explant based on model-dependent data. Once enough usage data has been collected, device-specific predictions will be calculated and displayed.

Charge Time

This section of the screen displays the amount of time it took the pulse generator to charge for the most recent maximum-energy shock or capacitor re-formation.

Battery Detail icon

When selected, this icon displays the Battery Detail Summary screen ("Battery Detail Summary Screen" on page 5-5).

Battery Status Indicators

The following battery status indicators appear in the battery status gauge. The indicated Approximate time to explant is calculated based on the pulse generator's current programmed parameters.

One Year Remaining—approximately one year of full pulse generator function remains (Approximate time to explant is one year).

Explant—the battery is nearing depletion, and pulse generator replacement must be scheduled. Once Explant status is reached there is sufficient battery capacity to monitor and pace 100% under existing conditions for three months and to deliver three maximum-energy shocks, or to

deliver six maximum energy shocks with no pacing. When Explant status is reached, 1.5 hours of ZIP telemetry remain. Consider using wanded telemetry.

NOTE: When the 1.5 hours of telemetry are exhausted, a LATITUDE alert is generated.

Battery Capacity Depleted—pulse generator functionality is limited, and therapies can no longer be guaranteed. This status is reached three months after Explant status is reached. The patient should be scheduled for immediate device replacement. Upon interrogation, the Limited Device Functionality screen is displayed (all other screens are disabled). This screen provides battery status information and access to remaining device functionality. ZIP telemetry is no longer available.

NOTE: A LATITUDE alert is generated, after which LATITUDE is no longer available.

When the device reaches Battery Capacity Depleted status, functionality is limited to the following:

- Brady Mode will be changed as described below:

Brady Mode prior to Battery Capacity Depleted Indicator	Brady Mode after Battery Capacity Depleted Indicator
DDD(R), DDI(R), VDD(R), VVI(R)	VVI
AAI(R)	AAI
Off	Off

- Brady Mode and Ventricular Tachy Mode can be programmed to Off; no other parameters are programmable
- One ventricular zone (VF) with a rate threshold of 165 min⁻¹
- Wanded telemetry only (RF telemetry is disabled)
- Maximum-energy shocks and manual capacitor reformations only (ATP therapy and low-energy shocks are disabled)
- An LRL of 50 min⁻¹

At Battery Capacity Depleted status, the following features are disabled:

- Daily measurement trends
- Brady enhancements (e.g., rate response, rate smoothing)
- Episode storage
- Diagnostic and EP tests
- Real-time EGMs
- Accelerometer

If the device reaches a point where insufficient battery capacity is available for continued operation, the device will revert to Storage Mode. In Storage Mode, no functionality is available.

NOTE: The device uses the programmed parameters and recent usage history to predict Approximate time to explant. Greater than normal battery usage may result in the subsequent day's Approximate time to explant to appear less than expected.

NOTE: As a backup method, Explant is declared when two consecutive charge times exceed 15 seconds each. If a charge time is greater than 15 seconds, the pulse generator schedules an automatic capacitor re-formation for one hour later. If the charge time during re-formation also exceeds 15 seconds, the battery status is changed to Explant.

Battery Detail Summary Screen

The Battery Detail summary screen provides the following information about pulse generator battery status (Figure 5-2 on page 5-6):

- Last Delivered Shock—date, energy, charge time, and shock impedance data.
- Beep When Explant Is Indicated—if this feature is programmed to On, the pulse generator emits 16 beeping tones every six hours after it reaches the Explant indicator. The tone can then be programmed to Off. Even when this feature is programmed to Off, it is automatically reactivated when the Battery Capacity Depleted indicator is reached.

CAUTION: Patients should be advised to contact their physician immediately if they hear tones coming from their device.

- Last Capacitor Re-form—date and charge time.
- Manual Re-form Capacitor—this feature is used to command a capacitor re-formation when needed.
- Charge Remaining (measured in ampere-hours)—the amount of charge remaining based on the pulse generator's programmed parameters until the battery is depleted.
- Power Consumption (measured in microwatts)—the average daily power being used by the pulse generator, based on currently programmed parameters. Power consumption is included in the calculations that determine Approximate time to explant and the needle position on the battery status gauge.
- Power Consumption Percentage—compares the power consumption at the pulse generator's currently programmed parameters with the power consumption of the standard parameters used to quote device longevity.

If any of the following parameters (which affect pacing output) are reprogrammed, the Power Consumption and Power Consumption Percentage values are adjusted accordingly:

- Amplitude
- Pulse Width
- Brady Mode
- LRL
- MSR

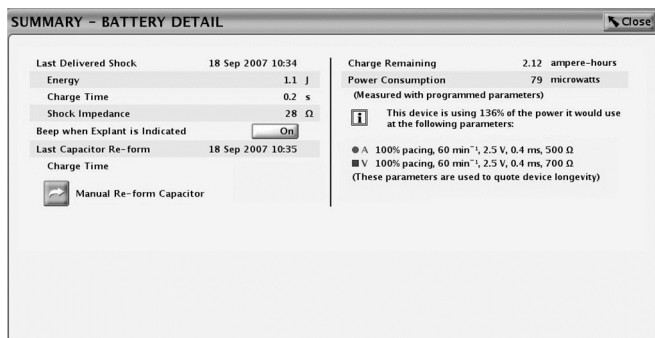


Figure 5-2. Battery Detail summary screen

Capacitor Re-formation

Automatic Capacitor Re-form. Capacitor deformation may occur during periods when no shocks are delivered, resulting in longer charge times. To reduce the effect of capacitor deformation on charge time, the capacitors are automatically re-formed. Tones will not be emitted from the pulse generator during automatic capacitor re-formations (even if the Beep During Capacitor Charge feature is programmed to On). During a capacitor re-formation, the Charge Time is measured and stored for later retrieval.

Manual Capacitor Re-form. Manual capacitor re-forms are not necessary, but may be commanded via the PRM as follows:

1. Select the Manual Re-form Capacitor button on the Battery Detail screen and ensure that telemetry communication is established. A message will appear indicating that the capacitors are charging. Warbling tones from the pulse generator (if the Beep During Capacitor Charge feature is programmed to On) will sound while the capacitors are charging.
2. The entire re-form cycle typically takes less than 15 seconds. After completion of the cycle, the capacitor energy is delivered to the pulse generator's internal test load. The initial Charge Time is displayed on the Battery Detail screen.

Charge Time Measurement

The pulse generator measures the Charge Time whenever its capacitors charge. The last measured value is stored in pulse generator memory and displayed by the PRM system on the Battery Detail screen.

Last Delivered Ventricular Shock

When a shock has been delivered to the patient, the following information from the last shock delivered is stored in the pulse generator's memory and displayed on the Battery Detail screen:

- Date
- Energy level
- Charge time
- Shocking lead impedance

This does not include auto capacitor re-forms or shocks that may have been diverted. If a fault condition is encountered (i.e., high or low impedance), the fault will be indicated so that corrective action may be taken.

NOTE: For shocks of 1.0 J or less, the accuracy of the impedance measurement decreases.

LEADS STATUS

Daily Measurements

The device performs the following measurements every 21 hours and reports them daily:

- Daily intrinsic amplitude measurement: the device will automatically attempt to measure the intrinsic P- and R- wave amplitudes for each cardiac chamber in which the daily intrinsic amplitude measurement is enabled regardless of the pacing mode. This measurement will not affect normal pacing. The device will monitor up to 255 cardiac cycles to find a sensed signal to obtain a successful measurement.
- Daily lead impedance measurement:
 - Pace lead(s)—the device will automatically attempt to measure the pace lead impedance for each chamber in which the daily pace impedance test is enabled, regardless of the pacing mode. To conduct the lead impedance test the device utilizes a sub-pacing threshold signal that will not interfere with normal pacing or sensing.
 - The High Impedance Limit is fixed at 2000 Ω . The Low Impedance Limit is nominally set to 200 Ω , and is programmable between 200 and 500 Ω in 50 Ω increments. Consider the following factors when choosing a value for the Low Impedance Limit:
 - For chronic leads, historical impedance measurements for the lead, as well as other electrical performance indicators such as stability over time
 - For newly implanted leads, the starting measured impedance value
 - **NOTE:** Depending on lead maturation effects, during follow-up testing the physician may choose to reprogram the Low Impedance Limit.
 - Pacing dependence of the patient
 - Recommended impedance range for the lead(s) being used, if available
 - Shock lead—the device will automatically attempt to measure the shock lead impedance. During a shock lead impedance test, the pulse generator delivers a subthreshold energy pulse through the shocking electrodes. These impedance measurements may show some variation over time, since they are taken every 21 hours and, thus, at different times of the day.
 - The pulse generator contains a beeper that emits audible tones to communicate status information. The beeper includes a programmable feature that, when programmed On, will cause the pulse generator to emit tones when Daily Impedance values are out of range. The Beep When Out-of-Range indicator consists of 16 tones repeated every six hours. When this feature is programmed Off, there is no audible indication of out of range Daily Impedance values. Refer to "Beeper Feature" on page 6-14.

Basic lead status information is displayed on the Summary screen. Detailed data are displayed in a graphical format on the Leads Status summary screen, which can be accessed by selecting the leads icon on the Summary screen (Figure 5-3 on page 5-8).

Possible leads status messages are as follows (Table 5-1 on page 5-8):

- Lead measurements are within range.


- Check Lead (message will specify which lead)—indicates daily lead measurement(s) are out of range. To determine which measurement is out of range, evaluate the corresponding lead’s daily measurement results.

Table 5-1. Lead measurement reporting

Lead Measurement	Reported Values	Out-of-Range Limits
A Pace Impedance (Ω)	200 to 2000	Low: \leq programmed Atrial Low Impedance Limit High: \geq 2000
RV Pace Impedance (Ω)	200 to 2000	Low: \leq programmed Right Ventricular Low Impedance Limit High: \geq 2000
Shock Impedance (Ω)	0 to 200	Low: \leq 20 High: \geq 125
P-Wave Amplitude (mV)	0.1 to 25.0	Low: 0.5 High: none
R-Wave (RV) Amplitude (mV)	0.1 to 25.0	Low: 3.0 High: none

The Leads Status summary screen provides daily measurement details for applicable leads (Figure 5-3 on page 5-8):

- The graph shows daily measurements from the past 52 weeks.
- Use the tabs across the top of the screen to view data for each lead. Select the Setup tab to enable or disable specific daily lead measurements or to set the Low Impedance Limit values.
- Each data point represents the daily measurement for a given day. To view specific results for a day, move the horizontal slider over the corresponding data point or gap.
- An out-of-range measurement will plot a point at the corresponding maximum or minimum value.
- A gap will be generated if the device is unable to obtain a valid measurement for that day.
- The most recent daily measurements are displayed at the bottom of the screen.

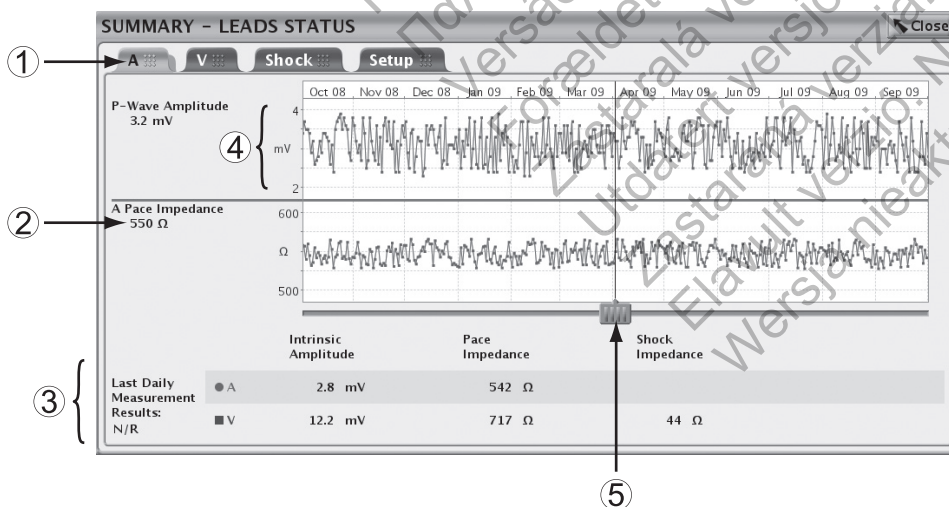

[1] Use tabs to select the appropriate lead [2] Results for the selected day [3] Results for most recent day [4] Y-axis adjusts based on measured results [5] Use horizontal slider to view data for a specific day

Figure 5-3. Leads Status summary screen

If the device is unable to obtain one or more daily measurements at the scheduled time, up to three re-attempts will be performed at one-hour intervals. Re-attempts do not change the timing of daily measurements. The next day's measurement will be scheduled 21 hours from the initial attempt.

If a valid measure is not recorded after the initial attempt plus three re-attempts, or is not recorded at the end of a 24-hour time block, the measurement will be reported as Invalid Data or No Data Collected (N/R).

Because eight measurements are recorded in seven days, one day will contain two measurements. If one measurement is valid and one invalid, the valid measurement will be reported. If both measurements are valid, the second value will be reported.

If the Summary screen indicates that a lead should be checked and the Intrinsic Amplitude and Impedance graphs do not show any out-of-range values or gaps, the test that resulted in the out-of-range value occurred within the current 24 hours and has not yet been saved with the daily measurements.

Table 5-2. Intrinsic Amplitude: Daily Measurement Conditions, Programmer Display, and Graphical Representation

Condition	Programmer Display	Graphical Representation
In-range amplitude measurement	Measurement value	Plotted point
Electrode configuration is programmed to Off/None	No Data Collected	Gap
All events during the test period are paced	Paced	Gap
Noise detected during the test period	Noise	Gap
Sensed events defined as a PVC	PVC	Gap
Sensed events defined as a PAC	PAC	Gap
Out-of-range amplitude measurements	0.1, 0.2, ..., 0.5 mV (RA lead) with attention icon	Plotted point
	0.1, 0.2, ..., 3.0 mV (ventricular lead) with attention icon	Plotted point at corresponding minimum
	< 0.1 mV with attention icon	Plotted point at corresponding maximum ^a

a. When the value measured is > 25 mV, an attention symbol is displayed on the graph even though no alert is generated on the summary screens.

Table 5-3. Lead Impedance: Daily Measurement Conditions, Programmer Display, and Graphical Representation

Condition	Programmer Display	Graphical Representation
In-range amplitude measurement	Measurement value	Plotted point
Electrode configuration is programmed Off/None	Invalid Data	Gap
Noise detected during the test period	Noise	Gap
> 2000 Ω with attention icon < 200 Ω with attention icon	Plotted point at corresponding minimum or maximum ^a	
Out-of-range impedance measurements (pace leads)	2000 Ω with attention icon Measured value less than or equal to the programmed Low Impedance Limit (200–500 Ω) with attention icon	Plotted point
	> 2000 Ω with attention icon < 200 Ω with attention icon	Plotted point at corresponding minimum or maximum ^a
Out-of-range impedance measurements (shock lead)	200 Ω; attention icon generated at ≥ 125 Ω 0 Ω; attention icon generated at ≤ 20 Ω	Plotted point
	> 200 Ω; attention icon generated	Plotted point at corresponding maximum ^b

a. Selecting these points will not display the numerical value, but will indicate that the value is above the upper range limit or below the lower range limit, as appropriate.

b. Selecting these points will not display the numerical value, but will indicate that the value is above the upper range limit.

Under the following conditions, Intrinsic Amplitude and Lead Impedance measurements will not be attempted. The programmer display will indicate No Data Collected or Invalid Data, and there will be a gap in the graphical representation:

- Tachy episode is in progress
- Tachy therapy is active
- Telemetry is active
- Post-Therapy parameters are in effect
- Device battery capacity is depleted
- LATITUDE interrogation is in progress
- Pulse generator is in Electrocautery Protection Mode

LEAD TESTS

The following lead tests are available (Figure 5-4 on page 5-10):

- Pace Impedance
- Shock Impedance
- Intrinsic Amplitude
- Pace Threshold

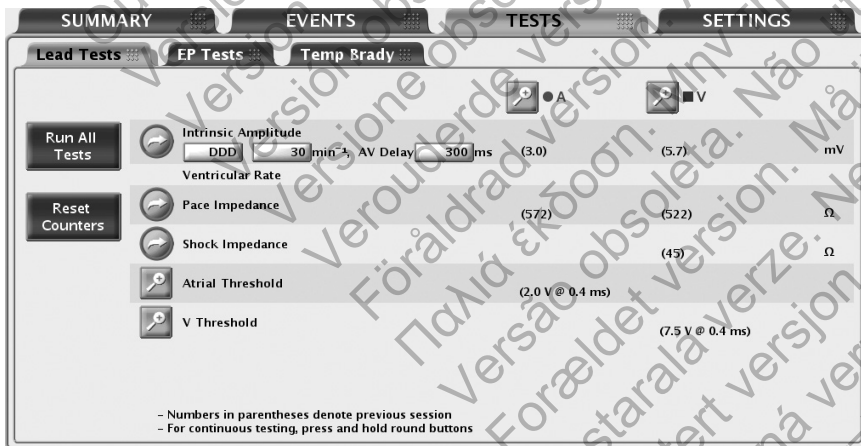


Figure 5-4. Lead Tests screen

Lead tests can be accessed by using the following steps:

1. From the main screen, select the Tests tab.
2. From the Tests screen, select the Lead Tests tab.

All lead tests may be performed following two different processes:

- Via the Lead Tests screen—allows you to perform the same lead tests across all chambers
- By selecting the desired chamber button—allows you to perform all tests on the same lead

Intrinsic Amplitude Test

The intrinsic amplitude test measures the intrinsic P- and R-wave amplitudes for the respective chambers.

An intrinsic amplitude test can be performed from the Lead Tests screen by completing the following steps:

1. You may change the following preselected values as necessary to elicit intrinsic activity in the chamber(s) being tested:
 - Programmed Normal Brady Mode
 - LRL at 30 min⁻¹
 - AV Delay at 300 ms
2. Select the Intrinsic Amplitude button. During the test, a window will display the test's progress. Selecting and holding the Intrinsic Amplitude Button will cause measurements to be repeated for up to 10 seconds or until the button is released. When the window closes, the same test can be performed again by selecting the Intrinsic Amplitude button. To cancel the test, select the Cancel button or press the DIVERT THERAPY key on the PRM.
3. When the test is complete, the intrinsic amplitude measurement will be displayed as the current measurement (not in parentheses). If the test is repeated during the same session, the current measurement will be updated with the new result. Note that the previous session measurement (displayed in parentheses) is from the most recent past session during which this test was performed.

NOTE: The test results from the last measurement are stored in pulse generator memory, retrieved during the initial interrogation, and displayed on the Lead Tests screen. The measurements are also provided on the Quick Notes report.

Lead Impedance Test

A lead impedance test can be performed and used as a relative measure of lead integrity over time.

If the lead integrity is in question, standard lead troubleshooting tests should be used to assess the lead system integrity.

Troubleshooting tests include, but are not limited to, the following:

- Electrogram analysis with pocket manipulation and/or isometrics
- X-ray or fluoroscopic image review
- Additional maximum-energy shocks
- Programming the Shock Lead Vector
- Invasive visual inspection

A shock impedance test is a useful tool in detecting shocking lead integrity changes over time. Evaluating this information together with the Last Delivered Shock impedance (displayed on the Battery Detail screen) or a subsequent high-energy shock impedance and other non-invasive diagnostic techniques may help troubleshoot potential lead system conditions.

A test result of NOISE is reported if a valid measurement could not be obtained (likely due to EMI).

Shorted and open shock lead failures will be reported as 0 Ω and > 200 Ω , respectively.

NOTE: *The shock impedance test may result in slightly higher values than delivered shock impedance measurements.*

Both low-energy and high-energy impedance tests have the following clinical limitations:

- A high- or maximum-energy shock test does not expose all forms of open lead conditions. Shock lead impedance measured during a commanded maximum-energy shock may appear normal when certain types of open lead conditions exist (e.g., lead conductor fracture or a loose setscrew), as the energy delivered could jump or arc across small gaps. A commanded low-energy impedance test is a more robust tool for identifying and verifying a potential open shocking lead condition.
- A low-energy lead impedance test does not expose all forms of shorted lead conditions. A low-energy impedance test may appear normal when certain types of shorted lead conditions exist (e.g., abraded lead body insulation or lead crushed between clavicle and first rib), as test energy is insufficient to jump or arc across small gaps between exposed conductors. A maximum-energy shock is a more robust tool for identifying and verifying a potential shorted shocking lead condition.

Pace and Shock lead impedance tests can be performed from the Lead Tests screen by completing the following steps:

1. Select the desired lead impedance test button. Selecting and holding a button will cause measurements to be repeated for up to 10 seconds or until the button is released.
2. During the test, a window will display the test progress. When the window closes, the same test can be performed by once again selecting the desired lead impedance test button. To cancel the test, select the Cancel button or press the DIVERT THERAPY key on the PRM.
3. When the test is complete, the impedance measurement will be displayed as the current measurement (not in parentheses). If the test is repeated during the same session, the current measurement will be updated with the new result. Note that the previous session measurement (displayed in parentheses) is from the most recent past session during which this test was performed.
4. If the test results in NOISE, consider the following mitigation options:
 - Repeat the test
 - Switch telemetry modes
 - Remove other sources of electromagnetic interference

NOTE: *The test results from the last measurement are stored in pulse generator memory, retrieved during the initial interrogation, and displayed on the Lead Tests screen. The measurements are also provided on the Quick Notes report.*

Pace Threshold Test

The Pace Threshold Test determines the minimum output needed for capture in a specific chamber.

Ventricular and atrial amplitude threshold tests are performed manually by selecting the Amplitude option on the Pace Threshold details screen.

Ventricular and atrial pulse width threshold tests are performed manually by selecting the Pulse Width option on the Pace Threshold details screen.

Manual Pace Threshold Test

The minimum 2X voltage or 3X pulse width safety margin is recommended for each chamber based on the capture thresholds, which should provide an adequate safety margin and help preserve battery longevity. The test begins at a specified starting value and steps that value down (amplitude or pulse width) as the test progresses. The PRM beeps with each decrement. The values used during the threshold test are programmable. The parameters are only in effect during the test.

NOTE: The starting values for Amplitude and Pulse Width values are automatically calculated. The device retrieves the stored results for the previous pace threshold measurement (for the parameter being tested) and sets the parameter at three steps above the previous threshold measurement. The LRL is preselected at 90 min^{-1} . For DDD mode, the LRL is further limited to 10 min^{-1} below the MTR.

NOTE: If DDD mode is chosen, selecting either the atrial or ventricular test will cause the pacing output to decrease only in the chamber selected.

Once the test is started, the device operates with the specified brady parameters. Using the programmed number of cycles per step, the device then decrements (steps down) the selected test type parameter (Amplitude or Pulse Width) until the test is complete. Real-time electrograms and annotated event markers, which include the values being tested, continue to be available during threshold testing. The display will automatically adjust to reflect the chamber being tested.

During the threshold test, the programmer displays the test parameters in a window while the test is in progress. To pause the test or perform a manual adjustment, select the Hold button on the window. Select the + or – button to manually increase or decrease the value being tested. To continue the test, select the Continue button.

The threshold test is complete and all parameters are returned to the normal programmed values when any of the following occur:

- The test is terminated via a command from the PRM (e.g., pressing the End Test button or DIVERT THERAPY key).
- The lowest available setting for Amplitude or Pulse Width is reached and the programmed number of cycles has completed.
- Telemetry communication is interrupted.

A pace threshold test can be performed from the Lead Tests screen using the following steps:

1. Select the desired chamber to be tested.
2. Select the Pace Threshold details button.
3. Select the test type.
4. Change the following parameter values as desired to elicit pacing in the chamber(s) being tested:
 - Mode
 - LRL
 - Paced AV Delay
 - Amplitude
 - Pulse Width
 - Cycles per Step

For DDD mode, the normal Brady MTR is used.

5. Watch the ECG display and stop the test by selecting the End Test button or pressing the DIVERT THERAPY key when loss of capture is observed. If the test continues until the programmed number of cycles at the lowest setting have occurred, the test is automatically terminated. The final threshold test value will be displayed (the value is one step above the value when the test was terminated).

NOTE: The threshold test result can be edited by selecting the Edit Today's Test button on the Threshold Test screen

6. When the test is complete, the threshold measurement will be displayed as the current measurement (not in parentheses). If the test is repeated during the same session, the current measurement will be updated with the new result. Note that the previous session measurement (displayed in parentheses) is from the most recent past session during which this test was performed.
7. To perform another test, make changes to the test parameter values if desired, then begin again. Results of the new test will be displayed.

NOTE: The test results from the most recent measurement are stored in pulse generator memory, retrieved during initial interrogation, and displayed on the Lead Tests screen and on the Lead Status screen. The measurements are also provided on the Quick Notes report.

PATIENT DIAGNOSTICS AND FOLLOW UP

CHAPTER 6

This chapter contains the following topics:

- "Therapy History" on page 6-2
- "Arrhythmia Logbook" on page 6-2
- "Histograms" on page 6-7
- "Counters" on page 6-8
- "Trends" on page 6-9
- "Post Implant features" on page 6-12

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolete. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

THERAPY HISTORY

The pulse generator automatically records data that can be helpful when evaluating the patient's condition and the effectiveness of pulse generator programming.

Therapy history data can be reviewed at various levels of detail using the PRM:

- Arrhythmia Logbook—provides detailed information for each detected episode ("Arrhythmia Logbook" on page 6-2)
- Histograms and Counters—displays the total number and percentage of paced and sensed events during a particular recording period ("Histograms" on page 6-7 and "Counters" on page 6-8)
- Trends—provides a graphical view of specific patient, pulse generator, and lead data ("Trends" on page 6-9)

NOTE: The Summary dialog and Summary tab display a prioritized list of events that have occurred since the last reset. This list will only include VF, and VT/VT-1 and ATR (if it lasted more than 48 hours) episodes.

ARRHYTHMIA LOGBOOK

The Arrhythmia Logbook provides access to the following detailed information about episodes of all types (Figure 6-1 on page 6-3):

- The number, date, and time of the event
- The type of event with tachyarrhythmia zone
- A summary of therapy delivered or attempted (if applicable)
- Duration of the event (when applicable)
- Electrograms with annotated markers
- Intervals

NOTE: The data include information from all active electrodes. The device compresses the history data to store a maximum of 17 minutes of electrogram data (13 minutes with Patient Triggered Monitor enabled). However, the amount of time actually stored may vary based on the data being compressed (e.g., noise on the EGM or an episode of VF).

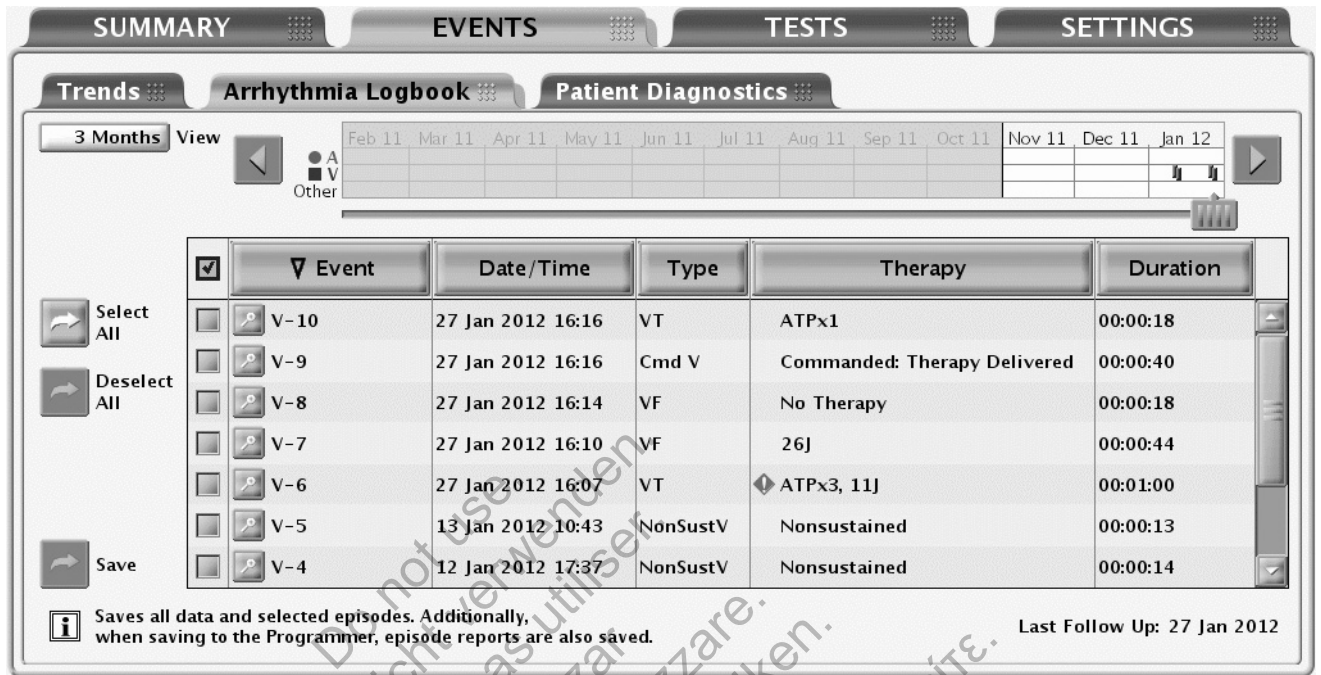


Figure 6-1. Arrhythmia Logbook screen

The priority, maximum number, and minimum number of episodes that the pulse generator stores under normal conditions varies by episode type (Table 6-1 on page 6-4). As long as device memory allocated for episode data is not full, the pulse generator stores up to the maximum number of episodes allowed for each episode type. The minimum number of episodes for each episode type ensures that all episode types are represented by protecting a few low priority episodes from being overwritten by high priority episodes when device memory is full.

Once device memory is full, the pulse generator attempts to prioritize and overwrite stored episodes according to the following rules:

1. If device memory is full, and there are episodes older than 18 months, then the oldest of the lowest priority episodes from these episode types will be deleted (regardless if the minimum number of episodes are stored).
 2. If device memory is full, and there are episode types that have more than the minimum number of episodes stored, then the oldest of the lowest priority episodes from these episode types will be deleted. In this case, the low priority episodes are not deleted if their number of stored episodes is less than the minimum number.
 3. If device memory is full, and there are no episode types that have more than the minimum number of episodes stored, then the oldest of the lowest priority episodes of all episode types will be deleted.
 4. If the maximum number of episodes has been reached within an episode type, the oldest episode of that type will be deleted.
- For non-commanded episodes, the episode type for VT-1, VT, and VF episodes is determined according to the zone Duration that expires first. If no zone Duration expires during an episode, the episode type is nonsustained.
 - An episode in progress has the highest priority until its type can be determined.

NOTE: Once history data is saved, it can be accessed at any time without device interrogation.

Table 6-1. Episode Priority

Episode Type	Priority	Maximum number of stored episodes	Minimum number of stored episodes with detailed reports	Maximum number of stored episodes with detailed reports
VF/VT/VT-1 with shock ^a	1	50	5	30
PTM (Patient Triggered Monitor)	1	5	1	1
VF/VT/VT-1 with ATP ^b	2	30	2	15
VF/VT/VT-1 with no therapy (Duration met) ^c	3	20	1	10
Cmd V	4	2	0	2
NonSustV (Duration not met)	4	10	0	2
ATR ^d	4	10	1	3
PMT ^d	4	5	1	3
APM RT ^e	4	1	1	1
RYTHMIQ ^d	4	10	1	3

- a. May also include ATP therapy.
- b. ATP without shock therapy
- c. No therapy defined as Duration met with: an Inhibit decision, detection in the Monitor Only zone, the last sensed interval not in the zone, or a Divert-Reconfirm.
- d. Not available in VR models.
- e. Advanced Patient Management real time (APM RT) events are presenting EGMs, captured and stored on the pulse generator during LATITUDE Communicator follow-ups.

To display Arrhythmia Logbook data, use the following steps:

1. From the Events tab, select Arrhythmia Logbook. If necessary, the pulse generator will be automatically interrogated and current data will be displayed. Saved patient data also can be displayed ("Data Storage" on page 1-15).
2. While retrieving the data, the programmer will display a window indicating the progress of the interrogation. No information will be displayed if you select the Cancel button before all of the stored data are retrieved.
3. Use the slider and View button to control the range of dates for the events you want to display in the table.
4. Select the Details button of an event in the table to display the event details. Event details, available if the details button is present, are useful in evaluating each episode. The Stored Event screen will appear, and you can browse between the following tabs for more information about the event:
 - Events Summary
 - EGM
 - Intervals
5. Select a column header button to sort the events by that column. To reverse the order, select the column header again.
6. To save specific events, select the event and choose the Save button. To print specific events, select the event and choose Reports from the toolbar. Choose the selected Episodes Report and select the Print button.

NOTE: An “in-progress” episode will not be saved; an episode must be complete before it will be saved by the application.

To view episode details, select the Details button next to the desired episode on the Arrhythmia Logbook screen. The Stored Event screen will appear, and you can browse between the Summary, EGM, and Intervals tabs.

Events Summary

The Events Summary screen displays additional details about the selected episode corresponding to the Arrhythmia Logbook.

The summary data may include the following:

- Episode number, date, time, type (e.g., VF, VT, VT-1, spontaneous/induced, or PTM)
- Average atrial and ventricular rates
- Type of therapy delivered
- For ATP therapy, the time of therapy delivery and the number of bursts
- For shock therapy, the start time of charging, charge time, impedance, and energy level
- Duration
- Atrial rate at PMT start (PMT events only)

Stored Electrograms with Annotated Markers

The pulse generator can store annotated electrograms sensed from the following leads prior to the onset of an episode, around duration met, and around therapy start and end:

- Shock lead
- RV pace/sense lead
- Atrial pace/sense lead

The particular annotated electrograms stored depend upon the episode type. In this section, EGM refers to both electrograms and the associated annotated markers. The EGM storage capacity varies depending on EGM signal condition and heart rate. The total amount of stored EGM data associated with an episode may be limited; EGMs from the middle of the episode may be removed for episodes greater than 4 minutes in duration.

When the memory allocated to EGM storage is full, the device overwrites older EGM data segments in order to store the new EGM data. The EGM is recorded in segments consisting of episode Onset, Attempt, and End EGM storage. Each segment of data is visible when the left caliper is in the specific section.

The following information is retained:

- Onset retains up to 25 seconds of data prior to Duration expiring
- Reconfirmation retains up to 20 seconds of data prior to therapy delivery
- Therapy data is displayed. In the case of ATP therapy, a maximum of 4 bursts and up to 20 seconds of data for each burst will be retained
- Post-therapy or diverted therapy retains up to 10 seconds of data

Episode Onset refers to the period of time (measured in seconds) of EGM prior to the first attempt.

Onset includes the following information:

- Type of event
- Average RA Rate at the start of Event
- Average RV Rate at the start of Event
- Programming of Detection Enhancements (Rate Only, Rhythm ID, or Onset/Stability)

Attempt information may be displayed as Attempt or In Progress, if an attempt is in progress. Attempt includes the following information:

- Detection information:
 - Average RA Rate at start of Attempt
 - Average RV Rate at start of Attempt
 - Rate Zone
- Measured Values of Detection Enhancements
- Therapy Attempt Information:
 - Attempt Number
 - Type (aborted, commanded, or inhibited)
 - Number of bursts (ATP attempt)
 - Charge time (shock attempt)
 - Lead impedance (shock attempt)
 - Lead polarity (shock attempt)
 - Shock faults (shock attempt)
 - Reason for No Therapy

The End EGM storage starts following therapy delivery and stores up to 10 seconds of EGM (Figure 6-2 on page 6-6).

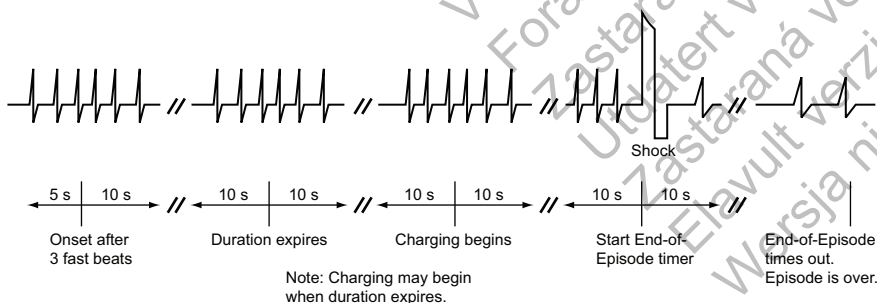


Figure 6-2. Relationship between ventricular tachy episode EGM storage and surface ECG strip chart recording

NOTE: Refer to "Use of Atrial Information" on page 2-4 for additional information about device performance when the atrial lead is programmed to Off.

To view the EGM data, select the Details button of the desired episode on the Arrhythmia Logbook screen.

Use the following steps to view specific details about each episode:

1. Select the EGM tab.

- EGM strips for the appropriate sources are displayed. Each strip includes the EGMs sensed during the episode with the corresponding annotated markers. Blue vertical bars indicate the segment (Onset, Attempt, End) boundaries.

NOTE: For marker definitions, select the Reports button on the PRM and view the Marker Legend Report.

- Use the slider under the upper display window to view different sections of the stored EGM.
- Improve visibility of EGM traces by adjusting the amplitude of each EGM. The adjustment controls and amplitude gain values are located on the right side of the trace display.
- Move the calipers along the trace to display the time interval between the calipers.
- As the left caliper is moved, the stored EGM voltage amplitude at the location of the caliper for each EGM is displayed along the left edge of the trace display. The EGM voltage amplitude is correct regardless of the amplitude gain value.
- A speed button changes the trace speed in millimeters/seconds.

2. Select the Previous Event or Next Event button to display a different event strip.

3. To print the entire episode report, select the Print Event button. To save the entire episode report, select the Save button.

Intervals

The pulse generator stores event markers and associated time stamps. The PRM derives event intervals from the event markers and time stamps.

To view the episode intervals, use the following steps:

1. From the Stored Event screen, select the Intervals tab. If all of the episode data is not visible in the window, use the scroll bar to view more data.
2. Select the Previous Event or the Next Event button to display a previous or more current episode, one episode at a time.
3. Select the Print Event button to print the entire episode report.
4. Select the Save button to save the entire episode report.

HISTOGRAMS

The Histograms feature retrieves information from the pulse generator and displays the total number and percentage of paced and sensed events for the chamber.

Histograms data can provide the following clinical information:

- The distribution of the patient's heart rates
- How the ratio of paced to sensed beats varies by rate

- How the ventricle responds to paced and sensed atrial beats across rates

Use the following steps to access the Histograms screen:

1. From the Events screen, select the Patient Diagnostics tab.
2. The initial display shows the paced and sensed data since the last time the counters were reset.
3. Select the Details button to display the data type and time period.
4. Select the Rate Counts button on the Details screen to view rate counts by chamber.

All Histograms and Counters can be reset by selecting the Reset button from any Patient Diagnostics Details screen.

COUNTERS

The following counters are recorded by the pulse generator and displayed on the Patient Diagnostics screen:

- Ventricular Tachy
- Brady

Ventricular Tachy Counters

Information about Ventricular Tachy Counters is available by selecting the Ventricular Tachy Counters Details button. This screen displays both Ventricular Tachy Episode and Therapy counters. For each counter, the number of events since last reset and device totals are displayed. Ventricular Tachy Episode counters contains the following data:

- Total episodes
- Treated—VF, VT, VT-1, and Commanded
- Nontreated—No Therapy Programmed, Nonsustained, and Other Untreated Episodes

Ventricular Tachy Therapy counters consist of ventricular shock and ATP therapy attempts. They can provide useful data about the effectiveness of a patient's therapy prescription. These counters include the following information:

- ATP Delivered
- ATP % Successful—the percent of time that the arrhythmia is converted and the episode ends without delivery of a programmed shock
- Shocks Delivered
- First Shock % Successful—the percent of time that the arrhythmia is converted and the episode ends without requiring a second programmed shock
- Shocks Diverted

The ventricular ATP counter is incremented at the start of the delivery of the first burst of an ATP scheme. Subsequent ATP bursts in the same scheme are not counted individually during the same episode.

An ATP scheme is counted as diverted only if it is diverted prior to delivery of the first burst.

Brady Counters

Information about Brady Counters is displayed by selecting the Brady Counters button. This screen displays the brady episode counters. For each counter, the number of events since last reset and reset before last are displayed. Brady counters contains the following details:

- Percent of atrial paced
- Percent of RV paced
- Intrinsic Promotion—includes Rate Hysteresis % successful and AV Search+ % successful
- Atrial burden—includes Percentage of time the device was in ATR, Episodes by Duration and Total PACs

NOTE: *Atrial Burden % records and displays data for a maximum of one year.*

- Ventricular counters—includes total PVCs and Three or More PVCs

All Histograms and Counters can be reset by selecting the Reset button from any Patient Diagnostic Details screen.

TRENDS

Trends provide a graphical view of specific patient, device, and lead data. This data can be useful when evaluating your patient's condition and the effectiveness of programmed parameters. Unless otherwise noted below, data for all trends is reported every 24 hours and is available for up to 1 year. For many trends, a value of "N/R" is reported if there is insufficient or invalid data for the collection period.

The following trends are available:

- Events—displays both atrial and ventricular events stored in the Arrhythmia Logbook, organized by date and type ("Arrhythmia Logbook" on page 6-2). This trend is updated whenever an episode is completed, and may contain data that is older than 1 year.
- Activity Level—displays a measure of the patient's daily activity.
- Atrial Burden—displays a trend of the total number of ATR Mode Switch events and the total amount of time spent in an ATR Mode Switch per day.
- Respiratory Rate—displays a trend of the patient's daily minimum, maximum, and median respiratory rate values ("Respiratory Rate Trend" on page 6-10).
- Heart Rate—displays a trend of the patient's daily maximum, mean, and minimum heart rate. Intervals used in this calculation must be valid sinus rhythm intervals.

The validity of an interval and the Heart Rate Trend data for the 24-hour collection period is determined by the Heart Rate Trend collection criteria.

- Lead impedance and amplitude—displays trends of the daily intrinsic amplitude and lead impedance measurements ("Leads Status" on page 5-7).

Follow the steps below to access Trends:

1. From the Events screen, select the Trends Tab.
2. Choose the Select Trends button to specify the trends you want to view. You can choose from the following categories:
 - Atrial Arrhythmia—includes Events, Heart Rate, and Atrial Burden trends
 - Activity—includes Heart Rate, Activity Level, and Respiratory Rate trends
 - Custom—allows you to select various trends to customize the information displayed on the Trends screen

The display on the screen can be viewed in the following manner:

- Select the desired time on the View button to choose the length of visible trend data.
- Adjust the start and end dates by moving the horizontal slider at the top of the window. You can also adjust these dates using the scroll left and scroll right icons.
- Move the vertical axis across the graph by moving the horizontal slider at the bottom of the display window.

Heart Rate Trend Collection Criteria

Only valid sinus rhythm intervals are used in the Heart Rate Trend data calculations. Valid intervals are those which include only valid Heart Rate Trend events.

Valid Heart Rate Trend events are listed below:

- AS with an interval not faster than MTR, followed by a VS
- AS followed by VP at the programmed AV Delay

Invalid Heart Rate Trend events are listed below:

- AP/VS or AP/VP
- AS with an interval faster than MTR
- Non-tracked VP events
- Consecutive AS events (no intervening V event)
- VP-Ns
- Rate Smoothing events (e.g., RVP↑)
- PVC

Heart Rate Trend data may not be reported for a variety of reasons; the most common are listed below:

- Less than 67% of the 24-hour collection period (approximately 16 hours) contains valid Heart Rate Trend events
- Brady parameters were programmed within the last 24 hours

Respiratory Rate Trend

The Respiratory Rate trend displays a graph of the patient's daily minimum, maximum, and median respiratory rate values. These daily values are stored for up to one year to create a longitudinal display of physiological data.

NOTE: *The American College of Cardiology (ACC)/American Heart Association (AHA) guidelines recommend the measurement and documentation of physiological vital signs including respiratory rate for cardiac patients¹.*

The Respiratory Sensor must be programmed to On for Respiratory Rate trend data to be collected and displayed ("Respiratory Sensor" on page 6-11).

Move the horizontal slider over a data point to view the values for a given date. At least 16 hours of data must be collected for values to be calculated and plotted to the Respiratory Rate trend. If insufficient data was collected, no data point will be plotted and there will be a gap in the trend line. This gap will be labeled as N/R to indicate that insufficient or no data was collected.

Respiratory Sensor

The Respiratory Sensor uses transthoracic impedance measurements to collect respiration-related data for use in generating the Respiratory Rate trend.

CAUTION: During mechanical ventilation, respiration-based trending may be misleading; therefore, the Respiratory Sensor should be programmed to Off.

Approximately every 50 ms (20 Hz), the pulse generator drives a current excitation waveform between the RV Coil electrode and Can. The application of the current between these electrodes creates an electrical field (modulated by respiration) across the thorax. During inspiration, the transthoracic impedance is high, and during expiration it is low. The pulse generator will detect the resulting voltage modulations between the RV Tip electrode and Can. Due to advanced filtering, breathing rates up to 65 breaths per minute are supported.

CAUTION: Any medical equipment, treatment, therapy, or diagnostic test that introduces electrical current into the patient has the potential to interfere with pulse generator function.

- External patient monitors (e.g., respiratory monitors, surface ECG monitors, hemodynamic monitors) may interfere with the pulse generator's impedance-based diagnostics (e.g., shock lead impedance measurements). To resolve suspected interactions, deactivate the Respiratory Sensor by programming it to Off.

NOTE: *The Respiratory Sensor signal does not cause an increase in heart rate.*

Consider the following when programming the sensor:

- Examine real-time EGMs before and after activating the sensor. The sensor signal can sometimes be observed on EGMs. If new artifacts are observed, and the leads are otherwise shown to be performing appropriately, consider programming the sensor to Off to prevent oversensing.
- Program the sensor to Off if you detect or suspect any loss of lead integrity.

CAUTION: Do not program the Respiratory Sensor to On until after the pulse generator has been implanted and system integrity has been tested and verified.

1. ACC/AHA Heart Failure Clinical Data Standards. *Circulation*, Vol. 112 (12), September 20, 2005.

The pulse generator may temporarily suspend the sensor in the following circumstances:

- Detection of 3 fast intervals on the RV sense channel (when Tachy Mode is set to Monitor Only or Monitor + Therapy)—The pulse generator immediately suspends the sensor. It is turned on again after 1 hour unless there is a ventricular tachy detection in progress, in which case the device waits another hour to reevaluate.

CAUTION: The Respiratory Sensor will not be suspended due to 3 fast intervals if the Tachy Mode is set to Off. Consider turning the Respiratory Sensor Off when Tachy Mode is Off to prevent potential oversensing and pauses in pacing.

NOTE: *If 3 fast ventricular beats are detected as a result of oversensing of the sensor signal, the patient may experience a short pause in pacing up to approximately twice the programmed LRL.*

- Excessive electrical noise levels—The pulse generator continuously monitors electrical noise levels. The sensor is temporarily suspended if noise is excessive, and is turned on again when noise decreases to an acceptable level.
- Loss of lead integrity—Lead impedances for the sensor are evaluated every 24 hours (separate from daily lead measurements). If either impedance measurement is out of range, the following occurs:
 - The pulse generator evaluates the lead impedance for a secondary vector driven from the RA Ring electrode to the Can, and measured from the RA Tip electrode to the Can. If this impedance measurement is in range, the sensor reverts to this secondary vector. If the lead impedance is also out of range with the secondary vector, the sensor is temporarily suspended until the next lead impedance evaluation in 24 hours.

NOTE: *If an RA lead is not used, the secondary vector is not available.*

- The pulse generator will continue to monitor lead impedance every 24 hours to determine if the sensor should be returned to the primary or secondary vector, or remain suspended. Acceptable lead impedance values are 200–2000 Ω for the tip to can and ring to can vectors and 20–200 Ω for the RV Coil to can vector.

To program the Respiratory Sensor, use the following steps:

1. From the Settings tab on the main screen, select Settings Summary.
2. From the Settings Summary tab, select the Normal Settings button.
3. From the Normal Settings screen, select the Accelerometer button.
4. On the Accelerometer screen, select the desired option for Respiratory Sensor.

POST IMPLANT FEATURES

Patient Triggered Monitor

Patient Triggered Monitor allows the patient to trigger the storage of EGMs, intervals, and annotated marker data during a symptomatic episode by placing a magnet over the device. Instruct the patient to place the magnet on the device briefly and one time only.

Patient Triggered Monitor is enabled by selecting Store EGM as the desired Magnet Response. This can be found in the Magnet and Beeper section on the V-Tachy Therapy Setup screen. When enabled, the device will store up to 2 minutes of patient monitor data prior to and up to 1

minute after triggering the monitoring. The stored data include the episode number, the rates at magnet application, and the start time and date of magnet application.

Once PTM is programmed on, only one EGM can be generated and stored. To store another EGM, the PTM feature must be re-enabled using the programmer.

When data are stored, the corresponding episode type is recorded as PTM in the Arrhythmia Logbook.

Use care when enabling Patient Triggered Monitor, because the following conditions will exist:

- All other magnet features are disabled, including inhibiting therapy (until the EGM is stored). The Magnet/Beeper feature will not indicate magnet position.
- Device longevity is impacted. Once the patient has triggered this feature to store episode data or the feature is disabled, the impact on device longevity is no longer present. To help reduce the longevity effect, this feature is automatically disabled after 60 days from the day it was enabled.
- Once the EGM is stored, the device Magnet Response automatically will be set to Inhibit Therapy. However, the pulse generator will not inhibit therapy until the magnet is removed for 3 seconds and placed on the device again.

To program the Patient Triggered Monitor feature, follow these steps:

1. From the Settings tab on the main screen, select Settings Summary.
2. From the Settings Summary tab, select Ventricular Tachy Therapy.
3. From Ventricular Tachy Therapy, select the V-Tachy Therapy Setup details button.
4. Program the Magnet Response to Store EGM.

CAUTION: Determine if the patient is capable of activating this feature prior to being given the magnet and prior to enabling Patient Triggered Monitor. Remind the patient to avoid strong magnetic fields so the feature is not inadvertently triggered.

CAUTION: Consider having the patient initiate a stored EGM at the time Patient Triggered Monitor is enabled to assist with patient education and feature validation. Verify the activation of the feature on the Arrhythmia Logbook screen.

WARNING: Ensure that Patient Triggered Monitor is enabled prior to sending the patient home by confirming the Magnet Response is programmed to Store EGM. If the feature is inadvertently left in the Inhibit Therapy setting, the patient could potentially disable tachyarrhythmia detection and therapy.

WARNING: Once the Patient Triggered Monitor feature has been triggered by the magnet and an EGM has been stored, or after 60 days have elapsed from the day that Store EGM was enabled, the Magnet Response programming automatically will be set to Inhibit Therapy. When this happens, the patient should not apply the magnet because tachyarrhythmia therapy could be inhibited.

NOTE: *When the Magnet Response programming has automatically been set to Inhibit Therapy, magnet application will cause the device to emit beeping tones. Inform the patient that if they hear tones coming from their device after applying the magnet, they should remove the magnet.*

5. Patient Triggered Monitor can only be enabled for a 60-day period of time. To disable the feature within the 60-day time period, reprogram the Magnet Response to a setting other

than Store EGM. When 60 days have passed since enabling Patient Triggered Monitor, the feature will automatically disable itself and the Magnet Response will revert to Inhibit Therapy. To re-enable the feature, repeat these steps.

For additional information, contact Boston Scientific using the information on the back cover.

Beeper Feature

The pulse generator contains a beeper that emits audible tones to communicate status information. The beeper includes both programmable and nonprogrammable features.

Programmable Features

The following beeper features are programmable:

- **Beep During Capacitor Charge**—When programmed to On, regardless of the Tachy mode, a warbling tone will sound continuously while the pulse generator is charging (except when charging during an auto capacitor re-form). The tone will continue until charging is complete. When this feature is programmed to Off, there is no audible indication that the pulse generator is charging. This feature is useful during EP testing.
- **Beep When Explant Is Indicated**—When this feature is programmed to On, the pulse generator emits tones upon reaching Explant. The Explant indicator consists of 16 tones repeated every six hours after the pulse generator reaches Explant until the feature is turned off via the programmer. When this feature is programmed to Off, there is no audible indication of Explant.
- **Beep When Out-of-Range**—When this feature is programmed to On, the pulse generator emits tones when Daily Impedance values are out of range. It is separately programmable for each pacing lead impedance as well as shock impedance. The Out-of-Range indicator consists of 16 tones repeated every six hours. When this feature is programmed to Off, there is no audible indication of Daily Impedance values.

Perform the following steps to program the magnet and beeper features:

Magnet and Beeper Response

1. Select the Settings tab.
2. From Ventricular Tachy, select the Therapy button.
3. Select the V-Tachy Therapy Setup button.
4. Enter the desired values.

Beep when Explant is indicated

1. Select the Summary tab.
2. Select the Battery button.
3. From the Battery Status summary screen, select the Battery Detail button.
4. From the Battery Detail summary screen, select the desired value for Beep when Explant is indicated.

Beep When Out-of-Range

1. Select the Summary tab.
2. Select the Leads button.
3. From the Leads Status summary screen, select the Setup tab.
4. Select the desired values for Beep When Out-of-Range.

NOTE: When the Magnet Response is programmed to Inhibit Therapy, magnet application will cause other types of beeping tones to be emitted, depending on the device mode. Refer to "Magnet Feature" on page 6-15 for more information.

Nonprogrammable Features

The following beeper features are nonprogrammable:

- Battery capacity depleted—Regardless of whether Beep When Explant Is Indicated is programmed to On or Off, once the battery capacity is depleted, the pulse generator will emit the explant indicator tones
- Fault code tones—For certain fault codes or when safety mode is entered, the pulse generator will beep 16 times every 6 hours

NOTE: Beeping tones may emit under nonprogrammable scenarios in response to device self-diagnostic testing. Advise patients to have their pulse generator checked whenever they hear tones coming from the device.

Contact Boston Scientific using the information on the back cover.

Magnet Feature

The magnet feature allows certain device functions to be triggered when a magnet is placed in close proximity to the pulse generator (Figure 6-3 on page 6-15).

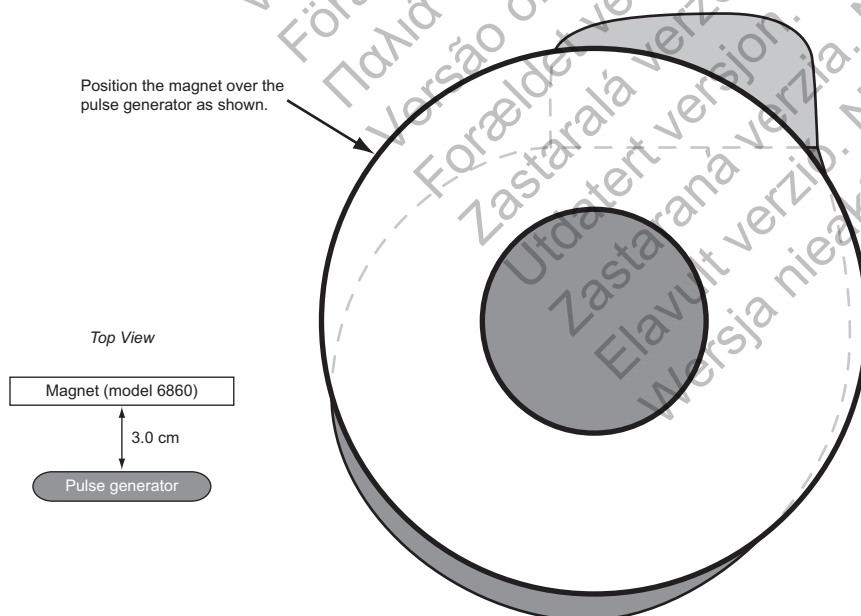


Figure 6-3. Proper position of magnet Model 6860 to activate the pulse generator magnet feature

The pulse generator Magnet Response settings can be programmed to control the behavior of the pulse generator when a magnet is detected. The Magnet Response settings are located in the Magnet and Beeper section of the V-Tachy Therapy Setup screen.

The following Magnet Response settings are available:

- Off—no response
- Store EGM—patient monitoring data will be stored
- Inhibit Therapy—therapy will be stopped

Off

When the Magnet Response is programmed to Off, application of the magnet will have no effect on the pulse generator.

Store EGM

When the Magnet Response is programmed to Store EGM, application of the magnet will activate the patient triggered monitor functionality ("Patient Triggered Monitor" on page 6-12).

Inhibit Therapy

When the Magnet Response is programmed to Inhibit Therapy, application of the magnet will inhibit and/or divert charging for a shock, divert a shock that is about to be delivered, or inhibit and/or divert further ATP therapy.

When Magnet Response is programmed to Inhibit Therapy, initiation of tachyarrhythmia therapy and arrhythmia induction is inhibited any time the magnet is properly positioned over the pulse generator. The tachyarrhythmia detection process continues, but therapy or induction cannot be triggered. When a magnet is placed over the pulse generator, the following will occur:

- If the Tachy mode is Monitor + Therapy or Off when the magnet is applied, the Tachy mode changes temporarily to Monitor Only mode and will remain in Monitor Only mode as long as the magnet is applied. Three seconds after the magnet is removed, the mode will return to the previously programmed mode.
- If the pulse generator is charging to deliver shock therapy when the magnet is applied, the charging continues but is then terminated within one to two seconds of magnet application, and the charge is diverted. (This delay occurs in case the magnet is inadvertently passed over the device when therapy inhibition is not desired.) The pulse generator remains in temporary Monitor Only mode while the magnet is applied. No further therapy is initiated until the magnet is removed; however, detection will continue.
- If charging is complete or completes within the 1–2 second delay period, holding the magnet over the pulse generator for more than two seconds will divert the shock. (If the magnet is removed during the delay period, the shock could still be delivered.) Shocks will not be delivered with the magnet in place.
- If the pulse generator is initiating fibrillation induction or ATP pulses, it terminates the delivery after one to two seconds of magnet application. No further induction or ATP pulse sequences are initiated until the magnet is removed.
- If the Tachy Mode is Monitor Only or Off, magnet application will produce a constant tone to indicate that the device is in a non-therapy mode.
- If the Tachy Mode is Monitor + Therapy, magnet application will cause the pulse generator to beep once per second to indicate that the device is in a therapy mode.

- If the pulse generator is in Electrocautery Protection Mode, magnet application will produce beeping consistent with whichever Tachy Mode was active when the pulse generator was placed into Electrocautery Protection Mode. For example, if Electrocautery Protection Mode was enabled when the Tachy Mode was set to Monitor + Therapy, magnet application will cause the pulse generator to beep once per second.

NOTE: *If tachy detection occurs while the magnet is in place, detailed therapy history will indicate that therapy was not delivered because the device was in Monitor Only mode.*

Outdated version. Do not use.
 Version überholt. Nicht verwenden.
 Version obsolète. Ne pas utiliser.
 Versión obsoleta. No utilizar.
 Versione obsoleta. Non utilizzate.
 Verouderde versie. Niet gebruiken.
 Föråldrad version. Använd ej.
 Παλιά έκδοση. Μην χρησιμοποιείτε.
 Versão obsoleta. Não utilize.
 Forældet version. Må ikke anvendes.
 Zastaralá verzia. Nepoužívať.
 Utdatert versjon. Skal ikke brukes.
 Zastaraná verzia. Nepoužívať.
 Elavult verzió. Ne használja!
 Wersja nieaktualna. Nie używać.

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verze. Nepoužívat.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

ELECTROPHYSIOLOGIC TESTING

CHAPTER 7

This chapter contains the following topics:

- "EP Test Features" on page 7-2
- "Induction Methods" on page 7-4
- "Commanded Therapy Methods" on page 7-8

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

EP TEST FEATURES

Electrophysiologic (EP) Testing features enable you to induce and terminate arrhythmias noninvasively in order to monitor and test the effectiveness of selected detection criteria and therapies. The EP Test features can be used in conjunction with the ECG display so that real-time traces may be viewed. The status of the pulse generator/patient interaction is also displayed.

WARNING: Always have external defibrillation protection available during implant and electrophysiologic testing. If not terminated in a timely fashion, an induced ventricular tachyarrhythmia can result in the patient's death.

The features allowing noninvasive EP testing of arrhythmias include the following:

- VFib induction
- Shock on T induction
- Programmed electrical stimulation (PES) induction/termination
- 50 Hz/Manual Burst pacing induction/termination
- Commanded Shock therapy
- Commanded ATP therapy

Temporary EP Mode

Temporary EP Mode allows you to program the device mode to a temporary value for EP test delivery, and ensures that the normal device mode remains unchanged.

EP Test Screen

The EP Test screen displays the real-time status of the detection and therapy process of the pulse generator when telemetry communication is occurring. Viewing this display allows you to induce and test either a programmed detection/therapy prescription or optional therapies while monitoring the pulse generator's progress.

Refer to the EP Test screen (Figure 7-1 on page 7-2):

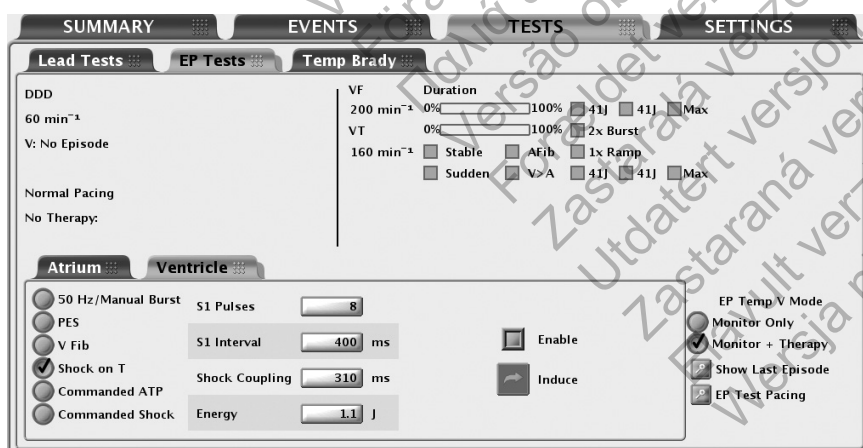


Figure 7-1. EP Test Screen

The screen provides the following information:

- Status messages indicate detection and therapy status and are described below:
 - Ventricular episode status—if an episode is occurring, the duration of the episode is displayed. (If it is greater than 10 minutes, then it is displayed as > 10:00 m:s).
 - Ventricular detection status—if an episode is occurring, it indicates whether ventricular detection is in Initial Detection, Redetection, or the zone in which that detection is met. If no episode is occurring, the programmer will also display the time (in minutes) since the last ventricular therapy (up to 10 minutes).
 - Brady pacing and SRD status.
 - The type of therapy initiated and the zone.
 - The status of the therapy such as In progress, Diverted, or Delivered.
- Duration timer—Progression of the duration timer is graphically displayed using a scale. The bar in the scale moves from left to right to show the percent complete of programmed duration. When duration is expired and therapy delivery begins, the bar is removed.
- Detection status—The status for each programmed detection enhancement is displayed. When enhancement criteria are met, a mark appears in the adjacent box.
- Therapy prescriptions—Only those therapy prescriptions that are programmed are displayed. As each therapy is delivered, a check mark or number will appear in the box adjacent to the respective therapy. ATP therapies indicate the scheme type as well as the programmed number of bursts in the scheme. A number will appear and increment (1, 2, etc.) in the ATP therapy box each time an ATP burst is delivered. Shock therapies indicate the programmed energy level for the programmable shocks. A number will appear and increment (1, 2, etc.) in the Max box each time a maximum-energy shock is delivered.

Follow the steps below to perform EP Test functions:

1. Select the Tests tab, then select the EP Tests tab.
2. Establish telemetry communication. Telemetry communication between the programmer and the pulse generator should be maintained throughout all EP test procedures.
3. Set backup pacing and EP Test Pacing outputs as desired.

NOTE: Backup pacing during EP testing is not available in single-chamber devices.

4. Program the EP Temp V mode appropriate to the EP Test method (Table 7-1 on page 7-3).

Table 7-1. EP Temp V Mode for EP Test Functions

EP Test Method ^a	EP Temp V Mode		
	Monitor + Therapy ^d	Monitor Only ^e	Off
50 Hz/Manual Burst ^b	X		
PES ^b	X		
VFib ^c	X		
Shock on T ^c	X		

Table 7-1. EP Temp V Mode for EP Test Functions (continued)

EP Test Method ^a	EP Temp V Mode		
	Monitor + Therapy ^d	Monitor Only ^e	Off
Commanded ATP ^c		X	
Commanded Shock ^c	X	X	

- a. EP functions cannot be performed if the pulse generator is in Storage Mode.
- b. Available method for both atrial and ventricular induction.
- c. Available method only for ventricular induction.
- d. The Ventricular Tachy Mode must be programmed to Monitor + Therapy.
- e. The Ventricular Tachy Mode must be programmed to Monitor Only or Monitor + Therapy.

INDUCTION METHODS

Each EP test method available from the EP Test screen is described below with instructions. During any type of induction/termination, the pulse generator performs no other activity until the test has ceased, at which time the programmed mode will take effect and the pulse generator will respond accordingly.

Consider the following information when using these methods:

- All inductions and tachycardia therapy delivery are inhibited when a magnet is positioned over the pulse generator (if magnet response is set to Inhibit Therapy)
- Pacing pulses during induction are delivered at the programmed EP Test pacing parameters

VFib Induction

VFib induction uses the shocking electrodes to stimulate the right ventricle at very fast rates.

The following settings are available to allow use of the minimum energy necessary for induction:

- VFib Low delivers a stimulation waveform of 9 volts
- VFib High delivers a stimulation waveform of 15 volts

Performing VFib Induction

NOTE: The patient should be sedated prior to delivery of fibrillation induction pulses. The large surface area of the shocking electrodes tends to stimulate the surrounding muscle and can be uncomfortable.

1. Select the VFib option. Buttons for each test and an Enable checkbox are displayed.
2. Select the Enable checkbox.
3. Select the desired Hold for Fib button to initiate delivery of the fibrillation induction train. The induction train is delivered up to 15 seconds as long as the button is held and the telemetry link is maintained.

During induction the pulse generator is automatically disabled from detecting, and automatically re-enabled following induction delivery. If VFib induction is initiated during an episode, the end-of-episode is declared before the VFib induction pulses are started. A new episode (with initial detection and therapy) can be declared after the VFib induction is completed. Event markers and EGMs are interrupted during VFib induction and will automatically restart following induction.

4. To stop the induction train, release the button (the button will become dimmed again).
5. To deliver another fibrillation induction, repeat these steps.

Shock on T Induction

A Shock on T wave induction method allows the device to deliver a drive train (up to 30 equally timed pacing pulses, or S1 pulses) through the ventricular pace/sense electrodes followed by shock delivery through the shocking electrodes (Figure 7-2 on page 7-5).

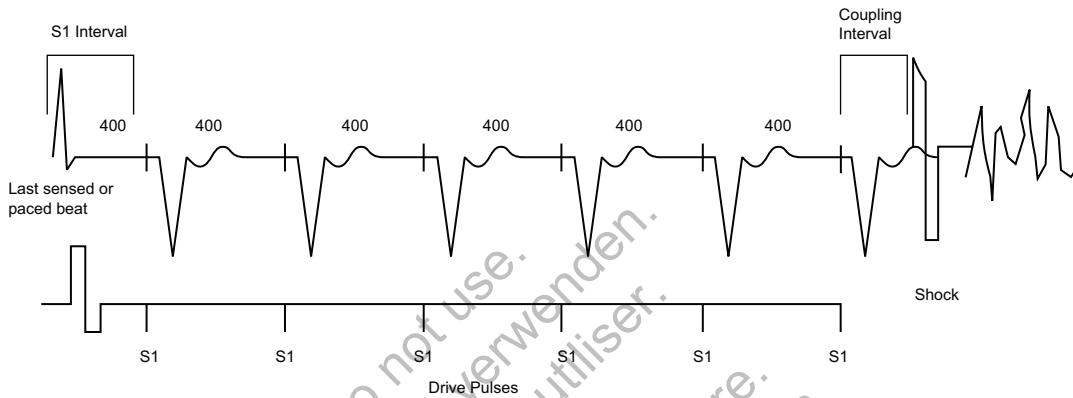


Figure 7-2. Shock on T induction drive train

The initial S1 pulse follows the last sensed or paced event at the S1 interval. The shock is coupled to the last S1 pulse of the drive train.

Performing Shock on T Induction

1. Select the Shock on T option. The programmable induction parameters will be displayed.
2. Select the desired value for each parameter.
3. Select the Enable checkbox. The Induce button will no longer be dimmed.
4. Select the Induce button to begin delivery of the drive train. The pulses are delivered in sequence until the programmed number of pulses is reached. Once induction is initiated, the drive train delivery will not stop if you interrupt telemetry communication. While telemetry is active, pressing the DIVERT THERAPY key will stop induction delivery.
5. Shock on T induction is complete when the drive train and shock are delivered, at which time the pulse generator automatically restarts detection.

NOTE: Prior to drive train delivery, tones will be heard indicating capacitor charging in preparation for shock delivery.

NOTE: The shock delivered during Shock on T induction does not increment episode or therapy counters.

Backup Ventricular Pacing During Atrial EP Testing

Backup ventricular pacing is available during atrial EP testing (PES, 50 Hz/Manual Burst) regardless of the programmed Normal or Post-therapy pacing modes.

NOTE: Backup pacing is performed in VOO mode.

NOTE: Backup pacing during EP testing is not available in single-chamber devices.

In dual-chamber devices, program the backup pacing parameters by selecting the EP Test Pacing button. Backup pacing parameters are independently programmable from the permanent

pacing parameters. Backup pacing can also be disabled by programming the Backup Pacing Mode to Off.

Programmed Electrical Stimulation (PES)

PES induction allows the pulse generator to deliver up to 30 equally timed pacing pulses (S1) followed by up to 4 premature stimuli (S2–S5) to induce or terminate arrhythmias. Drive pulses, or S1 pulses, are intended to capture and drive the heart at a rate slightly faster than the intrinsic rate. This ensures that the timing of the premature extra stimuli will be accurately coupled with the cardiac cycle (Figure 7-3 on page 7-6).

The initial S1 pulse is coupled to the last sensed or paced beat at the S1 interval. All pulses are delivered in XOO modes (where X is the chamber) at the programmed EP Test pacing parameters.

For Atrial PES, backup pacing parameters are provided.

NOTE: Backup pacing during EP testing is not available in single-chamber devices.

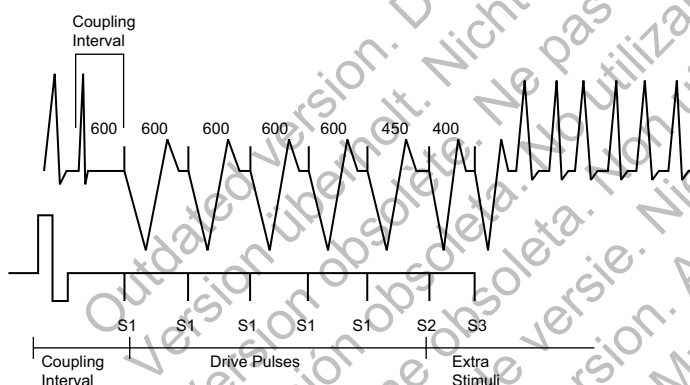


Figure 7-3. PES induction drive train

Performing PES Induction

1. In a dual-chamber device, choose the Atrium or Ventricle tab, depending on which chamber you want to pace.
2. Select the PES option. Buttons for the S1–S5 pulses and the corresponding burst cycle lengths are displayed.
3. Select the desired value for the S1–S5 intervals (Figure 7-4 on page 7-6). You can either select a value box for the desired S interval and choose a value from the box or use the plus or minus symbols to change the value visible in the value box.

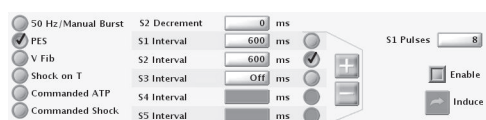


Figure 7-4. PES induction options

4. Select the Enable checkbox.
5. Select (do not hold) the Induce button to begin delivery of the drive train. When the programmed number of S1 pulses is delivered, the pulse generator will then deliver

the programmed S2–S5 pulses. The pulses are delivered in sequence until a pulse is encountered that is set to Off (e.g., if S1 and S2 are set to 600 ms, and S3 is Off, then S3, S4, and S5 will not be delivered). Once induction is initiated, the PES delivery will not stop if you interrupt telemetry communication. (While telemetry is active, pressing the DIVERT THERAPY key will stop induction delivery.)

6. PES induction is complete when the drive train and extra stimuli are delivered, at which time the pulse generator automatically restarts detection.

NOTE: Ensure the PES induction is complete before beginning another induction.

NOTE: When PES is used to terminate an arrhythmia that has been detected (and an episode declared), the episode is terminated when the PES is commanded regardless of whether it is successful or not. A new episode can be declared after the PES induction is completed. The PES itself is not recorded in therapy history; this may result in several episodes being counted in therapy history.

NOTE: Real-time EGMs and annotated event markers will continue to be displayed during the entire test sequence.

50 Hz/Manual Burst Pacing

50 Hz pacing and Manual Burst pacing are both used to induce or terminate arrhythmias when delivered to the desired chamber. Pacing parameters are programmable for Manual Burst but are fixed for 50 Hz pacing.

Manual Burst and 50 Hz pacing pulses are delivered in XOO mode (where X is the chamber) at the programmed EP Test pacing parameters. For Atrial Manual Burst and 50 Hz, backup pacing parameters are provided.

NOTE: Backup pacing during EP testing is not available in single-chamber devices.

Performing Manual Burst Pacing

1. In a dual-chamber device, choose the Atrium or Ventricle tab, depending on which chamber you want to pace.
2. Select the 50 Hz/Manual Burst option.
3. Select the desired value for the Burst Interval, Minimum, and Decrement. This indicates the cycle length of the intervals in the drive train.
4. Select the Enable checkbox.
5. To deliver the burst, select and hold the Hold for Burst button.

The ventricular Manual Burst will be delivered up to 30 seconds as long as the Hold for Burst button is held and the telemetry link is maintained.

The atrial Manual Burst will be delivered up to 45 seconds as long as the Hold for Burst button is held and the telemetry link is maintained.

The intervals will continue to be decremented until the Minimum interval is reached, then all further pulses will be at the Minimum interval.

6. To stop the burst delivery, release the Hold for Burst button. The Hold for Burst button will become dimmed again.

7. To deliver additional Manual Burst pacing, repeat these steps.

Performing 50 Hz Burst Pacing

1. In a dual-chamber device, choose the Atrium or Ventricle tab, depending on which chamber you want to pace.
2. Select the 50 Hz/Manual Burst option.
3. Select the Enable checkbox.
4. To deliver the burst, select and hold the Hold for 50 Hz Burst button.

The ventricular 50 Hz Burst will be delivered up to 30 seconds as long as the Hold for Burst button is held and the telemetry link is maintained.

The atrial 50 Hz Burst will be delivered up to 45 seconds as long as the Hold for Burst button is held and the telemetry link is maintained.

NOTE: During Hold for 50 Hz Burst pacing, the S1 interval is automatically set to 20 ms and the decrement to 0. These values will not be displayed on the screen.

5. To stop the burst delivery, release the Hold for 50 Hz Burst button. The Hold for 50 Hz Burst button will become dimmed again.
6. To deliver additional 50 Hz Burst pacing, repeat these steps.

NOTE: Real-time EGMs and annotated event markers will continue to be displayed during the entire test sequence.

COMMANDED THERAPY METHODS

The commanded EP test methods, Commanded Shock and Commanded ATP, may be delivered independently of the programmed detection and therapy parameters. If the pulse generator is in the process of delivering therapy when a commanded method is initiated, the EP Test function overrides and aborts the therapy in process. If an episode is not in progress, then a Commanded Ventricular Episode will be recorded in the Arrhythmia Logbook. Commanded Shock and Commanded ATP delivery is inhibited when a magnet is positioned over the pulse generator, if it is programmed to Inhibit Therapy.

Commanded Shock

The Commanded Shock feature allows delivery of a shock with programmable energy and coupling interval.

All Commanded Shocks are Committed and delivered R-wave synchronously when the coupling interval is programmed to Sync. Shock waveform and polarity are identical to detection-initiated shocks but a programmed coupling interval may be specified. The coupling interval is initiated at the point where the shock would have been delivered in Sync mode, but is instead delivered at the programmed coupling interval. Following any Commanded Shock delivery, Post-shock Redetection is used and post-shock pacing is activated.

Performing Commanded Shock Delivery

1. Select the Commanded Shock option.
2. Select the desired values for the Coupling interval and Shock Energy.

3. Select the Enable checkbox. The Deliver Shock button will become available.
4. Select the Deliver Shock button to initiate shock delivery. The Commanded Shock is recorded in therapy history.
5. To deliver subsequent shocks, repeat these steps.

Commanded ATP

Commanded ATP allows you to manually deliver ATP schemes, independent of the programmed detection and therapy parameters. You can configure the Commanded ATP by either selecting the type of ATP scheme or by programming ATP parameters on the Details screen in order to deliver Commanded ATP.

The EP Temp V Mode must be programmed to Monitor Only to ensure the Commanded ATP does not interfere with detection-initiated ATP.

Performing Commanded ATP

1. If the pulse generator Ventricular Tachy Mode is not currently programmed to Monitor Only, select the Monitor Only EP Temp V Mode option.
2. Select the type of ATP scheme and select the value for Number of Bursts.
3. Select the Start ATP button to initiate the first burst in the selected ATP scheme. The Bursts Remaining counter will decrement as each burst is completed.
4. Select the Continue button for each additional burst delivery desired. If all bursts in a scheme have been delivered, the Bursts Remaining counter will return to the initial count, and the Continue button will be dimmed.
5. Other ATP schemes may be selected at any time; select the desired scheme and repeat the above sequence. The Commanded ATP is recorded as a physician-commanded therapy counter and displayed on the counters screen.
6. After using Commanded ATP, remember to program the EP Temp V Mode to Monitor + Therapy or leave the screen so that the EP Temp V Mode is ended and the permanent Tachy Mode is resumed.

NOTE: If any button other than the Continue button is selected during delivery of a Commanded ATP scheme, the scheme will be reset and the Bursts Remaining box will be restored to its initial value. The Start ATP button must be reselected to initiate the scheme again.

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verze. Nepoužívat.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

PROGRAMMABLE OPTIONS

APPENDIX A

Table A-1. ZIP Telemetry settings

Parameter	Programmable Values	Nominal ^a
Communication Mode	Enable use of ZIP telemetry (May require limited use of wand), Use wand for all telemetry	Enable use of ZIP telemetry (May require limited use of wand)

a. If the Communication Mode is selected via the Utilities button on the PRM Startup screen, the Nominal setting within the ZOOMVIEW Programmer software application will correspond to the value chosen on the Startup screen.

Table A-2. Tachy Mode parameter

Parameter	Programmable Values	Nominal
Tachy Mode	Off, Monitor Only, Monitor + Therapy, Enable Electrocautery Protection	Storage

Table A-3. Ventricular Zones parameter

Parameter	Programmable Values	Nominal
Ventricular Zones	1, 2, 3	2

Table A-4. Detection parameters for 1-zone, 2-zone, and 3-zone configurations

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
Rate ^a (min ⁻¹) 3 zones (intervals in ms)	90, 95, ..., 200 (667–300)	110, 115, ..., 210 (545–286), 220 (273)	130, 135, ..., 210 (462–286), 220, 230, 240, 250 (273–240)	140 (Tolerance ± 5 ms) for VT-1 Zone 160 (Tolerance ± 5 ms) for VT Zone 200 (Tolerance ± 5 ms) for VF Zone
Rate ^a (min ⁻¹) 2 zones (intervals in ms)	--	90, 95, ..., 210 (667–286), 220 (273)	110, 115, ..., 210 (545–286), 220, 230, 240, 250 (273–240)	160 (Tolerance ± 5 ms) for VT Zone 200 (Tolerance ± 5 ms) for VF Zone
Rate ^a (min ⁻¹) 1 zone (intervals in ms)	--	--	90, 95, ..., 210 (667–286), 220 (273)	200 (Tolerance ± 5 ms)
Initial Duration ^b (sec) 3 zones	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0, 20.0, 25.0, ... 60.0	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0, 20.0, 25.0, 30.0	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0	2.5 (Tolerance ± 1 cardiac cycle) for VT-1 Zone 2.5 (Tolerance ± 1 cardiac cycle) for VT Zone 1.0 (Tolerance ± 1 cardiac cycle) for VF Zone
Initial Duration ^b (sec) 2 zones	--	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0, 20.0, 25.0, 30.0	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0	2.5 (Tolerance ± 1 cardiac cycle) for VT Zone 1.0 (Tolerance ± 1 cardiac cycle) for VF Zone
Initial Duration (sec) 1 zone	--	--	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0	1.0 (Tolerance ± 1 cardiac cycle)
Redetection Duration ^b (sec) 3 zones	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0	1.0 (nonprogrammable)	1.0 (Tolerance ± 1 cardiac cycle) for all zones
Redetection Duration (sec) 2 zones	--	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0	1.0 (nonprogrammable)	1.0 (Tolerance ± 1 cardiac cycle) for all zones
Redetection Duration (sec) 1 zone	--	--	1.0 (nonprogrammable)	1.0 (Tolerance ± 1 cardiac cycle)

Table A-4. Detection parameters for 1-zone, 2-zone, and 3-zone configurations (continued)

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
Post-shock Duration ^b (sec) 3 zones	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0, 20.0, 25.0, ..., 60.0	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0, 20.0, 25.0, 30.0	1.0 (nonprogrammable)	1.0 (Tolerance ± 1 cardiac cycle) for all zones
Post-shock Duration (sec) 2 zones	--	1.0, 1.5, ..., 5.0, 6.0, 7.0, ..., 15.0, 20.0, 25.0, 30.0	1.0 (nonprogrammable)	1.0 (Tolerance ± 1 cardiac cycle) for all zones
Post-shock Duration (sec) 1 zone	--	--	1.0 (nonprogrammable)	1.0 (Tolerance ± 1 cardiac cycle)

- a. The Rate difference between each tachy zone must be at least 20 min⁻¹. The lowest Tachy Rate Threshold must be ≥ 5 min⁻¹ higher than the Maximum Tracking Rate, Maximum Sensor Rate, and the Maximum Pacing Rate; and the lowest Tachy Rate Threshold must be ≥ 15 min⁻¹ higher than the Lower Rate Limit.
- b. The Duration in a zone must be equal to or greater than the Duration in the next highest zone.

Table A-5. Ventricular Detection Enhancement Type for 2-zone and 3-zone configurations

Parameter	Programmable Values	Nominal
Detection Enhancement Type	Off, Rhythm ID, Onset/Stability	Rhythm ID

Table A-6. Onset/Stability detection enhancement parameters for 2-zone and 3-zone configurations

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
V Rate > A Rate 3 zones ^a	Off, On	--	--	On
V Rate > A Rate 2 zones	--	Off, On	--	On
AFib Rate Threshold (min ⁻¹) 3 zones ^{a b}	Off, 100, 110, ..., 300	--	--	170 (Tolerance ± 5 ms)
AFib Rate Threshold (min ⁻¹) 2 zones ^b	--	Off, 100, 110, ..., 300	--	170 (Tolerance ± 5 ms)
Stability (ms) 3 zones ^a	Off, 6, 8, ..., 32, 35, 40, ..., 60, 70, 80, ..., 120	Off, 6, 8, ..., 32, 35, 40, ..., 60, 70, 80, ..., 120	--	20 (DR); 30 (VR) (Tolerance ± 5 ms)
Stability (ms) 2 zones	--	Off, 6, 8, ..., 32, 35, 40, ..., 60, 70, 80, ..., 120	--	20 (DR); 30 (VR) (Tolerance ± 5 ms)
Shock If Unstable (ms) 3 zones	--	Off, 6, 8, ..., 32, 35, 40, ..., 60, 70, 80, ..., 120	--	30 (Tolerance ± 5 ms)
Shock If Unstable (ms) 2 zones	--	Off, 6, 8, ..., 32, 35, 40, ..., 60, 70, 80, ..., 120	--	Off (Tolerance ± 5 ms)
Onset (% or ms) 3 zones ^a	Off, 9, 12, 16, 19, ..., 37, 41, 44, 47, 50% or 50, 60, ..., 250 ms	--	--	9% (Tolerance ± 5 ms)
Onset (% or ms) 2 zones	--	Off, 9, 12, 16, 19, ..., 37, 41, 44, 47, 50% or 50, 60, ..., 250 ms	--	9% (Tolerance ± 5 ms)
Stability And/Or Onset 3 zones ^a	And, Or	--	--	And
Stability And/Or Onset 2 zones	--	And, Or	--	And
Sustained Rate Duration (min:sec) 3 zones ^a	Off, 00:10, 00:15, ..., 00:55 01:00, 01:15, ..., 02:00 02:30, 03:00, ..., 10:00 15:00, 20:00, ..., 60:00	--	--	03:00 (Tolerance ± 1 cardiac cycle)
Sustained Rate Duration (min:sec) 2 zones	--	Off, 00:10, 00:15, ..., 00:55 01:00, 01:15, ..., 02:00 02:30, 03:00, ..., 10:00 15:00, 20:00, ..., 60:00	--	03:00 (Tolerance ± 1 cardiac cycle)
Detection Enhancement 3 zones	Off, On	Off, On	--	On

Table A-6. Onset/Stability detection enhancement parameters for 2-zone and 3-zone configurations (continued)

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
Detection Enhancement 2 zones	--	Off, On	--	On
Atrial Tachyarrhythmia Discrimination 3 zones ^a	Off, On	--	--	On
Atrial Tachyarrhythmia Discrimination 2 zones	--	Off, On	--	On
Sinus Tachycardia Discrimination 3 zones ^a	Off, On	--	--	On
Sinus Tachycardia Discrimination 2 zones	--	Off, On	--	On
Polymorphic VT Discrimination 3 zones	--	Off, On	--	On
Polymorphic VT Discrimination 2 zones	--	Off, On	--	Off

- a. If all VT-1 therapy is programmed to Off, detection enhancements will apply in the VT zone, not the VT-1 zone.
 b. All of the AFib Rate Thresholds are linked to the ATR Trigger Rate and Atrial Flutter Response Rate. If any one of these rates is reprogrammed, the others will automatically change to the same value.

Table A-7. Rhythm ID detection enhancement parameters for 2-zone and 3-zone configurations

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
Detection Enhancement 3 zones	Off, On	Off, On	--	On (VT-1) Off (VT)
Detection Enhancement 2 zones	--	Off, On	--	On
Sustained Rate Duration (min:sec) 3 zones	Off, 00:10, 00:15, ..., 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	Off, 00:10, 00:15, ..., 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	--	03:00 (VT-1 and VT) (Tolerance ± 1 cardiac cycle)
Sustained Rate Duration (min:sec) 2 zones	--	Off, 00:10, 00:15, ..., 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	--	03:00 (Tolerance ± 1 cardiac cycle)
Passive Method 3 zones (one value for all zones)	Off, On	Off, On	--	On
Passive Method 2 zones	--	Off, On	--	On
Active Method 3 zones (one value for all zones)	Off, On	Off, On	--	On
Active Method 2 zones	--	Off, On	--	On
Temp LRL (min ⁻¹) 3 zones (one value for all zones)	Use Norm LRL, 30, 35, ..., 105	Use Norm LRL, 30, 35, ..., 105	--	Use Norm LRL (Tolerance ± 5 ms)
Temp LRL (min ⁻¹) 2 zones	--	Use Norm LRL, 30, 35, ..., 105	--	Use Norm LRL (Tolerance ± 5 ms)
Atrial Tachy Discrimination 3 zones (one value for all zones)	Off, On	Off, On	--	On
Atrial Tachy Discrimination 2 zones	--	Off, On	--	On
AFib Rate Threshold (min ⁻¹) 3 zones (one value for all zones) ^{ac}	100, 110, ..., 300	100, 110, ..., 300	--	170 (Tolerance ± 5 ms)
AFib Rate Threshold (min ⁻¹) 2 zones ^{ac}	--	100, 110, ..., 300	--	170 (Tolerance ± 5 ms)

Table A-7. Rhythm ID detection enhancement parameters for 2-zone and 3-zone configurations (continued)

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
Stability (ms) 3 zones (one value for all zones) ^{ab}	6, 8, ..., 32, 35, 40, ..., 60, 70, ..., 120	6, 8, ..., 32, 35, 40, ..., 60, 70, ..., 120	--	20 (DR); 30 (VR) (Tolerance ± 5 ms)
Stability (ms) 2 zones ^{ab}	--	6, 8, ..., 32, 35, 40, ..., 60, 70, ..., 120	--	20 (DR); 30 (VR) (Tolerance ± 5 ms)

- a. This parameter is used in initial detection and Post-shock detection. Changing the value for initial detection will change the value for Post-shock Brady.
- b. The Stability parameter only applies in Post-shock for VR devices.
- c. All of the AFib Rate Thresholds are linked to the ATR Trigger Rate and Atrial Flutter Response Rate. If any one of these rates is reprogrammed, the others will automatically change to the same value.

Table A-8. Post-shock Onset/Stability detection enhancement parameters for 2-zone and 3-zone configurations

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
Post-shock V Rate > A Rate 3 zones ^a	Off, On	--	--	On
Post-shock V Rate > A Rate 2 zones	--	Off, On	--	On
Post-shock AFib Rate Threshold (min ⁻¹) 3 zones ^{a b}	Off, 100, 110, ..., 300	--	--	170 (Tolerance ± 5 ms)
Post-shock AFib Rate Threshold (min ⁻¹) 2 zones ^b	--	Off, 100, 110, ..., 300	--	170 (Tolerance ± 5 ms)
Post-shock Stability (ms) 3 zones ^a	Off, 6, 8, ..., 32, 35, 40, ..., 60, 70, 80, ..., 120	--	--	20 (DR); 30 (VR) (Tolerance ± 5 ms)
Post-shock Stability (ms) 2 zones	--	Off, 6, 8, ..., 32, 35, 40, ..., 60, 70, 80, ..., 120	--	20 (DR); 30 (VR) (Tolerance ± 5 ms)
Post-shock Sustained Rate Duration (min:sec) 3 zones ^a	Off, 00:10, 00:15, ..., 00:55, 01:00, 01:15, ..., 02:00, 02:30, 03:00, ..., 10:00, 15:00, 20:00, ..., 60:00	--	--	00:15 (Tolerance ± 1 cardiac cycle)
Post-shock Sustained Rate Duration (min:sec) 2 zones	--	Off, 00:10, 00:15, ..., 00:55, 01:00, 01:15, ..., 02:00, 02:30, 03:00, ..., 10:00, 15:00, 20:00, ..., 60:00	--	00:15 (Tolerance ± 1 cardiac cycle)

- a. If all VT-1 therapy is programmed to Off, detection enhancements will apply in the VT zone, not the VT-1 zone.
- b. All of the AFib Rate Thresholds are linked to the ATR Trigger Rate and Atrial Flutter Response Rate. If any one of these rates is reprogrammed, the others will automatically change to the same value.

Table A-9. Post-shock Rhythm ID detection enhancement parameters for 2-zone and 3-zone configurations

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
Post-shock Detection Enhancement 3 zones	Off, On	Off, On	--	Off
Post-shock Detection Enhancement 2 zones	--	Off, On	--	Off
Post-shock Sustained Rate Duration (min:sec) 3 zones	Off, 00:10, 00:15, ..., 00:55, 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	Off, 00:10, 00:15, ..., 00:55, 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	--	0:15 (Tolerance ± 1 cardiac cycle)
Post-shock Sustained Rate Duration (min:sec) 2 zones	--	Off, 00:10, 00:15, ..., 00:55, 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	--	0:15 (Tolerance ± 1 cardiac cycle)
AFib Rate Threshold (min ⁻¹) 3 zones (one value for all zones) ^{ac}	100, 110, ..., 300	100, 110, ..., 300	--	170 (Tolerance ± 5 ms)

Table A-9. Post-shock Rhythm ID detection enhancement parameters for 2-zone and 3-zone configurations (continued)

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
AFib Rate Threshold (min ⁻¹) 2 zones ^{ac}	--	100, 110, ..., 300	--	170 (Tolerance ± 5 ms)
Stability (ms) 3 zones (one value for all zones) ^{ab}	6, 8, ..., 32, 35, 40, ..., 60, 70, ..., 120	6, 8, ..., 32, 35, 40, ..., 60, 70, ..., 120	--	20 (DR); 30 (VR) (Tolerance ± 5 ms)
Stability (ms) 2 zones ^{ab}	--	6, 8, ..., 32, 35, 40, ..., 60, 70, ..., 120	--	20 (DR); 30 (VR) (Tolerance ± 5 ms)

- a. This parameter is used in initial detection and Post-shock detection. Changing the value for initial detection will change the value for Post-shock Brady.
- b. The Stability parameter only applies in Post-shock for VR devices.
- c. All of the AFib Rate Thresholds are linked to the ATR Trigger Rate and Atrial Flutter Response Rate. If any one of these rates is reprogrammed, the others will automatically change to the same value.

Table A-10. Ventricular ATP parameters (specified into a 750 Ω load)

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
ATP Type 3 zones	Off, Burst, Ramp, Scan, Ramp/Scan	Off, Burst, Ramp, Scan, Ramp/Scan	--	Off (VT-1); Burst (VT ATP1); Ramp (VT ATP2)
ATP Type 2 zones	--	Off, Burst, Ramp, Scan, Ramp/Scan	--	Burst (VT ATP1); Ramp (VT ATP2)
Number of Bursts (per scheme) 3 zones	Off; 1, 2, ..., 30	Off; 1, 2, ..., 30	--	Off (VT-1); 2 (VT ATP1); 1 (VT ATP2)
Number of Bursts (per scheme) 2 zones	--	Off; 1, 2, ..., 30	--	2 (VT ATP1); 1 (VT ATP2)
Initial Pulse (pulses) 3 zones	1, 2, ..., 30	1, 2, ..., 30	--	4 (VT-1); 10 (VT)
Initial Pulse (pulses) 2 zones	--	1, 2, ..., 30	--	10
Pulse Increment (pulses) 3 zones	0, 1, ..., 5	0, 1, ..., 5	--	0
Pulse Increment (pulses) 2 zones	--	0, 1, ..., 5	--	0
Maximum Number of Pulses 3 zones	1, 2, ..., 30	1, 2, ..., 30	--	4 (VT-1); 10 (VT)
Maximum Number of Pulses 2 zones	--	1, 2, ..., 30	--	10
Coupling Interval (% or ms) 3 zones	50, 53, 56, 59; 63, 66, ..., 84, 88, 91, 94, 97% or 120, 130, ..., 750 ms	50, 53, 56, 59; 63, 66, ..., 84, 88, 91, 94, 97% or 120, 130, ..., 750 ms	--	81% (Tolerance ± 5 ms)
Coupling Interval (% or ms) 2 zones	--	50, 53, 56, 59; 63, 66, ..., 84, 88, 91, 94, 97% or 120, 130, ..., 750 ms	--	81% (Tolerance ± 5 ms)
Coupling Interval Decrement (ms) 3 zones	0, 2, ..., 30	0, 2, ..., 30	--	0 (Tolerance ± 5 ms)
Coupling Interval Decrement (ms) 2 zones	--	0, 2, ..., 30	--	0 (Tolerance ± 5 ms)
Burst Cycle Length (BCL) (% or ms) 3 zones	50, 53, 56, 59; 63, 66, ..., 84, 88, 91, 94, 97% or 120, 130, ..., 750 ms	50, 53, 56, 59; 63, 66, ..., 84, 88, 91, 94, 97% or 120, 130, ..., 750 ms	--	81% (Tolerance ± 5 ms)
Burst Cycle Length (BCL) (% or ms) 2 zones	--	50, 53, 56, 59; 63, 66, ..., 84, 88, 91, 94, 97% or 120, 130, ..., 750 ms	--	81% (Tolerance ± 5 ms)
Ramp Decrement (ms) 3 zones	0, 2, ..., 30	0, 2, ..., 30	--	0 (VT-1); 0 (VT ATP1); 10 (VT ATP2) (Tolerance ± 5 ms)
Ramp Decrement (ms) 2 zones	--	0, 2, ..., 30	--	0 (VT ATP1); 10 (VT ATP2) (Tolerance ± 5 ms)

Table A-10. Ventricular ATP parameters (specified into a 750 Ω load) (continued)

Parameter	VT-1 Zone	VT Zone	VF Zone	Nominal
Scan Decrement (ms) 3 zones	0, 2, ..., 30	0, 2, ..., 30	--	0 (Tolerance ± 5 ms)
Scan Decrement (ms) 2 zones	--	0, 2, ..., 30	--	0 (Tolerance ± 5 ms)
Minimum Interval (ms) 3 zones	120, 130, ..., 400	120, 130, ..., 400	--	220 (Tolerance ± 5 ms)
Minimum Interval (ms) 2 zones	--	120, 130, ..., 400	--	220 (Tolerance ± 5 ms)
Right Ventricular ATP Pulse Width ^a (ms) 3 zones (one value for all zones)	0.1, 0.2, ..., 2.0	0.1, 0.2, ..., 2.0	--	1.0 (Tolerance ± 0.03 ms at < 1.8 ms; ± 0.08 ms at ≥ 1.8 ms)
Right Ventricular ATP Pulse Width ^a (ms) 2 zones (one value for all zones)	--	0.1, 0.2, ..., 2.0	--	1.0 (Tolerance ± 0.03 ms at < 1.8 ms; ± 0.08 ms at ≥ 1.8 ms)
Right Ventricular ATP Amplitude ^a (V) 3 zones (one value for all zones)	0.1, 0.2, ..., 3.5, 4.0, ..., 7.5	0.1, 0.2, ..., 3.5, 4.0, ..., 7.5	--	5.0 (Tolerance ± 15% or ± 100 mV, whichever is greater)
Right Ventricular ATP Amplitude ^a (V) 2 zones (one value for all zones)	--	0.1, 0.2, ..., 3.5, 4.0, ..., 7.5	--	5.0 (Tolerance ± 15% or ± 100 mV, whichever is greater)
ATP Time-out ^b (min:sec) 3 zones	Off, 00:10, 00:20, ..., 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	Off, 00:10, 00:20, ..., 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	--	01:00 (Tolerance ± 1 cardiac cycle)
ATP Time-out (min:sec) 2 zones	--	Off, 00:10, 00:20, ..., 01:00, 01:15, ..., 02:00, 02:30, ..., 10:00, 15:00, ..., 60:00	--	01:00 (Tolerance ± 1 cardiac cycle)
QUICK CONVERT ATP (VF Only) 1, 2, or 3 zones	--	--	Off, On	On

- a. The programmed Amplitude and Pulse Width values affect Post Therapy Brady Pacing, but are separately programmable from Normal Brady Pacing, Temporary Brady Pacing, and EP Test.
- b. The VT-1 ATP Time-out must be greater than or equal to the VT ATP Time-out.

Table A-11. Ventricular Shock Parameters

Parameter	Programmable Values	Nominal
Shocks 1 and 2 energy (J) ^{a b c} (stored energy)	Off, 0.1, 0.3, 0.6, 0.9, 1.1, 1.7, 2, 3, 5, 6, 7, 9, 11, 14, 17, 21, 23, 26, 29, 31, 36, 41	41 J (Tolerance ± 60% for ≤ 0.3 J, ± 40% for 0.6–3 J, ± 20% for 5–36 J, ± 10% for 41 J)
Energy of Remaining Shocks (J) ^{a c} (stored energy)	Off, 41	41 J (Tolerance ± 10% for 41 J)
Lead Polarity ^d	Initial, Reversed	Initial
Committed Shock	Off, On	Off
Shock Lead Vector	RV Coil to RA Coil and Can, RV Coil to Can, RV Coil to RA Coil	RV Coil to RA Coil and Can

- a. Biphasic energy is specified.
- b. The Shock 2 energy level must be greater than or equal to the Shock 1 energy level.
- c. In a VT-1 zone of a 3-zone configuration or a VT zone of a 2-zone configuration, all or some of the shocks may be programmed to Off while other shocks in that zone are programmed in joules.
- d. A commanded STAT SHOCK is delivered at the programmed Polarity.

Table A-12. Pacing therapy parameters (Normal, Post-Therapy, and Temporary) (specified into a 750 Ω load)

Parameter	Programmable Values	Nominal
Mode ^{a b f j}	DDD(R), DDI(R), VDD(R), VVI(R), AAI(R), Off; Temporary: DDD, DDI, DOO, VDD, VVI, VOO, AAI, AOO, Off	DDD (DR); VVI (VR)

Table A-12. Pacing therapy parameters (Normal, Post-Therapy, and Temporary) (specified into a 750 Ω load) (continued)

Parameter	Programmable Values	Nominal
RYTHMIQ ^{f i}	Off, AAI(R) with VVI Backup	Off
Lower Rate Limit (LRL) ^{a c} (min ⁻¹)	30, 35, ..., 185	60 (Tolerance ± 5 ms)
Maximum Tracking Rate (MTR) ^{f i} (min ⁻¹)	50, 55, ..., 185	130 (Tolerance ± 5 ms)
Maximum Sensor Rate (MSR) ^{f i} (min ⁻¹)	50, 55, ..., 185	130 (Tolerance ± 5 ms)
Pulse Amplitude ^{a d e} (atrium) (V)	0.1, 0.2, ... 3.5, 4.0, ..., 5.0	3.5 (5.0 post-therapy) (Tolerance ± 15% or ± 100mV, whichever is greater)
Pulse Amplitude ^{a d e} (right ventricle) (V)	0.1, 0.2, ..., 3.5, 4.0, ..., 7.5	3.5 (5.0 post-therapy) (Tolerance ± 15% or ± 100mV, whichever is greater)
Pulse Width ^{a d e} (atrium, right ventricle) (ms)	0.1, 0.2, ..., 2.0	0.4 (1.0 post-therapy) (Tolerance ± 0.03 ms at < 1.8 ms; ± 0.08 ms at ≥ 1.8 ms)
Atrial Pace/Sense Configuration ^{a f}	Bipolar, Off	Bipolar
Activity Threshold ^{f i}	Very High, High, Medium High, Medium, Medium Low, Low, Very Low	Medium
Reaction Time ^{f i} (sec)	10, 20, ..., 50	30
Response Factor ^{f i}	1, 2, ..., 16	8
Recovery Time ^{f i} (min)	2, 3, ..., 16	2
Maximum PVARP ^{a f} (ms)	150, 160, ..., 500	280 (Tolerance ± 5 ms)
Minimum PVARP ^{a f} (ms)	150, 160, ..., 500	240 (Tolerance ± 5 ms)
PVARP After PVC ^{a f} (ms)	Off, 150, 200, ..., 500	400 (Tolerance ± 5 ms)
V-Blank After A-Pace ^{a g} (ms)	45, 65, 85, Smart	65 (Tolerance ± 5 ms)
A-Blank After V-Pace ^{a g} (ms)	45, 65, 85, Smart	Smart (Tolerance ± 5 ms)
A-Blank After V-Sense ^{a g} (ms)	45, 65, 85, Smart	Smart (Tolerance ± 5 ms)
Maximum VRP (right ventricle) ^{a h} (ms)	150, 160, ..., 500	250 (Tolerance ± 5 ms)
Minimum VRP (right ventricle) ^{a h} (ms)	150, 160, ..., 500	230 (DR); 250 (VR) (Tolerance ± 5 ms)
Maximum Paced AV Delay ^{a f} (ms)	30, 40, ..., 400	180 (Tolerance ± 5 ms)
Minimum Paced AV Delay ^{a f} (ms)	30, 40, ..., 400	80 (Tolerance ± 5 ms)
Maximum Sensed AV Delay ^{a f} (ms)	30, 40, ..., 400	150 (Tolerance ± 5 ms)
Minimum Sensed AV Delay ^{a f} (ms)	30, 40, ..., 400	65 (Tolerance ± 5 ms)
AV Search + ^{f i}	Off, On	Off
AV Search + Search Interval ^{f i} (cycles)	32, 64, 128, 256, 512, 1024	32 (Tolerance ± 1 cycle)
AV Search + Search AV Delay ^{f i} (ms)	30, 40, ..., 400	300 (Tolerance ± 5 ms)
Respiratory Sensor ^{a f}	Off, On	Off
Rate Hysteresis Hysteresis Offset ^{f i} (min ⁻¹)	-80, -75, ..., -5, Off	Off (Tolerance ± 5 ms)
Rate Hysteresis Search Hysteresis ^{f i} (cycles)	Off, 256, 512, 1024, 2048, 4096	Off (Tolerance ± 1 cycle)
Rate Smoothing (up, down) ^{f i} (%)	Off, 3, 6, 9, 12, 15, 18, 21, 25	Off (Tolerance ± 1%)
Rate Smoothing Maximum Pacing Rate (MPR) ^{a f} (min ⁻¹)	50, 55, ..., 185	130 (Tolerance ± 5 ms)

Table A-12. Pacing therapy parameters (Normal, Post-Therapy, and Temporary) (specified into a 750 Ω load) (continued)

Parameter	Programmable Values	Nominal
Noise Response ^{a f k}	AOO, VOO, DOO, Inhibit Pacing	DOO for DDD(R) and DDI(R) modes; VOO for VDD(R) and VVI(R) modes; AOO for AAI(R) mode
Post-therapy Pacing Period (min:sec) (available post-shock only)	00:15, 00:30, 00:45, 01:00, 01:30, 02:00, 03:00, 04:00, 05:00, 10:00, 15:00, 30:00, 45:00, and 60:00	00:30 (Tolerance ± 1 cardiac cycle)

- a. The programmed Normal Brady values will be used as the nominal values for Temporary Brady pacing.
- b. Refer to the NASPE/BPEG codes below for an explanation of the programmable values. The identification code of the North American Society of Pacing and Electrophysiology (NASPE) and the British Pacing and Electrophysiology Group (BPEG) is based on the categories listed in the table.
- c. The basic pulse period is equal to the pacing rate and the pulse interval (no hysteresis). Runaway protection circuitry inhibits bradycardia pacing above 205 min⁻¹. Magnet application does not affect pacing rate (test pulse interval).
- d. Separately programmable for ATP/Post-Shock, Temporary Brady, and EP Test.
- e. Values are not affected by temperature variation within the range 20°–43°C.
- f. This parameter is used globally in Normal Brady pacing and Post-shock Brady pacing. Changing the value for Normal Brady will change the value for Post-shock Brady.
- g. This parameter is automatically set to at least 85 ms for Post-Shock Brady.
- h. This parameter is automatically adjusted in Post-Shock Brady to allow appropriate sensing.
- i. This parameter is disabled during Temporary Brady.
- j. The programmable values for VR devices only include VVI(R), Off; Temporary: VVI, VOO, Off
- k. The programmable values for VR devices only includes VOO and Inhibit Pacing and as such the nominal is VOO.

Table A-13. Atrial Tachy Parameters

Parameter	Programmable Values	Nominal
ATR Mode Switch ^{ab}	Off, On	On
ATR Trigger Rate ^{a bd} (min ⁻¹)	100, 110, ..., 300	170 (Tolerance ± 5 ms)
ATR Duration ^{ab} (cycles)	0, 8, 16, 32, 64, 128, 256, 512, 1024, 2048	8 (Tolerance ± 1 cardiac cycle)
Entry Count ^{ab} (cycles)	1, 2, ..., 8	8
Exit Count ^{ab} (cycles)	1, 2, ..., 8	8
ATR Fallback Mode ^{eb}	VDI, DDI, VDIR, DDIR	DDI
ATR Fallback Time ^{ab} (min:sec)	00:00, 00:15, 00:30, 00:45, 01:00, 01:15, 01:30, 01:45, 02:00	00:30
ATR/VTR Fallback LRL ^{ab} (min ⁻¹)	30, 35, ..., 185	70 (Tolerance ± 5 ms)
ATR Ventricular Rate Regulation (VRR) ^{ab}	Off, On	On
ATR Maximum Pacing Rate (MPR) ^{a b} (min ⁻¹)	50, 55, ..., 185	130 (Tolerance ± 5 ms)
Atrial Flutter Response ^{cb}	Off, On	Off
Atrial Flutter Response Trigger Rate ^{bdc} (min ⁻¹)	100, 110, ..., 300	170 (Tolerance ± 5 ms)
PMT Termination ^{cb}	Off, On	On
Ventricular Rate Regulation (VRR) ^{cb}	Off, On	Off
VRR Maximum Pacing Rate (MPR) ^{bc} (min ⁻¹)	50, 55, ..., 185	130 (Tolerance ± 5 ms)

- a. The programmed Normal Brady values will be used as the nominal values for Temporary Brady pacing.
- b. This parameter is used globally in Normal Brady pacing and Post-shock Brady pacing. Changing the value for Normal Brady will change the value for Post-shock Brady.
- c. This parameter gets disabled during Temporary Brady.
- d. ATR Trigger Rate and Atrial Flutter Response Rate are linked to all the AFib Rate Thresholds. If any one of these rates is reprogrammed, the others will automatically change to the same value.
- e. If Normal Brady ATR Fallback Mode is DDIR or DDI, then Temporary Brady ATR Fallback Mode is DDI and If Normal Brady ATR Fallback Mode is VDIR or VDI, then Temporary Brady ATR Fallback Mode is VDI.

Table A-14. Brady Mode values based on NASPE/BPEG codes

Position	I	II	III	IV	V
Category	Chambers Paced	Chambers Sensed	Response to Sensing	Programmability, rate modulation	Antitachyarrhythmia Functions
Letters	0–None	0–None	0–None	0–None	0–None
	A–Atrium	A–Atrium	T–Triggered	P–Simple Programmable	P–Pacing (Antitachyarrhythmia)

Table A-14. Brady Mode values based on NASPE/BPEG codes (continued)

Position	I	II	III	IV	V
	V–Ventricle	V–Ventricle	I–Inhibited	M–Multiprogrammable	S–Shock
	D–Dual (A&V)	D–Dual (A&V)	D–Dual (T&I)	C–Communicating	D–Dual (P&S)
				R–Rate Modulation	
Mfrs. Designation Only	S–Single (A or V)	S–Single (A or V)			

Table A-15. Magnet and Beeper functions

Parameter	Programmable Values	Nominal
Magnet Response	Off, Store EGM, Inhibit Therapy	Inhibit Therapy
Beep During Capacitor Charge	Off, On	Off
Beep When Explant is Indicated	Off, On	On
Atrial Beep When Out-of-Range	Off, On	Off
Ventricular Beep When Out-of-Range	Off, On	Off
Shock Beep When Out-of-Range	Off, On	Off

Table A-16. Sensitivity Adjustment

Parameter	Programmable Values	Nominal
Atrial Sensitivity ^a (mV)	AGC 0.15, AGC 0.2, AGC 0.25, AGC 0.3, AGC 0.4, ..., AGC 1.0, AGC 1.5	AGC 0.25
Right Ventricular Sensitivity ^a (mV)	AGC 0.15, AGC 0.2, AGC 0.25, AGC 0.3, AGC 0.4, ..., AGC 1.0, AGC 1.5	AGC 0.6

a. With CENELEC waveform, per EN 45502-2-2:2008.

Table A-17. Daily Lead Measurements

Parameter	Programmable Values	Nominal
Atrial Daily Intrinsic Amplitude	On, Off	On
Ventricular Daily Intrinsic Amplitude	On, Off	On
Atrial Daily Impedance	On, Off	On
Ventricular Daily Impedance	On, Off	On
Shock Daily Impedance	On, Off	On
Atrial Low Impedance Limit (Ω)	200, 250, ..., 500	200
Ventricular Low Impedance Limit (Ω)	200, 250, ..., 500	200

Table A-18. Ventricular Commanded ATP

Parameter ^a	Programmable Values	Nominal
Commanded Ventricular ATP (Type)	Burst, Ramp, Scan, Ramp/Scan	Burst
Number Of Bursts	1, 2, ..., 30	30
Initial Pulses per Burst (pulses)	1, 2, ..., 30	4
Pulse Increment (pulses)	0, 1, ..., 5	0
Maximum Number of Pulses	1, 2, ..., 30	4
Coupling Interval (% or ms)	50, 53, 56, 59; 63, 66, ..., 84; 88, 91, 94, 97% or 120, 130, ..., 750 ms	81% (Tolerance ± 5 ms)
Coupling Interval Decrement (ms)	0, 2, ..., 30	0 (Tolerance ± 5 ms)

Table A-18. Ventricular Commanded ATP (continued)

Parameter ^a	Programmable Values	Nominal
Burst Cycle Length (BCL) (% or ms)	50, 53, 56, 59; 63, 66, ..., 84; 88, 91, 94, 97% or 120, 130, ..., 750 ms	81% (Tolerance \pm 5 ms)
Ramp Decrement (ms)	0, 2, ..., 30	0 (Tolerance \pm 5 ms)
Scan Decrement (ms)	0, 2, ..., 30	0 (Tolerance \pm 5 ms)
Minimum Interval (ms)	120, 130, ..., 400	200 (Tolerance \pm 5 ms)

a. The ventricular Commanded ATP Pulse Width and Amplitude values are the same as programmed for ventricular ATP therapy.

Table A-19. 50 Hz/Manual Burst Pacing

Parameter ^a	Programmable Values	Nominal
Burst Interval (ms)	20, 30, ..., 750	600 (Tolerance \pm 5 ms)
Minimum Interval (ms)	20, 30, ..., 750	200 (Tolerance \pm 5 ms)
Decrement (ms)	0, 10, ..., 50	50 (Tolerance \pm 5 ms)

a. Applied to the atrium or ventricle depending on the chamber selected.

Table A-20. Ventricular Commanded Shock

Parameter	Programmable Values	Nominal
Shock (stored energy) (J)	0.1, 0.3, 0.6, 0.9, 1.1, 1.7, 2, 3, 5, 6, 7, 9, 11, 14, 17, 21, 23, 26, 29, 31, 36, 41	41 (Tolerance \pm 60% for \leq 0.3 J; \pm 40% for 0.6–3 J; \pm 20% for 5–36 J, \pm 10% for 41 J)
Coupling Interval (ms)	Sync, 50, 60, ..., 500	Sync

Table A-21. VFib (Ventricular Fibrillation) Induction

Parameter	Values
VFib High	15V (nonprogrammable) (Tolerance \pm 10V)
VFib Low	9V (nonprogrammable) (Tolerance \pm 7V)

Table A-22. Shock on T Induction

Parameter	Programmable Values	Nominal
Shock (stored energy) (J)	0.1, 0.3, 0.6, 0.9, 1.1, 1.7, 2, 3, 5, 6, 7, 9, 11, 14, 17, 21, 23, 26, 29, 31, 36, 41	1.1 J (Tolerance \pm 60% for \leq 0.3 J; \pm 40% for 0.6–3 J; \pm 20% for 5–36 J, \pm 10% for 41 J)
Number of S1 Pulses	1, 2, ..., 30	8
S1 Interval (ms)	120, 130, ..., 750	400
Coupling Interval (ms)	Sync, 10, 20, ..., 500	310

Table A-23. PES (Programmed Electrical Stimulation)

Parameter ^a	Programmable Values	Nominal
Number of S1 Intervals (pulses)	1, 2, ..., 30	8
S2 Decrement (ms)	0, 10, ..., 50	0
S1 Interval (ms)	120, 130, ..., 750	600 (Tolerance \pm 5 ms)
S2 Interval (ms)	Off, 120, 130, ..., 750	600 (Tolerance \pm 5 ms)
S3 Interval (ms)	Off, 120, 130, ..., 750	Off (Tolerance \pm 5 ms)
S4 Interval (ms)	Off, 120, 130, ..., 750	Off (Tolerance \pm 5 ms)
S5 Interval (ms)	Off, 120, 130, ..., 750	Off (Tolerance \pm 5 ms)

a. Applied to the atrium or ventricle as commanded by the programmer.

PACEMAKER INTERACTION

APPENDIX B

There may be instances where patients may have a separate temporary or permanent pacemaker. Temporary or permanent pacemakers can interact with an ICD and can interfere with the identification of tachyarrhythmias in the following ways:

- During a tachyarrhythmia, if the pacemaker does not sense the arrhythmia and paces, and the pacing pulse detected from the ICD rate-sensing electrode is large enough, it could cause the ICD to interpret the pacing as a normal rhythm at the rate of the pacemaker. The ICD would neither detect the arrhythmia nor deliver therapy.
- The pacemaker could present signals to the ICD resulting from the following events:
 - Inappropriate sensing
 - Lead dislodgment
 - Failure to capture

This could cause the ICD's rate measurement to be faster than the patient's actual heart rate. As a result, the ICD could deliver inappropriate therapy.

- Conduction delay could cause the ICD to count both pacemaker artifact and ventricular depolarization. This could result in inappropriate ICD therapy.

For these reasons, the use of a pacemaker that results in pacemaker and ICD interaction is not recommended. Unipolar pacemakers are contraindicated for use with an ICD.

Consider the following actions if a separate pacemaker is used:

- Always deactivate a patient's ICD when:
 - Using temporary bipolar or temporary AV sequential pacing
 - Reprogramming a separate implanted pacemaker

If cardioversion or defibrillation is needed at such times, use an external defibrillator.

- The ICD rate-sensing electrodes should be as far from the pacing electrodes as possible.
- After implanting the pacing leads, examine the signals from the ICD rate-sensing electrodes to ensure that minimal pacemaker artifacts are present.
- Since it is difficult to predict the relative magnitudes of pacemaker artifacts and various tachyarrhythmia electrograms that may occur chronically or during EP testing, it is important to reduce artifacts to a minimum.
- All of the patient's rhythms should be induced while the ICD is activated and the separate pacemaker is programmed to an asynchronous mode at maximum output. This should provide the greatest opportunity for inhibition of arrhythmia detection due to pacemaker artifacts. Leads may have to be repositioned to eliminate artifacts.

- To reduce the possibility of pacemaker interaction, consider testing the separate pacemaker by programming it to the following settings:


- The lowest amplitude allowable for safe capture in the chronic state
- The maximum sensitivity to ensure that pacing is inhibited during VF
- The minimum cardiac rate acceptable for the patient

Also consider using rate-sensing and pacemaker leads with close intraelectrode spacing (e.g., 11 mm).

- Consider turning off the bradycardia pacing function of the ICD or programming that function to a rate less than the separate pacemaker rate.
- For pacing control following any shock therapy, consider whether to use the ICD's post-therapy bradycardia pacing feature at higher rates and outputs or the separate pacemaker.

The following test procedure aids in determining the potential for pacemaker/ICD interaction (Figure B-1 on page B-3).

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolete. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult versio. Ne használja!
Wersja nieaktualna. Nie używać.

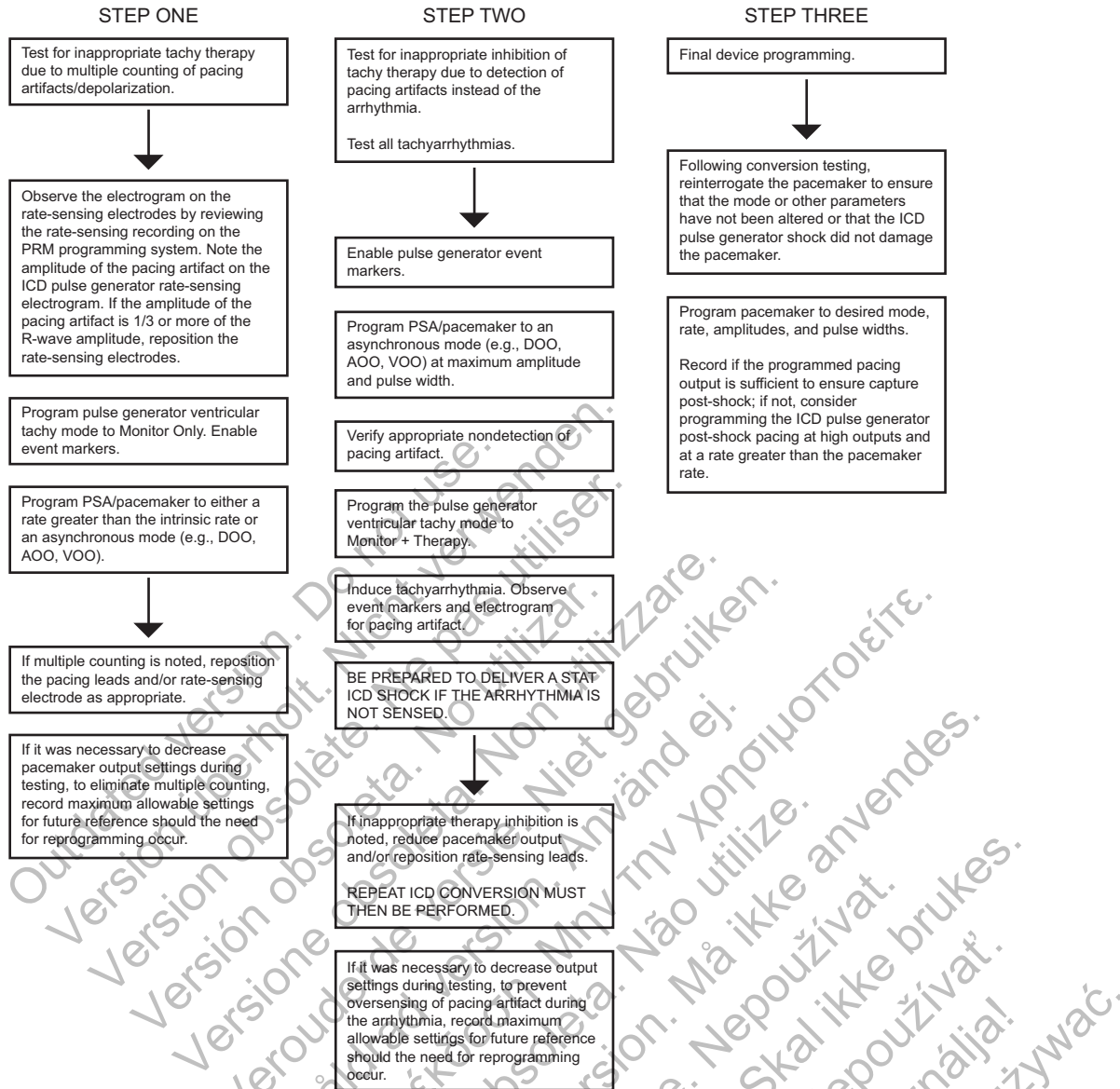

NOTE: The Guidant Model 6860 Magnet also can be used to assess pacemaker interaction if the magnet function is enabled. Placing the magnet over a device in a ventricular Monitor + Therapy mode should produce tones.

Figure B-1. Test procedure for pacemaker-ICD interaction

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verze. Nepoužívat.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.

SYMBOLS ON PACKAGING

APPENDIX C

SYMBOLS ON PACKAGING

The following symbols may be used on packaging and labeling (Table C-1 on page C-1):

Table C-1. Symbols on packaging























Symbol	Description
	Reference number
	Package contents
	Pulse generator
	Torque wrench
	Literature enclosed
	Serial number
	Use by
	Lot number
	Date of manufacture
	Non-ionizing electromagnetic radiation
	Sterilized using ethylene oxide
	Do not resterilize
	Do not reuse
	Do not use if package is damaged
	Dangerous voltage
	Consult instructions for use
	Temperature limitation
	CE mark of conformity with the identification of the notified body authorizing use of the mark
	RTTE designation for radio equipment with a use restriction

Table C-1. Symbols on packaging (continued)

Symbol	Description
	Ward placement indicator for interrogation
	Opening instruction
	Authorized Representative in the European Community
	Manufacturer

Outdated version. Do not use.
 Version überholt. Nicht verwenden.
 Version obsolète. Ne pas utiliser.
 Versión obsoleta. No utilizar.
 Versione obsoleta. Non utilizzare.
 Verouderde versie. Niet gebruiken.
 Föråldrad version. Använd ej.
 Παλιά έκδοση. Μην χρησιμοποιείτε.
 Versão obsoleta. Não utilize.
 Forældet version. Må ikke anvendes.
 Zastaralá verze. Nepoužívat.
 Utdatert versjon. Skal ikke brukes.
 Zastaraná verzia. Nepoužívať.
 Elavult verzió. Ne használja!
 Wersja nieaktualna. Nie używać.

INDEX

Symbols

50 Hz/manual burst pacing 7-7

A

A-blank

after RV-sense 4-37

after V-pace 4-37

A-tachy response (ATR)

mode switch 4-19

Accelerate, in zone 3-2

Accelerometer 4-13

activity threshold 4-15

reaction time 4-16

recovery time 4-17

response factor 4-14

Activity threshold 4-15

Adaptive-rate pacing 4-13

AFib rate threshold 2-20, 2-25, 2-26

AGC (automatic gain control) 4-9

Amplitude 4-8

ATP (antitachycardia pacing) 3-13

intrinsic test 5-11

Application screen 1-2

Arrhythmia logbook 6-2

episode detail 6-5

events summary 6-5

interval 6-7

stored EGM 6-5

ATP (antitachycardia pacing) 3-7

amplitude 3-13

burst cycle length (BCL) 3-10

burst scheme 3-11

commanded, EP test 7-9

coupling interval 3-9

minimum interval 3-10

number of bursts 3-8

pulse count 3-8

pulse width 3-13

ramp scheme 3-11

ramp/scan scheme 3-12

redetection after ATP 2-14

scan scheme 3-12

time-out 3-13

ATR (atrial tachy response)

atrial flutter response 4-23

duration 4-20

end of ATR episode 4-22

entry count 4-20

exit count 4-21

LRL, fallback 4-21

maximum pacing rate 4-22

mode switch 4-19

mode, fallback 4-21

PMT termination 4-23

rate threshold 4-20

time, fallback 4-21

ventricular rate regulation 4-22

VTR (ventricular tachy response) 4-22

ATR Trigger Rate 4-20

Atrial

refractory period, post ventricular atrial (PVARP) 4-33

refractory period, same chamber 4-35

use of atrial information 2-4

Atrial flutter response 4-23

Atrial tachy

ATR mode switch 4-19

atrial flutter response 4-23

PMT termination 4-23

ventricular rate regulation 4-22

Attention conditions, yellow 1-7

Automatic Intrinsic Rhythm ID 2-6

AV delay 4-28

paced 4-28

RYTHMIQ 4-32

sensed 4-29

AV Delay

Search 4-31

AV Search + 4-31

Search AV Delay 4-31

Search Interval 4-31

B

Backup ventricular pacing during atrial stimulation,

EP test 7-5

Battery

Explant status 5-3

icon 1-5

indicator 5-3

status 5-2

Beep

during capacitor charge 5-6

feature setup 6-14

Blanking 4-36

A-blank after RV-sense 4-37

A-blank after V-pace 4-37

RV-blank after A-pace 4-36

Burst

ATP (antitachycardia pacing) 3-8

cycle length (BCL) 3-10

minimum interval 3-10

number of bursts 3-8

pacing, 50 Hz/manual burst 7-7

parameter 3-8

pulse count 3-8

scheme 3-11

Buttons, software 1-5

C

Capacitor

deformation 3-16, 5-6

re-formation 5-6

Charge time 3-15
 measurement 5-6
Check
 icon 1-5
Commanded
 ATP, EP test 7-9
 shock, EP test 7-8
 therapy, EP test 7-8
Committed shock 2-9, 3-17
Communication, telemetry
 Radio frequency (RF) 1-8
Continue
 icon 1-6
Counter
 brady 6-9
 therapy history 6-8
 ventricular 6-8
Coupling interval 3-9
 decrement 3-9

D

Daily measurements 5-7
Data
 disk 1-15
 patient 1-15
 storage 1-15
 USB 1-15
Decelerate, in zone 3-2
Decrement
 coupling interval 3-9
 ramp scheme 3-11
 scan scheme 3-12
Defibrillation
 backup defibrillator, safety mode 1-17
Demonstration
 Programmer/recorder/monitor (PRM) mode 1-3, 1-7
Detail icon 1-5
Detection
 AFib rate threshold 2-20
 duration 2-11
 enhancement 2-6, 2-17
 episode 2-14
 onset 2-24
 rate sensing 2-3
 rate threshold 2-4
 reconfirmation/committed shock 3-17
 redetection 2-9
 stability 2-22
 sustained rate duration (SRD) 2-24
 tachyarrhythmia 2-1
 tachyarrhythmia, safety mode 1-17
 V rate > A rate 2-19
 vector timing and correlation 2-19
 ventricular, initial 2-5
 window 2-10
Device
 memory 1-16
 mode 2-2
Diagnostic

battery status 5-2
histogram 6-7
lead test 5-10
patient triggered monitor 6-12
Disk
 data 1-15
 read 1-15
 save 1-15
DIVERT THERAPY 1-13
Duration 2-11
 ATR (atrial tachy response) 4-20
 post-shock 2-14
 redetection 2-14
Dynamic Noise Algorithm 4-11, 4-40

E

ECG (electrocardiogram)
 display 1-3
 surface 1-3
EGM (electrogram)
 display 1-3
 real-time 1-3
Electrocautery
 mode 2-2
Electrode, lead configuration 4-28
End of ATR episode 4-22
Energy
 shock 3-15
Enhancement
 detection 2-6, 2-17
Entry count 4-20
EP test (electrophysiologic test) 7-2
 ATP, commanded 7-9
 backup ventricular pacing during atrial stimulation 7-5
 burst pacing, 50 Hz/manual 7-7
 commanded therapy 7-8
 fibrillation 7-4
 induction 7-4
 mode, temporary 7-2
 programmed electrical stimulation (PES) 7-6
 shock on T 7-5
 shock, commanded 7-8
 VFib 7-4
Episode 2-14
 end of ATR 4-22
 nontreated 2-15, 6-8
 treated 2-15, 6-8
 ventricular 2-14
Event
 counter 6-8
 icon 1-5
 summary 6-5
 therapy history 6-2
Exit count 4-21

F

Fallback, atrial mode switch
LRL 4-21
mode 4-21
time 4-21
Fibrillation
VFib induction 7-4
Follow-up
Lead status 5-7

H

Histogram 6-7
Hold
icon 1-6
Horizontal slider
icon 1-6
Hysteresis, rate 4-25

I

Icon
battery 1-5
check 1-5
continue 1-6
details 1-5
event 1-5
hold 1-6
horizontal slider 1-6
increment and decrement 1-6
information 1-5
lead 1-5
patient 1-5
patient information 1-15
Programmer/recorder/monitor (PRM) mode
indicator 1-3
run 1-6
scrolling 1-6
sorting 1-6
vertical slider 1-6
Impedance test, lead 5-11
Implant
post, information 6-12
Increment and decrement
icon 1-6
Indications Based Programming (IBP) 1-11
Induction, EP test 7-4
Information
icon 1-5
implant 1-15
lead 1-15
patient 1-15
Interrogate 1-8
Interval
arrhythmia logbook 6-7
coupling, ATP 3-9
minimum, burst cycle length 3-10

Intrinsic amplitude test 5-11

L

Last delivered shock 5-6
Lead
configuration 4-28
Daily measurements 5-7
icon 1-5
impedance 5-11
intrinsic amplitude 5-11
Lead status 5-7
pace threshold 5-12
test 5-10
Logbook 6-2
Lower rate limit (LRL) 4-4

M

Magnet
feature setup 6-15
inhibit tachy therapy 6-16
Manual programming 1-13
Manual/50 Hz burst pacing 7-7
Maximum
pacing rate 4-22
sensor rate (MSR) 4-6
tracking rate (MTR) 4-4
Maximum pacing rate
rate smoothing 4-27
Memory, device 1-16
Minimum
interval 3-10
Mode
Demonstration 1-7
device 2-2
electrocautery 2-2
fallback ATR (atrial tachy response) 4-21
pacing 4-3
Programmer/recorder/monitor (PRM) 1-3
temporary, EP test 7-2
ventricular tachy 2-2

N

Noise
blanking and noise rejection 4-36
Dynamic Noise Algorithm 4-11, 4-40
response 4-39
Nominal parameter setting A-1
Number of bursts 3-8
pulse count 3-8

O

Onset 2-8, 2-24, 2-26, 2-27

P

Pace

STAT PACE 1-14

Pace threshold test 5-12

Pacing

adaptive-rate 4-13

amplitude 4-8

ATR mode switch 4-19

AV delay 4-28

backup during atrial stimulation 7-5

backup pacemaker in safety mode 1-17

burst, 50 Hz/manual 7-7

Indications Based Programming (IBP) 1-11

lower rate limit (LRL) 4-4

maximum sensor rate (MSR) 4-6

maximum tracking rate (MTR) 4-4

mode 4-3

noise response 4-39

parameter, basic 4-2

post therapy 4-12

post-therapy 4-11

pulse width 4-7

refractory 4-33

runaway protection 4-7

sensitivity 4-8

sensor 4-18

temporary 4-12

therapy 4-2

Package

symbol on C-1

Patient

information icon 1-5

Patient Information 1-15

Patient triggered monitor 6-12

PES (programmed electrical stimulation) 7-6

PMT (pacemaker-mediated tachycardia)

termination 4-23

Polarity

shock 3-16

Post implant information 6-12

beeper feature 6-14

magnet feature 6-15

Post-shock

detection parameter 2-9

duration 2-14

pacing 4-12

Post-therapy pacing 4-11

Premature ventricular contraction (PVC) 4-34

Prescription

therapy 3-2

Print

report 1-16

Printer

external 1-16

Program 1-11

Programmable option, parameter A-1

Programmer/recorder/monitor (PRM) 1-2

controls 1-2, 1-13

Demonstration mode 1-7

modes 1-3

software terminology 1-2

use of color 1-7

Programming recommendation 1-11, 1-13

Protection

runaway 4-7

Pulse amplitude 4-8

Pulse count 3-8

Pulse generator (PG)

memory 1-16

replacement indicators 5-3

Pulse width 4-7

ATP (antitachycardia pacing) 3-13

PVARP (post ventricular atrial refractory period) 4-33

after PVC (premature ventricular contraction) 4-34

dynamic PVARP 4-34

PVC (premature ventricular contraction) 4-34

Q

QUICK CONVERT ATP 3-14

Quit

ending a telemetry session 1-9

R

Radio frequency (RF)

interference 1-9

operating temperature, telemetry 1-9

starting telemetry 1-8

telemetry 1-8

Ramp scheme 3-11

Ramp/Scan scheme 3-12

Rate

adaptive 4-13

AFib threshold 2-20

calculation 2-3

lower limit (LRL) 4-4

maximum sensor 4-6

maximum tracking 4-4

sensing 2-3

sustained rate duration (SRD) 2-24

threshold, ventricular 2-4

V rate > A rate 2-19

ventricular 2-4

zone 2-4

Rate adaptive pacing 4-13

Rate enhancement, pacing

rate hysteresis 4-25

rate smoothing 4-25

Rate Hysteresis 4-25

hysteresis offset 4-25

search hysteresis 4-25

Rate smoothing 4-25

down 4-27
Maximum pacing rate 4-27
up 4-27
Rate threshold, ATR 4-20
Reaction time 4-16
Read data 1-15
Reconfirmation 2-9, 3-17
Recovery time 4-17
Red warning conditions 1-7
Redetection 2-9
 after ATP delivery 2-14, 3-6
 after shock delivery 2-14, 3-7
 duration 2-14
 ventricular 3-6
Reform, capacitor 5-6
Refractory
 atrial, post ventricular (PVARP) 4-33
 atrial, same chamber 4-35
 blanking and noise rejection 4-36
 PVARP after PVC 4-34
 right ventricular (RVRP) 4-35
Refractory; pacing
 refractory 4-33
Replacement Indicators 5-3
Report, printed 1-3, 1-15
 ECG/EGM 1-3
Response factor, accelerometer 4-14
Rhythm
 ID, automatic intrinsic 2-6
Right ventricular refractory (RVRP) 4-35
Run
 icon 1-6
Runaway protection 4-7
RV-blank after A-pace 4-36
RYTHMIQ 4-32

S

Safety core 1-17
Safety mode 1-17
Safety Tachy Mode 1-17
Save data 1-15
Scan scheme 3-12
Screen, programmer application 1-2
Scrolling
 icon 1-6
Search +, AV 4-31
Search AV Delay 4-31
Search Interval 4-31
Security
 ZIP telemetry 1-9
Sensing, rate 2-3
Sensitivity 4-8
 AGC (automatic gain control) 4-9
Sensor and trending, pacing 4-18
 accelerometer 4-13
 adaptive-rate 4-13
 maximum sensor rate (MSR) 4-6
Setting
 parameter value A-1

zone configuration 2-4
Shock
 charge time, energy 3-15, 5-6
 commanded, EP test 7-8
 committed 3-17
 diverting 1-13
 energy 3-15
 impedance 5-11
 last delivered 5-6
 on T induction 7-5
 polarity 3-16
 post-shock pacing 4-12
 redetection 2-14
 selection 3-2
 sequence 3-2
 STAT SHOCK 1-14
 therapy 3-15
 ventricular therapy 3-15
 waveform 3-16
Shock if unstable 2-23
Shock on T induction 7-5
Software terminology 1-2
Sorting
 icon 1-6
Stability 2-8, 2-22, 2-25, 2-26, 2-27
STAT PACE 1-14
STAT SHOCK 1-14
Stimulation, PES induction 7-6
Stored EGM
 arrhythmia logbook 6-5
Sustained rate duration (SRD) 2-24
Symbol
 on package C-1

T

Tabs, software 1-5
Tachy mode 2-2
 Safety Mode 1-17
Tachyarrhythmia
 detection 2-1
 detection in safety mode 1-17
 Indications Based Programming (IBP) 1-12
 therapy 3-2
 therapy in safety mode 1-17
 zone 2-4
Telemetry
 ending a telemetry session 1-9
 operating temperature, ZIP 1-9
 starting ZIP 1-8
 wand 1-7
 wanded 1-8
 ZIP 1-8
Temporary
 pacing 4-12
Test
 EP (electrophysiologic) 7-2
 intrinsic amplitude 5-11
 lead 5-10
 lead impedance 5-11

pace threshold 5-12
Therapy
ATP (antitachycardia pacing) 3-7
pacing 4-2
post-shock pacing 4-12
prescription 3-2
selection 3-2
shock 3-15
tachyarrhythmia 3-2
tachyarrhythmia, safety mode 1-17

Therapy history 6-2
arrhythmia logbook 6-2
counter 6-8
histogram 6-7
patient triggered monitor 6-12

Threshold
AFib rate 2-20
rate 2-4

Threshold, activity 4-15

Time-out, ATP 3-13

Timing
blanking 4-36
PVARP after PVC 4-34

Timing, pacing 4-33

Toolbar 1-4

Trending
sensor 4-18

Trends 6-9
heart rate 6-10
respiratory rate 6-10
respiratory sensor 6-11

U

Upper Rate Behavior 4-5
USB 1-15

V

V rate > A rate 2-19
Vector timing and correlation 2-19, 2-25
Ventricular
ATP (antitachycardia pacing) 3-7
detection, tachyarrhythmia 2-5
redetection after ventricular ATP therapy 3-6
redetection after ventricular shock therapy 3-7
redetection after ventricular therapy delivery 3-6
shock therapy 3-15
tachy mode 2-2
tachyarrhythmia therapy 3-2
Ventricular rate regulation 4-22
maximum pacing rate 4-22
Ventricular shock vector 3-15
Vertical slider
icon 1-6
VFib induction 7-4
VTR (ventricular tachy response) 4-22

W

Wand, telemetry 1-2, 1-7, 1-8
Warning conditions, red 1-7
Waveform, shock 3-16
Wenckebach 4-26
Window
detection 2-10

Y

Yellow attention conditions 1-7


Z

ZIP telemetry 1-8
advantages 1-8
indicator light 1-8
interference 1-9
operating temperature 1-9
radio frequency (RF) 1-8
security 1-9
session 1-8
Zone
configuration 2-4
ventricular 2-4
ventricular tachyarrhythmia 2-4
ZOOMVIEW Software Application 1-2
purpose 1-2
screens and icons 1-2
use of color 1-7

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzare.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verze. Nepoužívat.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.



Boston Scientific
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA

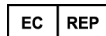

Guidant Europe NV/SA; Boston Scientific
Green Square, Lambroekstraat 5D
1831 Diegem, Belgium

www.bostonscientific.com

1.800.CARDIAC (227.3422)

+1.651.582.4000

© 2012 Boston Scientific or its affiliates.

All rights reserved.
358427-018 EN Europe 2012-04

Outdated version. Do not use.
Version überholt. Nicht verwenden.
Version obsolète. Ne pas utiliser.
Versión obsoleta. No utilizar.
Versione obsoleta. Non utilizzate.
Verouderde versie. Niet gebruiken.
Föråldrad version. Använd ej.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Versão obsoleta. Não utilize.
Forældet version. Må ikke anvendes.
Zastaralá verzia. Nepoužívať.
Utdatert versjon. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Elavult verzió. Ne használja!
Wersja nieaktualna. Nie używać.


CE0086
Authorized 2010

