
LATITUDE™ Communicator Patient Manual Supplement

USB Ethernet Adapter

en

This supplement to your patient manual explains how to connect the USB Ethernet Adapter to your LATITUDE Communicator. The steps below assume that your Communicator is already set up and connected to an electrical outlet. Your Communicator must be placed near an Ethernet (Internet) connection in order to use the USB Ethernet Adapter.

1. Insert the narrow end of the USB cable (included with the USB Ethernet Adapter) into one of the USB ports on the Communicator labeled , as shown below.
 2. Insert the square end of the USB cable into the end of the USB Ethernet Adapter nearest the adapter light.
 3. Insert the Ethernet cable (included with the USB Ethernet Adapter) into the opposite end of the adapter.
 4. Plug the Ethernet cable into an Ethernet port for your Internet service, such as a modem, router, or Ethernet wall jack. Confirm that the USB Ethernet Adapter is properly connected by verifying the green light on its front is lit (solid or flashing).
- **Important:** Your Communicator should remain connected to the electrical outlet and your Internet service.
5. Refer to your patient manual for additional instructions.
 6. To return or dispose of the USB Ethernet Adapter, follow the same instructions in your patient manual as for the Communicator.

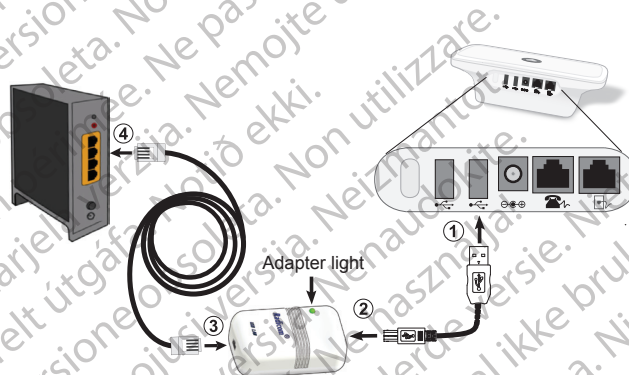The following information applies to the LATITUDE Communicator:

Boston Scientific
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA
1.800.CARDIAC (227.3422)
+1.651.582.4000
www.bostonscientific.com

Boston Scientific
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA
Marketing Authorization Holder
Boston Scientific Japan K.K.
www.bostonscientific.jp

Manufacturer
Boston Scientific
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA

Authorized Representative in the European Community
Guidant Europe NV/SA
Boston Scientific
Green Square,
Lambroekstraat 5D
1831 Diegem, Belgium
1.800.CARDIAC (227.3422)
+1.651.582.4000
www.bostonscientific.com

Australian Sponsor Address
Boston Scientific (Australia) Pty Ltd
PO Box 332
BOTANY NSW 1455 Australia
Free Phone 1 800 676 133
Free Fax 1 800 836 666

Cardiac Pacemakers Incorporated
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA

Manufactured at:
Jabil Circuit (Shanghai) Ltd.
600 Tian Lin Road
Shanghai 200233, P.R.
China

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© 2014 Boston Scientific Corporation or its affiliates. All rights reserved.