Important – Change to your LATITUDE™ Communicator Connection Method

Currently your Communicator may send information using a standard (analog) telephone line. The analog-to-wireless cellular converter that you are receiving is an update to your Communicator. It will send data via a cellular connection instead of a telephone connection.

Before setting up your new analog-to-wireless converter you may need to disconnect the telephone line.

1. Disconnecting the Telephone Line

Unplug the AC Adapter, the telephone cord and your telephone (if used) from the back of your Communicator.

Look for the corresponding symbols to indicate what should be disconnected.

Input from telephone jack

Output to telephone (optional)

AC/DC adapter power input

Important: Do not unplug anything from either USB port, as indicated by the corresponding symbol.

Universal serial bus (USB) connector

If you disconnect a telephone from your Communicator:

Before setting up your analog-to-wireless converter, please connect the telephone directly to a telephone jack.

2. Setting up the Analog-to-Wireless Converter

Follow the instructions that are included with your analog-to-wireless converter.

Note: Your Communicator may look different than the picture.

For additional assistance, contact LATITUDE Patient Services or LATITUDE Customer Support. Phone numbers are on the following page.

The following information applies to LATITUDE Communicator Models **REF**:

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LATITUDE is a trademark of Boston Scientific Corporation.