**INSTRUCTIONS FOR USE / GEBRAUCHSANWEISUNG / INSTRUCTIONS
D'UTILISATION / INSTRUCCIONES DE USO / ISTRUZIONI PER L'USO /
GEBRUIKSAANWIJZING / INSTRUÇÕES DE UTILIZAÇÃO / ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ /
INSTRUKSJONER FOR BRUK / BRUGSANVISNING / BRUKSANVISNING /
INSTRUKCJA UŻYTKOWANIA / NÁVOD NA POUŽITIE / POKYNY K POUŽITÍ /
HASZNÁLATI ÚTMUTATÓ / ИНСТРУКЦИИ ПО ЭКСПЛУАТАЦИИ / KULLANIM
TALÍMATLARI / 사용 설명서 / INSTRUÇÕES DE USO / ИНСТРУКЦИИ
ЗА УПОТРЕБА / UPUTE ZA KORIŠTENJE / KÄYTTÖOHJEET / LIETOŠANAS
INSTRUKCIJA / INSTRUCȚIUNI DE UTILIZARE / KASUTUSJUHEND /
NAUDOJIMO INSTRUKCIJOS / NAVODILA ZA UPORABO**

**PSA CABLE / PSA-KABEL / CÂBLE PSA / CABLE DE PSA / CAVO PSA / PSA-KABEL / CABO PARA PSA /
ΚΑΛΩΔΙΟ PSA / PSA-KABEL / PSA-KABEL / PSA-KABEL / KABEL PSA / KÁBEL PSA / KABEL PŘÍSTROJE
PSA / PSA-KÁBEL / КАБЕЛЬ АНАЛИЗАТОРА СИСТЕМЫ СТИМУЛЯЦИИ (PSA) / PSA KABLOSU /
PSA 케이블 / CABO PSA / PSA КАБЕЛ / PSA KABEL / PSA-KAABELI / KIA KABELIS / CABLU ASS /
PSA-KAABEL / SSA KABELIS / KABEL ZA ASS**

REF

6763

Остаряла версия. Да не се използва.
Zastaralá verze. Nepoužívat.
Forældet version. Må ikke anvendes.
Version überholt. Nicht verwenden.
Aegunud versioon. Ärge kasutage.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Outdated version. Do not use.
Version obsolete. Ne pas utiliser.
Version périmée. Nemojte upotrebljavati.
Zastarjela verzija. Neizmantot.
Úřetl útɡáfa. Notið ekki.
Versione obsoleta. Ne používajte.
Novecojusi versija. Ne utilizzare.
Pasenusi versija. Neizmantot.
Elavult verzió. Ne használja!
Dit is een verouderde versie. Niet gebruiken.
Utdatert version. Skal ikke brukes.
Wersja przeterminowana. Nie używać.
Versão obsoleta. Não utilize.
Zastarana verzija. Nepoužívat.
Zastarela različica. Ne uporabite.
Vanhentunut versio. Älä käytä.
Föråldrad version. Använd ej.
Güncel olmayan sürüm. Kullanmayın.

SYMBOLS USED ON PACKAGING / SYMBOLE AUF DER VERPACKUNG / SYMBOLES UTILISÉS SUR L'EMBALLAGE / SÍMBOLOS UTILIZADOS EN EL ENVASE / SIMBOLI RIPIORTATI SULLA CONFEZIONE / SYMBOLEN DIE OP DE VERPAKKING WORDEN GEBRUIKT / SÍMBOLOS UTILIZADOS NA EMBALAGEM / ΣΥΜΒΟΛΑ ΠΟΥ ΧΡΗΣΙΜΟΠΟΙΟΥΝΤΑΙ ΣΤΗ ΣΥΣΚΕΥΑΣΙΑ / SYMBOLER PÅ EMBALLASJEN / SYMBOLER ANVENDT PÅ EMBALLAGEN / SYMBOLER PÅ FÖRPACKNINGEN / SYMBOLE ZNAJDUJĄCE SIĘ NA OPAKOWANIU / SYMBOLY NA BALENÍ / SYMBOLY NA BALENÍ / A CSOMAGOLÁSON TALÁLHATÓ SZIMBÓLUMOK / УСЛОВНЫЕ ОБОЗНАЧЕНИЯ НА УПАКОВКЕ / AMBALAJ ÜZERİNDE KULLANILAN SİMGELER / 포장재 기호 / SÍMBOLOS UTILIZADOS NA EMBALAGEM / СИМВОЛИ, ИЗПОЛЗВАНИ ВЪРХУ ОПАКОВКАТА / SIMBOLI NA PAKIRANJU / PAKKAUKSESSA KÄYTETYT SYMBOLIT / UZ IERAKOJUMA LIETOTIE SIMBOLI / SIMBOLURI FOLOSITE PE AMBALAJ / PAKENDIL KASUTATUD SÜMBOLID / SIMBOLIAI, NAUDOJAMI ANT PAKUOTĖS / SIMBOLI, UPORABLJENI NA EMBALAŽI

Symbol / Symbol / Symbole / Símbolo / Símbolo / Symbol / Símbolo / Σύμβολο / Symbol / Symbol / Symbol / Symbol / Symbol / Symbol / Szimbólum / Условное обозначение / Simge / 기호 / Símbolo / Символ / Simbol / Symboli / Simbols / Simbol / Sümbol / Simbolis / Simbol	Definition / Definition / Signification / Definición / Definizione / Definitie / Definição / Επεξήγηση / Definisjon / Definition / Definition / Definícia / Definícia / Definicie / Meghatározás / Значение / Tanim / 정의 / Definição / Дефиниция / Definicija / Määritelmä / Definičija / Definiție / Tāhendus / Apibrēztis / Opredelitev
REF	Reference Number / Bestellnummer / Numéro de référence / Número de referencia / Numero di riferimento / Referentienummer / Número de Referência / Αριθμός αναφοράς / Referansnummer / Referencenummer / Referensnummer / Numer referencyjny / Referenčné číslo / Referenční číslo / Katalógusszám / Справочный номер / Referans Numarası / 참조 번호 / Número de referência / Номер за справка / Referentni broj / Viitenumero / Atsauces numurs / Număr de referință / Viitenumber / Nuorodos numeris / Referenčna številka
	Date of Manufacture / Herstellungsdatum / Date de fabrication / Fecha de fabricación / Data di fabbricazione / Productiedatum / Data de fabrico / Ημερομηνία κατασκευής / Produktionsdato / Produktionsdato / Tillverkningsdatum / Data produkcji / Datum výroby / Gyártás időpontja / Дата производства / Üretim Tarihi / 제조일자 / Data de manufatură / Дата на производство / Datum proizvodnje / Valmistuspäivä / Ražošanas datums / Data fabricării / Tootmiskuupäev / Pagaminimo data / Datum proizvodnje

LOT	Lot Number / Chargenbezeichnung / Numéro de lot / Número de lote / Numero di lotto / Lotnummer / Número de lote / Αριθμός παρτίδας / Lotnummer / Partinummer / Lotnummer / Numer serii / Číslo šarže / Číslo šarže / Tételszám / Номер партии / Lot Numarası / 로트 번호 / Número de lote / Партиден номер / Lot broj / Eränumero / Partijas numurs / Numār lot / Partii number / Partijos numeris / Številka serije
PN	Part Number / Teilenummer / Référence / Número de pieza / Codice parte / Onderdeelnummer / Número de peça / Αριθμός εξαρτήματος / Delenummer / Delnummer / Artikelnummer / Numer wydania / Číslo dielu / Číslo dílu / Cikkszám / Номер детали / Parça Numarası / 부품 번호 / Número da peça / Номер на част / Broj dijela / Osanumero / Daļas numurs / Număr piesă / Osa number / Dalies numeris / Številka dela
	Follow Instructions For Use / Gebrauchsanweisung befolgen / Se conformer aux instructions d'utilisation / Siga las instrucciones de uso / Seguire le istruzioni per l'uso / Volg de gebruiksaanwijzing / Siga as instruções de utilização / Ακολουθήστε τις οδηγίες χρήσης / Følg instruksjoner for bruk / Følg brugsanvisningen / Følj bruksanvisningen / Należy postępować zgodnie z instrukcją obsługi / Postupujte podľa návodu na použitie / Dodržujte pokyny k použití / Kövesse a használati útmutatót / Следуйте Инструкции по эксплуатации / Kullanma Talimatlarına Uyun / 사용 지침 준수 / Seguir as instruções de uso / Следвайте инструкциите за употреба / Slijedite upute za korištenje / Noudata käyttöohjeita / Ievērojiet lietošanas instrukciju / Urmați instrucțiunile de utilizare / Jārgaida kasutusuheniti / Vykdysite naudojimo instrukcijas / Upoštečajte navodila za uporabo
	Manufacturer / Hersteller / Fabricant / Fabricante / Produttore / Fabrikant / Fabricante / Κατασκευαστής / Producent / Producent / Tilverker / Wytwórca / Výrobca / Výrobce / Gyártó / Изготовитель / Üretici / 제 조 사 / Fabricante / Производител / Proizvodač / Tootja / Ražotājs / Producător / Tootja / Gamintojas / Proizvajalec

EC REP	Authorized Representative in the European Community / Autorisierter Repräsentant in der Europäischen Gemeinschaft / Représentant autorisé dans la Communauté européenne / Representante autorizado en la Comunidad Europea / Rappresentante autorizzato nella Comunità Europea / Geautoriseerde vertegenwoordiger in de Europese Gemeenschap / Representante autorizado na Comunidade Europeia / Εξουσιοδοτημένος Αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Autorisert representant i Europa / Autoriseret repræsentant i den Europæiske Union / Auktoriserad representant inom Europeiska unionen / Upoważniony przedstawiciel w Unii Europejskiej / Splnomocnený zástupca v Európskom spoločenstve / Autorizovaný zástupce pro Evropské společenství / Hivatalos képviselő az Európai Közösségekben / Уполномоченный представитель в Европейском сообществе / Avrupa Topluluğu Yetkili Temsilcisi / 유럽의 인가 대표자 / Representante autorizado na Comunidade Europeia / Упълномощен представител в Европейската общност / Ovlašteni predstavnik u Europskoj uniji / Valtuutettu edustaja Euroopan yhteisössä / Pilnvarotais pārstāvis Eiropas Kopienā / Représentant autorizat în Comunitatea Europeană / Volitatud esindaja Euroopa Ühenduses / Įgaliotasis atstovas Europos Bendrijoje / Pooblaščený zastopnik v Evropski skupnosti
AUS	Australian Sponsor Address / Adresse des australischen Verantwortlichen / Adresse du sponsor australien / Dirección del patrocinador australiano / Indirizzo sponsor australiano / Adres Australische sponsor / Endereço do patrocinador da Austrália / Διεύθυνση Πρωτοβητή στην Αυστραλία / Australsk sponsoradresse / Australsk sponsoradresse / Address till australiensisk sponsor / Adres australijskiego sponsora / Adresa sponsoru pre Austráliu / Adresa zastoupení pro Austrálii / Ausztráliei szponzor címe / Адрес спонсора в Австралии / Avustralya Sponsor Adresi / 호주 후원인 주소 / Endereço do patrocinador australiano / Австралийский адрес на спонсора / Adresa australiskog narucitelja / Australialaisen sponsorin osoite / Austrālijas sponsora adrese / Adresa sponsorului australian / Austrālija toetaja address / Austrālijos užsakovs adresas / Naslov avstralskega sponzorja
 	Separate Collection / Sonderabfall / Élimination séparée / Recogida separada / Raccolta differenziata / Gescheiden inzameling / Recolha separada / Εξχωριστή συλλογή / Separat innsamling / Separat opsamling / Separat insamling / Oddzielną zbiórka / Separovaný zber / Oddělený sběr / Elkülönített gyűjtés / Раздельный сбор / Аyrı Toplama / 분리수거 / Coleta separada / Разделно събиране / Zasebno prikupljanje / Erilliskeräys / Atseviška savākšana / Colectare separată / Eraldi kogumine / Atskiras surinkimas / Ločeno zbiranje
	Non-Sterile / Nicht steril / Non stérile / No estéril / Non sterile / Niet steril / Não Esterilizado / Μη αποστειρωμένο / Ikke steril / Ikke-sterilt / Osterilt / Niejałowe / Nesterilne / Nesterilni / Nem steril / Нестерильно / Steril deşildir / 비열균 / Não estéril / Нестерильно / Nesterilno / Epāsteriili / Nesterils / Nesteril / Mittesteriiline / Nesterilu / Nesterilno

	Medical Device under EU Legislation / Medizinprodukt nach EU-Gesetzgebung / Dispositif médical soumis à la législation de l'UE / Dispositivo médico sujeto a legislación de la UE / Dispositivo medico in base alla legislazione UE / Medisch apparaat onder EG-wetgeving / Dispositivo Médico sob a Legislação da UE / Ιατρική συσκευή βάσει της νομοθεσίας της ΕΕ / Medicinsk utstyr i følge EU-lovgivning / Medicinsk udstyr i henhold til EU-lovgivningen / Medicínsk enhet som oppfyller EU-bestämmelser / Wyrób medyczny objęty prawodawstwem UE / Zdravotnícka pomôcka podľa právnych predpisov EU / Zdravotnícký prostriedek podle právnych předpisů EU / EU-s jogszabályoknak megfelelő orvostechnikai készülék / Медицинское устройство в соответствии с законодательством ЕС / AB Yasa ları altında Tibbi Cihaz / EU 법률에 따른 의료 기기 / Dispositivo médico de acordo com a legislação da UE / Медицинско устройство според законодательството на ЕС / Medicinski proizvod u skladu sa zakonodavstvom EU-a / EU:n lainsäädännön mukainen lääkinnällinen laite / Mediciniska ierice saskaņā ar ES tiesību aktiem / Dispozitív medic conform legislației UE / Meditsiinisade EI-i õigusaktide kohaselt / Medicinos prietaisas pagal ES teisės aktus / Medicínski pripomoček v skladu z zakonodajo EU
	Contents / Inhalt / Contenu / Contenido / Contenuto / Inhoud / Conteúdo / Περιεχόμενα / Innhold / Indhold / Innehåll / Zawartość / Obsah / Obsah / Tartalom / Содержание / İçindekiler / 내용물 / Conteúdo / Съдържание / Sadržaj / Sisältö / Saturs / Conținut / Sisukord / Turinys / Vsebina
	Temperature limitation / Temperaturbegrenzung / Limites de température / Limite de temperatura / Limiti di temperatura / Temperaturslimitet / Limites de temperatura / Περιορισμοί θερμοκρασίας / Temperaturbegrensning / Temperaturbegrænsning / Temperaturbegrænsning / Granice temperatury / Teplotné rozmedzie od do / Teplotní rozmezí od do / Hőmérséklet-tartomány / Температурные ограничения / Sıcaklık kısıtlaması / 온도 제한 / Limites de temperatura / Температурно ограничение / Temperaturno ograničenje / Rajoitettu lämpötila / Temperatūras ierobežojums / Limitarea temperaturii / Temperaturipiirang / Temperatūros ribojimas / Omejeitev temperature
	Humidity limitation / Feuchtigkeitsbeschränkung / Limites d'humidité / Limite de humedad / Limiti di umidità / Vochtigheidsbeperking / Limite de humidade / Περιορισμός υγρασίας / Fuktighetsgrænser / Fugtighedsbegrænsning / Luftfuktighetsbegrænsning / Granice wilgotności / Obmedzenia wilkości / Omezení vlhkosti / Páratartalom korlátozása / Ограничения влажности / Nem kısıtlaması / 습도 제한 / Limites de umidade / Ограничение за влажност / Ograničenje vlažnosti / Kosteusrajoituis / Mitruma ierobežojums / Limitarea de umiditate / Niiskuse piirang / Drėgmės apribojimas / Omejeitev vlažnosti

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE / ÜBERBLICK ÜBER DIE SICHERHEIT UND DIE KLINISCHE LEISTUNG / RÉSUMÉ SUR LA SÉCURITÉ ET LES PERFORMANCES CLINIQUES / RESUMEN SOBRE SEGURIDAD Y RENDIMIENTO CLÍNICO / RIEPILOGO DELLE PRESTAZIONI DI SICUREZZA E CLINICHE / OVERZICHT VAN VEILIGHEID

EN KLINISCHE PRESTATIES / RESUMO DA SEGURANÇA E DO DESEMPENHO CLÍNICO / ΣΥΝΟΨΗ ΑΣΦΑΛΕΙΑΣ ΚΑΙ ΚΛΙΝΙΚΗΣ ΑΠΟΔΟΣΗΣ / SAMMENDRAG AV SIKKERHET OG KLINISK YTELSE / OPSUMMERING AF SIKKERHED OG KLINISK PRÆSTATION / ÖVERSİKT ÖVER SÄKERHET OCH KLINISKA PRESTANDA / PODSUMOWANIE BEZPIECZEŃSTWA I DZIAŁANIA KLINICZNEGO / SÚHRN BEZPEČNOSTI A KLINICKÉHO VÝKONU / SOUHRN BEZPEČNOSTI A KLINICKÉHO VÝKONU / A BIZTONSÁG ÉS A KLINIKAI TELJESÍTMÉNY ÖSSZEFOGLALÁSA / РЕЗЮМЕ БЕЗОПАСНОСТИ И КЛИНИЧЕСКИХ ХАРАКТЕРИСТИК / GÜVENLİK VE KLİNİK PERFORMANS ÖZETİ / 안전성 및 임상 성능 요약 / RESUMO DE DESEMPENHO CLÍNICO E DE SEGURANÇA / ОБОБЩЕНИЕ ЗА БЕЗОПАСНОСТТА И КЛИНИЧНАТА ЕФЕКТИВНОСТ / SAŽETAK SIGURNOSTI I KLINIČKE UČINKOVITOSTI / YHTEENVETO TURVALLISUUDESTA JA KLIINISESTÄ SUORITUSKYVYSTÄ / KOPSAVILKUMS PAR DROŠĪBU UN KLĪNISKO VEIKTSPĒJU / REZUMATUL SECȚIUNII DE SIGURANȚĂ ȘI PERFORMANȚĂ CLINICĂ / OHUTUSE JA KLIINILISTE OMADUSTE KOKKUVÕTE / SAUGUMO IR KLINIKINIO VEIKSMINGUMO SANTRAUKA / POVZETEK VARNOSTNIH INFORMACIJ IN KLINIČNIH LASTNOSTI

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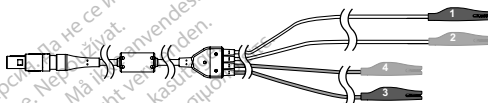
(1) - PSA Cable

(1) - Literature

[1] reverse side of black clip is marked V-

[2] reverse side of red clip is marked V+

[3] reverse side of black clip is marked A-

[4] reverse side of red clip is marked A+

Figure 1. PSA Cable, Model 6763

This literature is intended for use in hospital and clinical environments by professionals trained or experienced in device implant and/or follow-up procedures. For clinical benefits of the PSA, refer to the instructions for use provided with the PSA.

INTENDED USE

The PSA cable is an accessory indicated for use with PSA equipment distributed by Boston Scientific and is used for the transmission of sensing signals and pacing pulses for evaluation of lead placement characteristics during implant of pacemakers and defibrillators and for the support of other pacemaker functionality.

INSTRUCTIONS FOR USE

CAUTION: The PSA cable is shipped nonsterile. If the PSA cable is to be used in a sterile field, it must be actively sterilized before use.

Specifications and Operating Conditions

Table 1. PSA Cable Specifications

Specification	Value
Length (overall)	2.845 m (112 in.)
Clip protective covers (4)	Elastosil R 401, (silicone rubber)

Table 2. Operating and Storage Ranges

Parameter	Operating Range	Storage Range (3 days maximum for ranges beyond operating range)
Temperature range	10 to 40 °C (50 to 104 °F)	-40 to 70 °C (-40 to 158 °F)
Humidity range	20 to 95% non-condensing	5 to 95% non-condensing

Cleaning Instructions

CAUTION: Do not use an abrasive cloth or volatile solvents to clean any portion of the PSA cable.

- DO NOT immerse the PSA cable.
- DO NOT use an ultrasonic cleaner or automated washer/disinfectors.

At point of use:

When necessary, to prevent drying of any possible contaminants immediately after use, clean with a soft cloth dampened with sterile water.

Thorough Cleaning:

Thoroughly clean the cable with a soft cloth dampened with a mild cleaning solution such as green soap, green soap tincture (U.S. Pharmacopeia), sodium tetraborate decahydrate (such as Borax or equivalent), or alcohol-free hand soap. Prepare and use the cleaning agent per detergent manufacturer's instructions. Use a fresh soft cloth dampened with sterile water to remove residue. Towel-dry or air-dry the cable. Visually inspect the cable to ensure contaminants are removed. Repeat the cleaning step until all visible contaminants are removed.

NOTE: Discard the PSA cable any time surface cracks appear in the cable and/or the cable discolors, becomes visibly worn, or if labeling becomes unreadable.

Ethylene Oxide (EO) Sterilization Instructions

- Prior to sterilization, thoroughly clean the PSA cable as indicated in the "Cleaning Instructions" on page 2.
- Individually wrap each cable in no more than two layers of 1-ply polypropylene wrap (Kimberly-Clark Kimguard KC600 wrap or equivalent) ensuring no exposed surfaces of the device or seal in an EO permeable pouch.
- Follow the recommendations of the EO sterilization equipment manufacturer and allow the specified aeration time in Table 3 to fully elapse prior to use.

Table 3. EO Sterilization Parameters

Parameter	Value
Temperature	50 °C (122 °F) minimum and 60 °C (140 °F) maximum
Humidity	40% minimum and 80% maximum; non-condensing
EO dwell time	2 hours
EO concentration minimum	450 mg/L

Parameter	Value
Minimum aeration time	12 hours at 60 °C (140 °F)

Steam Sterilization Instructions

- Prior to sterilization, thoroughly clean the PSA cable as indicated in the “Cleaning Instructions” on page 2.
- Individually wrap each cable in no more than two layers of 1-ply polypropylene wrap (Kimberly-Clark Kimguard KC600 wrap, or equivalent) ensuring no exposed surfaces of the device.
- Follow the recommendations of the steam sterilization equipment manufacturer and allow the specified drying time in Table 4 to fully elapse prior to use.

Table 4. Steam Sterilization Parameters

Parameter	Value (gravity sterilizers)		Value (dynamic air-removal sterilizers)			
Temperature	132 °C (269.6 °F)	135 °C (275.0 °F)	132 °C (269.6 °F)	134 °C (273.2 °F)	135 °C (275.0 °F)	138 °C (280.4 °F)
Steam dwell time	15 minutes	10 minutes	4 minutes	18 minutes	3 minutes	4 minutes
Drying time ¹	30 minutes	30 minutes	30 minutes	30 minutes	16 minutes	16 minutes

¹ This represents the typical drying time. Once sterilized, ensure that the device is dry prior to removing it from the chamber.

NOTE: The Model 6763 PSA cable has been tested for 50 sterilization cycles; more than this number of sterilization cycles is not recommended.

Cable Connection

Refer to the operator's manual for the Pacing System Analyzer (PSA) for cable connection instructions.

Disposal

Use waste collection that is appropriate for electronic equipment.

Service

For service or technical assistance, contact Boston Scientific using the information on the back of this manual. Save packaging materials and containers to return device if service is needed.

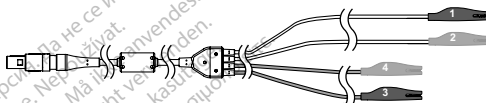
WARRANTY DISCLAIMER

Except as otherwise provided herein, Boston Scientific disclaims all express and implied warranties for this product, including without limitation any implied warranties of merchantability or fitness for a particular purpose. Boston Scientific's obligations under any warranty provided herein shall be limited strictly to replacement of the product. Buyer assumes all risk of loss or damages arising from use of this product.

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All Rights Reserved.
Boston Scientific (Australia) Pty Ltd
© 2020 Boston Scientific Corporation or its affiliates.