EXALT™

Monitor

User's Manual	1
Manual del usuario	47
Manuel d'utilisation	94
Bedienungsanleitung	142
Manuale dell'utente	190
Gebruikershandleiding	238
Manual do Utilizador	282

TABLE OF CONTENTS

INTRODUCTION	3
Where to Get Help	3
DEVICE DESCRIPTION	3
Model Numbers	3
Contents (M00542760 and M00542770)	3
Contents (M00542780 and M00542790)	3
Operating Principle	3
Display Specifications	4
Memory	4
Battery Specifications	4
Mounting	4
Ingress Protection Information	4
Power Adapter Electrical Specifications	4
Power Cord Specifications	4
Physical Specifications	5
User Information	5
INTENDED USE /INDICATIONS FOR USE	5
Clinical Benefit Statement	5
CONTRAINDICATIONS	5
WARNINGS	5
PRECAUTIONS	6
ADVERSE EVENTS	7
CONFORMANCE TO STANDARDS	7
Essential Performance Statement	7
HOW SUPPLIED	7
Device Details	7
Handling and Storage	7
Environmental Limits During Transportation, Use, and Storage	7
EXALT MONITOR COMPATIBILITY	8
OPERATIONAL INSTRUCTIONS	8
Physical EXALT Monitor Features	8
Additional Required Items	20
Installation, Calibration, and Servicing	20
Preparation	20
EXALT Monitor Setup	20

Attaching the EXALT Monitor to an IV Pole	21
Starting the EXALT Monitor	27
Powering On the Monitor	27
Checking the Battery Status	29
Adjusting the Brightness of the Video Image and Operating Light On/Off Button	29
Procedure	30
Using the EXALT Monitor in an Endoscopy Procedure	30
Notifications and Errors.....	30
Recovering from an EXALT Monitor Failure.....	32
Recovery from Unintended Shutdown.....	32
Recovery from an EXALT Monitor that Becomes Unresponsive	32
Recovery from EXALT Monitor Hardware Failure.....	32
Troubleshooting and Responding to Error Codes.....	32
Troubleshooting Chart	32
Post-Procedure.....	35
Reviewing, Exporting, Deleting, and Shutting Down after Procedure Completion	35
Shutdown of the EXALT Monitor	37
Charging the EXALT Monitor	39
Cleaning of the EXALT Monitor	39
Disposal	39
Information to Brief the Patient.....	39
WARRANTY	39

Rx ONLY

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

INTRODUCTION

This user manual describes how to appropriately and safely use, maintain, and troubleshoot the EXALT Monitor (hereafter called the Monitor).

The monitor is used with a Boston Scientific single-use endoscope. Refer to "EXALT Monitor Compatibility" for compatible Boston Scientific single-use endoscopes.

Where to Get Help

For technical support, ordering, service, and return authorization, contact Boston Scientific at 888-272-1001.

DEVICE DESCRIPTION

Model Numbers

EXALT Monitor	M00542760
EXALT Monitor (Additional Language)	M00542770
EXALT Monitor – No Mount	M00542780
EXALT Monitor (Additional Language) – No Mount	M00542790

Contents (M00542760 and M00542770)

- One (1) EXALT Monitor
- For M00542760 – One (1) North American Power Cable
- For M00542770 – One (1) EU Power Cable
- AC Power Adapter
- One (1) DVI cable
- One (1) SDI cable
- One (1) USB drive
- One (1) EXALT Monitor mount

Contents (M00542780 and M00542790)

- One (1) EXALT Monitor
- For M00542780 – One (1) North American Power Cable
- For M00542790 – One (1) EU Power Cable
- AC Power Adapter
- One (1) DVI cable
- One (1) SDI cable
- One (1) USB drive

Ensure the package contains the components listed above.

Operating Principle

The Monitor is an electronic device that:

- Receives video signals from a Boston Scientific single-use endoscope,
- Processes the video signals,
- Displays image on integrated display
- Outputs video images to a secondary video monitor when connected to AC mains,
- Outputs electrical signal(s) that interface with external image capture systems, and
- Saves and exports procedure images and videos.

The Monitor also controls the light transmitted by the tip of the single-use endoscope to illuminate the area of interest within the anatomy. Buttons on the Monitor touch screen enable the user to control the brightness level of the light. To use the Monitor, connect a Boston Scientific single-use endoscope to the Monitor. Optionally the Monitor can be connected to a secondary video monitor with a video cable when connected to AC mains. The Monitor provides direct visualization during an endoscopic procedure.

The Monitor interfaces with external media capture equipment via the Monitor's rear panel connectors. Image capture is initiated via a button on the Boston Scientific single-use endoscope or a button on the Monitor's user interface. The Monitor displays an indicator notifying the user that an image capture has been initiated.

Display Specifications

Display Type	12.3" PixelSense™ Display
Max Resolution	2736 x 1824 (267 PPI)
Orientation	Landscape

Memory

Storage	85 GB
---------	-------

Battery Specifications

Battery Life	1.5 hours
--------------	-----------

Mounting

Type	VESA Compatible Rear Panel Mount
------	----------------------------------

Ingress Protection Information

IP Rating	IP30
-----------	------

Power Adapter Electrical Specifications

Input voltage	100 to 240 VAC, 50/60 Hz
Input current	1.5A
Fuse rating	250 V, 3.15A
Output voltage	15 VDC
Output current	4.0 A

Note: The ground terminal of the power inlet in the external power supply is for Functional Earthing Only of the end use equipment; this end use equipment is Class II as per IEC 60601-1.

Power Cord Specifications

115 VAC Domestic (US)	Length—3.1 meters (10 feet) Voltage Rating—125 VAC Current Rating—10 amps Connector Type—IEC 60320 C13 Plug Type—NEMA 5-15P Hospital Grade
250 VAC International	Length—3.0 meters (9.8 feet) Voltage Rating—250 VAC Current Rating—10 amps Connector Type—IEC 60320 C13 Plug Type—CEE 7/7

Physical Specifications

Height	8.4 in (21.3 cm)
Width	13.6 in (34.5 cm)
Depth	2.3 in (5.8 cm)
Weight (unpackaged)	4.7 lbs (2.1 kg)

User Information

The Monitor and these instructions are intended for use by physicians trained in endoscopic procedures.

A thorough understanding of the techniques, principles, clinical applications and risks associated with endoscopic procedures is required before using the Monitor with the single-use endoscope.

INTENDED USE /INDICATIONS FOR USE

The EXALT Monitor is intended for use with a Boston Scientific single-use endoscope for endoscopic diagnosis, treatment, and video observations.

Clinical Benefit Statement

Provide visualization of and access to the airways and tracheobronchial tree to facilitate the diagnosis and management of a spectrum of inflammatory, infectious, and malignant diseases of the airway and lungs via a sterile single-use bronchoscope to eliminate the risk of exogenous micro-organisms that may be transmitted by ineffective reprocessing of reusable bronchoscopes.

The EXALT Monitor is a key component of bronchoscopy procedure to display live imaging data from EXALT Model B Single-Use Bronchoscope.

CONTRAINDICATIONS

None known.

WARNINGS

- Do not use with High Frequency ancillary devices.
- EXALT Monitor images must not be used as an independent diagnostic of any pathology. Physicians must interpret and substantiate any finding by other means and in the light of the patient's clinical characteristics.
- The EXALT Monitor is not to be used in a MRI environment.
- Read this user manual, the single-use endoscope instructions for use and the user manual for any accessories, including a secondary video monitor, and any external media capture devices before using the EXALT Monitor. Failure to follow any instructions or failure to heed any warnings or precautions may result in harm or injury to patient.
- Do not use the EXALT Monitor in the presence of flammable fluids and gases such as alcohol or oxygen. Doing so can result in fire and burns to the operator and patient.
- Do not perform diagnostic or therapeutic procedures without a clear and adequate video display. Doing so can result in adverse events.
- Placing the EXALT Monitor where other electrical medical devices can degrade the video image can delay the procedure and result in adverse events. In addition, placing the EXALT Monitor where it can degrade the performance of other equipment in the endoscopy suite due to EMI emissions can delay the procedure or result in adverse events. To ensure the EXALT Monitor displays a clear and adequate video image and does not degrade the performance of other equipment, locate the EXALT Monitor as described in Table 12, Table 13, Table 14, Table 15, and Table 16 in the Appendices. Verify operation in the endoscopy suite environment before starting a procedure. Follow ancillary equipment instructions for use to locate ancillary equipment.

- Using the EXALT Monitor without cleaning the exterior surfaces including touch screen user interface can expose the operator to biohazardous materials. To prevent exposure to biohazardous materials, clean the device between uses, following the procedure described in "Cleaning."
- If the EXALT Monitor is connected to an improperly grounded power supply, electrical leakage can result in electrical shock to the user. To avoid risk of electrical shock, this equipment must only be connected to a supply mains with protective earth.
- Do not touch connecting devices for electrical connections between the different components (such as signal input and output connections for video signals, data exchange, control circuits, etc.) and the patient at the same time. Doing so can result in electric shock to the patient.
- If the EXALT Monitor experiences an unintended shutdown or lock up during a procedure, follow the procedure described in "Recovering from an EXALT Monitor Failure." Failure to follow this recovery procedure after an EXALT Monitor failure can result in patient injury.
- No modification of this equipment is allowed.
- The use of accessories and cables other than those specified or supplied as spare parts from Boston Scientific may result in increased emissions or decreased immunity of the EXALT Monitor or single-use endoscope.
- Components connected to the EXALT Monitor by the user must be certified to the respective IEC standards (IEC 60601-1 for medical equipment, IEC 60950 for data processing equipment, and IEC 60065 for A/V equipment). In addition, the user must ensure the new configuration complies with the IEC 60601-1 standard.
- The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.
- Do not connect the output of any equipment to the EXALT Monitor's video outputs.
- Only USB flash drives should be connected to the EXALT Monitor's USB port. Boston Scientific has tested the included Kingston (DTSE9H or DTKN) 32 GB for compatibility. No other items should be connected via the USB port.
- For use in a professional healthcare environment.
- The external temperature of the Monitor may exceed 48°C (118°F) during charging. Surface temperatures over 48°C (118°F) may cause tissue burns if constant contact is made for a prolonged period of time. Always ensure that the operator does not rest on the Monitor during use. Also, it is always recommended to fully charge the monitor before beginning a procedure.

PRECAUTIONS

- Spilling liquids on the EXALT Monitor can damage it or cause it to shut down. Do not place liquids above or near the EXALT Monitor.
- Opening the device for repair purposes can damage the EXALT Monitor. The EXALT Monitor does not use operator-serviceable components. To prevent damage, do not access the EXALT Monitor device. Opening the EXALT Monitor voids any remaining warranty.
- Connecting an incompatible single-use endoscope to the EXALT Monitor can damage the EXALT Monitor. Only connect a single-use endoscope listed in the "EXALT Monitor Compatibility" section.
- Locate the EXALT Monitor appropriately to avoid accidentally pulling cable connections, which can result in disconnection and loss of visualization.
- If placing the EXALT Monitor on an IV pole, do not place an IV bag on the same IV pole.
- Before starting a procedure, ensure components such as the secondary video monitor and irrigation pump that support the EXALT Monitor and single-use endoscope are present and operational. Starting a procedure without the supporting components present and operational can prolong the procedure.

- Do not use cleaning solutions that contain long-life surfactants. Doing so can leave conductive residues on the contacts of the single-use endoscope connector receptacle. The conductive residues can lead to malfunctions of the EXALT Monitor.
- Use of a cardiac defibrillator while a connected single-use endoscope remains in a patient can damage the EXALT Monitor. To prevent damage to the EXALT Monitor when using a defibrillator, remove the single-use endoscope before using the defibrillator.
- Do not insert a wet connector into the EXALT Monitor receptacle as poor video performance or damage to the EXALT Monitor may result.
- Applied parts of other electrical medical equipment in application with this equipment have to be type BF. Hence, only connect a single-use endoscope listed in the “EXALT Monitor Compatibility” section.
- The batteries in the EXALT Monitor are not changeable and should not be removed or replaced.
- Pay attention to the battery symbol indicator on the EXALT Monitor. Recharge EXALT Monitor when the battery level is low.
- Use only 15 to 70 percent isopropyl alcohol in purified water solution for cleaning.

ADVERSE EVENTS

Please refer to single-use endoscope instructions for use.

CONFORMANCE TO STANDARDS

Essential Performance Statement

Per IEC 60601-1, the EXALT Monitor does not have any functions that would present an unacceptable risk if failure occurred.

HOW SUPPLIED

Device Details

Device is supplied non-sterile in a single-pack configuration. Inspect the Monitor and cables for damage. Do not use if package is opened or damaged. Do not use if labeling is incomplete or illegible. Do not use a component if it appears damaged.

Handling and Storage

Environmental Limits During Transportation, Use, and Storage

Table 1. Transportation, use, and storage environmental limits

Environmental Limits during Use	
Ambient temperature (°C)	10 to 33
Relative humidity (%)	30 to 85 (non-condensing)
Atmospheric pressure (hPa)	700 to 1090
Environmental Limits during Transport and Storage	
Ambient temperature (°C)	-40 to 70
Relative humidity (%)	10 to 90 (non-condensing)
Atmospheric pressure (hPa)	500 to 1090

Note: Use of the Monitor at an ambient temperature above the maximum rated operating temperature of 33°C can cause the Monitor to overheat. This can lead to reduced video frame rate or can cause the system to power off. If this is observed, remove the endoscope from the patient and unplug from the Monitor. Then ensure the Monitor is powered off and move the Monitor to a cooler location and allow time for the Monitor to return to standard operating temperature.

EXALT MONITOR COMPATIBILITY

All attached equipment must comply with its applicable IEC standards. The Monitor is compatible with:

- EXALT Model B Single-use Bronchoscope (M00542710, M00542711, M00542730, M00542731, M00542750, M00542751).
- Medical grade video monitors with standard video signal inputs (at a minimum: DVI 1.0 and/or 3G-SDI). Monitor incorporates standard video signal outputs (i.e. DVI 1.0, 3G-SDI) to allow connection to a secondary video monitor. Monitor was tested for compatibility with the NDS Radiance G3 26" medical grade monitor at the following factory default settings:

Note: If planning to use a secondary video monitor, ensure the Monitor is connected to AC mains, as this is required.

Input: DVI – 1920 x 1080p @ 60.00Hz

Parameter	Setting
Brightness	50
Contrast	50
Sharpness	0
Overscan	0
Gamma	2.2
Color Temperature	Default
Red	50
Green	45
Blue	46
Saturation	50
Hue	50
Video Level	Normal
Color Correction	Bypass
Backlight Control	Off

- External media capture devices that have a 9-pin RS-232 compliant DB9 signal input connector and/or a TRS jack with a maximum rating of 5 VDC. Monitor was tested for compatibility with media capture devices listed in the following table. Call Boston Scientific for system specific setup instructions.

Image Capture Device	Software Version
Medicapture USB-300	170510
TEAC™ UR-4MD	a1.23
ProVation™ MD by ProVation Medical	5.0.410.23

OPERATIONAL INSTRUCTIONS

Physical EXALT Monitor Features

Figures 1, 2, 3, 4 and 5 and Table 2 illustrate and describe EXALT Monitor's top, front, right, bottom, and back panel features.

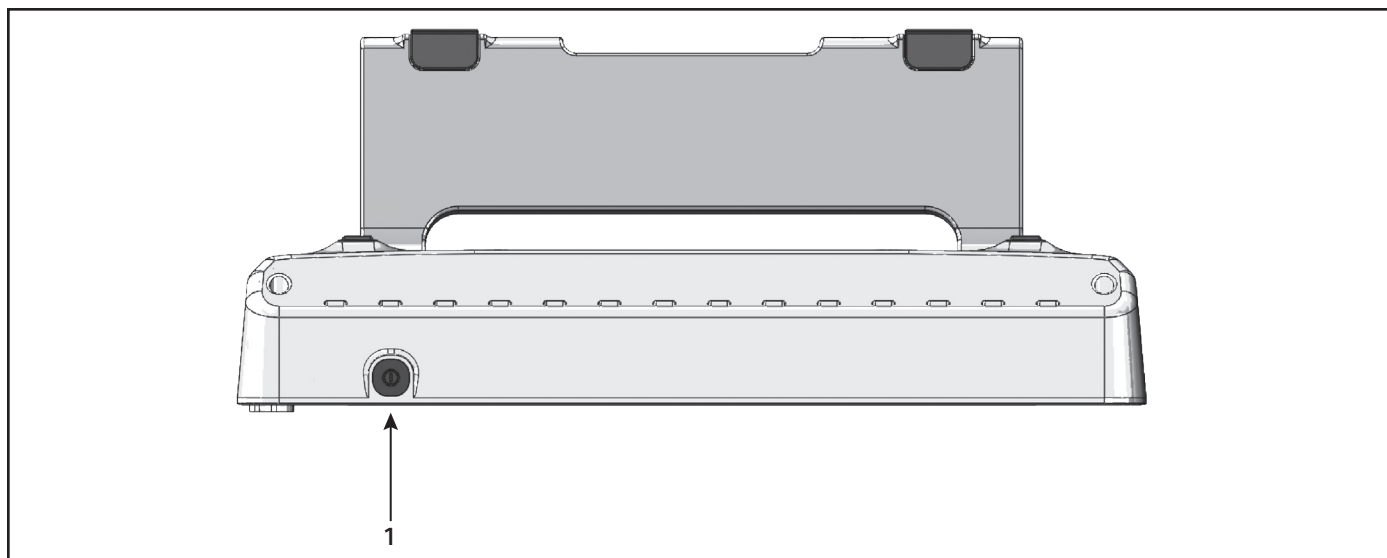
Figure 1. Top Panel Features

Figure 2. Front Panel Features

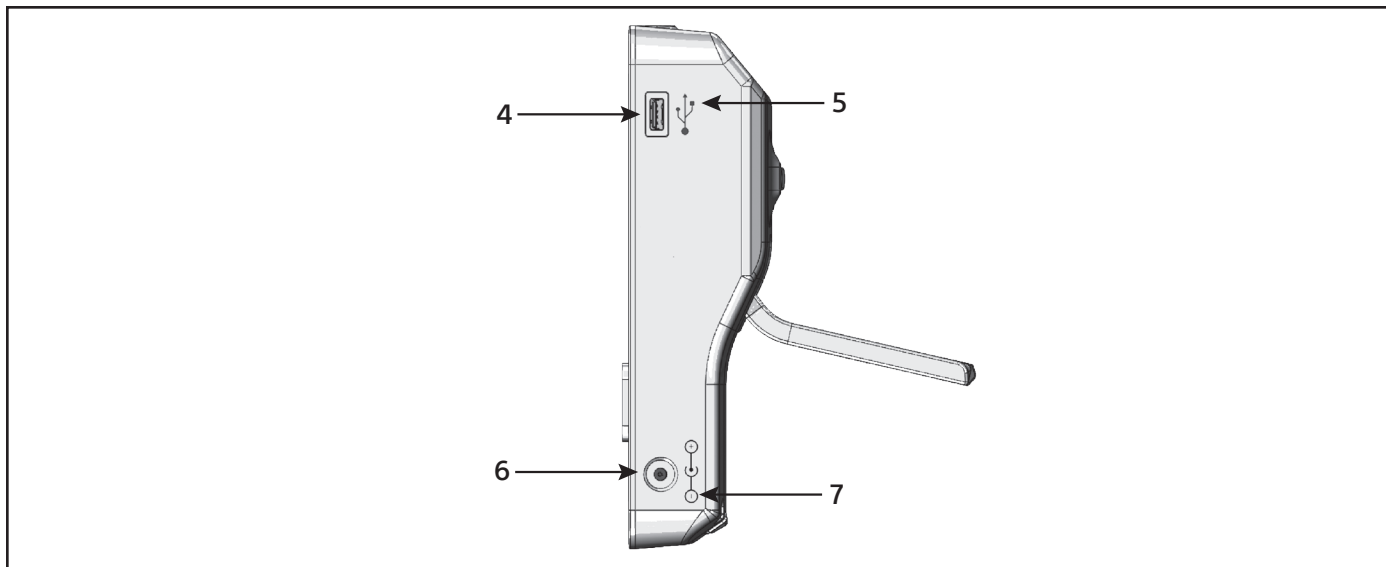
Figure 3. Right Panel Features

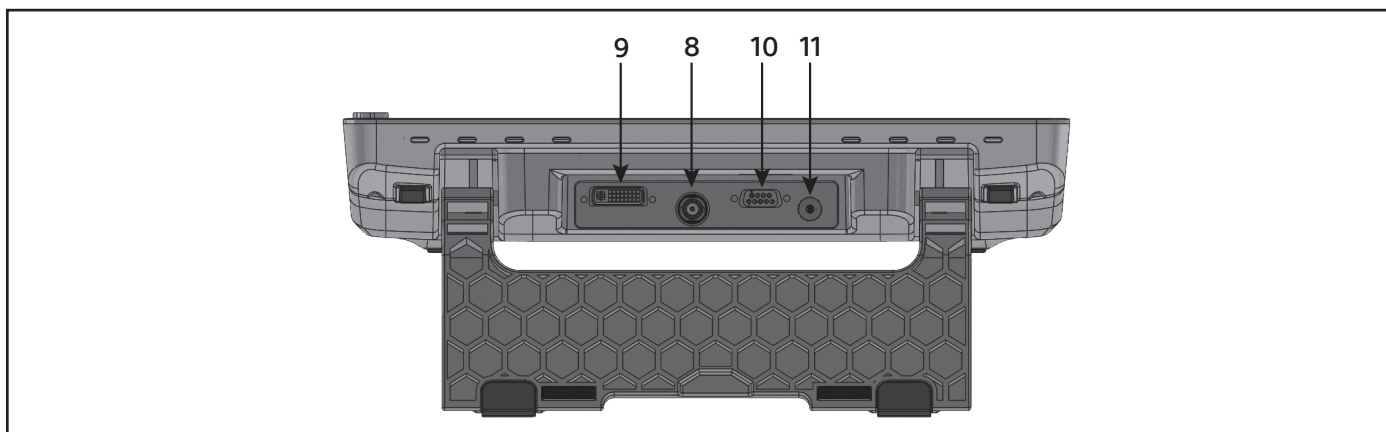
Figure 4. Bottom Panel Features

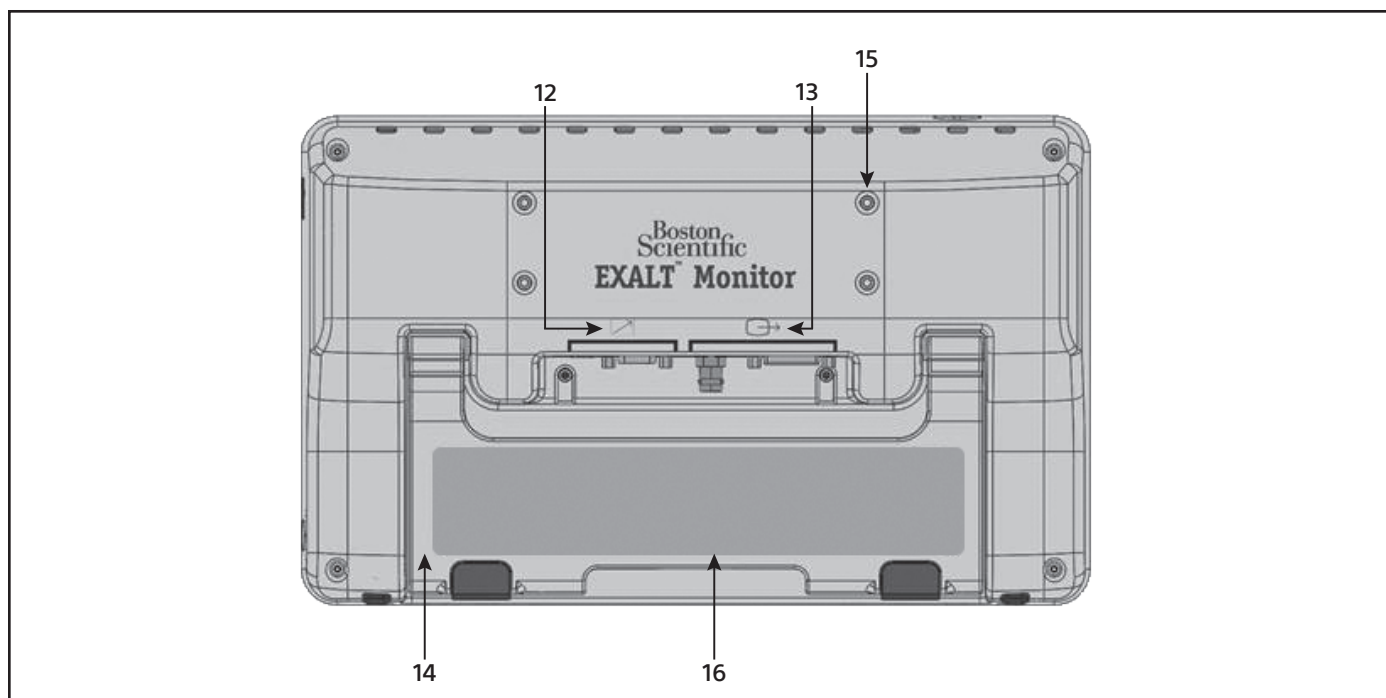
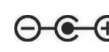
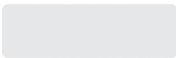
Figure 5. Back Panel Features

Table 2. Description of Features

Feature Number	Feature Illustration	Description	Location
1		Power Button – Pushing the power button alternates between powering up and powering down the EXALT Monitor.	Top Panel
2		Connection Cable Receptacle – User plugs the umbilical cable from the single-use endoscope into the receptacle during setup.	Front Panel
3		BF Symbol – Indicates user must use BF applied parts.	Front Panel
4		USB Port – Inserting a USB flash drive into the USB port gives users the ability to export images and videos from the EXALT Monitor.	Right Panel
5		USB Port Symbol – Indicates a USB port.	Right Panel
6		Power Inlet – Connects EXALT Monitor to AC mains through the provided AC Power Adapter.	Right Panel
7		Power Inlet Symbol – Indicates a center positive DC power connection.	Right Panel

Feature Number	Feature Illustration	Description	Location
8		SDI Connector – Connector for SDI Video Output. Note: Video outputs are only active when the EXALT Monitor is connected to AC mains.	Bottom Panel
9		DVI Connector – Connector for DVI Video Output . Note: Video outputs are only active when the EXALT Monitor is connected to AC mains.	Bottom Panel
10		RS232 DB9 Connector – Image capture output connector for connection with external media capture devices. To configure system for connection call Boston Scientific for support.	Bottom Panel
11		3.5mm TRS Connector Jack – Image capture output connector for connection with external media capture devices. To configure system for connection call Boston Scientific for support.	Bottom Panel
12		Remote Control Lead Symbol – Indicates remote control lead.	Back Panel
13		Video Output Symbol – Indicates video outputs.	Back Panel
14	N/A	Stand – Used to place the EXALT Monitor on a solid surface.	Back Panel
15		Tapped Holes – Used to attach Monitor mount.	Back Panel
16		Label – Provides regulatory and manufacturing information.	Back Panel

EXALT Monitor Battery Display

The EXALT Monitor is a battery powered device. The battery level is indicated using an icon displayed in the upper right corner of each screen. In addition to the battery icon, a battery percentage is displayed. Table 3 the battery display symbology of the EXALT Monitor.

Table 3. Battery Display Symbology

Battery Percentage	Symbol Displayed When EXALT Monitor Is Not Charging	Symbol Displayed When EXALT Monitor Is Charging
91-100%		
64-90%		
38-63%		
20-37%		
0-19%		

A charging symbol is also displayed next to the battery icon to indicate that the EXALT Monitor is connected to AC mains and is charging.

EXALT Monitor Home Screen Features

Figure 6 illustrates the Home Screen of the EXALT Monitor. The Scope Connect graphic located in the center of the screen indicates that a single-use endoscope needs to be connected to begin the procedure. Table 4 describes additional features found on the Home Screen.

Figure 6. EXALT Home Screen

Table 4. Home Screen Features

Feature Illustration	Description
	Procedure History Button – Pressing the procedure history button results in navigation to the procedure history screen.
	Settings Button – Pressing the settings button results in navigation to the settings screen.
	Shutdown Button – Pressing the shutdown button.

EXALT Monitor Procedure Mode Screen Features

Figure 7 and Table 5 illustrate and describe the features located on the Procedure Mode Screen of the EXALT Monitor.

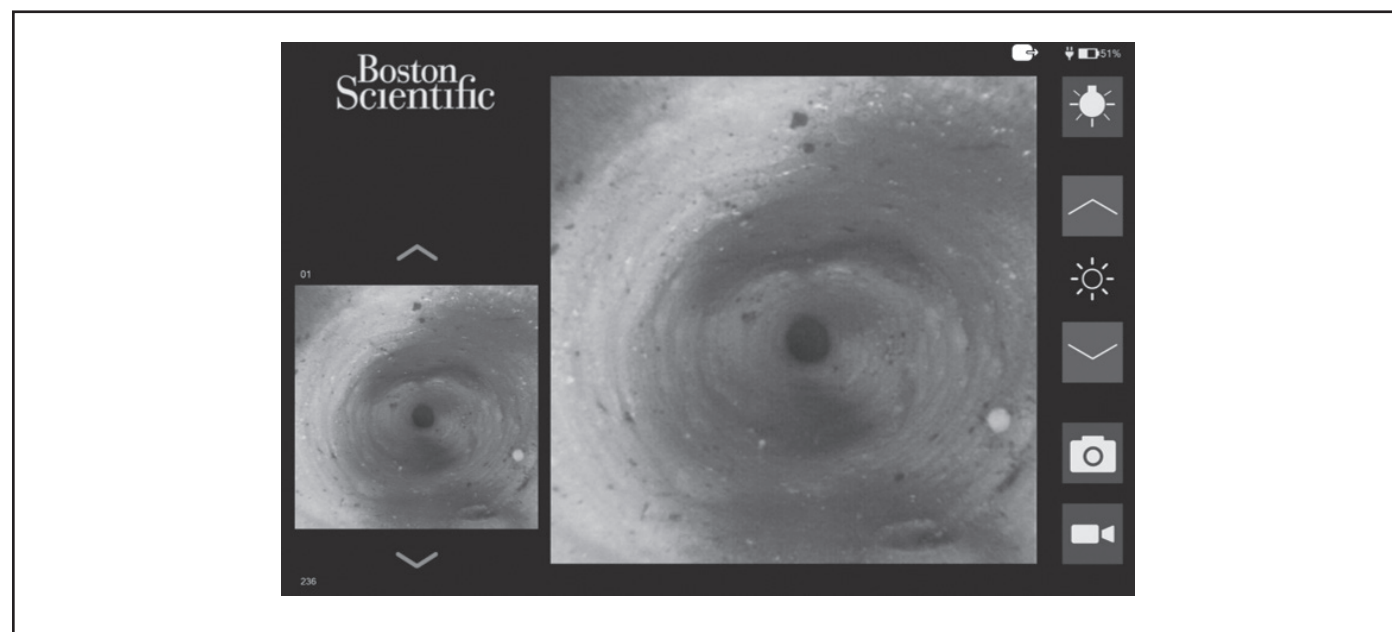
Figure 7. Procedure Mode Screen of the EXALT Monitor

Table 5. Procedure Mode Screen Features

Feature Illustration	Description
	Video Output Enabled Symbol - Indicates that video outputs are enabled. EXALT Monitor must be connected to AC mains to enable video outputs.
	Video Output Disabled Symbol - Indicates that video outputs are disabled. EXALT Monitor must be connected to AC mains to enable video outputs.
	LED Control Button - Indicates that endoscope light source is ON. Pressing will turn endoscope light source OFF.
	LED Control Button - Indicates that endoscope light source is OFF. Pressing will turn endoscope light source ON.
	Brightness Up Button – Pressing increases the image brightness.
	Brightness Down Button – Pressing decreases the image brightness.

Feature Illustration	Description
	Image Capture Button – Pressing captures an image. A thumbnail of the image is displayed on the left side of the screen to indicate successful image capture.
	Video Record Button – Pressing begins video recording. A video thumbnail is displayed on the left side of the screen to indicate successful video capture. Note: Individual video files are limited to 10 minutes in length. Multiple video files can be recorded in a single procedure.
	Video Stop Button – Pressing stops video recording.
	Video Thumbnail – Pressing opens the video on the File Review Screen.
N/A	Image Thumbnail – Pressing opens the image on the File Review Screen.

EXALT Monitor Procedure History Screen Features

Figure 8 and Table 6 illustrate and describe the features located on the Procedure History Screen of the EXALT Monitor.

Note: The EXALT Monitor will only display up to 72 thumbnails within each procedure. Additional images and videos can be captured during a single procedure, but they must be exported via USB in order to be viewed.

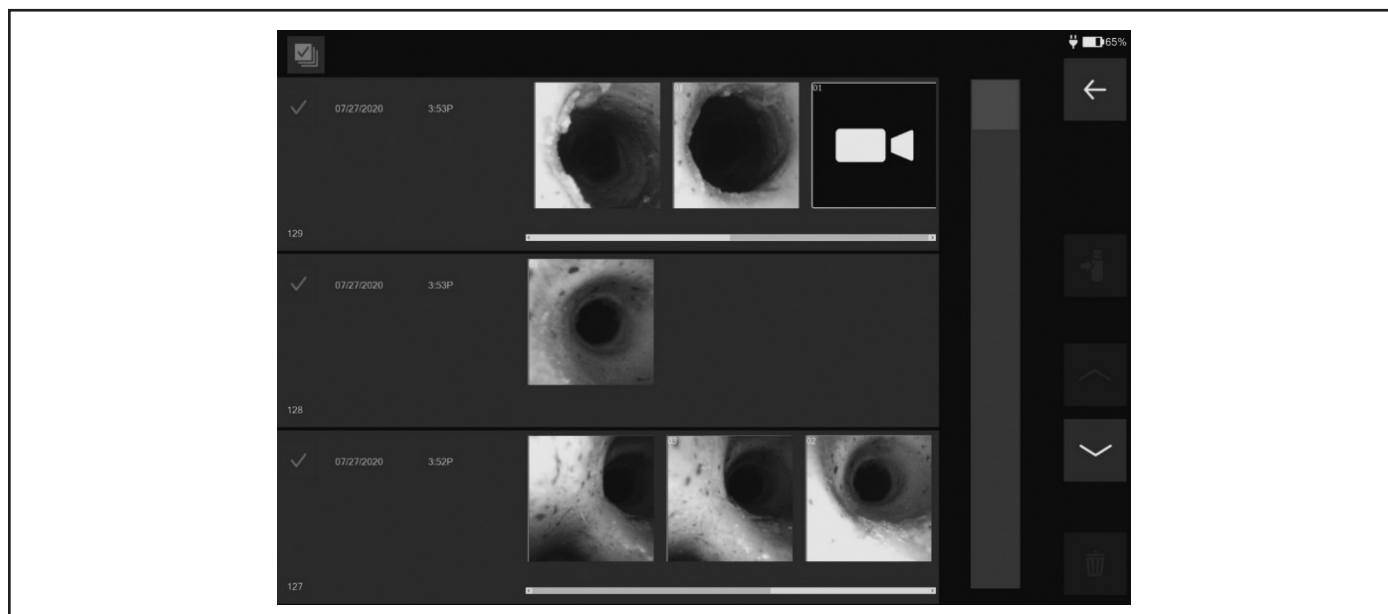
Figure 8. Procedure History Screen

Table 6. Procedure History Screen Features

Feature Illustration	Description
	Back Button – Pressing navigates back to the Home Screen.
	Procedure History Selection – Pressing selects all procedures on the procedure screen.
	Procedure History Deselect – Pressing deselects all procedures on the procedure history screen.
	Procedure Selection Button – Pressing selects all images/videos in procedure.
	Procedure Deselection Button – Pressing deselects all images/videos in procedure.
	USB Export Button – Pressing exports selected images and videos. Export option is only available when a USB flash drive is connected to the EXALT Monitor and images or videos are currently selected.
	Procedure Up Arrow – Pressing scrolls the procedure list up one row. Holding for 1.5 seconds initiates repeat mode. During repeat mode, procedure list will scroll up 1 row every 300 ms. This function is not available if the first procedure is currently being displayed on the screen.
	Procedure Down Arrow – Pressing scrolls the procedure list down one row. Holding for 1.5 seconds initiates repeat mode. During repeat mode, procedure list will scroll down 1 row every 300 ms. This function is not available if the last procedure is being displayed on the screen.
	Deletion Button – Pressing deletes selected images and videos. Delete option is only available when images or videos are currently selected.
	Video Thumbnail – Pressing opens the video on the File Review Screen.
N/A	Image Thumbnail – Pressing opens image on the File Review Screen.

EXALT Monitor Review Mode Screen Features

Figure 9 and Table 7 illustrate and describe the features located on the Review Mode Screen of the EXALT Monitor.

Note: The EXALT Monitor will only display up to 72 thumbnails within each procedure. Additional images and videos can be captured during a single procedure, but they must be exported via USB in order to be viewed.

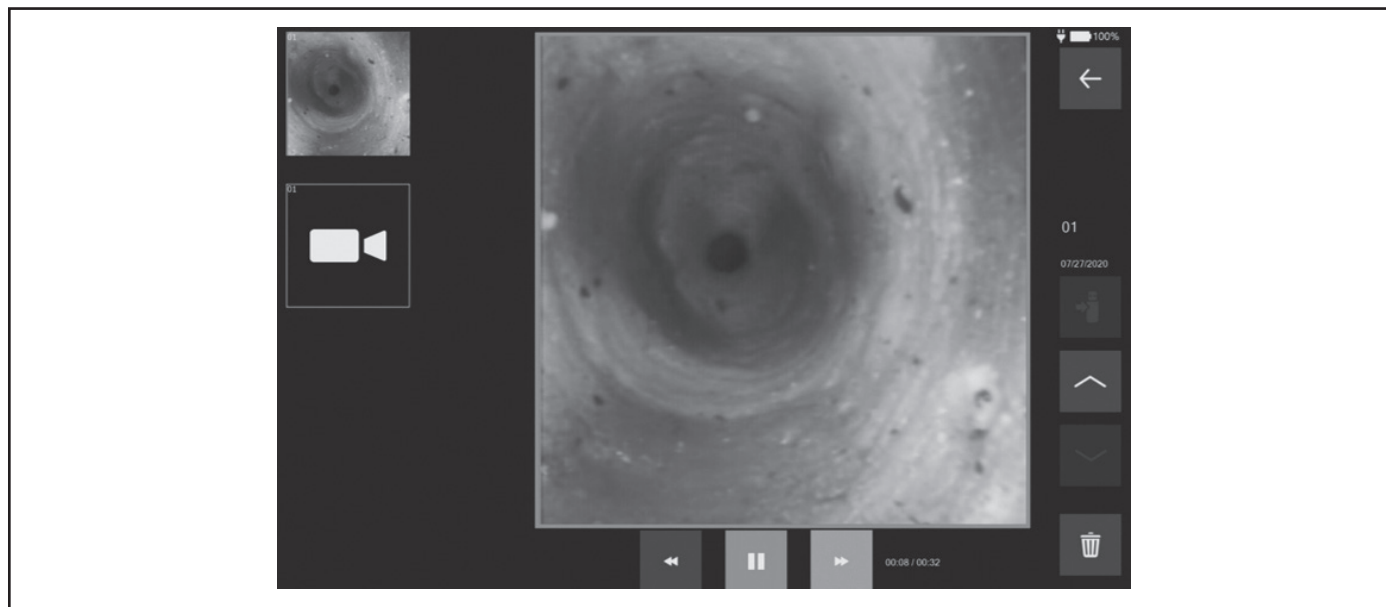
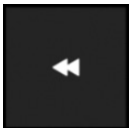
Figure 9. Review Mode Screen

Table 7. Image Review Screen Features

Feature Illustration	Description
	Back Button – Pressing navigates back to the Procedure History Screen.
	USB Export Button – Pressing exports selected image/video. Export option is only available when a USB flash drive is connected to the EXALT Monitor.
	Procedure Up Arrow – Pressing selects the previous image/video in the procedure. This function is not available if the first image/video in the procedure is already selected.
	Procedure Down Arrow – Pressing selects the next image/video in the procedure. This function is not available if the last image/video in the procedure is already selected.
	Deletion Button – Pressing deletes selected images and videos. Delete option is only available when images or videos are currently selected.

Feature Illustration	Description
	Fast Forward Button - When the video is in playback mode, pressing will cause the video to forward faster. Successive presses cause the frame rate to increase. The speed of play is indicated by X2, X4 or X10 being displayed within the border of this button. When the video is stopped, pressing will cause the next video frame to be displayed.
	Play Button – Pressing causes a paused video to play.
	Pause Button – Pressing causes a video to be paused when in playback mode.
	When the video is in playback mode, pressing will cause the video to play in reverse. Successive presses cause the frame rate to increase. The speed of play is indicated by X2, X4 or X10 being displayed within the border of this button. When the video is stopped, pressing will cause the previous frame to be displayed.
	Video Thumbnail – Pressing opens the video on the File Review Screen.
N/A	Image Thumbnail – Pressing opens image on the File Review Screen.

EXALT Monitor Settings Screen Features

Figure 10 and Table 8 illustrate and describe the features located on the Settings Screen of the EXALT Monitor.


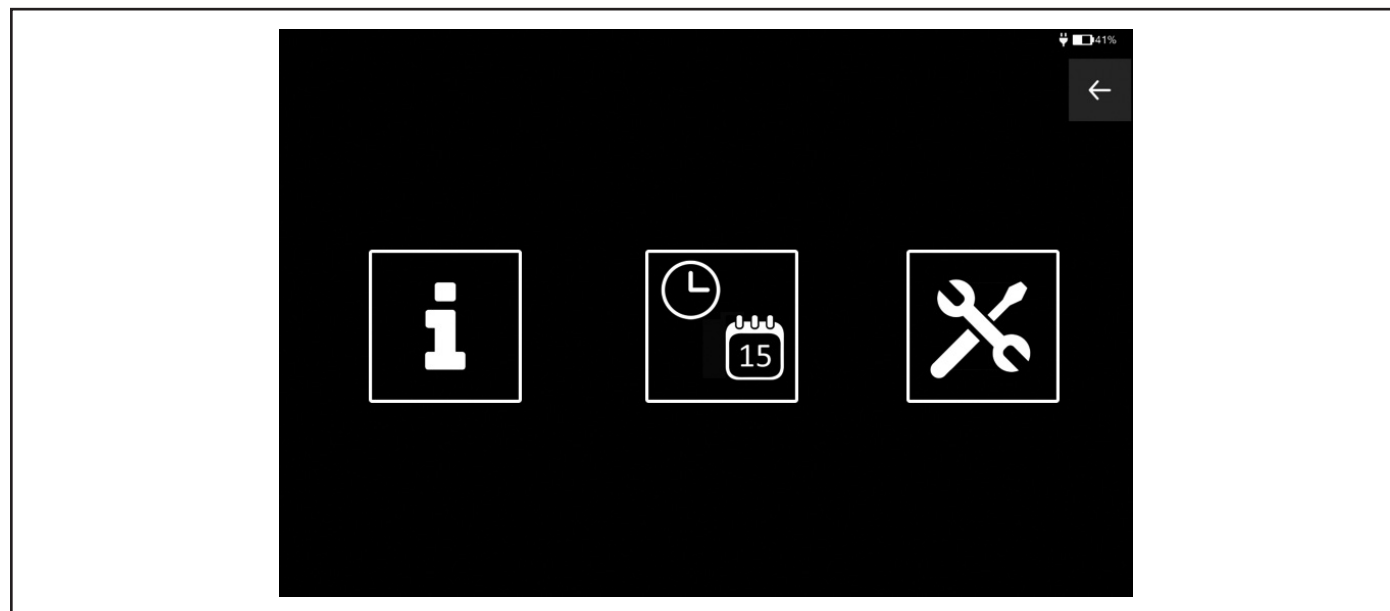
Figure 10. Settings Screen

Table 8. Settings Screen Features

Feature Illustration	Description
	Back Button – Pressing navigates back to the Home Screen.
	Information Button – Pressing displays information about the EXALT Monitor.
	Set Date and Time Button – Pressing navigates to the Set Date and Time Screen.
	Service Panel Button – Pressing navigates to the Service Panel login screen. Only BSC personnel can access the Service Panel of the EXALT Monitor. Users should not attempt to access the Service Panel.

EXALT Monitor About Screen

The software version of the EXALT Monitor is displayed on the About Screen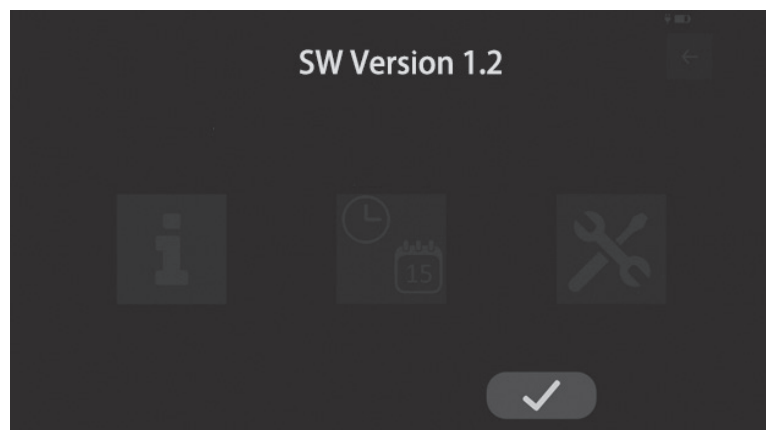

EXALT Monitor Set Date and Time Screen

Figure 11 and Table 9 illustrate and describe the features located on the Set Date and Time Screen of the EXALT Monitor.

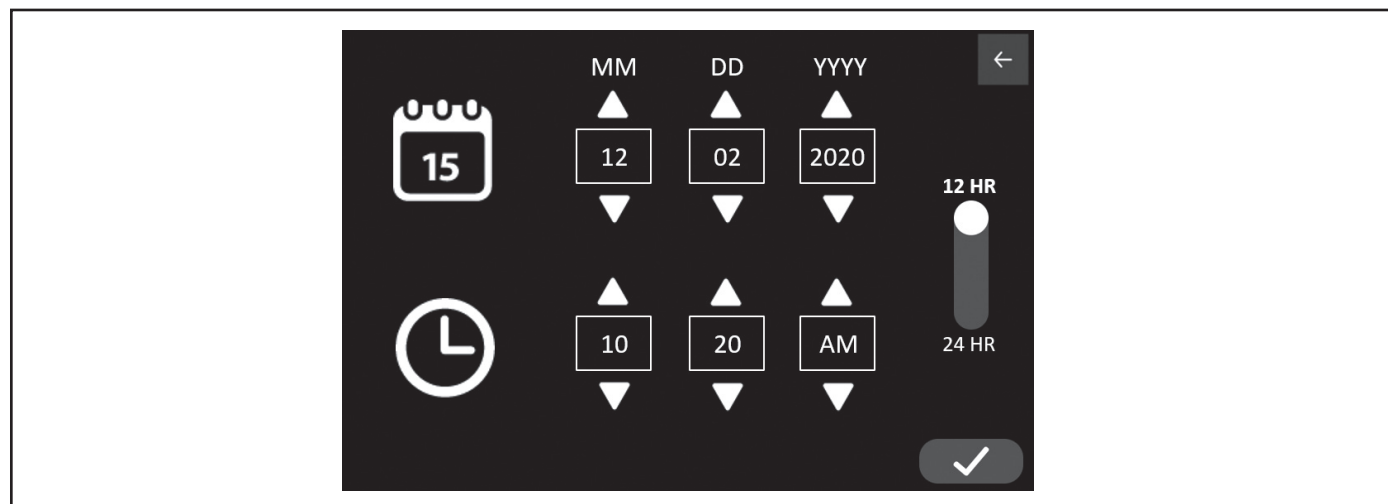
Figure 11. Set Date and Time Screen

Table 9. Set Date and Time Screen Features

Feature Illustration	Description
	Back Button – Pressing navigates back to the Settings Screen.
	12/24 HR Clock Button – Pressing toggles between 12 hour clock setting and 24 hour clock setting. Note: The EXALT Monitor does not automatically adjust for daylight savings.
	Calendar Icon – Indicates that buttons to the right of the icon control date setting.
	Clock Icon – Indicates that the buttons to the right of the icon control the time setting.
	Up Arrow – Pressing increases year, month, day, hour, or minute. Toggles between AM and PM. The AM/PM toggle is only available when in 12 hr clock mode.
	Down Arrow – Pressing decreases year, month, day, hour, or minute. Toggles between AM and PM. The AM/PM toggle is only available when in 12 hr clock mode.
	Confirmation Button – Pressing confirms date and time settings.

Additional Required Items

Items to be used with EXALT Monitor:

- Boston Scientific single-use endoscope

Installation, Calibration, and Servicing

The EXALT Monitor does not require routine maintenance and calibration. Periodically check the power cord assembly for damage to the insulation or connectors. In the event that the EXALT Monitor requires repair or replacement, contact Boston Scientific. No service shall be performed while the EXALT Monitor is in use with a patient.

Preparation

EXALT Monitor Setup

Upon receiving the EXALT Monitor, complete these initial setup steps:

1. Inspect the Monitor and its components to ensure that they are not damaged.
2. Clean the Monitor following the instructions in the “Cleaning” section.
3. Place the EXALT Monitor on a stable, level surface or attach the EXALT Monitor to an IV pole using the Monitor mount.
4. If desired, use the DVI or SDI cable, supplied by Boston Scientific, to connect one of the video output connectors of the EXALT Monitor to one of the video input connectors of a secondary video monitor.

Note: EXALT Monitor must be connected to AC mains in order to activate video outputs. See the “EXALT Monitor Compatibility” section.

Note: Connecting the EXALT Monitor to AC mains while powered on will result in the Monitor screen flashing momentarily. This is not an indication of an issue with the Monitor. To avoid this, ensure the Monitor is connected to AC mains prior to powering on the device.

5. If desired, connect the EXALT Monitor to an external image capture device (see the “EXALT Monitor Compatibility” section for additional details).
6. Power on the EXALT Monitor using the steps outlined in the “Starting the EXALT Monitor” section.

Attaching the EXALT Monitor to an IV Pole

Note: Do not place an IV bag on the same IV pole as the EXALT Monitor.

Note: The EXALT Monitor was tested for compliance to IEC 60601-1 using Pryor 135T IV pole.

Note: The Monitor mount is compatible with IV pole diameters between $\frac{3}{4}$ " and $1\frac{1}{2}$ ".

If desired, the EXALT Monitor can be mounted on a 60601-1 compliant IV pole using these steps:

1. Ensure the kickstand of the EXALT Monitor is in the closed position.
2. Place the EXALT Monitor screen-side down on a surface that will not damage the screen.
3. Align the Monitor mount's screws (Figure 12) with the tapped holes on the back of the EXALT Monitor (Figure 13).

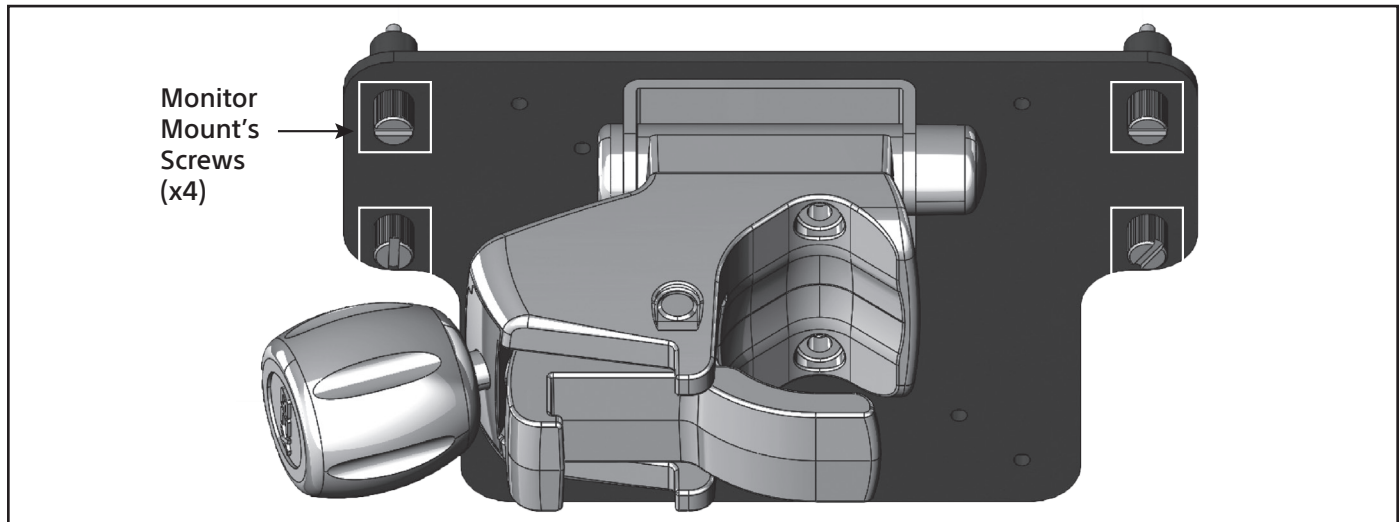
Figure 12. Align Monitor Mount's Screws

Monitor
Tapped
Holes
(x4)

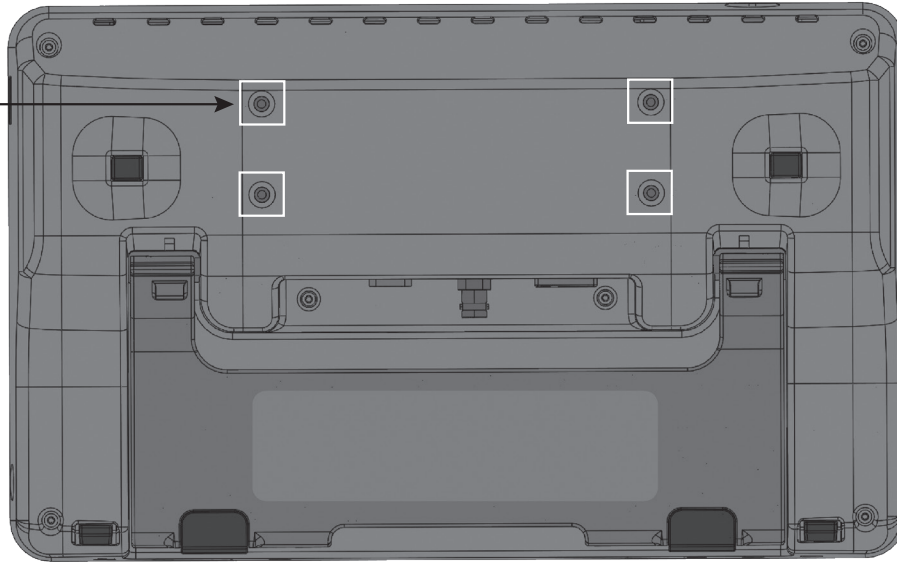
Figure 13. Monitor Tapped Holes

Note: The Monitor mount can be attached in 2 configurations. Figure 14 shows configuration 1. This configuration allows for access to the video outputs of the EXALT Monitor and provides greater downward tilt of the EXALT Monitor when attached to an IV pole. Figure 15 shows configuration 2, which provides greater upward tilt.

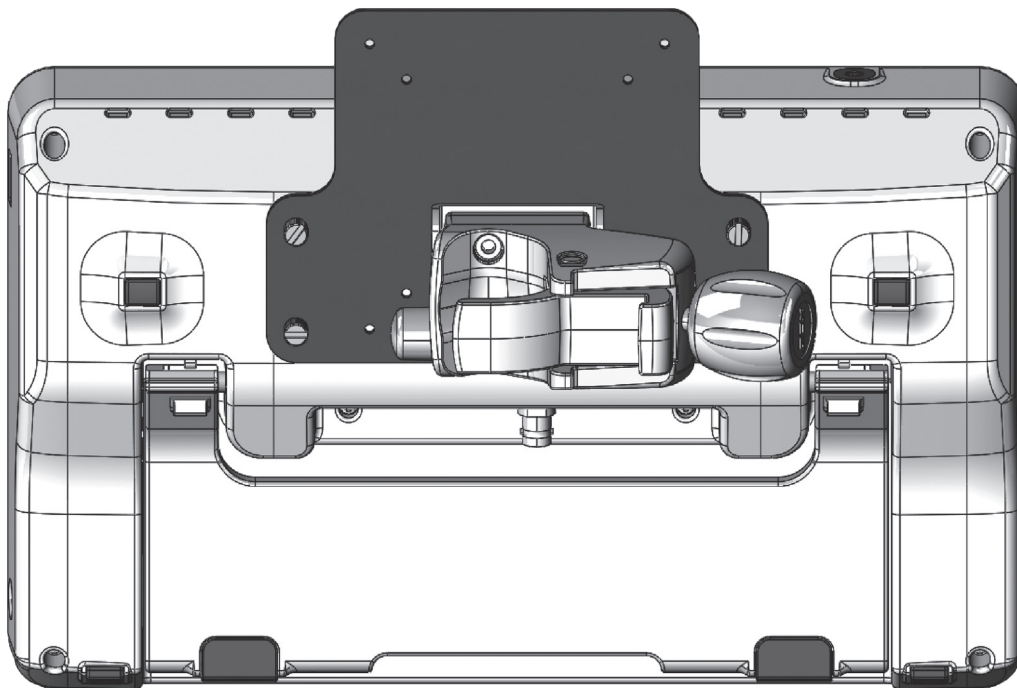
Figure 14. Monitor Mount Configuration 1

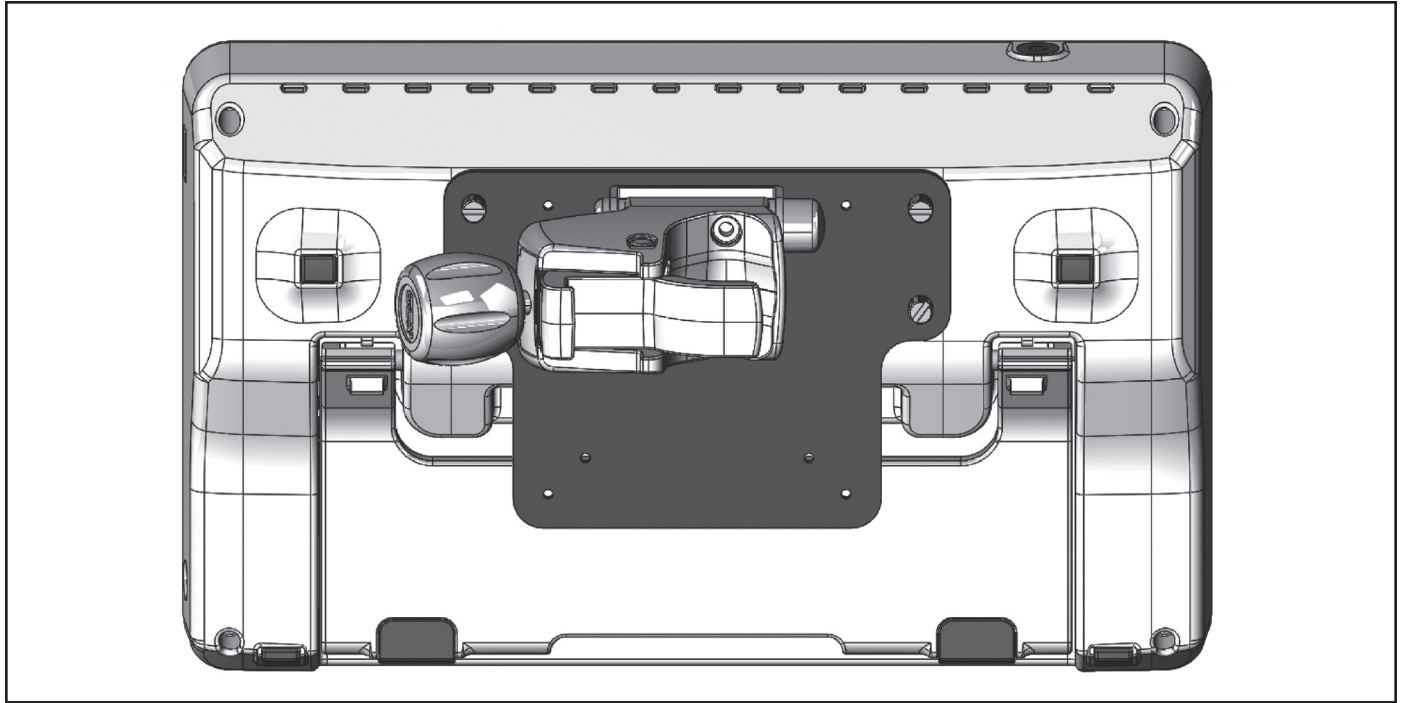
Figure 15. Monitor Mount Configuration 2

4. Hand tighten the screws until the mount is securely attached to the EXALT Monitor.
5. Rotate the handle of the clamp (Figure 16) to loosen the clamp.

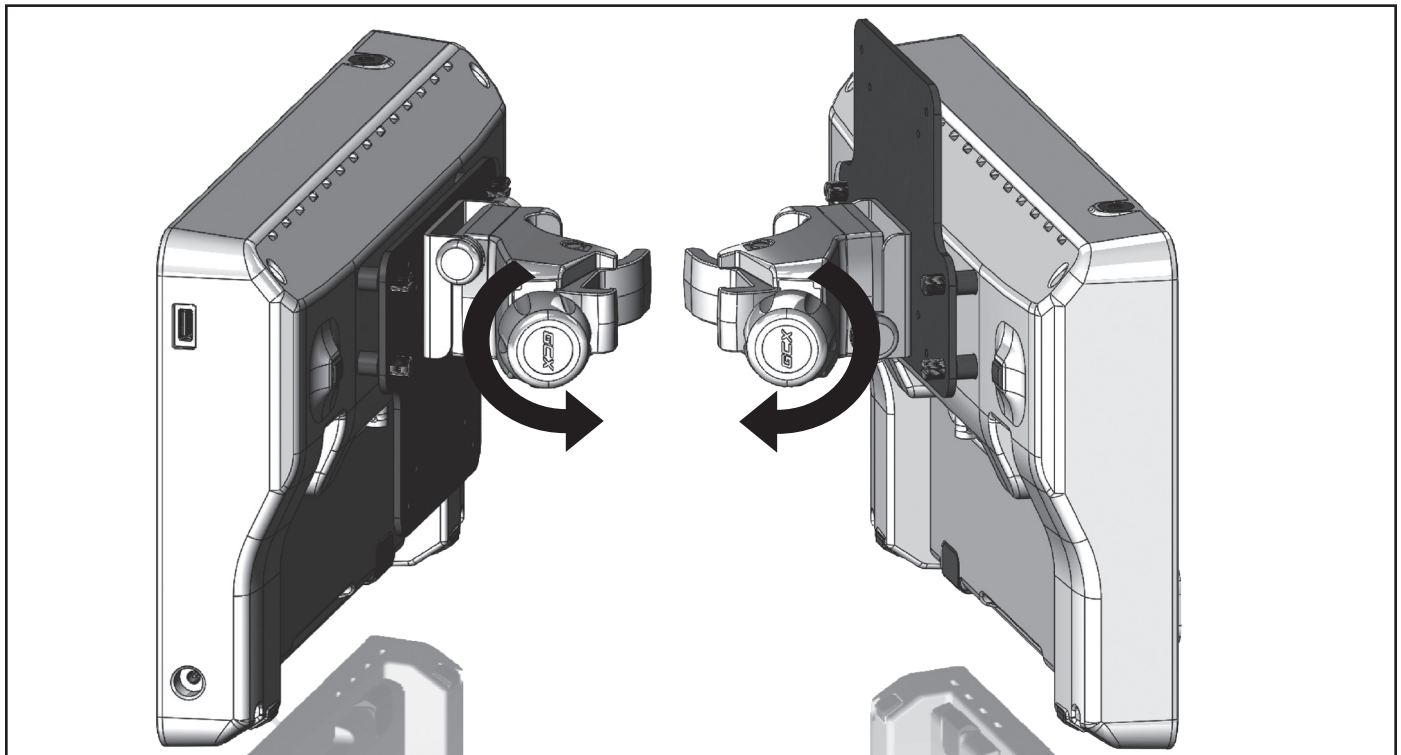
Figure 16. Rotate Clamp

6. Place the clamp around an IV pole.
7. Rotate the handle of the clamp in the opposite direction to tighten the clamp until the clamp securely grips the IV pole.

If desired, the EXALT Monitor can be mounted on a VESA mount using these steps:

1. Ensure the kickstand of the EXALT Monitor is in the closed position.
2. Open the swivel mechanism of the IV pole clamp 90° as shown in Figure 17.

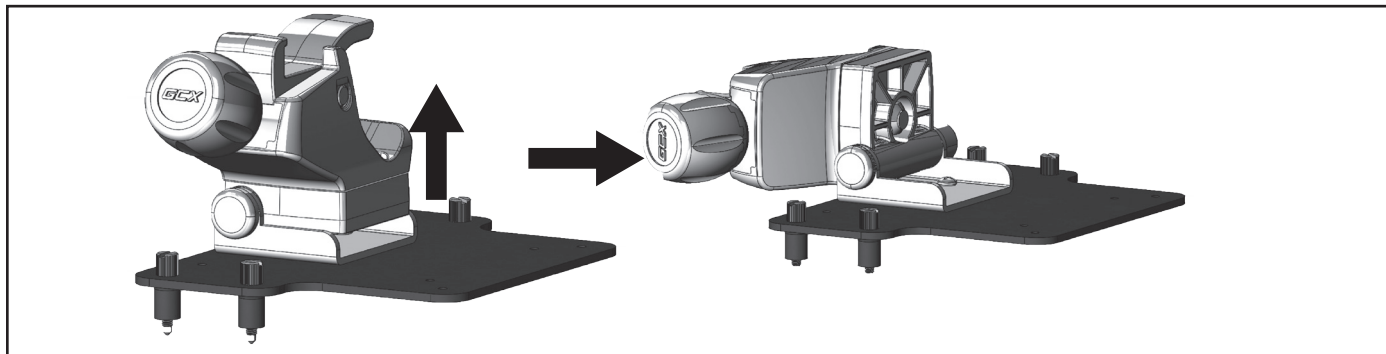
Figure 17. Open Swivel Mechanism

3. Using a #2 Philips head screwdriver, remove the screws shown in Figure 18.

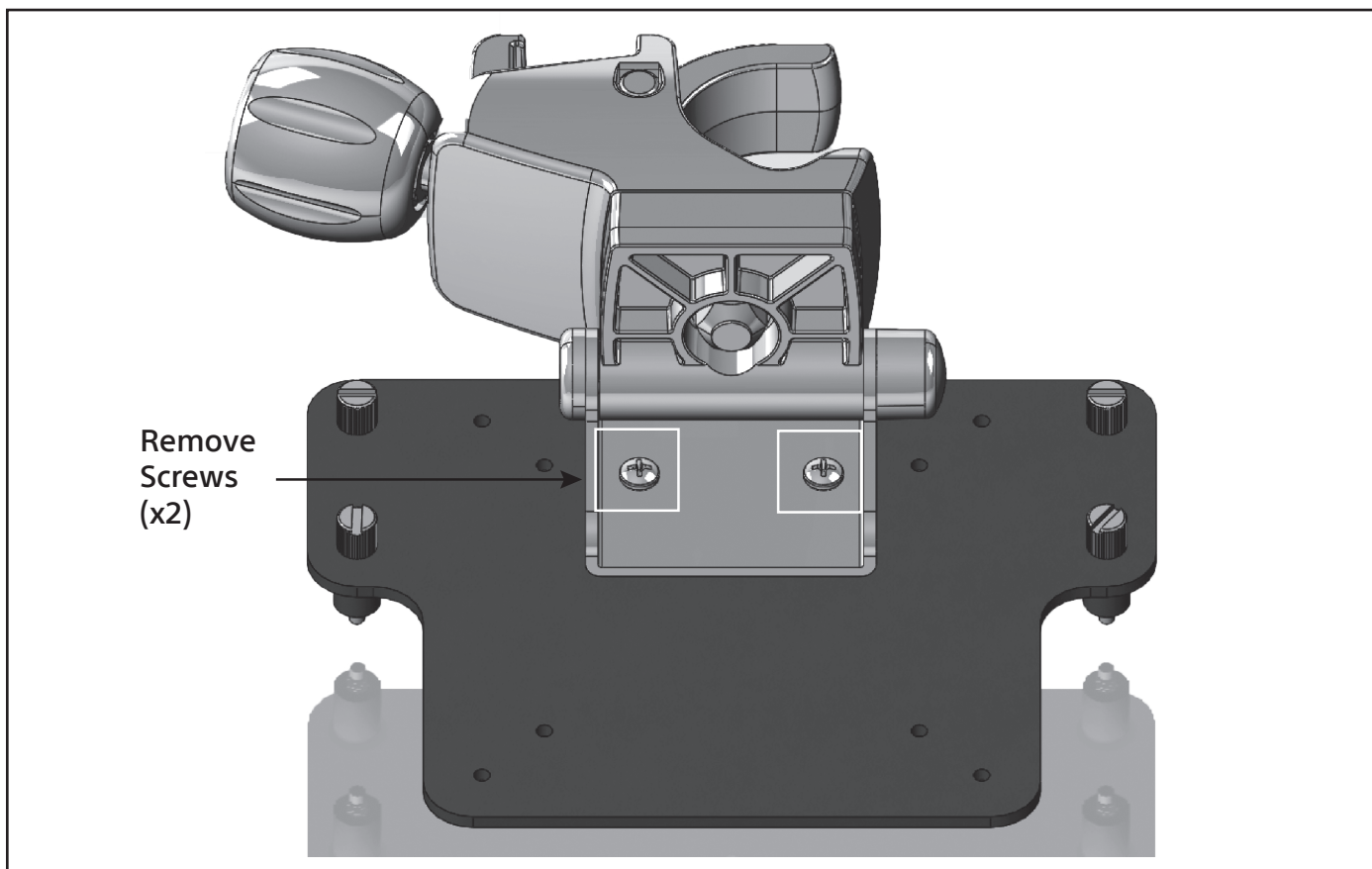
Figure 18. Remove Screws

4. Align the Monitor mount's screws (Figure 19) with the tapped holes on the back of the EXALT Monitor (Figure 20).

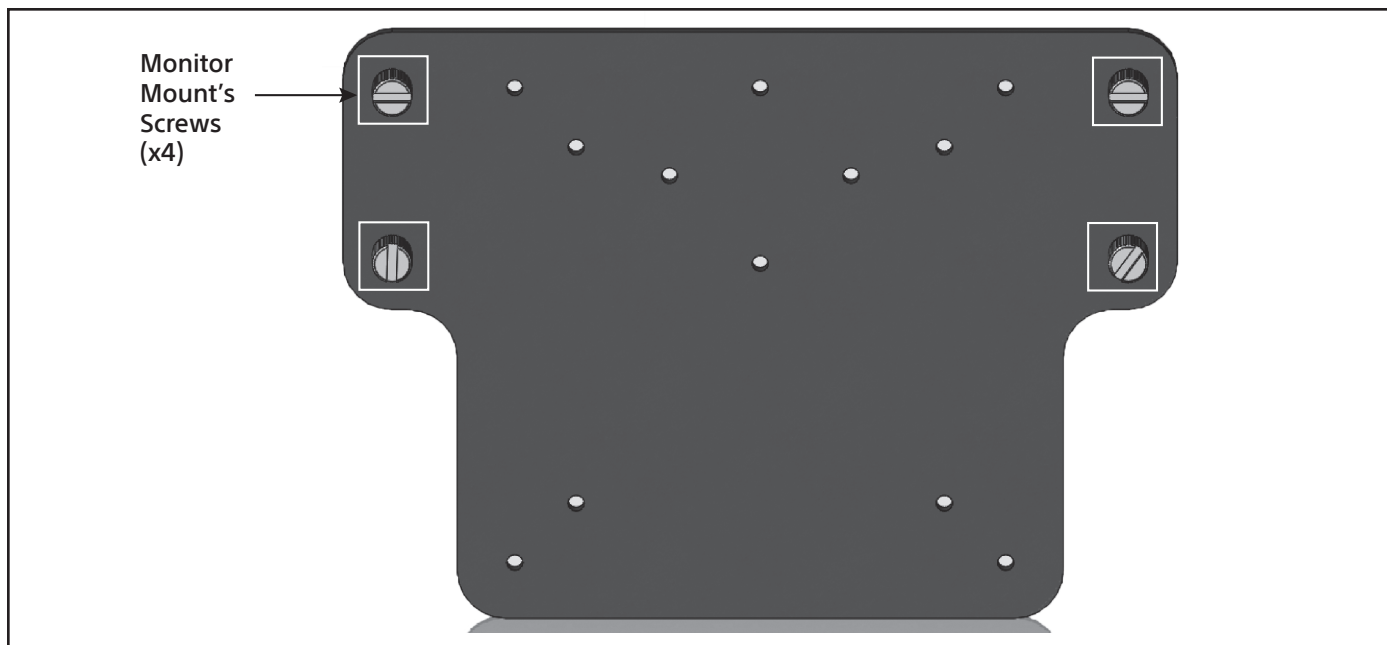
Figure 19. Align Monitor Mount's Screws

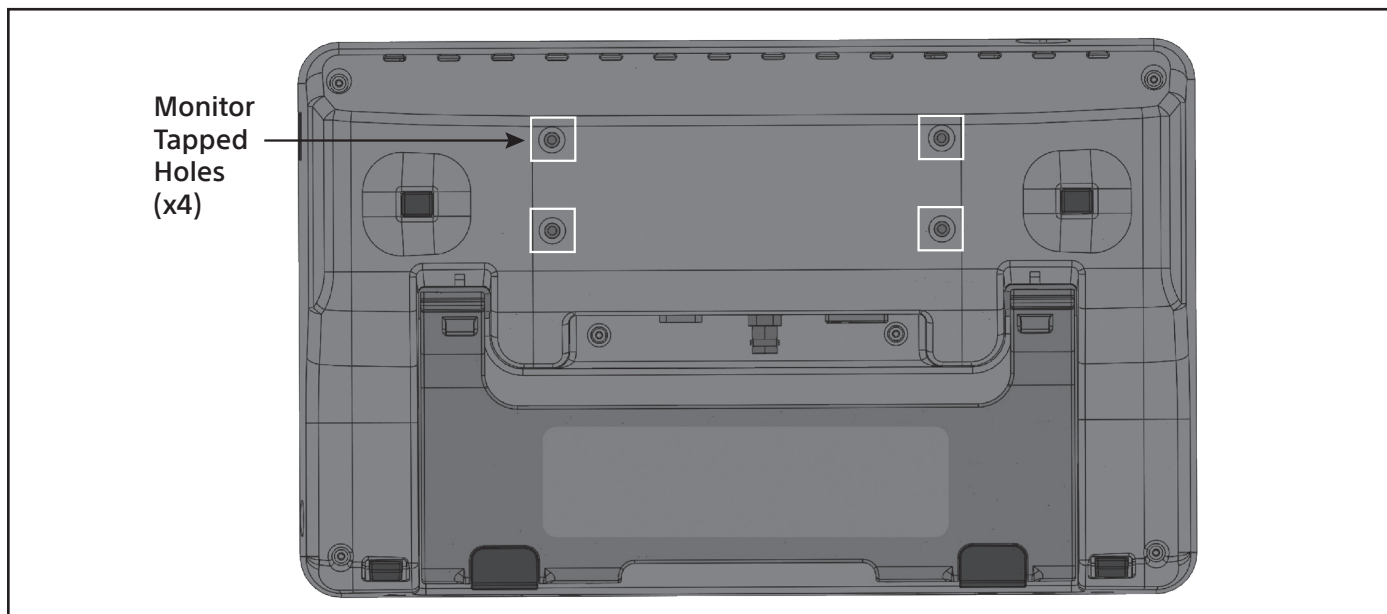
Figure 20. Monitor Tapped Holes

Note: The Monitor mount can be attached in 2 configurations. Figure 21 shows configuration 1 and Figure 22 shows configuration 2.

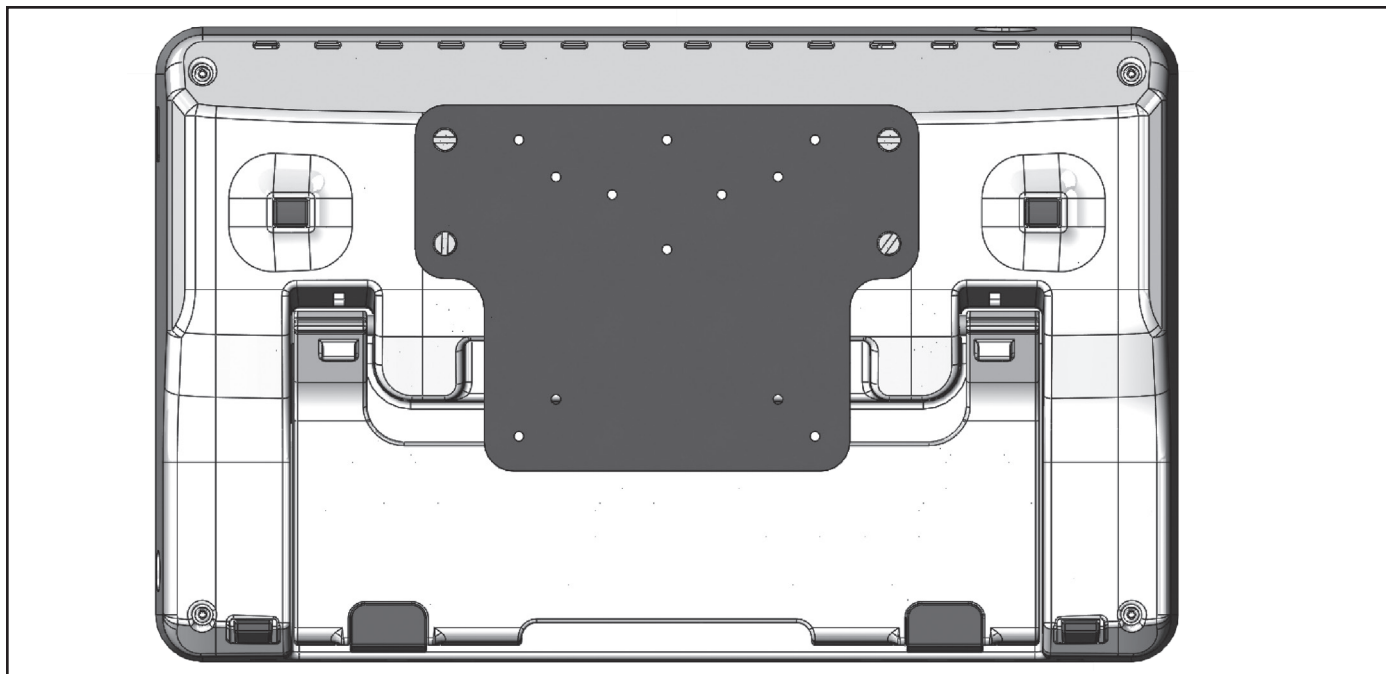
Figure 21. Monitor Mount Configuration 1

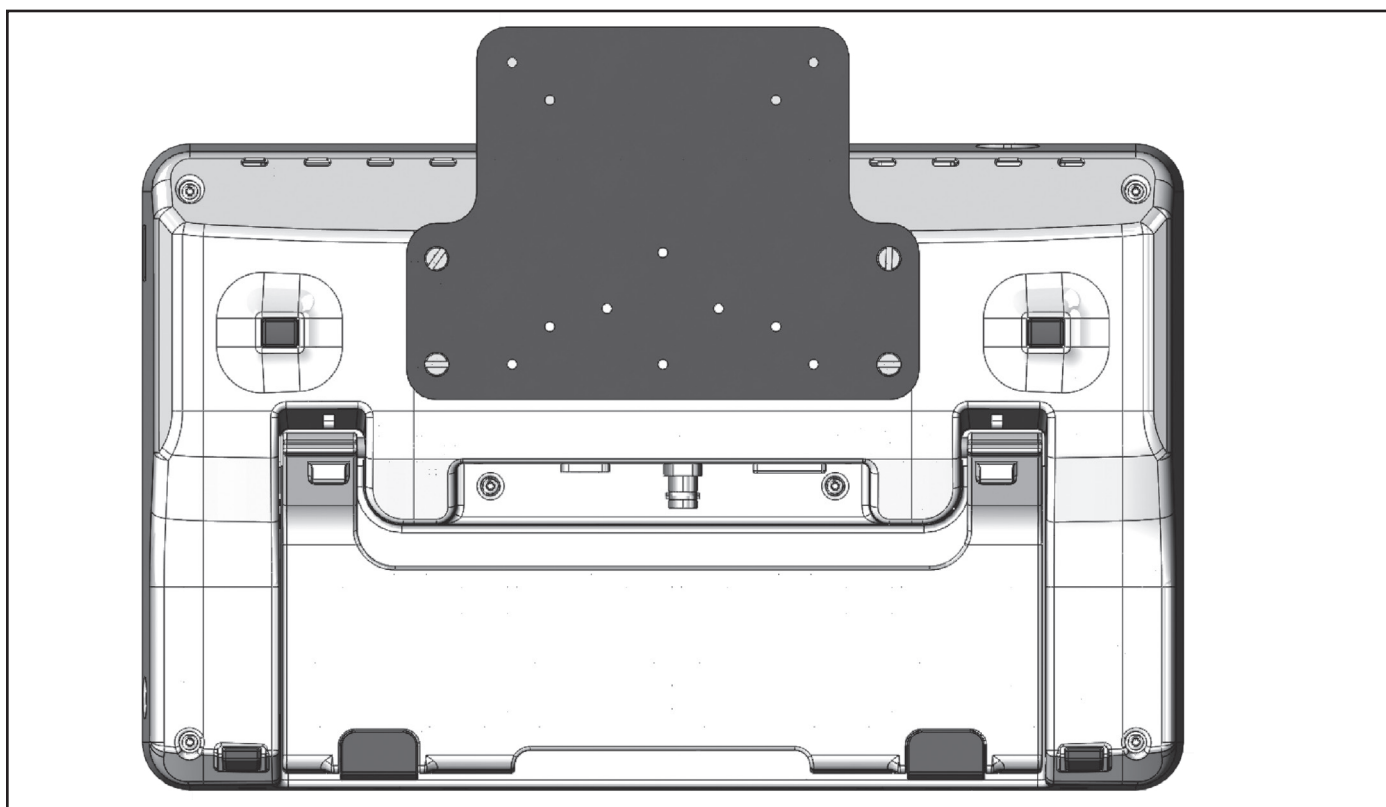
Figure 22. Monitor Mount Configuration 2

5. Hand tighten the screws until the mount is securely attached to the EXALT Monitor.
6. Using the tapped holes (shown in Figure 23), attach the EXALT Monitor to a VESA monitor mount.


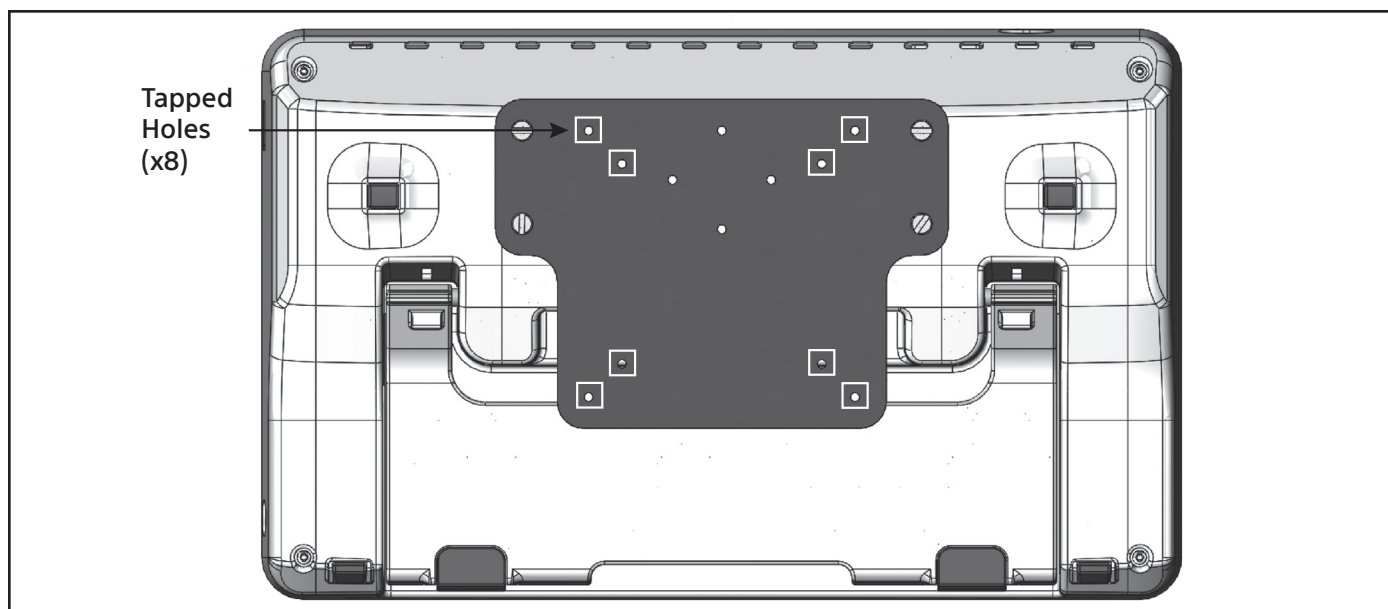
Figure 23. Tapped Holes

Starting the EXALT Monitor


Follow the steps below to start the EXALT Monitor. The user can start the EXALT Monitor with or without the single-use endoscope umbilicus cable plugged into the Monitor.

Powering On the Monitor

1. Press the power button located on the top panel to start the EXALT Monitor. The EXALT Monitor will then complete the bootup sequence. The EXALT Monitor will display the primary Windows boot screen (a), the login screen (b), and the Boston Scientific boot screen (c). If the EXALT Monitor does not perform the anticipated bootup sequence, consult the troubleshooting section.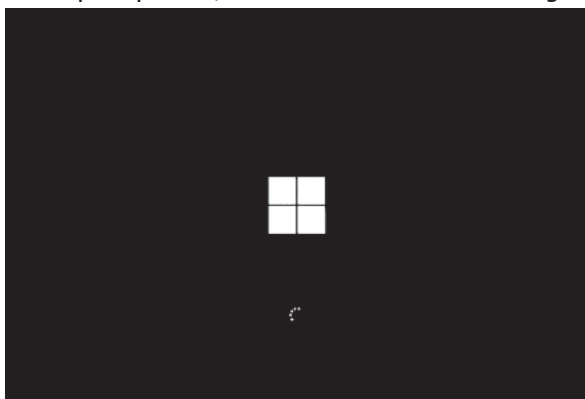

← (a) Windows boot screen

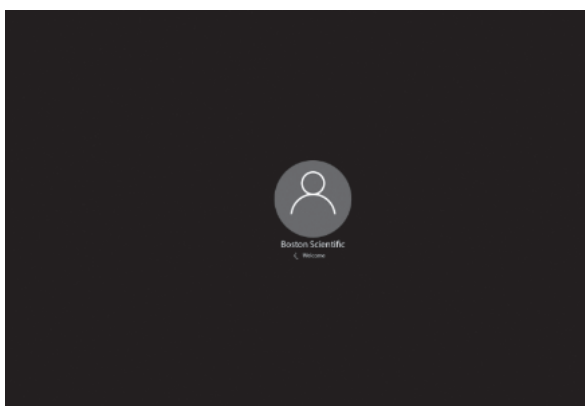
← (b) login screen

← (c) BSC boot screen

2. If the EXALT Monitor displays the Dead Battery Boot Up Screen (d), the EXALT Monitor needs to be charged before usage.

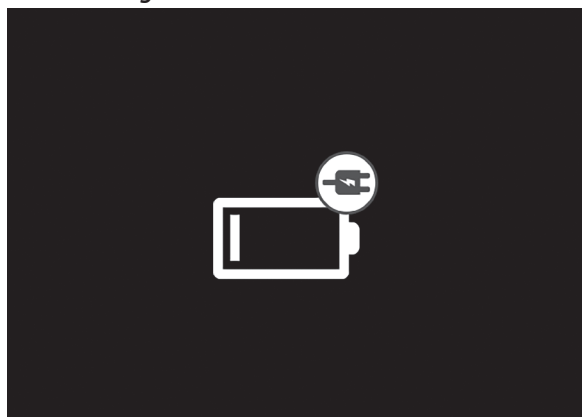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
3. Upon a successful bootup, the EXALT Monitor displays the Home Screen (e). Connect the single-use endoscope to the EXALT Monitor. If a single-use endoscope is already connected, proceed to step 4.

← (e) home screen

4. The Monitor will then display a live video image on the Procedure Mode Screen (f). If the live image does not appear, consult the troubleshooting section.

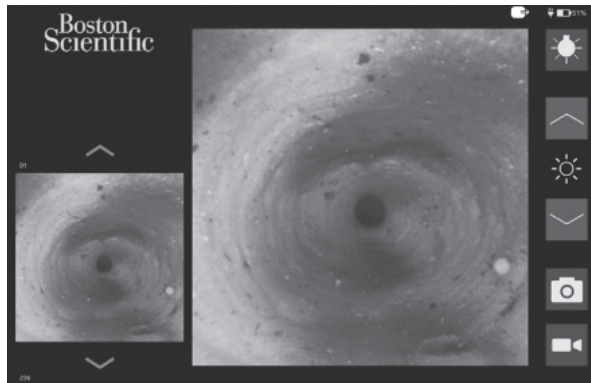← (f) procedure mode screen

Checking the Battery Status

Check the battery indicator on the EXALT Monitor before beginning a procedure. The battery status is displayed in the upper right corner of each screen. If necessary, connect the EXALT Monitor power supply (included in package contents) to a wall socket and insert the power plug into the power inlet of the EXALT Monitor. It is recommended to locate the EXALT Monitor power supply and the nearest wall socket before beginning the procedure.

Adjusting the Brightness of the Video Image and Operating Light On/Off Button

If desired, adjust the brightness of the image using the lighting controls on the Procedure Mode Screen of the EXALT Monitor.

Pressing the light on/off button (figure below) toggles the light source between the ON and OFF state. The On/Off state of the light source is indicated by the background color of the light on/off button.


← light source on


← light source off

To increase the brightness of the video image as displayed on the live video screen, press the brightness up button. To decrease the brightness of the video image, press the brightness down button. The relative brightness is indicated via the number located in the upper right of the brightness up button if the brightness setting is greater than 0 (default) and the lower right of the brightness down button.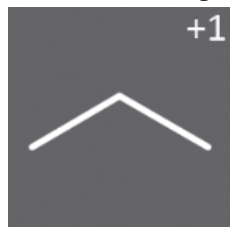

← brightness up button with
relative brightness indication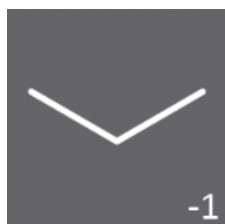

← brightness down button with relative brightness indication

Procedure

The procedure below describes the procedural use of the EXALT Monitor and is to be performed only once the user has received, inspected, assembled, and tested the EXALT Monitor following instructions in the "Preparation" section.


Using the EXALT Monitor in an Endoscopy Procedure

1. Clean the EXALT Monitor as directed in the "Cleaning" section.
2. Power on the EXALT Monitor by pressing the power button.
3. Connect a single-use endoscope to the front panel connector.
4. Complete the procedure as outlined in the single-use endoscope instructions for use.
5. If desired, use the image and video capture buttons to capture images and videos throughout the procedure.

Notifications and Errors

Notifications may appear on the Home Screen and Procedure Mode Screen of the EXALT Monitor while in use. Notifications will appear in the upper left corner on the Home Screen and the Procedure Mode Screen. Table 10 illustrates and details possible notifications.

Table 10. Description of Notifications and Errors

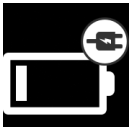Notification	Description
	Battery Low - Indicates that battery level is low and EXALT Monitor should be plugged into provided AC power adapter.
	Battery Critical - Indicates that battery level is critical and EXALT Monitor should be plugged into provided AC power adapter immediately. Continuing use may result in unintended shutdown of EXALT Monitor.
	Media Storage Low - Indicates that media storage space is low and videos/images should be deleted from the Procedure History Screen.
	Media Storage Critical - Indicates that media storage space is critical and videos/images should be deleted from the Procedure History Screen immediately. Continuing use may result in the inability to capture images/videos.
	SUD Reuse Notification - Indicates that expiration of the use time of the SUD will occur in ≤ 30 minutes.
	Hardware Error Notification - Indicates a hardware error. If this error appears, shutdown and restart the EXALT Monitor. If the issue is not resolved, contact Boston Scientific.

Errors may appear on the Procedure Mode Screen of the EXALT Monitor while in use. Figure 24 illustrates the Scope Malfunction Error, which indicates that the single-use endoscope is broken or defective. If this error appears, record the error number (located in the bottom right) and replace the single-use endoscope.

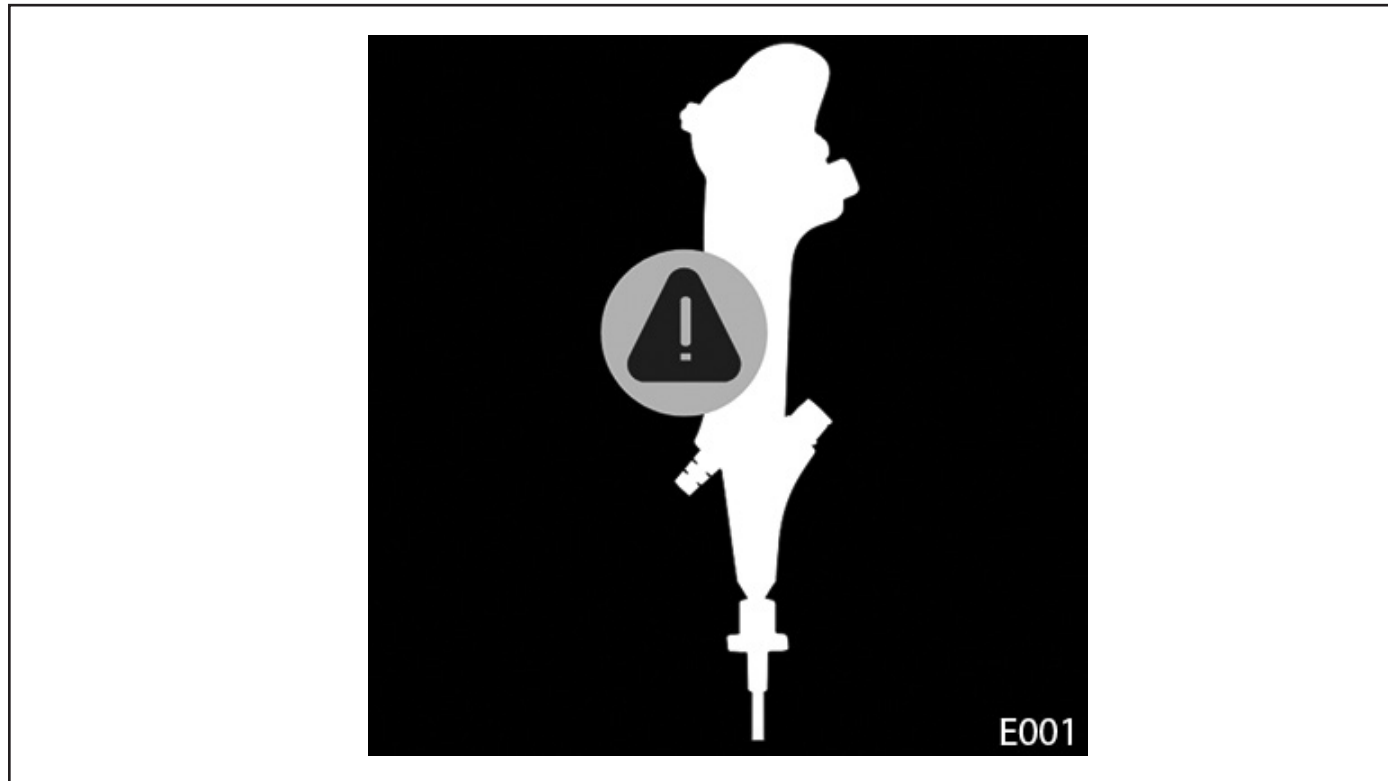
Figure 24. Scope Malfunction Error

Figure 25 illustrates the SUD Reuse Error. This error appears if the SUD is disconnected and reconnected after 12 hours of use. If this error appears, replace the single-use endoscope.

Figure 25. SUD Reuse Error

Recovering from an EXALT Monitor Failure

There are three possible failure modes: (1) unintended EXALT Monitor shutdown, (2) EXALT Monitor that becomes unresponsive, and (3) EXALT Monitor Hardware Failure.

Recovery from Unintended Shutdown

Follow these steps to recover from an unintended EXALT Monitor shutdown:

1. Press the Monitor's power button to restart the EXALT Monitor.
2. If the EXALT Monitor displays the No Power Boot Screen, connect the EXALT Monitor power supply (included in package contents) to a wall socket and insert the power plug into the power inlet of the EXALT Monitor.
3. If the EXALT Monitor fails to start, contact Boston Scientific.

Recovery from an EXALT Monitor that Becomes Unresponsive

1. Unplug the single-use endoscope from the EXALT Monitor and plug the single-use endoscope back into the EXALT Monitor.
2. If step 1 does not resolve the issue, press the EXALT Monitor's power button to power off the EXALT Monitor.
3. Once the EXALT Monitor has fully shut down, press the power button to restart the EXALT Monitor.
4. If the EXALT Monitor fails to start, contact Boston Scientific.

Recovery from EXALT Monitor Hardware Failure

1. Press the Monitor's power button to restart the EXALT Monitor.
2. If the EXALT Monitor fails to start, contact Boston Scientific.

Troubleshooting and Responding to Error Codes

Troubleshooting Chart

If the EXALT Monitor does not operate as expected, try to resolve the problem with the troubleshooting chart in Table 11 before contacting Boston Scientific for technical support.

Table 11. Troubleshooting Chart

Symptom	Possible Cause	Corrective Action
EXALT Monitor does not start properly.	EXALT Monitor battery not charged.	Connect EXALT Monitor to power supply and ensure supply is properly connected to wall socket.
	Boot problem.	Hold the power button down for 15 full seconds to force the Monitor to Reboot.
	EXALT Monitor is damaged.	Call Boston Scientific for support.
Single-use endoscope umbilicus cable is plugged into the EXALT Monitor, but no video image is displayed.	Single-use endoscope umbilicus cable is not properly connected to EXALT Monitor.	Ensure the plug on the single-use endoscope umbilicus is firmly inserted into the connector. Ensure connection terminals are dry and clean.
	Single-use endoscope is broken or defective.	Replace single-use endoscope.
	Illumination light is off.	Press LED control button to turn on light.

Symptom	Possible Cause	Corrective Action
Single-use endoscope connection cable is plugged into the EXALT Monitor, but Home Screen is displayed.	Single-use endoscope cable is not properly connected to EXALT Monitor.	Ensure the plug on the single-use endoscope umbilicus is firmly inserted into the connector. Ensure connection terminals are dry and clean.
	Single-use endoscope is broken or defective.	Replace single-use endoscope.
Single-use endoscope experiences image loss and the scope error screen is displayed.	Single-use endoscope umbilicus cable is not properly connected to EXALT Monitor.	Ensure the plug on the single-use endoscope umbilicus is firmly inserted into the connector. Ensure connection terminals are dry and clean.
	Single-use endoscope is broken or defective.	Replace single-use endoscope.
EXALT Monitor experiences image loss and the hardware error screen is displayed.	EXALT Monitor is malfunctioning.	Reboot the EXALT Monitor by powering it down and restarting it. If the problem reoccurs, call Boston Scientific for support.
EXALT Monitor experiences image loss and unexpected video is displayed.		
Video image is blurry, scrambled, inadequate, distorted, or otherwise not acceptable.	EXALT Monitor is too close to other electrical medical equipment.	Ensure EXALT Monitor is placed as directed in Table 9 or Table 10 of Appendix 4. Shutdown other medical electrical equipment to determine which equipment is causing the symptom. Properly place other electrical medical devices according to their instructions for use.
	Debris covering the distal tip of the single-use endoscope.	Clean the distal tip using a purified water solution and a cotton swab.
	Single-use endoscope is broken or defective.	Replace single-use endoscope.
	EXALT Monitor is malfunctioning.	Reboot the EXALT Monitor by powering it down and restarting it. If the problem reoccurs, call Boston Scientific for support.
EXALT Monitor touch screen is unresponsive.	Debris covering the touch screen.	Clean the touch screen using 15 to 70 percent isopropyl alcohol in purified water solution and a soft cloth.
	EXALT Monitor is malfunctioning.	Reboot the EXALT Monitor by powering it down and restarting it. If the problem reoccurs, call Boston Scientific for support.

Symptom	Possible Cause	Corrective Action
EXALT Monitor is connected to power supply but charging symbol is not displayed.	Power supply is not properly connected to EXALT Monitor.	Ensure that power supply is properly connected to the power inlet of the EXALT Monitor.
	Power supply is not properly connected to wall socket.	Ensure that the EXALT Monitor power supply is properly connected to wall socket.
	Wall socket is dysfunctional.	Ensure that wall socket is functional.
	EXALT Monitor is malfunctioning.	Reboot the EXALT Monitor by powering it down and restarting it. If the problem reoccurs, call Boston Scientific for support.
Video is displayed on the EXALT Monitor, but not on secondary video monitor.	Secondary video monitor is not powered up or not connected properly.	Power up the secondary video monitor. Check that the video cable is properly connected to the EXALT Monitor and secondary video monitor. Verify that the secondary video monitor is set to the correct video input.
	EXALT Monitor is running off battery power.	Video outputs are disabled when the EXALT Monitor is running off battery power. Connect the EXALT Monitor power supply (included in package contents) to a wall socket and insert the power plug into the power inlet of the EXALT Monitor.
Captured images and videos are not being properly stored.	EXALT Monitor's internal storage is full.	Delete unwanted procedures from the Procedure History Screen.
	EXALT Monitor is malfunctioning.	Reboot the EXALT Monitor by powering it down and restarting it. If the problem reoccurs, call Boston Scientific for support.
Captured images and videos are not being properly exported.	USB flash drive is full.	Replace full USB flash drive with flash drive with sufficient space.
	EXALT Monitor is malfunctioning.	Reboot the EXALT Monitor by powering it down and restarting it. If the problem reoccurs, call Boston Scientific for support.
	USB flash drive is malfunctioning.	Replace USB Flash Drive. Boston Scientific has tested the included Kingston (DTSE9H or DTKN) 32 GB for compatibility.

Symptom	Possible Cause	Corrective Action
Video image is too dark.	Brightness setting is too low.	Adjust brightness using brightness control buttons.
	Debris covering the distal tip of the single-use endoscope.	Clean the distal tip using a purified water solution and a cotton swab.
	Single-use endoscope illumination elements are damaged.	Replace single-use endoscope.
	EXALT Monitor is malfunctioning.	Reboot the EXALT Monitor by powering it down and restarting it. If the problem reoccurs, call Boston Scientific for support.
Video image is too bright.	Brightness setting is too high.	Adjust brightness using brightness control buttons.
	Single-use endoscope illumination elements are damaged.	Replace single-use endoscope.
	EXALT Monitor is malfunctioning.	Reboot the EXALT Monitor by powering it down and restarting it. If the problem reoccurs, call Boston Scientific for support.
Monitor screen flashing momentarily.	Connecting the Monitor to AC mains while the Monitor is powered on.	Ensure the Monitor is connected to AC mains prior to powering on.

If any of these problems persist, please contact Boston Scientific for repair or replacement information.

Post-Procedure

Following procedure completion, remove any accessory and the single-use endoscope from the patient following the instructions in the single-use endoscope instructions for use.

Any serious incident that occurs in relation to this device should be reported to the manufacturer, and to the relevant local regulatory authority.


Reviewing, Exporting, Deleting, and Shutting Down after Procedure Completion

To review images and videos captured during the procedure, follow these steps:

1. Disconnect the single-use endoscope cable from the front of the EXALT Monitor. There is no need to have an SUD connected to work with previous procedures, images and videos.
2. Press the procedure history button on the Home Screen to navigate to the Procedure History Screen.

3. Use the scrolling features on the Procedure History Screen to scroll through procedures and images/videos within a single procedure.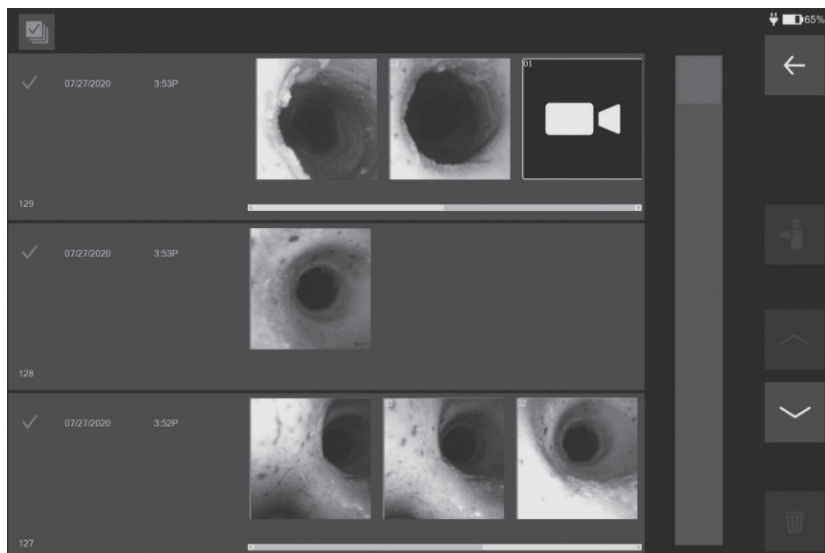

4. To review a single image or video, click on the thumbnail of the image/video to open it on the Review Mode Screen.
5. To select a subsequent image/video while on the Review Mode Screen:
 - a. Use the scrolling features on the Review Mode Screen OR
 - b. Select an image/video thumbnail
6. Press the back button on the Review Mode Screen to navigate back to the Procedure History Screen.

Images and videos can be exported from either the Procedure History Screen or the Review Mode Screen. While on the Procedure History Screen, the user can export entire procedures. While on the Review Mode Screen, the user can export individual images and videos within a procedure.

To export an image, video, or procedure, follow these steps:


1. Insert a USB flash drive into the USB port located on the right panel of the EXALT Monitor.
2. If exporting from the Procedure History Screen, select procedure(s) by pressing the procedure selection button located in the upper right corner of each procedure panel OR


3. Select all procedures by pressing the procedure history selection button.

4. A blue procedure background indicates that the procedure has successfully been selected.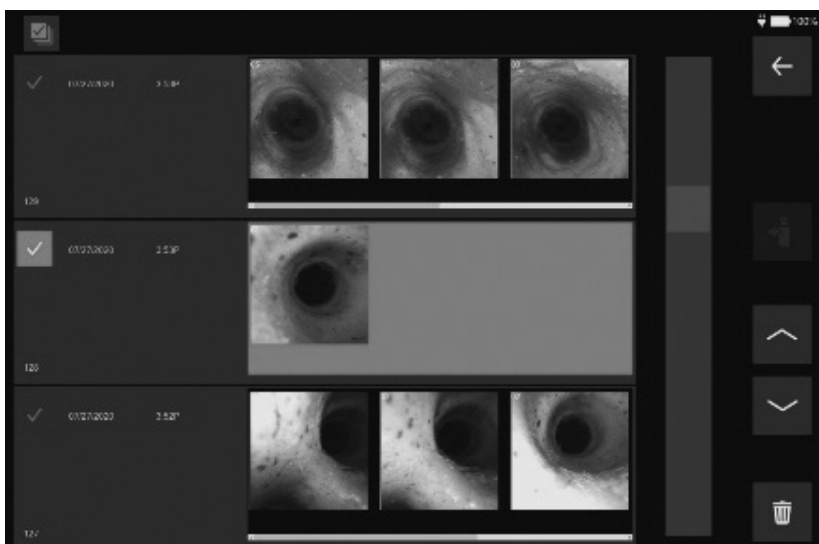

5. Press the USB Export button to begin exporting selected procedure(s).
6. The USB Export popup indicates that USB export is in progress. Exporting large video files may take up to 5 minutes per video file to transfer. Do not disconnect the USB during file transfer.

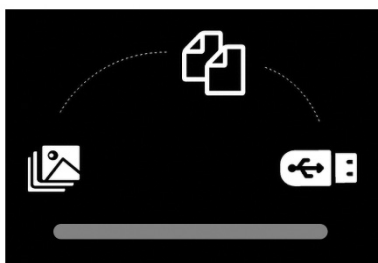7. USB Full notification indicates that the USB is full and files will not be exported.

Images and videos can be deleted from either the Procedure History Screen or the Review Mode Screen. While on the Procedure History Screen, the user can delete entire procedures. While on the Review Mode Screen, the user can delete individual images and videos within a procedure.

To delete an image, video, or procedure while on the Procedure History Screen, follow these steps:

1. Press the Delete button.
2. Confirm the deletion by pressing the checkmark on the Deletion Confirmation Screen.

Note: Recommended players for exported videos are VLC Media Player and Windows Media Player (VLC Media Player v3.0.11 or later, Windows Media Player v12 or later).

Shutdown of the EXALT Monitor

There are 3 methods that can be used to shut down the EXALT Monitor.

Method 1 (Preferred Method):


1. Disconnect the single-use endoscope cable from the front of the EXALT Monitor.
2. Press the Power Button on the Home Screen.

3. Confirm the shutdown by pressing the checkmark on the Shutdown Confirmation Screen.

4. Wait for the EXALT Monitor to complete the shutdown sequence.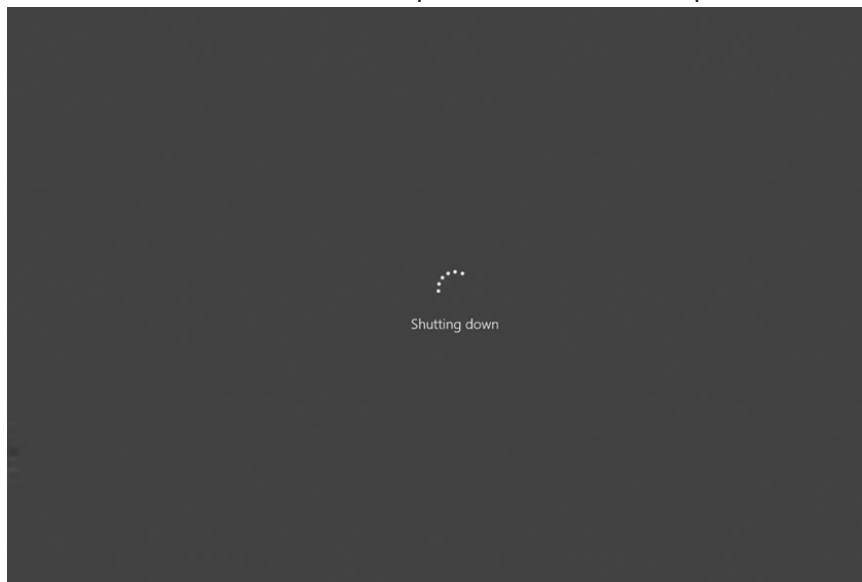

Method 2:

1. Press the physical power button located on the top panel of the EXALT Monitor.
2. Wait for the EXALT Monitor to complete the shutdown sequence.

Method 3:

1. Press and hold the physical power button located on the top panel of the EXALT Monitor.
2. Slide down the Windows slider to confirm shutdown.

3. Wait for the EXALT Monitor to complete the shutdown sequence.

Charging the EXALT Monitor

Ensure that the EXALT Monitor is sufficiently charged before use.

Cleaning of the EXALT Monitor

Disconnect the power cord before cleaning the unit. Use a 15-70 percent isopropyl alcohol in purified water solution and a soft cloth to clean the Monitor enclosure, front panel, video cables, power adapter, and power cable. Do not allow fluids to enter the enclosure, power cable connections, single-use endoscope cable receptacle, or component/ accessory connections. Do not spray cleaning solution directly onto EXALT Monitor. Do not clean using abrasive materials as this may scratch the EXALT Monitor's touchscreen. Do not attempt to clean the unit while it is plugged into an electrical outlet. Ensure that the EXALT Monitor is completely dry before reconnecting to power.

Disposal

All external and accessible surfaces of this device, video cables, power cable, and power adapter should be cleaned and disinfected per the disinfection instructions included in the user manual. Use of recycling service suppliers familiar with medical electrical equipment is recommended, but not required. Do not dispose of by incineration, burial or placement into common waste stream. Device should be safely disposed of in accordance with hospital, administrative, and/or local government policy.

INFORMATION TO BRIEF THE PATIENT

Please refer to single-use endoscope instructions for use.

WARRANTY

For device warranty information, visit (www.bostonscientific.com/warranty).

EXALT is a registered trademark of Boston Scientific Corporation or its affiliates.

All other trademarks are the property of their respective owners.

APPENDICES

The appendices include:

- Appendix 1—Guidance and Manufacturer's Declaration-Electromagnetic Emissions
- Appendix 2—Guidance and Manufacturer's Declaration-Electromagnetic Immunity
- Appendix 3—Guidance and Manufacturer's Declaration— electromagnetic IMMUNITY – for ME Equipment and ME Systems that are not LIFE-SUPPORTING
- Appendix 4—Recommended Separation Distance Between Portable and Mobile RF Communications Equipment and the EXALT Monitor
- Appendix 5—Medical Design Criteria and Specifications

The Monitor and Single-use Endoscope were tested for compliance to IEC 60601-1. Any additions or modifications to the Monitor, Single-use Endoscope, or previously configured accessories require the end user to ensure the system remains compliant with the requirements of IEC 60601-1.

Note: The ME System includes the EXALT Monitor AC power adapter and a single-use endoscope.

Appendix 1—Guidance and Manufacturer's Declaration-Electromagnetic Emissions

The EXALT System is intended for use in the electromagnetic environment specified in Table 12. The customer or the user of the EXALT Monitor should assure that it is used in such an environment.

Table 12. Guidance and manufacturer's declaration – electromagnetic emissions

Emissions Test	Compliance	Electromagnetic Environment - Guidance
RF Emissions CISPR 11	Group 1	The EXALT Monitor uses RF energy only for its internal function Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class A	The EXALT Monitor is suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/Flicker Emissions IEC 61000-3-3	Per section 5 of IEC 61000-3-3	

Appendix 2—Guidance and Manufacturer's Declaration-Electromagnetic Immunity 2nd and 3rd Edition

The EXALT System is intended for use in the electromagnetic environment specified in Table 13. The customer or the user of the EXALT Monitor should assure that it is used in such an environment.

Table 13. Guidance and manufacturer's declaration – electromagnetic immunity IEC 60601-1-2 (2nd and 3rd Edition)

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment (Guidance)
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV/V air	± 6 kV contact ± 8 kV/V air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% UT (> 95% dip in UT) for 0.5 cycle 40% U (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles < 5% UT (> 95% dip in UT) for 5 sec	< 5% UT (> 95% dip in UT) for 0.5 cycle 40% U (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles < 5% UT (> 95% dip in UT) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the EXALT Monitor requires continued operation during power mains interruptions, it is recommended that the EXALT Monitor be powered from the battery.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment (Guidance)
Power frequency (50/60 Hz magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note—UT is the a.c. mains voltage prior to application of the test level.			
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the EXALT Monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d=1.2\sqrt{P}$ $d=1.2\sqrt{P}$ 80 MHz to 800 MHz $d=2.3\sqrt{P}$ 800 MHz to 2.5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey (a), should be less than the compliance level in each frequency range (b). Interference may occur in the vicinity of equipment marked with the following Symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	
Note 1 —At 80 MHz and 800 MHz, the higher frequency range applies.			
Note 2 —These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the EXALT Monitor is used exceeds the applicable RF compliance level above, the EXALT Monitor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the EXALT Monitor. b) Over the frequency range 150 kHz to 80 MHz, the field strengths should be less than 3 V/m.			

Appendix 3—Guidance and Manufacturer's Declaration – Electromagnetic Immunity – for ME Equipment 4th Edition and ME Systems that are not Life-Supporting

The EXALT System is intended for use in the electromagnetic environment specified in Table 14. The customer or the user of the EXALT Monitor should assure that it is used in such an environment.

Table 14. Guidance and manufacturer's declaration – electromagnetic immunity IEC 60601-1-2 (4th Edition)

Immunity Test	IEC 60601 Test Level/Compliance Level	Electromagnetic Environment (Guidance)
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV/V air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips IEC 61000-4-11	0% UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315°	Mains power quality should be that of a typical commercial or hospital environment. If the user of the EXALT Monitor requires continued operation during power mains interruptions, it is recommended that the EXALT Monitor be powered from the battery.
	0% UT; 1 cycle and 70% UT; 25/30 cycles Single phase: at 0°	
Voltage interruptions IEC 61000-4-11	0% UT; 250/300 cycle	Mains power quality should be that of a typical commercial or hospital environment. If the user of the EXALT Monitor requires continued operation during power mains interruptions, it is recommended that the EXALT Monitor be powered from the battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note—UT is the a.c. mains voltage prior to application of the test level.		

Immunity Test	IEC 60601 Test Level/Compliance Level				Electromagnetic Environment (Guidance)
Conducted RF IEC 61000-4-6	3 V 0.15 MHz -80 MHz 6 V in ISM bands between 0.15 MHz -80 MHz 80% AM at 1 kHz				Portable and mobile RF communications equipment should be used no closer to any part of the EXALT Monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz 80% AM at 1 kHz				
Proximity fields from RF wireless communications equipment IEC 61000-4-3	Test Frequency (MHz)	Band (MHz)	Modulation	Immunity Test Level (V/m)	Recommended separation distance $d=2\sqrt{P}$ 80 MHz to 2.7 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey (a), should be less than the compliance level in each frequency range (b). Interference may occur in the vicinity of equipment marked with the following Symbol: 
	385	380 - 390	Pulse modulation 18 Hz	27	
	450	430 - 470	FM ± 5 kHz deviation 1 kHz sine	28	
	710	704-787	Pulse modulation 217 Hz	9	
	745				
	780				
	810	800-960	Pulse Modulation 18 Hz	28	
	870				
	930				
	1720	1700-1990	Pulse modulation 217 Hz	28	
	1845				
	1970				
	2450	2400-2570	Pulse modulation 217 Hz	28	
	5240	5100-5800	Pulse modulation 217 Hz	9	
5500					
5785					

Note —These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the EXALT Monitor is used exceeds the applicable RF compliance level above, the EXALT Monitor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the EXALT Monitor.

b) Over the frequency range 150 kHz to 80 MHz, the field strengths should be less than 3 V/m.

Appendix 4—Recommended Separation Distance Between Portable and Mobile RF Communications Equipment and the EXALT System

The EXALT Monitor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the EXALT Monitor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the EXALT Monitor as recommended below, according to the maximum output power of the communications equipment (Table 15 or Table 16).

Table 15. Recommended separation distance between portable and mobile RF communications equipment and the EXALT Monitor IEC 60601-1-2 (2nd and 3rd Edition)

Max Output Power (Watts)	Separation (m) 150 kHz to 80 MHz $D = 1.2(\sqrt{P})$	Separation (m) 80 MHz to 800 MHz $D = 1.2(\sqrt{P})$	Separation (m) 800 MHz to 2.5 GHz $D = 2.3(\sqrt{P})$
0.01	.12	.12	.23
0.1	.38	.38	.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power of the transmitter in watts (W) according to the manufacturer.

NOTE 1—At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2—These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Table 16. Recommended separation distance between portable and mobile RF communications equipment and the EXALT Monitor IEC 60601-1-2 (4th Edition)

Max Output Power (Watts)	Separation Distance $D = 2\sqrt{P}$
0.01	0.2
0.1	0.6
1	2.0
10	6.3
100	20.0
0.01	0.2

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum power of the transmitter in watts (W) according to the manufacturer.

NOTE – These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Appendix 5—Medical Design Criteria and Specifications

The EXALT Monitor meets medical design criteria and specifications (Table 17).

Table 17. Medical design criteria and specification

Design Criteria for EXALT Monitor	Specification
Type of protection against electric shock	Class II equipment
Degree of protection against electric shock	Type BF applied part
Mode of operation	Continuous operation
Installation and use	Portable equipment

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All rights reserved.
Boston Scientific (Australia) Pty Ltd
© 2021 Boston Scientific Corporation or its affiliates.