# Koala™

Cleaning Sponge plus Pure™ Enzymatic

Directions for Use	2
Instrucciones de uso	4
Mode d'emploi	6
Gebräuchsanweisung	8
Istruzioni per l'uso	10
Gebruiksaanwijzing	12
Instruções de Utilização	14
Brugsanvisning	16
Οδηγίες χρήσης	18
Instruções de uso	20
Bruksanvisning	22
Használati útmutató	24
Návod k použití	26
Instrukcja obsługi	28
Bruksanvisning	30

사용 지침	32
Kullanma Talimatları	34
Käyttöohjeet	36
Instructiuni de utilizare	38
Návod na použitie	40
Указания за употреба	42
Upute za uporabu	44
Kasutusjuhised	46
Lietošanas norādījumi	48
Naudojimo nurodymai	50
Petunjuk Penggunaan	52
Navodila za uporabo	54

51096726-01

2020-03

# Koala™

## Cleaning Sponge plus Pure™ Enzymatic

### R ONLY

**Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician.

#### REUSE WARNING

For single use only. Do not reuse or reprocess. Reuse, reprocessing or sterilizing may compromise the structural integrity of the device and/or lead to device failure. Reuse of reprocessing may also create a risk of contamination of the device or may lead to improper scope cleaning.

#### DEVICE DESCRIPTION

The Koala Cleaning Sponge plus Pure enzymatic is a single use, non-sterile, hand-held device made of absorbent polyester polyurethane intended to be soaked in water while squeezing the sponge to release the enzymatic detergent. The surface of the endoscope is wiped with the enzymatic detergent soaked sponge after the endoscopic procedure as a pre-cleaning step prior to disinfection or sterilization of the endoscope.

#### Contents

Package contents include:

(1) Koala Cleaning Sponge plus Pure Enzymatic

#### INDICATIONS FOR USE / INTENDED USE

The Koala Cleaning Sponge plus Pure Enzymatic is intended to be soaked in water to wipe the surface of the endoscope after the endoscopic procedure as a pre-cleaning step prior to disinfection or sterilization of the endoscope.

#### Clinical Benefit Statement

Remove debris and reduce bioburden from the exterior surface of endoscopes during pre-cleaning steps of endoscope reprocessing.

#### CONTRAINdicATIONS

None known.

#### WARNINGS

#### DANGER

May cause allergy or asthma symptoms or breathing difficulties if inhaled.

#### PRECAUTIONS

- Ensure proper personal protective equipment (PPE) is worn during use of this device.
- Avoid contact with eyes and skin. In case of contact, flush with water. If irritation persist, contact a physician.
- Avoid breathing enzymatic dust. If inhaled: If breathing is difficult, move person to fresh air and keep comfortable for breathing. If experiencing respiratory systems: contact a physician.

#### ADVERSE EVENTS

- Allergic Reaction
- Asthma Exacerbation

#### HOW SUPPLIED

The device is supplied non-sterile and intended for single use only. The packaging and device should be inspected prior to use.

Do not use if package is damaged or unintentionally opened before use.

Do not use if labeling is incomplete or illegible.

#### Handling and Storage

This product has no special handling or storage requirements.

#### OPERATIONAL INSTRUCTIONS

- After the endoscopy procedure, create an enzymatic solution by submerging the enzymatic sponge into the EcoBowl, or equivalent, filled with approximately 300 ml of water while squeezing the sponge vigorously at least three times to release the enzymatic detergent.
- Next, suction the enzymatic solution through the endoscope and wipe the exterior surfaces of the endoscope with the wet sponge.
- Place the sponge over the tip of the endoscope during endoscope transport.

## **Disposal**

To minimize the risk of infection or microbial hazards after use, dispose device and packaging as follows:

After use, device may contain biohazardous substances. The device and packaging should be treated and disposed of as biohazardous waste or have them treated and disposed of in accordance with any applicable hospital, administrative, and/or local government regulations. Use of a biohazardous container with biological hazard symbol is recommended. Untreated biohazardous waste should not be disposed of in the municipal waste system.

## **Post-Procedure**

Any serious incident that occurs in relation to this device should be reported to the manufacturer and relevant local regulatory authority.

## **WARRANTY**

For device warranty information, visit ([www.bostonscientific.com/warranty](http://www.bostonscientific.com/warranty)).

EU Importer: Boston Scientific International B.V., Vestastraat 6, 6468 EX Kerkrade, The Netherlands

Koala is a registered trademark of Boston Scientific Corporation or its affiliates.

All other trademarks are the property of their respective owners.

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After use, the device may contain biological risk substances. The device and its packaging must be treated and disposed of as medical waste in accordance with applicable hospital, administrative and/or governmental local regulations. It is recommended to use a receptacle for medical waste containing a biological risk symbol. Medical waste not treated as such must be disposed of in the municipal waste system.

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All rights reserved.
© 2020 Boston Scientific Corporation or its affiliates.