Boston Scientific

Advancing science for life™

en

fr

ru

ar

Vercise™ Deep Brain Stimulation System Physician Manual

91098825-10

Content: MP91098825-10 REV A

How to Use this Manual

This manual provides information about the Vercise™ Boston Scientific Deep Brain Stimulation (DBS) System.

Read all instructions carefully before using the Vercise™ DBS System. For other device-specific information not included in this manual, refer to the appropriate DFU for your Boston Scientific DBS System as listed in your *DBS Reference Guide*.

Guarantees

Boston Scientific Corporation reserves the right to modify, without prior notice, information relating to its products in order to improve their reliability or operating capacity.

Trademarks

All trademarks are the property of their respective holders.

Contacting Boston Scientific

To contact Boston Scientific, see the “*Technical Support*” section of this manual.

Patient Identification Card

Ensure that the patient receives a completed Temporary Identification Card after their surgery. Permanent ID cards will be provided by your local sales representative after patient registration.

Product Model Numbers

Model	Description
DB-1110-C	Vercise IPG Kit
DB-2201-30AC	DBS Lead Kit, 30cm
DB-2201-45BC	DBS Lead Kit, 45cm
DB-2201-30DC	DBS Lead Kit, 30cm
DB-2201-45DC	DBS Lead Kit, 45cm
DB-2201-30EC	DBS Lead Kit, 30cm
DB-2201-45EC	DBS Lead Kit, 45cm
NM-3138-55	55cm 8 Contact Extension Kit
DB-2500-C	Vercise Physician’s Spare Kit
SC-4252	Tunneling Tool, 28cm Straw
SC-4254	Tunneling Tool, 35cm Long
SC-4100A	OR Cable & Extension
DB-5110-C	Vercise ETS

Table of Contents

Introduction	1
Overview of Manual	1
Intended Use / Indications for Use.....	1
System Description	1
Safety Information	2
Contraindications	2
Warnings	3
Precautions	6
Adverse Events.....	11
Package Contents	13
Contents of the Implantable Pulse Generator Kit	13
Contents of the Lead Kit	13
Contents of the Extension Kit	13
Contents of the Tunneling Tool Kit.....	13
Contents of the Spares Kit.....	14
Other Components	14
DBS Lead Implantation	15
Pre-Conditions	16
Implanting the DBS Lead	16
Intraoperative Stimulation Testing.....	18
Securing the DBS Lead	20
Stimulator Implantation	24
Exposing the DBS Lead.....	24
Connecting the DBS Lead to the DBS Extension	25
Assembling the Tunneling Tool	27
Tunneling the DBS Lead and Extension	28
Connecting the Stimulator	30
Vercise™ DBS System Revisions & Explantation	34
Explanting the DBS Lead.....	34
Explanting the DBS Extensions	35
Explanting or Replacing the Stimulator.....	36
The Vercise™ Stimulator	37
Stimulator Battery	38
Stimulator Battery Messages	38
The Charging System	39
Charging the Charger	40
Charging the Stimulator	40
Using the Charging Collar.....	41
Using the Adhesive Patch.....	42
Properly Aligning the Charger	43
The Remote Control	45
Basic Operation	45

Remote Control Position	46
Stimulator Linking	46
Stimulation On/Off.....	48
Signal Strength	48
Stimulation Amplitude	49
Stimulator Battery	49
Program Selection	50
Modifying and Saving Programs.....	51
Individual Area Amplitude	52
Restore Option.....	52
Searching.....	52
Clinician Options.....	53
Stimulation Guidelines.....	57
Charge Density	57
Detailed Device Description	58
General Component Description	58
Stimulator Physical Characteristics	59
Stimulator Programmable Characteristics	60
DBS Lead	61
DBS Extension.....	62
DBS Lead Boot.....	63
Suture Sleeve	64
Tunneling Tool.....	65
Programmer Communication.....	66
Vercise™ DBS System Registration	67
Technical Support	68
Electromagnetic Compatibility.....	70
Essential Performance.....	72
Telemetry Information	72
Wireless Security	72
Limited Warranty	73
IPG.....	73
Leads	74
Externals.....	75
Appendix	77
Rechargeable Stimulator Battery Longevity	77

Introduction

Overview of Manual

This manual describes the implantation and usage of the Vercise™ Deep Brain Stimulation (DBS) System. The Vercise DBS System features a Stimulator coupled with DBS Leads and DBS Extensions. In this manual you will also find detailed descriptions of each system component. Additional information about programming the Vercise DBS System programming can be found in the *DBS Programming Manual*.

Intended Use / Indications for Use

The Vercise Deep Brain Stimulation System is indicated for use in the following:

- Unilateral or bilateral stimulation of the subthalamic nucleus (STN) or internal globus pallidus (GPi) for treatment of levodopa-responsive Parkinson's disease which is not adequately controlled with medication.
- Unilateral or bilateral stimulation of the subthalamic nucleus (STN) or internal globus pallidus (GPi) for treatment of intractable primary and secondary dystonia, for persons 7 years of age and older.
- Thalamic stimulation for the suppression of tremor not adequately controlled by medications in patients diagnosed with Essential Tremor or Parkinson's disease.

System Description

The Vercise DBS System includes a Stimulator with DBS Leads for unilateral or bilateral stimulation. There are also DBS Extensions that allow the DBS Leads mounted in the skull to be extended to reach the Stimulator implanted near the clavicle.

The rechargeable Vercise DBS System utilizes current steering across eight contacts per DBS Lead to provide precise positioning of stimulation. The Stimulator is controlled by a handheld Remote Control, and can be interfaced with a Clinician's Programmer using the Bionic Navigator™ Software. Periodically, the Stimulator battery must be replenished with an RF charging device provided in the Patient DBS Charging Kit.

Note: *This product contains no detectable latex.*

Safety Information

Contraindications

The Boston Scientific Vercise™ DBS System, or any of its components, is contraindicated for:

Diathermy. Shortwave, microwave, and/or therapeutic ultrasound diathermy. The energy generated by diathermy can be transferred to the Vercise DBS System, causing tissue damage at the contact site resulting in severe patient injury or death.

Magnetic Resonance Imaging (MRI). Patients implanted with the full Vercise DBS System (Leads, Extensions, and Stimulator) should not be subjected to MRI. MRI exposure may result in:

- Dislodgement of implanted components.
- Heating of the contacts, or other system components, causing permanent tissue lesioning.
- Damage to the Stimulator's electronics.
- Current induction through the DBS Leads and Vercise DBS System causing unpredictable levels of stimulation.
- Distortion of the diagnostic image.
- Personal injury or even death.

Note: *Vercise DBS Lead-only system (before Stimulator is implanted) is MR conditional. An MRI examination can be conducted safely when all instructions in the supplemental manual MRI Guidelines for Boston Scientific DBS Systems are followed.*

For the latest version of the manual go to www.bostonscientific.com/manuals.

Patient Incapability. Patients who are unable to properly operate the Remote Control and Charging System should not be implanted with the Vercise DBS System.

Poor Surgical Risks. The Vercise DBS System is not recommended for patients who are poor surgical risks.

Warnings

Automobiles and Equipment. Patients should operate automobiles, other motorized vehicles, or potentially dangerous machinery/equipment with caution after receiving the Vercise DBS System. Performing activities that would be dangerous if treated symptoms were to return, or instances in which stimulation changes occur, should be avoided.

Charge Density. High levels of stimulation may damage brain tissue. Whenever possible, the current amplitude and pulse width should be programmed such that the charge density is below $30 \mu\text{C}/\text{cm}^2$ per stimulation phase. To maintain safety limits, the software will display a warning when the level of stimulation exceeds $30 \mu\text{C}/\text{cm}^2$ per stimulation phase; however, the software allows the stimulation to be adjusted above this level by the physician.

Patients may have the ability to change the amplitude with the Remote Control. Set and verify the maximum and minimum amplitude levels allowed by the Remote Control to ensure that current levels remain safe.

DBS Extension Connector and Stimulator Placement. Implanting the DBS Extension connector in the soft tissue of the neck may increase the chance of DBS Lead breakage. Boston Scientific recommends placing the DBS Extension connector behind the ear such that glasses or headgear do not interfere with the system. Boston Scientific recommends that the Stimulator be placed subclavicularly.

Diathermy. Shortwave, microwave and/or therapeutic ultrasound diathermy should not be used on patients implanted with the Vercise™ DBS System, or any of the system components. The Stimulator, whether it is turned on or off, may be critically damaged by the use of diathermy. The energy generated by diathermy can be transferred to the Vercise™ DBS System, causing tissue damage at the contact site resulting in severe injury or death.

Electromagnetic Interference. Strong electromagnetic fields can potentially turn the Stimulator off, cause temporary unpredictable changes in stimulation, or interfere with the Remote Control communication. Patients should be counseled to avoid or exercise care around:

- Theft detectors, tag deactivators and RFID devices, such as those used at department stores, libraries, and other public establishments. The patient should proceed with caution, ensuring to move through the center of the detector as quickly as possible.
- Security screeners, such as those used in Airport Security or at entrances to government buildings, including hand-held scanners. The patient should request assistance to bypass the device. If the patient must pass through the security screener, they should move quickly through the device staying as far from the physical device as allowable.
- Power lines or power generators.
- Electric steel furnaces and arc welders.
- Large magnetized stereo speakers.
- Strong magnets.

- Automobiles or other motorized vehicles using a LoJack system or other anti-theft systems that can broadcast a radio frequency (RF) signal. The high energy fields produced by these systems may interfere with the operation of the Remote Control and its ability to control stimulation.
- Other sources of electromagnetic interference, such as RF transmitters at television or radio broadcast stations, Amateur Radio or Citizens Band radio transceivers, or Family Radio Service band transceivers.

Note: *When in close proximity, equipment that generates strong electromagnetic fields might cause unintended stimulation or interfere with wireless communication even if they comply with International Special Committee on Radio Interference (CISPR) requirements.*

Heat Due to Charging. The Charger may become warm while charging the Stimulator. The Charger should be handled with care. Failure to use either the Charging Collar or an Adhesive Patch while charging, as directed, may result in a burn. The Patient should not charge while sleeping. This may result in a burn. If the patient experiences pain or discomfort, they should cease charging and contact their physician.

Intracranial Hemorrhage. Special precautions should be taken for patients who are prone to hemorrhage including patients with coagulopathy, with high blood pressure, or who are using prescribed anticoagulants. Microelectrode penetration and DBS Lead insertion can put patients who have a likelihood of intracranial hemorrhages at greater risk.

Magnetic Resonance Imaging. Patients implanted with the full Vercise DBS System (Leads, Extensions, and Stimulator) should not be subjected to Magnetic Resonance Imaging (MRI). MRI exposure may result in:

- Dislodgement of implanted components.
- Heating of the contacts, or other system components, causing permanent tissue lesioning.
- Damage to the Stimulator's electronics.
- Current induction through the DBS Leads and Vercise DBS System components causing unpredictable levels of stimulation.
- Distortion of the diagnostic image.
- Personal injury or even death.

Note: *Vercise DBS Lead-only system (before Stimulator is implanted) is MR conditional. An MRI examination can be conducted safely when all instructions in the supplemental manual MRI Guidelines for Boston Scientific DBS Systems are followed.*

For the latest version of the manual go to www.bostonscientific.com/manuals.

Other Active Implantable Devices. Stimulators, such as the Vercise™ Stimulator, may interfere with the operation of implanted sensing stimulation devices such as pacemakers or cardioverter defibrillators. The effects of implanted stimulation devices on neurostimulators, such as the Vercise DBS System, are unknown.

Pregnancy. It is unknown whether this device may cause complications with pregnancy and/or hurt an unborn baby.

Status Dystonicus (Dystonia Indication). Patients with dystonia require careful monitoring for increase in symptom severity. Life-threatening status dystonicus (also called dystonic storm or dystonic crisis) can rarely occur. Status dystonicus may be triggered by the loss of DBS therapy as well as by other factors and may lead to respiratory distress or failure, rhabdomyolysis, multi-organ failure, and death. Ensure that patients and caregivers understand the importance of maintaining DBS therapy and that its loss for any reason (accidental turn-off, battery depletion, system failure) could cause symptom return and possible rebound, including status dystonicus.

Patients and caregivers should know how to use the Remote Control to ensure the system is on and providing stimulation. Patients and caregivers also need to understand the correct method for recharging the Stimulator. If symptoms return, the patient/caregiver should contact the healthcare provider immediately.

Stimulator Damage. Chemical burns may result if the Stimulator housing is ruptured or pierced, exposing the patient's tissue to battery chemicals. Do not implant the Stimulator if the housing is damaged.

Suicide. Depression, suicidal ideation, and suicide are known risks of DBS. Consider adjustment of stimulation, discontinuing stimulation, adjusting medication, and/or psychiatric referral.

Unauthorized Modification. Unauthorized modification to the medical devices is prohibited. System integrity could be compromised and harm or injury to the patient could occur if the medical devices are subjected to unauthorized modification.

Precautions

Physician training is required for usage of the Vercise™ DBS System. The implanting physician should be experienced in the subspecialty of Stereotactic and Functional Neurosurgery. The following is a list of precautions that should be taken when implanting or using the DBS Stimulator.

Bathing. Patients should exercise reasonable caution when bathing.

Cell Phones and other Portable RF Communication Devices. While interference caused by cell phones is not anticipated, the full effects of interaction with cell phones are unknown at this time. Patients should be instructed that portable RF communications equipment (for example, mobile phones) should be kept a minimum distance of 6 inches (15 cm) from the area of the implanted device. If interference does occur, move the cell phone away from the implanted Stimulator or turn off the phone. If there is a concern or a problem is encountered, patients should contact their physician.

Cleaning the Charging Collar. Hand wash the Charging Collar with mild soap and warm water. Do not machine wash the charging collar. Let the Charging Collar air dry. Be sure to remove the Charger and Counterweight from the Charging Collar before washing the Charging Collar.

Cleaning the Remote Control, Charger, Base Station, & Power Supply. The components can be cleaned using alcohol or a mild detergent applied with a cloth or tissue. Residue from soapy detergents should be removed with a damp cloth. Do not use abrasive cleansers for cleaning. Do not clean the Charger, Base Station, or Power Supply while they are directly or indirectly connected to a power outlet.

Component Removal & Disposal. Any explanted components should be returned to Boston Scientific. The Stimulator should be explanted in the case of cremation and returned to Boston Scientific. Cremation may cause the Stimulator battery to explode.

The Remote Control or Charging System should not be disposed of in fire, as these components contain batteries which may explode causing injury when exposed to fire. Used batteries should be disposed of in accordance with local laws and regulations.

Components. The use of components other than those supplied by Boston Scientific and intended for use with the Vercise DBS System may: damage the system, diminish the effectiveness of therapy, and/or put the patient at unknown risk.

Connections. Before inserting any DBS Lead or DBS Extension into any connector or header ports, including the Stimulator header, DBS Extension connectors, and operating room cable assembly, always wipe the DBS Lead with a dry cotton sponge. Contamination inside the ports may be difficult to remove and can cause high impedances, preventing electrical connectivity which may compromise the integrity of the stimulation circuit.

Delayed Effectiveness. Patients may experience a delayed effect in symptom control from DBS therapy. Patients and caregivers should be made aware that optimal symptom control may require frequent programming adjustments over a period of several weeks to months.

Device Failure. Implants can fail at any time due to random component failure, loss of battery functionality, or DBS Lead breakage. Suddenly stopping brain stimulation can cause serious reactions to develop. If the Stimulator stops working even after complete charging (up to four hours when properly aligned), patients should be instructed to turn off the Stimulator and contact their physician immediately so that the system can be evaluated and appropriate medical care given to manage the return of symptoms.

Environmental Precautions. Patients should avoid activities that could potentially involve large amounts of electromagnetic interference. Devices that contain permanent magnets, such as speakers, should not be placed near the Stimulator because they may cause the system to turn on or off.

Excess DBS Extension. Coil excess DBS Extension around or below the Stimulator. Excess wire on top of the Stimulator may increase the potential for tissue erosion or damage during Stimulator replacement surgery and may interfere with charging.

Massage Therapy. Patients should avoid receiving massage therapy near the implanted system components. If a patient does receive massage therapy, the patient should inform the masseuse that they have an implanted device and show him/her where the Stimulator, DBS Extension, and DBS Leads are located. The patient should have the masseuse avoid these areas and proceed with caution.

Medical Devices/Therapies. The following medical therapies or procedures may turn stimulation off, cause permanent damage to the Stimulator, or may cause injury to the patient:

- Electrocautery – Electrocautery can transfer destructive current into the DBS Leads and/or Stimulator.
- External Defibrillation – Safe usage of external defibrillation has not been established and damage should be ascertained following defibrillation.
- Lithotripsy – High frequency signals directed near the Stimulator may damage circuitry.
- Radiation Therapy – Lead shielding should be used over the Stimulator to prevent damage from high radiation. Any damage to the device by radiation may not be immediately detectable.
- Transcranial Stimulation – Safe use of electromagnetic therapies, such as transcranial magnetic stimulation, have not been established.
- MRI – Patients implanted with the full Vercise DBS System (Leads, Extensions, and Stimulator) should not be subjected to MRI to avoid damage to the device and patient harm.

Note: *Vercise DBS Lead-only system (before Stimulator is implanted) is MR conditional. An MRI examination can be conducted safely when all instructions in the supplemental manual MRI Guidelines for Boston Scientific DBS Systems are followed.*

For the latest version of the manual go to www.bostonscientific.com/manuals.

- Diathermy – The energy generated by diathermy can be transferred to the Vercise™ DBS System and may result of damage to the device or patient harm.

If any of the above is required by medical necessity, the procedure(s) should be performed as far from the implanted components as possible. Ultimately, however, the Stimulator may require explantation as a result of damage to the device or patient harm.

Operating Temperature. The operating temperature of the External Trial Stimulator and Remote Control is 5 °C to 40 °C (41 °F to 104 °F). For proper operation, do not use the Charger if the ambient temperature is above 35 °C (95 °F).

Other Models of External Devices. Only the Remote Control, Clinician Programmer, and Charger that were provided with the Boston Scientific Vercise™ DBS System should be used with the Vercise DBS System. Other models of these devices will not function with the Vercise DBS System.

Patient Activities Requiring Coordination. Loss of coordination is a potential side effect of DBS therapy. Patients should exercise reasonable caution when participating in activities requiring coordination, including those that they were able to perform prior to receiving DBS therapy (e.g., swimming).

Patient Activity Following Surgery. During the two weeks following surgery, it is important for the patient to exercise extreme care so that appropriate healing will secure the implanted components. During this period, the patient should not attempt to move heavy objects. Instruct the patient to restrict head movements, including extension or flexion of the neck and rotation of the head, until healing is complete.

Resterilization. For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient. If resterilization is desired, obtain a new sterile device for implantation.

After use, return the Stimulator, DBS Leads, and DBS Extensions to Boston Scientific and dispose of other components and packaging in accordance with hospital, administrative and/or local government policy.

Check the expiration date on the package before opening the sterile package and using the contents. Do not use the contents if the current date is past the expiration date, if the package is opened or damaged, or if contamination is suspected because of a defective sterile package seal.

- Inspect the seal integrity of the outer tray before use.
- Inspect the seal integrity and sterile indicator on the inner tray. The sterile indicator will be green with red stripes if sterile. Yellow stripes indicate the tray is not sterile. If the tray is not sterile, do not use the components and return to Boston Scientific.
- Open the inner tray in the sterile field.

- If the Stimulator was dropped, do not implant it in a patient. The dropped Stimulator may have lost sterility, experienced a loss of hermeticity, or been otherwise damaged. Replace the dropped Stimulator with a new, sterile Stimulator prior to implantation. Return the damaged Stimulator to Boston Scientific.
- Do not use any component that shows signs of damage.
- Do not use if “Use By” date has expired.

Setscrews. Before tightening Setscrews, always test impedance to confirm electrical connectivity. Tightening a Setscrew onto a contact may damage the contact and may result in the need to replace the DBS Lead or DBS Extension.

Sterilization. Contents of the surgical kits are supplied sterile using an ethylene oxide process. Do not use if sterile barrier is damaged. If damage is found, call your Boston Scientific representative and return the damaged part to Boston Scientific.

Stimulator Orientation. To ensure proper charging, orient the Stimulator parallel to the skin surface and at a depth less than 2 cm below the skin. The etched writing “This Side Up” must be facing out of the pocket towards the patients skin. Suboptimal placement of the Stimulator may result in the inability to recharge and/or a revision surgery.

Never attempt to change the orientation of or turn over the Stimulator. Patients should avoid touching the Stimulator site or incisions. If the Stimulator flips over in the body, then it cannot be charged. If stimulation cannot be turned on after charging, the Stimulator may have changed orientation or rotated; patients should contact their physician to arrange an evaluation of the system.

If a patient notices a change in appearance of the skin at the Stimulator location, such as the skin becoming thin over time, they should contact their physician.

Storage, Handling and Transport. Store implanted components like Stimulators, Leads, and Extensions between 0 °C to 45 °C (32 °F to 113 °F) in an area where they are not exposed to liquids or excessive moisture. Temperatures outside of the stated range can cause damage. If stored in conditions beyond the required storage temperature, do not use the components and return to Boston Scientific.

Do not expose the Remote Control or Charging System components to excessively hot or cold conditions. Do not leave the devices in your car or outdoors for extended periods of time. The sensitive electronics can be damaged by temperature extremes, particularly high heat. If the Remote Control or the Charging System is to be stored for a period of time, be careful that the storage temperature does not exceed -20 °C to 60 °C (-4 °F to 140 °F).

Handle the system components and accessories with care. Do not drop them or submerge them in water. Accessories, including the Remote Control and Charger, must be kept dry and not exposed to moisture. Although reliability testing has been performed to ensure quality manufacturing and performance, dropping the devices on hard surfaces or in water, or other rough handling, can permanently damage the components. Keep the Remote Control and Charger away from pets, pests, and children to avoid damage to the devices.

Care must be taken to avoid damaging the DBS Lead with sharp instruments or excessive force during surgery. The following guidelines will help to ensure the longevity of components:

- Do not sharply bend or kink the DBS Lead or Extension.
- Do not tie suture(s) directly to the DBS Lead or Extension body.
- Avoid pulling an implanted DBS Lead taut; stress relief loops may help to minimize tension on the DBS Lead.
- Avoid handling the DBS Lead with sharp instruments; use only rubber-tipped forceps.
- Take care when using sharp instruments, such as hemostats or scalpels to prevent damaging the DBS Lead.

Surgical Tape. If tape is used to temporarily secure the DBS Lead during surgery, caution should be used to ensure the Lead is not cut or damaged when removing the tape.

Sutures. Do not apply sutures tightly around the DBS Leads, as this may damage the DBS Leads' insulation and may result in DBS Lead failure.

Tissue Reaction. Temporarily, there may be some pain in the area of the Stimulator as the incisions heal. If there is excessive redness around the wound area, it should be checked for infection. In rare cases, adverse tissue reaction to implanted materials can occur.

Adverse Events

The following is a list of known risks with the use of deep brain stimulation. There may be risks that are unknown. Specific adverse events may be more relevant depending on the indication and site of stimulation. Note that some of these symptoms may be resolved or reduced by current steering, changing stimulation parameters, or by changing the position of the Lead during surgery.

If any of these events occur, patients should contact their physician as soon as possible to inform them.

- Allergic or immune system response
- Anesthesia/neurosurgery risks, including unsuccessful implant, exposure to bloodborne pathogens
- CSF leak
- Death, including suicide
- Embolism, including air embolism and pulmonary embolism
- Failure or malfunction of any of the device components or the battery, including but not limited to Lead or Extension breakage, hardware malfunctions, loose connections, electrical shorts or open circuits and Lead insulation breaches, whether or not this requires explant and/or reimplantation
- Hemorrhagic or ischemic stroke, immediate or delayed, which could result in temporary or permanent neurologic deficits such as muscle weakness, paralysis or aphasia
- Implant site complications such as pain, poor healing, wound reopening
- Infection
- Injury to tissues adjacent to implant or within surgical field, such as blood vessels, peripheral nerves, brain (including pneumocephalus), or pleura (including pneumothorax)
- Interference from external electromagnetic sources
- Lead, Extension (including Extension header) and neurostimulator erosion or migration
- Loss of adequate stimulation
- Mentation impairment such as attention or cognitive deficits, memory disturbances, or confusion
- Motor problems such as paresis, weakness, incoordination, restlessness, muscle spasms, postural and gait disorders, tremor, dystonia, or dyskinesias, and falls or injuries resulting from these problems
- Musculoskeletal stiffness
- Neuroleptic malignant syndrome or acute akinesia can occur very rarely
- Overstimulation or undesirable sensations, such as paresthesia, transient or persistent
- Pain, headache or discomfort, transient or persistent, including symptoms due to neurostimulation

- Poor initial Lead location
- Psychiatric disturbances such as anxiety, depression, apathy, mania, insomnia, suicide, or suicidal ideation or attempts
- Radiation exposure due to imaging (CT, fluoroscopy x-ray)
- Seizures
- Sensory changes
- Seroma, edema or hematoma
- Skin irritation or burns at neurostimulator site
- Speech or swallowing problems such as dysphasia, dysarthria or dysphagia, as well as complications of dysphagia such as aspiration pneumonia
- Status dystonicus
- Systemic symptoms-autonomic (tachycardia, sweating, fever, dizziness), changes in renal function, urinary retention, sexual effects, gastrointestinal (nausea, bowel retention, bloating)
- Thrombosis
- Visual disturbances or periorbital symptoms, such as diplopia, eyelid movement difficulty, oculomotor difficulties, or other visual field effects
- Weight changes

Package Contents

Contents of the Implantable Pulse Generator Kit

- Stimulator
- Stimulator Template
- Stimulator Header Plugs (2)
- Torque Wrench

Note: All contents of the inner package (or tray) are sterile.

Contents of the Lead Kit (30 cm and 45 cm)

- DBS Lead with preloaded Straight Stylet
- Torque Wrench
- Lead Boot
- Lead Stop – Screw and Ring
- 1 cm Suture Sleeve
- 1 cm Split Suture Sleeve
- 2.3 cm Suture Sleeve
- 4 cm Suture Sleeve

Note: All contents of the inner package (or tray) are sterile and non-pyrogenic.

Contents of the Extension Kit (55 cm)

- DBS Extension
- Torque Wrench
- Tunneling Tool Shaft (with Pre-Loaded Straw)
- Tunneling Tool Handle

Note: All contents of the inner package (or tray) are sterile.

Contents of the Tunneling Tool Kit (28 cm, Straw and 35 cm, Long)

- Tunneling Tool Shaft (with Pre-Loaded Straw)
- Tunneling Tool Handle

Note: All contents of the inner package (or tray) are sterile.

Contents of the Spares Kit


- Lead Boot
- Lead Stop – Screw and Ring
- Torque Wrench
- 1 cm Suture Sleeve
- 1 cm Split Suture Sleeve
- 2.3 cm Suture Sleeve
- 4 cm Suture Sleeve

Note: All contents of the inner package (or tray) are sterile.

Other Components

- Remote Control with Batteries
- Remote Control Case
- Charger
- Base Station
- Power Supply
- Charging Collar (Small and Medium)
- Adhesive Patches
- Charging Spacer and Counterweight
- Clinician Programmer
- External Trial Stimulator
- External Trial Stimulator Battery
- OR Cable and Extension
- IR Interface
- IR Interface Holder
- USB to Serial Cable
- Patient Travel Case
- International Power Converter
- Remote Control Kit (Remote Control, Batteries, Case)
- Charging Kit (Charger, Base Station, Power Supply, Charging Collar)

DBS Lead Implantation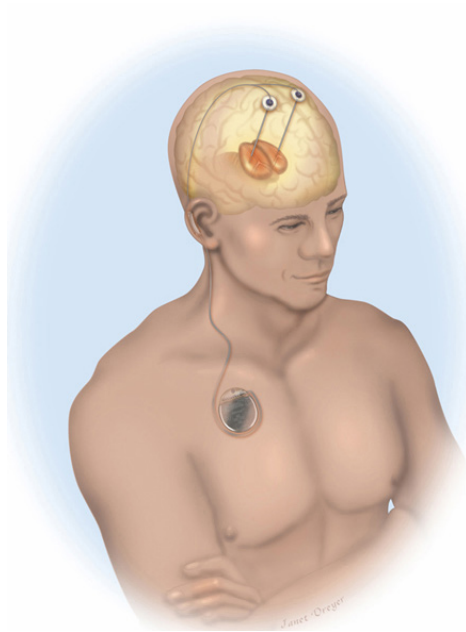

This section describes the recommended procedures for implanting the Vercise™ DBS System. Procedures for DBS Lead insertion and intra-operative testing are outlined, followed by DBS Extension tunneling and Stimulator placement.

Note: *Utilize meticulous care during implantation of the Vercise DBS System to prevent infection.¹*

¹ **Note:** For additional information regarding recommended practices for the DBS procedure, please see the following references -

Seijo F, Alvarez-Vega M, Lozano B, et al. (2009). “Common Questions and Answers to Deep Brain Stimulation Surgery” In Rogers & Anderson (eds.), *Deep Brain Stimulation*. (pp. 1-29). New York, NY: Nova Science Publishers, Inc.

Starr PA & Sillay K (2008). “Complication Avoidance and Management in Deep Brain Stimulation Surgery” In Tarsy, Vitek, Starr & Okun (eds.), *Deep Brain Stimulation in Neurological and Psychiatric Disorders*. (pp. 135-150). Totowa, NJ: Humana Press.


Umemura A (2007). “Complications and Avoidance” In Baltuch & Stern (eds.), *Deep Brain Stimulation for Parkinson’s Disease*. (pp. 103-112). New York, NY: Informa Healthcare

Pre-Conditions

The described implant procedures start with implanting the DBS Lead. It is assumed that the following procedures have been completed:

- The stereotactic frame and/or fiducials of a frameless system are attached to the patient.
- The desired trajectory of the DBS Lead insertion path has been determined.
- The incision in the scalp has been made and the burr hole drilled.
- If using the SureTek™ Burr Hole Cover, the Base of the Burr Hole Cover has been affixed over the burr hole. (See the manual provided with the Burr Hole Cover Kit for instructions for use.)
- The desired trajectory and DBS Lead depth may have been verified by microelectrode recording or an appropriate means.

Implanting the DBS Lead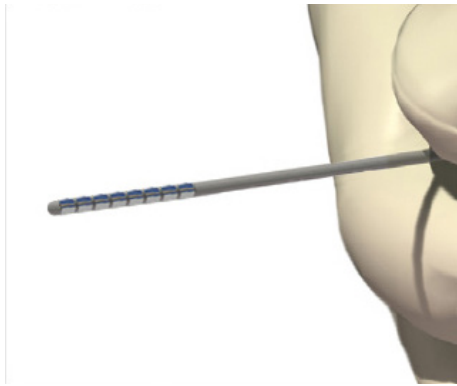

The DBS Leads will be implanted and therapeutic benefit verified.


1. Prepare DBS Lead for implant. Visually inspect the DBS Lead and determine it to be acceptable for implantation.
2. Pass the DBS Lead through the cannula to ensure proper fit.
3. Insert the cannula (with stylet) into the brain to the desired depth.

Note: *Cannula depth depends on physician's preference.*

4. Assemble the Lead Stop by partially screwing the threaded portion of the Screw into the threaded hole in the Ring.
5. Measure the desired depth of DBS Lead with a gauge or ruler and apply the DBS Lead Stop at that length.

To apply the DBS Lead Stop, push the DBS Lead to the center of the Lead Stop and then tighten the Screw. This will ensure that the DBS Lead will be inserted to the proper depth.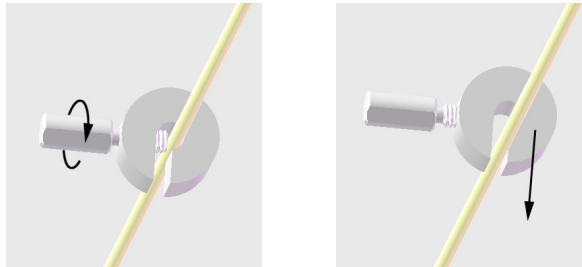


Note: Make sure the Lead Stop will not slide on the DBS Lead when engaged.

6. Insert the DBS Lead, with the stylet in place, into the cannula.
7. Insert the DBS Lead and the cannula into the cannula guide on the microdrive.
8. Attach the DBS Lead to the microdrive.
9. Check that the locking lever on the OR Cable connector is in the open (0) position.
10. Slide the proximal end of the DBS Lead, with stylet, into the open port on the OR Cable connector.

Note: The descriptors “proximal” and “distal” use the Stimulator as the reference throughout this manual.

11. Push the end of the DBS Lead into the port until it stops. Hold the DBS Lead in place while sliding the locking lever to the locked (1) position.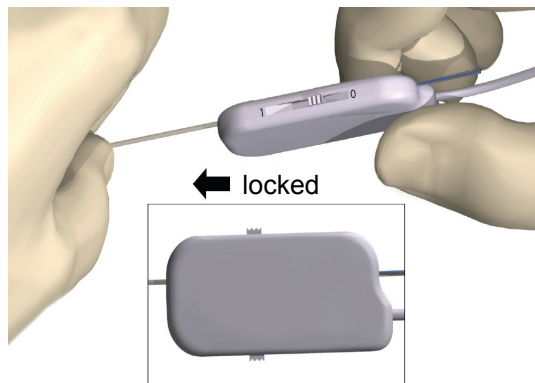


12. Secure the OR Cable connector to the microdrive.

Note: Make sure the stylet is inside the DBS Lead before advancing the Lead to the desired target.

13. Slowly advance the DBS Lead to the desired target using the microdrive.

Intraoperative Stimulation Testing

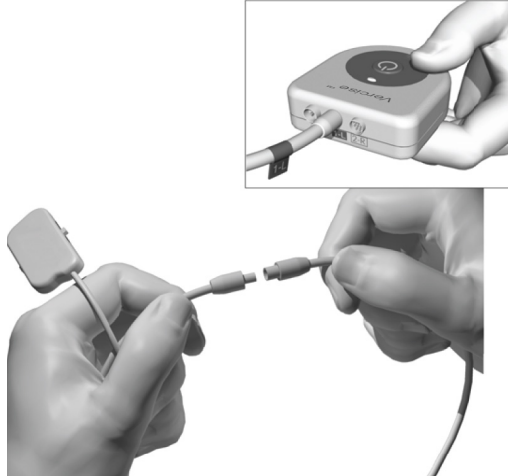The External Trial Stimulator, OR Cable, OR Cable Extension, and Clinician Programmer (CP) are used to conduct intraoperative stimulation testing during the procedure. The OR Cable Extension is designed for temporary connection to the OR Cable to facilitate stimulation testing outside of the sterile field.

Note: *The following steps are for procedural reference only. Please refer to the DBS Programming Manual for detailed stimulation testing procedures and guidelines.*

CAUTION: Do not immerse the OR Cable connector or plug in water or other liquids. The OR Cable is intended for one-time use only; do not resterilize.

1. Attach the OR Cable Extension to the OR Cable.
2. Verify that the External Trial Stimulator is off.

WARNING: Always turn the External Trial Stimulator off before connecting or disconnecting the Cable Assemblies to prevent unexpected stimulation.

3. Plug the OR Cable into the External Trial Stimulator socket labeled “1-L.” If two DBS Leads are being used, connect the left DBS Lead to socket “1-L” and the right DBS Lead to socket “2-R.”

4. Verify impedances are acceptable by using the CP or Remote Control to measure monopolar impedances.

If using the CP, high impedance contacts will contain a red X. If using the Remote Control, high impedance contacts are represented by hollow rectangles.

Note: If using the Remote Control, press ▼ to view the impedance values.

5. Evaluate DBS Lead placement.
 - a. Evaluate whether stimulation is appropriate to reduce symptoms.
 - b. Adjust the DBS Lead location or stimulation parameters if necessary.

Note: The stylet shall remain in place throughout DBS Lead insertion and adjustments.

WARNING: High charge density can cause permanent tissue damage. A warning will pop-up on the Clinician Programmer screen if the stimulation parameters will cause the charge density to exceed $30 \mu\text{C}/\text{cm}^2$.


WARNING: Increasing the number of DBS Lead penetrations increases the probability of hemorrhage. The necessity for an acute DBS Lead revision should be minimized using techniques of target localization, such as microelectrode recordings and imaging, to correctly place the DBS Leads on the first attempt.

6. Turn off the External Trial Stimulator.

WARNING: A sudden increase in stimulation may occur if External Trial Stimulator is ON while disconnecting the OR Cables.

7. Disconnect the OR Cable and OR Cable Extension from the proximal end of the DBS Lead.
8. Verify that the DBS Lead has not moved from the desired location.

Securing the DBS Lead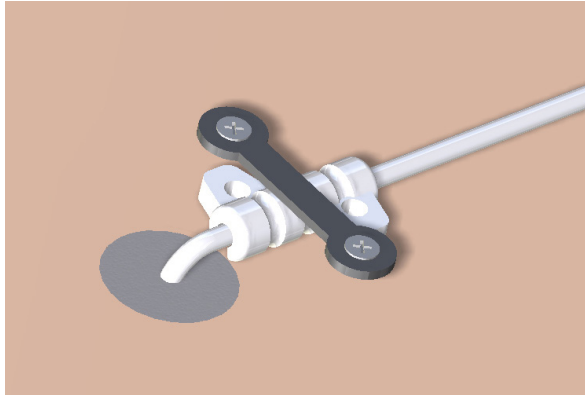

Once a DBS Lead has been placed, it should be secured.

CAUTION: While securing the DBS Lead, use care not to move it.

1. Remove the Lead Stop by unscrewing the Screw and detaching the Lead Stop from the DBS Lead.
2. Slowly retract the cannula to just above the burr hole by sliding it over the proximal portion of the DBS Lead. Be careful not to move the DBS Lead.

Note: The descriptors “proximal” and “distal” use the Stimulator as the reference throughout this manual.

3. Fix the Lead in place. The SureTek™ Burr Hole Cover Kit is recommended for use with the Vercise™ System. (An appropriate commercially available filler may also be used.²) Fill the burr hole with an appropriate commercially available filler, such as hydroxyapatite (not provided), using the manufacturer’s instructions.³

Note: Remove any bone wax from the bone interface before applying the adhesive.

Note: Ensure that the adhesive is level with the superior surface of the skull.


4. Allow the adhesive to set according to the manufacturer’s instructions.
5. While holding the DBS Lead with rubber-shod forceps, remove the DBS Lead implant components.
 - a. Remove the cannula.
 - b. Remove the microdrive system.
 - c. Remove the stereotactic frame.
 - d. Remove the stylet.

WARNING: Do not reinsert the stylet into the DBS Lead while the DBS Lead is in the brain, as this may damage the DBS Lead and/or cause patient harm.

² DBS Lead secured and tested utilizing Biomet Mimix QS bone void filler, a Stryker 12 mm titanium mini plate, Stryker titanium screws, and a Boston Scientific 1 cm split suture sleeve. Data on file.

³ DBS Lead secured and tested utilizing Biomet Mimix QS bone void filler, a Stryker 12 mm titanium mini plate, Stryker titanium screws, and a Boston Scientific 1 cm split suture sleeve. Data on file.

6. Verify that the DBS Lead has not moved from the desired location.
7. Place the split suture sleeve onto the DBS Lead. Be careful not to move the DBS Lead when placing the suture sleeve.
8. Rotate the suture sleeve until the slit is facing the skull.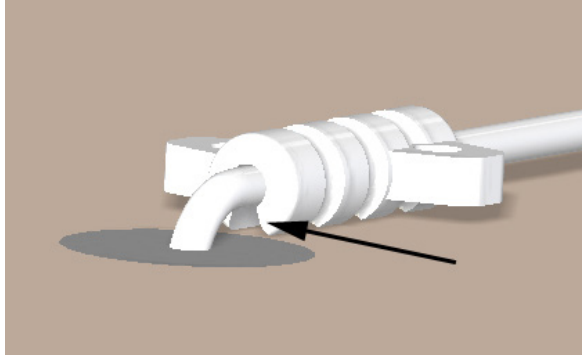


Note: *If the slit is not located properly, the DBS Lead may not be protected by the suture sleeve and may be in direct contact with the mini plate.*

9. Place a mini plate (not provided) over the suture sleeve and DBS Lead.

CAUTION: Do not place the mini plate in direct contact with the DBS Lead. Ensure that the suture sleeve lies between the mini plate and the DBS Lead.


10. Secure the mini plate to the skull per the manufacturer's instructions.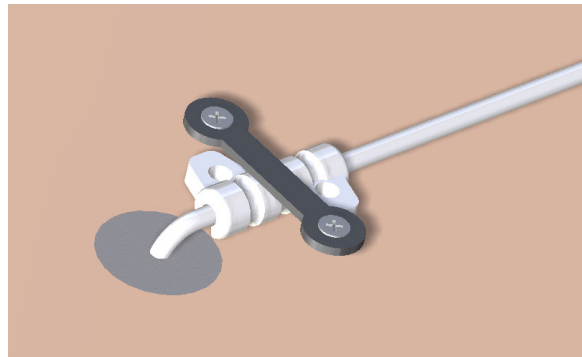


11. If the Stimulator will be implanted during a separate surgery, prepare the DBS Lead for the “*Stimulator Implantation*” procedure.

- a. Place a Suture Sleeve on the left DBS Lead to differentiate the Leads.
- b. Insert proximal end of the DBS Lead into the Lead Boot until it stops.

Note: *Be sure to fully insert the proximal tip of the DBS Lead into the Lead Boot (see below) so that the retention sleeve is located under the Setscrew.*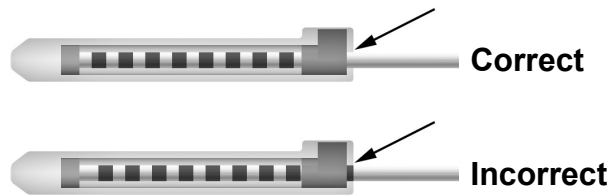

Note: *The retention sleeve is easily distinguishable from the contacts by its length (see below).*

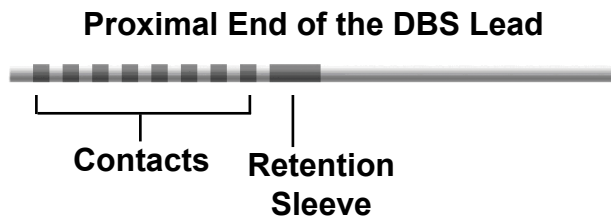- c. Pass the Torque Wrench through the slit in the septum located on the top of the Lead Boot.
- d. Tighten the Setscrew until the Torque Wrench clicks, indicating the Setscrew is fully secured.

Note: *To tighten the Setscrew, use one hand to grasp the Lead Boot at the base and the other to rotate the Torque Wrench clockwise until it clicks, indicating the Setscrew is fully secured. To loosen the Setscrew, rotate the Torque Wrench counterclockwise.*

CAUTION: The wrench is torque limiting, such that the Setscrew cannot be over tightened. Use only the wrench provided, as other tools may over-tighten the Setscrew and damage the DBS Lead.

- e. Create a tunnel, for the proximal end of the DBS Lead, to a place closer to the desired DBS Extension connector location.

Note: *Boston Scientific recommends placing the DBS Extension connector behind the ear.*

CAUTION: Placement of the DBS Extension connector in the neck region can increase the risk of device failure due to repetitive movement of the neck.

- f. Create a pocket under the skin for the excess DBS Lead and Lead Boot.
- g. Coil excess DBS Lead material under the scalp, in the pocket, until it is ready to be connected to the DBS Extension.

Note: *The DBS Lead may be connected to the DBS Extension and Stimulator during a separate surgery at a later time. See “Stimulator Implantation” on page 24.*

12. Repeat the “*Implanting the DBS Lead*” procedure for the second DBS Lead. Use the Tunneling Tool to tunnel the second DBS Lead to the same side as the first Lead.

13. Close the incisions.

Stimulator Implantation

The Stimulator may be implanted immediately following DBS Lead implantation and intra-operative testing or during a separate surgery.


Exposing the DBS Lead

1. Palpate the DBS Lead Boot and DBS Lead under the scalp.
2. Mark and create an incision in the scalp to expose the Lead Boot. Be careful not to damage or cut the DBS Lead.
3. Expose the DBS Lead and Lead Boot through the incision.
4. Using the Torque Wrench, remove and discard the Lead Boot.


Note: *To loosen the Setscrew, rotate the Torque Wrench counterclockwise. To tighten the Setscrew, rotate the Torque Wrench clockwise.*

5. Dry the proximal end of the DBS Lead.

Connecting the DBS Lead to the DBS Extension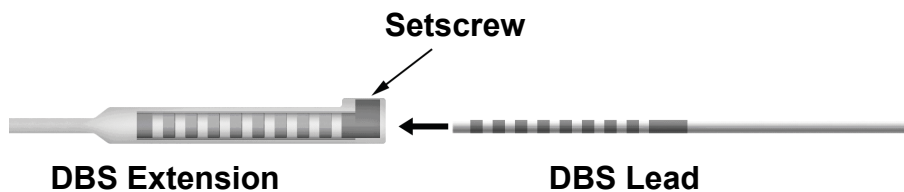

1. Check to ensure that the Setscrew is not restricting the entry port on the DBS Extension connector by unscrewing the Setscrew one to two turns with the Torque Wrench. Grip the DBS Lead next to the Retention Sleeve.

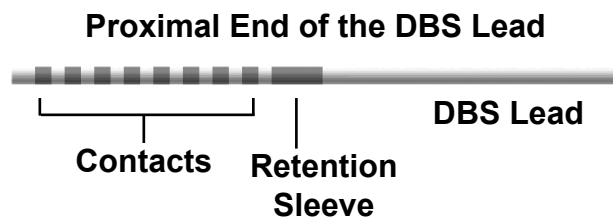Note: The retention sleeve is easily distinguishable from the contacts by its extended length.

2. Push the DBS Lead into the DBS Extension connector until the DBS Lead electrodes line up with the DBS Extension contacts. Some resistance may be felt as each electrode enters into the DBS Extension Connector. You should be able view the DBS Lead electrodes as they pass through the DBS Extension Connector. Some additional resistance may be felt as the last electrode aligns into place.

3. Visually check to ensure that the DBS Lead electrodes are aligned with the DBS Extension contacts. If they are not aligned, continue to grip the DBS Lead next to the Retention Sleeve and push to advance the electrodes into alignment with the DBS Extension contacts. If necessary, back out the Lead slightly and then advance the electrodes into alignment again, until proper alignment can be confirmed.

Note: Be sure to fully insert the DBS Lead into the connector so that the retention sleeve is located under the Setscrew.

4. Do not tighten the Setscrew at this time.
5. Repeat steps 1 through 3 to connect the second DBS Lead to the second DBS Extension.

6. Test the impedance of the connection to ensure that you have properly aligned the DBS Lead within the DBS Extension connector. Use the External Trial Stimulator, OR Cable, OR Cable Extension, and Clinician Programmer (CP) to test the impedance. The OR Cable Extension is designed for temporary connection to the OR Cable to facilitate stimulation testing outside of the sterile field.

Note: *The following steps are for procedural reference only. Please refer to the DBS Programming Manual for detailed impedance testing procedures and guidelines.*

CAUTION: Do not immerse the OR Cable connector or plug in water or other liquids. The OR Cable is intended for one-time use only; do not resterilize.

7. Attach the OR Cable Extension to the OR Cable.

8. Verify that the External Trial Stimulator is off.

WARNING: Always turn the External Trial Stimulator off before connecting or disconnecting the Cable Assemblies to prevent unexpected stimulation.

9. Plug the OR Cable into the External Trial Stimulator socket labeled “1-L.” If two DBS Leads are being used, connect the left DBS Lead to socket “1-L” and the right DBS Lead to socket “2-R.”


10. Verify impedances are acceptable by using the CP or Remote Control to measure monopolar impedances.

If using the CP, high impedance contacts will contain a red X. If using the Remote Control, high impedance contacts are represented by hollow rectangles.

Note: *If using the Remote Control, press ▼ to view the impedance values.*

Assembling the Tunneling Tool

A Tunneling Tool and Straw are provided to facilitate tunneling of the DBS Extension.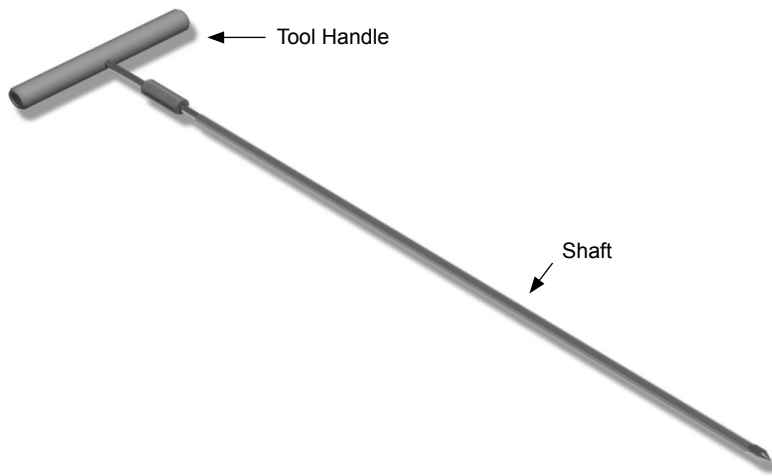

1. Attach the Tunneling Tool Handle to the Shaft by turning the locking mechanism clockwise.
 - a. Push the locking mechanism at the base of the Tool Handle onto the Shaft.
 - b. Grasping the Tool Handle and the Tip of the Tunneling Tool, rotate the Shaft back and forth until the handle seats onto the Shaft.
 - c. While firmly grasping the Tip of the Tunneling Tool to hold the Shaft stationary, turn the locking mechanism clockwise until secure.

Tunneling the DBS Lead and Extension

1. Create a pocket for the Stimulator under the skin in a location inferior to the clavicle on the same side as the DBS Lead and Extensions.

Note: *Boston Scientific recommends implanting the Stimulator subclavicularly.*

- a. Mark the location of the pocket.
- b. Use the template to outline the intended pocket to guide the optimal pocket sizing.
- c. Make the pocket no deeper than 2 cm; Stimulator charging could become ineffective at greater depths.

Note: *It is important to keep the pocket small to prevent the Stimulator from turning over.*

2. Mark a tunneling route from the location of the subclavicular pocket to the incision superior to the ear.
3. Administer appropriate local anesthetic along the tunneling route.

CAUTION: Be sure not to puncture or damage the DBS Lead or other components when administering local anesthetic.


4. Create a subcutaneous tunnel from the incision above the ear, along the tunneling path to the Stimulator pocket.

WARNING: Be careful not to puncture or damage important structures along the tunneling path, such as the brachial plexus and jugular, as this may cause patient harm.

5. If desired, bend the Tunneling Tool to an appropriate shape.

CAUTION: Do not bend locking joints.

6. Once the Tip of the Tunneling Tool is completely exposed, unscrew and remove the Tunneling Tool Handle.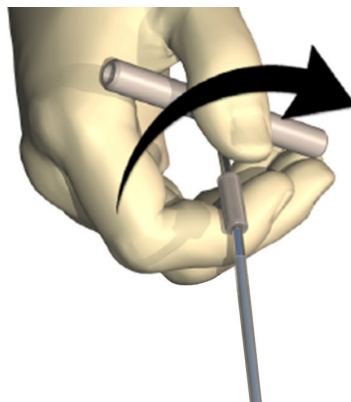

7. Grasp the Tip firmly with one hand and, while holding the Straw in place with the other hand, pull the Shaft out of the Straw.
8. Push the proximal ends of the DBS Extensions through the Straw, and then withdraw the Straw.
9. Optionally secure the DBS Extension connector to the fascia using sutures and/or suture sleeves.

CAUTION: Do not use polypropylene sutures as they may damage the suture sleeve. Do not suture directly onto the DBS Extension or use a hemostat on the DBS Extension body. This may damage the DBS Extension insulation.

Connecting the Stimulator


Dual DBS Lead Connection

Connect Left DBS Extension to port 1-L

Connect Right DBS Extension to port 2-R


1. Fully insert the male end of the DBS Extension into the Stimulator until it stops.
 - a. Insert the header plug to verify no Setscrews obstruct the socket.
 - b. Wipe the DBS Extension contacts before inserting.
 - c. Insert the DBS Extensions into the header. When fully inserted, the tip of the DBS Extension will slide into the back of the port and the retention sleeve on the DBS Extension will be located under the Setscrew.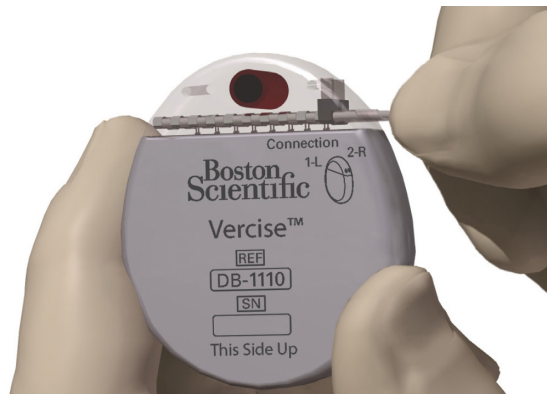


CAUTION: Verify proper DBS Extension insertion by checking impedances before tightening the Setscrew. Tightening the Setscrew down onto a contact can damage the DBS Extension.

2. Check impedances to verify connections before tightening the Setscrew.
 - a. Place the Stimulator partially in the subcutaneous pocket.
 - b. Test impedances using the Remote Control or Clinician Programmer.

- Verify that the retention sleeve on the DBS Extension is located directly under the Setscrew in the Stimulator header.

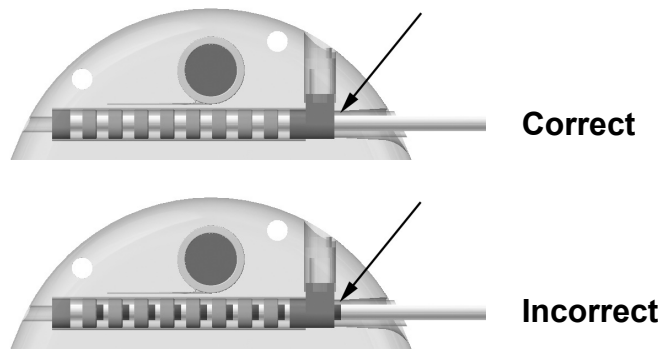
Note: The retention sleeve is easily distinguishable from the contacts by its length (see below).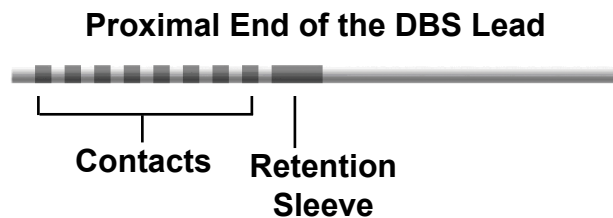

- Pass the Torque Wrench through the slit in the septum located on the top of the Stimulator header.
- Tighten the Setscrew in the Stimulator until the Torque Wrench clicks, indicating the Setscrew is fully secured.

Note: To tighten the Setscrew, rotate the Torque Wrench clockwise. To loosen the Setscrew, rotate the Torque Wrench counterclockwise.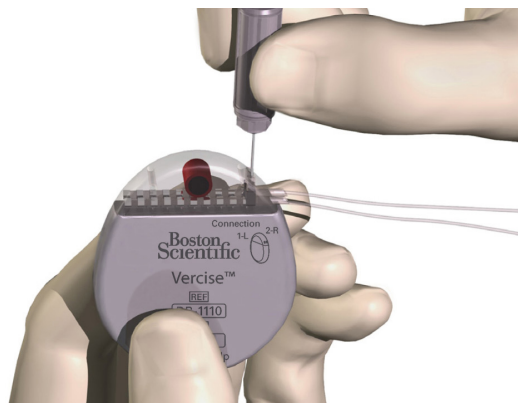

CAUTION: The wrench is torque limiting, such that the Setscrew cannot be over tightened. Use only the wrench provided, as other tools may over-tighten the Setscrew and damage the DBS Lead.


6. Visually check to ensure that the DBS Lead electrodes are aligned with the DBS Extension contacts. If they are not aligned, continue to grip the DBS Lead next to the Retention Sleeve and push to advance the electrodes into alignment with the DBS Extension contacts. If necessary, back out the Lead slightly and then advance the electrodes into alignment again, until proper alignment can be confirmed.

7. Check impedances to verify connections before tightening the Setscrew.

CAUTION: Verify proper DBS Extension insertion by checking impedances before tightening the Setscrew. Tightening the Setscrew down onto a contact can damage the DBS Extension.

8. Pass the Torque Wrench through the slit in the septum located on the top of the DBS Extension connector.
9. Tighten the Setscrew in the DBS Extension connector until the Torque Wrench clicks, indicating the Setscrew is fully secured.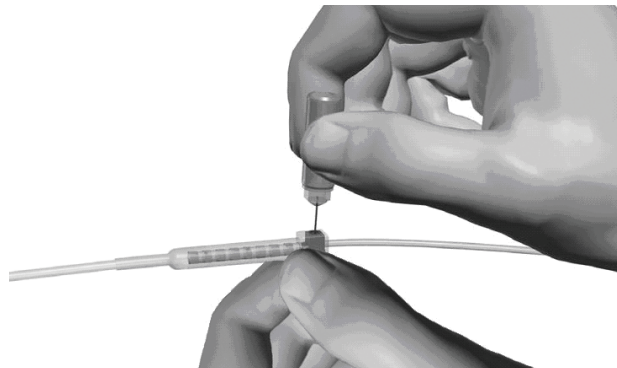

CAUTION: The wrench is torque limiting, such that the Setscrew cannot be over tightened. Use only the wrench provided, as other tools may over-tighten the Setscrew and damage the DBS Lead.

Note: To tighten the Setscrew, use one hand to grasp the Extension at the base and the other to rotate the Torque Wrench clockwise. To loosen the Setscrew, rotate the Torque Wrench counterclockwise.

10. Repeat steps 1 – 9 to tighten second Stimulator Setscrew and the Setscrew on the second DBS Extension.

11. Place the Stimulator in the subcutaneous pocket with the etched writing “This Side Up” facing the skin, and parallel to the skin surface.

WARNING: Failure to orient the correct side of the Stimulator towards the skin may result in the inability to charge and/or a revision surgery.

a. Coil the excess DBS Extension length around the Stimulator perimeter.

WARNING: Avoid placing the excess DBS Extension length on the superficial surface of the Stimulator, as this may increase the potential for tissue erosion or damage during Stimulator replacement surgery and may interfere with charging.

b. Optionally secure the Stimulator to the fascia by suturing through the holes in the Stimulator header.

12. Close the incisions.

CAUTION: Be careful not to damage the DBS Lead, Stimulator, or other implanted components when closing the incisions.

Note: *When closing the incision over the extension connector, orient the extension connector to minimize the profile under the skin.*

Vercise™ DBS System Revisions & Explantation

If the entire Vercise DBS System (Stimulator, DBS Extensions, and DBS Leads) is to be removed, then the DBS Leads should be removed first (as described below) followed by the DBS Extensions, and lastly the Stimulator. This order should reduce the potential spread of infection toward the skull opening.

Explanting the DBS Lead

WARNING: When explanting the Vercise DBS System, the DBS Lead should be pulled from the site above the ear and not the site near the burr hole to avoid a potential spread of infection toward the skull opening.

1. Turn off the Stimulator.
 2. Palpate the scalp to locate the mini plate.
 3. Make an incision near the mini plate to expose the mini plate and DBS Lead. Be careful not to damage or cut the DBS Lead or suture sleeve.
 4. Cut the DBS Lead at a distance about 2-3 cm from the mini plate, leaving enough length to grasp the Lead.
 5. Unscrew the screws anchoring the mini plate and gently remove the mini plate.
 6. Use appropriate methods to remove the DBS Lead from the adhesive.
 7. Slowly and gently retract the DBS Lead from the neural tissue, pulling as close to perpendicular to the skull as possible. The DBS Lead should experience minimal resistance when retracted.
 8. Palpate the region under the scalp to locate the DBS Extension connector.
 9. Create an incision to expose the DBS Lead and DBS Extension connector. Be careful not to damage the implanted components to allow for proper analysis following explant.
 10. Loosen the connector Setscrew on the DBS Extension using the Torque Wrench provided.
- Note:** *Be sure to fully insert the Torque Wrench before loosening the Setscrew.*
- Note:** *To tighten the Setscrew, rotate the Torque Wrench clockwise. To loosen the Setscrew, rotate the Torque Wrench counterclockwise.*
11. Remove the DBS Lead from the DBS Extension.
 12. Gently pull the remainder of the DBS Lead through the incision behind the ear.

WARNING: The DBS Lead should be pulled from the site behind the ear and not the site near the burr hole to avoid a potential spread of infection toward the skull opening.

13. If you are replacing the DBS Lead, follow the instructions in the “*Implanting the DBS Lead*” section.

If you are explanting the entire Vercise™ DBS System, continue on to the “*Explanting the DBS Extensions*” procedure.

Otherwise, close the incisions.

14. Ship the explanted DBS Leads to Boston Scientific.

Explanting the DBS Extensions

1. Turn off the Stimulator.
2. Palpate the region under the scalp to locate the DBS Extension connector.
3. Create an incision to expose the DBS Lead and DBS Extension connector. Be careful not to damage the implanted components to allow for proper analysis following explant.
4. Cut the DBS Extension(s) at the tapered (proximal) end of the connector.
5. Loosen the connector Setscrew using the Torque Wrench provided.

CAUTION: Loosen the Setscrew only as much as necessary to remove the DBS Lead. Loosening the Setscrew too much will cause it to fall out.

Note: *To tighten the Setscrew, rotate the Torque Wrench clockwise. To loosen the Setscrew, rotate the Torque Wrench counterclockwise.*

6. Disconnect the DBS Extension connector. Return the DBS Extension connector to Boston Scientific.
7. Expose and disconnect the DBS Extensions from the Stimulator by following the procedure in “*Explanting or Replacing the Stimulator*”.
8. Gently pull the DBS Extension through the tunnel from the Stimulator site.

WARNING: Avoid pulling towards the ear to reduce the potential for infection of the DBS Leads.

9. Ship the explanted DBS Extensions to Boston Scientific.

Note: *If the DBS Extension has broken, then it may be necessary to make additional incisions or to pull one end of the DBS Extension out at the Stimulator site and the other end from the DBS Extension connector site.*

Explanting or Replacing the Stimulator

1. Turn off the Stimulator.
2. Palpate the subclavicular area to locate the Stimulator.
3. Surgically open the pocket where the Stimulator is located. Be careful not to damage the implanted components to allow for proper analysis following explant.

CAUTION: Do not use electrocautery as it will damage the Stimulator.

Note: *The incision should be large enough to remove the Stimulator from the pocket.*

4. Withdraw the Stimulator from the pocket.
5. Using the Torque Wrench, unscrew the header Setscrews to release the DBS Extensions.

CAUTION: Loosen the Setscrew only as much as necessary to remove the DBS Extension. Loosening the Setscrew too much will cause it to fall out.

Note: *To tighten the Setscrew, rotate the Torque Wrench clockwise. To loosen the Setscrew, rotate the Torque Wrench counterclockwise.*

6. Remove the DBS Extensions from the Stimulator.
7. If the Stimulator is to be replaced, reconnect the new Stimulator by following the procedures in *Connecting the Stimulator*.
8. If the DBS Extensions will remain implanted, you may optionally clean the proximal ends of the DBS Extensions, attach Lead Boots (as described in the “*Securing the DBS Lead*” section), and coil the excess DBS Extension material in the pocket.
9. Close the incision.
10. Ship the explanted Stimulator to Boston Scientific.

CAUTION: Be careful not to damage any remaining implanted components when closing the incision.

The Vercise™ Stimulator

The Vercise Stimulator is rechargeable. Patient's should expect a daily recharging time of 15 to 30 minutes or a periodic recharging time of 3 to 4 hours every 1 to 2 weeks, but their recharge routine may vary depending on their stimulation parameters. High power users will require more frequent charging. Boston Scientific recommends any recharge schedule that fits the patient's schedule and lifestyle while maintaining sufficient charge to maintain stimulation.

Note: *Frequent charging does not affect or diminish the battery life of the Stimulator.*

Developing a patient's recharge schedule involves finding the right balance between four factors:

- How much power is required for the patient to experience effective therapy.
- How often the patient wants to recharge.
- How long the patient wants to recharge.
- How the patient would like to manage their personal schedule.

The Clinician Programmer (CP) will estimate charging time based on 24 hours per day of stimulation at the programmed settings. Patients should be instructed to charge either a little every day (for 15 to 30 minutes) or fully charge once every 1 to 2 weeks (for 3 to 4 hours). If fully charging the Stimulator, patient should be instructed to charge until the Charger emits the end of charge double beep. The recharging process is simple, but important.

The rechargeable Stimulator battery should provide at least five years of service. In many cases, the Stimulator battery should provide at least 25 years of service. Battery life is dependent on the stimulation settings and conditions. To determine the expected battery life for specific settings and conditions, please see the "Appendix" on page 77 of this manual.

Over time and with repeated charging, the battery in the Stimulator will lose its ability to recover its full capacity. As a result, over time, the Stimulator may require shorter intervals between charges. For example, a Stimulator with settings requiring charging every 22 days during the first year, is expected to require charging every 20 days after five years of continuous use (amplitude = 3 mA, pulse width = 60 μ s, rate = 185 Hz, impedance = 1500 Ohms). The Stimulator will need replacement when stimulation can no longer be maintained with routine charging.

Stimulator Battery

When the Remote Control communicates with the Stimulator, the battery status is sent to the Remote Control.

The Remote Control displays a battery graphic near the top center of the screen to indicate the battery level of the Stimulator. The graphic is easy to understand: three filled-in bars means that the Stimulator has a fully-charged battery.

Note: *The battery graphic indicates the battery level of the Stimulator, not the Remote Control.*

As the battery strength wears down, depending on the stimulation parameters and usage, the bars will empty, as shown in the chart below. For complete information on maintaining the Stimulator's battery, see “Charging the Stimulator” on page 40.

Battery Graphic	Description
	The Stimulator's battery is <i>full</i> .
	The Stimulator's battery is <i>ok</i> .
	The Stimulator's battery is <i>low</i> .

Stimulator Battery Messages

When the Stimulator's battery is low, the Remote Control will display a “Recharge Stimulator Soon” message. The Stimulator should be recharged as soon as possible.

If the Stimulator is not recharged and the Stimulator's battery becomes very low, the Remote Control will display a “Recharge Stimulator Now” message. The Stimulator should be recharged immediately. Failure to recharge the Stimulator will lead to loss of stimulation within less than 24 hours. If the Stimulator turns off due to a very low battery, it will need to be charged for approximately two hours (with the Charger properly aligned) before it can be turned on again.

If the patient reports that stimulation has stopped, instruct the patient to check the Stimulator battery status with the Remote Control.

The Charging System


WARNINGS:

- Do not charge while sleeping. This may result in a burn.
- While charging, the Charger may become warm. It should be handled with care.
- Failure to use either the Charging Collar or an Adhesive Patch while charging, as shown in the “*Charging the Stimulator*” section, may result in a burn. If pain or discomfort is felt, the patient should cease charging and contact their physician.

CAUTION:

- Keep the Charger and charging components dry. They should not be exposed to moisture.
- Do not connect the Charger to any other device except to the Base Station provided with it.
- Keep the Charger and charging components away from pets, pests and children to avoid damage to the devices.

The Charging System for the Stimulator consists of the Charger unit, a Base Station, and a Power Supply. The Base Station is designed to remain connected to a power outlet at all times. When it is not being used, keep the Charger on the Base Station so that it is always ready to deliver a charge.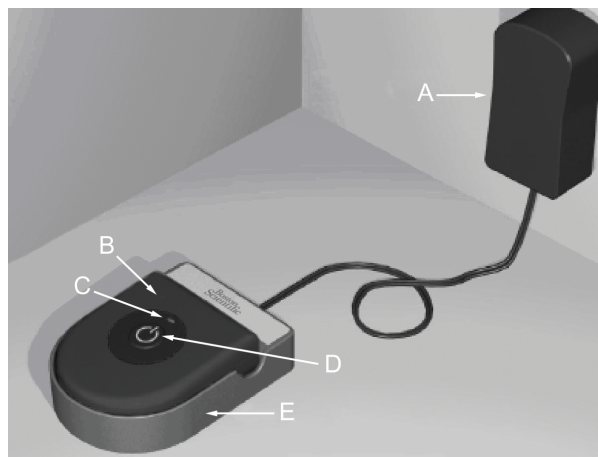

- | | |
|--------------------|-----------------|
| A. Power Supply | D. Power Button |
| B. Charger | E. Base Station |
| C. Indicator Light | |

Charging the Charger

1. Find a convenient electrical outlet, one that will not expose the Charging System components to water or direct heat.
2. Plug in the Power Supply.
3. Connect the Power Supply to the Base Station.
4. Place the Base Station on a flat surface.
5. Finally, place the Charger in the Base Station with the power button facing up until the indicator light turns green.

Indicator Light Status	Description
Green	The Charger is fully charged and ready to charge the Stimulator.
Yellow	The Charger is partially charged. The Charger may still be used, but it may not be able to fully charge the Stimulator. To fully charge the Charger, place it in the Base Station.
Off	The Charger is off –or– the Charger battery is empty. If pressing the Power Button does not turn on the Charger, place the Charger in the Base Station to charge.

Charger Battery Service Life

The Charger battery has a typical service life of 500 charging cycles.

Charging the Stimulator

When charging the Stimulator, the patient must use the Charger with either the Charging Collar or an Adhesive Patch to hold the Charger over the Stimulator. The Adhesive Patches are made of non-reactive and Latex-free material suitable for most sensitive skin types.


Using the Charging Collar

1. When the indicator light is green, remove the Charger from the Base Station. *(The indicator light will go off, regardless of the ready status of the Charger.)*
2. If the patient's Stimulator is in a shallow location or the patient has thin skin, instruct the patient to place the Charging Spacer at the back of the pocket in the Charging Collar. Patients with shallow Stimulators or thin skin will be able to charge faster with the Charging Spacer.

Note: *Patients with Stimulators in a deep location should not use the Charging Spacer, as it may slow the speed of charging.*


3. Place the Charger into the appropriate pocket on the Charging Collar with the power button facing out. If the Stimulator is on the right side of the chest, place the Charger in the right pocket. If the Stimulator is on the left side of the body, place the Charger in the left pocket.
4. If using the Charging Spacer, make sure it is between the Charger and the back of the pocket.
5. Place the Counterweight in the pocket opposite of the Charger.
6. Place the Charging Collar over the neck with the pockets facing out, as shown.
7. For best charging results, make sure the Charger is centered over the Stimulator. If the Charger is not centered, charging time may increase. See “*Properly Aligning the Charger*” on page 43 to ensure you center the Charger over the Stimulator.
8. If the Charger is not centered over the Stimulator, you may need to adjust the length of the Charging Collar using the straps. Occasionally checking that the Charger is aligned over the Stimulator during your charging session is recommended.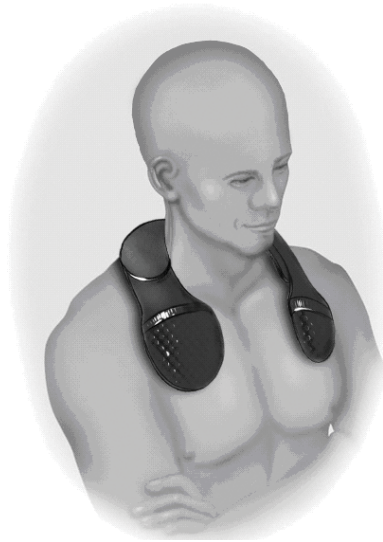

Note: *The Charging Collar can be placed under or over clothing. You should not wear tight fitting or heavy clothing over the Charger while charging to allow air flow around the Charger.*

Using the Adhesive Patch

1. When the indicator light is green, remove the Charger from the Base Station. *(The indicator light will go off, regardless of the ready status of the Charger.)*
2. Apply the Adhesive Patch to the backside of the Charger by peeling the clear liner from the patch and applying the white side with the blue stripe to the rear of the Charger, as shown to the right.
3. Remove the skin side beige liner from the adhesive (only good for one fixation.).
4. For best charging results, make sure the Charger is centered over the Stimulator. If the Charger is not centered, charging time may increase. See “*Properly Aligning the Charger*” on page 43 to ensure you center the Charger over the Stimulator.

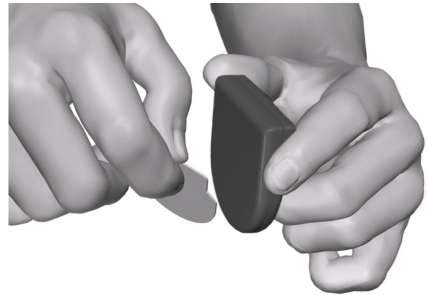WARNING: Do not put the Charger directly on the skin (e.g., without an Adhesive Patch). This may result in uncomfortable heating of the skin or a burn.

Properly Aligning the Charger

1. Press the power button. The indicator light will come on again, and the Charger will begin beeping steadily to signal that it is searching for the Stimulator.
2. Place the Charger on the chest in the area of the Stimulator.

3. Position the Charger over the Stimulator.

The beeping will stop when the Charger is partially aligned with the Stimulator, but the Charger should be fully aligned with the Stimulator for optimal charging. Charging time may increase if the Charger is not fully aligned with the Stimulator. To fully align the Charger over the Stimulator, position the bottom curve of the Charger approximately 1 cm lower than the bottom curve of the Stimulator, as shown below.

You may use the tips of your index and middle fingers to help locate the Stimulator under the skin. Lay your fingers gently on the skin without applying pressure or gripping. You may use this method to locate the center of the Stimulator.

4. Secure the Charger over the Stimulator either by pressing the adhesive to the skin over the Stimulator, or letting go of the Charging Collar.

Note: *If the Adhesive Patch is accidentally located in the wrong place, or if the Charging Collar moves out of alignment, the Charger will start beeping again. Use a new Adhesive Patch or readjust the Charging Collar to place the Charger back into the fully aligned position.*

Note: *Moving around while charging is acceptable, but be aware that the Charger must stay centered over the Stimulator for proper charging. Excessive movement may cause the Charger and Stimulator to become misaligned.*

5. When the Charger emits a distinct double beep, press the power button to turn off the Charger and check the Stimulator's battery with the Remote Control.
6. If the Stimulator is fully charged, remove the Charging Collar or Adhesive Patch from your body.
7. Separate the Charger from the Charging Collar or Adhesive Patch. Set aside the Charging Collar or discard the Adhesive Patch.
8. Return the Charger to the Base Station.

Do not confuse the end of charge signal (a distinct double beep) with the steady, continuous misalignment signal.

Note: *Patients should expect a daily recharging time of 15 to 30 minutes or a periodic recharging time of 3 to 4 hours every 1 to 2 weeks, but their recharge routine may vary depending on their stimulation parameters.*

Note: *The end of a charge signal is a distinct double beep, and the alignment indicator light is a steady, continuous signal.*


The Remote Control

CAUTION:

- Keep the Remote Control dry. It should not be exposed to moisture.
- Keep the Remote Control away from pets, pests, and children to avoid damage to the device.

Basic Operation

The Remote Control communicates with the Stimulator through a radio frequency (RF) telemetry link from a distance of 18 inches (45 cm). When it is not being used, the Remote Control is in sleep mode from which it can be reactivated by any button press. During normal patient use, the Remote Control will transition to sleep mode automatically after 60 seconds of non-use.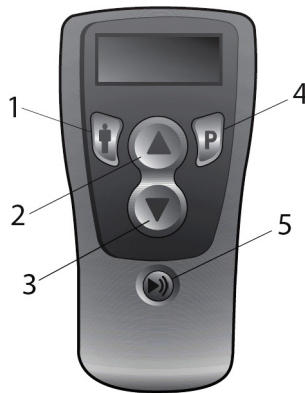


- | | |
|----------------|-----------------------|
| 1. Area Button | 4. Program Button |
| 2. Up/Activate | 5. Stimulation On/Off |
| 3. Down/Save | |

Some Remote Control functions (i.e., the Restore option and Clinician Mode) are accessed by pressing and holding a button (for approximately three seconds). These are identified in the appropriate following sections.

Remote Control Position

You will achieve a strong communication link between the Remote Control and the implanted Stimulator if the Remote Control is positioned as shown in the picture below.

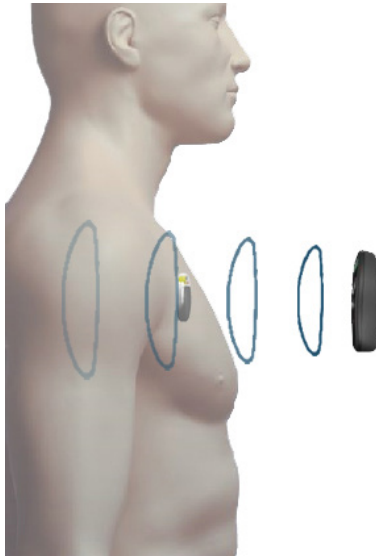In some situations, you may need to bring the Remote Control closer to the Stimulator. The distance at which you will be able to achieve communication depends on your environment and the relative orientation of the Remote Control to the Stimulator.

Stimulator Linking

A Remote Control is restricted to communicating with one Stimulator at a time in order to prevent it from inadvertently controlling an unintended device. Therefore, the Remote Control must be linked with the Stimulator before use.

If the Remote Control is linked to a different Stimulator, you will need to clear the link before linking it to the new Stimulator. See “*Selection #3 – Clear Link*” in the “*Clinician Options*” section.

The initial step in the linking process involves the Remote Control identifying, by telemetry, the intended Stimulator for communication. The second step in establishing the link depends on whether:

- The Stimulator is an External Trial Stimulator or an implantable Stimulator.
- Programs are stored in either the Remote Control or Stimulator.
- The Remote Control and Stimulator contain different program sets (if so, one set will need to be cleared).

The Remote Control will display the following message upon first activation (when any button is pressed):

1. Ensure the Remote Control is within telemetry range of the Stimulator (60 cm or 2 ft).
2. Press **P** to initiate communication between the Remote Control and the Stimulator. The Remote Control will identify the Stimulator by ID number.

3. Press **P** to confirm and continue.

The Remote Control will automatically detect available program sets in the Remote Control or Stimulator. If there are no available programs, the link will be completed immediately and you will see the screen below:

However, if programs are present in either device (Remote Control or Stimulator) during linking, you will be required to respond to one or more decision screens. Your decisions will guide the Remote Control to complete the linking with the desired program set saved to the desired device (Remote Control or Stimulator). You may also need to enter the clinician's password.

If an error occurs during the process or if the password is incorrect linking will be aborted. For further information, including information about the clinician's password, see "*Selection #3 – Clear Link*" in the "*Clinician Options*" section.

Stimulation On/Off

Stimulation is turned on and off via a dedicated power switch on the Remote Control keypad. Simply press the stimulation on/off button  at any time – even when the Remote Control is in sleep mode – to change the stimulation status of the Stimulator.

Signal Strength

The Remote Control indicates the signal strength between itself and the Stimulator in the upper right corner of the display:

Signal Strength	Description
	The Remote Control is achieving <i>optimal</i> communication with your Stimulator.
	The Remote Control is achieving <i>good</i> communication with your Stimulator.
	The Remote Control is achieving <i>OK</i> communication with your Stimulator.
	The Remote Control is achieving <i>weak</i> communication with your Stimulator.
	The Remote Control is achieving either <i>very weak</i> or <i>no</i> communication with your Stimulator. Communication between the Remote Control and the Stimulator may or may not be achieved. To improve the communication speed, move the Remote Control closer to the Stimulator and position it as shown in “ <i>Remote Control Position</i> ” on page 46.

After missing a telemetry message, the Remote Control will display the “Searching” screen. The Remote Control will then start looking for signal strength every second. Move to a better spot and wait for the signal strength to display. To cancel searching press .

Note: *Avoid common sources of interference, such as televisions and computer monitors when trying to use the Remote Control.*

Stimulation Amplitude

Whenever stimulation is turned on, or after the Remote Control is awakened from sleep mode, the Remote Control display defaults to the Level screen.

Note: *When there is no button activity for more than 60 seconds, the Remote Control will transition to sleep mode and the display screen will be blank.*

Usually, changes to the amplitude of stimulation will not be permitted using the Remote Control. In these cases, the physician should determine the optimal amplitude for stimulation therapy. However, for certain patients, physicians may allow patients to change stimulation levels.

If the physician has allowed control of the stimulation amplitude, press the ▲ or ▼ button from the Level screen to increase or decrease the amplitude. The triangular ramp in the center of the screen will fill as the stimulation amplitude increases.

Stimulator Battery

The Level screen also displays a battery graphic near the top center of the screen to indicate the battery level of the Stimulator. The graphic is easy to understand: three filled-in bars means that the Stimulator has a fully-charged battery.

Note: *The battery graphic indicates the battery level of the Stimulator, not the Remote Control.*

As the battery strength wears down, depending on the stimulation parameters and usage, the bars will empty, as shown in the chart below. For complete information on maintaining the Stimulator's battery, see "Charging the Stimulator" on page 40.

Battery Graphic	Description
	The Stimulator's battery is <i>full</i> .
	The Stimulator's battery is <i>ok</i> .
	The Stimulator's battery is <i>low</i> .

Program Selection

A stimulation program is a set of stimulation parameters that determine the patient's therapy. A program may apply stimulation to up to four independent stimulation fields or areas, depending on how the program is set up. For example, one area may correspond to a right brain target, while another area may correspond to a left brain target. The Remote Control can store up to four programs – numbered 1 through 4 – for the patient to select and activate at any time.

Graphic	Description
	A blank program slot.
	A full program slot
	The most recently activated or saved program.
	The currently selected program slot
	A reminder that the P button changes the selected program.

Note: If a patient tries to activate an empty program, nothing will happen.

To select a program:

1. Press the **P** button from the Level screen to go to the Program screen.

From the Program screen, each additional **P** button press selects the next program number. Press **⏮** to select the previous program number. The black highlighted box shows where you are in the cycle. Pressing **P** from program 4 returns you to the Level screen. Pressing **⏮** from program 1 also returns you to the Level screen.

2. Press **P** or **⏮** to move until the program you want is highlighted.
3. Press **▲** to activate the program.

Once you have selected and activated a program, the Remote Control will return to the Level screen.

Modifying and Saving Programs

1. Press **P** as needed from the Level screen to cycle to the desired program.

Note: *The most recently retrieved or saved program will be underlined(_).
Empty program slots are denoted by a blank box with no program number [].
If a patient tries to activate an empty program, nothing will happen.*

2. Press **▲** to activate the selected program.

3. After the Remote Control times out to the Level screen, you may change the amplitude of all areas in the program.
4. To save/store changes, select the program again and press **▼**. You will be required to confirm the adjustment first.
5. Press **P** to confirm, or press the **⏏** button to cancel the operation.

Note: *To save the modifications as a new program, simply select an empty [] program slot and press **▼** instead of overwriting the existing program.*

Individual Area Amplitude

A program may apply stimulation to one area, or up to four areas, depending on how the program it set up. To adjust the amplitude of each area independently:

1. Press the **I** button as needed from the Level screen to cycle to a specific stimulation area.
2. Press the **▲** or **▼** button to adjust the amplitude of the selected area.

Restore Option

With the Vercise™ DBS System, the “Restore” is readily available to all patients. The Restore feature allows patients to return a program to the original settings you programmed for them at the initial fitting or at a follow-up.

To access the Restore option:

1. Press and hold **P** approximately 3 seconds to reach Restore Program 1.
2. If necessary, press the **P** button again (normal press) to cycle through the programs and select the program to be restored.
3. Press **▲** to restore the last clinic-programmed settings.

Searching

In the event of communication interference, the Remote Control will automatically begin “*searching*” for the Stimulator. Try to reposition the Remote Control closer to the Stimulator to help it locate the Stimulator.

Clinician Options

Clinicians and physicians can access additional options in the Remote Control to set up the Vercise™ DBS System. These options are not available to patients.

Patient-restricted clinician screens provide access to:

- Remote Control and Stimulator clearing and relinking
- Communication with the clinician programmer
- Contact impedance monitoring

To access Clinician Options menu:

1. Press the **I** and **P** buttons simultaneously for approximately three seconds. The Enter Clinician Options menu will appear as shown below.
2. Press the **▲** or **▼** button to navigate the Clinician Options menu.

Each option is discussed in sequence below.

Selection #1 – Clinician Programmer (CP) Mode

1. From the Enter Clinician Mode menu, press the **P** button to prepare the Remote Control for communication with the Clinician's Programmer (CP).

Note: The Remote Control will remain CP Ready for 15 minutes.

2. Place the Remote Control and the IR Interface in the IR Interface Holder with their communication ports facing.
3. Plug the serial end of the USB to Serial Adapter into Interface Holder.
4. Plug the USB to Serial Adapter into the USB port on the CP.
5. Power-on the CP.
6. Launch the Bionic Navigator™ Software and wait for the IR Communication Established display.

en

Selection #2 – Impedances


When the “Impedances” option is selected from the Enter Clinician Options menu, the Remote Control will take the measurements via telemetry; this will take a few seconds.

Note: During this measurement the serial number of the Stimulator will be displayed on the screen.


Eventually, the Remote Control will display the Contact Impedance Status screen.

Contacts 1 through 8 (Lead position 1-L) are represented by the rectangles in the top row; contacts 9 through 16 (Lead position 2-R) are represented by the rectangles in the bottom row.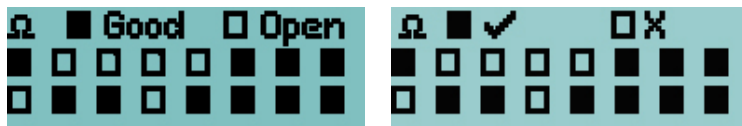

Contacts within the acceptable impedance range are displayed as solid rectangles; high impedance contacts (above 4500 Ohms) are represented by hollow rectangles.

Pressing any key, except the ►) Stim On/Off button, will display the value of the reading of contacts 1 – 8. Subsequent pressing of any key, except the ►) Stim On/Off button, will show values for contacts 9 – 16.

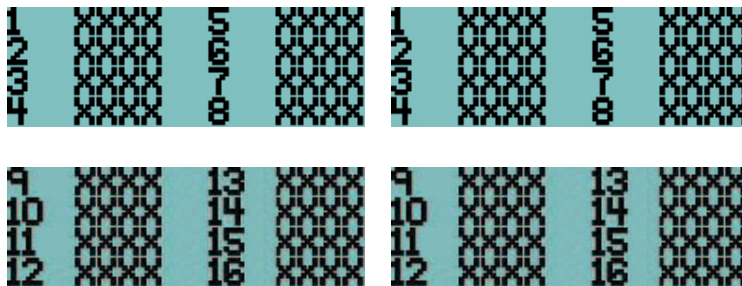Any key press other than the ►) Stim On/Off button will show the Measure Again screen which allows for repeating the measurement or exiting this function.

Note: The Remote Control will terminate the CP Ready state and transition to the sleep mode if there is no IR signal after 15 minutes.

All buttons are active during CP Ready and pressing any button returns the Remote Control to the Level screen.

Stimulation may be turned on or off during CP Ready.

Once IR Communication is established, the Remote Control will terminate communication and transition to the sleep mode if there is no IR activity after 15 minutes.

All buttons are active during IR Communication and pressing any button returns the Remote Control to the Level screen.

Whenever the Remote Control is re-activated from sleep mode the display will default to the Level screen.

Selection #3 – Clear Link

When the “Clear Link” option is selected from the Enter Clinician Options menu, you will immediately be required to enter the clinician’s password in order to continue.

The password for the Clear Link option is A-B-C.

To enter the password:

The first character is highlighted when the Enter Password screen opens. To select/confirm any character and/or move to the next character position, press **P**. To scroll through possible characters, use **▲** or **▼**.

Note: *Do not share this password with patients, as they may unknowingly clear the link between their Remote Control and Stimulator.*

When clearing a link:

If the password is entered correctly, the link between the Remote Control and the “previous” Stimulator is broken immediately, and the Remote Control displays the Not Linked screen. The Remote Control’s programs remain intact. If the password is entered *incorrectly*, the process is aborted and the Remote Control will return to the Enter Clinician Options screen.

Note: *If the password is entered incorrectly during an attempt to link the Remote Control, the process is aborted and the Remote Control returns to the Sleep screen.*

Selection #4 – Language

When the “Language” option is chosen from the Enter Clinician Options menu, you may scroll through the six language screens by pressing ▲ or ▼.

Use the **P** key to select English, Spanish, French, Italian, German, or Dutch when the language is highlighted.

Then, press **P** on the next screen to confirm the selection. The Remote Control language will now change to your selected language.

Remote Control Battery Life


Under typical usage, the Remote Control battery will last 15 days before replacement.

Stimulation Guidelines

Charge Density

The Clinician Programmer (CP) software provides a warning when the charge density at a contact is about to be programmed to exceed $30 \mu\text{C}/\text{cm}^2$ per stimulation phase, as there is evidence of tissue damage above this level. However, the CP allows the stimulation to continue above this level.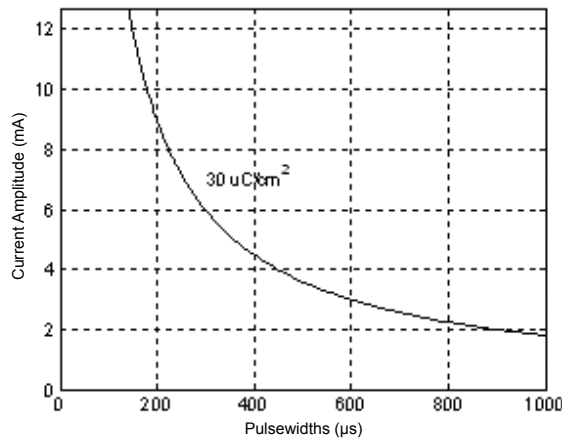

This graph displays the recommended maximum charge density of $30 \mu\text{C}/\text{cm}^2$ for different combinations of current amplitude (mA) and pulse width (μs). These estimates of charge density are only for the Boston Scientific DBS Leads.

WARNING: Patients may have the ability to change the amplitude with the Remote Control. The physician should set and verify the maximum and minimum amplitude levels allowed by the Remote Control to ensure that current levels remain safe. Do not set the default amplitude outside of the minimum and maximum amplitude range.

Note: A warning will pop-up on the Clinician Programmer screen if the stimulation parameters will cause the charge density to exceed $30 \mu\text{C}/\text{cm}^2$.

Detailed Device Description

General Component Description

The implantable portion of the Vercise™ DBS System includes a **Stimulator** and two **DBS Leads**. Other implantable components include the **DBS Extensions** that extend the DBS Leads to the Stimulator, a **DBS Lead Boot** to protect the proximal end of the DBS Lead between surgeries, and **Sutures Sleeves** to protect the DBS Lead and/or to anchor the DBS Leads and DBS Extensions.

The **Tunneling Tool** is a surgical instrument used to create a subcutaneous tunnel for the DBS Leads and DBS Extensions.

The patient will be provided with external devices including a **Remote Control** to communicate with the Stimulator and a **Charger** to recharge the battery of the Stimulator.

Stimulator Physical Characteristics

The physical characteristics of the Stimulator are outlined in Table 1. The Stimulator contains a radiopaque identification tag. The identification tag is visible using standard x-ray procedures.

Table 1: Stimulator Physical Properties

Feature	Description
Case	Titanium
Header	Epoxy
Dimensions	55 mm x 45 mm x 11 mm
Volume	20.7 cm ³

Stimulator Programmable Characteristics

The stimulation parameters are independent for the two DBS Leads such that stimulation of two different brain targets can have different amplitudes, pulse widths, stimulation rates, and contact configurations. The two DBS Leads can also have differing monopolar and multipolar configurations; however, a given DBS Lead contact cannot be programmed as both monopolar and multipolar.

The programmable parameter ranges for the Stimulator are shown in Table 2.

Note: *Some frequency combinations may not be used. See the DBS Programming Manual for specific frequency combinations.*

Parameter	Range
Amplitude ⁴	0.1 mA to 20 mA
Rate ^{5, 6}	2 Hz to 255 Hz
Pulse Width ⁷	10 μ s to 450 μ s
Cycle (On/Off)	1 s to 90 min
Stim Ramp On	1 s to 10 s
Contact Connections	16
Independent Areas of Stim (4 Programs with 4 Areas per Program)	16

Note: *While it is possible to program stimulation rates higher than 255 Hz, clinical effectiveness and safety of these cumulative rates has not been demonstrated and is not recommended.*

WARNING: Although the CP utilizes warnings to aid clinicians in their determination of safe charge density levels, the primary responsibility of keeping safe current levels resides with the clinician. Additionally, maximum and minimum amplitude levels allowed by the Remote Control should be set and verified by the physician to ensure that these levels are safe.

4 The programmable coverage for each individual contact is limited to 12.7 mA. A programming interlock is enforced to limit the total output current to 20 mA or less per coverage area. For example, a maximum current output of 12.7 mA on one contact would limit the total summed current output on the remaining contacts to 7.3 mA within one coverage area.

5 The rate is limited to 255 Hz for a given area.

6 The use of higher frequencies than the ones that are established (250 Hz) is the sole responsibility of the user.

7 The use of lower pulse widths than established (60 μ s to 450 μ s) is the sole responsibility of the user.

DBS Lead

The DBS Lead consists of 8 cylindrical contacts. The outer diameter of the DBS Lead and contacts is 1.3 mm and is compatible with existing commercially available DBS implantation tools. The DBS Lead can be implanted and attached to the Stimulator for both unilateral stimulation (on either the left or right side) and bilateral stimulation.

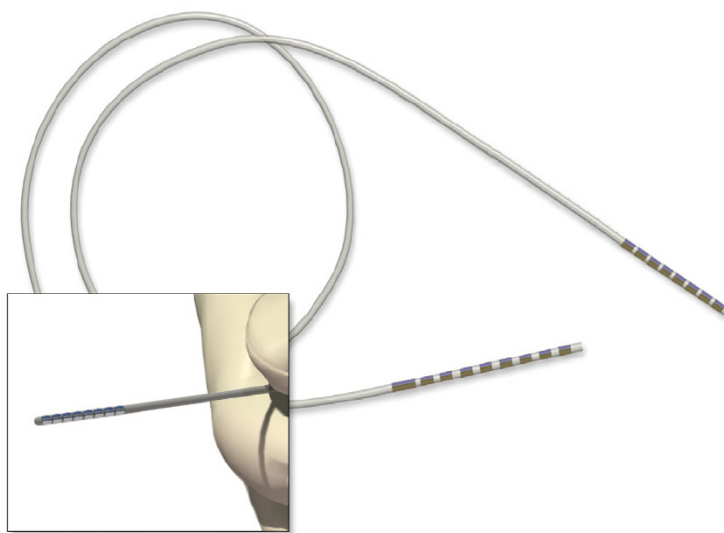
Table 3: DBS Lead

Feature	Description
Number of Contacts	8
Contact Length	1.5 mm
Contact Spacing (Center-to-Center)	2.0 mm
Contact Span	15.5 mm
Distal Contact to Tip Length	< 1.3 mm
Diameter	1.3 mm
Overall Length	30 cm, 45 cm
Outer Jacket Tubing (Insulation)	Polyurethane
Contact Material	Platinum/Iridium

DBS Extension

The DBS Extension consists of a connector at the distal end and 8 cylindrical contacts at the proximal end. The DBS Lead may be inserted and secured into the connector, which also contains 8 contacts that align with the contacts on the DBS Lead to form electrical connections. The DBS Extension can be implanted and attached to the Stimulator and the DBS Lead for both unilateral stimulation (on either left or right side) and bilateral stimulation.

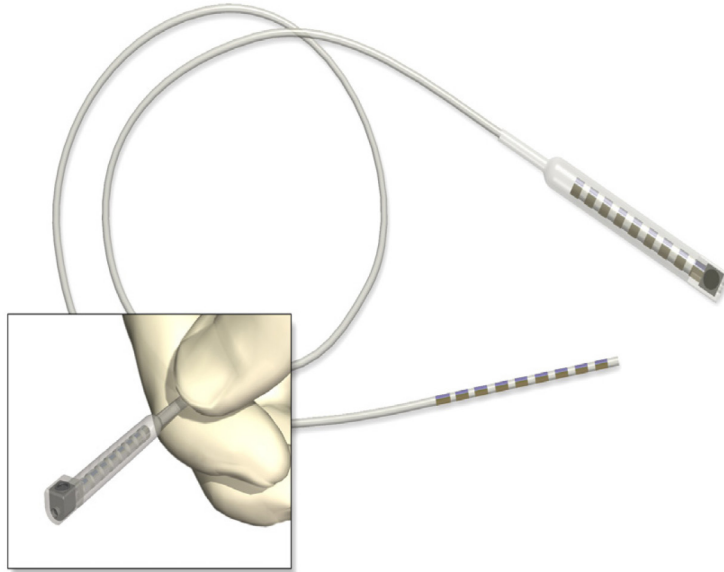
Table 4: DBS Extension

Feature	Description
Overall Length	55 cm
Outer Diameter	1.35 mm
Number of Contacts	8
Contact Material	Platinum/Iridium
Insulation Material	Polyurethane, Silicone

DBS Lead Boot

The DBS Lead Boot protects the proximal end of the DBS Lead prior to the Stimulator implant surgery. The Setscrew, when engaged, secures the DBS Lead in the Lead Boot.

en

Feature	Description
Overall Length	3.3 cm
Setscrew	Titanium
Connector Block	Stainless Steel
End Stop	Stainless Steel
Insulation Material	Silicone

Suture Sleeve

The Suture Sleeve is placed between the DBS Lead and the mini plate to protect the DBS Lead. The Suture Sleeve may also be used to anchor the DBS Lead or DBS Extension to the fascia.

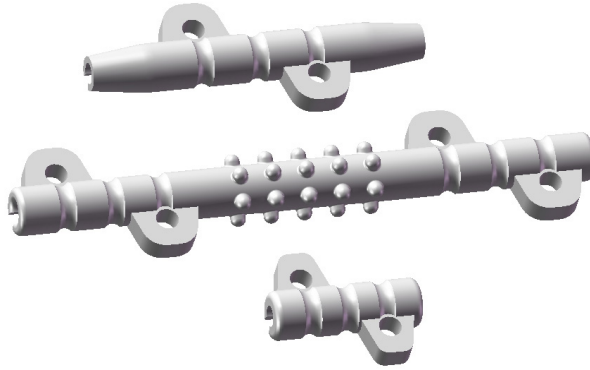
Table 6: Suture Sleeve

Feature	Description
Overall Length	1 cm, 2.3 cm, 4 cm
Material	Silicone

Tunneling Tool

The Tunneling Tool is used to create a path for the DBS Lead and DBS Extension in the subcutaneous tissue.

Table 7: Tunneling Tool

Feature	Description
Length	28 cm (Straw), 35 cm (Long)
Shaft Material	Stainless Steel
Straw Material	PTFE
Handle Material	Stainless Steel, Ultem

Programmer Communication

The Clinician Programmer (CP) can communicate with a Stimulator. In order to begin a programming session, the CP and the Remote Control Infrared (IR) windows must be aligned.


1. Turn on the CP and the Bionic Navigator™ Software will boot-up.
2. Enter the Clinician Options menu on the Remote Control by pressing the **I** and **P** buttons simultaneously.
3. Select CP Mode.
4. When the Remote Control displays “CP Ready,” put the IR Interface and Remote Control into the IR Interface Holder to keep the IR windows aligned.

Note: *Orient the IR Interface with the yellow circle facing down to maximize communication.*

5. Plug the serial end of the USB to Serial Adapter into the IR Interface.
6. Plug the USB to Serial Adapter into the USB port on the CP.
7. Position the patient within 60 cm (2 ft) of the Remote Control to ensure a complete communication link from the programmer to the Stimulator (see “Remote Control Position” on page 46).

Note: *A message will be displayed on the Clinician Programmer if the communication range is not satisfactory.*

For more detailed instructions on how to use the Clinician Programmer and Bionic Navigator Software with the Vercise™ DBS System, see the *DBS Programming Manual*.

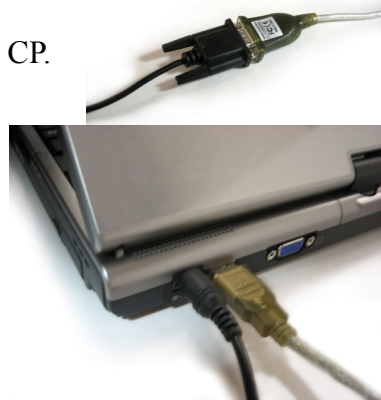Vercise™ DBS System Registration

In accordance with international practice and regulatory legislation in some countries, a registration form is packed with each Boston Scientific Stimulator, DBS Lead, and DBS Extension.

The purpose of this form is to maintain traceability of all products and to secure warranty rights. It also allows the institution involved in the evaluation or replacement of a specific implanted DBS Lead, accessory, or device to gain quick access to pertinent data from the manufacturer.

Fill out the registration form included in the package contents. Return one copy to the Boston Scientific Customer Service Department, keep one copy for patient records, provide one copy to the patient, and save one copy for the physician.

Boston Scientific
Neuromodulation Corporation
Attention: Customer Service Department
25155 Rye Canyon Loop
Valencia, CA 91355, USA

Technical Support

Boston Scientific Corporation has highly trained service professionals to assist you. The Technical Support Department is available to provide technical consultation 24 hours a day. To speak to a representative, please choose your locality from the following list:

Argentina

T: +5411 4896 8556 F: +5411 4896 8550

Australia / New Zealand

T: 1800 676 133 F: 1800 836 666

Austria

T: +43 1 60 810 F: +43 1 60 810 60

Balkans

T: 0030 210 95 37 890 F: 0030 210 95 79 836

Belgium

T: 080094 494 F: 080093 343

Brazil

T: +55 11 5853 2244 F: +55 11 5853 2663

Bulgaria

T: +359 2 986 50 48 F: +359 2 986 57 09

Canada

T: +1 888 359 9691 F: +1 888 575 7396

Chile

T: +562 445 4904 F: +562 445 4915

China – Beijing

T: +86 10 8525 1588 F: +86 10 8525 1566

China – Guangzhou

T: +86 20 8767 9791 F: +86 20 8767 9789

China – Shanghai

T: +86 21 6391 5600 F: +86 21 6391 5100

Colombia

T: +57 1 629 5045 F: +57 1 629 5082

Czech Republic

T: +420 2 3536 2911 F: +420 2 3536 4334

Denmark

T: 80 30 80 02 F: 80 30 80 05

Finland

T: 020 762 88 82 F: 020 762 88 83

France

T: +33 (0) 1 39 30 97 00 F: +33 (0) 1 39 30 97 99

Germany

T: 0800 072 3301 F: 0800 072 3319

Greece

T: +30 210 95 42401 F: +30 210 95 42420

Hong Kong

T: +852 2960 7100 F: +852 2563 5276

Hungary

T: +36 1 456 30 40 F: +36 1 456 30 41

India – Bangalore

T: +91 80 5112 1104/5 F: +91 80 5112 1106

India – Chennai

T: +91 44 2648 0318 F: +91 44 2641 4695

India – Delhi

T: +91 11 2618 0445/6 F: +91 11 2618 1024

India – Mumbai

T: +91 22 5677 8844 F: +91 22 2617 2783

Italy

T: +39 010 60 60 1 F: +39 010 60 60 200

Korea

T: +82 2 3476 2121 F: +82 2 3476 1776

Malaysia

T: +60 3 7957 4266 F: +60 3 7957 4866

Mexico

T: +52 55 5687 63 90 F: +52 55 5687 62 28

Middle East / Gulf / North Africa

T: +961 1 805 282 F: +961 1 805 445

The Netherlands

T: +31 30 602 5555 F: +31 30 602 5560

Norway

T: 800 104 04 F: 800 101 90

Philippines

T: +63 2 687 3239 F: +63 2 687 3047

Poland

T: +48 22 435 1414 F: +48 22 435 1410

Portugal

T: +351 21 3801243 F: +351 21 3801240

Singapore

T: +65 6418 8888 F: +65 6418 8899

South Africa

T: +27 11 840 8600 F: +27 11 463 6077

Spain

T: +34 901 11 12 15 F: +34 902 26 78 66

Sweden

T: 020 65 25 30 F: 020 55 25 35

Switzerland

T: 0800 826 786 F: 0800 826 787

Taiwan

T: +886 2 2747 7278 F: +886 2 2747 7270

Thailand

T: +66 2 2654 3810 F: +66 2 2654 3818

Turkey – Istanbul

T: +90 216 464 3666 F: +90 216 464 3677

Uruguay

T: +59 82 900 6212 F: +59 82 900 6212

UK & Eire

T: +44 844 800 4512 F: +44 844 800 4513

Venezuela

T: +58 212 959 8106 F: +58 212 959 5328

Note: Phone numbers and fax numbers may change. For the most current contact information, please refer to our website at <http://www.bostonscientific-international.com/> or write to the following address:

Boston Scientific Neuromodulation Corporation

25155 Rye Canyon Loop
Valencia, CA 91355 USA

Electromagnetic Compatibility

EN 60601-1-2 Classification Information

- Internally powered
- Continuous operation
- Ordinary equipment
- Class II

Guidance and Manufacturer's Declaration – Electromagnetic Emissions		
The Vercise DBS System is intended for use in electromagnetic environment specified below. The customer or the user of the Vercise DBS System should ensure that it is used in such an environment.		
Emissions Test	Compliance	Electromagnetic Environment – Guidance
RF emissions CISPR 11	Group 1	The Vercise DBS System uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Vercise DBS System is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not Applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not Applicable	

Guidance and Manufacturer's Declaration - Electromagnetic Immunity			
The Vercise DBS System is intended for use in the electromagnetic environment specified below. The customer or the user of the Vercise DBS System should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	Air: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV Contact: ± 8 kV	Air: Remote Control and Charger: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV ETS: ± 2 kV, ± 4 kV, ± 8 kV Contact: Remote Control and Charger: ± 8 kV ETS: ± 6 kV	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %. Note: Applies to external devices.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. Magnetic fields from common appliances are not expected to affect the device.

Guidance and Manufacturer's Declaration – Electromagnetic Immunity

The Vercise DBS System is intended for use in the electromagnetic environment specified below. The customer or the user of the Vercise DBS System should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Conducted RF IEC 61000-4-6 (ETS only)	3 Vrms 150 kHz to 80 MHz	3 Vrms 150 kHz to 80 MHz 6 Vrms in ISM and amateur radio bands between 150 kHz and 80 MHz	Professional healthcare facility environment and home healthcare environment.
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2,7 GHz	10 V/m 80 MHz to 2,7 GHz	Professional healthcare facility environment and home healthcare environment. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the symbol shown below: 

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Vercise DBS System is used exceeds the applicable RF compliance level above, the Vercise DBS System should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Vercise DBS System.

Manufacturer's Declaration – Proximity Fields			
The Vercise DBS System is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The users of the Vercise DBS System can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Vercise DBS System as recommended below, according to the maximum output power of the communications equipment.			
Proximity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment Guide
IEC 61000-4-3	385 MHz: 27 V/m @ 18 Hz pulse modulation	27 V/m	Recommended separation distance d = 30 cm
	450 MHz: 28 V/m @ FM modulation	28 V/m	
	710 MHz, 745 MHz, 780 MHz: 9 V/m @ 217 Hz pulse modulation	9 V/m	
	1720 MHz, 1845 MHz, 1970 MHz: 28 V/m @ 217 Hz pulse modulation	28 V/m	
IEC 61000-4-3	1720 MHz, 1845 MHz, 1970 MHz: 28 V/m @ 217 Hz pulse modulation	28 V/m	Recommended separation distance d = 30 cm
	2450 MHz: 28 V/m @ 217 Hz pulse modulation	28 V/m	
	5240 MHz, 5500 MHz, 5785 MHz: 9 V/m @ 217 Hz pulse modulation	9 V/m	
<p>Note: For the frequency bands in this table, use the specified recommended separation distance. The recommended minimum separation distance of 30 cm between portable and mobile RF communications equipment (transmitters) and the Vercise DBS System apply to all other frequencies within the specified ranges.</p> <p>Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			

Essential Performance

Failure of the external electrical components will not result in an unacceptable risk to the user.

Telemetry Information

The following parameters describe the wireless communication link between the Stimulator and the Remote Control.

- Frequency Band: 119 kHz to 131 kHz
- Modulation type: FSK
- Effective Radiated Power: 0.4 mW (-4 dBm) maximum
- Magnetic Field Strength (at 3 m distance): 94 μ A/m

Wireless Security

The Vercise DBS System has a short range inductively coupled telemetry system. A Remote Control has to be linked with a Stimulator to allow communication. The Stimulator will not respond to any unlinked device. There are additional mechanisms that ensure the integrity of the communicated data.

Limited Warranty

IPG

Boston Scientific Corporation (referred to as Boston Scientific) warrants to the patient who receives a Vercise™ Implantable Pulse Generator (referred to as the IPG) that the IPG will be free from defects in workmanship and materials for a period of five (5) years from the date of surgical implant of the IPG. This warranty applies only to the patient who has the IPG implanted and no other person or entity. This warranty does not apply to the leads, extensions, or surgical accessories used with the IPG.

If the IPG fails to function within normal ranges within five (5) years after the date it is implanted, Boston Scientific will replace the IPG with a functionally equivalent IPG made by Boston Scientific. No other relief whatsoever is available under this limited warranty. The limited warranty for a replacement IPG will last only for five years from the date of surgical implant of the original IPG. Claims under this limited warranty are subject to the following additional conditions and limitations:

1. The product registration card must be completed and returned to Boston Scientific within 30 days of surgery.
2. The IPG must be purchased after January 1, 2005 and implanted before the “use by” date.
3. Failure of the IPG must be confirmed by Boston Scientific.
4. The IPG must be returned to Boston Scientific (or a Boston Scientific authorized agent) within 30 days after it fails to function within normal ranges. That IPG will be the property of Boston Scientific.
5. This limited warranty does not include failures to function within normal ranges caused by:
 - (a) Fire, floods, lightning, natural disasters, water damage, and other calamities commonly defined as “Acts of God”;
 - (b) Accident, misuse, abuse, negligence, or the customer’s failure to operate the IPG in accordance with manufacturer’s instructions;
 - (c) Unauthorized attempts to repair, maintain, or modify the IPG by the patient or any unauthorized third party; or
 - (d) Attaching equipment to the IPG that is not supplied or expressly authorized by Boston Scientific.

This limited warranty is the only warranty that applies to the IPG, and Boston Scientific expressly disclaims any other warranty, express or implied, including any warranty of merchantability or fitness for a particular purpose.

Under this limited warranty, Boston Scientific will be responsible only for replacement of the IPG with a functionally equivalent IPG made by Boston Scientific and will not be liable for any damages (whether direct, indirect, consequential, or incidental) caused by the IPG, whether the claim is based on warranty, contract, tort or any other theory.

Boston Scientific assumes no liability with respect to instruments reused, reprocessed or resterilized and makes no warranties, express or implied, including but not limited to merchantability or fitness for a particular purpose, with respect to such instruments.

Leads

Boston Scientific Corporation (referred to as Boston Scientific) warrants to the patient that the DBS Leads, Lead Extensions, and Lead Boots are free from defects in workmanship and materials for a period of one (1) year from the date of implantation.

A Lead, Extension, or Boot that fails to function within normal tolerances within (1) year from the date of surgery is covered under this Limited Warranty. The liability of Boston Scientific under this warranty shall be limited to: (a) replacement with a functionally equivalent Lead, Extension, or Boot; or (b) full credit equal to the original purchase price to be applied towards the purchase of a new Lead, Extension, or Boot. Product claims under Boston Scientific Limited Warranty are subject to the following conditions and limitations:

1. The product registration card must be completed and returned to Boston Scientific within 30 days of surgery in order to obtain warranty rights.
2. The Lead, Extension, or Boot must be returned to Boston Scientific (or authorized agent) within 30 days of malfunction or discovery of defect, and shall be the property of Boston Scientific.
3. The Lead, Extension, or Boot must be implanted prior to the “use by” date.
4. Failure of the Lead, Extension, or Boot must be confirmed by Boston Scientific. This warranty specifically excludes defects or malfunctions caused by: (a) fire, floods, lightning, natural disasters, water damage, and other calamities commonly defined as “Acts of God”; (b) accident, misuse, abuse, negligence, or the customer’s failure to operate the Lead, Extension, and Boot in accordance with manufacturer’s instructions; (c) unauthorized attempts to repair, maintain, or modify the equipment by the customer or any unauthorized third party; or (d) attachment of any equipment not supplied by Boston Scientific without prior approval.
 - a. This warranty does not include surgical accessories used with the Lead, Extension, or Boot.
5. The decision as to product replacement or credit shall be made solely at the discretion of Boston Scientific. For a replacement Lead, Extension, or Boot, the warranty will run only to the end of the warranty period for the original Lead, Extension, or Boot that was replaced.

This warranty is in lieu of any other warranty, expressed or implied, including any warranty of merchantability or fitness for intended use. Except as expressly provided by this Limited Warranty, Boston Scientific shall not be responsible or liable for any direct, consequential or incidental damages caused by device malfunction, failure or defect, whether the claim is based on warranty, contract, tort, or otherwise.

Boston Scientific assumes no liability with respect to instruments reused, reprocessed or resterilized and makes no warranties, express or implied, including but not limited to merchantability or fitness for a particular purpose, with respect to such instruments.

Externals

Boston Scientific Corporation (referred to as Boston Scientific) warrants to the patient that the Remote Control device and Charging System (Charger and/or Charger Base Station) are free from defects in workmanship and materials for a period of one (1) year from the date of purchase.

If a Remote Control Device or Charging System component fails to function within normal ranges within one year after the date of purchase, Boston Scientific will replace the device or component with a functionally equivalent device or component made by Boston Scientific. No other relief whatsoever is available under this limited warranty. The limited warranty for a replacement device or component will last only for one year after the date of purchase. Claims under this limited warranty are subject to the following additional conditions and limitations:

1. The product registration card must be completed and returned to Boston Scientific within 30 days of purchase.
2. Boston Scientific must confirm the device or component failure.
3. The device or component must be returned to Boston Scientific (or Boston Scientific's authorized agent) within 30 days after it fails to function within normal ranges. That device or component will be Boston Scientific's property.
4. This limited warranty does not include failures to function within normal ranges caused by:
 - (a) fire, floods, lightning, natural disasters, water damage, and other calamities commonly defined as "Acts of God";
 - (b) accident, misuse, abuse, negligence, or the customer's failure to operate the device or component in accordance with manufacturer's instructions;
 - (c) unauthorized attempts to repair, maintain, or modify the device or component by the patient or any unauthorized third party; or
 - (d) attaching equipment to the device or component that is not supplied or expressly authorized by Boston Scientific.

This limited warranty is the only warranty that applies to the device or component, and Boston Scientific expressly disclaims any other warranty, express or implied, including any warranty of merchantability or fitness for a particular purpose.

Under this limited warranty, Boston Scientific will be responsible only for replacement of the device or component with a functionally equivalent device or component made by Boston Scientific and will not be liable for any damages (whether direct, indirect, consequential, or incidental) caused by the device or component, whether the claim is based on warranty, contract, tort, or any other theory.

Appendix

Rechargeable Stimulator Battery Longevity

The rechargeable Stimulator battery should provide at least five years of service. In many cases, the Stimulator battery should provide at least 25 years of service. Battery-longevity is dependent on the stimulation settings and conditions. The battery-longevity projections provided in the table below are based on continuous stimulation with one area and one program. The longevity estimates are generated using projections of typical results based on Boston Scientific internal testing data. Actual results may vary. To determine battery-longevity with multiple-area stimulation, please use the Multiple-Area Longevity Worksheet provided below. To determine battery-longevity with multiple stimulation programs, please use the Multiple-Program Longevity Worksheet provided below. If stimulation is not used continuously, the projected battery longevity should exceed the value shown.

Defining End of Battery Longevity

The expected battery longevity is defined as the longer of either:

- a) Typical case: the time from initiating therapy to the time at which therapy cannot be maintained with daily charging

OR

- b) High energy case: when the maximum time between charging has decreased by more than 50% from the initial maximum time between charging.

Effect of Impedance on Battery-Longevity Projections

This chart assumes a maximum impedance of 1500 Ohms between any 2 contacts in use. For lower impedance values, the projected battery longevity should slightly exceed the value shown. For higher impedances, the projected battery longevity should be slightly less than the value shown.

Determining Longevity

1. Fit the Vercise™ DBS System to the patient, make note of the amplitude, rate and pulse width for each of the areas. For multi-area stimulation, please use the Multiple-Area Battery-Longevity Worksheet.
2. In the following chart, look up the battery-longevity based on the amplitude, rate, and pulse width values that correspond most closely to the settings from step 1. (Interpolation may be used for improved accuracy.) The battery-longevity projections are given in years and represent the longevity of a typical Vercise™ Stimulator battery when used continuously. If the rechargeable Stimulator battery is not used continuously, the battery-longevity should be greater than the value given.

Note: Cells in the table that are marked with an “*” represent stimulation settings that cannot be attained with the Vercise DBS System. Interpolation between a cell with an “*” and another cell is not possible.

Note: *The longevity and recharge interval estimates in this document were generated using projections of typical results based on Boston Scientific's internal testing data. Actual results may vary.*

Multiple-Area Battery-Longevity Worksheet

Use this method when more than one area is used.

1. Calculate the TOTAL RATE of the stimulation frequencies by adding together the rate from each area that is used:

$$\text{TOTAL RATE} = \text{Rate 1} \underline{\hspace{1cm}} + \text{Rate 2} \underline{\hspace{1cm}} \\ + \text{Rate 3} \underline{\hspace{1cm}} + \text{Rate 4} \underline{\hspace{1cm}}$$

$$\text{TOTAL RATE} = \underline{\hspace{1cm}} \text{ Hz}$$

2. Calculate the Rate-Weighted Pulse Width (RWPW) for each area used by multiplying the Pulse Width value for each area used by the Rate of that area and then dividing the result by the TOTAL RATE that was calculated in step 1.

$$\text{RWPW1} = \text{PW1} \times \text{Rate 1} / \text{TOTAL RATE} = \underline{\hspace{1cm}}$$

$$\text{RWPW2} = \text{PW2} \times \text{Rate 2} / \text{TOTAL RATE} = \underline{\hspace{1cm}}$$

$$\text{RWPW3} = \text{PW3} \times \text{Rate 3} / \text{TOTAL RATE} = \underline{\hspace{1cm}}$$

$$\text{RWPW4} = \text{PW4} \times \text{Rate 4} / \text{TOTAL RATE} = \underline{\hspace{1cm}}$$

3. Calculate the Total-Weighted Pulse Width (TWPW) by adding together the Rate-Weighted Pulse Widths calculated in step 2.

$$\text{TWPW} = \text{RWPW1} \underline{\hspace{1cm}} + \text{RWPW2} \underline{\hspace{1cm}} \\ + \text{RWPW3} \underline{\hspace{1cm}} + \text{RWPW4} \underline{\hspace{1cm}}$$

$$\text{TWPW} = \underline{\hspace{1cm}} \mu\text{s}$$

4. Determine the area with the largest amplitude.

$$\text{Largest amplitude} = \underline{\hspace{1cm}} \text{ mA}$$

5. Look up the battery-longevity projection in the table using the Total Rate, TWPW, and largest amplitude. (Interpolation may be used for improved accuracy.) The battery-longevity projection is given in years and represents the longevity of a typical rechargeable Stimulator battery when used continuously. If the rechargeable Stimulator battery is not used continuously, the battery-longevity should be greater than the value given.

Multiple-Area Battery Longevity Worksheet Example 1

In this example two stimulation areas are used with the following settings:

Area 1:

Amplitude = 3.0 mA, Rate = 130 Hz, PW = 60 μ s

Area 2:

Amplitude = 3.0 mA, Rate = 185 Hz, PW = 90 μ s

1. Calculate the TOTAL RATE (TR) of the stimulation frequencies by adding together the rate from each area that is used:

$$\text{TOTAL RATE} = \text{Rate 1 } \underline{130 \text{ Hz}} + \text{Rate 2 } \underline{185 \text{ Hz}}$$

$$\text{TOTAL RATE} = \underline{\mathbf{315 \text{ Hz}}}$$

2. Calculate the Rate Weighted Pulse Width (RWPW) for each area used by multiplying the pulse width value for each area used by the Rate of that area and then dividing the result by the TOTAL RATE that was previously calculated.

$$\begin{aligned} \text{RWPW1} &= \text{PW1} \times \text{Rate 1} / \text{TOTAL RATE} \\ &= \underline{(60 \times 130) / 315} = \underline{24.76} \end{aligned}$$

$$\begin{aligned} \text{RWPW2} &= \text{PW2} \times \text{Rate 2} / \text{TOTAL RATE} \\ &= \underline{(90 \times 185) / 315} = \underline{52.85} \end{aligned}$$

3. Calculate the Total-Weighted Pulse Width (TWPW) by adding the rate weighted pulse widths.

$$\begin{aligned} \text{TWPW} &= \text{RWPW } \underline{24.76} + \text{RWPW2 } \underline{52.85} \\ \text{TWPW} &= \underline{\mathbf{77.61 \mu\text{s}}} \end{aligned}$$

4. Determine the area with the largest amplitude.

$$\text{Largest amplitude} = \underline{\mathbf{3.0 \text{ mA}}}$$

5. Use TOTAL RATE, TWPW, and largest amplitude in the chart to determine the battery-longevity projection.

The closest values on the table using 315 Hz, 77.61 μ s, and 3.0 mA are found in the box associated with 510 Hz, 100 μ s, and 3.0 mA. The result is a battery-longevity projection of more than 25 years.

Battery Longevity Projection (Years)											
Amplitude	Rate	Pulse Width (µs)									
		40	60	100	150	200	250	300	350	400	450
mA	Hz										
1	95	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	130	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	185	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	255	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	397	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	510	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	1020	>25	>25	>25	*	*	*	*	*	*	*
2	95	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	130	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	185	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	255	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	397	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	510	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	1020	>25	>25	>25	*	*	*	*	*	*	*
3	95	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	130	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	185	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	255	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	397	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	510	>25	>25	>25	>25	>25	>25	>25	>25	22.6	20.4
	1020	>25	>25	>25	*	*	*	*	*	*	*
4	95	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	130	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	185	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	255	>25	>25	>25	>25	>25	>25	>25	>25	>25	22.6
	397	>25	>25	>25	>25	>25	>25	23.1	19.1	16.7	14.6
	510	>25	>25	>25	>25	>25	>25	21.8	17.3	14.6	12.3
	1020	>25	>25	>25	*	*	*	*	*	*	*
5	95	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	130	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	185	>25	>25	>25	>25	>25	>25	>25	>25	24.4	22.2
	255	>25	>25	>25	>25	>25	>25	23.8	20.9	18.5	15.4
	397	>25	>25	>25	>25	23.1	19.0	15.7	12.1	10.8	10.3
	510	>25	>25	>25	24.8	18.5	13.6	11.0	10.3	*	*
	1020	>25	>25	18.5	*	*	*	*	*	*	*
6	95	>25	>25	>25	>25	>25	>25	>25	>25	>25	>25
	130	>25	>25	>25	>25	>25	>25	>25	>25	23.4	23.4
	185	>25	>25	>25	>25	>25	>25	24.4	21.5	19.1	17.0
	255	>25	>25	>25	>25	>25	21.8	18.5	15.7	13.4	11.3
	397	>25	>25	>25	23.1	17.8	13.9	11.1	10.4	9.8	*
	510	>25	>25	>25	18.5	13.4	10.8	10.0	*	*	*
	1020	>25	22.6	13.4	*	*	*	*	*	*	*
8	95	>25	>25	>25	>25	>25	>25	>25	>25	>25	23.9
	130	>25	>25	>25	>25	>25	>25	>25	22.7	20.3	18.1
	185	>25	>25	>25	>25	>25	22.4	19.1	16.3	14.0	11.9
	255	>25	>25	>25	>25	20.6	16.6	13.4	11.1	10.5	10.0
	397	>25	>25	>25	17.8	12.7	10.6	9.8	*	*	*
	510	>25	>25	20.6	13.4	10.5	*	*	*	*	*
	1020	24.7	17.3	10.5	*	*	*	*	*	*	*
10	95	>25	>25	>25	>25	>25	>25	>25	24.4	21.9	19.8
	130	>25	>25	>25	>25	>25	24.8	21.5	18.7	16.3	14.2
	185	>25	>25	>25	>25	22.4	18.4	15.1	12.4	10.9	10.4
	255	>25	>25	>25	21.8	16.6	12.7	10.8	10.1	*	*
	397	>25	>25	21.1	13.9	10.6	9.6	*	*	*	*
	510	>25	>25	16.6	10.8	*	*	*	*	*	*
	1020	20.6	13.4	*	*	*	*	*	*	*	*
13	95	>25	>25	>25	>25	>25	>25	22.4	19.6	17.2	15.1
	130	>25	>25	>25	>25	24.1	20.0	16.7	14.0	11.7	10.8
	185	>25	>25	>25	22.9	17.7	13.7	11.0	10.3	9.8	*
	255	>25	>25	24.5	17.1	12.0	10.4	9.6	*	*	*
	397	>25	>25	16.4	10.7	*	*	*	*	*	*
	510	>25	21.1	12.0	9.6	*	*	*	*	*	*
	1020	15.9	10.6	*	*	*	*	*	*	*	*
16	95	>25	>25	>25	>25	>25	21.9	18.6	15.9	13.5	11.5
	130	>25	>25	>25	>25	20.3	16.3	13.0	11.0	10.4	9.9
	185	>25	>25	>25	19.1	14.0	10.9	10.1	*	*	*
	255	>25	>25	20.6	13.4	10.5	*	*	*	*	*
	397	>25	21.9	12.7	9.8	*	*	*	*	*	*
	510	24.7	17.3	10.5	*	*	*	*	*	*	*
	1020	12.3	9.7	*	*	*	*	*	*	*	*
20	95	>25	>25	>25	>25	21.9	17.9	14.6	11.9	10.8	10.3
	130	>25	>25	>25	21.5	16.3	12.3	10.7	10.0	9.4	8.9
	185	>25	>25	22.4	15.1	10.9	9.9	*	*	*	*
	255	>25	>25	16.6	10.8	*	*	*	*	*	*
	397	>25	17.8	10.6	*	*	*	*	*	*	*
	510	20.6	13.4	*	*	*	*	*	*	*	*
	1020	10.5	*	*	*	*	*	*	*	*	*

Rates are the sum of all active Areas.

*Parameter combination is not allowed by Clinician Programmer.

The following is federal government communications regulation information about the Vercise™ DBS System.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received including interference that may cause undesired operation.

The Vercise DBS System components should only be serviced by Boston Scientific. Do not attempt to open or repair any of the components.

Changes or modifications to this product not authorized by Boston Scientific Corporation could void the FCC Certification and negate your authority to operate this product.

This page intentionally left blank.

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All rights reserved.
Boston Scientific (Australia) Pty Ltd
Corporation or its affiliates.