Advancing science for life™

Bionic Navigator™ 2.03
Software
Windows 10 (Win 10)
(DB-7101-203-W10)
Installation Guide

Installation Guide

en

Guide d'installation

fr

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es

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pt-BRA

91111264-10

Content: MP91111264-10 REV A

How to Use this Manual

This manual describes the installation of the Boston Scientific Bionic Navigator Software. Read all instructions carefully before using the DBS Systems.

For indications for use, contraindications, warnings, precautions, adverse events, sterilization, component disposal, storage and handling, and warranty information, refer to the *Physician Manual* for your Boston Scientific DBS System as listed in your *DBS Reference Guide*. For other device-specific information not included in this manual or labeling symbols, refer to the appropriate DFU for your Boston Scientific DBS System as listed in your *DBS Reference Guide*.

Guarantees

Boston Scientific Corporation reserves the right to modify, without prior notice, information relating to its products in order to improve their reliability or operating capacity.

Drawings are for illustration purposes only.

Trademarks

Vercise™ and Bionic Navigator™ are trademarks of Boston Scientific Corporation or its affiliates.

All other trademarks are the property of their respective owners.

Contacting Boston Scientific

To contact Boston Scientific, refer to the “*Technical Support*” section of this manual.

Product Model Numbers

Installer Model Number	Description
DB-7101-203-W10	Bionic Navigator™ 2.03 Installer
Compatible Clinician Programmer Model Number	Description
DB-7164 and NM-7164	Clinician Programmer
DB-7164-R and NM-7164-R	Clinician Programmer (Refurbished)

Table of Contents

Bionic Navigator™ 2.03 Software (Win 10) (DB-7101-203-W10) Installation Guide	1
Description	1
Package Contents	1
Software Installation.....	1
Software Installation.....	1
Setup for Software Installation.....	2
Software Installation on the CP.....	3
Downloading Software Over the Internet.....	3
Installation from USB	6
Complete the Software Installation on the CP	7
Uninstall and Remove Software or Delete the Database	10
Appendix A - McAfee Security Policies	12
Appendix B - Web Activation Portal	13
Appendix C - Web Activation Portal, Confirm Activation	14
Appendix D - Accessing the Software Label.....	15
Technical Support	16

en

This page intentionally left blank.

Bionic Navigator™ 2.03 Software (Win 10) (DB-7101-203-W10) Installation Guide

Description

An executable file will install Bionic Navigator 2.03 Software (version 9028287-203) onto a Clinician Programmer (CP). The executable file is provided via AirWave, and in cases where internet connectivity is not available, a USB Installer.

en

Package Contents

(1) Bionic Navigator 2.03 Installer USB drive (DB-7101-203-W10) containing an executable installation file for software version 9028287-203.

This device is to be disposed of per local regulatory requirements.

Note: *Software can be distributed via an Installer USB drive or via the internet. If distributed over the internet, there will be no physical package contents.*

Software Installation

Important: *Copying or moving of the files on the Installer USB is strictly prohibited. Do not copy or move any files from the Installer USB onto the CP or to any other electronic device.*

Important: *If installation of Neural Navigator Software is also desired on the same CP, it is recommended that Bionic Navigator 2.03 Software be installed prior to installation of Neural Navigator Software.*

Software Installation

Boston Scientific Neuromodulation (BSN) Employees:

To complete the installation of Bionic Navigator 2.03 Software (version 9028287-203), you will need your CP, another VPN-enabled BSN computer, and if internet connectivity is not available, an Installer USB. This installation may only be performed on a compatible model CP (see the “Product Model Numbers” section of this manual). On your BSN computer, complete the VPN connection process, then follow the steps outlined in “Appendix B - Web Activation Portal” and “Appendix C - Web Activation Portal, Confirm Activation” to receive an activation key and provide a confirmation key to allow BSN to track the installation. The steps below must be repeated in order to install 9028287-203 on each compatible CP as each installation requires a unique activation key.

If you encounter any issues during installation, contact Technical Support.

Non-BSN Employees:

To complete the installation of Bionic Navigator 2.03 Software (version 9028287-203), you will need your CP and, if internet connectivity is not available, an Installer USB. This installation may only be performed on a compatible model CP (see the “Product Model Numbers” section of this manual). To obtain activation keys, contact your local sales representative. The steps below must be repeated in order to install 9028287-203 on each compatible CP as each installation requires a unique activation key.

If you encounter any issues during installation, contact Technical Support.

Setup for Software Installation

1. Plug the CP into a power source.
2. Turn on the CP.
3. Log in as ClinicUser. If this is the first login, use password “bsn”. You will then be prompted to set up a new ClinicUser password when logging into the CP for the first time. The new ClinicUser password must be 10 or more characters in length.

Note: *In regions where software is installed by a third party, follow region-specific guidance for the initial ClinicUser password.*

4. Ensure the power level of the CP is at least 30% or greater. If the power level is below 30%, ensure the CP is charging by checking for a white light in the charger connector. Wait until the power level is above 30%.

Note: *In order to perform software download or installation, the CP must be plugged in and have a power level of 30% or greater.*

5. Navigate to the CP desktop and ensure that all applications and programs, including any previous versions of Bionic Navigator Software, are closed.
6. From the CP desktop, launch AirWave .

Software Installation on the CP

Software is installed after downloading the software via the AirWave portal. If internet connectivity is not available, refer to the “*Installation from USB*” section of this manual.

Downloading Software Over the Internet

1. Connect to WiFi.
2. Navigate to the **CP Registration Tab** within AirWave (Figure 1).
3. Review Registration Data, including: Account Number, Account Name, Address, and Country (Figure 1).
 - a. If all data is correct, select **Confirm** (Figure 1).
 - b. If any information is incorrect, select **Request Update** (Figure 1). You will be presented with a Registration Data Update Request pop-up. Within the text fields, enter the correct information for the CP and select **Submit** (Figure 2).
 - c. If you are unsure if the Registration Data is correct, contact your BSC representative. If you do not know how to contact your BSC representative, refer to the “*Technical Support*” section of this manual.

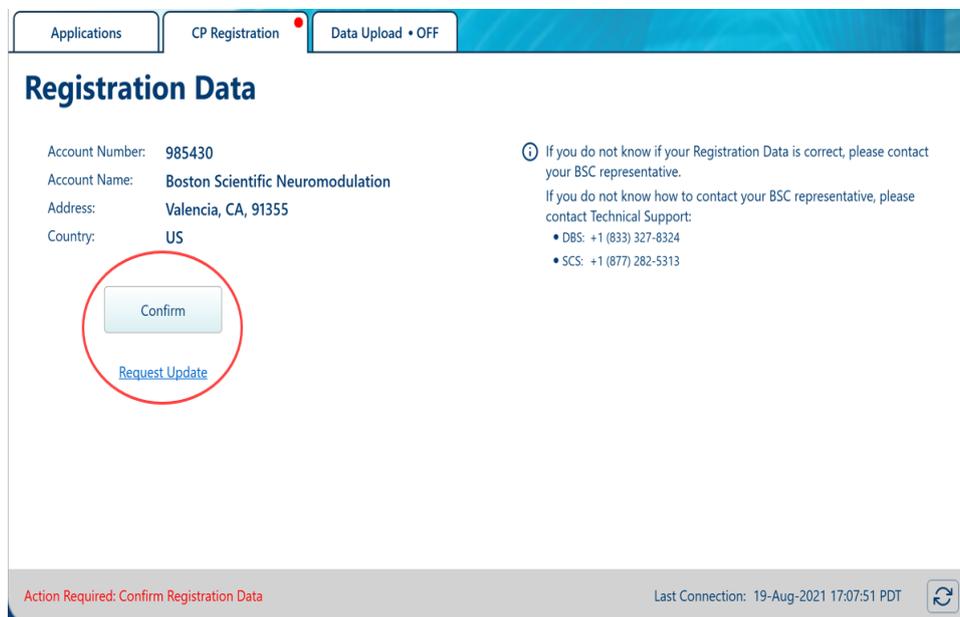
Figure 1. CP Registration Tab

Note: The presence of a red dot on a tab indicates that an action is required within that tab.

Figure 2. Registration Data Update Request

Note: Software cannot be downloaded over the internet until Registration Data is confirmed.

4. Check the bottom left corner of AirWave to confirm the status of your Registration is correct.

- a. If you selected **Confirm**, your status will say “Registration Confirmed: [Date and Time Confirmation Received]” (Figure 3).

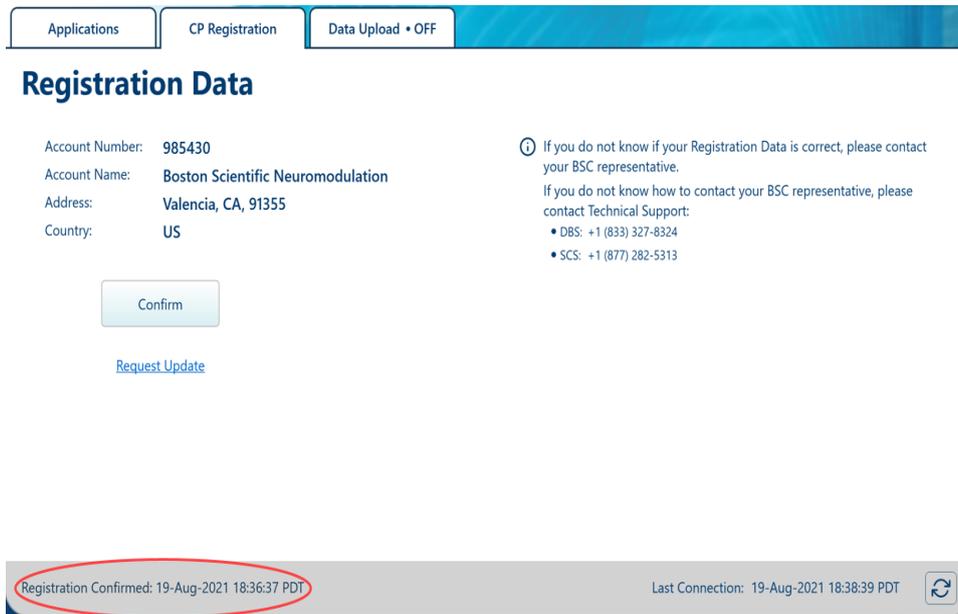
Figure 3. Registration Confirmed Status

- b. If you selected **Request Update** and submitted updated data within the pop-up window, your status will say “Processing Registration Updates: Requested [Date and Time Updates Requested]” (Figure 4).

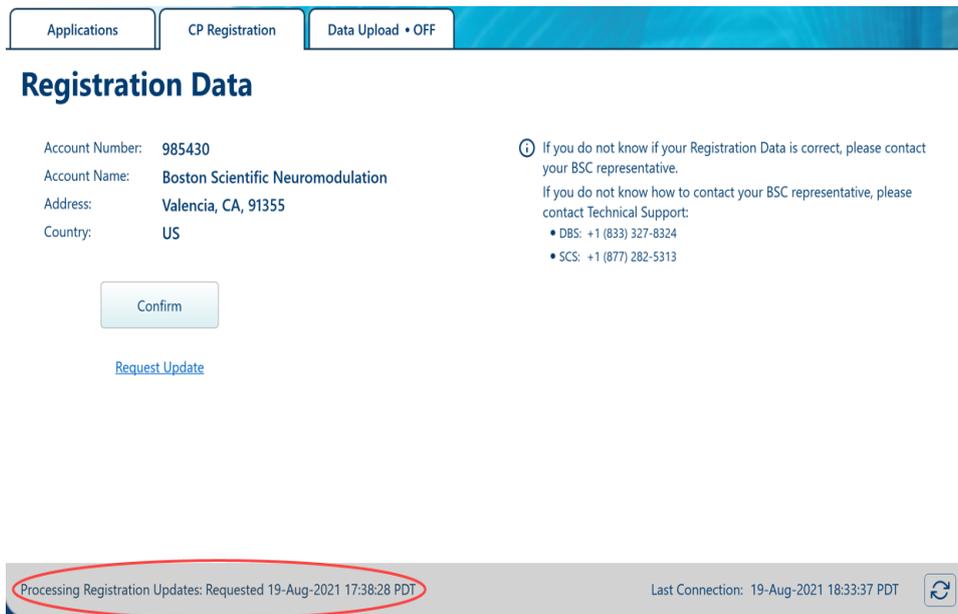
Figure 4. Registration Processing Status

5. If you selected **Request Update**, check back in 24 to 48+ hours to ensure the data has been updated. Once updated, select **Confirm** (Figure 1).

6. Navigate to the **Applications Tab** within AirWave (Figure 5).
7. Ensure the radio button filter for **All** applications is selected (Figure 5).
8. Find “Bionic Navigator 2.03” in the software list. The software status will display “Ready to Download.” Select **Download** (Figure 5).

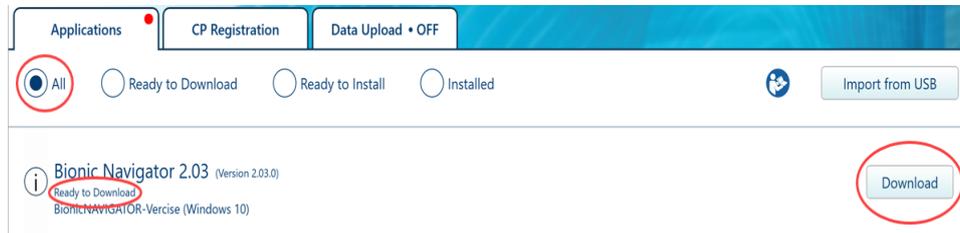
Figure 5. Download BN 2.03

9. Once Bionic Navigator 2.03 has completed downloading, you will be asked if you want to install the application now. Select **Yes**.

Note: If you select **No**, the application can be installed at a later time by selecting the **Install** button for the appropriate software application (Figure 6).

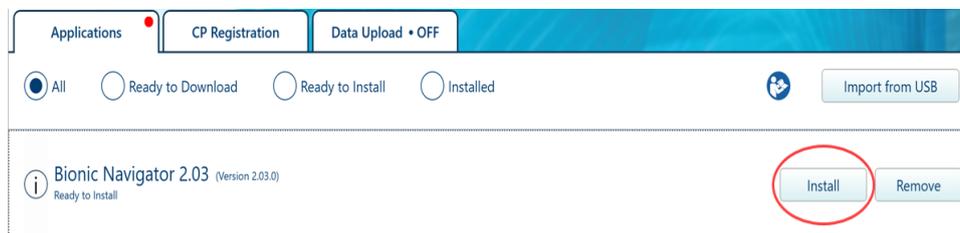
Figure 6. Install BN 2.03

10. Proceed to the “Complete the Software Installation on the CP” section of this manual.

Installation from USB

1. Insert the Installer USB into an available USB port.
2. Select the **Import from USB** button in AirWave (Figure 7).

Figure 7. AirWave Import from USB

3. Within the file explorer window pop-up, navigate to the USB drive and then select the installation file for Bionic Navigator 2.03 Software named “DB7101203W10.bsnpkg” (Figure 8).
4. Select **Open** for the installation file for Bionic Navigator 2.03 Software (DB7101203W10.bsnpkg) (Figure 8).

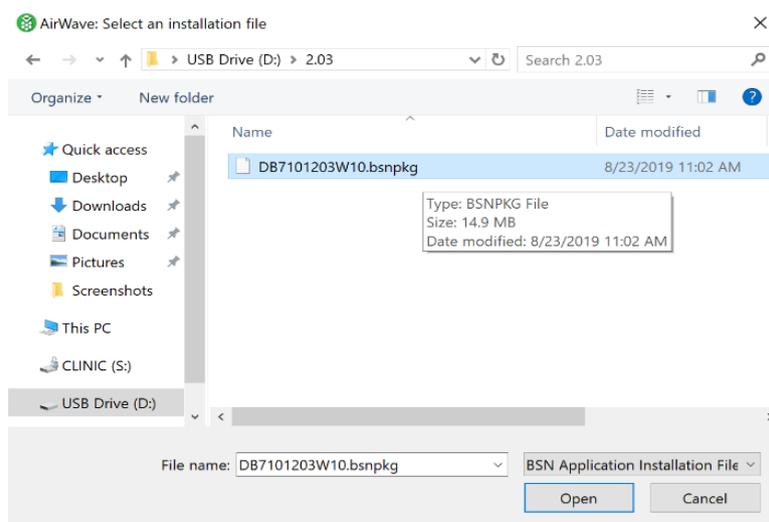
Figure 8. Select Installation File

5. You will be asked if you want to install the application now. Select **Yes**.

Note: If you select **No**, the application can be installed at a later time by selecting the **Install** button for the appropriate software application (Figure 9).

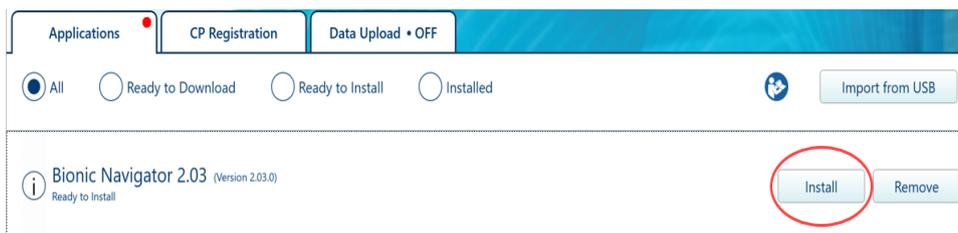
Figure 9. Install BN 2.03

6. Proceed to the “Complete the Software Installation on the CP” section of this manual.

Complete the Software Installation on the CP

1. Verify that the Software Part Number is 9028287-203, as shown in Figure 10.

BSN CP Product Activation

For assistance contact Customer Service at +1 (866) 360-4747 or contact your local representative.

Please visit the Boston Scientific Neuromodulation activation website, select Activate Clinician Programmer, and enter the information provided below. Then enter the Activation Key in the space below and click Activate to continue with the installation.

CP Model:	SURFACE PRO 6		
CP Serial Number:	PB005428284053		
Software Part Number:	9028287	-	203
Activation Key:	PVBPO	-	CGIGB

Figure 10. Clinician Programmer Product Activation

Note: The CP serial number and activation key displayed here are for illustration only.

BSN employees: To obtain an activation key, see the “Appendix B - Web Activation Portal” section of this manual, then continue to Step 2.

Non-BSN employees: Contact your local sales representative to obtain an activation key, then continue to Step 2.

2. On the CP, enter the activation key generated from the web activation portal or provided by your local sales representative (Figure 10).
3. Select the **Activate** button (Figure 10). This process may take several minutes as the system is being reconfigured to support the new application. Various status messages will be displayed as the Installer installs the Bionic Navigator 2.03 Software on the CP (Figure 11). Select **Next** or **Finish** as necessary.

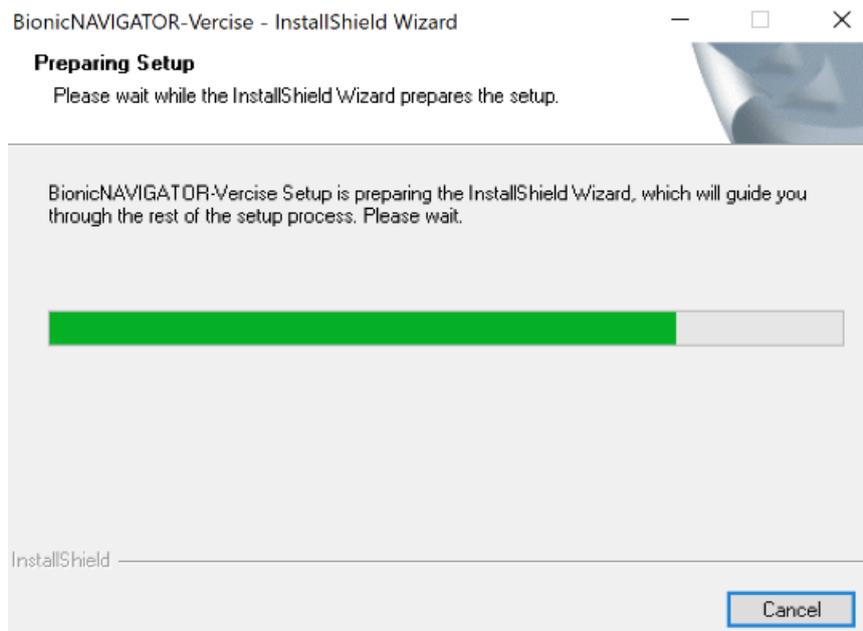
Figure 11. Software Installation on CP

4. After a successful installation, a confirmation key will be displayed (Figure 12).

Note: The confirmation key displayed here is for illustration only.

5. Record the confirmation key displayed on the screen and confirm the activation (Figure 12).

BSN employees: To confirm activation, see the “**Appendix C - Web Activation Portal, Confirm Activation**” section of this manual, then continue to **Step 6**.

Non-BSN employees: Contact your local sales representative to provide the confirmation key then continue to **Step 6**.

6. Select **OK** (Figure 12).

BSN CP Product Activation

For assistance contact Customer Service at +1 (866) 360-4747 or contact your local representative.

Please confirm this installation by reporting the Confirmation Key through the Boston Scientific Neuromodulation activation website.

Confirmation Key

KAHCV - MCVNC

OK

Figure 12. Confirmation Key

7. Navigate to the the **Applications Tab** within AirWave. Confirm the software status is “Installed” and that there are no error messages displayed in red under the “Installed” status (Figure 13).

Note: In addition, to view further details regarding the application, select the **Information** button  (Figure 13).

Note: If software was not successfully installed, refer to the “**Appendix A - McAfee Security Policies**” section of this manual and complete the steps provided to troubleshoot.

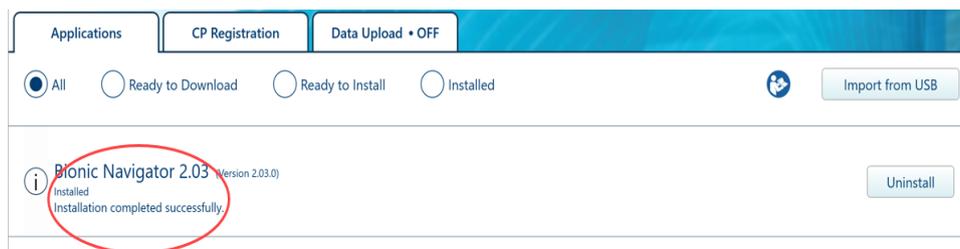
Figure 13. Installation Confirmation

8. Restart the CP to complete the installation.

- Once the CP restarts, double-click the Bionic Navigator 2.03 icon  on the desktop to launch the Bionic Navigator Software. Select **Tools** from the toolbar on the Start Screen and select **About Bionic Navigator** from the drop-down menu. The Software Part Number on the screen should now read 9028287-203 (Figure 14).

Note: *If the Vercise Neural Navigator Software application is also installed on the same CP, Bionic Navigator 2.03 Software can be accessed using the Vercise Launcher . Double-click the Vercise Launcher icon and select the Vercise DBS System icon  to start Bionic Navigator 2.03 Software.*

en

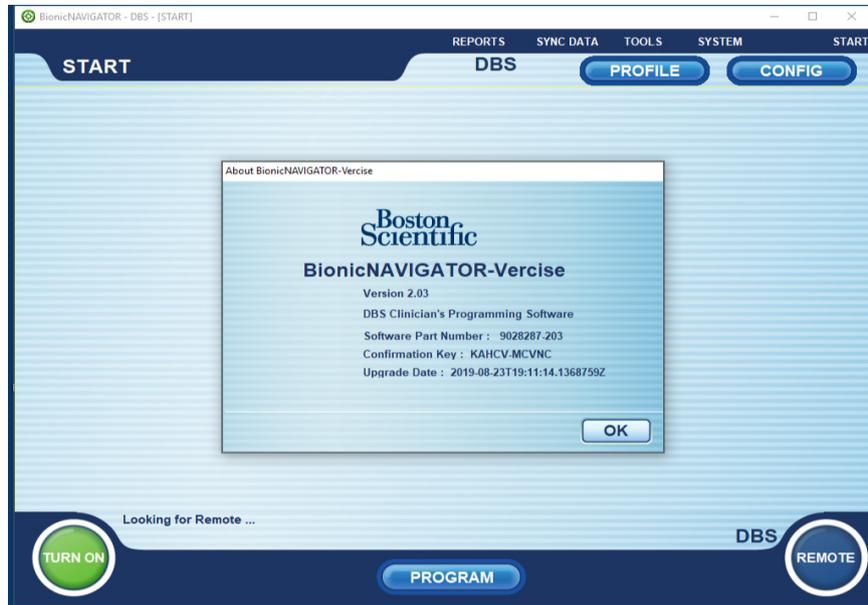
Figure 14. Software Part Number Confirmation

Note: *If you do not see a successful installation message, or the Software Part Number does not display correctly, contact Technical Support.*

- If software was installed via USB per the “Installation from USB” section of this manual, remove the Installer USB.

Uninstall and Remove Software or Delete the Database

To uninstall and, if desired, remove software from the CP, or to delete the database, follow the steps below:

1. Navigate to the CP desktop and ensure that all applications and programs are closed.
2. Double-click the AirWave icon  to launch AirWave.
3. Within the AirWave window, identify which software you would like to uninstall. Select the **Uninstall** button adjacent to the software application title that you wish to uninstall (Figure 15).

Note: After the software is uninstalled, the software can be reinstalled at a later date without downloading the software again and without an Installer USB.

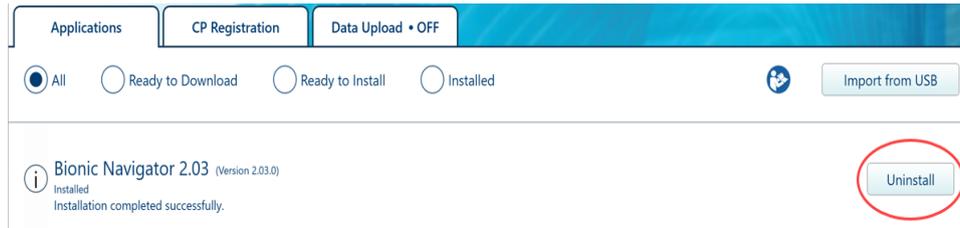
Figure 15. Software Uninstallation

4. Select **Yes** to proceed with the uninstallation (Figure 16).

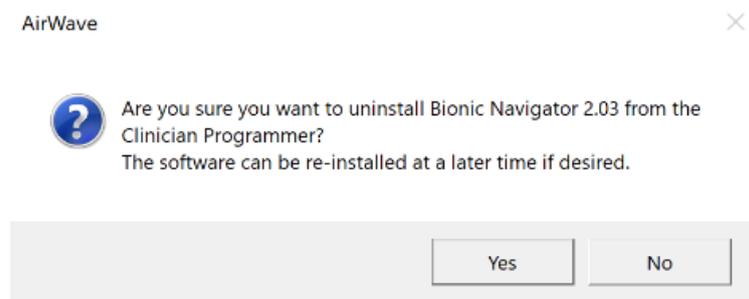
Figure 16. Uninstallation Prompt

5. Select **Remove BionicNAVIGATOR-Vercise** and then select **Next** to proceed with the uninstallation (Figure 17).

Note: If you would like to delete the database, select **Delete Database Only** and proceed through the prompts to complete database deletion.

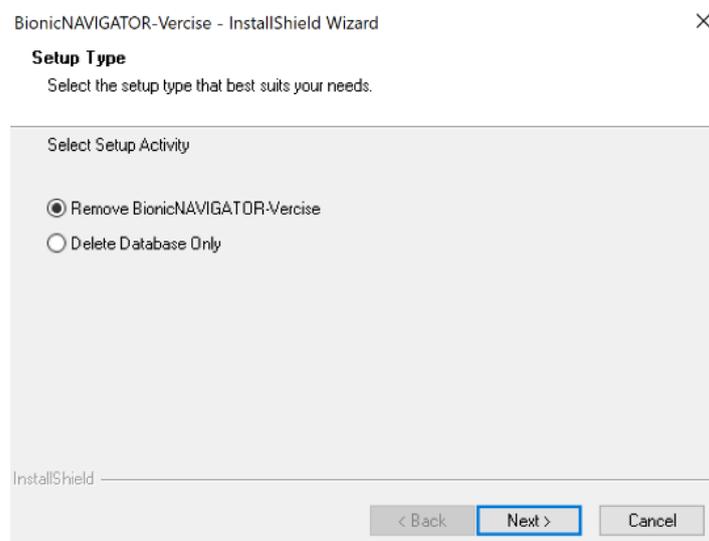
Figure 17. Uninstallation Setup Type

6. When prompted, select **Yes** to proceed with the uninstallation and then select **Finish** once the uninstallation is complete.

7. When prompted select **OK** to restart the CP.
8. Navigate to the the **Applications Tab** within AirWave and confirm the software was successfully uninstalled (Figure 18).

Note: Once the software is uninstalled, the two buttons displayed next to the software title will be “Install” and “Remove” (Figure 18).

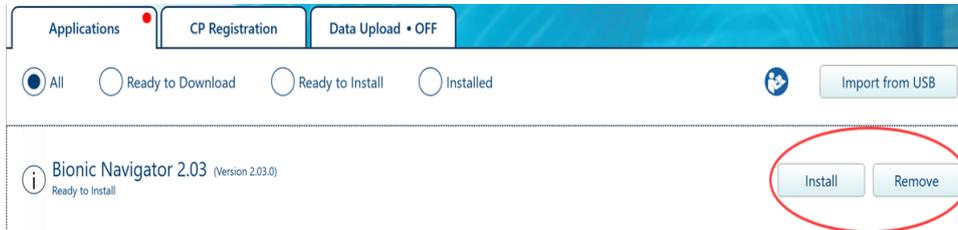
Figure 18. Uninstallation Confirmation

9. If desired, the software can be completely removed from the CP after uninstallation by selecting the **Remove** button adjacent to the software title (Figure 19).

Note: After the software has been removed from the CP, the software can be reinstalled at a later date by downloading again directly via AirWave or by using an Installer USB.

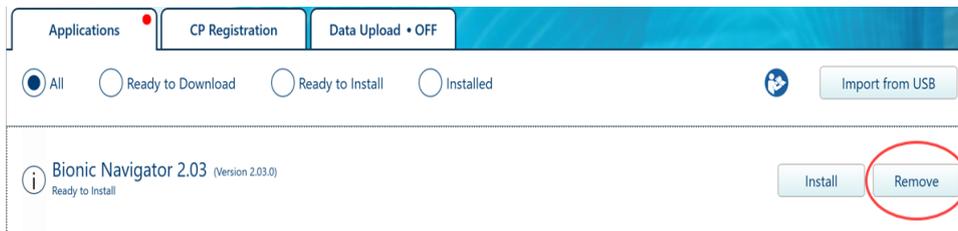
Figure 19. Remove Software

10. Select **Yes** to proceed with the software removal (Figure 20).

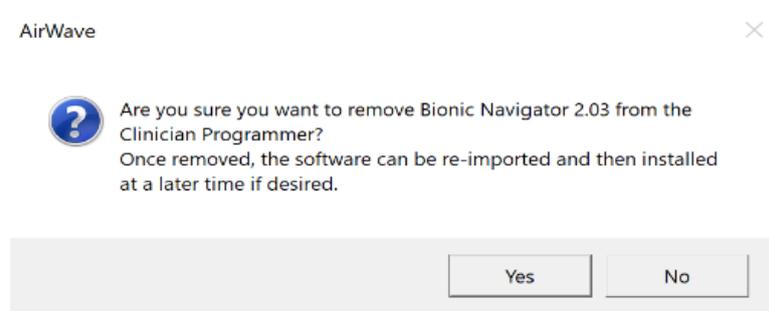
Figure 20. Software Removal Prompt

Appendix A - McAfee Security Policies

The following instructions outline how to update McAfee Security Policies.

1. Connect to WiFi.
2. Using the touch pad on the keyboard, move the pointer to the bottom of the screen to bring up the Status Bar.
3. If necessary, select  to show hidden icons (Figure 21).
4. Select the McAfee Icon  (Figure 21).
5. Select **McAfee Agent Status Monitor** (Figure 21).

Figure 21. McAfee Agent Status Monitor

6. Select **Collect and Send Props** on the McAfee Agent Monitor dialog (upper right button) (Figure 22).

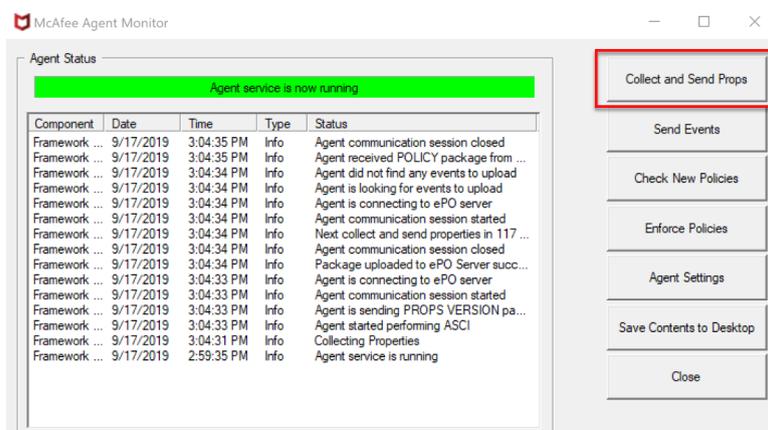
Figure 22. Collect and Send Props

7. Wait for the “Agent communication session closed” message to appear in the Agent Status List (about 1 to 2 minutes) (Figure 23).

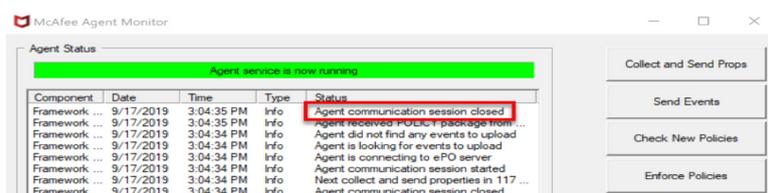
Figure 23. Agent Status List

8. Restart the CP.
9. Proceed to install software beginning at Step 4 of the “Setup for Software Installation” section of this manual.

Appendix B - Web Activation Portal

1. On a BSN computer, log onto the BSN VPN.
2. Once connected to the VPN, open a web browser and connect to the BSN Product Activation Portal using the following URL:

<http://nmdpascpwp00001/Default>

3. Click on **Activate CP Software** (Figure 24).

[Confirm Activation](#)

Figure 24. Web Activation Portal

4. Use the drop-down arrow to select the CP model, then enter the CP serial number and Software Part Number displayed on the CP screen into the appropriate fields on the web page (Figure 25).

Activate CP Software

CP Model:

CP Serial Number:

Software Part Number: -

[Return to Main Menu](#)

Figure 25. Activate CP Software Portal

Note: Serial numbers are case-sensitive. Note also that the letter “O” and the number “0” look similar and must be distinguished.

5. Click **Get Activation Key** to generate the activation key (Figure 25). Record the activation key, to be used in Step 5 of the “Complete the Software Installation on the CP” section of this manual. The activation key is valid for 5 days.
6. Click on the **Return to Main Menu** link.

Appendix C - Web Activation Portal, Confirm Activation

1. On the Product Activation Portal, click on **Confirm Activation** (Figure 26).

Figure 26. Web Activation Portal

2. Enter the confirmation key from the CP into the displayed field on the Product Activation Portal (Figure 27).

Figure 27. Confirm Activation Portal

3. Click **Confirm Activation** button (Figure 27).

Note: The confirmation key is displayed in a pop-up message upon activation on the CP. In addition, the confirmation key can be found by selecting the **Information** button ⓘ within AirWave for the applicable software.

Appendix D - Accessing the Software Label

1. Plug the CP into a power source.
2. Turn on the CP.
3. Log in as ClinicUser.
4. Navigate to the CP desktop and ensure that all applications and programs, including any previous versions of Bionic Navigator Software and Vercise Neural Navigator Software, are closed.
5. From the CP desktop, launch AirWave .
6. Navigate to the **Applications Tab** within AirWave and select the **Information** button  adjacent to the software application title for which you wish to view the label (Figure 28). To enlarge the label, select the **Label** button (Figure 28).

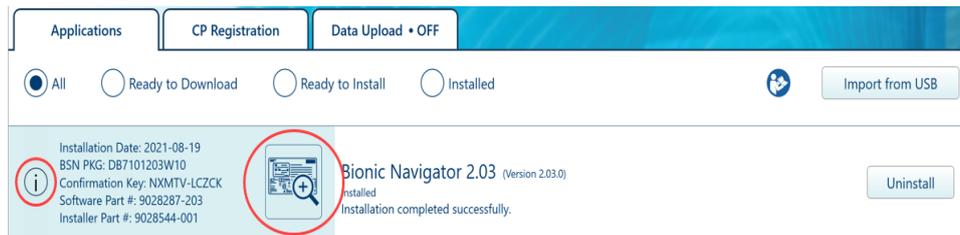
Figure 28. Software Label

7. To close the label image, select the  icon at the top of the label image.

Technical Support

There are no user serviceable parts. If you have a specific question or issue, contact your sales representative. To contact Boston Scientific for any other reason, use the contact information provided for your locality in the list below.

en

Argentina

T: +5411 4896 8556 F: +5411 4896 8550

Australia / New Zealand

T: 1800 676 133 F: 1800 836 666

Austria

T: +43 1 60 810 F: +43 1 60 810 60

Balkans

T: 0030 210 95 37 890 F: 0030 210 95 79 836

Belgium

T: 080094 494 F: 080093 343

Brazil

T: +55 11 5853 2244 F: +55 11 5853 2663

Bulgaria

T: +359 2 986 50 48 F: +359 2 986 57 09

Canada

T: +1 888 359 9691 F: +1 888 575 7396

Chile

T: +562 445 4904 F: +562 445 4915

China – Beijing

T: +86 10 8525 1588 F: +86 10 8525 1566

China – Guangzhou

T: +86 20 8767 9791 F: +86 20 8767 9789

China – Shanghai

T: +86 21 6391 5600 F: +86 21 6391 5100

Colombia

T: +57 1 629 5045 F: +57 1 629 5082

Czech Republic

T: +420 2 3536 2911 F: +420 2 3536 4334

Denmark

T: 80 30 80 02 F: 80 30 80 05

Finland

T: 020 762 88 82 F: 020 762 88 83

France

T: +33 (0) 1 39 30 97 00 F: +33 (0) 1 39 30 97 99

Germany

T: 0800 072 3301 F: 0800 072 3319

Greece

T: +30 210 95 42401 F: +30 210 95 42420

Hong Kong

T: +852 2960 7100 F: +852 2563 5276

Hungary

T: +36 1 456 30 40 F: +36 1 456 30 41

India – Bangalore

T: +91 80 5112 1104/5 F: +91 80 5112 1106

India – Chennai

T: +91 44 2648 0318 F: +91 44 2641 4695

India – Delhi

T: +91 11 2618 0445/6 F: +91 11 2618 1024

India – Mumbai

T: +91 22 5677 8844 F: +91 22 2617 2783

Italy

T: +39 010 60 60 1 F: +39 010 60 60 200

Korea

T: +82 2 3476 2121 F: +82 2 3476 1776

Malaysia

T: +60 3 7957 4266 F: +60 3 7957 4866

Mexico

T: +52 55 5687 63 90 F: +52 55 5687 62 28

Middle East / Gulf / North Africa

T: +961 1 805 282 F: +961 1 805 445

The Netherlands

T: +31 30 602 5555 F: +31 30 602 5560

Norway

T: 800 104 04 F: 800 101 90

Philippines

T: +63 2 687 3239 F: +63 2 687 3047

Poland

T: +48 22 435 1414 F: +48 22 435 1410

Portugal

T: +351 21 3801243 F: +351 21 3801240

Singapore

T: +65 6418 8888 F: +65 6418 8899

South Africa

T: +27 11 840 8600 F: +27 11 463 6077

Spain

T: +34 901 11 12 15 F: +34 902 26 78 66

Sweden

T: 020 65 25 30 F: 020 55 25 35

Switzerland

T: 0800 826 786 F: 0800 826 787

Taiwan

T: +886 2 2747 7278 F: +886 2 2747 7270

Thailand

T: +66 2 2654 3810 F: +66 2 2654 3818

Turkey – Istanbul

T: +90 216 464 3666 F: +90 216 464 3677

Uruguay

T: +59 82 900 6212 F: +59 82 900 6212

UK & Eire

T: +44 844 800 4512 F: +44 844 800 4513

Venezuela

T: +58 212 959 8106 F: +58 212 959 5328

Note: *Phone numbers and fax numbers may change. For the most current contact information, please refer to our website at*

<http://www.bostonscientific.com>

or write to the following address:

Boston Scientific Neuromodulation Corporation

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All rights reserved.
Boston Scientific (Australia) Pty Ltd
Boston Scientific Limited