**Boston  
Scientific**

Advancing science for life™

**Vercise™ DBS  
Charging  
Handbook**

en

fr

es

pt-BRA

91171709-16

Content: MP91171709-16 REV A

en

## Trademarks

All trademarks are the property of their respective holders.

## Guarantees

Boston Scientific Corporation reserves the right to modify, without prior notice, information relating to its products in order to improve their reliability or operating capacity.

Drawings are for illustration purposes only.

## Additional Information

Refer to the appropriate *Information for Patients DFU and Remote Control Handbook* for additional instructions and information about your DBS System. Refer to the *Labeling Symbols* document for an explanation of labeling symbols.

References to the Charging System or charging process are applicable only when using a rechargeable DBS Stimulator.

## Contacting Boston Scientific

To contact Boston Scientific, see the “*Technical Support*” section of this manual.

## Product Model Numbers

Model Number	Product Name
DB-64125	Vercise Charging System
DB-6300-C	Vercise Charging Collar Accessories
DB-6200-C	Vercise Charging Collar, Small
DB-6100-C	Vercise Charging Collar, Medium
NM-6350	Adhesive Kit
NM-5312	Charger
NM-5305	Base Station
NM-6305	Power Supply & Adapter For Charger (EU)

---

# Table of Contents

**Introduction**..... 1

**Using the Charging System**..... 2

    Getting Set Up.....4

    Charging the Charger.....5

    Understanding the Charger Sound Alerts.....7

    Charging Your Stimulator .....8

    Using the Charging Collar .....9

    Using the Adhesive Patch .....11

    Properly Aligning the Charger .....12

    Tips for Properly Aligning the Charger .....14

    Cleaning Instructions.....16

**Technical Support** ..... 17

*This page intentionally left blank.*

# Introduction

The Boston Scientific Deep Brain Stimulation (DBS) System provides a reversible therapy that stimulates structures in the brain with small electrical pulses. The Boston Scientific DBS System includes a rechargeable Stimulator that requires periodic recharging. The Charging System allows you to recharge the Stimulator battery as needed.

The Charging System for the Stimulator consists of the Charger, a Base Station, and a Power Supply. The Charging System also includes a Charging Collar that holds the Charger over the Stimulator.

This manual provides you with instructions for charging your Stimulator. We encourage you to read this and your other Boston Scientific manuals entirely. If you have any questions, or need clarification on anything contained in this manual, contact Technical Support.

# Using the Charging System

You should expect a daily recharging time of 5 to 30 minutes per day or a periodic recharging time of 30 minutes to 4 hours every 1 to 2 weeks. Your recharge routine may vary depending on your stimulation settings. Some stimulation settings will require more frequent charging. Boston Scientific recommends any recharge routine that fits your schedule and lifestyle while maintaining sufficient charge to maintain stimulation.

Developing a recharge routine involves finding the right balance between four factors:

- How much power is required to experience effective therapy
- How often you want to recharge
- How long you want to recharge
- How you would like to manage your personal schedule

The Boston Scientific DBS System's programming software gives your healthcare provider a conservative recommendation for how often to charge. This estimate assumes stimulation is on 24 hours per day, 7 days per week at the programmed energy output level. While you may want to follow these recommendations, you and your physician can also develop an appropriate charge routine that best fits your schedule.

The Remote Control provides an easy-to-view Stimulator battery meter on the Home screen and messages to inform you of the battery's condition. When the Remote Control indicates a low battery, the Stimulator should be recharged as soon as possible. Failure to recharge may result in loss of stimulation. After stimulation stops, communication with the Stimulator will also stop until the Stimulator is sufficiently charged. Refer to your *Remote Control Handbook* for more information.

The rechargeable Stimulator battery should provide at least five years of service. In many cases, the Stimulator battery should provide at least 25 years of service. Battery life is dependent on the stimulation settings and conditions.

Over time and with repeated charging, the battery in the Stimulator will lose its ability to recover its full capacity. As a result, the Stimulator may require shorter intervals between charges. The Stimulator will need replacement when stimulation can no longer be maintained with a routine charging.

## Getting Set Up

Check that you have received all the components included in your DB-64125 Vercise Charging System (see Table 1). If any item is missing, contact Technical Support for a replacement.

en

Table 1: Components of the Vercise Charging System	
Label	Description
A	Charging Collar (2)
B	Base Station (1)
C	Power Supply (1) and Adapter (1)
D	Counterweight (1)
E	Charger (1)
F	Charging Spacer (1)

Also, be sure you received your Patient Identification Card from your healthcare provider.

## Charging the Charger

The Charger contains a rechargeable battery that is used to recharge your Stimulator's battery (see Table 2).

### Cautions:


- Keep the Charging System dry. It should not be exposed to moisture.
- Do not connect the Charger to any other device except to the Base Station provided with it.
- Keep the Charging System away from pets, pests, and children to avoid damage to the device.

To charge the Charger:

1. Find a convenient electrical outlet that will allow you to place the Base Station on a clear, flat surface and will not expose the Charging System to water or direct heat.
2. Plug the Power Supply into a standard AC wall outlet.
3. Connect the Power Supply to the Base Station.
4. Place the Charger in the Base Station with the power button facing up.

**Note:** *The Base Station is designed to remain connected to a power outlet at all times. When it is not being used, keep the Charger on the Base Station so that it is always ready to deliver a charge.*

**Table 2: Vercise Charger System Components**

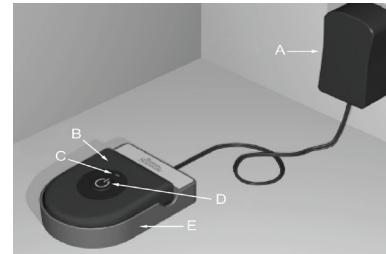Label	Description
A	Power Supply
B	Charger
C	Indicator Light
D	Power Button
E	Base Station

The Charger is ready to fully charge your Stimulator when the Charger's indicator light is green. If the indicator light is yellow, the Charger can only partially charge the Stimulator. While the Charger may be used when the indicator light is yellow, it may not be able to fully charge your Stimulator and you may need to charge sooner than usual.

en


Charger Indicator Light	Description
Green	The Charger is fully charged and ready to charge the Stimulator.
Yellow	The Charger is partially charged. The Charger may still be used, but it may not be able to fully charge the Stimulator. To fully charge the Charger, place it in the Base Station.
Off	The Charger is off – or – the Charger battery is empty. If pressing the Power Button does not turn on the Charger, place the Charger in the Base Station to charge.

### Charger Battery Service Life

The Charger battery has a typical service life of 500 charging cycles.

## Understanding the Charger Sound Alerts

The Charger emits different sounds to alert you of specific situations. Before you begin charging your Stimulator, it is important to understand these sounds. The following chart describes what each sound means.

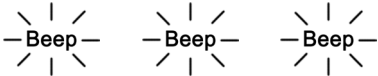Sound	Description	
	Repeated Single Beep	The Charger is searching for your Stimulator and is not properly aligned with the Stimulator. Align the Charger as shown in the <i>“Properly Aligning the Charger”</i> section of this manual.
	Repeated Double Beep	The Charger has fully charged the Stimulator battery. Turn off the Charger and return it to the Base Station.

## Charging Your Stimulator

When charging, you must use the Charger with either the Charging Collar or an Adhesive Patch to hold the Charger over the Stimulator. The Adhesive Patches are made of non-reactive and natural Latex-free material suitable for most sensitive skin types.

### **WARNINGS:**

- Do not charge while sleeping. This may result in a burn.
- While charging, the Charger may become warm. It should be handled with care.
- Failure to use either the Charging Collar or an Adhesive Patch while charging may result in a burn. If pain or discomfort is felt, stop charging and contact your healthcare provider.

## Using the Charging Collar

There are two lengths of the Charging Collar available (Small and Medium). If you need to order a different size Charging Collar or a replacement Charging Collar, contact Technical Support.

1. When the indicator light is green, remove the Charger from the Base Station.

**Note:** *Once the Charger is removed from the Base Station, the Charger indicator light will turn off regardless of the ready status of the Charger.*

2. If instructed to do so by your healthcare provider, place the Charging Spacer at the back of the pocket (Figure 1).
3. Place the Charger into the appropriate pocket on the Charging Collar with the power button facing out (Figure 1). If your Stimulator is on the right side of your chest, place the Charger in the right pocket. If the Stimulator is on the left side of your chest, place the Charger in the left pocket.
4. If instructed to use the Charging Spacer, make sure it is between the Charger and the back of the pocket (Figure 1).
5. Place the Counterweight in the pocket opposite of the Charger.

**Figure 1. Placing the Charger and the Charging Spacer in the Pocket**

6. Put the Charging Collar over your neck, with the pockets facing out (Figure 2).
7. For best charging results, make sure the Charger is centered over the Stimulator. If the Charger is not centered, charging time may increase. See the “*Properly Aligning the Charger*” and “*Tips for Properly Aligning the Charger*” sections of this manual to ensure you center the Charger over the Stimulator. It is recommended that you follow the instructions in both sections the first time you charge your Stimulator.
8. If the Charger is not centered over the Stimulator, you may need to adjust the length of the Charging Collar using the straps. Occasionally checking that the Charger is aligned over the Stimulator during your charging session is recommended.

**Note:** *The Charging Collar can be placed under or over clothing. You should not wear tight fitting or heavy clothing over the Charger while charging to allow air flow around the Charger.*

**Note:** *Do not use the Charging Collar when it is wet or damp.*

**Figure 2. Charging Collar Placed Over the Neck**


## Using the Adhesive Patch

1. When the indicator light is green, remove the Charger from the Base Station.

**Note:** *Once the Charger is removed from the Base Station, the Charger indicator light will turn off regardless of the ready status of the Charger.*

2. Apply the Adhesive Patch to the backside of the Charger by peeling the clear liner from the patch and applying the white side with the blue stripe to the rear of the Charger (Figure 3).
3. Remove the skin side beige liner from the adhesive (only good for one fixation).
4. For best charging results, make sure the Charger is centered over the Stimulator. If the Charger is not centered, charging time may increase. See the “*Properly Aligning the Charger*” and “*Tips for Properly Aligning the Charger*” sections of this manual to ensure you center the Charger over the Stimulator. It is recommended that you follow the instructions in both sections the first time you charge your Stimulator.

**Note:** *Do not put the Charger directly on the skin (e.g., without an Adhesive Patch).*

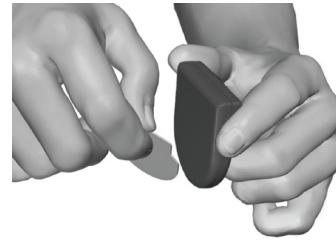**Figure 3. Peel the Clear Liner from the Adhesive Patch**

## Properly Aligning the Charger

1. Press the power button. The indicator light will come on again, and the Charger will begin beeping repeatedly to signal that it is searching for the Stimulator.
2. Position the Charger over the Stimulator.

The beeping will stop when the Charger is partially aligned with the Stimulator. For optimal charging, make sure the Charger is centered over the Stimulator (Figure 4).

Charging time may increase if the Charger is not fully aligned with the Stimulator. To fully align the Charger over the Stimulator, position the bottom curve of the Charger approximately 1 cm lower than the bottom curve of the Stimulator.

If you are having trouble properly aligning the Charger or experiencing longer than normal charging times, see the “*Tips for Properly Aligning the Charger*” section of this manual.

**Figure 4. Position the Charger Over the Stimulator**

3. Secure the Charger over the Stimulator by aligning the Charging Collar, or by pressing the adhesive to the skin over the Stimulator.

**Note:** *If the Charging Collar moves out of alignment, or if the Adhesive Patch is accidentally located in the wrong place, the Charger will start beeping again. Realign the Charging Collar, or use a new Adhesive Patch to place the Charger back into the fully aligned position.*

**Note:** *Moving around while charging is acceptable, but be aware that the Charger must stay centered over the Stimulator for proper charging. Excessive movement may cause the Charger and Stimulator to become misaligned.*

4. When the Charger emits a distinct double beep, press the power button to turn off the Charger and check the Stimulator's battery with the Remote Control.
5. If the Stimulator is fully charged, remove the Charging Collar or Adhesive Patch from your body.

**Note:** *Do not confuse the end of charge signal (a distinct double beep) with repeated beeping indicating misalignment.*

6. Separate the Charger from the Charging Collar or Adhesive Patch.  
Set aside the Charging Collar or discard the Adhesive Patch.
7. Return the Charger to the Base Station.

## Tips for Properly Aligning the Charger

If you are having trouble properly aligning the Charger or are experiencing longer than normal charging times, you can use the following procedure to help you properly align the Charger.

1. Lightly press on your skin until you find the implanted Stimulator.
2. Press the Power Button on the Charger. The Charger will begin beeping repeatedly to signal that it is searching for the Stimulator.
3. Place the Charger to the left of the Stimulator.
4. Slowly move the Charger to the right, towards the Stimulator.
5. Stop moving the Charger when it stops beeping.
6. Make a mark on your skin with a soft-tipped marker at the right edge of the Charger (Figure 5).
7. Place the Charger to the right of the Stimulator.
8. Slowly move the Charger to the left, towards the Stimulator.
9. Stop moving the Charger when it stops beeping.
10. Make a mark on your skin at the left edge of the Charger (Figure 6).

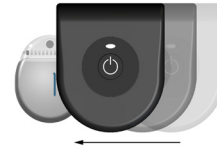
Figure 5. Right Edge Alignment of Charger

Figure 6. Left Edge Alignment of Charger

11. Place the Charger above the Stimulator.
12. Slowly move the Charger down, towards the Stimulator.
13. Stop moving the Charger when it stops beeping.
14. Make a mark on your skin at the bottom edge of the Charger (Figure 7).
15. Place the Charger below the Stimulator.
16. Slowly move the Charger up, towards the Stimulator.
17. Stop moving the Charger when it stops beeping.
18. Make a mark on your skin at the top edge of the Charger (Figure 8).

The marks on your skin should create a square or crosshair. The center of this crosshair indicates the center of the Stimulator.

19. Now that you have located the center of the Stimulator, properly align the Charger over the Stimulator (Figure 9).

**Figure 7. Top Edge Alignment of Charger**

**Figure 8. Bottom Edge Alignment of Charger**

**Figure 9. Proper Alignment of Charger**

## Cleaning Instructions

**Cleaning the Charger, Base Station, and Power Supply.** The components can be cleaned using alcohol or a mild detergent applied with a cloth or tissue. Residue from soapy detergents should be removed with a damp cloth. Do not use abrasive cleansers for cleaning. Do not clean the Charger, Base Station, or Power Supply while they are connected to a power outlet.

**Cleaning the Charging Collar.** Hand wash the Charging Collar with mild soap and warm water. Do not machine wash the charging collar. Let the Charging Collar air dry. Be sure to remove the Charger and Counterweight from the Charging Collar before washing the Charging Collar.

# Technical Support

There are no user serviceable parts. If you have a specific question or issue, contact your healthcare provider. To contact Boston Scientific for any other reason, use the contact information provided for your locality in the list below.

**Argentina**

T: +5411 4896 8556 F: +5411 4896 8550

**Australia / New Zealand**

T: 1800 676 133 F: 1800 836 666

**Austria**

T: +43 1 60 810 F: +43 1 60 810 60

**Balkans**

T: 0030 210 95 37 890 F: 0030 210 95 79 836

**Belgium**

T: 080094 494 F: 080093 343

**Brazil**

T: +55 11 5853 2244 F: +55 11 5853 2663

**Bulgaria**

T: +359 2 986 50 48 F: +359 2 986 57 09

**Canada**

T: +1 888 359 9691 F: +1 888 575 7396

**Chile**

T: +562 445 4904 F: +562 445 4915

**China – Beijing**

T: +86 10 8525 1588 F: +86 10 8525 1566

**China – Guangzhou**

T: +86 20 8767 9791 F: +86 20 8767 9789

**China – Shanghai**

T: +86 21 6391 5600 F: +86 21 6391 5100

**Colombia**

T: +57 1 629 5045 F: +57 1 629 5082

**Czech Republic**

T: +420 2 3536 2911 F: +420 2 3536 4334

**Denmark**

T: 80 30 80 02 F: 80 30 80 05

**Finland**

T: 020 762 88 82 F: 020 762 88 83

**France**

T: +33 (0) 1 39 30 97 00 F: +33 (0) 1 39 30 97 99

**Germany**

T: 0800 072 3301 F: 0800 072 3319

**Greece**

T: +30 210 95 42401 F: +30 210 95 42420

**Hong Kong**

T: +852 2960 7100 F: +852 2563 5276

**Hungary**

T: +36 1 456 30 40 F: +36 1 456 30 41

**India – Bangalore**

T: +91 80 5112 1104/5 F: +91 80 5112 1106

**India – Chennai**

T: +91 44 2648 0318 F: +91 44 2641 4695

**India – Delhi**

T: +91 11 2618 0445/6 F: +91 11 2618 1024

**India – Mumbai**

T: +91 22 5677 8844 F: +91 22 2617 2783

**Italy**

T: +39 010 60 60 1 F: +39 010 60 60 200

**Korea**

T: +82 2 3476 2121 F: +82 2 3476 1776

**Malaysia**

T: +60 3 7957 4266 F: +60 3 7957 4866

**Mexico**

T: +52 55 5687 63 90 F: +52 55 5687 62 28

**Middle East / Gulf / North Africa**

T: +961 1 805 282 F: +961 1 805 445

**The Netherlands**

T: +31 30 602 5555 F: +31 30 602 5560

**Norway**

T: 800 104 04 F: 800 101 90

**Philippines**

T: +63 2 687 3239 F: +63 2 687 3047

**Poland**

T: +48 22 435 1414 F: +48 22 435 1410

**Portugal**

T: +351 21 3801243 F: +351 21 3801240

**Singapore**

T: +65 6418 8888 F: +65 6418 8899

**South Africa**

T: +27 11 840 8600 F: +27 11 463 6077

**Spain**

T: +34 901 11 12 15 F: +34 902 26 78 66

**Sweden**

T: 020 65 25 30 F: 020 55 25 35

**Switzerland**

T: 0800 826 786 F: 0800 826 787

**Taiwan**

T: +886 2 2747 7278 F: +886 2 2747 7270

**Thailand**

T: +66 2 2654 3810 F: +66 2 2654 3818

**Turkey – Istanbul**

T: +90 216 464 3666 F: +90 216 464 3677

**Uruguay**

T: +59 82 900 6212 F: +59 82 900 6212

**UK & Eire**

T: +44 844 800 4512 F: +44 844 800 4513

**Venezuela**

T: +58 212 959 8106 F: +58 212 959 5328

**Note:** *Phone numbers and fax numbers may change. For the most current contact information, please refer to our website at <http://www.bostonscientific-international.com/> or write to the following address:*

**Boston Scientific Neuromodulation Corporation**

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or its affiliates. All rights reserved.
Boston Scientific (Australia) Pty Ltd
Boston Scientific Limited
© 2020 Boston Scientific Corporation