Boston Scientific

Advancing science for life™

Vercise™ DBS
Remote Control 4
Directions for Use

Directions for Use

en

Mode d'emploi

fr

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es

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pt-BRA

Rx ONLY CAUTION: Federal law restricts this device to sale, distribution and use by or on the order of a physician.

92378769-04

Content: MP92378769-04 REV A

How to Use this Manual

This manual describes the usage of the Boston Scientific Vercise DBS Remote Control. Read all instructions carefully before using the DBS Systems. Throughout this manual, the term “Boston Scientific DBS System” refers to the Vercise Genus™ Deep Brain Stimulation (DBS) System.

Read all instructions carefully before using the DBS System. For indications for use, contraindications, warnings, precautions, adverse events, storage and handling, sterilization, and warranty information refer to the *DBS Information for Prescribers DFU*. For other device-specific information not included in this manual, refer to the appropriate DFU for your Boston Scientific DBS System as listed in your *DBS Reference Guide*.

Note: *Images of the Remote Control Home Screen within this manual are representative of those displayed for a rechargeable system unless specified otherwise. The Home Screen for a non-rechargeable Stimulator does not include the battery level of the Stimulator.*

Guarantees

Boston Scientific Corporation reserves the right to modify, without prior notice, information relating to its products in order to improve their reliability or operating capacity.

Drawings are for illustration purposes only.

Trademarks

All trademarks are the property of their respective holders.

The **Bluetooth®** word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Boston Scientific Neuromodulation Corporation is under license.

Contacting Boston Scientific

To contact Boston Scientific, see the “*Technical Support*” section of this manual.

Product Model Numbers

Model Number	Description
DB-5572-1A and DB 5572-1	Vercise DBS Remote Control 4 Kit
DB-5270	Vercise DBS Remote Control 4
NM-6210	Remote Control Holster
DB-6315 and NM-6315	USB Power Supply, Remote Control, Universal
NM-6310	USB Power Supply, Remote Control, North America
DB-6386	DBS Pairing Magnet

Note: *Only device models that are appropriate for your region will be provided.*

Table of Contents

Description	1
Cautions.....	1
Remote Control Features	2
Basic Operation	3
Unlocking the Remote Control.....	3
Linking a Remote Control to a Patient's Stimulator.....	4
Establishing Communication Between Stimulator and Clinician Programmer.....	6
The Home Screen.....	7
Turning Stimulation ON/OFF.....	8
Selecting Your Language/Screen Preference.....	8
Accessing Additional Options from the Main Menu	9
Accessing the Main Menu.....	9
Program Selection and Activation.....	10
Restoring Programs.....	11
Changing Stimulation Level.....	12
Accessing System Settings	13
System Settings Menu.....	13
Remote Settings.....	13
System Info.....	16
Accessing the Clinician Menu	18
Stimulator Search and Linking.....	20
Impedances.....	24
Reset Stimulator.....	26
CP Mode: Stimulator.....	27
Exit Clinician Mode.....	28
Remote Control Battery	29
Remote Control Battery Status.....	29
Recharging the Remote Control.....	29
MRI Mode	30
Enabling MRI Mode.....	30
Disabling MRI Mode.....	33
MRI Mode Error Screens.....	36
ERI or EOS Screens During MRI Mode.....	40
Stimulator Battery Messages	43
ERI (Elective Replacement Indicator) Screen (For Non-Rechargeable Stimulators Only).....	43

en

Vercise™ DBS Remote Control 4 Directions for Use

End of Service (EOS) (For Non-Rechargeable Stimulators Only)	44
Stimulator Low Battery Screens (For Rechargeable Stimulators Only)	45
ETS Low Battery Screens (For ETS 3 Only).....	46
Troubleshooting	47
Bluetooth Help	47
No Program to Run Screen.....	48
Invalid Program Screen	49
Unsupported Stimulator Screen.....	49
System Settings Reset Screen	50
Stimulator Error Screen	50
Remote Error Screen.....	51
Icon Description	52
Technical Support	54

en

Description

The Remote Control is a hand-held device that communicates with the Stimulator through a Bluetooth Low Energy telemetry link. The Vercise Remote Control 4 is only compatible with Vercise Genus Stimulators and the Vercise External Trial Stimulator 3 (ETS 3). The Remote Control functions include the following:

- Linking/De-Linking to a Stimulator
- Turning stimulation ON and OFF
- Increasing stimulation amplitude
- Changing and restoring Programs
- Accessing Clinician Menu Options
- Customizing Remote Control settings: language, volume, brightness
- Enabling/Disabling MRI Mode for MR conditional Stimulators


In addition to the functions listed above, the Remote Control is used to establish communication between the Stimulator and Clinician Programmer (CP) for device programming. See the “*Establishing Communication Between Stimulator and Clinician Programmer*” section of this manual for instructions.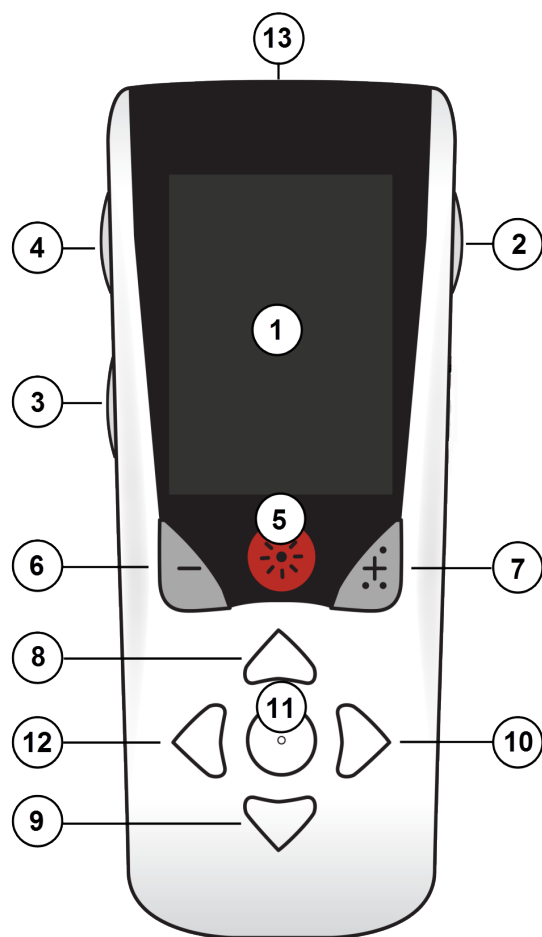

Cautions

- Keep the Remote Control dry. It should not be exposed to moisture.
- Do not connect the Remote Control USB port to any other device except to the Remote Control charger provided with it.
- Keep the Remote Control away from pets, pests, and children to avoid damage to the device.

Remote Control Features

en

1. **Screen:** Displays all menus, icons, and messages.
2. **Lock/Unlock Button** : Unlocks the Remote Control from sleep mode.
3. **Programs Button P**: Displays the Programs menu. Also acts as a shortcut (press + hold) to prepare the Stimulator for connection to the CP.
4. **Home Button** : Displays the Home screen.
5. **Stimulation Button** : Turns stimulation ON or OFF.
6. **Minus Button** : Decreases selected parameter.
7. **Plus Button** : Increases selected parameter.
8. **Navigation Button** : Allows scrolling through a selected menu.
9. **Navigation Button** : Allows scrolling through a selected menu.
10. **Navigation Buttons** : Makes a selection.
11. **Navigation Buttons** : Makes a selection.
12. **Navigation Button** : Returns to the previous screen.
13. **USB Port (not shown):** Allows recharging of the Remote Control with the USB power supply.

The Remote Control has a range of 3 meters (9.8 ft) with implanted Vercise Genus Stimulators under normal conditions. The Remote Control has a range of 6 meters (19.7 ft) with the External Trial Stimulator 3 (ETS 3) under normal conditions. When using the Remote Control, avoid common sources of interference, such as Wi-Fi routers, cordless phones, Bluetooth wireless streaming devices, baby monitors, and microwave ovens. For additional telemetry information, see the “Bluetooth Help” section of this manual.

Basic Operation

Unlocking the Remote Control

The Remote Control goes into idle (or sleep) mode when not in use. It can be reactivated by pressing the **Lock/Unlock**  button on the right side of the Remote Control.

en

Note: The Remote Control will not communicate with an implanted rechargeable Stimulator during charging.


Linking a Remote Control to a Patient's Stimulator

A new Remote Control that is intended to be given to a patient for their use must be linked to the patient's Stimulator as the 'Patient Remote.'


To link a Remote Control to the patient's stimulator:

en

1. Place the Stimulator into Pairing Mode by placing the DBS Pairing Magnet directly over the Stimulator for at least 2 seconds and then remove the magnet.
2. Unlock the Remote Control.
3. The  **Link to Stimulator?** screen will appear. Select  **Yes** to link to the Stimulator. The **Stimulator Search** screen displays. If the  **Link to Stimulator?** screen does not automatically display, you can manually initiate linking from the  **Clinician Menu**. See the “*Stimulator Search and Linking*” section of this manual for further information.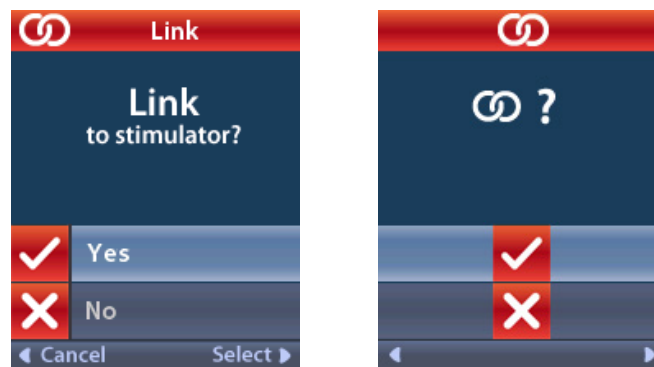

4. Select the Stimulator that you wish to link to. The  **Link as Patient Remote?** screen displays with the Stimulator serial number.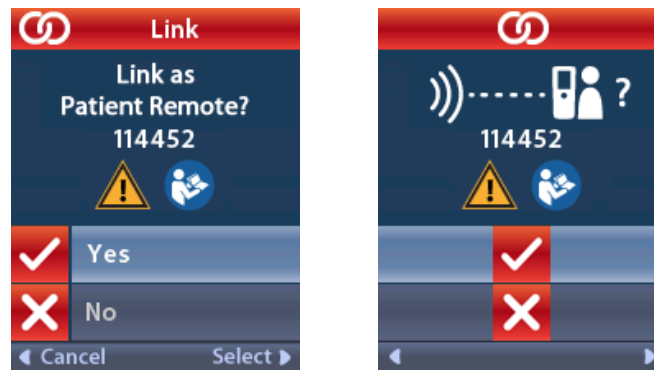

5. Select  **Yes** to link as the Patient Remote. A Remote Control that is intended to be utilized by the patient must be linked as the Patient Remote.

or


Select  **No** to cancel the action or to temporarily link to the Stimulator for use in the clinic. See the “*Stimulator Search and Linking*” section of this manual for further information.

Establishing Communication Between Stimulator and Clinician Programmer

The Remote Control can be used to establish communication between the Stimulator and Clinician Programmer (CP) for device programming. The Remote Control must be linked to the Stimulator to perform this function.

From the **Lock** screen, press and hold the **P** button (approximately 5 seconds) until the screen displays “Stimulator Ready for CP Connection.”

en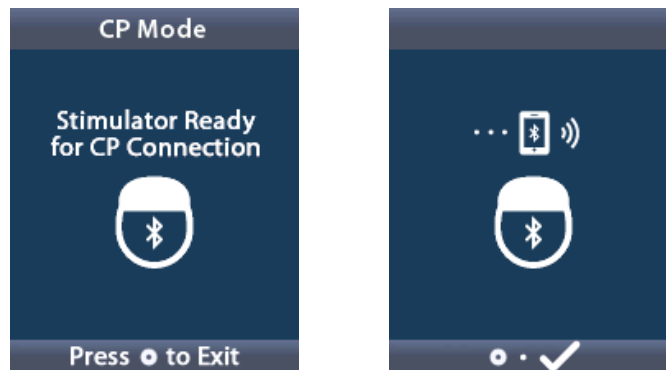


This will place the Stimulator into CP Mode (Pairing Mode) so that a CP may connect to it. Then proceed to connect the CP to the Stimulator. See the “*CP Mode: Stimulator*” section of this manual for additional information.

The Home Screen

After unlocking the Remote Control, the **Home** screen will appear if the Remote Control was previously linked to a Stimulator. The **Home** screen shows the number of the active program, stimulation status (ON/OFF), navigation to the Main Menu, and the Stimulator battery level for rechargeable Stimulators. It also shows the battery level of the Remote Control (see the “*Remote Control Battery*” section of this manual). The **Home** screen can be accessed from any other screen by pressing the **Home**  button.


1. Press the **Home**  button on the left side of the Remote Control to view the **Home** screen.

Home Screen for Rechargeable Stimulator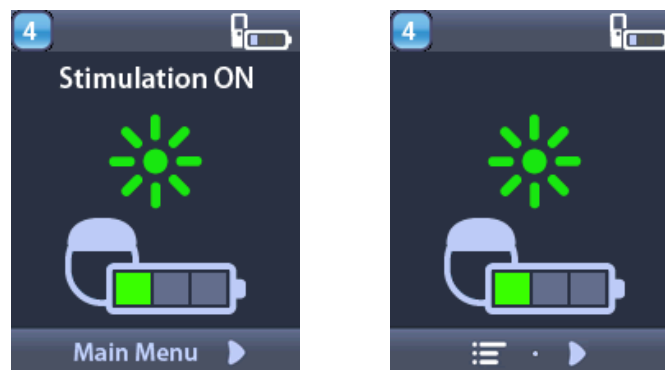

Note: When the Remote Control connects to a rechargeable Stimulator, it will display the current battery level of the Stimulator on the Home screen.

Home Screen for Non-Rechargeable Stimulator

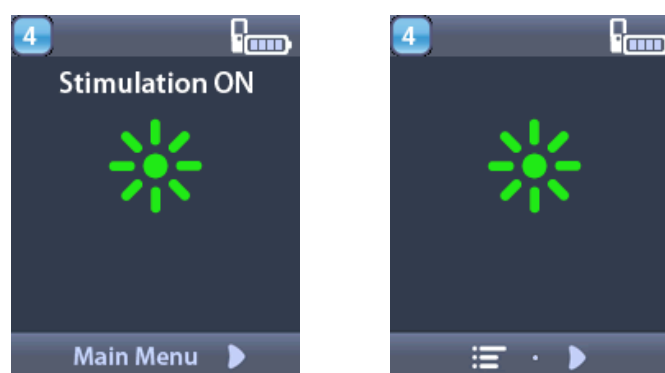Note: When the Remote Control connects to a non-rechargeable Stimulator, the battery level of the Stimulator will not be displayed on the screen.

Turning Stimulation ON/OFF

After unlocking the Remote Control, press the **Stimulation ON/OFF**  button to turn the Stimulator ON or OFF.

Selecting Your Language/Screen Preference

en

The Remote Control allows you to choose either an iconic screen or a text screen in one of the languages provided. See the “*Languages*” section of this manual for additional instructions.

Note: *Throughout this manual, both the English text screens and the iconic screens are shown.*



Accessing Additional Options from the Main Menu

The  **Main Menu** gives you access to the following options:

-  **Programs**
-  **Stimulation**
-  **System Settings**

Accessing the Main Menu

1. Press the **Home**  button on the left side of the Remote Control to display the **Home** screen.
2. From the **Home** screen, press the **Right Arrow**  button to select the  **Main Menu**.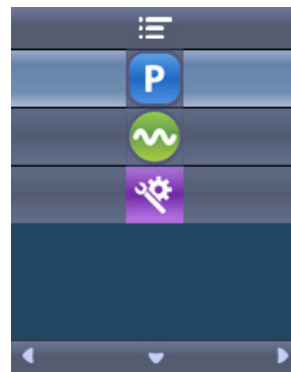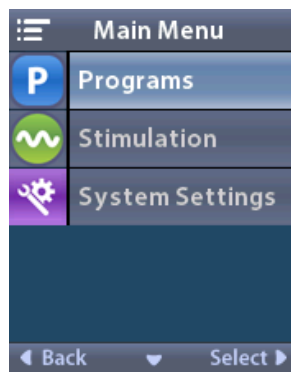

Note: When the Stimulator has no programs saved on it, the Main Menu will only display the System Settings option.

en


Program Selection and Activation

A stimulation program is a set of stimulation parameters that determine the patient’s therapy. A program may apply stimulation to as many as four independent stimulation fields or areas, depending on how the program is set up. For example, one area may correspond to a right brain target, whereas another area may correspond to a left brain target. The Remote Control can store up to four programs – numbered 1 through 4 – for the patient to select and activate at any time.


en

1. Press the **P** button on the left side of the Remote Control to display the **Programs** screen.

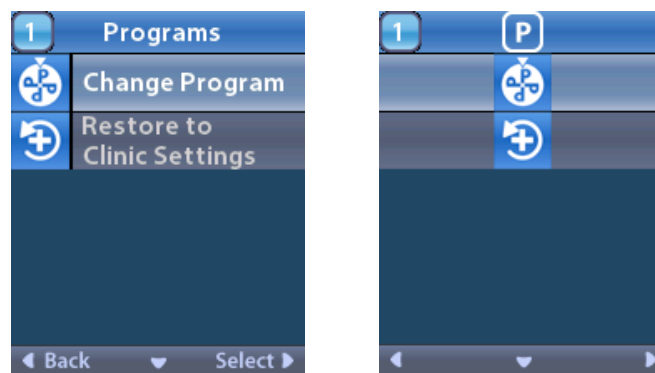Note: You can also navigate to the Programs menu from the Main Menu. See the “Accessing the Main Menu” section of this manual for additional instructions.

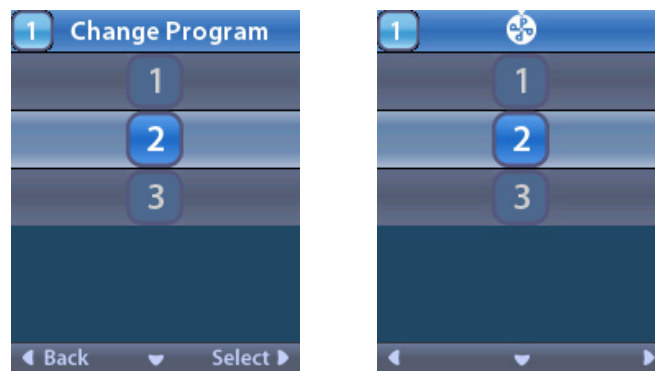2. Select  **Change Program**.

3. Press the **Up**  or **Down**  Arrow buttons to navigate to the desired program.
4. Press the **Right Arrow**  button to select the program.

Restoring Programs


If after changing a program, you decide that you want to revert the program to its Clinic Settings, you can do so by restoring the program.

1. Press the **P** button on the left side of the Remote Control.


or

From the  **Main Menu** screen, select  **Programs**.

Select  **Restore to Clinic Settings**. The **Restore** screen displays a list of programs to restore.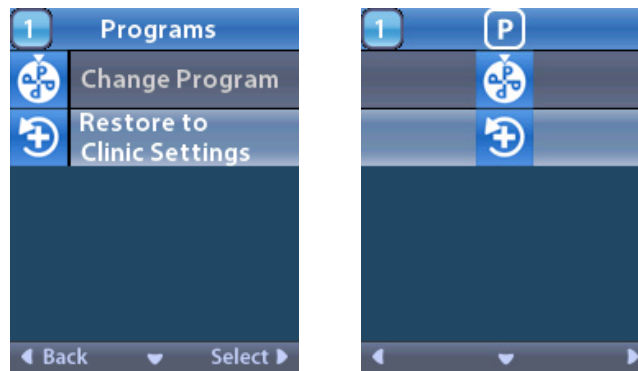

2. Select the program you want to restore. The **Restore Confirmation** screen displays.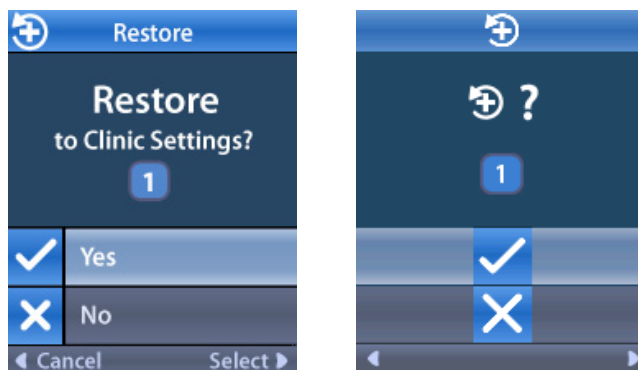

3. Select  **Yes** to restore the program to the Clinic Settings.

or


Select  **No** to cancel the action.

en

Changing Stimulation Level

A program may apply stimulation to one area, or up to four areas, depending on how the program is set up. The stimulation level for an area may be changed if allowed by the clinician using the Remote Control.

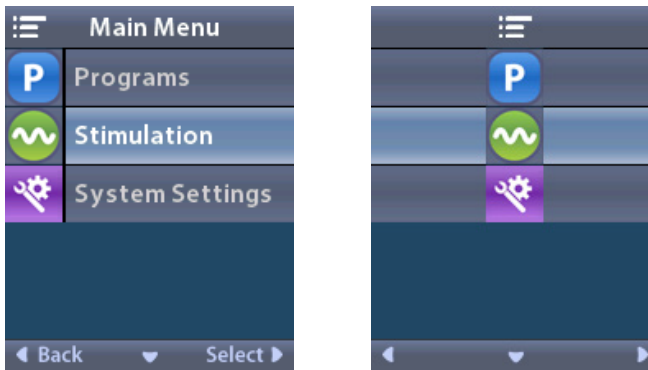1. From the  **Main Menu**: Select  **Stimulation**.

2. Use the **Navigation** buttons () to select the desired area.

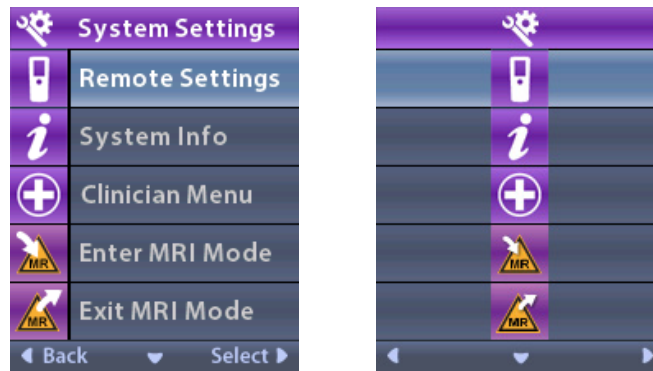
3. Press the **Minus**  button to decrease the level of stimulation.
4. Press the **Plus**  button to increase the level of stimulation.

Accessing System Settings

System Settings Menu

From the  **Main Menu**:

1. Select  **System Settings** to display the **System Settings** menu.

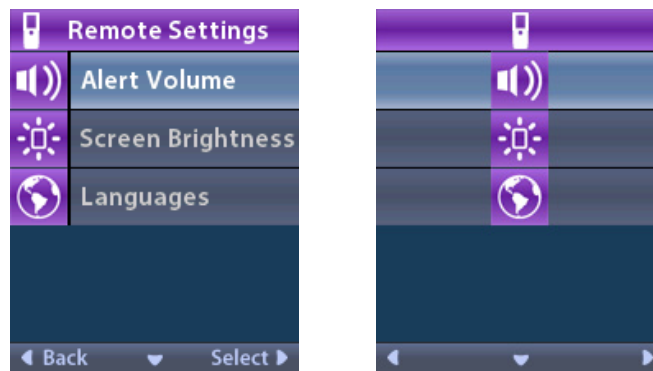
en

Remote Settings

The **Remote Settings** menu allows you to adjust the alert volume and screen brightness, select your language preference, transfer Stimulator data, erase Remote Control data, and view when Stimulator data was last synced.

From the  **System Settings** menu:


1. Select  **Remote Settings** to display the **Remote Settings** menu.

Alert Volume

From the  **Remote Settings** menu:

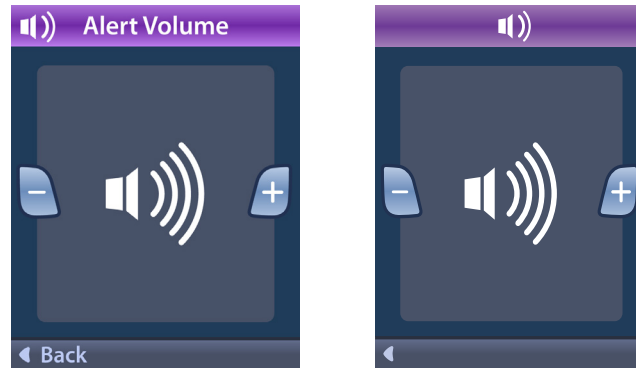1. Select  **Alert Volume**.

2. Press the **Minus**  button to decrease the alert volume.
3. Press the **Plus**  button to increase the alert volume.

Screen Brightness

From the  **Remote Settings** menu:

1. Select  **Screen Brightness**.

2. Press the **Minus**  button to decrease the brightness.
3. Press the **Plus**  button to increase the brightness.

Languages

The Remote Control allows you to configure your system to display one the following:

- Text screen in one of the available languages
- Iconic screen

From the  **Remote Settings** menu:

1. Select  **Languages**.

2. Select the desired language, such as  **English** or      iconic display.

The **Languages Confirmation** screen displays to confirm your selection.


en

System Info

The **i** **System Info** menu gives you the option of viewing information regarding the Remote Control or the Stimulator.


From the **i** **System Settings** menu, select **i** **System Info**.

en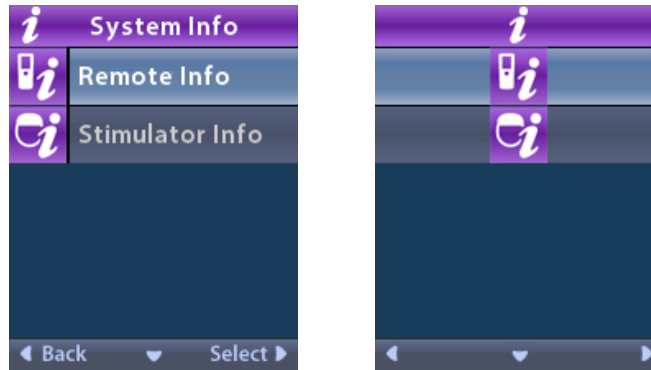

Remote Info


Select **i** **Remote Info** to view the **Remote Info** screen.

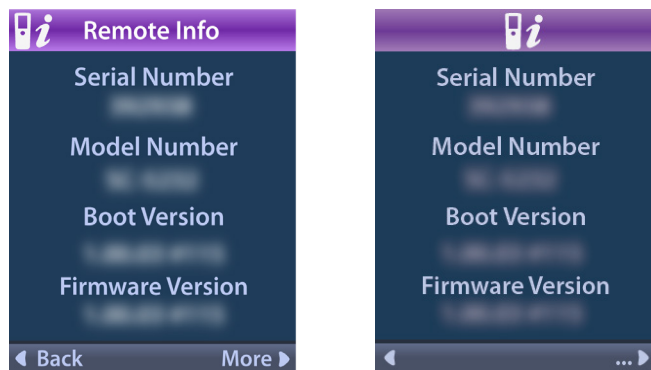The **Remote Info** screen displays the following information:

- Serial Number
- Model Number
- Boot Version
- Firmware Version


Select **More** to view additional Remote Info, such as LCD Boot Version and LCD Firmware.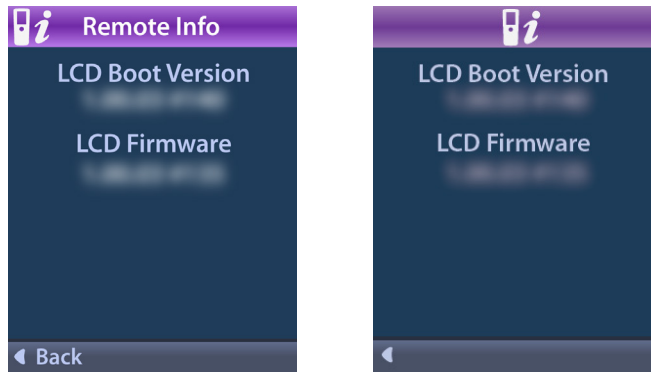

en

Stimulator Info

Select  **Stimulator Info** to view the **Stimulator Info** screen.

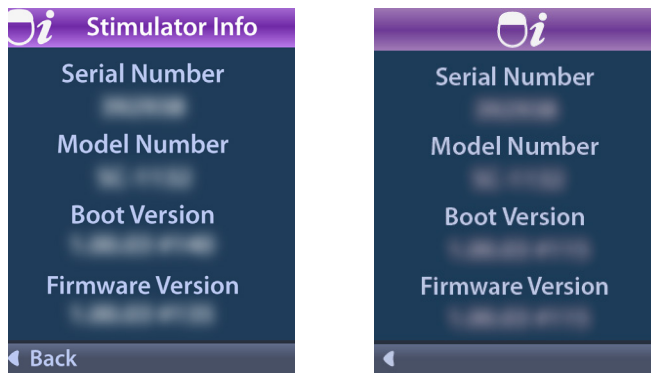The **Stimulator Info** screen displays the following information:

- Serial Number
- Model Number
- Boot Version
- Firmware Version



Accessing the Clinician Menu

The **Clinician Menu** allows you to link to a Stimulator, clear the link to a Stimulator, check impedances, and reset the Stimulator. To enter the Clinician Menu you must enter a password. Contact Boston Scientific Technical Support for the Clinician Password.

From the  **System Settings** menu:

en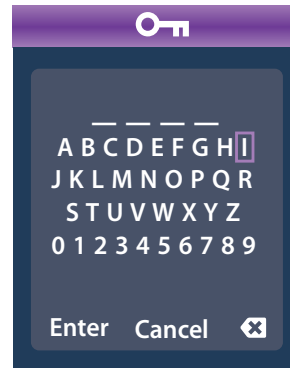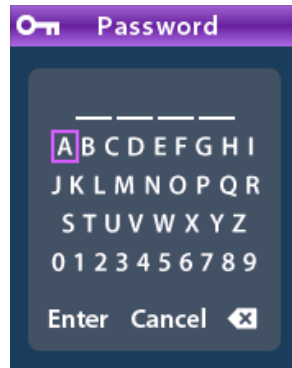

1. Select the  **Clinician Menu**. The  **Password** screen displays.


2. Use the **Navigation**  buttons to enter your password.

Or

- Use the **Navigation**  buttons to cancel.

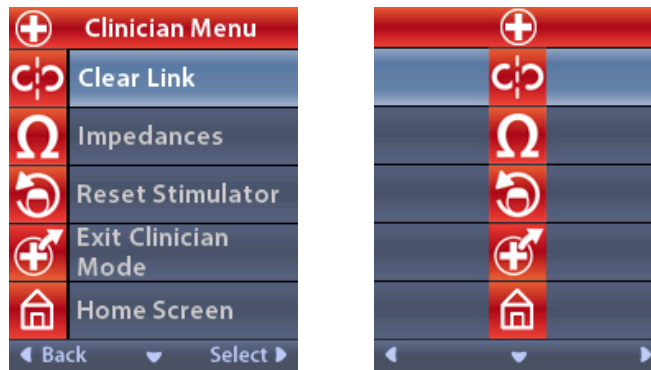If the password is incorrect, the  **Invalid Password** screen displays. If the password is correct, the  **Clinician Menu** displays.

en

Note: If the Remote Control is not linked to a Stimulator, the  **Stimulator Search** option displays on the Clinician Menu instead of the Clear Link option.

Select  to view additional **Clinician Menu** options, such as  **CP Mode: Stimulator**.

Stimulator Search and Linking

A Stimulator and Remote Control must be linked in order to communicate. A Remote Control that is intended to be utilized by the patient must be linked as the 'Patient Remote.' The Stimulator will only recognize one remote control device at a time as the Patient Remote; however, the Stimulator can be linked to a second Remote Control by establishing a temporary link.

en

A temporary link is intended only for clinician use when the Patient Remote is not available. A temporary link does not interfere with the link to the Patient Remote. Do not utilize the **Clear Link** option on the Patient Remote unless a new Remote Control device is to be assigned as the Patient Remote. If a new Remote Control device is assigned as the Patient Remote, any devices previously linked as the Patient Remote will be cleared. Once a link between a Stimulator and Remote Control has been established, they will remain linked unless the link is cleared.

The **Clinician Menu** allows you to search for a Stimulator and link to it. Stimulators that communicate using Bluetooth Low Energy, such as Vercise Genus, must be placed into Pairing Mode in order to link to a Remote Control.

In order to place an IPG (Implantable Pulse Generator) into Pairing Mode:

- Place the DBS Pairing Magnet directly over the Stimulator for at least 2 seconds.
- Then remove the DBS Pairing Magnet.
 - Magnet pairing may be used on an IPG that is implanted or in the packaging.


Note: *Once the magnet is removed, the Stimulator will automatically complete a device reset cycle. This device reset cycle will take several seconds to complete. During this reset cycle, the Stimulator will not provide stimulation and cannot communicate. Once the reset is complete, the Stimulator will return to normal operation.*

In order to place ETS 3 into Pairing Mode:

- Press and hold  on the ETS until the battery indicator light flashes, alternating between green and yellow.

If the Remote Control is not currently linked to a Stimulator, when the Remote Control is unlocked, the  **Link to Stimulator?** screen may automatically display.

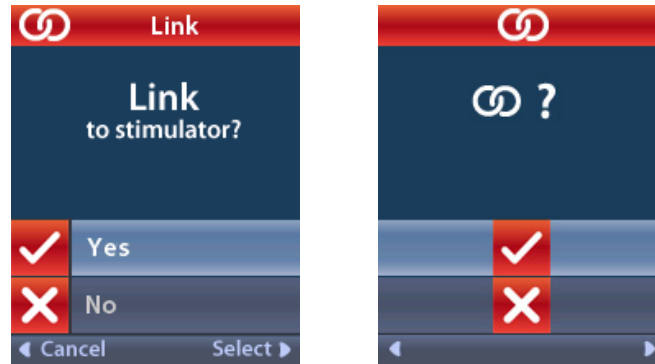
1. Select  **Yes** to link to the Stimulator. The **Stimulator Search** screen displays. Skip to Step 3 of this section.

en

If the  **Link to Stimulator?** screen does not automatically display, you can manually initiate linking from the  **Clinician Menu**.

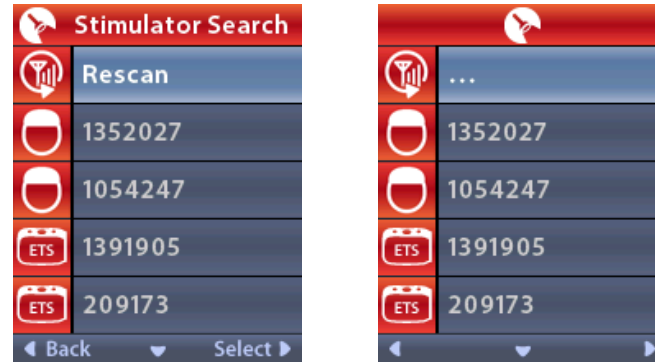2. From the  **Clinician Menu**, select  **Stimulator Search**. The **Stimulator Search** screen displays.


3. If the desired Stimulator has not been located, select  **Rescan**.

Note: Stimulators that utilize Bluetooth Low Energy communication must be in Pairing Mode in order to appear on the Stimulator Search screen and link.

4. Select the Stimulator that you wish to link to. The  **Link as Patient Remote?** screen displays with the Stimulator serial number. Only a Remote Control that is intended to be utilized by the patient at home should be linked as the Patient Remote. Selecting  **No** will create a temporary link with the Stimulator for use in the clinic.

en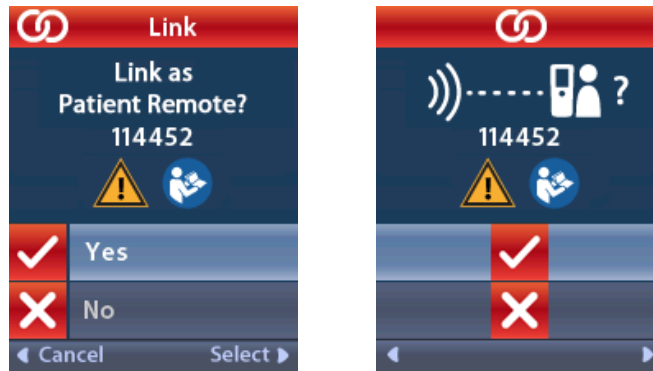


5. Select  **Yes** to link as Patient Remote to the Stimulator.

or

Select  **No** to temporarily link to the Stimulator for use in the clinic or to cancel the action.


Clear Link

If the Remote Control is linked to a Stimulator, the  **Clear Link** option displays on the **Clinician Menu**.

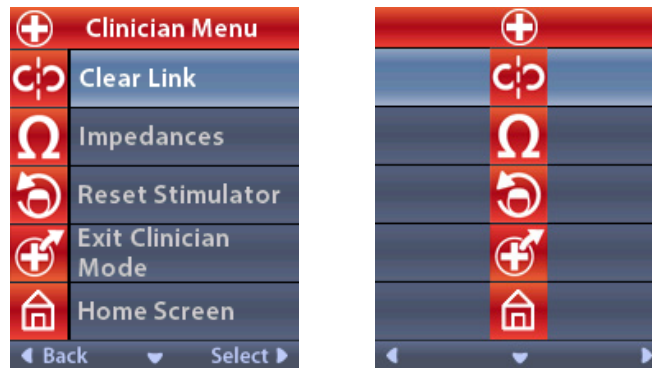From the  **Clinician Menu**:

1. Select  **Clear Link**. The **Clear Link to Stimulator?** screen displays.

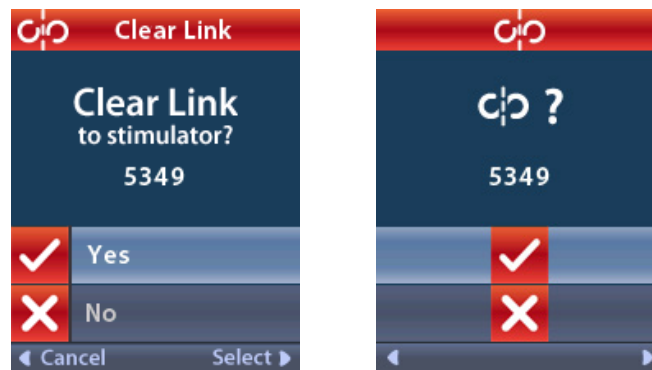2. Select  **Yes** to clear the link to the Stimulator.

or

Select  **No** to cancel the action.


en

Impedances

You can use the Remote Control to check impedances.


From the  **Clinician Menu**:

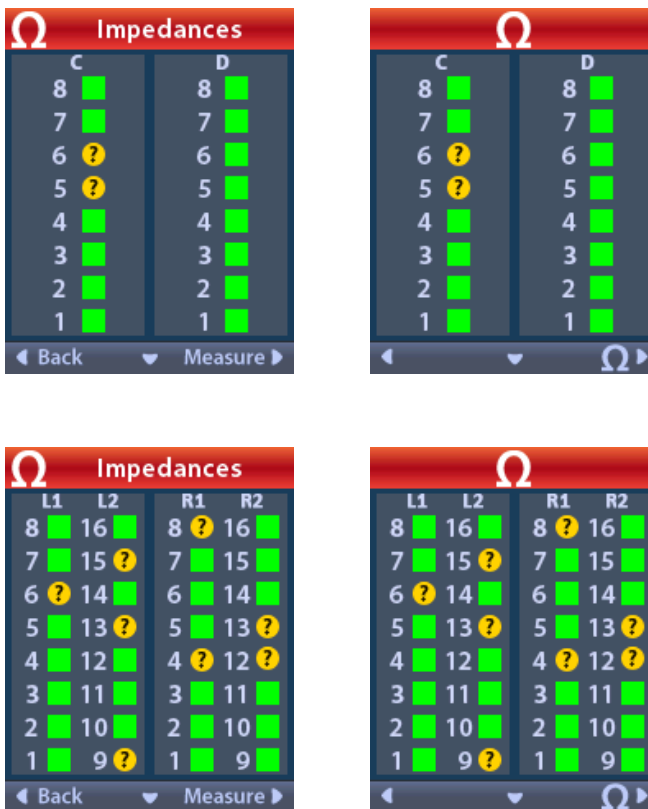1. Select  **Impedances**. An impedance measurement is taken and the **Impedances** screen is displayed.

When an impedance measurement is taken, impedances are assessed between a Contact and the case (monopolar), and between pairs of Contacts (bipolar). A green square indicates that impedance is within the acceptable range. A yellow dot with a question mark indicates that impedance is outside of the acceptable range (200 to 8000 Ohms).

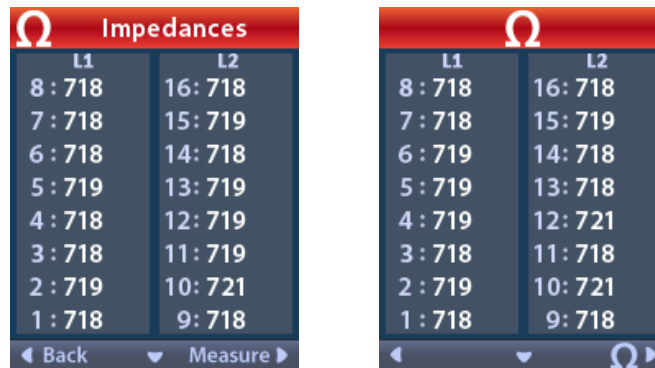Monopolar impedance values can be displayed by pressing the **Down Arrow**  button from the  **Impedances** screen.

en

Note: For 32 Contact Stimulators, the impedances are displayed on two screens. Press the **Down Arrow**  button to view the second screen.


2. Select **Measure**  to check impedances again.
3. Select  **Back** to go back to the **Clinician Menu**.

Reset Stimulator

You can use the Remote Control to reset the Stimulator.

From the  **Clinician Menu**:

1. Select  **Reset Stimulator**. The **Reset Stimulator?** screen displays.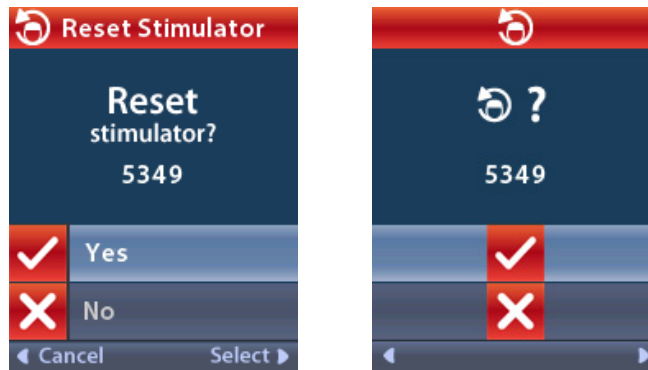

2. Select  **Yes** to reset the Stimulator.

or

Select  **No** to cancel the action.


en

CP Mode: Stimulator


The Remote Control can be used to establish communication between the implanted Stimulator (IPG) and Clinician Programmer (CP) for device programming. The Stimulator can be placed into CP Mode (Pairing Mode) by following the steps below. Alternatively, you may utilize the shortcut method described in the “Basic Operation” section of this manual.

From the  **Clinician Menu:**

1. Select  **CP Mode: Stimulator.**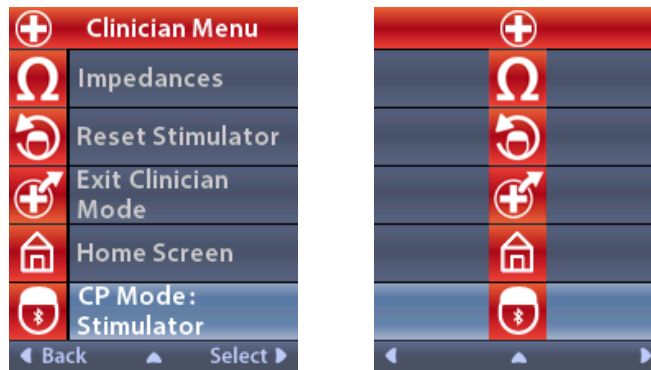

2. Connect the CP to the Stimulator and proceed with device programming.

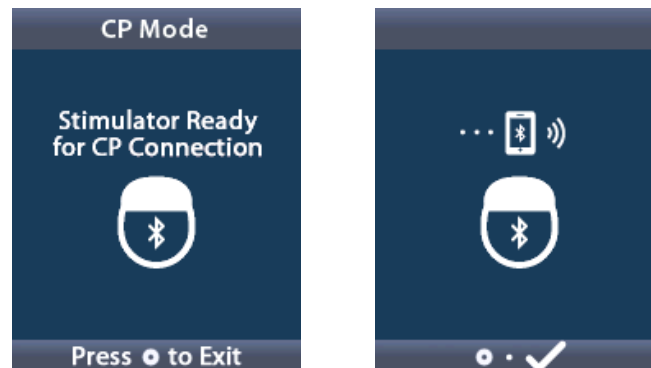3. Press  to exit **CP Mode.**

Note: *CP Mode (Pairing Mode) must be enabled for the CP to connect to and program the Stimulator using Bluetooth communication.*

en

Exit Clinician Mode

From the  **Clinician Menu**:


1. Select  **Exit Clinician Mode**.
2. The **System Settings** screen will display.

en

Remote Control Battery

Remote Control Battery Status


The  icon at the top right of the **Home** screen indicates the charge level of the Remote Control battery. The Remote Control will display messages reminding you to recharge the Remote Control if the battery power is low.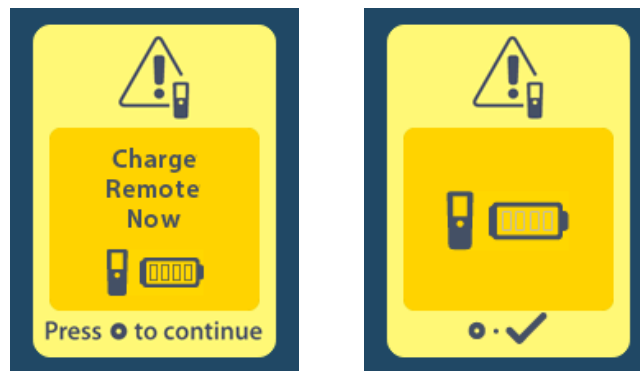

en

Recharging the Remote Control

Recharge the Remote Control battery by connecting the Remote Control to the USB power supply provided with the Remote Control. The Remote Control battery has a typical service life of 400 charging cycles.

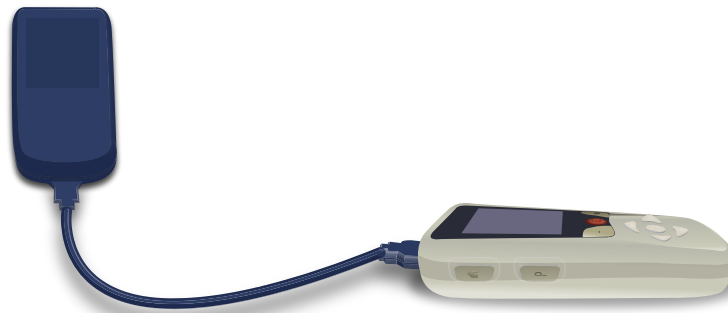MRI Mode

A patient implanted with an MR Conditional DBS System can undergo an MR Conditional Scan when all instructions listed in the *ImageReady™ MRI Guidelines for Boston Scientific DBS Systems* are followed, including enabling MRI Mode on the Stimulator. Contact Boston Scientific Technical Support or go to www.bostonscientific.com/manuals for the latest version of this manual.

en Enabling MRI Mode

When the Remote Control is linked to an MR Conditional Stimulator, the **Enter MRI Mode** icon  will appear on the **System Settings** screen. The Remote Control must be used to enable MRI Mode on the Stimulator before performing an MRI scan on a patient. Once the MRI scan is complete, disable MRI Mode. Do not leave the Stimulator in MRI Mode for extended periods of time beyond what is necessary to perform the MRI scan. The stimulation is automatically turned OFF when MRI Mode is enabled.

Warning: *Do not perform an MRI scan if MRI Mode is not enabled. Scanning under different conditions may result in patient injury or device malfunction.*

To enable MRI Mode:



1. Unlock the Remote Control by pressing the **Lock/Unlock**  button on the right side of the Remote Control.
2. After unlocking the Remote Control, the **Home** screen appears.



Note: *The Remote Control may display either a text screen in one of the languages provided or an iconic screen.*

3. Press the **Right Arrow**  button to navigate to the **Main Menu**. 



4. Select  **System Settings**.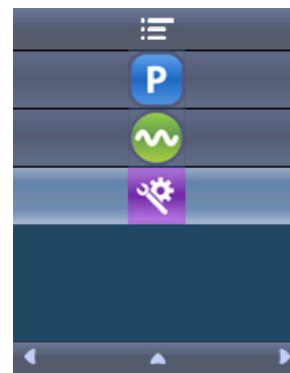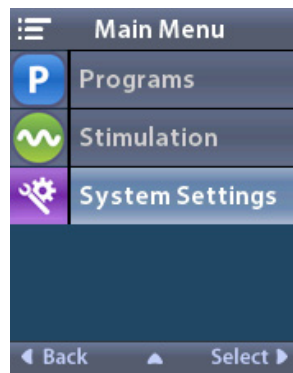

5. Select  **Enter MRI Mode**.

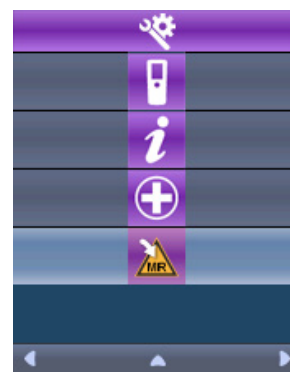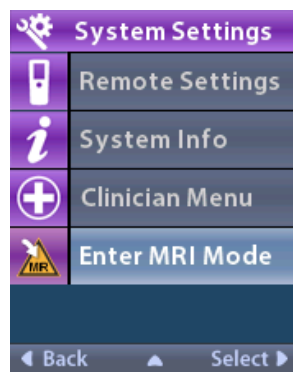
6. Select  **Yes** to enter MRI Mode or  **No** to cancel the action.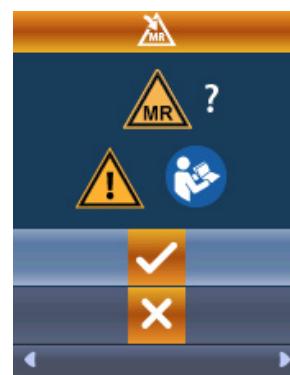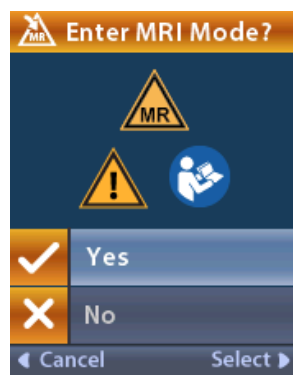


en

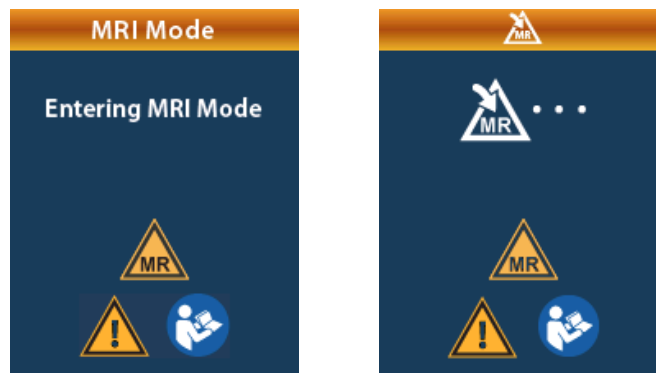7. The System performs a series of checks before MRI Mode is enabled.


en

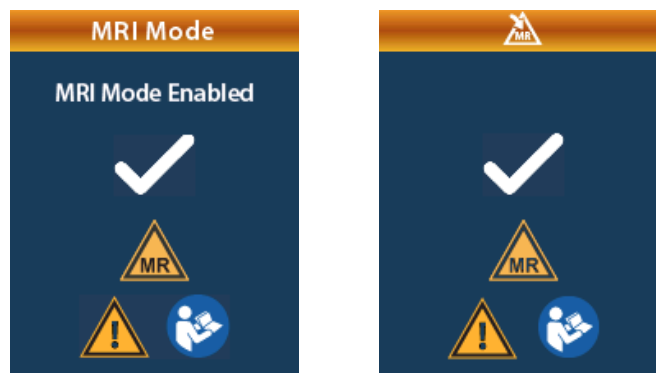8. If MRI Mode is enabled, stimulation is turned OFF and the **MRI Mode Enabled** confirmation screen is displayed.

9. The **Home** screen on the Remote Control will display the **MR Conditional Symbol**  if MRI Mode is enabled. Always confirm that the **Home** screen of the Remote Control displays the **MR Conditional Symbol**  before performing an MRI scan on the patient.

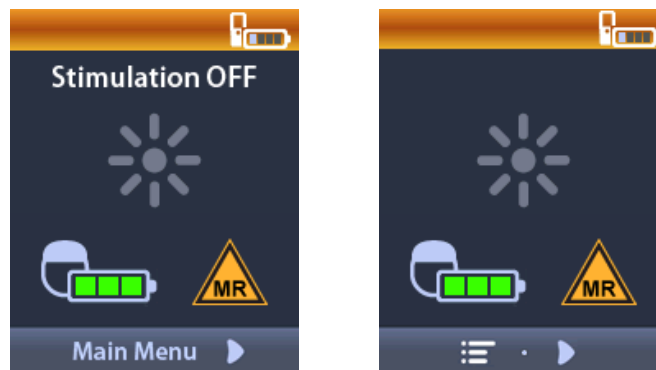Disabling MRI Mode


Upon completion of the MRI scan, the Remote Control must be used to disable the MRI Mode.

To disable MRI Mode:

1. Unlock the Remote Control by pressing the **Lock/Unlock**  button on the right side of the Remote Control.
2. After unlocking the Remote Control, the **Home** screen appears.


3. Press the **Right Arrow**  button to navigate to the **Main Menu** .
4. Select  **System Settings**.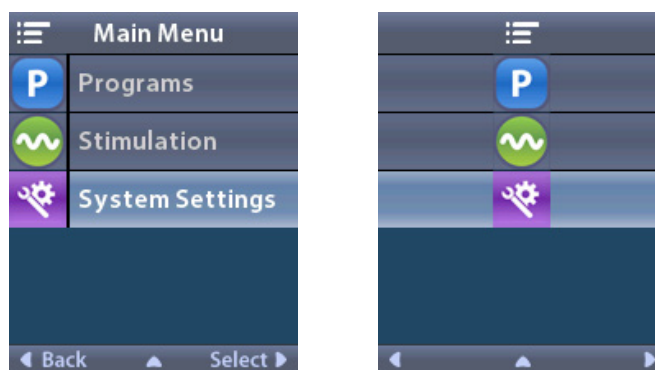

en


5. Select  **Exit MRI Mode**.

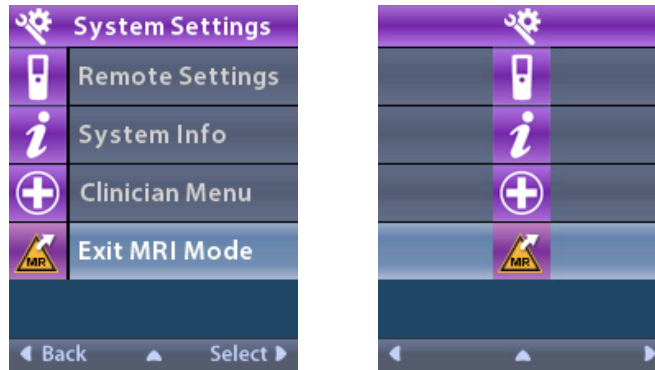
6. Select  **Yes** to Exit MRI Mode or  **No** to cancel the action.

7. The Stimulator performs a series of checks before disabling MRI Mode.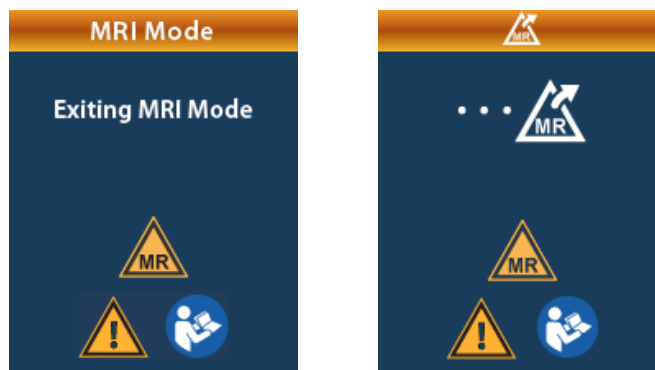

8. If MRI Mode is disabled, the **MRI Mode Disabled** confirmation screen is displayed.

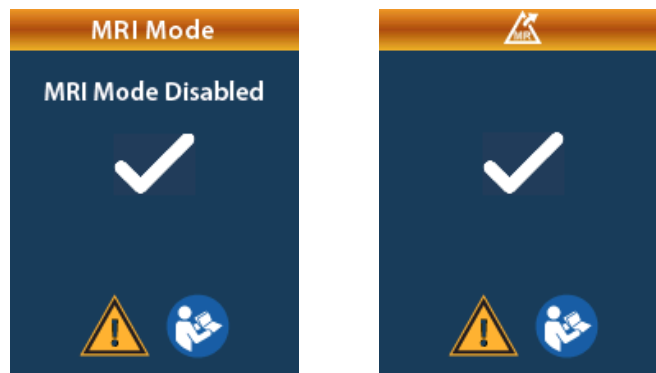Note: The Stimulator will retain the stimulation and program settings that were set before MRI Mode was enabled. If the stimulation was ON before MRI Mode was enabled, then disabling MRI Mode turns the stimulation back ON. If stimulation was OFF before MRI Mode was enabled, then disabling MRI Mode keeps the stimulation OFF.

9. The **Home** screen on the Remote Control will not display the **MR Conditional Symbol**  once MRI Mode is disabled.

MRI Mode Error Screens


The System performs a series of checks prior to entering MRI Mode. These checks are performed once  **Enter MRI Mode** is selected from the  **System Settings**. The Remote Control will display error screens if:

en

- The Stimulator battery is low.
- The Impedance check detects an anomaly.
- There is an error in the Stimulator.

Charge Stimulator Now Screen (Rechargeable Stimulators Only)

The Stimulator battery must be fully charged before MRI Mode is enabled. If the Stimulator battery is not fully charged, the Remote Control will display one of the following messages instructing the patient to charge the Stimulator before enabling MRI Mode.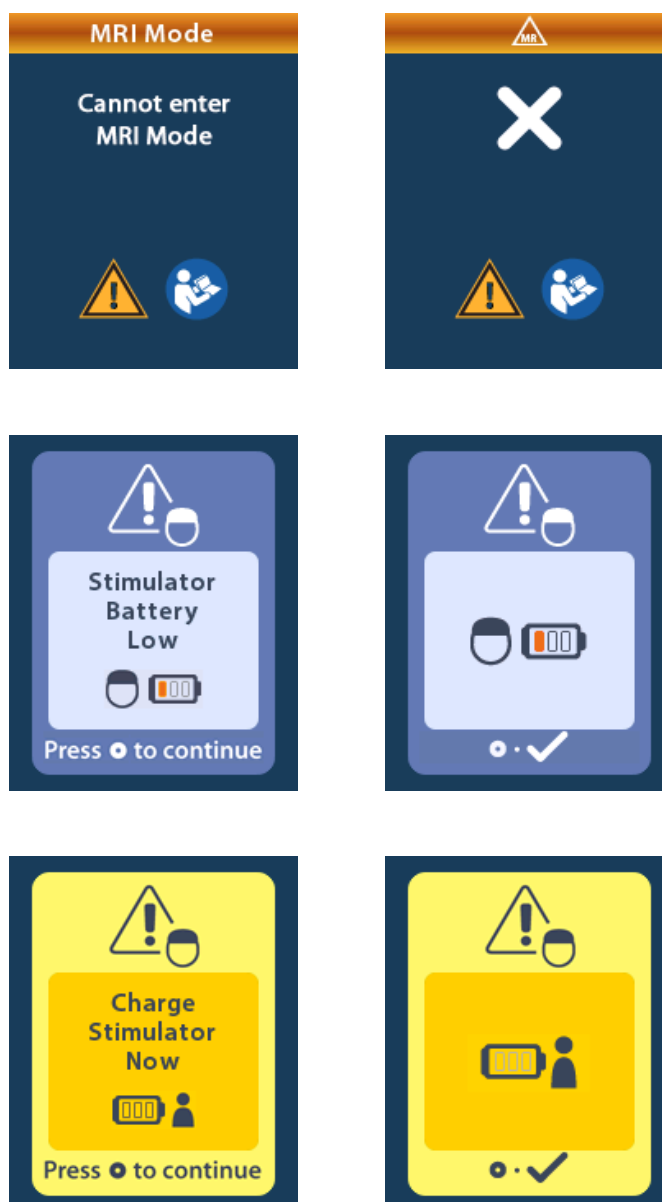


en

Warning: Always check the Stimulator battery to ensure that it is fully charged before performing a scan on the patient.

1. Press  to dismiss the error message and return to the Remote Control **Home** screen.
2. Instruct the patient to charge the Stimulator.
3. Enable MRI Mode once the Stimulator is fully charged.

Charge Stimulator Now or Disable MRI Mode Screen (Rechargeable Stimulators Only)

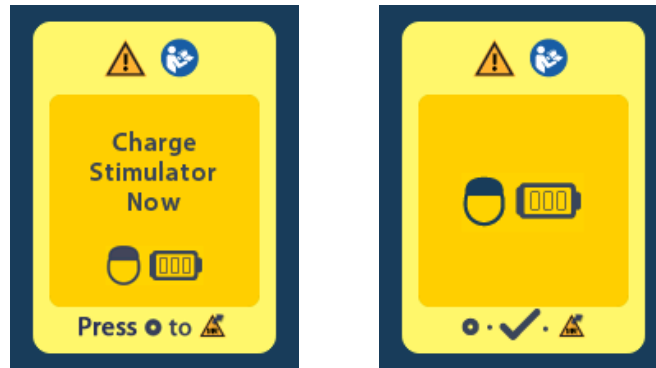If MRI Mode has already been enabled and the Stimulator battery power falls below the recommended value, the Remote Control will display a message instructing the patient to charge the Stimulator.

To charge the Stimulator without disabling MRI Mode:

1. Do not press .
2. Instruct the patient to charge the Stimulator.
3. Check the Remote Control to confirm that the error message has cleared.
4. Navigate to the **Home** screen on the Remote Control by pressing  on the side panel of the Remote Control and confirm that the **MR Conditional Symbol**  is displayed on the **Home** screen.

The patient can also disable the MRI Mode before charging the Stimulator:

1. Press  to disable the MRI Mode.
2. Instruct the patient to fully charge the Stimulator.
3. Check the Remote Control to confirm that the error message has cleared.
4. Enable MRI Mode by following instructions in the “*Enabling MRI Mode*” section of this manual.

Caution: *The Charger and Remote Control are MR Unsafe and must not be brought into the MRI scanner room.*

Stimulator Battery Low Screen Due to ERI or EOS (Non-Rechargeable Stimulators Only)

A Stimulator that has entered the Elective Replacement Indicator (ERI) or End of Service (EOS) period cannot be placed into MRI Mode. MRI Mode will not be enabled and the Remote Control will display “Cannot enter MRI Mode” and then “Stimulator Battery Low” messages.

Warning: Do not perform an MRI scan if MRI Mode is not enabled. Scanning under different conditions may result in patient injury or device malfunction.

ERI or EOS Screens During MRI Mode

If MRI Mode has already been enabled and the Stimulator battery power falls below the threshold, the Remote Control will display a message informing the patient that the Stimulator has entered the Elective Replacement Indicator (ERI) period or has reached End of Service (EOS) of the device.

The patient can disable MRI Mode:

en

1. Press  to disable the MRI Mode.

2. Check the Remote Control to confirm that the Stimulator battery error message still appears. See the “ERI (Elective Replacement Indicator) Screen (For Non-Rechargeable Stimulators Only)” section of this manual.

Caution: *The Remote Control is MR Unsafe and must not be brought into the MRI scanner room.*

Warning: *Do not perform an MRI scan if MRI Mode is not enabled. Scanning under different conditions may result in patient injury or device malfunction.*

Impedances Out Of Range Screen

The impedances must be within the acceptable range before MRI Mode is enabled. If the impedances are not within the acceptable range, the Remote Control will display an error message.


1. Press  to continue.
2. The Remote Control displays a new message instructing the user to review the MRI scan risks related to abnormal impedances. Review the "Impedance Out of Range" section in the *ImageReady™ MRI Guidelines* before proceeding. Press  to continue.

en

3. Select **Yes** to proceed with enabling MRI Mode or **No** to cancel the action.

en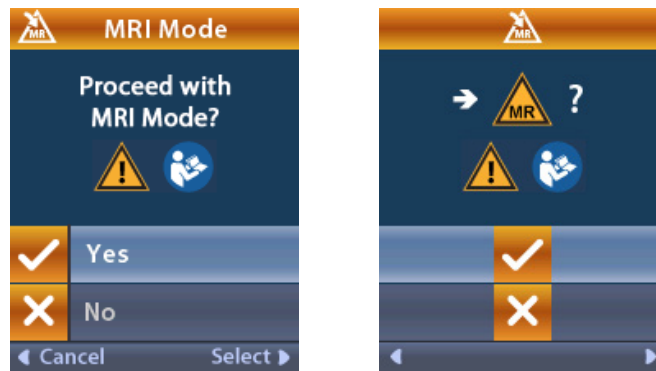

Warning: An MRI scan is not recommended when the impedances are not within the acceptable range. Higher or lower than normal impedances could indicate compromised Stimulator-Lead integrity. Scanning under these conditions may increase the risk of potential adverse effects that are listed in the “Safety Information” section of the ImageReady™ MRI Guidelines.

Stimulator Error Screen

If the system check fails due to a Stimulator error, MRI Mode will not be enabled and the Remote Control will display the **Stimulator Error** screen. This screen may be displayed when there is a communication error between the Remote Control and Stimulator. Press  to acknowledge the message and retry entering MRI Mode. If this error screen immediately displays again, do not perform an MRI scan and instruct the patient to contact the healthcare provider managing their DBS System or Boston Scientific Technical Support.




Stimulator Battery Messages

ERI (Elective Replacement Indicator) Screen (For Non-Rechargeable Stimulators Only)

This screen displays when the implanted non-rechargeable Stimulator is nearing end of service. Stimulation is provided during the ERI period. Changes made to the stimulation will not be saved, and stimulation will not be available soon. Batteries that have lasted one year or more without entering the ERI period will have a minimum of four weeks between entering the ERI period and reaching end of battery life. Patients should be advised to contact their healthcare provider to report this message screen. Press  to clear this information screen. This screen will appear whenever the Remote Control is activated.





en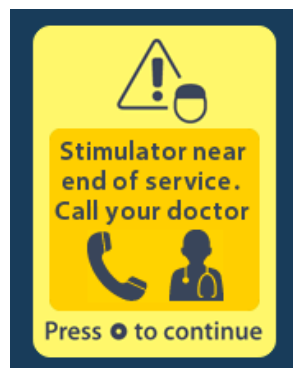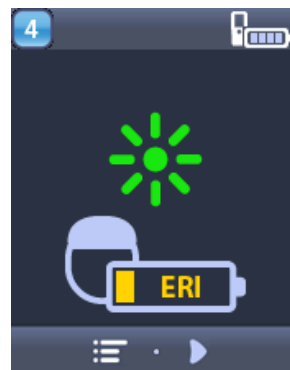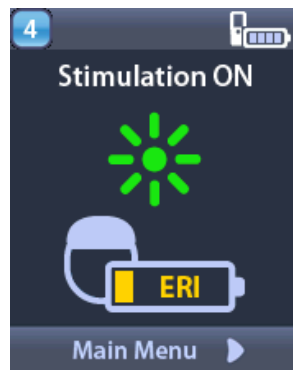

End of Service (EOS) (For Non-Rechargeable Stimulators Only)

This screen displays when the battery of the implanted non-rechargeable Stimulator is fully depleted. Stimulation is not available. Patients should be advised to call their healthcare provider when this message appears.

en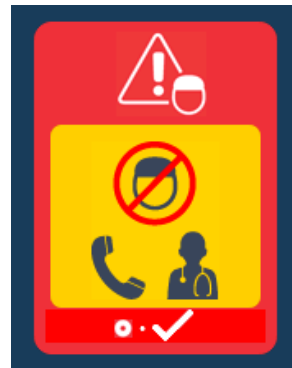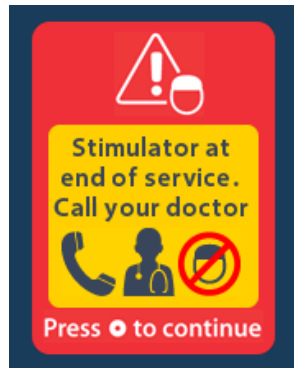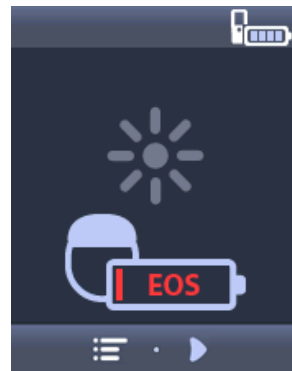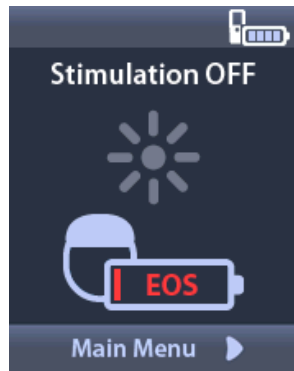

Stimulator Low Battery Screens (For Rechargeable Stimulators Only)

The Remote Control will display the following messages reminding the patient to recharge the Stimulator if the battery power is low.




en

ETS Low Battery Screens (For ETS 3 Only)

The Remote Control will display the following messages indicating that the ETS batteries need to be replaced.

en

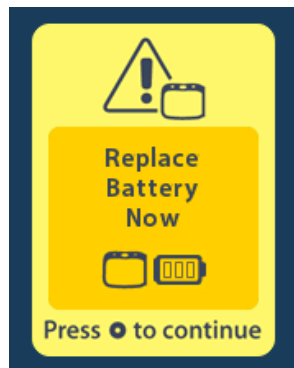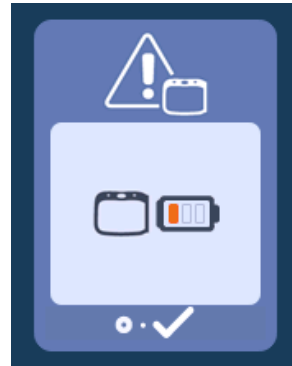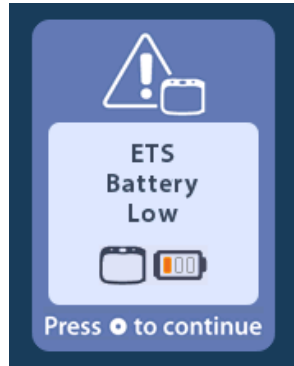
Troubleshooting

Bluetooth Help

Common sources of interference, such as Wi-Fi routers, cordless phones, Bluetooth wireless streaming devices, baby monitors, and microwave ovens, can affect wireless communication. You will continue to have access to the Remote Control menus even if the Remote Control is outside its effective communication range with the Stimulator. If you enter a command to change your settings or programs while the Remote Control is outside of its effective communication range, you will have several seconds to move the Remote Control within range of the Stimulator to send the command. After the command is received by the Stimulator, the Remote Control will make a double-beep then display the requested change on the screen. If the Remote Control is unable to communicate with the Stimulator, the Bluetooth Error message will appear:


en

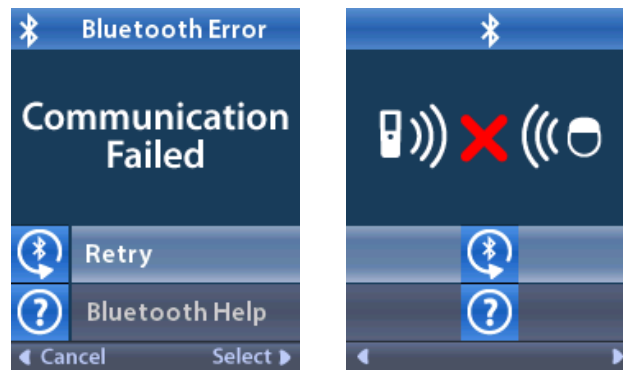To retry communication with the Stimulator – this resends the last command – select  **Retry**. To cancel communication and return to the screen, press .

Vercise™ DBS Remote Control 4 Directions for Use

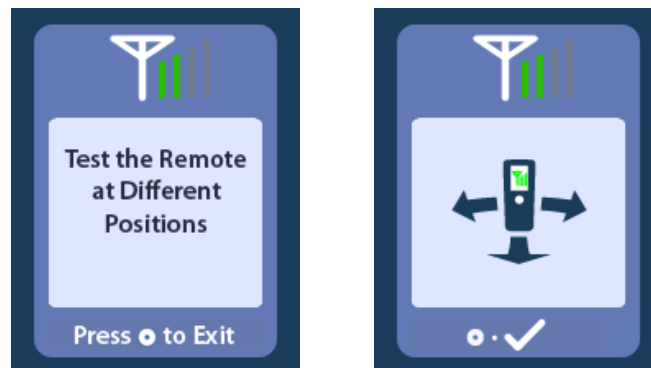
The Bluetooth Help tool is also available to assist you in improving communication between the Remote Control and the Stimulator. To use the Bluetooth Help tool, select  **Bluetooth Help** . The following screen will be displayed:

Once in the  **Bluetooth Help** screen, the Remote Control will automatically scan for the connected Stimulator. Once communication is established, the telemetry bars will indicate the strength of the wireless communication. The number of bars will change depending on the distance and orientation of the Remote Control from the Stimulator. This tool will continue to measure the strength of communication for 60 seconds or until  is pressed.


No Program to Run Screen

This screen is displayed when the Stimulator has no programs to run.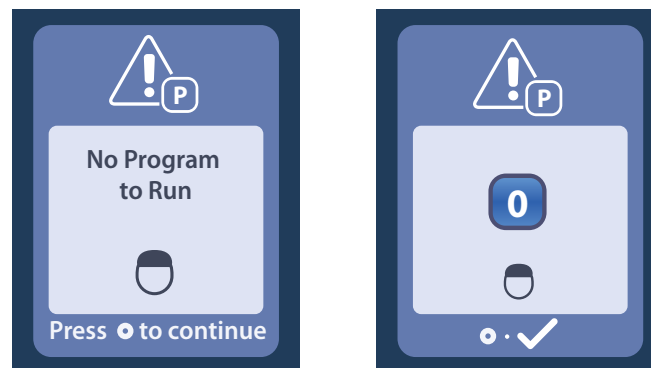

Invalid Program Screen


This screen is displayed when the Stimulator attempts to run an invalid program.

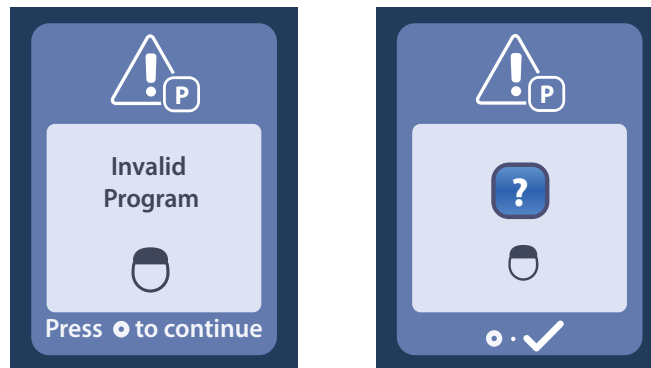en

Unsupported Stimulator Screen


This screen is displayed when the Remote Control detects an incompatible Stimulator.

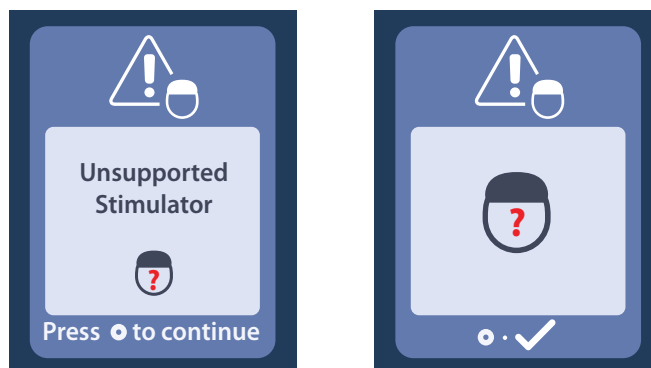System Settings Reset Screen


This screen is displayed when the Remote Control experiences an error and a reset of the Remote Control is required. Press  to continue. Programs will not be deleted, but all system settings (for example, language and volume settings) will be restored to the default factory settings.

en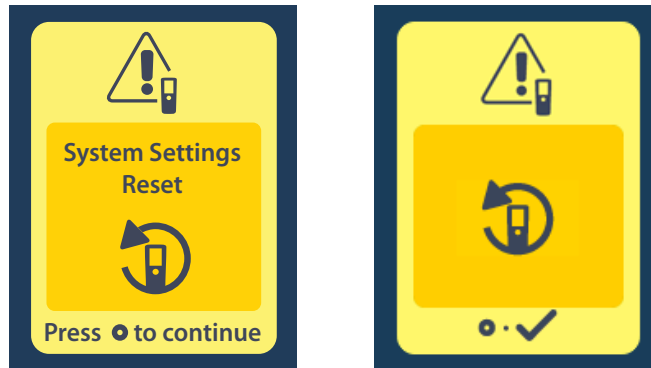


Stimulator Error Screen

This screen is displayed when there is a non-critical Stimulator error. This screen may also be displayed when there is a communication error between the Remote Control and Stimulator. Press  to acknowledge the message and continue. If desired, retry the action that was interrupted by the error.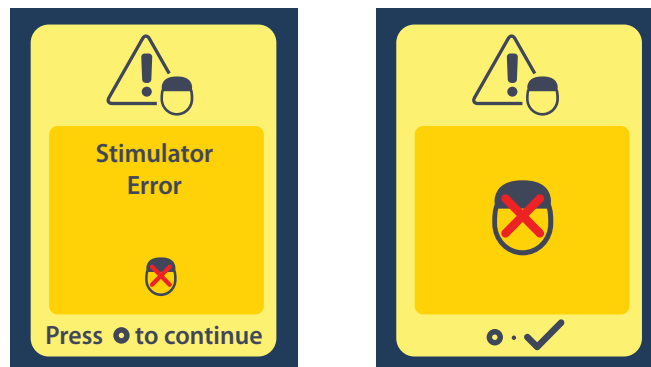

Remote Error Screen

This screen is displayed when there is a critical error on the Remote Control, which makes the Remote Control unusable. Please contact Boston Scientific Technical Support to order a replacement Remote Control.

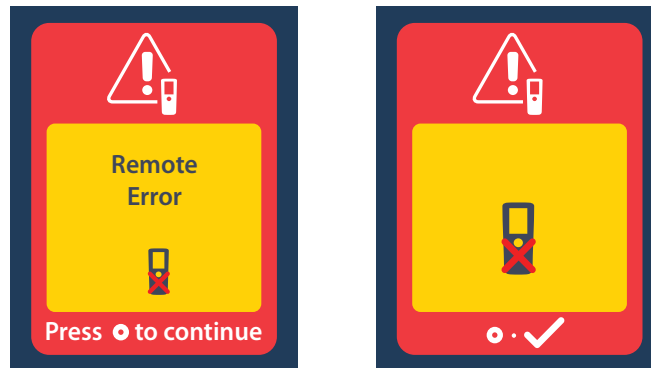en

Icon Description

en

A

Alert Volume

B

Bluetooth Help

C

Change Program

Clear Link

Clinician Menu

CP Mode: Stimulator

E

Exit Clinician Mode

Enter MRI Mode

Exit MRI Mode

External Trial Stimulator

F

Follow Instructions for Use

H

Home

I

Impedances

Invalid Password

L

Language Setting

Link

Lock/Unlock

M

Main Menu

Minus Button

MR Conditional

N

Navigation

No

P

Password

Program

Plus Button

R

-  Remote
-  Remote Battery
-  Remote Information
-  Remote Settings
-  Rescan
-  Reset Stimulator
-  Restore to Clinic Settings
-  Retry

S

-  Screen Brightness
-  Stimulation Area
-  Stimulation Button
-  Stimulation OFF
-  Stimulation ON
-  Stimulator
-  Stimulator Battery
-  Stimulator Information
-  Stimulator Search
-  System Information
-  System Settings

Y

-  Yes

en

Technical Support

Boston Scientific Corporation has highly trained service professionals to assist you. The Technical Support Department is available to provide technical consultation 24 hours a day.

In the US, please call (866) 360-4747 to speak to a representative. Outside the US, please choose your locality from the following list:

en

Argentina

T: +5411 4896 8556 F: +5411 4896 8550

Australia / New Zealand

T: 1800 676 133 F: 1800 836 666

Austria

T: +43 1 60 810 F: +43 1 60 810 60

Balkans

T: 0030 210 95 37 890 F: 0030 210 95 79 836

Belgium

T: 080094 494 F: 080093 343

Brazil

T: +55 11 5853 2244 F: +55 11 5853 2663

Bulgaria

T: +359 2 986 50 48 F: +359 2 986 57 09

Canada

T: +1 888 359 9691 F: +1 888 575 7396

Chile

T: +562 445 4904 F: +562 445 4915

China – Beijing

T: +86 10 8525 1588 F: +86 10 8525 1566

China – Guangzhou

T: +86 20 8767 9791 F: +86 20 8767 9789

China – Shanghai

T: +86 21 6391 5600 F: +86 21 6391 5100

Colombia

T: +57 1 629 5045 F: +57 1 629 5082

Czech Republic

T: +420 2 3536 2911 F: +420 2 3536 4334

Denmark

T: 80 30 80 02 F: 80 30 80 05

Finland

T: 020 762 88 82 F: 020 762 88 83

France

T: +33 (0) 1 39 30 97 00 F: +33 (0) 1 39 30 97 99

Germany

T: 0800 072 3301 F: 0800 072 3319

Greece

T: +30 210 95 42401 F: +30 210 95 42420

Hong Kong

T: +852 2960 7100 F: +852 2563 5276

Hungary

T: +36 1 456 30 40 F: +36 1 456 30 41

India – Bangalore

T: +91 80 5112 1104/5 F: +91 80 5112 1106

India – Chennai

T: +91 44 2648 0318 F: +91 44 2641 4695

India – Delhi

T: +91 11 2618 0445/6 F: +91 11 2618 1024

India – Mumbai

T: +91 22 5677 8844 F: +91 22 2617 2783

Italy

T: +39 010 60 60 1 F: +39 010 60 60 200

Korea

T: +82 2 3476 2121 F: +82 2 3476 1776

Malaysia

T: +60 3 7957 4266 F: +60 3 7957 4866

Mexico

T: +52 55 5687 63 90 F: +52 55 5687 62 28

Middle East / Gulf / North Africa

T: +961 1 805 282 F: +961 1 805 445

The Netherlands

T: +31 30 602 5555 F: +31 30 602 5560

Norway

T: 800 104 04 F: 800 101 90

Philippines

T: +63 2 687 3239 F: +63 2 687 3047

Poland

T: +48 22 435 1414 F: +48 22 435 1410

Portugal

T: +351 21 3801243 F: +351 21 3801240

Singapore

T: +65 6418 8888 F: +65 6418 8899

South Africa

T: +27 11 840 8600 F: +27 11 463 6077

Spain

T: +34 901 11 12 15 F: +34 902 26 78 66

Sweden

T: 020 65 25 30 F: 020 55 25 35

Switzerland

T: 0800 826 786 F: 0800 826 787

Taiwan

T: +886 2 2747 7278 F: +886 2 2747 7270

Thailand

T: +66 2 2654 3810 F: +66 2 2654 3818

Turkey – Istanbul

T: +90 216 464 3666 F: +90 216 464 3677

Uruguay

T: +59 82 900 6212 F: +59 82 900 6212

UK & Eire

T: +44 844 800 4512 F: +44 844 800 4513

Venezuela

T: +58 212 959 8106 F: +58 212 959 5328

Note: *Phone numbers and fax numbers may change. For the most current contact information, please refer to our website at <http://www.bostonscientific-international.com/> or write to the following address:*

Boston Scientific Neuromodulation Corporation

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Boston Scientific (Australia) Pty Ltd
Boston Scientific Limited