




# 1 Remote Control Buttons



# Home Screen Display

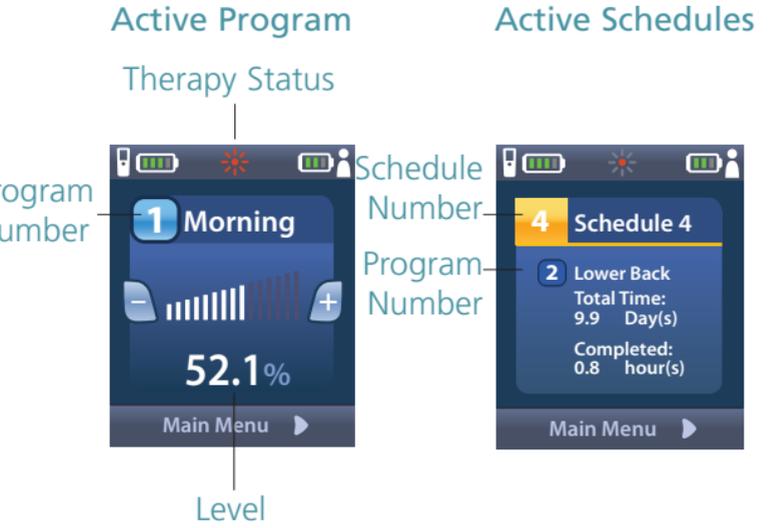

Note: You will see the icon for the Stimulator battery only if you have a rechargeable battery.

Note: The Home Screen Displayed is dependent on the therapy prescribed by your physician.

# 2 Getting Started



**1** Unlock/ Lock

**2** Start/Stop Therapy

**3** Decrease Increase Stimulation Strength

**4** Main menu

The settings displayed are dependent on your Stimulator and the therapy configured by your healthcare provider.

# 3 Accessing Programs

If your healthcare provider has configured Programs that can be changed, there are two ways to view the Programs menu:



- Press the Programs button on the left side of the Remote
- From the Main Menu , select the Programs menu and press

# 4 Changing Programs



**1** From the Programs Menu press to enter the Change Program menu

**2** Press to select the desired Program

**3** Press to activate the selected Program

My Healthcare Provider

Name

Phone

Hospital