**Boston
Scientific**
Advancing science for life™

Vercise™ DBS
Charging Handbook
Charging System with Belt

92357527-06

Content: MP92357527-06 REV A

en
fr
ru
ar
he

Trademarks

All trademarks are the property of their respective holders.

Guarantees

Boston Scientific Corporation reserves the right to modify, without prior notice, information relating to its products in order to improve their reliability or operating capacity.

Drawings are for illustration purposes only.

Additional Information

Refer to the appropriate *Information for Patients DFU* and *Remote Control Handbook* for additional instructions and information about your DBS System. Refer to the *Labeling Symbols* document for an explanation of labeling symbols.

References to the Charging System or charging process are applicable only when using a rechargeable DBS Stimulator.

Contacting Boston Scientific

To contact Boston Scientific, see the “*Technical Support*” section of this manual.

Product Model Numbers

Model Number	Product Name
DB-64126	Vercise Charging Belt System
DB-6358-0BK	Vercise Charging Belt, Small
DB-6358-1BK	Vercise Charging Belt, Medium
DB-6358-5BK	Vercise Charging Belt Extension
DB-6358-9BK	Vercise Charging Belt Loop Replacement
NM-6350	Adhesive Kit
NM-5312	Charger
NM-5305	Base Station
NM-6305	Power Supply & Adapter For Charger (EU)

Table of Contents

Introduction.....1

Using the Charging System.....2

 Getting Set Up.....4

 Charging the Charger.....5

 Understanding the Charger Sound Alerts.....7

 Charging Your Stimulator.....8

 Using the Charging Belt.....9

 Using the Adhesive Patch.....14

 Properly Aligning the Charger.....15

 Tips for Properly Aligning the Charger.....17

 Cleaning Instructions.....19

Technical Support.....20

This page intentionally left blank.

Introduction

The Boston Scientific Deep Brain Stimulation (DBS) System provides a reversible therapy that stimulates structures in the brain with small electrical pulses. The Boston Scientific DBS System includes a rechargeable Stimulator that requires periodic recharging. The Charging System allows you to recharge the Stimulator as needed.

The Charging System for the Stimulator consists of the Charger, a Base Station, and a Power Supply. The Charging System also includes a Charging Belt that holds the Charger over the Stimulator.

This manual provides you with instructions for charging your Stimulator. We encourage you to read this and your other Boston Scientific manuals entirely. If you have any questions, or need clarification on anything contained in this manual, contact Technical Support.

Using the Charging System

You should expect a daily recharging time of 5 to 30 minutes per day or a periodic recharging time of 30 minutes to 4 hours every 1 to 2 weeks. Your recharge routine may vary depending on your stimulation settings. Some stimulation settings will require more frequent charging. Boston Scientific recommends any recharge routine that fits your schedule and lifestyle while maintaining sufficient charge to maintain stimulation.

Developing a recharge routine involves finding the right balance between four factors:

- How much power is required to experience effective therapy
- How often you want to recharge
- How long you want to recharge
- How you would like to manage your personal schedule

The Boston Scientific DBS System's programming software gives your healthcare provider a conservative recommendation for how often to charge. This estimate assumes stimulation is on 24 hours per day, 7 days per week at the programmed energy output level. While you may want to follow these recommendations, you and your physician can also develop an appropriate charge routine that best fits your schedule.

The Remote Control provides an easy-to-view Stimulator battery meter on the Home screen and messages to inform you of the battery's condition. When the Remote Control indicates a low battery, the Stimulator should be recharged as soon as possible. Failure to recharge may result in loss of stimulation. After stimulation stops, communication with the Stimulator will also stop until the Stimulator is sufficiently charged. Refer to your *Remote Control Handbook* for more information.

The rechargeable Stimulator battery should provide at least five years of service. In many cases, the Stimulator battery should provide at least 25 years of service. Battery life is dependent on the stimulation settings and conditions.

Over time and with repeated charging, the battery in the Stimulator will lose its ability to recover its full capacity. As a result, the Stimulator may require shorter intervals between charges. The Stimulator will need replacement when stimulation can no longer be maintained with a routine charging.

Getting Set Up

Check that you have received all the components included in your DB-64126 Vercise Charging Belt System (see Table 1). If any item is missing, contact Technical Support for a replacement.

Table 1: Components of the Vercise Charging Belt System

	
Label	Description
A	Charging Belt (1)
B	Base Station (1)
C	Charger (1)
D	Power Supply (1) and Adapter (1)

Charging the Charger

The Charger contains a rechargeable battery that is used to recharge your Stimulator's battery (see Table 2).


Cautions:

- Keep the Charging System dry. It should not be exposed to moisture.
- Do not connect the Charger to any other device except to the Base Station provided with it.
- Keep the Charging System away from pets, pests, and children to avoid damage to the device.

To charge the Charger:

1. Find a convenient electrical outlet that will allow you to place the Base Station on a clear, flat surface and will not expose the Charging System to water or direct heat.
2. Plug the Power Supply into a standard AC wall outlet.
3. Connect the Power Supply to the Base Station.
4. Place the Charger in the Base Station with the power button facing up.

Table 2: Vercise Charger System Components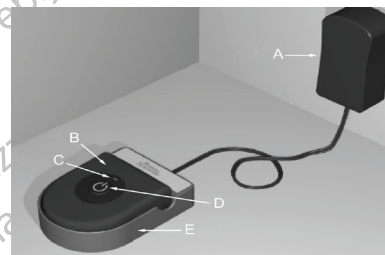

Label	Description
A	Power Supply
B	Charger
C	Indicator Light
D	Power Button
E	Base Station

Note: The Base Station is designed to remain connected to a power outlet at all times. When it is not being used, keep the Charger on the Base Station so that it is always ready to deliver a charge.

The Charger is ready to fully charge your Stimulator when the Charger's indicator light is green. If the indicator light is yellow, the Charger can only partially charge the Stimulator. While the Charger may be used when the indicator light is yellow, it may not be able to fully charge your Stimulator and you may need to charge sooner than usual.

Charger Indicator Light	Description
Green	The Charger is fully charged and ready to charge the Stimulator.
Yellow	The Charger is partially charged. The Charger may still be used, but it may not be able to fully charge the Stimulator. To fully charge the Charger, place it in the Base Station.
Off	The Charger is off – or – the Charger battery is empty. If pressing the Power Button does not turn on the Charger, place the Charger in the Base Station to charge.

Charger Battery Service Life

The Charger battery has a typical service life of 500 charging cycles.

Understanding the Charger Sound Alerts

The Charger emits different sounds to alert you of specific situations. Before you begin charging your Stimulator, it is important to understand these sounds. The following chart describes what each sound means.

Sound	Description
	Repeated Single Beep The Charger is searching for your Stimulator and is not properly aligned with the Stimulator. Align the Charger as shown in the “<i>Properly Aligning the Charger</i>” section of this manual.
	Repeated Double Beep The Charger has fully charged the Stimulator battery. Turn off the Charger and return it to the Base Station.

Charging Your Stimulator

When charging, you must use the Charger with either the Charging Belt or an Adhesive Patch to hold the Charger over the Stimulator. The Adhesive Patches are made of non-reactive and natural Latex-free material suitable for most sensitive skin types.

WARNINGS:

- Do not charge while sleeping. This may result in a burn.
- While charging, the Charger may become warm. It should be handled with care.
- Failure to use either the Charging Belt or an Adhesive Patch while charging may result in a burn. If pain or discomfort is felt, stop charging and contact your health care provider.

Using the Charging Belt

There are two lengths of the Charging Belt available (Small and Medium). If you need to order a Charging Belt Extension, a different size Charging Belt, a Charger Spacer, or a replacement Belt Loop, contact Technical Support.

Note: A Charging Belt Extension is available if the Charging Belt is too small. If using the extension, insert one strap through the loop on the extension and fasten the strap onto the belt (Figure 2).

1. When the indicator light is green, remove the Charger from the Base Station.

Note: Once the Charger is removed from the Base Station, the Charger indicator light will turn off regardless of the ready status of the Charger.

2. Place the Charger in the pouch of the Charging Belt so that the power button is visible through the mesh fabric (Figure 1).


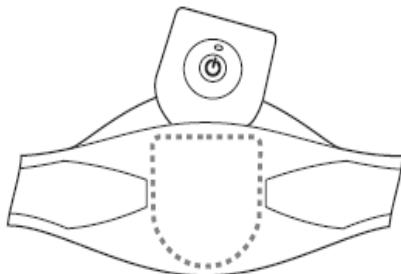
Figure 1. Placing the Charger in the Pouch

3. Close the pouch.
4. Place the Charging Belt around your body so that the power button and mesh fabric are facing out (Figure 1).

5. To set the length of the Charging Belt, place the Charging Belt around your body and insert each strap through the Belt Loop. Affix the straps using fasteners. Each strap is adjustable. See Figure 2 and Table 3 for reference.

Table 3: Charging Belt Key Features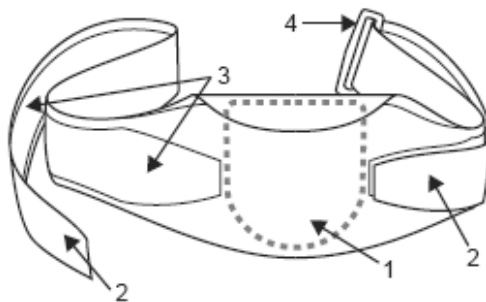

Label	Description
1	Pouch for Charger
2	Adjustable Straps
3	Hook-and-Loop Fasteners
4	Belt Loop
Note: In this image, the right strap is affixed and the left strap is not affixed.	

Affix the adjustable straps so that the Belt is slightly loose at first. In order to properly use the Charging Belt, both of the straps must go through the Belt Loop (Figure 2).

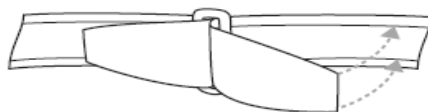
Figure 2. Adjustable Strap Placed Through Belt Loop and Affixed Using Fasteners

6. Position the Charger over the Stimulator.

When the Charger is aligned over the Stimulator, the beeping will stop. For the best charging results, make sure the Charger is well ventilated and centered over the Stimulator (Figure 3). See the “*Properly Aligning the Charger*” and “*Tips for Properly Aligning the Charger*” sections of this manual to ensure you center the Charger over the Stimulator.

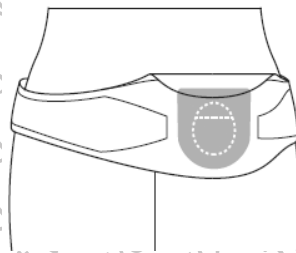
Figure 3. Ideal Alignment of Charger (Outlined) Over Stimulator (Dotted Line)

7. Secure the Charger over the Stimulator by securing the Charging Belt.

Be careful to maintain the Charger's alignment as you tighten the Belt. Take care not to overtighten the Belt, which may result in discomfort or accidental movement of the Charger.

Note: Wearing the Charging Belt over loose fitting clothes may result in more frequent movement of the Charger. For the best results, occasionally check the Charger to make sure that it is centered over the Stimulator. If the Charger is not centered over the Stimulator, charging time may increase. If the Charger significantly moves out of alignment, it will start beeping again (Figure 4).

Note: The Charging Belt can be placed under or over clothing. You should not wear tight fitting or heavy clothing over the Charger while charging to allow air flow around the Charger.

Note: Do not use the Charging Belt when it is wet or damp.

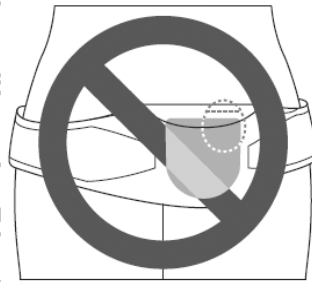
Figure 4. Poor Alignment of Charger (Outlined in Grey) Over the Stimulator (Dotted Line)

Using the Adhesive Patch

1. When the indicator light is green, remove the Charger from the Base Station.

Note: *Once the Charger is removed from the Base Station, the Charger indicator light will turn off regardless of the ready status of the Charger.*

2. Apply the Adhesive Patch to the backside of the Charger by peeling the clear liner from the patch and applying the white side with the blue stripe to the rear of the Charger (Figure 5).
3. Remove the skin side beige liner from the adhesive (only good for one fixation).
4. For best charging results, make sure the Charger is centered over the Stimulator. If the Charger is not centered, charging time may increase. See the “*Properly Aligning the Charger*” and “*Tips for Properly Aligning the Charger*” sections of this manual to ensure you center the Charger over the Stimulator. It is recommended that you follow the instructions in both sections the first time you charge your Stimulator.

Note: *Do not put the Charger directly on the skin (e.g., without an Adhesive Patch).*

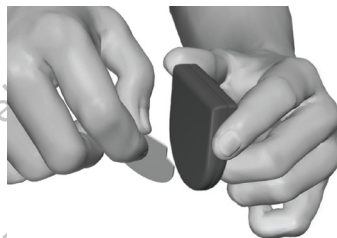
Figure 5. Peel the Clear Liner from the Adhesive Patch

Properly Aligning the Charger

1. Press the power button. The indicator light will come on again, and the Charger will begin beeping repeatedly to signal that it is searching for the Stimulator.
2. Position the Charger over the Stimulator (Figure 6).

The beeping will stop when the Charger is partially aligned with the Stimulator. For optimal charging, make sure the Charger is centered over the Stimulator.

Charging time may increase if the Charger is not fully aligned with the Stimulator. To fully align the Charger over the Stimulator, position the bottom curve of the Charger approximately 1 cm lower than the bottom curve of the Stimulator.

If you are having trouble properly aligning the Charger or experiencing longer than normal charging times, see the “*Tips for Properly Aligning the Charger*” section of this manual.

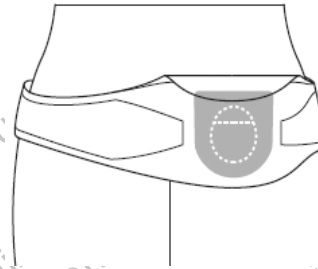
Figure 6. Position the Charger Over the Stimulator

3. Secure the Charger over the Stimulator by aligning the Charging Belt, or by pressing the adhesive to the skin over the Stimulator.

Note: *If the Charging Belt moves out of alignment, or if the Adhesive Patch is accidentally located in the wrong place, the Charger will start beeping again. Realign the Charging Belt, or use a new Adhesive Patch to place the Charger back into the fully aligned position.*

Note: *Moving around while charging is acceptable, but be aware that the Charger must stay centered over the Stimulator for proper charging. Excessive movement may cause the Charger and Stimulator to become misaligned.*

4. When the Charger emits a distinct double beep, press the power button to turn off the Charger and check the Stimulator's battery with the Remote Control.
5. If the Stimulator is fully charged, remove the Charging Belt or Adhesive Patch from your body.

Note: *Do not confuse the end of charge signal (a distinct double beep) with repeated beeping indicating misalignment.*

6. Separate the Charger from the Charging Belt or Adhesive Patch.

Set aside the Charging Belt or discard the Adhesive Patch.

7. Return the Charger to the Base Station.

Tips for Properly Aligning the Charger

If you are having trouble properly aligning the Charger or are experiencing longer than normal charging times, you can use the following procedure to help you properly align the Charger.

1. Lightly press on your skin until you find the implanted Stimulator.
2. Press the Power Button on the Charger. The Charger will begin beeping repeatedly to signal that it is searching for the Stimulator.
3. Place the Charger to the left of the Stimulator.
4. Slowly move the Charger to the right, towards the Stimulator.
5. Stop moving the Charger when it stops beeping.
6. Make a mark on your skin with a soft-tipped marker at the right edge of the Charger (Figure 7).
7. Place the Charger to the right of the Stimulator.
8. Slowly move the Charger to the left, towards the Stimulator.
9. Stop moving the Charger when it stops beeping.
10. Make a mark on your skin at the left edge of the Charger (Figure 8).

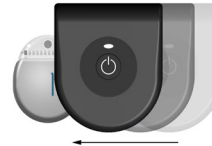
Figure 7. Right Edge Alignment of Charger

Figure 8. Left Edge Alignment of Charger

11. Place the Charger above the Stimulator.
12. Slowly move the Charger down, towards the Stimulator.
13. Stop moving the Charger when it stops beeping.
14. Make a mark on your skin at the bottom edge of the Charger (Figure 9).
15. Place the Charger below the Stimulator.
16. Slowly move the Charger up, towards the Stimulator.
17. Stop moving the Charger when it stops beeping.
18. Make a mark on your skin at the top edge of the Charger (Figure 10).

The marks on your skin should create a square or crosshair. The center of this crosshair indicates the center of the Stimulator.

19. Now that you have located the center of the Stimulator, properly align the Charger over the Stimulator (Figure 11).

Figure 9. Top Edge Alignment of Charger

Figure 10. Bottom Edge Alignment of Charger

Figure 11. Proper Alignment of Charger

Cleaning Instructions

Cleaning the Charger, Base Station, and Power Supply. The components can be cleaned using alcohol or a mild detergent applied with a cloth or tissue. Residue from soapy detergents should be removed with a damp cloth. Do not use abrasive cleansers for cleaning. Do not clean the Charger, Base Station, or Power Supply while they are connected to a power outlet.

Cleaning the Charging Belt. The Charging Belt can be hand washed, air dried, and stored until the next use. Be sure to remove the Charger from the Belt pouch before washing the Charging Belt. Washing instructions for the Charging Belt can be found on the tag in the Belt pouch.

Technical Support

There are no user serviceable parts. If you have a specific question or issue, contact your healthcare provider. To contact Boston Scientific for any other reason, use the contact information provided for your locality in the list below:

Argentina

T: +5411 4896 8556 F: +5411 4896 8550

Australia / New Zealand

T: 1800 676 133 F: 1800 836 666

Austria

T: +43 1 60 810 F: +43 1 60 810 60

Balkans

T: 0030 210 95 37 890 F: 0030 210 95 79 836

Belgium

T: 080094 494 F: 080093 343

Brazil

T: +55 11 5853 2244 F: +55 11 5853 2663

Bulgaria

T: +359 2 986 50 48 F: +359 2 986 57 09

Canada

T: +1 888 359 9691 F: +1 888 575 7396

Chile

T: +562 445 4904 F: +562 445 4915

China – Beijing

T: +86 10 8525 1588 F: +86 10 8525 1566

China – Guangzhou

T: +86 20 8767 9791 F: +86 20 8767 9789

China – Shanghai

T: +86 21 6391 5600 F: +86 21 6391 5100

Colombia

T: +57 1 629 5045 F: +57 1 629 5082

Czech Republic

T: +420 2 3536 2911 F: +420 2 3536 4334

Denmark

T: 80 30 80 02 F: 80 30 80 05

Finland

T: 020 762 88 82 F: 020 762 88 83

France

T: +33 (0) 1 39 30 97 00 F: +33 (0) 1 39 30 97 99

Germany

T: 0800 072 3301 F: 0800 072 3319

Greece

T: +30 210 95 42401 F: +30 210 95 42420

Hong Kong

T: +852 2960 7100 F: +852 2563 5276

Hungary

T: +36 1 456 30 40 F: +36 1 456 30 41

India – Bangalore

T: +91 80 5112 1104/5 F: +91 80 5112 1106

India – Chennai

T: +91 44 2648 0318 F: +91 44 2641 4695

India – Delhi

T: +91 11 2618 0445/6 F: +91 11 2618 1024

India – Mumbai

T: +91 22 5677 8844 F: +91 22 2617 2783

Italy

T: +39 010 60 60 1 F: +39 010 60 60 200

Korea

T: +82 2 3476 2121 F: +82 2 3476 1776

Malaysia

T: +60 3 7957 4266 F: +60 3 7957 4866

Mexico

T: +52 55 5687 63 90 F: +52 55 5687 62 28

Middle East / Gulf / North Africa

T: +961 1 805 282 F: +961 1 805 445

The Netherlands

T: +31 30 602 5555 F: +31 30 602 5560

Norway

T: 800 104 04 F: 800 101 90

Philippines

T: +63 2 687 3239 F: +63 2 687 3047

Poland

T: +48 22 435 1414 F: +48 22 435 1410

Portugal

T: +351 21 3801243 F: +351 21 3801240

Singapore

T: +65 6418 8888 F: +65 6418 8899

South Africa

T: +27 11 840 8600 F: +27 11 463 6077

Spain

T: +34 901 11 12 15 F: +34 902 26 78 66

Sweden

T: 020 65 25 30 F: 020 55 25 35

Switzerland

T: 0800 826 786 F: 0800 826 787

Taiwan

T: +886 2 2747 7278 F: +886 2 2747 7270

Thailand

T: +66 2 2654 3810 F: +66 2 2654 3818

Turkey – Istanbul

T: +90 216 464 3666 F: +90 216 464 3677

Uruguay

T: +59 82 900 6212 F: +59 82 900 6212

UK & Eire

T: +44 844 800 4512 F: +44 844 800 4513

Venezuela

T: +58 212 959 8106 F: +58 212 959 5328

Note: Phone numbers and fax numbers may change. For the most current contact information, please refer to our website at <http://www.bostonscientific-international.com/> or write to the following address:

Boston Scientific Neuromodulation Corporation

25155 Rye Canyon Loop
Valencia, CA 91355 USA

This page intentionally left blank.

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or its affiliates. All rights reserved.
Boston Scientific (Australia) Pty Ltd
Boston Scientific Limited
© 2020 Boston Scientific Corporation