**Boston
Scientific**

Advancing science for life™

Vercise™

Deep Brain Stimulation Systems

Remote Control 3 Handbook

92753346-06

Content: MP92753346-06 REV A

- en
- fr
- ru
- ar
- he
- uk

How to Use this Manual

This manual provides information about the Vercise™ Deep Brain Stimulation (DBS) System Remote Control (RC). Read this manual carefully before using the Vercise DBS System. References to the Charging System or charging processes are applicable only when using a rechargeable DBS Stimulator. If you have any questions or concerns about your DBS System, contact your healthcare provider.

Note: *Images of the Remote Control Home screen within this manual are representative of those displayed for a rechargeable system unless specified otherwise. The Home screen for non-rechargeable Stimulators does not include the battery level of the Stimulator.*

More Information

Refer to your *Information for Patients* manual for more information on your Deep Brain Stimulation System, such as:

- Indications for use
- Contraindications
- Warnings
- Precautions

- Adverse events
- Storage and handling
- Device disposal

Refer to your *Charging Handbook* for charging instructions for rechargeable Stimulators.

Refer to your *Labeling Symbols* document for an explanation of labeling symbols.

More information may be available online, visit www.bostonscientific.com/patientlabeling.

Guarantees

Boston Scientific Corporation reserves the right to modify, without prior notice, information relating to its products in order to improve their reliability or operating capacity.

Drawings are for illustration purposes only.

Trademarks

Vercise™, Vercise Gevia™, and ImageReady™ are trademarks of Boston Scientific Corporation or its affiliates.

All other trademarks are the property of their respective owners.

Warranty

For device warranty information, visit www.bostonscientific.com/warranty.

Summary of Safety and Clinical Performance

For customers in the European Union: visit <https://ec.europa.eu/tools/eudamed> to view a summary of this device's safety and clinical performance information on the EUDAMED website.

Technical Support

There are no user serviceable parts. If you have a specific question or issue, contact your healthcare provider. To contact Boston Scientific for any other reason, use the contact information provided for your locality via www.bostonscientific.com.

Product Model Numbers

Model Number	Description	Compatible System
DB-5552-1A	Vercise™ Remote Control Kit, RC 3	<ul style="list-style-type: none"> • Vercise Gevia™ Deep Brain Stimulation System • Vercise™ PC Deep Brain Stimulation System
DB-5250	Vercise™ Remote Control, RC 3	
NM-6210	Remote Holster	
DB-6315, NM-6315	Remote Power Supply, Universal	
NM-6600	Travel Case	

Table of Contents

Introduction	8
Using the Remote Control	13
How To Use Programs	21
About System Settings	31
About MRI Mode	36
Reminders and Messages	47
Troubleshooting	55

This manual is color-coded to help you use the Remote Control while referencing the manual.

Each section is labeled and color-coded

INTRODUCTION

Introduction

CAUTIONS

- Keep the Remote Control dry. It should not be exposed to moisture.
- Do not connect the Remote Control USB port to any other device except to the Remote Control charger provided with it.
- Keep the Remote Control away from pets, pests, and children to avoid damage to the device.

The Remote Control is a hand-held device that allows you to communicate with your Stimulator. Typically, you will use the Remote Control to turn stimulation ON or OFF.

Your Remote Control will display important messages about your Stimulator and Remote Control battery status. You should check your Remote Control at least once per month. The Remote Control will not communicate with an implanted rechargeable Stimulator during charging.

Keep your Remote Control with you at all times. Be sure to bring your Remote Control with you to appointments with your healthcare provider, including visits to program your Stimulator.

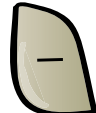
Intended Use

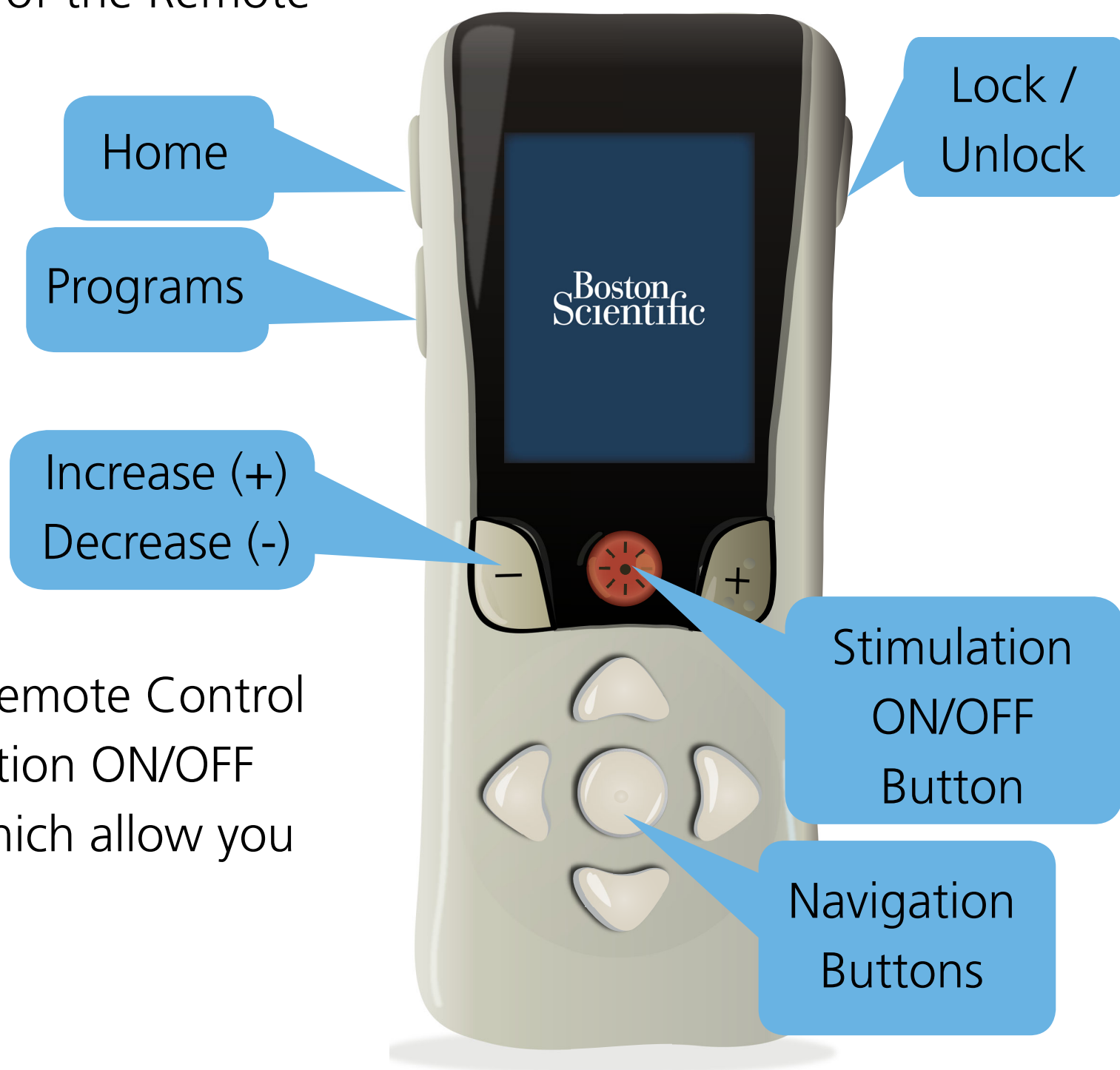
The Remote Control is a medical device that is intended to communicate with and control the compatible Boston Scientific Stimulator.

Remote Control Buttons

Buttons are located on the front and sides of the Remote Control as shown in the picture.

On the right side of the Remote Control, you will find the Lock/Unlock button . On the left side, you will find the HOME button  and Programs button **P**.

On the front of the Remote Control, you will find the Stimulation ON/OFF button , and the  and  buttons that allow you to adjust features, such as the Remote Control volume and brightness. Below the Stimulation ON/OFF button are the Navigation buttons , which allow you to navigate through various menus.

Remote Control Screen

The Remote Control screen displays menus, icons, status messages, error messages, and other information.

When the Remote Control is not in use (for example, the buttons are not being pressed), the screen automatically darkens after approximately 1 minute to preserve the Remote Control's battery. The screen lights up when the Remote Control is unlocked and remains on while the Remote Control is being used. To change the brightness of the screen, see the *"About System Settings"* section of this manual.

Typically, when the Remote Control is in use, the Home screen displays.

Home Screen

The Home screen indicates whether stimulation is ON  or OFF . If you have a rechargeable Stimulator, the Home screen will display the current battery level of the Stimulator.

Icons at the top of the Home screen indicate the charge level of the Remote Control battery  , and the number of the Program currently running.

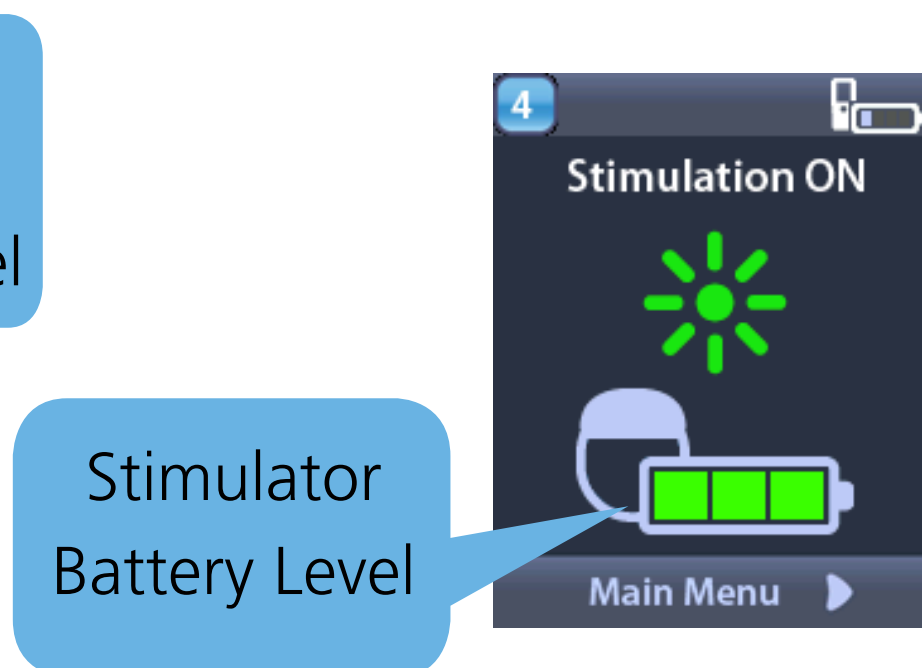
Note

Pressing the Home button  from almost all other screens will return you to the Home screen.

Non-Rechargeable Stimulator

Rechargeable Stimulator

Using the Remote Control

Quick Start Guide	14
Turn Stimulation ON/OFF	16
Charging the Remote Control	17
Accessories	18

Quick Start Guide

Unlock 

To turn Stimulation ON
press 

To turn Stimulation OFF
press 

To access the Main Menu press 

Lock 

QUICK START

Turn Stimulation ON/OFF

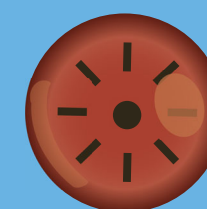
Note

Stimulation will not automatically turn OFF when the Remote Control is locked or when the screen darkens to preserve power in the Remote Control battery.

Stimulation can be turned ON and OFF by pressing the Stimulation ON/OFF button . To turn stimulation ON or OFF, you must unlock the Remote Control first; then, press the Stimulation ON/OFF button.



To
START/STOP
stimulation,
press

Charging the Remote Control

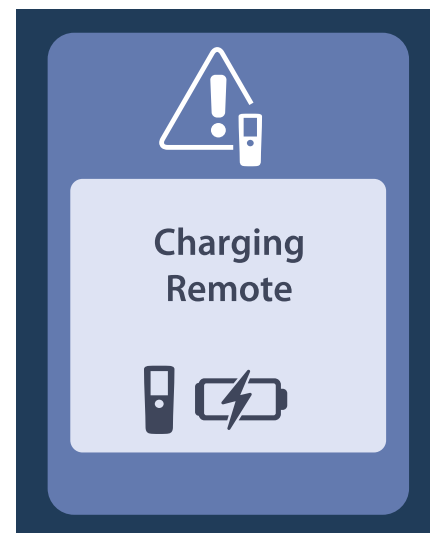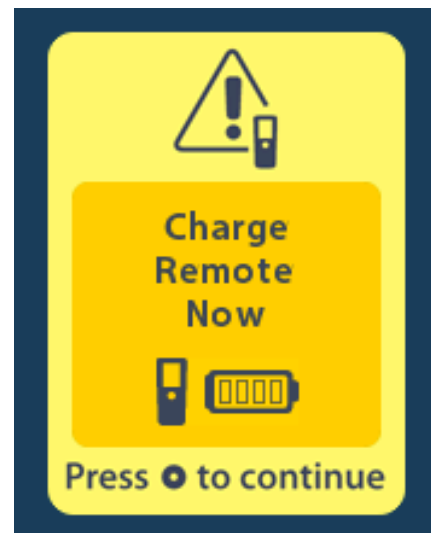The Remote Control displays messages reminding you to charge the battery when the battery power is low.

QUICK START

Charge the Remote Control battery by connecting the Remote Control to the Remote Power Supply provided. Then, find a convenient electrical outlet that will not expose the Remote Control or Power Supply parts to water or direct heat. Connect the power plug on the Power Supply to an electrical outlet. Any use of an electrical outlet has the risk of an electric shock. Be careful when plugging in the Power Supply.

Remote Control Battery Service Life

The Remote Control has a typical battery service life of 400 charging cycles.

Accessories

CAUTION

Always place the Remote Control in the Silicone Case so that the Stimulation button  is on the open side of the case and remains uncovered.

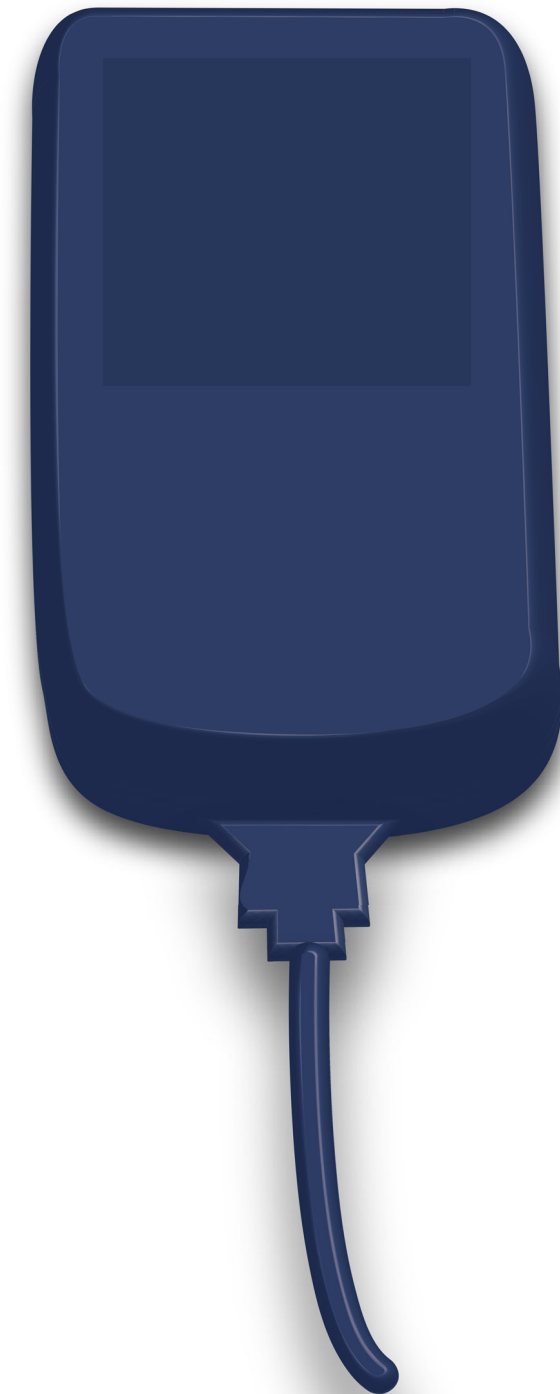
The following accessories are provided with the Remote Control:

Silicone Case - The Silicone Case allows you to customize your Remote Control. You can access all the buttons, the USB port, and see the screen while the device is in the Silicone Case. The side buttons that are covered by the Silicone Case are still available for use.

Holster - The Remote Control holster allows you to carry the Remote Control on a belt. You can only access the Stimulation ON/OFF button and the USB port of the Remote Control while it is enclosed in the holster.

Remote Power Supply - The Remote Power Supply is used to charge the Remote Control battery. The USB port on the Remote Control is designed to accept a standard 5V Mini-B USB power supply.

This page intentionally left blank.

How To Use Programs

What Is a Program?	22
Changing Stimulation Level	23
Accessing Programs	24
Overview of the Programs Menu	25
Change to a Different Program	26
Restore a Program	28

What Is a Program?

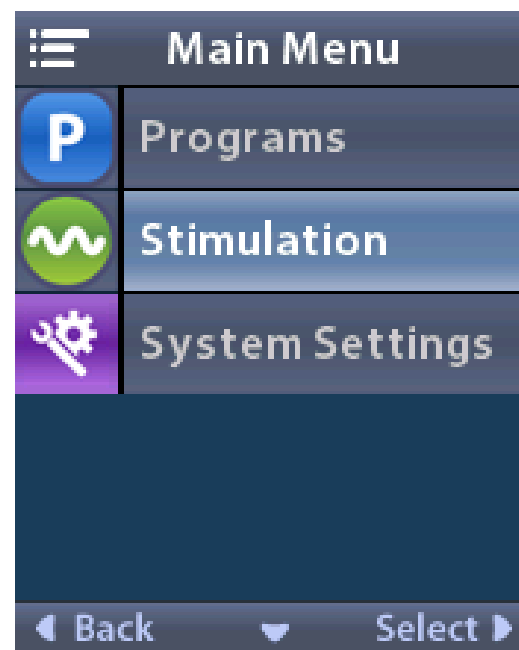
A Stimulation Program **P** is a group of configured settings that produce the stimulation to help you manage your symptoms. One or more Programs were configured by your healthcare provider. Whenever you are using your Stimulator, one of these Programs is active, and stimulation is delivered according to the configured settings. The Remote Control can store up to four Programs – numbered 1 through 4 – for you to select and activate at any time. Each Program is identified by a number.

Each Program **P** may apply stimulation to as many as four stimulation areas. For example, one area may correspond to a right brain target, while another area may correspond to a left brain target. These stimulation areas control additional parameters that can affect your stimulation therapy.

Changing Stimulation Level


Healthcare providers may allow certain patients to change the stimulation level of individual areas of a Program.

If your healthcare provider allows you to change your stimulation level, you may adjust stimulation level from the Main Menu. Select Stimulation  then select the amplitude that you wish to change.

Use the  or  button to adjust stimulation. On the Amplitude screen, L corresponds to the left side of the body and R corresponds to the right side of the body.

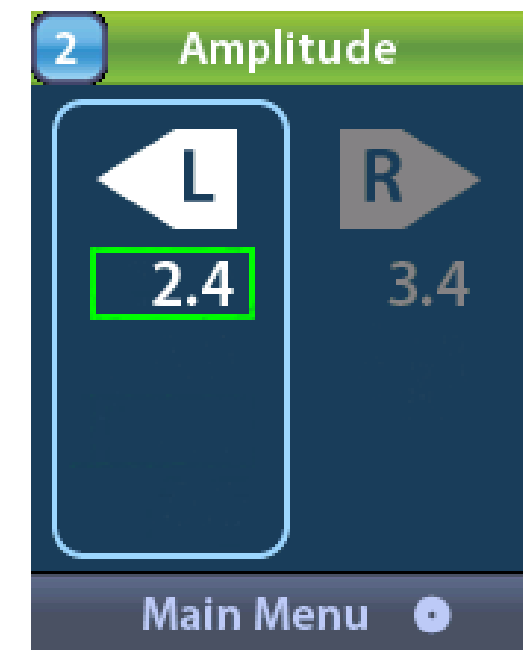If you adjust stimulation to the limit of the allowable range, the “Limit Reached” screen will appear. Press the  button to return to the Main Menu.

Accessing Programs

If your healthcare provider has configured Programs that can be changed, there are two ways to view the Programs menu:

Press the Programs button **P** on the left side of the Remote Control.

From the Main Menu , select Programs  by pressing .

Note

After entering the Main Menu , the  button can also be pressed to make a selection.

Overview of the Programs Menu

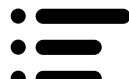
If your healthcare provider has configured Programs that can be changed, you can do any of the following operations:

- Turn Stimulation ON/OFF  and adjust stimulation level for the Program currently running
- Change  the currently running Program to a different Program (if available)
- Restore  a Program to the settings that your healthcare provider last configured for you

If any changes were made to your stimulation (for example, changing stimulation strength), the changes will save automatically.

Change to a Different Program

From the Home screen, press  to display the Main Menu .

Press  again to enter the Programs menu .

Press  again to enter the menu .

4 Press   to select the desired Program.

5 Press  to activate the selected Program.

PROGRAMS

Restore a Program

If you decide that you want to change the Program back to the way it was configured by your healthcare provider, you can do so by restoring the Program.

From the Main Menu
☰, press  to select Programs .

Press  to highlight Restore to Clinic Settings . Press  to select Restore to Clinic Settings .

**3**

The screen displays all Programs configured by your healthcare provider. Navigate to the Program you wish to restore and press .

**4**

Select Yes  to restore the Program or No  to cancel.

**5**

The Restore confirmation screen displays.

PROGRAMS

This page intentionally left blank.

About System Settings

About System Settings	32
Adjust Screen Brightness	33
Adjust Volume	34
View Information about Your Remote Control or Stimulator	35

About System Settings

Using the System Settings , you can set or change Remote Control interface settings, like Alert Volume , Screen Brightness , and the Language setting .

In addition, you can view information about the Remote Control  and the Stimulator . Your healthcare provider can access special functions through the Clinician Menu  (password protected) to aid you in your therapy.

If you are implanted with a Vercise Gevia rechargeable Stimulator, the Enter or Exit MRI Mode icon  or  will appear on the System Settings screen. See the “About MRI Mode” section of this manual for information on MRI Mode.

Adjust Screen Brightness

1

From the Main Menu
, select System
 Settings .

2

Select Remote
 Settings  by
 pressing .

3

Select Screen
 Brightness 
 by pressing .

4

To decrease or increase
 Screen Brightness,
 press the  or the 
 button.

SYSTEM SETTINGS

Adjust Volume

The Remote Control emits a single, short “beep” whenever you press a button. The volume of the alert can be adjusted or turned off.

1

From the Main Menu , select System Settings .

2

Select Remote Settings  by pressing .

3

Select Alert Volume  by pressing .

4

To increase or decrease Alert Volume, press the  or the  button.

View Information about Your Remote Control or Stimulator

You can view basic information about your Remote Control and Stimulator that may be helpful for your healthcare provider when troubleshooting an issue.

From the Main Menu
, select System
 Settings .

Select System Info
 by pressing .

Select Remote Info
 by pressing .

Select Stimulator Info
 by pressing .

SYSTEM SETTINGS

About MRI Mode (For Rechargeable Vercise Gevia Stimulator Only)

About MRI Mode	37
Enabling MRI Mode	38
Disabling MRI Mode	40
MRI Mode Error Messages	42

About MRI Mode

If you have a Vercise Gevia rechargeable Stimulator, you must use the Remote Control to enable MRI Mode on the Stimulator before an MRI scan is performed and exit MRI Mode after the MRI scan has been completed. Do not leave the Stimulator in MRI Mode for extended periods of time beyond what is necessary to perform the MRI scan. The Stimulator battery must be fully charged before MRI Mode is enabled.

Your healthcare provider and MRI center must ensure all guidelines listed in *ImageReady™ MRI Guidelines for Boston Scientific DBS Systems* are followed before an MRI scan is performed. Ask your healthcare provider or MRI center to contact Boston Scientific Customer Support or go to www.IFU-BSCI.com for the latest version of the manual.

Note

Always carry your Remote Control and Charger to the MRI center.

WARNING

The Remote Control and Charger are MR unsafe and must not be taken inside the scan room.

Enabling MRI Mode

1

From the Home screen, select  to display the Main Menu .

2

From the Main Menu, select System Settings .

3

Press  to navigate to the Enter MRI Mode  option and then press .

4

Select  Yes to enter MRI mode or  No to cancel the action.

**5**

The Stimulator performs a series of checks before MRI Mode is enabled.

**6**

The MRI Mode Enabled confirmation screen displays and Stimulation is turned OFF.

**7**

The Home screen displays the MR Conditional Symbol  when MRI Mode is enabled. Always check the MRI Home screen before entering the MRI scan room.

WARNING

The Remote Control is MR Unsafe and must not be taken into the MRI Scanner Room.

Note

Stimulation is automatically turned OFF when MRI Mode is enabled.

MRI MODE

Disabling MRI Mode

Upon completion of the MRI scan, the Remote Control is used to disable MRI Mode.

From the Home screen, select to display the Main Menu .

From the Main Menu, select System Settings .

Press to navigate to the Exit MRI Mode option and then press .

Select Yes to exit MRI mode or No to cancel the action.

**5**

The Stimulator performs a series of checks before MRI Mode is disabled.

**6**

MRI Mode disabled confirmation screen displays.

**7**

The Home screen on the Remote Control will not display the MR Conditional Symbol  once MRI Mode is disabled.

Note

The Stimulator retains the settings that were set before enabling MRI Mode. If stimulation was ON before MRI Mode was enabled, then disabling MRI Mode turns stimulation back ON. If stimulation was OFF before MRI Mode was enabled, then disabling MRI Mode keeps stimulation off.

MRI Mode Error Messages

WARNING

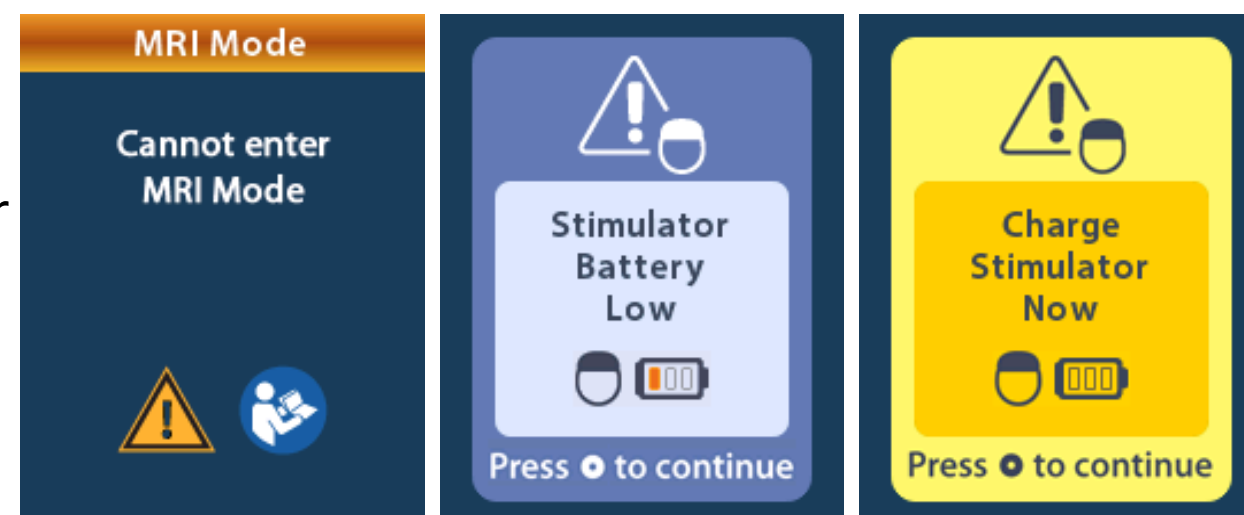
The Charger is MR Unsafe and must not be taken into the MRI Scanner Room.

Note

Always make sure to fully charge your Stimulator before an MRI scan. Depending on the Stimulator battery level, it may take a few hours for your Stimulator to be fully charged.

Charge Stimulator Now Screen

The Stimulator battery must be fully charged before MRI Mode is enabled. If the Stimulator battery is not fully charged, one of the following messages displays.

Press  to dismiss the error and charge the Stimulator without enabling MRI Mode. Enable MRI Mode after the Stimulator is fully charged.

Charge Stimulator Now or Disable MRI Mode Screen

A Charge Stimulator Now error message appears when MRI Mode is enabled and your Stimulator battery level falls below the recommended value.

You can either disable MRI Mode before charging your Stimulator or charge your Stimulator without disabling MRI Mode.

Disable MRI Mode before charging your Stimulator

Press  to disable MRI Mode. Charge your Stimulator and enable MRI Mode after the error message has disappeared and your Stimulator is fully charged.

OR

Charge your Stimulator without disabling MRI Mode

Do not press  and charge your Stimulator. The error message disappears after your Stimulator is fully charged. Make sure the Home screen still displays the MR Conditional symbol  before performing an MRI.

Impedances out of Range Screen

On enabling MRI Mode, the system performs a check of Lead impedances. If the impedances are not within the acceptable range, the Remote Control displays an error message.

Press  to acknowledge the message.

2

Press  to acknowledge the message. Your healthcare provider must check the integrity of the Leads and review the risks listed in *ImageReady™ MRI Guidelines for Boston Scientific DBS Systems* before proceeding with an MRI scan.

3

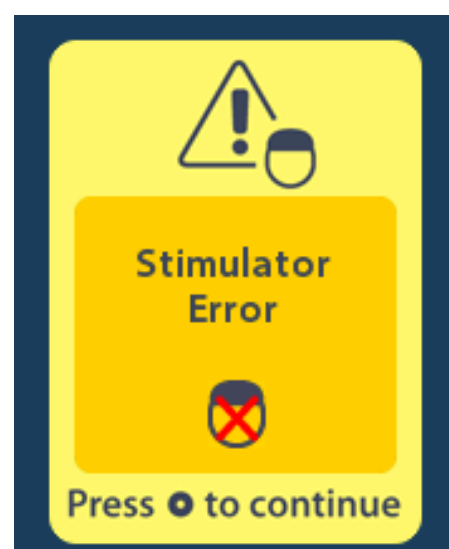
Select  Yes after healthcare provider approval to proceed with Enabling MRI Mode or  No to cancel the action.

MRI MODE

Stimulator Error Screen

If the system check fails due to a Stimulator error, MRI Mode will not be enabled and the Remote Control displays the Stimulator Error screen. This screen may display when there is a communication error between the Remote Control and Stimulator.

Press  to acknowledge the message and retry entering MRI Mode. If this error screen immediately displays again, your healthcare provider should not perform an MRI scan. Contact the healthcare provider managing your DBS System or Boston Scientific.



Reminders and Messages

Stimulator Low Battery Messages (Rechargeable Stimulators Only)	48
Remote Control Low Battery Message	49
Charging Remote Screen	50
Charger Disconnect Screen	50
Remote Battery Full Screen	51
Button Stuck	51
End of Programmed Service (Non-Rechargeable Stimulators Only)	52
ERI Screen (Non-Rechargeable Stimulators Only)	53
End of Service (EOS) Screen (Non-Rechargeable Stimulators Only)	54

Stimulator Low Battery Messages (Rechargeable Stimulators Only)

For a rechargeable Stimulator, the Remote Control provides messages to remind you when your Stimulator battery is low. One of the following messages displays:

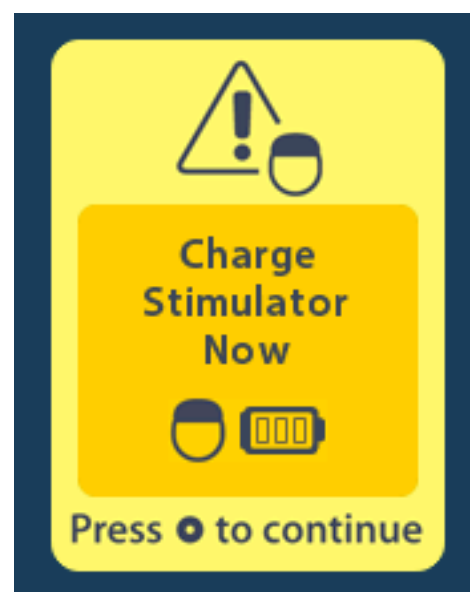
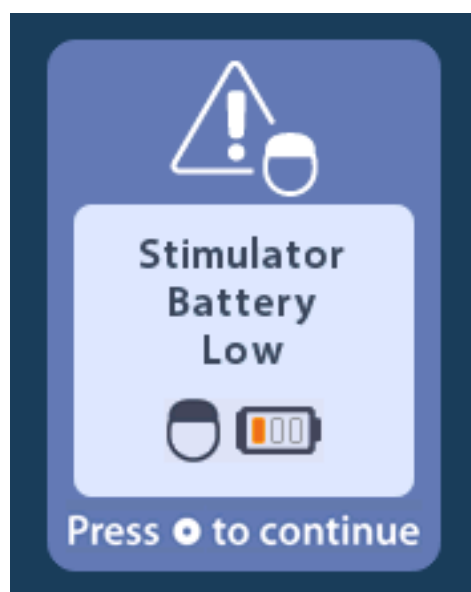
For instructions on charging your rechargeable Stimulator, refer to the *Vercise DBS Charging Handbook*.

Remote Control Low Battery Message


The Remote Control provides messages to remind you when your Remote Control battery is low. The following messages display:

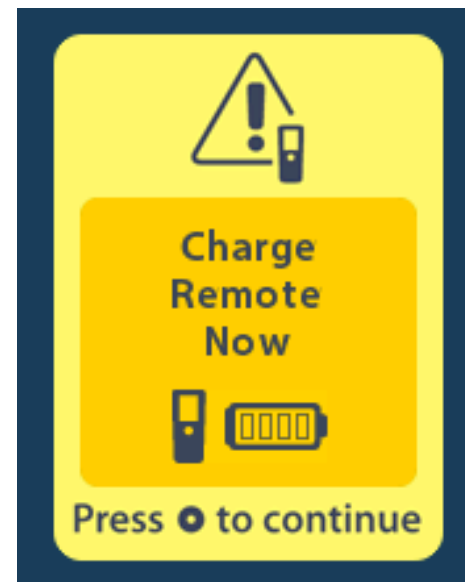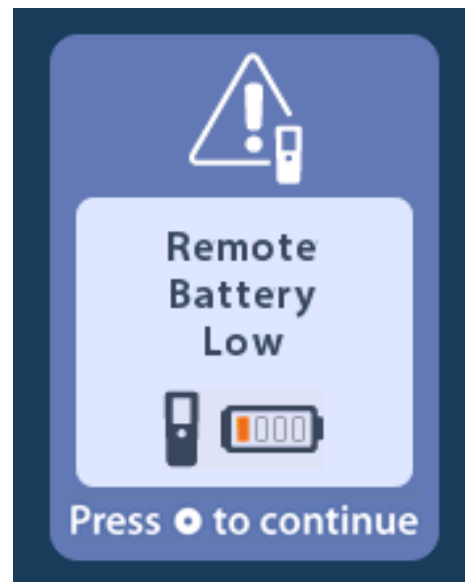See the “*Charging the Remote Control*” section of this manual for more information on charging your Remote Control.


Charging Remote Screen

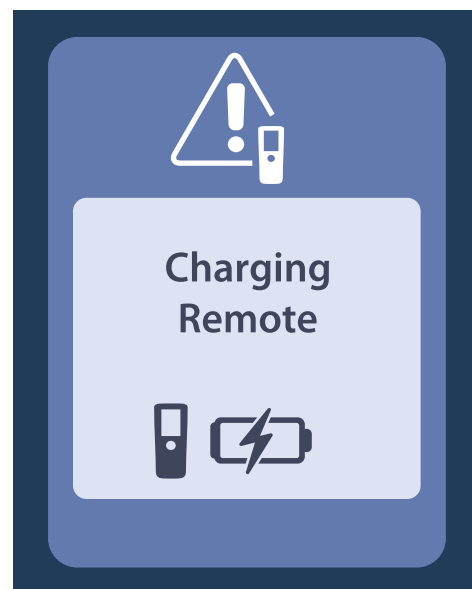This screen displays when the Remote Control is being used and the Remote Power Supply is plugged in.


Charger Disconnect Screen

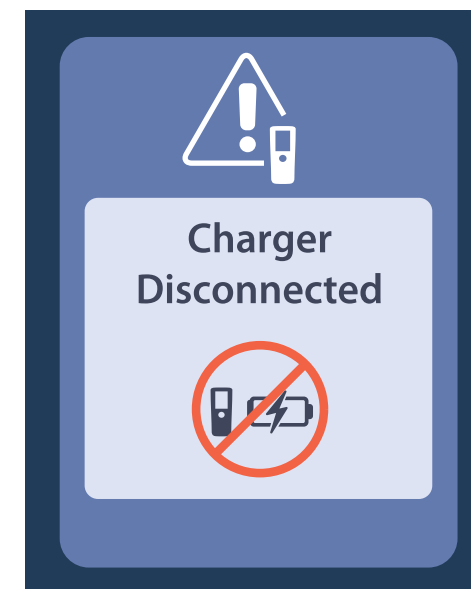This screen displays when the Remote Control is being used and the Remote Power Supply is disconnected.


Remote Battery Full Screen

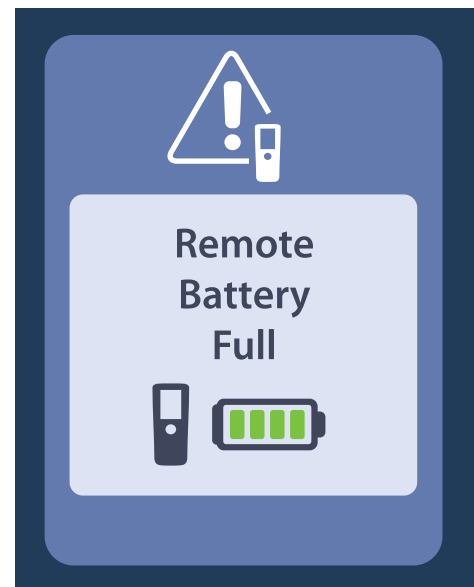This screen displays when the Remote Control is being used and the Remote Control battery becomes fully charged.

Button Stuck

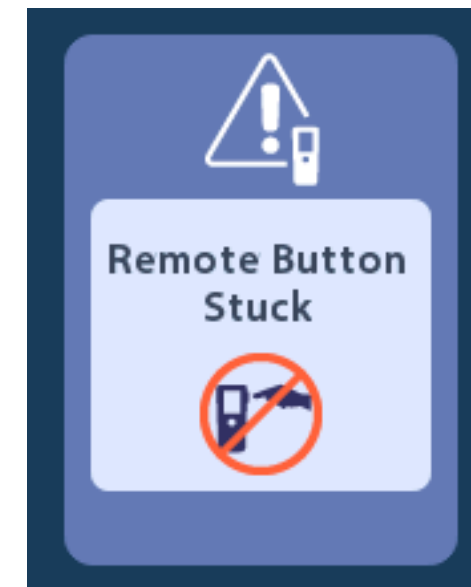If a button is stuck on the Remote Control, other button presses may not function as intended. The Remote Control provides the following message if it detects a stuck button:


Refer to your *Information for Patients* manual for Remote Control cleaning instructions. If the button is still stuck, contact Boston Scientific.


End of Programmed Service (Non-Rechargeable Stimulators Only)

The software for non-rechargeable Stimulators is programmed to end service after a defined period. If your non-rechargeable Stimulator is within approximately 180 days of the end of its programmed period, the Remote Control displays the following message:

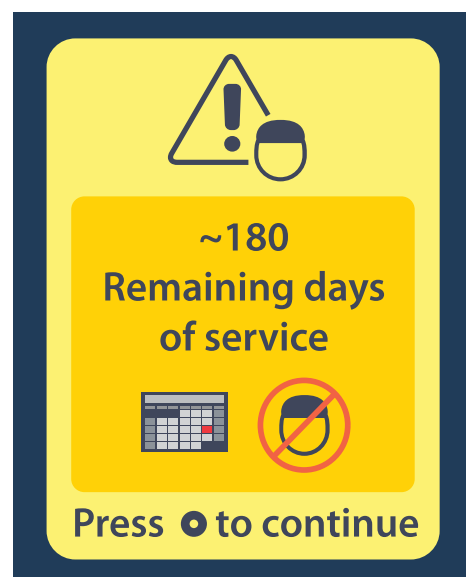Press  to clear the message.

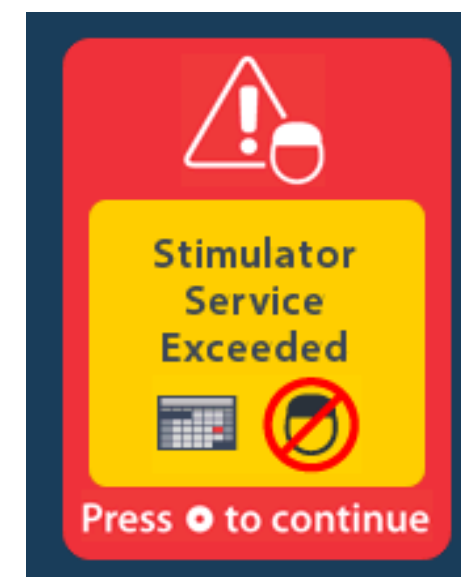The message will reappear approximately weekly until the end of the non-rechargeable Stimulator's programmed period is reached. Surgery is required to replace the implanted non-rechargeable Stimulator. Call your healthcare provider to report this message screen. When the end of its programmed period is reached, the Remote Control displays the following message:

ERI Screen (Non-Rechargeable Stimulators Only)

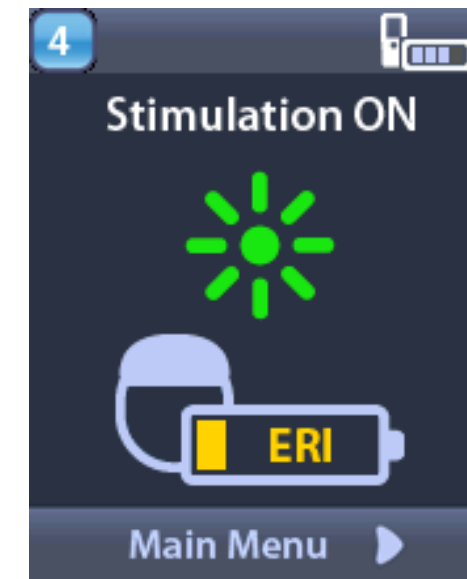
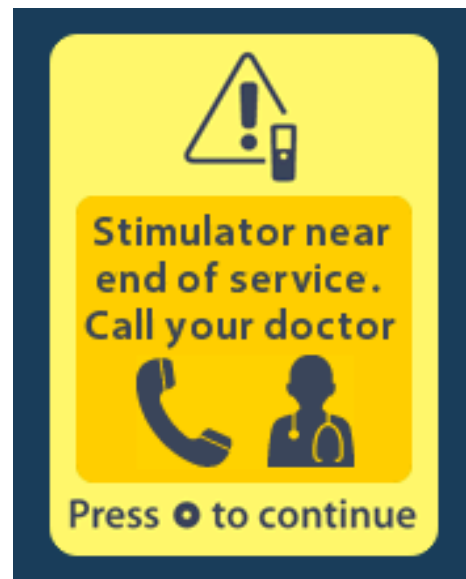
When the battery of a non-rechargeable Stimulator nears depletion, it enters Elective Replacement Indicator (ERI) mode and displays the following message:

“Stimulator near end of service. Call your doctor.”

Surgery is required to replace the implanted non-rechargeable Stimulator.

Call your healthcare provider to report the ERI message screen. Your Stimulator will still provide the programmed stimulation until the battery reaches End of Service (see the “*End of Service (EOS) Screen (Non-Rechargeable Stimulators Only)*” section of this manual).

Press  to clear this information screen. This screen appears whenever the Remote Control is activated.



End of Service (EOS) Screen (Non-Rechargeable Stimulators Only)

When the battery of an implanted non-rechargeable Stimulator is fully depleted, the Remote Control displays the following EOS message:

“Stimulator at end of service. Call your doctor.”

Surgery is required to replace the implanted non-rechargeable Stimulator. Call your healthcare provider to report the EOS message screen.

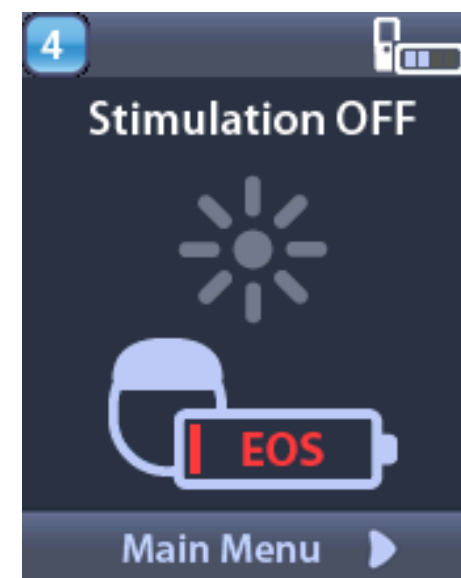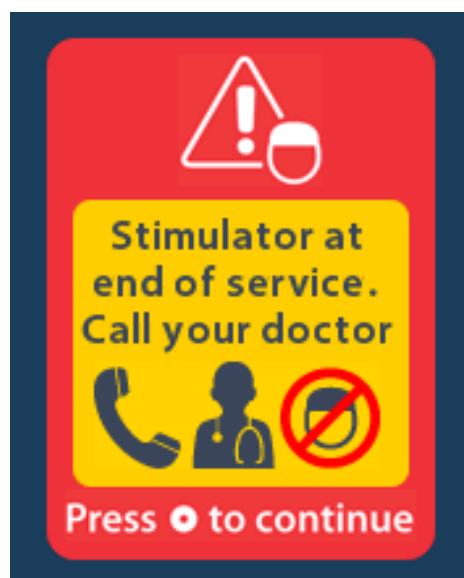The Home screen also displays the EOS indicator icon.

Troubleshooting

Telemetry Help	56
Telemetry Help Tool	58
Unlinked Remote Control	59
No Program to Run Screen	60
Invalid Program Screen	60
Unsupported Stimulator Screen	61
System Settings Reset Screen	61
Stimulator Error Screen	62
Remote Error Screen	62
Icon Description	63

Telemetry Help

The Remote Control has an average communication range of 22 in (55.8 cm) with the implanted non-rechargeable Stimulator and an average communication range of 36 in (91.4 cm) with the implanted rechargeable Stimulator.

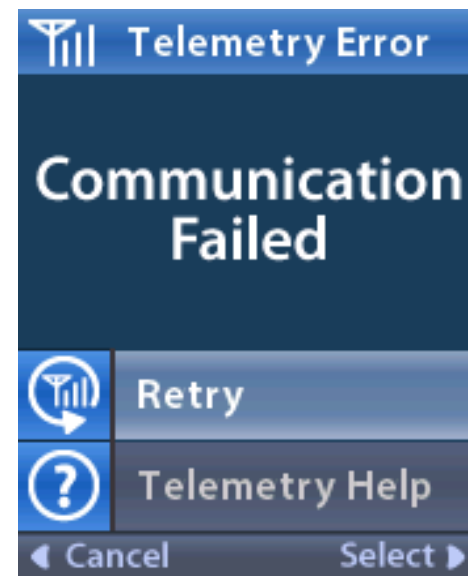
When using the Remote Control, avoid common sources of interference, such as television and computer monitors, short range RFID electronic tracking systems such as badge scanners and parking lot scanners, power adapters, and wireless chargers. These can affect wireless communication.

You will continue to have access to the Remote Control menus even if the Remote Control is outside its effective communication range with the Stimulator.

If you enter a command to change your settings or Programs while the Remote Control is outside its effective communication range, you have several seconds to move the Remote Control within range of the Stimulator to send the command. After the command is received by the Stimulator, the Remote Control makes a “double beep” and then displays the requested change on the screen.

If the Remote Control is unable to communicate with the Stimulator, the Remote Control displays the following Telemetry Error message:


“Communication Failed”

To retry communication with the Stimulator, select Retry . This resends the last command. To cancel communication and return to the System Settings screen, press .

Telemetry Help Tool

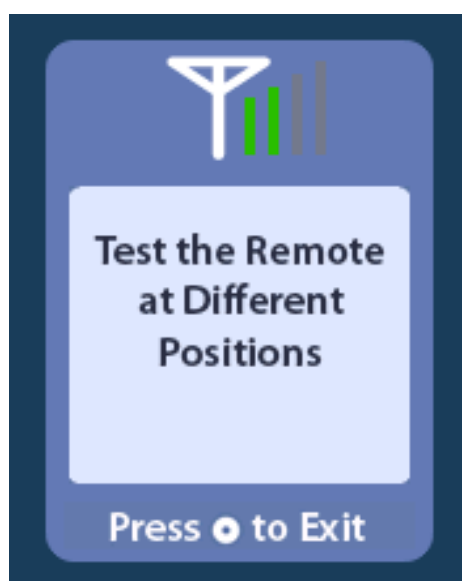
The Telemetry Help tool is also available to assist you in improving communication between the Remote Control and the Stimulator. To use the Telemetry Help tool, select Telemetry Help . The following screen displays:

The telemetry bars indicate the strength of wireless communication. The number of bars change depending on the distance and orientation of the Remote Control from the Stimulator. This tool continues to measure the strength of communication for 30 seconds or until  is pressed.

Unlinked Remote Control

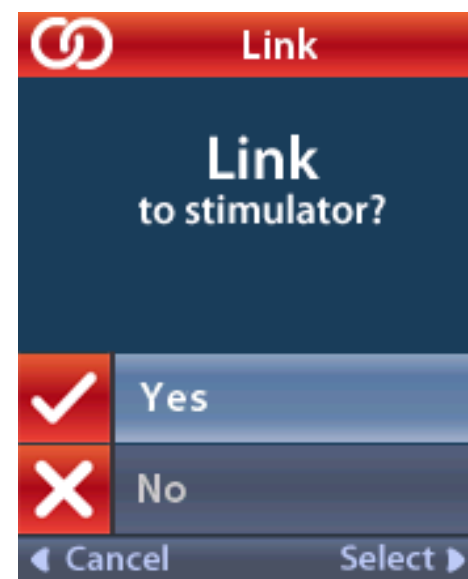
The Remote Control and Stimulator must establish a one-to-one link in order to communicate. This prevents a Remote Control from accidentally communicating with another Stimulator. If your Remote Control is not linked to your Stimulator (i.e., when you have a brand new Remote Control), the following screen appears and prompts you to link the Remote Control to the Stimulator:

Contact your healthcare provider if you see this message. Your healthcare provider can access special functions through the Clinician Menu to link your Remote Control to your Stimulator.


No Program to Run Screen

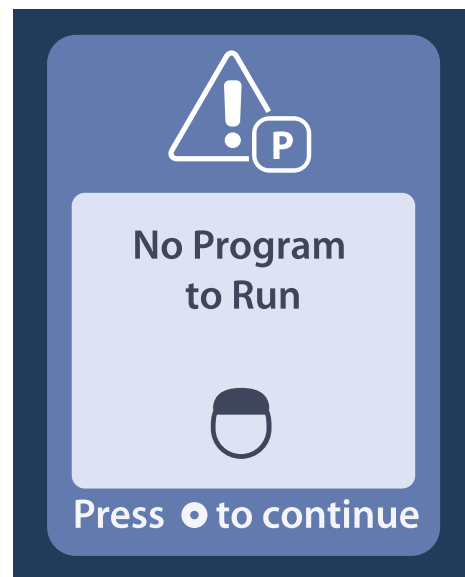This screen displays when the Stimulator has no Programs to run.


Invalid Program Screen

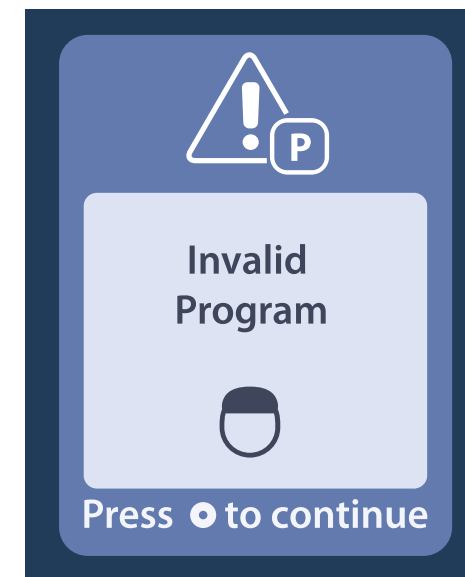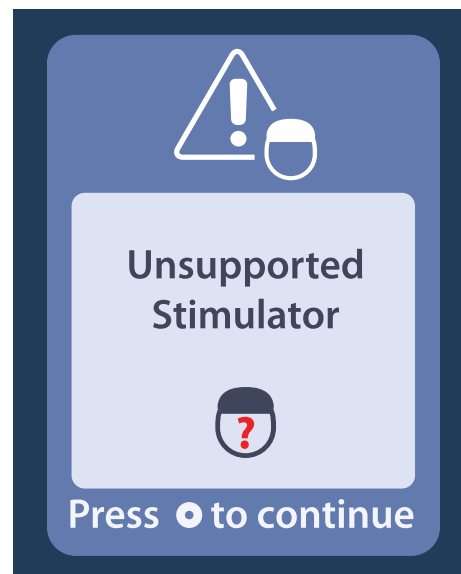This screen displays when the Stimulator attempts to run an invalid Program.

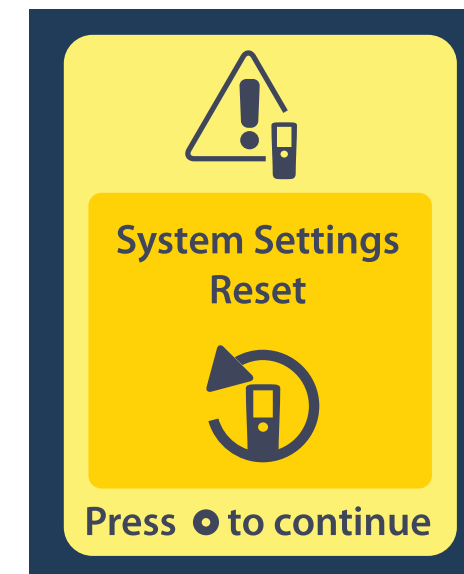
Unsupported Stimulator Screen

This screen displays when the Remote Control detects an incompatible Stimulator.

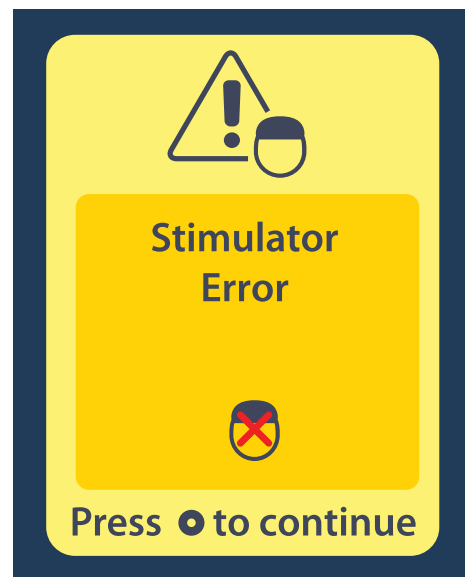
System Settings Reset Screen

This screen displays when the Remote Control experiences an error and a reset of the Remote Control is required. Press  to continue. All Remote Control settings will be restored to the default factory settings.

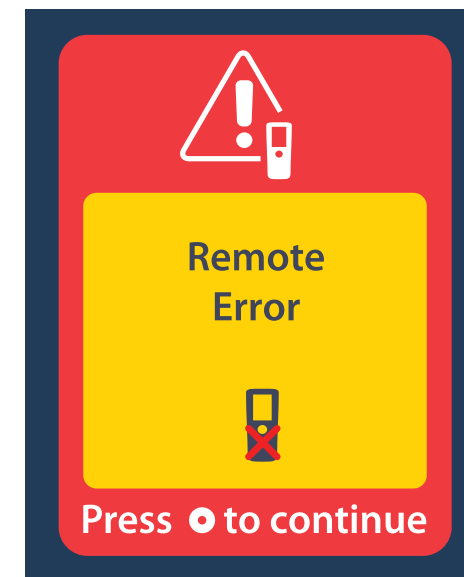
Stimulator Error Screen

The Stimulator Error is a non-critical error. This screen may also display when there is a communication error between the Remote Control and Stimulator. Press  to acknowledge the message and continue. If desired, retry the action that was interrupted by the error.

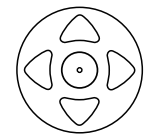
Remote Error Screen

This screen displays when there is a critical error on the Remote Control which makes the Remote Control unusable. Contact Boston Scientific or your healthcare provider to order a replacement Remote Control.

Icon Description

	A Alert Volume		H Home
	C Change Program		L Language Setting
	E Enter MRI Mode		Lock/Unlock
	Exit MRI Mode		M Main Menu
	F Follow Instructions for Use		Minus Button
			MR Conditional Symbol

N

Navigation

No

P

Program

Plus Button

R

Remote

Remote Battery

Remote Information

Remote Settings

Restore to Clinic Settings

Retry

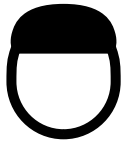
S

Screen Brightness

Stimulation Button

	Stimulation Level		T
	Stimulation OFF		Telemetry Help
	Stimulation ON		Y
	Stimulator		Yes
	Stimulator Battery		
	Stimulator Information		
	System Information		
	System Settings		

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or its affiliates. All rights reserved.
Boston Scientific Limited
Boston Scientific (Australia) Pty Ltd
© 2023 Boston Scientific Corporation