Bead Block™

Bland Embolic Bead

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2021-03

TABLE OF CONTENTS

REUSE WARNING	4
DEVICE DESCRIPTION	4
Table 1. Bead size ranges and indications	4
Contents	5
Operating Principle	5
Materials	5
Non-pyrogenic	5
User Information	5
INDICATIONS FOR USE	5
Recommended Catheters and Contrast Agents	5
Table 2. Size range and catheter compatibility	6
Summary of Safety and Clinical Performance	6
CONTRAINDICATIONS	6
Uterine Fibroid Embolization (UFE)-Specific Contraindications	7
Prostatic Artery Embolization (PAE)-Specific Contraindications	7
WARNINGS	7
UFE-Specific Warnings	8
Warnings about UFE and Pregnancy	8
PAE-Specific Warnings	9
PRECAUTIONS	9
UFE-Specific Precautions	9
PAE-Specific Precautions	9
ADVERSE EVENTS	9
UFE-Specific Potential Adverse Events	10
PAE-Specific Potential Adverse Events	11

HOW SUPPLIED	11
Device Details.....	11
Handling and Storage.....	11
OPERATIONAL INSTRUCTIONS	11
Instructions for Use.....	11
UFE-Specific Instructions for Use.....	12
PAE-Specific Instructions for Use.....	12
Preparation.....	12
Procedure.....	12
Disposal.....	13
Post-Procedure.....	13
Implantable Device Patient Information.....	13
INFORMATION TO BRIEF THE PATIENT	13
UFE and PAE Patient Counseling Information.....	14
REFERENCES	14
WARRANTY	14

Bead Block™

Bland Embolic Bead

REUSE WARNING

Reuse, reprocessing, or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing, or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.

DEVICE DESCRIPTION

Bead Block Bland Embolic Bead (Bead Block Device) comprises a range of hydrogel beads that are biocompatible, hydrophilic, nonresorbable, and precisely-calibrated. Bead Block Device is produced from polyvinyl alcohol (PVA) and is available in the following size ranges:

Table 1. Bead size ranges and indications

Bead Size Range		Indication		
Nominal Bead Size	Color Coding	Hypervascular Tumours/ Arteriovenous Malformations (AVMs)	Uterine Fibroid	Benign Prostatic Hyperplasia (BPH)
100 µm – 300 µm	Yellow	Yes	No	Yes
300 µm – 500 µm	Blue	Yes	No	Yes
500 µm – 700 µm	Red	Yes	Yes	No
700 µm – 900 µm	Green	Yes	Yes	No
900 µm – 1,200 µm	Purple	Yes	Yes	No

Bead Block Device is supplied in a 20 mL syringe, pre-filled with 2 mL Bead Block Device suspended in non-pyrogenic, sterile, phosphate-buffered saline transport solution. The total volume of Bead Block Device, including the solution, is approximately 7 mL. The label indicates the specific size of bead product contained in the syringe.

Contents

1 20 mL pre-filled syringe

Operating Principle

Bead Block Device uses mechanical obstruction of the target vessel as the mechanism of action resulting in ischemia of the tissue.

Materials

Bead Block Device consists of a copolymer derived from a PVA macromer modified with sulphonated groups from 2-acrylamido-2-methyl-1-propanesulfonic acid sodium salt.

Non-pyrogenic

This device meets pyrogen limit specifications.

User Information

Bead Block Device should only be used by physicians who are experienced in interventional vascular embolization procedures.

INDICATIONS FOR USE

Bead Block Device is indicated for the embolization of hypervascular tumors (HVTs), including uterine fibroids and arteriovenous malformations (AVMs). Bead Block Device is also indicated for the treatment of symptomatic benign prostatic hyperplasia (BPH).

Recommended Catheters and Contrast Agents

Bead Block Device has been tested and shown to be successfully delivered using the following combinations of bead size, contrast medium, and microcatheters shown in the table below.

Table 2. Size range and catheter compatibility

Product size range	Recommended catheter (internal diameter)	Recommended contrast agents
100 μm – 300 μm	$\geq 2.4\text{F}$ (0.42 mm/0.016 in)	Omnipaque 300 (Iohexol 300) Omnipaque 350 (Iohexol 350) Visipaque 320 (Iodixanol 320) Iomeron 350 (Iomeprol 350)
300 μm – 500 μm		
500 μm – 700 μm		
700 μm – 900 μm	$\geq 4.0\text{F}$ (1.03 mm/0.041 in)	Niopam/Optiray 300 (Ioversol 300) Niopam 370 (Ioversol 370) Isovue 300 (Solutrast 300)
900 μm – 1,200 μm		

Note: Other contrast agents have not been tested in conjunction with Bead Block Device.

Summary of Safety and Clinical Performance

For customers in the European Union: visit (<https://ec.europa.eu/tools/eudamed>) to view a summary of this device's safety and clinical performance information on the EUDAMED website.

CONTRAINDICATIONS

- Patients intolerant to occlusion procedures.
- Vascular anatomy or blood flow that precludes catheter placement or injection of embolics.
- Presence or likely onset of vasospasm.
- Presence or likely onset of hemorrhage.
- Presence of severe atheromatous disease.
- Presence of feeding arteries smaller than the succeeding distal branches.
- Presence of patent extra-to-intracranial anastomoses or shunts.
- Presence of collateral vessel pathways potentially endangering normal territories during embolization.
- Presence of end arteries leading directly to cranial nerves.

- Presence of arteries supplying the lesion/tumor not large enough to accept the beads.
- Vascular resistance peripheral to the feeding arteries precluding passage of the beads into the lesion/tumor.
- Do not use Bead Block Device in the following applications:
 - Embolization of large-diameter arteriovenous (AV) shunts (i.e., where the blood does not pass through the arterial/capillary/venous transition but directly from artery to vein).
 - Pulmonary arterial vasculature.
 - Any vasculature where the beads could pass directly into the internal carotid artery, the central circulatory system, or other non-target territories.
 - Any neurovascular location.

Uterine Fibroid Embolization (UFE)-Specific Contraindications

- Pregnant women.
- Active or suspected pelvic inflammatory disease.
- Malignancy of the pelvic region.
- Endometrial neoplasia or hyperplasia.
- Presence of submucosal fibroids with greater than 50 % growth into the uterine cavity.
- Presence of pedunculated serosal fibroid as the dominant fibroid(s).
- Fibroids with significant collateral feeding by vessels other than the uterine arteries.

Prostatic Artery Embolization (PAE)-Specific Contraindications

- Presence of prostate cancer.
- Presence of prostatic or lower urinary tract infection.
- Prior prostatitis.

WARNINGS

The following warning statements provide important information for safe use of Bead Block Device. Observe all warnings provided in these instructions for use. Failure to do so may result in patient injury or product damage.

- Studies have shown that Bead Block Device does not form aggregates and, as a result, penetrate deeper into the vasculature as compared to similarly-sized PVA particles. Care must be taken to choose a larger-sized Bead Block Device when embolizing AVMs with large shunts to avoid passage of the product into the pulmonary or coronary circulation.
- The color of the beads could be visible through the skin if injected into arteries feeding superficial tissues.

UFE-Specific Warnings

Warnings about UFE and Pregnancy

- There are no long-term data on the effects of UFE on the ability to become pregnant and carry a fetus to term, and on the development of the fetus.
- This procedure should only be performed on women who do not intend future pregnancy.
- Women who become pregnant following UFE may be at increased risk for the following:
 - Postpartum hemorrhage.
 - Preterm delivery.
 - Cesarean delivery.
 - Abnormal presentation at birth.
- Devascularization of the uterine myometrium resulting from UFE may increase the risk of uterine rupture of women who subsequently become pregnant following UFE.
- When using Bead Block Device for UFE, do not use beads smaller than 500 μm .
- An appropriate gynecologic work-up should be performed on all patients presenting for embolization of uterine fibroids (e.g., gynecologic history, fibroid imaging, or endometrial sampling to rule out carcinoma in patients with abnormal menstrual bleeding).
- The diagnosis of uterine sarcoma could be delayed by taking a non-surgical approach (such as UFE) to treating fibroids. It is important to pay close attention to warning signs for sarcoma (e.g., rapid tumor growth, post-menopausal with new uterine enlargement, or magnetic resonance imaging [MRI] findings), and to conduct a more thorough work-up of such patients prior to recommending UFE. Recurrent or continued tumor growth following UFE should be considered a potential warning sign for sarcoma and surgery should be considered.

PAE-Specific Warnings

- PAE does not reduce symptoms in all cases despite decreased prostate volume.
 - The prostate may continue to grow (with or without a reduction in symptoms).
-

PRECAUTIONS

The following precaution statements provide important information for safe use of Bead Block Device. Observe all precautions provided in these instructions for use. Failure to do so may result in patient injury or product damage.

- Do not use if the syringe or packaging appear damaged.
- Select the size and quantity of Bead Block Device appropriate for the pathology to be treated.
- Embolization with Bead Block Device should only be performed by physicians who are experienced in interventional occlusion in the region intended to be embolized.
- Do not reuse due to risk of infection.

UFE-Specific Precautions

- There is an increased chance of reflux of Bead Block Device into unintended blood vessels as uterine artery flow diminishes. Comparison of angiographic endpoint and infarction rate in individual patients indicates that best results were obtained with an endpoint close to stasis.
- The long-term outcome of UFE is at present unknown.

PAE-Specific Precautions

- Patients with tortuous arterial supply or atherosclerotic arteries should be excluded from treatment.
- The long-term outcome of PAE is at present unknown.

ADVERSE EVENTS

Potential adverse events which may be associated with vascular embolization procedures or the use of Bead Block Device include but are not limited to:

- Capillary bed saturation and tissue damage
- Clot formation at the tip of the catheter and its subsequent dislodgement
- Death
- Foreign body reactions necessitating medical intervention

- Infection necessitating medical intervention
- Ischemia at an undesirable location
- Ischemic stroke or ischemic infarction
- Liver abscess
- Neurological deficits, including cranial nerve palsies
- Non-target embolization, for example:
 - Pulmonary embolism
 - Pancreatitis
- Post-embolization syndrome (PES)
- Recanalization
- Undesirable reflux or passage of beads into normal arteries adjacent to the targeted lesion/tumor, or through the lesion/tumor into other arteries or arterial beds
- Vasospasm
- Vessel or lesion/tumor rupture and hemorrhage

UFE-Specific Potential Adverse Events

- Abdominal pain
- Amenorrhea
- Constipation
- Deep vein thrombosis with or without pulmonary embolism
- Discomfort
- Fever
- Hysterectomy
- Infection of the pelvic region
- Local vascular inflammation
- Nausea
- Post-UFE intervention to remove necrotic fibroid tissue
- Premature ovarian failure (e.g., menopause)
- Tissue passage, fibroid sloughing, or fibroid expulsion post-UFE
- Transient hypertensive episode
- Uterine/ovarian necrosis
- Vaginal discharge
- Vasovagal reaction

PAE-Specific Potential Adverse Events

- Genitourinary tract infection
- Post-procedural abdominal pain
- Transient hematospermia
- Transient hematuria
- Transient pollakiuria (frequent urge to urinate)
- Transient urinary retention

HOW SUPPLIED

Device Details

The pre-filled syringes are packaged in a sterile, sealed, pre-formed polycarbonate tray with a peel-off Tyvek lid.

Contents supplied STERILE using a steam process. Do not use if sterile barrier is damaged. If damage is found, call your Boston Scientific representative. For single use only. Do not reuse, reprocess, or resterilize.

Do not use if package is damaged or unintentionally opened before use.

Do not use if labeling is incomplete or illegible.

Handling and Storage

Protect the product from light. This product has no other special handling or storage requirements.

OPERATIONAL INSTRUCTIONS

Instructions for Use

- Carefully evaluate the vascular network associated with the lesion/tumor using high-resolution imaging prior to beginning the embolization procedure.
- The beads are available in a range of sizes. Choose the appropriate size bead that best matches the pathology (i.e., vascular target/vessel size) and provides the desired clinical outcome.
- When embolizing AVMs, choose a bead size that will occlude the nidus without passing through the AVM.
- Choose a delivery catheter based on the size of the target vessel.
- Introduce the delivery catheter into the target vessel according to standard techniques. Position the catheter tip as close as possible to the treatment site to avoid inadvertent occlusion of normal vessels.

- The beads are not radiopaque. It is recommended to monitor the embolization under fluoroscopic visualization by adding the desired amount of contrast medium to the phosphate-buffered saline.

UFE-Specific Instructions for Use

- An endpoint of stasis or near stasis is recommended, with the main uterine artery remaining patent, but with negligible residual flow toward the uterus. Bead sizes 500 μm – 700 μm and greater are shown to be effective in the treatment of uterine fibroids; however, 700 μm – 900 μm beads are recommended, with upsizing to 900 μm – 1,200 μm if required.
- This endpoint corresponds to an angiographic image of a patent horizontal segment with absent flow in the ascending segment of the uterine artery.
- At the discretion of the physician, pneumatic compression devices may be used for patients currently taking hormone therapy, uterine volume > 1,000 cm^3 , and patients that are overweight to lower the risk of deep vein thrombosis.

PAE-Specific Instructions for Use

- There is some clinical evidence to date for the use of smaller bead sizes; however, the majority has documented the use of 300 μm – 500 μm beads for PAE.

Preparation

1. To obtain a homogeneous suspension, directly aspirate the contrast medium into the syringe containing the beads in phosphate-buffered saline and remove all air from the syringe. The volume of contrast medium to be added varies depending on the contrast medium viscosity and the bead size. The typical volume of contrast medium required is 3 mL – 6 mL.
2. To evenly suspend the beads in the contrast medium, gently invert the 20 mL syringe several times. Attach the 20 mL syringe to 1 port of the luer-lock three-way stopcock. If desired, a delivery catheter may be attached to the remaining port on the stopcock. Wait several minutes to allow the beads to suspend properly.
3. Draw the bead/contrast medium into the injection syringe slowly and gently to minimize the potential of introducing air into the system. Purge all air from the system prior to injection.

Procedure

1. Inject the bead/contrast medium from the injection syringe under fluoroscopic visualization using a slow pulsatile action, while observing the contrast medium flow rate. If there is no effect on the flow rate, repeat

the delivery process with additional injections of bead/contrast medium, or larger-sized beads may be considered. If the bead/contrast medium requires re-suspension, gently invert the 20 mL syringe several times.

2. Upon completion of the treatment, remove the catheter while maintaining gentle suction so as not to dislodge any beads still within the catheter lumen.

Disposal

To minimize the risk of infection or microbial hazards after use, dispose device and packaging as follows.

After use, device may contain biohazardous substances. The device and packaging should be treated and disposed of as biohazardous waste or have them treated and disposed of in accordance with any applicable hospital, administrative, and/or local government regulations. Use of a biohazardous container with biological hazard symbol is recommended. Untreated biohazardous waste should not be disposed of in the municipal waste system

Post-Procedure

Assess patient for hematoma and/or other signs of bleeding at the puncture site.

Any serious incident that occurs in relation to this device should be reported to the manufacturer and relevant local regulatory authority.

Implantable Device Patient Information

Advise the patient that additional information may be available to them on the Boston Scientific website (www.bostonscientific.com/patientlabeling).

INFORMATION TO BRIEF THE PATIENT

The physician should consider the following points while counseling patients on the use of Bead Block Device in association with the interventional procedure:

- Discuss the risks and benefits, including review of potential adverse events listed in this document, both for Bead Block Device and for other interventional treatments likely to be employed.
- Inform the patient that the color of Bead Block Device could be visible through the skin if injected into arteries feeding superficial tissues.
- Discuss post-procedure instructions, including any follow-up appointments, lifestyle changes, medications, and home-care or rehabilitation guidelines.

- Provide the patient with the completed implant card to carry and advise the patient that additional information, including MRI conditions, may be available on the Boston Scientific website (www.bostonscientific.com/patientlabeling).
- Instruct the patient to present the implant card to their Healthcare Professionals (doctors, dentist, technicians) so they can take the necessary precautions.
- Inform the patient that the beads are a permanent implantable. The materials of the device are nonbiodegradable and are intended to last for the lifetime of the patient.

UFE and PAE Patient Counseling Information

- Patients should have a clear understanding prior to embolization of who will provide their post-procedure care, and whom to contact in case of an emergency after embolization.
- UFE and PAE candidates should have an understanding of the potential benefits, risks, and adverse events associated with the embolization procedure. In particular, patients should understand that there is a chance their fibroid or prostatic symptoms will not improve following embolization.

REFERENCES

The physician should consult recent literature on current medical practice on embolization.

WARRANTY

For device warranty information, visit (www.bostonscientific.com/warranty).

Bead Block is a registered trademark of Boston Scientific Corporation or its affiliates.

Iomeron is a trademark of Bracco Suisse S.A.

Isovue is a trademark of Bracco Diagnostics Inc.

Niopam is a trademark of Bracco UK Limited.

Optiray is a trademark of Liebel-Flarsheim Company LLC.

Omnipaque is a trademark of GE Healthcare AS.

Tyvek is a trademark of E. I. du Pont de Nemours and Company.

Visipaque is a trademark of GE Healthcare.

EU Importer: Boston Scientific International B.V., Veststraat 6, 6468 EX Kerkrade, The Netherlands

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INSTRUCTIONS FOR USE

en

The Instructions for Use (IFU) for this product are supplied in electronic form over the Internet.

Visit www.bostonscientific-elabeling.com to access the IFU in Adobe® Portable Document Format (PDF), and be sure to have the product label available for reference. The free Adobe Reader® program, used for accessing PDF files, is available at www.Adobe.com.

If you have difficulty accessing the IFU online, or would prefer to receive a paper copy, please contact Boston Scientific Customer Service or your local country contact. A copy will be sent to you at no charge and should arrive within seven days.

Please note that this IFU is subject to periodic updates. Visit the above website for the most current revision.

EU Importer: Boston Scientific International B.V., Vestastraat 6, 6468 EX Kerkrade, The Netherlands

Adobe® and Reader® are trademarks of Adobe Systems Incorporated.

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© 2021 Boston Scientific Corporation or its affiliates.
All rights reserved.
Boston Scientific (Australia) Pty Ltd