

## Details About Your Boston Scientific ALTRUA Pacemaker

### What is a pacemaker?

A pacemaker is a small, battery-powered device that holds a tiny computer. It is implanted in the body to watch for and treat abnormally slow heart rhythms. Your doctor programs your pacemaker to respond appropriately based on your individual heart rhythm needs.

### Treating slow heart rhythms

If an abnormally slow heart rhythm occurs, your ALTRUA pacemaker delivers small electrical signals that help your heart beat more normally. Your pacemaker can also help increase your heart rate to meet your body's needs when you are active.

### How long will my pacemaker last?

Like anything that operates on a battery, the life of your pacemaker will depend on how much the battery is used. The battery is used every time your device delivers therapy. So the amount of pacing you receive, and the amount of energy your doctor programs into the pacing therapy, affects the life of the battery. An ALTRUA pacemaker typically lasts 5–7 years.



ACTUAL SIZE  
(when printed at 100%)

### What happens after my implant procedure?

#### Care and recovery at home

Your doctor or nurse will give you specific instructions to follow when returning home. Full recovery from surgery can take from several days to a few months, depending on your condition. It is important that you are actively involved in your own recovery. Follow your doctor's instructions about caring for the implant site and letting your other doctors, dentists, and emergency personnel know that you have an implanted pacemaker system.

#### Follow-up visits

Once you have a pacemaker, it is important to follow your doctor's guidelines for follow-up visits. Your doctor uses the follow-up visits to check your device and monitor your heart condition, which can change over time. Your doctor may also adjust your device settings.

### How small is my pacemaker?

Model	Height	Width	Thickness	Weight
<b>S201</b> <b>S401</b> <b>S601</b>	1.65 in. 42 mm	1.65 in. 42 mm	0.31 in. 8 mm	0.83 oz. 23.4 g
<b>S203</b> <b>S403</b> <b>S603</b>	1.73 in. 44 mm	1.65 in. 42 mm	0.31 in. 8 mm	0.90 oz. 25.4 g
<b>S204</b>	1.73 in. 44 mm	1.65 in. 47 mm	0.31 in. 8 mm	0.90 oz. 24.9 g
<b>S202</b> <b>S208</b> <b>S402</b> <b>S404</b> <b>S602</b> <b>S606</b>	1.93 in. 49 mm	1.69 in. 43 mm	0.31 in. 8 mm	1.04 oz. 29.6 g
<b>S205</b>	2.13 in. 54 mm	1.93 in. 49 mm	0.31 in. 8 mm	1.04 oz. 32.3 g



Models S201, S401, S601, S203, S403, S603, S202,  
S402, S602, S204, S205, S208, S404, S606

## ***Important Safety Information***

### **Pacemakers**

A pacemaker system is designed to monitor and treat your heart rhythm problems, greatly reducing the risks associated with them. But it is not for everyone, including patients with certain steroid allergies. Procedure risks include infection, tissue damage, and kidney failure. Patients who have additional medical conditions that may not allow the pacemaker to function appropriately should not receive a device. In rare cases severe complications or device failures can occur. Electrical or magnetic fields can affect the device. Only your doctor knows what is right for you. This device is available by prescription only. Individual results may vary. Your physician should discuss all potential benefits and risks with you. For further safety information, refer to the Physicians Instructions for Use or call 1.866.484.3268.

### **Device Quality and Reliability**

It is Boston Scientific's intent to provide implantable devices of high quality and reliability. However, these devices may exhibit malfunctions that may result in lost or compromised ability to deliver therapy. Refer to Boston Scientific's CRM product performance report on [www.bostonscientific.com](http://www.bostonscientific.com) for more information about device performance, including the types and rates of malfunctions that these devices have experienced historically. While historical data may not be predictive of future device performance, such data can provide important context for understanding the overall reliability of these types of products. Also, it is important that you talk with your doctor about the risks and benefits associated with the implantation of a device.

(Rev. F)



*Defining tomorrow, today.™*

#### **Cardiology, Rhythm and Vascular**

One Boston Scientific Place  
Natick, MA 01760-1537 USA  
[www.bostonscientific.com](http://www.bostonscientific.com)

*Medical Professionals:*

*1.800.CARDIAC (227.3422)*

*Patients and Families:*

*1.866.484.3268*

© 2011 by Boston Scientific Corporation  
or its affiliates. All rights reserved.

CRM-15306-AA AUG2011